

SOMATOM Definition/Definition AS



DICOM Conformance Statement

Version syngo CT 2008G

Siemens AG Wittelsbacherplatz 2 D-80333 Muenchen Germany

Headquarters

Siemens AG, Medical Solutions Henkestr. 127, D-91052 Erlangen Germany Telephone: +49 9131 84-0 www.siemens.com/medical

Contact Address

Siemens AG, Medical Solutions Computed Tomography Siemensstr. 1, D-91301 Forchheim Germany

Telephone: +49 9191 18-0



© 2002-2008, Siemens AG Order No. C2-028.610.08.01.02 Printed in Germany 05/08



Manufacturer's note:

This product bears a CE marking in accordance with the provisions of the Council Directive 93/42/EEC of June 14th, 1993 concerning medical devices.

The CE marking applies only to medico-technical products/ medical products introduced in connection with the abovementioned comprehensive EC directives.

Table of Contents

Introduction

A DICOM Conformance Statement Network

- A.1 Implementation Models
- A.2 AE Specifications
- A.3 Communication Profiles
- A.4 Extensions/Specializations/Privatizations
- A.5 Configuration
- A.6 Support of Extended Character Sets

B DICOM Conformance Statement PartII-Media Storage

- B.1 Implementation Model
- B.2 AE Specifications
- B.3 Augmented and Private Profiles
- B.4 Extensions, Specializations and Privatizations of SOP Classes and Transfer Syntaxes
- B.5 Configuration
- B.6 Support of Extended Character Sets

Annex

Table of Contents DICOM

Introduction

Overview

This DICOM Conformance Statement is written according to part PS 3.2 of [1].

The applications described in this conformance statement are implemented in the Siemens SOMATOM products using software Somaris/7 *syngo* CT 2008G (internal: VA20A).

The Somaris/7 DICOM network implementation acts as SCU and SCP for the DICOM Verification, Storage, Storage Commitment Push Model, and Query/Retrieve Services. It acts as SCU for the DICOM Print Management Service, the Modality Worklist Service, and the Performed Procedure Step Service.

These services are described in "Part I".

The Somaris/7 DICOM Media Storage Service implementation acts as FSC, FSU, and FSR for the specified application profiles and the related SOP Class instances. These services are described in "Part II".

Somaris/7 is based on a Siemens common medical platform. This platform is shared with other Siemens modalities in order to provide a common look and feel and common interoperability features.

In this document parts of the Siemens common medical platform are referenced by the terms "syngo®"1, "MedCom", and "CSA".

As Somaris/7 is a *syngo* based product, this DICOM Conformance Statement is based on the corresponding *syngo* template [2].

^{1. &#}x27;syngo' and 'We speak syngo' are registered trademarks of Siemens AG

Audience

This document is intended for hospital staff, health system integrators, hospital IT-managers, and software designers or implementors. It is assumed that the reader has a working understanding of DICOM.

Scope

This DICOM Conformance Statement refers to Siemens SOMATOM products using software Somaris/7. The following table relates software names to Siemens products.

| Software Name | SIEMENS Products |
|-----------------------------|-----------------------|
| syngo CT 2008G | SOMATOM Definition |
| (internal: Somaris/7 VA20A) | SOMATOM Definition AS |

Definitions, Acronyms and Abbreviations

Definitions

- DICOM: Digital Imaging and Communications in Medicine
 DIMSE: DICOM Message Service Element
 DIMSE-C: DICOM Message Service Element with
 Composite information objects
- Acronyms and Abbreviations

□ UID: Unique Identifier□ VR: Value Representation

|) | ACR: American College of Radiology |
|---|-------------------------------------------------------------------|
|) | AE: DICOM Application Entity |
|) | ASCII : American Standard Code for Information Interchange |
|) | FSC: File Set Creator |
| 1 | FSR: File Set Reader |
|) | FSU: File Set Updater |
|) | HIS: Hospital Information System |
|) | IOD: DICOM Information Object Definition |
|) | ISO: International Standard Organization |
|) | NEMA : National Electrical Manufacturers Association |
|) | O: Optional Key Attribute |
|) | R: Required Key Attribute |
|) | RIS: Radiology Information System |
|) | RWA: Real-World Activity |
|) | PDU: DICOM Protocol Data Unit |
|) | SCU: DICOM Service Class User (DICOM client) |
|) | SCP: DICOM Service Class Provider (DICOM server) |
| 1 | SOP: DICOM Service-Object Pair |
|) | U: Unique Key Attribute |

References

[1]: Digital Imaging and Communications in Medicine (DICOM) 3.0, NEMA PS 3.1-3.14

[2]: syngo VE32B DICOM Conformance Statement

Connectivity and Interoperability

The implementation of the Somaris/7 DICOM interface has been carefully tested to assure correspondence with this Conformance Statement. But the Conformance Statement and the DICOM standard do not guarantee interoperability of Siemens modalities and modalities of other vendors. The user must compare the relevant Conformance Statements and if a successful interconnection should be possible, the user is responsible to specify an appropriate test suite and to validate the interoperability, which is required. A network environment may need additional functions out of the scope of DICOM.



DICOM Conformance Statement

Network



Implementation Models

Verification


The Verification service class defines an application-level class of service which allows for the operator to verify the ability of an application on a remote node to respond to DICOM messages. The DICOM Service Tool application supports the Verification service to act as SCU.

The other direction – responding to Verification requests from remote applications – is handled by the Storage SCP application.

Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU for the C-ECHO DICOM network service. The product target operating system is Windows XP.

DICOM Standard Interface

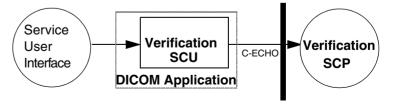

Functional Definitions of Application Entities

The DICOM Service Tool application opens an association when a "verification" of a remote application is requested during a configuration session. This can be done when entering new data for remote application configuration or to verify existing configuration data.

Sequencing of Real-World Activities

Newly configured data of a remote AE have to be saved first, before a "verification" of these data is possible.

Storage

The Somaris/7 DICOM Application Entity both originates associations for Storage of DICOM Composite Information Objects in Remote Application Entities and accepts association requests for Storage from Remote Application Entities.

Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU and SCP for the C-STORE DICOM network service and as SCP for the C-ECHO DICOM network service. The product target operating system is Windows XP.

"Send to ..." User Interface DICOM Standard Interface Storage SCU C-STORE DICOM Receiver (Storage SCU)

Database Storage SCP Verification SCP DICOM Sender (Storage SCU) DICOM Application

Functional Definitions of Application Entities

The Storage SCU is invoked by the job control interface that is responsible for processing network archiving tasks. The job consists of data describing the composite image objects selected for storage and the destination. An association is negotiated with the destination application entity and the image data is transferred using the C-STORE DIMSE-Service. Status of the transfer is reported to the job control interface.

The Storage SCP component of the Siemens Somaris/7 DICOM application is operating as background server process. It is existing, when the machine is powered on and waits for Storage association requests. Upon accepting an association with a negotiated Presentation Context it starts to receive the Composite Image Objects and imports them to local database. Verification requests will be processed and responded to by Storage SCP component, too.

Sequencing of Real-World Activities

Not applicable.




Network – Implementation Models

Storage Commitment

The Storage Commitment service class defines an application-level class of service which facilitates the commitment to storage. It performs an additional task of commitment of composite objects apart from the network based storage of images as defined by the Storage Service class. The Somaris/7 DICOM implementation supports the Storage Commitment Push Model as SCU and SCP.

Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU for the Storage Commitment Push Model Service using the Storage Commitment Service Class. The product target operating system is Windows XP.

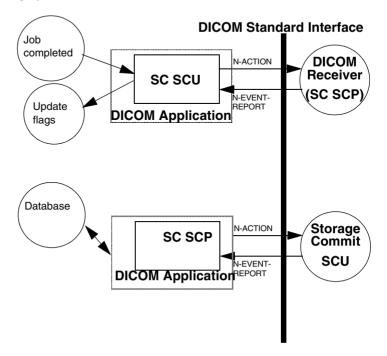

Functional Definitions of Application Entities

With each successfully completed send job, the Somaris/7 DICOM application will create a Storage Commitment Push Model Identifier from the SOP Instances sent. Then a Storage Commit Request is triggered. Depending on configuration, the Somaris/7 DICOM application will keep the association open for responses with a configurable time-out, or closes the association and expects responses on a different association that has to be established by the remote Storage Commitment SCP.

The commitment status derived from the related trigger response will be indicated in the Status Flags of the related entrity. It is possible to create triggers ("auto rules") from this event.

The Transaction UIDs of the pending commitment request are kept "open" for a configurable amount in time (default: 1h). If the "open time" for a pending commitment request has elapsed without a related response from the provider, the Transaction UID is removed and the related entities are indicated as "commit failed"

In any case, commitment will only be requested for previously and successfully sent images.

Sequencing of real World Activities

The Storage Commitment trigger is automatically derived from the successful completion of a Send job.


Query/Retrieve

The query/retrieve service class defines an application-level class of services which facilitates the management of images and patient data against the well defined information model of DICOM and allows a DICOM AE to retrieve images from a remote DICOM node or to request a remote DICOM AE to initiate a transfer of images to another DICOM AE. The DICOM query/retrieve application supports the query/retrieve services to act as SCU and SCP.


Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU and SCP for the query/retrieve network service. The product target operating system is Windows XP.

DICOM Standard Interface

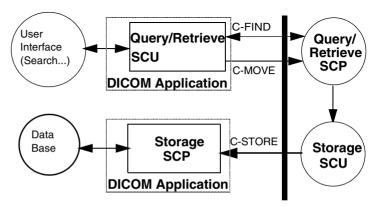

DICOM Standard Interface

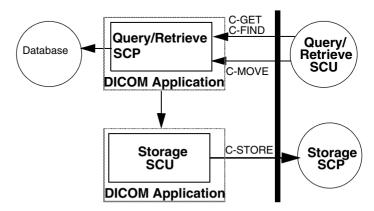

Functional Definitions of Application Entities

The query/retrieve SCU requests the remote query/retrieve SCP to perform a search and match to the keys specified in the request in order to display the results in the user interface. Depending on user action (Import) the Somaris/7 DICOM SCU sends a C-MOVE DIMSE service to initiate a C-STORE suboperation on the SCP to start an image transfer from the remote Storage SCU to the Somaris/7 DICOM Storage SCP.

The query/retrieve SCP responds to C-FIND DIMSE services from remote SCU applications. Depending on further remote request, a C-GET or a C-MOVE involves the Somaris/7 DICOM query/retrieve SCP application to initiate a C-STORE association (by triggering and parameterizing the own Storage SCU) to send image objects to a remote Storage SCP.

All components of the Somaris/7 DICOM query/retrieve SCP application are operating as background server processes. They are existing when the machine is powered on and then respond to queries based on the records stored in its database.

Note

The Somaris/7 DICOM query/retrieve SCU application will execute new queries based upon the data found in the higher level query. For details see → Page A.2–54.

Sequencing of Real-World Activities

Retrieve of images is only possible if results from a previous "Search..." operation exist and those entities can be selected for "Import".


Print

The Print Management Service Classes define an application-level class of services which facilitate the printing of images on a hardcopy medium. The print management SCU and print management SCP are peer DICOM print management application entities. The DICOM print application supports the print management DIMSE services to act as SCU.

Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU for the print management network service. The product target operating system is Windows XP.

DICOM Standard Interface

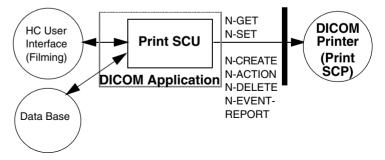

Functional Definitions of Application Entities

The Print SCU is invoked by the user interface to set up film sheet layout and whenever an image is ready to be printed on film. The print SCU will hold and maintain all data needed to compile a complete film sheet from the data (images, layout, configuration) received. Whenever a film sheet is ready to print the related data is used to supply the information to the SOP Classes of the Print Management Service Class. A queue is maintained in order to intermediately store several film sheets in case of resource problems on printer. The SCU will only supply and require the mandatory SOP Classes of the Print Management Service Class.

Sequencing of Real-World Activities


Not applicable.

Model Worklist

The Basic Worklist Service Class defines an application-level class of service which facilitates the transfer of worklists from the information system to the imaging modality. The worklist is queried by the AE and supplies the SCU with the scheduled tasks, which have to be performed on the modality. The DICOM worklist application supports the worklist service as SCU.

Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU for the Basic Worklist Service using the Modality Worklist SOP Class. The product target Operating System is Windows XP.

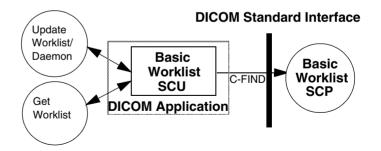

Note: It is configurable to get the worklist updates either automatically (in a configurable time interval) or manually (initiated by the user). There are two kinds of query the user can do: broad worklist query (all jobs for the own modality or own application entity) and patient based worklist query (where more search keys can be given, including Patient Name and Patient ID).

Functional Definitions of Application Entities

The worklist SCU ("broad query") is invoked from the patient browser user interface or by timer to request the worklist from a remote Information System (Modality Worklist SCP). This is done to perform a match to the internal worklist query keys specified in the C-FIND DIMSE service issued for the Modality Worklist Model

The Worklist SCP responds to the C-FIND query and scheduled imaging service requests (scheduled procedure steps) and patient demographic information will be downloaded from the information system to the Somaris/7 modality. All information retrieved will be held in the scheduling database for usage during patient registration procedure.

Sequencing of Real-World Activities


The "narrow" (patient based) Worklist Query requires that sufficient matching keys or a unique matching key are/is entered before the query is issued. Only then a single response can be expected to complete the registration dialog.

Modality Performed Procedure Step

The Modality Performed Procedure Step service class defines an application-level class of service which facilitates the transfer of billing and radiation dose information from the imaging modality to the information system. The performed procedure step is created and set by the AE and supplies the SCP with the information about a real world procedure which is performed on the modality. The DICOM Modality Performed Procedure Step application supports the MPPS service as SCU.

Application Data Flow Diagram

The Somaris/7 DICOM network implementation acts as SCU for the worklist network service. The target operating system is Windows XP.

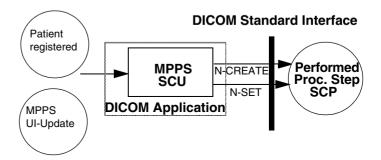

Functional Definitions of Application Entities

With registration of a patient (i.e. a Scheduled Procedure Step from Worklist), the Somaris/7 DICOM application will create a MPPS instance and communicate it to the MPPS SCP.

Furthermore a manual update can be performed with the Somaris/7 MPPS user interface. Only there it is possible to set the state of the MPPS to "Completed" or "Discontinued". If done so, the DICOM application will no longer allow updates on the related MPPS Instance.

Multiple requested procedures scheduled for a CT examination will result in multiple studies and MPPS, each requested procedure (RP) related to one study.

Somaris/7 supports creation of "unscheduled cases" by allowing MPPS instances to be communicated for locally registered patients.

Sequencing of real-World Activities

Not applicable.



AE Specifications

Verification AE Specification

Association Establishment Policies

General

The Somaris/7 DICOM Service Tool application attempts to initiate a new open an association for verification request whenever the "verification" function is activated during network configuration of a remote DICOM application.

Number of Associations

The Somaris/7 DICOM Service Tool application initiates one association at a time to request verification.

Asynchronous Nature

The Somaris/7 DICOM implementation Service Tool application does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

| Implementation | Class | UID | is: |
|----------------|-------|-----|-----|
|----------------|-------|-----|-----|

"1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS S7VA20A".

Association Initiation by Real-World Activity

The Somaris/7 DICOM Service Tool application attempts to initiate a new association for

□ DIMSE C-ECHO

service operations.

Associated Real-World Activity - Verification

Associated Real-World Activity - Verification SCU

The associated Real-World activity is a C-ECHO request initiated by the Service and Configuration SW environment whenever a "verification" is requested. If an association to a remote Application Entity is successfully established, Verification with the configured AET is requested via the open association. If the C-ECHO Response from the remote Application contains a status other than "Success" this will be indicated in the service environment and the association is closed.

Proposed Presentation Contexts - Verification SCU

The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | |
|--------------|----------------------------|-------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------|------|-------------|
| Abs | tract Syntax | Transfer Syntax | | Role | Extended |
| Name | UID | Name List | UID List | | Negotiation |
| Verification | 1.2.840.10008.1.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1. 2 1.2.840.10008.1. 2.2 1.2.840.10008.1. 2.1 | SCU | None |

SOP Specific Conformance Statement - Verification SCU The Application conforms to the definition of a Verification SCU in accordance to the DICOM Standard.

Association Acceptance Policy

The Verification SCP is part of the Storage SCP – see → page A.2–19, *Association Acceptance Policy*.

Storage AE Specification

The Somaris/7 Storage service class user applications and service class provider applications use one AE when initiating/receiving associations to/from remote DICOM nodes.

The SIEMENS Somaris/7 DICOM implementation provides Standard Conformance to the following DICOM V3.0 SOP Classes as both an SCU and SCP:

| SOP Class Name | SOP Class UID |
|-----------------------------------------------------------------------------|-------------------------------|
| Computed Radiography Image Storage | 1.2.840.10008.5.1.4.1.1.1 |
| D igital X -Ray Image Storage - For Presentation | 1.2.840.10008.5.1.4.1.1.1.1 |
| D igital X -Ray Image Storage - For Processing | 1.2.840.10008.5.1.4.1.1.1.1.1 |
| Digital M ammo G raphy Image Storage - For Presentation | 1.2.840.10008.5.1.4.1.1.1.2 |
| Digital M ammo G raphy Image Storage - For Processing | 1.2.840.10008.5.1.4.1.1.1.2.1 |
| Digital Intra-oral X-Ray Image Storage - For Presentation | 1.2.840.10008.5.1.4.1.1.3 |
| Digital Intra-oral X-Ray Image Storage - For Processing | 1.2.840.10008.5.1.4.1.1.3.1 |
| Computed Tomography Image Storage | 1.2.840.10008.5.1.4.1.1.2 |
| UltraSound Multi-Frame Image Storage (retired) (SCP only) | 1.2.840.10008.5.1.4.1.1.3 |

| SOP Class Name | SOP Class UID |
|---------------------------------------------------|-----------------------------------|
| UltraSound Multi-Frame Image Storage | 1.2.840.10008.5.1.4.1.1.3.1 |
| Magnetic Resonance Image Storage | 1.2.840.10008.5.1.4.1.1.4 |
| UltraSound Image Storage (retired) (SCP only) | 1.2.840.10008.5.1.4.1.1.6 |
| UltraSound Image Storage | 1.2.840.10008.5.1.4.1.1.6.1 |
| Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7 |
| 12-lead ECG Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.1.1 |
| General ECG Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.1.2 |
| Ambulatory ECG Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.1.3 |
| Hemodynamic Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.2.1 |
| Cardiac Electrophysiology Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.3.1 |
| Basic Voice Audio Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.4.1 |
| X-Ray Angiographic Image Storage | 1.2.840.10008.5.1.4.1.1.12.1 |
| X-Ray RadioFluoroscopic Image Storage | 1.2.840.10008.5.1.4.1.1.12.2 |
| N uclear M edicine Image Storage | 1.2.840.10008.5.1.4.1.1.20 |
| Basic Text SR | 1.2.840.10008.5.1.4.1.1.88.1 1 |

| SOP Class Name | SOP Class UID |
|-------------------------------------------------------|-----------------------------------|
| Enhanced SR | 1.2.840.10008.5.1.4.1.1.88.2 2 |
| Comprehensive SR | 1.2.840.10008.5.1.4.1.1.88.3 |
| Procedure Log | 1.2.840.10008.5.1.4.1.1.88.4 0 |
| Mammography CAD SR | 1.2.840.10008.5.1.4.1.1.88.5 0 |
| Chest CAD SR | 1.2.840.10008.5.1.4.1.1.88.6 5 |
| X_Ray Radiation Dose SR | 1.2.840.10008.5.1.4.1.1.88.6 7 |
| PET Image Storage | 1.2.840.10008.5.1.4.1.1.128 |
| R adio T herapy Image Storage | 1.2.840.10008.5.1.4.1.1.481. 1 |
| Radio Therapy Dose Storage | 1.2.840.10008.5.1.4.1.1.481. 2 |
| Radio Therapy Structure Set Storage | 1.2.840.10008.5.1.4.1.1.481. 3 |
| Radio Therapy Plan Storage | 1.2.840.10008.5.1.4.1.1.481. 5 |
| Radio Therapy Ion Plan Storage | 1.2.840.10008.5.1.4.1.1.481. 8 |
| Radio Therapy Ion Beams Treatment Record Storage | 1.2.840.10008.5.1.4.1.1.481. 9 |
| Radio Therapy Ion Brachy Treatment Record Storage | 1.2.840.10008.5.1.4.1.1.481. 6 |
| Radio Therapy Ion Treatment Summary Record Storage | 1.2.840.10008.5.1.4.1.1.481. 7 |

| SOP Class Name | SOP Class UID |
|-------------------------|-------------------|
| Verification (only SCP) | 1.2.840.10008.1.1 |

Somaris/7 DICOM implementation provide Private Conformance to the following DICOM V3.0 conform Private SOP Classes as both an SCU and SCP:

| SOP Class Name | SOP Class UID |
|-----------------------|---------------------|
| CSA Non-Image Storage | 1.3.12.2.1107.5.9.1 |

Association Establishment Policies

General

The existence of a job queue with network destination or an internal trigger from processing a retrieve request will activate the DICOM Storage Application. An association request is sent to the destination AE and upon successful negotiation of a Presentation Context the transfer is started

The default PDU size will be 28 KB.

Number of Associations

The Somaris/7 DICOM application initiates several associations at a time, one for each destination to which a transfer request is being processed in the active job queue list.

The Somaris/7 DICOM application is able to accept multiple associations at a time. It can handle up to 10 associations in parallel.

The number of Simultaneous DICOM associations can be configured via the Service-UI. The dialog can be found in Configuration / DICOM / General.

Asynchronous Nature

The Somaris/7 DICOM application does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

The Implementation Class UID of

□ "1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS S7VA20A".

Association Initiation Policy

If a job with network destination becomes active in the job list or a retrieve sub-operation is processed the Somaris/7 DICOM application attempts to initiate a new association for

□ DIMSE C-STORE

service operations.

Associated Real-World Activity - Send (Storage SCU)

Associated Real-World Activity - Send Image Objects to a Network Destination The associated Real-World activity is a C-STORE request initiated by an internal daemon process triggered by a job with network destination or the processing of an external C-MOVE retrieve request. If the process successfully establishes an association to a remote Application Entity, it will transfer each image one after another via the open association. If the C-STORE Response from the remote Application contains a status other than "Success" or "Warning", the association is aborted

Proposed Presentation Contexts - Send Images (Storage SCU) The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | | | |
|------------------------------------------------------------|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|------|-------------|--|--|
| Abs | tract Syntax | Transfer S | yntax | Role | Extended | | |
| Name | UID | Name List | UID List | | Negotiation | | |
| Computed Radiography Image | 1.2.840.10008.5.1.4.1.1.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None | | |
| Digital X-Ray Image for pre- sentation | 1.2.840.10008.5.1.4.1.1.1. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None | | |
| D igital X -Ray Image for pro- cessing | 1.2.840.10008.5.1.4.1.1.1. 1.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None | | |

| MammoGra- phy Image for presentation | 1.2.840.10008.5.1.4.1.1.1. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
|---------------------------------------------------------------|-----------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| MammoGra- phy Image for processing | 1.2.840.10008.5.1.4.1.1.1. 2.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
| Digital Intra- oral X-Ray Im- age for presen- tation | 1.2.840.10008.5.1.4.1.1.1. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
| Digital Intra- oral X-Ray Im- age for pro- cessing | 1.2.840.10008.5.1.4.1.1.1. 3.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |

| Computed To- mography Im- age | 1.2.840.10008.5.1.4.1.1.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
|--------------------------------------|---------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| UltraSound Multi-Frame Image | 1.2.840.10008.5.1.4.1.1.3. 1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 | SCU | None |
| Magnetic Resonance Image | 1.2.840.10008.5.1.4.1.1.4 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
| U ltra S ound Image | 1.2.840.10008.5.1.4.1.1.6. 1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |

| Secondary Capture Image | 1.2.840.10008.5.1.4.1.1.7 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
|------------------------------------------------------|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| W aveform | 1.2.840.10008.5.1.4.1.1.9. 1.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| W aveform 12-lead ECG Object | 1.2.840.10008.5.1.4.1.1.9. 1.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| Waveform General ECG Object | 1.2.840.10008.5.1.4.1.1.9. 1.2 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| Waveform Ambulatory ECG Object | 1.2.840.10008.5.1.4.1.1.9. 1.3 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| Waveform He- modynamic Object | 1.2.840.10008.5.1.4.1.1.9. 2.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| Waveform Cardiac Elec- trophysiology Object | 1.2.840.10008.5.1.4.1.1.9. 3.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| Waveform Basic Audio Object | 1.2.840.10008.5.1.4.1.1.9. 4.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCU | None |
| X-Ray Angio- graphic Image | 1.2.840.10008.5.1.4.1.1.1 2.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |

| X-Ray RadioF- luoroscopic Image | 1.2.840.10008.5.1.4.1.1.1 2.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 .70 | SCU | None |
|---------------------------------------|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Nuclear Medicine Image | 1.2.840.10008.5.1.4.1.1.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
| Basic Text SR | 1.2.840.10008.5.1.4.1.1.88 .11 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| Enhanced SR | 1.2.840.10008.5.1.4.1.1.88 .22 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| Comprehensive SR | 1.2.840.10008.5.1.4.1.1.88 .33 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| Mammo- graphy CAD SR | 1.2.840.10008.5.1.4.1.1.88 .50 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| Chest CAD SR | 1.2.840.10008.5.1.4.1.1.88 .65 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| Procedure Log | 1.2.840.10008.5.1.4.1.1.88 .40 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| X-Ray Radia- tion Dose SR | 1.2.840.10008.5.1.4.1.1.88 .67 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |

| PET Image | 1.2.840.10008.5.1.4.1.1.1 28 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
|----------------------------------|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| RT Image | 1.2.840.10008.5.1.4.1.1.4 81.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
| RT Dose | 1.2.840.10008.5.1.4.1.1.4 81.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) ^a JPEG Lossy Extended (2 & 4) ^a JPEG Lossless, Process 14 (selection value 1) | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4 .50 1.2.840.10008.1.2.4 .51 1.2.840.10008.1.2.4 | SCU | None |
| RT Structure Set | 1.2.840.10008.5.1.4.1.1.48 1.3 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| RT Beams Treatment Record | 1.2.840.10008.5.1.4.1.1.48 1.4 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| RT Plan | 1.2.840.10008.5.1.4.1.1.48 1.5 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| RT Brachy Treatment Record | 1.2.840.10008.5.1.4.1.1.48 1.6 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |

| RT Treatment Summary Record | 1.2.840.10008.5.1.4.1.1.48 1.7 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
|----------------------------------------|-----------------------------------|----------------------------------------------------------------------------------|-----------------------------------------------------------------|-----|------|
| RT Ion Plan | 1.2.840.10008.5.1.4.1.1.4 81.8 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| RT Ion Beams Treatment Record | 1.2.840.10008.5.1.4.1.1.4 81.9 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |
| CSA Non-Image | 1.3.12.2.1107.5.9.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None |

a. The Transfer Syntax used is strongly influenced by the fact of "how was the accepted Transfer Syntax at the time when the Instance was received". e.g. the Instances received with JPEG Lossy Transfer Syntaxes will not be converted and can only be sent out with the same Transfer Syntax

Note

The proposed Transfer Syntax is highly restricted for images stored internally in lossy compressed format. E.g. instances received with JPEG Loss Transfer Syntaxes will not be converted and can only be sent out with the same Transfer Syntax.

The "MOVE" destinations must be configured as Storage destinations. This would include the configuration of Transfer Syntax capabilities.

Not all the listed transfer syntaxes will be proposed all the time. For some abstract syntaxes only a list of uncompressed (UC) transfer syntaxes (one or more) will be proposed, for other abstract syntaxes also JPEG Lossless (LL) syntax will be proposed and/or a list of JPEG Lossy (LY) transfer syntaxes. The contents of this lists is configurable, e.g. UC could be configured to contain only Implicit Little Endian for instance. For further configuration details see→ page A.5–3.

It is not possible to send an image that is locally stored in a Lossy compressed format using an uncompressed or lossless transfer syntax.

Depending on the real world activity initiating the C-STORE, we have the following behaviors:

- "if the C-STORE is initiated by a user, a configuration parameter called QualityFactor(Q) will be used to decide which transfer syntax lists will be proposed. Q can take values between 0 and 100. If Q=0, only UC will be proposed. If Q = 100, UC and LL will be proposed. Else UC and LY will be proposed.
- "if the C-STORE is initiated by the C-MOVE SCP, there is another configuration parameter called Compression Types Supported (CTS) which will be used to decide what transfer syntaxes are proposed. CTS can take integer values. If CTS=0 or CTS > 3, UC will be proposed. If CTS=1, UC and LY will be proposed. If CTS = 2, UC and LL will be proposed. If CTS >= 3, UC, LL and LY will be proposed.

The compression types JPEG lossy and JPEG lossless are parameters, which are part of the Application Entity Properties configuration (storage checked). It can by reached via the Service-UI: Configuration / DICOM / Network nodes

SOP Specific Conformance Statement - Storage SCU

The DICOM images created by Somaris/7 DICOM application conform to the DICOM IOD definitions (Standard Extended IODs). But they will contain additional private elements which have to be discarded by a DICOM system when modifying the image.

The DICOM nodes are responsible for data consistency when modifying images. All unknown private attributes have to be removed upon modification!

Somaris/7 does not change private attributes if no modification is done. But during a *Save as* operation all non *syngo* defined private attributes will be removed. For association and DIMSE level time outs refer

→ page A.5–3, Configurable Parameters

Association Acceptance Policy

The Somaris/7 DICOM application attempts to accept a new association for

□ DIMSE C-ECHO

□ DIMSE C-STORE

service operations. Any Information Object transmitted on that association will be checked for conformance and stored in the database if check was successful.

Real-World Activity - Receive (Storage SCP)

Associated Real-World Activity - Receiving Images from Remote Note (Storage SCP) The daemon receiving process will accept an association and will receive any images transmitted on that association and will store the images on disk in the own database if the conformance check is performed successfully.

Proposed Presentation Contexts - Receiving Images (Storage SCP) The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | | | |
|--------------------------------------------|----------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|-------------|--|--|
| Abs | tract Syntax | Transfer S | syntax | Role | Extended | | |
| Name | UID | Name List | UID List | | Negotiation | | |
| Computed Radiography Image | 1.2.840.10008.5.1.4.1.1.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1)\ JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None | | |
| Digital X-Ray Image for presentation | 1.2.840.10008.5.1.4.1.1.1. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None | | |

| Digital X-Ray Image for processing | 1.2.840.10008.5.1.4.1.1.1. 1.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
|----------------------------------------------------|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Digital MammoGraph y X-Ray Image for presentation | 1.2.840.10008.5.1.4.1.1.1. 2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossly RLE Lossless | 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |

| Digital Mammo Graph y X-Ray Image for processing | 1.2.840.10008.5.1.4.1.1.1. 2.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
|-----------------------------------------------------------|-----------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Digital Intra- oral X-Ray Image for presentation | 1.2.840.10008.5.1.4.1.1.1. 3 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |

| Digital Intra- oral X-Ray Image for processing | 1.2.840.10008.5.1.4.1.1.1. 3.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
|---------------------------------------------------------|-----------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Computed Tomography Image | 1.2.840.10008.5.1.4.1.1.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossly RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |

| UltraSound Multi-Frame Image | 1.2.840.10008.5.1.4.1.1.3. 1 1.2.840.10008.5.1.4.1.1.3 (retired) *1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
|------------------------------------|---------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Magnetic Resonance Image | 1.2.840.10008.5.1.4.1.1.4 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |

| U ltra S ound Image | 1.2.840.10008.5.1.4.1.1.6. 1 1.2.840.10008.5.1.4.1.1.6 (retired) *1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
|--------------------------------------|---------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Secondary Capture Image | 1.2.840.10008.5.1.4.1.1.7 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossly RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
| W aveform | 1.2.840.10008.5.1.4.1.1.9. 1.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCP | None |
| Waveform General ECG Object | 1.2.840.10008.5.1.4.1.1.9. 1.2 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCP | None |
| Waveform Ambulatory ECG Object | 1.2.840.10008.5.1.4.1.1.9. 1.3 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCP | None |

| Waveform Hemodynamic Object | 1.2.840.10008.5.1.4.1.1.9. 2.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCP | None |
|-----------------------------------------------------|-----------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Waveform Cardiac Electrophysiol ogy Object | 1.2.840.10008.5.1.4.1.1.9. 3.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCP | None |
| Waveform Basic Audio Object | 1.2.840.10008.5.1.4.1.1.9. 4.1 | Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 | SCP | None |
| X-Ray Angiographic Image | 1.2.840.10008.5.1.4.1.1.1 2.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossly RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
| X-Ray RadioFluorosc opic Image | 1.2.840.10008.5.1.4.1.1.1 2.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2.1 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |

| Nuclear Medicine Image | 1.2.840.10008.5.1.4.1.1.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) JPEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.4. 90 1.2.840.10008.1.2.4. 91 1.2.840.10008.1.2.5 | SCP | None |
|------------------------------|-----------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| Basic Text SR | 1.2.840.10008.5.1.4.1.1.8 8.11 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| Enhanced SR | 1.2.840.10008.5.1.4.1.1.8 8.22 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| Comprehensiv e SR | 1.2.840.10008.5.1.4.1.1.8 8.33 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| Mammography CAD SR | 1.2.840.10008.5.1.4.1.1.8 8.55 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| Chest CAD SR | 1.2.840.10008.5.1.4.1.1.8 8.65 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |

| PET Image | 1.2.840.10008.5.1.4.1.1.1 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCP | None |
|-----------|---------------------------|---------------------------|----------------------|-----|------|
| | 28 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |
| | | JPEG Lossy Baseline | 1.2.840.10008.1.2.4. | | |
| | | (Process 1) | 50 | | |
| | | JPEG Lossy Extended (2 | | | |
| | | & 4) | 1.2.840.10008.1.2.4. | | |
| | | JPEG Lossless, Process | 51 | | |
| | | 14 | 1.2.840.10008.1.2.4. | | |
| | | (selection value 1) | 70 | | |
| | | PEG Lossless, Process | | | |
| | | 14 | 1.2.840.10008.1.2.4. | | |
| | | Non-HIER | 90 | | |
| | | JPEG 2000 Lossless | 1.2.840.10008.1.2.4. | | |
| | | JPEG 2000 Lossy | 91 | | |
| | | RLE Lossless | 1.2.840.10008.1.2.5 | | |
| RT Image | 1.2.840.10008.5.1.4.1.1.4 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCP | None |
| | 81.1 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |
| | | JPEG Lossy Baseline | 1.2.840.10008.1.2.4. | | |
| | | (Process 1) | 50 | | |
| | | JPEG Lossy Extended (2 | | | |
| | | & 4) | 1.2.840.10008.1.2.4. | | |
| | | JPEG Lossless, Process | 51 | | |
| | | 14 | 1.2.840.10008.1.2.4. | | |
| | | (selection value 1) | 70 | | |
| | | PEG Lossless, Process | | | |
| | | 14 | 1.2.840.10008.1.2.4. | | |
| | | Non-HIER | 90 | | |
| | | JPEG 2000 Lossless | 1.2.840.10008.1.2.4. | | |
| | | JPEG 2000 Lossy | 91 | | |
| | | RLE Lossless | 1.2.840.10008.1.2.5 | | |
| | | | | | |

| RT Dose | 1.2.840.10008.5.1.4.1.1.4 81.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian JPEG Lossy Baseline (Process 1) JPEG Lossy Extended (2 & 4) JPEG Lossless, Process 14 (selection value 1) PEG Lossless, Process 14 Non-HIER JPEG 2000 Lossless JPEG 2000 Lossy RLE Lossless | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2.4. 50 1.2.840.10008.1.2.4. 51 1.2.840.10008.1.2.4. 70 1.2.840.10008.1.2.2 | SCP | None |
|----------------------------------------|-----------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|------|
| RT Structure Set | 1.2.840.10008.5.1.4.1.1.4 81.3 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| RT Beams Treatment Record | 1.2.840.10008.5.1.4.1.1.4 81.4 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| RT Plan | 1.2.840.10008.5.1.4.1.1.4 81.5 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| RT Brachy Treatment Record | 1.2.840.10008.5.1.4.1.1.4 81.6 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| RT Treatment Summary Record | 1.2.840.10008.5.1.4.1.1.4 81.7 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| RT Ion Plan | 1.2.840.10008.5.1.4.1.1.4 81.8 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| RT Ion Beams Treatment Record | 1.2.840.10008.5.1.4.1.1.4 81.9 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |
| CSA Non-Image | 1.3.12.2.1107.5.9.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | None |

| Verification | 1.2.840.10008.1.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2.1 | SCP | None |
|--------------|-------------------|----------------------------------------------------------------------------------|---------------------|-----|------|

*1: US Retired and US Multiframe Retired images are converted to US images/US Multiframe images before storing them into the local database. The conversion creates new images, which implies new UIDs.

Note

With RLE Lossless Transfer Syntax and JPEG Lossless, Process14,Non_HIER Transfer Syntax the DICOM application will decompress the image before storing it into the database.

Note

JPEG 2000 decompression supported only for import in connection with COSMOS / syngo Imaging workplace.

Note

Private attributes in sequence items will be removed during import in *syngo*.

SOP Specific Conformance Statement - Receiving Images

The Somaris/7 DICOM application conforms to the Full Storage Service Class at Level 2.

Upon successfully receiving a C-STORE-RQ, the SIEMENS Somaris/7 DICOM receiver performs a quick plausibility test on the received image and available system resources. If this test succeeds, it returns the status SUCCESS, otherwise one of the following status codes is returned and the association is aborted:

☐ Refused (A700):

This error status indicates a lack of Resources (e.g. not enough disk space) on the Somaris/7 modality.

☐ Invalid Dataset (A900):

The dataset is not containing one of the attributes "Study Instance UID", "Series UID" or "SOP Instance UID", or one of them has an invalid value

☐ Processing Error (0110):

An error occurred while processing the image which makes it impossible to proceed.

Attention! Only after sending the response, the image will be saved into the database. If an error occurs during this operation, the association will be aborted. This implies that a C-STORE-RSP with status SUCCESS does not mean that the image was successfully stored into the database.

In order to confirm that the images sent were successfully stored into the database, the sending application should use Storage Commitment Service.

If an image instance is received that is identified by a SOP Instance UID which is already used by an Instance stored in the database then the received image will be discarded. The existing image is not superseded. So if a remote node sends twice the same image (same SOP Instance UID) then there will still be only one image (the first) in the database of the DICOM receiver.

The following sections will differentiate the attribute contents required for Image Viewing. The Somaris/7 DICOM application supports more formats for Storage of images than Viewing.

Image Pixel Attribute Acceptance Criterion for Grayscale Images

The Display application accepts the MONOCHROME1 and MONOCHROME2 photometric interpretation pixel format and graphic overlay with unsigned integer and 8 or 16 bits allocated. Accepted values:

| Pixel plane |
|-----------------------------------------------------------------------------------------------------|
| ☐ samples per pixel (attribute 0028, 0002) = 1 |
| □ photometric interpretation (attribute 0028,0004) = "MONOCHROME1" |
| □ photometric interpretation (attribute 0028,0004) = "MONOCHROME2" |
| □ pixel representation (attribute 0028, 0103) = 0 (unsigned) |
| ☐ bits allocated (attribute 0028, 0100) = 8, 16 |
| ☐ bits stored (attribute 0028,0101) = 8, 10, 12,14,15,16 |
| ☐ high bit (attribute 0028,0102) = 7,9,11 |
| ☐ only aspect ratio (attribute 0028,0034) 1:1 is supported |
| Overlay plane ☐ overlay type (attribute 60xx, 0040) = "G" |
| □ overlay bits allocated (attribute 60xx, 0100) = 16 |
| □ overlay bit position (attribute 60xx, 0102) = 12, 13, 14, 15 (only bits above high bit permitted) |
| ☐ Graphic Overlay will be shifted to fill Overlay Planes from Bit 12 and consecutive. |
| Overlay plane ☐ overlay type (attribute 60xx, 0040) = "G" |
| u overlay bits allocated (attribute 60xx, 0040) = 0 |
| overlay bits anocated (attribute 60xx, 6100) = 1 |
| ☐ overlay data (attribute 60xx, 3000) = supported |
| The Somaris/7 DICOM application accepts also the |
| MONOCHROME1 and MONOCHROME2 photometric inter- |
| pretation pixel format with binary 2's complement integer and |

16 bits allocated. Accepted values:

| Pixel plane □ samples per pixel (attribute 0028, 0002) = 1 □ photometric interpretation (attribute 0028,0004) = "MONOCHROME1" |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| □ photometric interpretation (attribute 0028,0004) = "MONOCHROME2" |
| □ pixel representation (attribute 0028, 0103) = 1 (signed) □ bits allocated (attribute 0028, 0100) = 16 □ bits stored (attribute 0028,0101) = 16 □ high bit (attribute 0028,0102) = 15 □ only aspect ratio (attribute 0028,0034) 1:1 is supported |
| Overlay plane □ overlay type (attribute 60xx, 0040) = "G" □ overlay bits allocated (attribute 60xx, 0100) = 1 □ overlay bit position (attribute 60xx, 0102) = 0 □ overlay data (attribute 60xx, 3000) = supported |
| For MOD LUT both the linear LUT (Rescale Slope/Intercept) and the MOD LUT SQ are supported and considered when pixel data is displayed. However there are two limitations. The MOD LUT SQ will be ignored in the following cases: |
| □ 8_Bit signed pixels |
| $\hfill \Box$ The pixel format is changed by the MOD LUT (e.g. 8 bit -> 16 bit). |
| ☐ If the MOD LUT SQ contains multiple LUTs then only the first one is used. |
| For VOI LUT also both the linear LUT (Window Center/Width) and the VOI LUT SQ are supported (VOI LUT SQ with 8 or 16 bit LUT data). |
| But if both a VOI LUT SQ and a linear MOD LUT are specified |

If the VOI LUT SQ contains multiple LUTs, then only the first one is used by default. The other VOI LUTs are selectable.

within one image, then the value of Rescale Slope is restricted to

1.

In this version the Display application supports only rectangular and circular Shutters. Images with other shutter types will be displayed without Shutter.

Image Pixel Attribute Acceptance Criterion for Color Images

| The Siemens Somaris/7 Multi_Modality Viewing applications supports the RGB color image description with the unsignation integer 24 bit color image plane pixel format. Accepted values amples per pixel (attribute 0028, 0002) = 3 □ photometric interpretation (attribute 0028,0004) = "RGB" □ pixel representation (attribute 0028, 0103) = 0 □ bits allocated (attribute 0028, 0100) = 8 □ bits stored (attribute 0028,0101) = 8 □ high bit (attribute 0028,0102) = 7 □ planar configuration (attribute 0028,0006) = 0 (pixel interleave) or 1 (plane interleave) | ec |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----|
| The Siemens Somaris/7 Multi_Modality Viewing application supports the "Palette Color" color image description wounsigned integer and 2's complement pixel format: □ samples per pixel (attribute 0028, 0002) = 1 □ photometric interpretation (attribute 0028,0004) = "PALETT COLOR" □ pixel representation (attribute 0028, 0103) = 0 □ bits allocated (attribute 0028, 0100) = 8 and bits stored (attribute 0028,0101) = 8 bits allocated (attribute 0028,0101) = 16 and bits stored (attribute 0028,0101) = 16 □ high bit (attribute 0028,0102) = 7,15 | ith |
| Both 8bit and 16bit palettes are supported - but no Segment Palette Color LUTs. | ec |

The Siemens Somaris/7 Multi-Modality Viewing application supports the YBR color image description with the unsigned integer pixel format. Accepted values:

□ samples per pixel (attribute 0028, 0002) = 3

□ photometric interpretation (attribute 0028,0004) = "YBR_FULL" or "YBR_FULL_422"

 \Box pixel representation (attribute 0028, 0103) = 0

□ bits allocated (attribute 0028, 0100) = 8 and bits stored (attribute 0028,0101) = 8

 \Box high bit (attribute 0028.0102) = 7

If Somaris/7 software is making any persistent changes on a YBR image, the resulting new image will be saved with Photometric Interpretation "RGB".

Presentation Context Acceptance Criterion -Storage SCP The Somaris/7 DICOM application will accept any number of Verification or storage SOP classes that are listed above. The number of presentation contexts accepted is limited to the maximum of 127 (DICOM limit). In case the Siemens Somaris/7 DICOM application runs out of resources, it will reject the association request.

Transfer Syntax Selection Policies - Storage SCP

The Somaris/7 DICOM application supports

- the Implicit VR Little Endian, the Explicit VR Little Endian and Explicit VR Big Endian transfer syntaxes
- the JPEG Baseline and JPEG Extended transfer syntaxes (JPEG lossy)
- the JPEG lossless Non-Hierarchical transfer syntax
- the RLE Lossless Transfer Syntax
- the JPEG 2000 Lossless and Lossy Transfer Syntax

Any proposed presentation context which including one of these transfer syntaxes will be accepted. Any proposed presentation context that does not include one of these transfer syntaxes will be rejected.

The order of preference in accepting Transfer Syntaxes within Presentation Contexts or Presentation Contexts with single Transfer Syntaxes is:

- 1. JPEG Lossy Extended
- 2. JPEG Lossless Non-hierarchical
- 3. JPEG Lossy Baseline
- 4. RLE Lossless
- 5. Explicit VR Little Endian
- 6. Explicit VR Big Endian
- 7. Implicit VR Little Endian
- 8. JPEG 2000 Lossy
- JPEG 2000 Lossless

With RLE Lossless Transfer Syntax the Somaris/7 application will decompress the image before storing it to the database.

With Implicit VR Little Endian Transfer Syntax the Somaris/7 DICOM application will remove any private attributes not known to the application. Decision on removal of a Private Element is done if there is NO entry in the attribute dictionary of the Somaris/7 DICOM application.

Thus any Explicit VR Transfer Syntax shall preferrably be used by the Storage SCUs when sending Composite Image Instances to the Somaris/7 application.

Storage Commitment AE Specification

The Somaris/7 DICOM application provides Standard Conformance to the following DICOM V3.0 SOP Classes as SCU.

| SOP Class Name | SOP Class UID |
|----------------------------------|----------------------|
| Storage Commitment Push Model | 1.2.840.10008.1.20.1 |

Association Establishment Policies

General

With a Send Job successfully completed, the DICOM application will generate a Storage Commitment Identifier which references all instances of the processed job. The Commit Request is then sent over a single opened association. Somaris/7 will wait for Status responses of the Storage Commitment Request. If the Provider accepts the Storage Commitment with Success Status, the generated Transaction UID, together with study identification data and time stamp, is kept. Depending on configuration, the association is closed when the configured time out has elapsed or a response was received before. If the association is closed before a response was received, the response is then expected on a different association. Multiple Storage Commitment Requests can be pending.

The default PDU size will be 28 KB

Number of Associations

The Somaris/7 DICOM application initiates several association at a time, one for each destination to which a transfer request is being processed in the active job queue list.

The Somaris/7 DICOM application is able to accept multiple association at a time. It can handle up to 10 associations in parallel.

Asynchronous Nature

The Somaris/7 DICOM application does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

The Somaris/7 DICOM application provides the Implementation Class UID of

"1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS S7VA20A".

Association Initiation Policy

The Somaris/7 Storage Commitment Application Entity acts as a Service Class User (SCU) for the

☐ Storage Commitment Push Model Service Class (to request commitment for storage of instances previously sent)

To do so. Somaris/7 will issue a

- □ N-ACTION DIMSE to request commitment or a
- □ N-EVENT-REPORT DIMSE to respond to a received storage commitment request and the association was closed by the remote system prior to response.

Real-World Activity - Send Storage Commitment Request

Associated Real-World Activity - Job Completed

The Somaris/7 Storage Commitment application sends the commit request (N-ACTION_RQ) message and waits for acceptance of this request (N_ACTION-RSP). After receiving this, the transaction is marked as "waiting".

Depending on a configuration value, the association will then be closed or kept open. In the first case there is another configurable timeout giving the number of hours (h) and minutes (m) (by default 1h:0m) to wait for the corresponding commit response (N_EVENT-REPORT). In the second case, this time is the (also configurable) time out for the association. For both cases, if the commit response (N-EVENT-REPORT) does not arrive during the configured time, the transaction will be marked as failed. Somaris/7 does not re-send objects because of a failed Storage Commitment result in any case.

If the commit response (N-EVENT-REPORT) received has the status of "complete - failure exists", the transaction is marked as failed, else the transaction is marked as "completed"; in both cases, a message is shown to the user.

Proposed Presentation Contexts - Job Completed The Somaris/7 Storage Commitment AE will propose Presentation Contexts as shown in the following table:

| Presentation Context Table | | | | | | |
|---------------------------------|----------------------|---------------------------|---------------------|-----|-------------|--|
| Abstract Syntax Transfer Syntax | | | | | Extended | |
| Name | UID | Name List | UID List | | Negotiation | |
| Storage | 1.2.840.10008.1.20.1 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU | None | |
| Commitment | | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | | |
| Push | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | | |
| Model | | | | | | |

SOP Specific Conformance Statement - Job Completed

Storage Commitment is supported for all the SOP class UIDs as mentioned in table \rightarrow page A.2–20.

The Referenced Study Component Sequence is not supported.

Storage Media File-Set ID and UID Attributes will not be supported in the commitment request (N-ACTION primitive) invoked by the Storage Commitment SCU.

Association Acceptance Policy

The Somaris/7 Storage Commitment AE accepts an association in this case: when acting as SCU if configured to receive N-EVENT-REPORT on a separate association.

Associated Real World Activity - Receive Storage Commitment SCU

Associated Real World Activity - Update Flags

The Somaris/7 Storage Commitment AE has sent a Storage Commitment Request and, being configured to receive response on a separate association, has closed the association, and now it gets an association request from the Storage Commitment SCP that wants to send the results. The application will wait for Storage Commitment Notification triggers. Any incoming notification will be checked for validity, that is, if the related Transaction UID is still part of the Pending Request Queue.

If the Notification is valid, the Notification Identifier is evaluated and the related Instances marked with the related status. The over all Commit Status of the higher Information Entities is derived from propagation of all Image entities included in a study.

The status flags directly affected by Storage Commitment results and indicated in the different entities of the Patient Browser list can be one of (English UI assumed):

- □ "AC" or "SC" Successful Commitment. A means archived to configured archive destination, whereas S means sent to any other destination.
- ☐ "Af" or "Sf" Commitment failed
- □ "A?" or "S?" Commitment request is sent, response is pending

In case of failure the user has to repeat the transfer of images to the Archive destination. Another Storage Commitment will be performed after sending is completed successfully.

Note: The flags A (Archived) and S (Sent) respectively only indicate the receipt of the images by remote AE. They do not indicate successful storage in the intended archive. The data may be lost if it is deleted by the sender e.g., by an auto delete mechanism and if it cannot be stored by the receiver.

Source of danger: Misleading/misinterpretation of the flags AC/SC

Flags "AC" & "SC" depict receipt and storage on hard disk on the receiver side which may be not sufficient to fulfill the regulatory requirements of long-term archiving.

Consequence: Loss of data within the required period for retention

Remedy: Sending data with the attributes AC or SC via network indicates a safe data transfer but does not fulfill the regulatory requirements of long-term archiving. Objects with the "committed" flag may be deleted by the user. Observe the regulatory requirements regarding the archiving procedure.

Accepted Presentation Contexts - Update Flags

The Somaris/7 Storage Commitment AE will accept Presentation Contexts as shown in the following table:

| Presentation Context Table | | | | | | | |
|----------------------------|-----------------------------|---------------------------|---------------------|-----|-------------|--|--|
| Abst | Role | Extended | | | | | |
| Name | Name UID Name List UID List | | | | Negotiation | | |
| Storage | 1.2.840.10008.1.20.1 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU | None | | |
| Commitment | | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | | | |
| Push | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | | | |
| Model | | | | | | | |

SOP Specific Conformance Statement - Receive Storage Commitment Response If the Commitment Response (N_EVENT_REPORT) received has the status of "complete - failure exists", the transaction is marked as failed, else the transaction is marked as "completed"; In both cases, a message is shown to the user.

The related status flags are set for the committed images in the local database.

The Somaris/7 DICOM application will NOT support the Storage Media File Set ID attributes.

Query/Retrieve AE Specification

The Query/Retrieve SCU requests that the remote SCP performs a match of all keys specified in the request against the information in its database and the identified images will be moved over a different (C-MOVE) storage association.

The Query/Retrieve SCP responds to queries of the records based on its database and images will be sent to the requesting SCU or to a different storage destination.

The Somaris/7 DICOM application provides Standard Conformance to the following DICOM V3.0 SOP Classes as SCU:

| SOP Class Name | SOP Class UID |
|-------------------------------------------------------------------|-----------------------------|
| Patient Root Query/Retrieve Information Model - FIND | 1.2.840.10008.5.1.4.1.2.1.1 |
| Patient Root Query/Retrieve Information Model - MOVE | 1.2.840.10008.5.1.4.1.2.1.2 |
| Study Root Query/Retrieve Information Model - FIND | 1.2.840.10008.5.1.4.1.2.2.1 |
| Study Root Query/Retrieve Information Model - MOVE | 1.2.840.10008.5.1.4.1.2.2.2 |
| Patient/Study Only Query/ Retrieve Information Model - FIND | 1.2.840.10008.5.1.4.1.2.3.1 |
| Patient/Study Only Query/ Retrieve Information Model - MOVE | 1.2.840.10008.5.1.4.1.2.3.2 |

Somaris/7 DICOM implementation provides Standard Conformance to the following DICOM V3.0 SOP Classes as an SCP:

| SOP Class Name | SOP Class UID |
|-------------------------------------------------------------------|-----------------------------|
| Patient Root Query/Retrieve Information Model - FIND | 1.2.840.10008.5.1.4.1.2.1.1 |
| Patient Root Query/Retrieve Information Model - MOVE | 1.2.840.10008.5.1.4.1.2.1.2 |
| Patient Root Query/Retrieve Information Model - GET | 1.2.840.10008.5.1.4.1.2.1.3 |
| Study Root Query/Retrieve Information Model - FIND | 1.2.840.10008.5.1.4.1.2.2.1 |
| Study Root Query/Retrieve Information Model - MOVE | 1.2.840.10008.5.1.4.1.2.2.2 |
| Study Root Query/Retrieve Information Model - GET | 1.2.840.10008.5.1.4.1.2.2.3 |
| Patient/Study Only Query/ Retrieve Information Model - FIND | 1.2.840.10008.5.1.4.1.2.3.1 |
| Patient/Study Only Query/ Retrieve Information Model - MOVE | 1.2.840.10008.5.1.4.1.2.3.2 |
| Patient/Study Only Query/ Retrieve Information Model - GET | 1.2.840.10008.5.1.4.1.2.3.3 |

Note

See also the Storage DICOM Conformance Statement (→ page A.2–19, Association Acceptance Policy) to compare for conformance of the C-STORE sub-operation generated by the C-MOVE and C-GET DIMSE services. Furthermore compare the supported Storage Service SOP classes described in the Storage DICOM Conformance Statement of the Modality to which the images shall be transferred to.

Association Establishment Policies

General

With the "Search..." function the query data are entered and the DICOM Query/Retrieve application is started. A query request will be sent out to one remote node that can be selected from a list of configured Query Providers and the response data will be displayed for the user. In order to provide detailed information early additional queries are sent for the more detailed levels not yet covered by the first results. Upon request (Import), the retrieval of selected items is initiated

The default PDU size used will be 28 KB.

Number of Associations

The Somaris/7 DICOM Query/Retrieve application initiates several association at a time, one for each query/retrieve request being processed.

The Somaris/7 DICOM Query/Retrieve application is able to accept multiple associations at a time. It can handle up to 10 associations in parallel.

Asynchronous Nature

The Somaris/7 DICOM Query/Retrieve AE does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

The Somaris/7 DICOM Query AE provides the Implementation Class UID of

1 "1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS S7VA20A".

Association Initiation Policy

The Query/Retrieve user interface will request the query data from the user and triggers C-FIND requests to the selected remote node. The response data will be displayed in the query UI for further data navigation.

When requesting Import of related items the browser requests the retrieve application to send a C_MOVE request to the related remote node. Images will then be received by the Storage SCP as described in the related section.

The following DIMSE-C operations are supported as SCU:

□ C-FIND

□ C-MOVE

Real-World Activity - Find SCU

Associated Real-World Activity - Find SCU "Search" The associated Real-World activity is to fill out a query form with search data and pass it as queries to the network application which issues a C-FIND over a previously built association. The remote SCP will respond with related data entries that will be passed to a browser application. When data transfer is finished the association is closed

Proposed Presentation Contexts - Find SCU

The Somaris/7 DICOM Query application will propose Presentation Contexts as shown in the following table:

| Abstract Syntax Transfer Syntax | | | | | Extended |
|------------------------------------------------|---------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------|-----|-------------|
| Name | UID | Name List | UID List | | Negotiation |
| Patient Root Query/Retrieve Model - FIND | 1.2.840.10008.5.1.4.1.2.1. 1 | DICOM Implicit VR Little Endian Transfer Syntax DICOM Explicit VR Little Endian Transfer Syntax DICOM Explicit VR Big Endian Transfer Syntax | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | See Note |

| Study Root | 1.2.840.10008.5.1.4.1.2.2. | DICOM Implicit VR Little | 1.2.840.10008.1.2 | SCU | See Note |
|----------------|----------------------------|--------------------------|---------------------|-----|----------|
| Query/Retrieve | 1 | Endian Transfer Syntax | | | |
| Model - FIND | | DICOM Explicit VR Little | 1.2.840.10008.1.2.1 | | |
| | | Endian Transfer Syntax | | | |
| | | DICOM Explicit VR Big | 1.2.840.10008.1.2.2 | | |
| | | Endian Transfer Syntax | | | |
| Patient Study | 1.2.840.10008.5.1.4.1.2.3. | DICOM Implicit VR Little | 1.2.840.10008.1.2 | SCU | See Note |
| only Query/ | 1 | Endian Transfer Syntax | | | |
| Retrieve Model | | DICOM Explicit VR Little | 1.2.840.10008.1.2.1 | | |
| - FIND | | Endian Transfer Syntax | | | |
| | | DICOM Explicit VR Big | 1.2.840.10008.1.2.2 | | |
| | | Endian Transfer Syntax | | | |

It is configurable which of the three query models are to be used by the Somaris/7 Query SCU application. If Patient and Study Root Abstract Syntaxes are configured, the C_FIND SCU will use the Patient Root Model only for C-FIND requests on PATIENT level. For all other levels it will use the STUDY Root Model.

It is highly recommended to configure only the Study Root Model if the corresponding Query SCP supports this Model. If the Query SCP does not support queries on Series Level, Patient Study Only Model should be used.

SOP Specific Conformance Statement - Find SCU

The Somaris/7 DICOM Query/Retrieve SCU supports hierarchical queries with all mandatory search keys. On each level, the unique attributes of all previous levels are also sent (values are provided by the user by selection from a list). For instance for a query using the patient root model on Series level, the Patient ID of the current selected patient and the Study Instance UID of the current selected study are included in the message. The interactive querying of attributes on IMAGE level is not supported by the Query SCU. Nevertheless, retrieval of individual Objects is possible.

The Query dialog in Somaris/7 offers search keys on different levels (Patient, Study, Series). Depending on the used Query Model (Patient Root, Study Root, Patient Study Only) the Somaris/7 DICOM Query/Retrieve SCU will execute multiple queries sequentially.

E.g. Patient Root Model:

The first query is performed on patient level with the entered patient level query keys. For each query result a new query is performed on study level with the entered study level query keys. At last for each result on study level a query is performed on series level with the entered series level keys.

The following table *Query attributes* describes the search keys for the three query models (Patient Root, Study Root and Patient Study Only) that the Somaris/7 Query/Retrieve application supports as an SCU. Matchings are either wildcard, which means that the user can supply a string containing wildcards, either universal, or means that the attribute is returned no matter what value it has

| Attribute name | Tag | Туре | Matching | user input | return value displayed |
|-------------------------------------|-------------|------|-----------------------|-----------------------------|------------------------------|
| Patient level ^a | | | | | |
| Patient name | (0010,0010) | R | wildcard ^d | enter value | yes |
| Patient ID | (0010,0020) | U | wildcard ^d | enter value | yes |
| Patient's birth date | (0010,0030) | 0 | universal (NULL) | enter value | yes |
| Patient's sex | (0010,0040) | 0 | universal (NULL) | enter value | yes |
| Number of Patient related studies | (0020,1200) | 0 | universal (NULL) | - | yes ^b |
| Number of Patient relates series | (0020,1202) | 0 | universal (NULL) | - | no |
| Number of Patient related instances | (0020,1204) | 0 | universal (NULL) | - | no |
| Study level | | | | | |
| Patient Name ^c | (0010,0010) | R | wildcard ^d | enter value | yes |
| Patient ID | (0010,0020) | R | wildcard ^d | enter value | yes |
| Patient's Birth date ^c | (0010,0030) | 0 | universal (NULL) | enter value | yes |
| Patient's Sex ^c | (0010,0040) | 0 | universal (NULL) | enter value | yes |
| Study Instance UID | (0020,000D) | U | single value | - | no |
| Study ID | (0020,0010) | R | universal (NULL) | enter value ^d | yes |
| Study Date | (0008,0020) | R | universal (NULL) | enter value ^e | yes |

| Attribute name | Tag | Туре | Matching | user input | return value displayed |
|------------------------------------|-------------|------|---------------------|-----------------------------|------------------------------|
| Study Time | (0008,0030) | R | universal (NULL) | - | yes |
| Accession Number | (0008,0050) | R | universal (NULL) | enter value ^d | yes |
| Study Description | (0008,1030) | 0 | universal (NULL) | enter value ^d | yes |
| Referring Physician's name | (0008,0090) | 0 | universal (NULL) | enter value ^d | yes |
| Name of Physician Reading Study | (0008,1060) | 0 | universal (NULL) | enter value ^d | yes |
| Modalities in Study | (0008,0061) | 0 | universal (NULL) | enter value ^d | yes |
| Storage Media File Set ID | (0008,0130) | 0 | universal (NULL) | - | no |
| Retrieve AE Title | (0008,0054) | 0 | universal (NULL) | - | no |
| Number of Study related Series | (0020,1206) | 0 | universal (NULL) | - | yes ^f |
| Number of Study related instances | (0020,1208) | 0 | universal (NULL) | - | no |
| Series level | | | | | |
| Series Instance UID | (0020,000E) | U | single value | - | no |
| Series Number | (0020,0011) | R | universal (NULL) | - | yes |
| Modality | (0008,0060) | R | universal (NULL) | enter value | yes |

| Attribute name | Tag | Туре | Matching | user input | return value displayed |
|-------------------------------------|-------------|------|---------------------|-----------------------------|------------------------------|
| Series Date | (0008,0021) | 0 | universal (NULL) | - | yes |
| Series Time | (0008,0031) | 0 | universal (NULL) | - | yes |
| Study ID | (0020,0010) | 0 | universal (NULL) | - | yes |
| Series Description | (0008,103E) | 0 | universal (NULL) | enter value ^d | yes |
| Storage Media File Set ID | (0008,0130) | 0 | universal (NULL) | - | yes |
| Retrieve AE Title | (0008,0054) | 0 | universal (NULL) | - | yes |
| Body Part Examined | (0018,0015) | 0 | universal (NULL) | enter value ^d | yes |
| Scan Options | (0018,0022) | 0 | universal (NULL) | _d | yes |
| Protocol name | (0018,1030) | 0 | universal (NULL) | - | no |
| Performing Physician | (0018,1050) | 0 | universal (NULL) | enter value ^d | yes |
| Performed procedure step start date | (0040,0244) | 0 | universal (NULL) | - | yes |
| Performed procedure step start time | (0040,0245) | 0 | universal (NULL) | - | yes |
| Request Attribute Sequence | (0040,0275) | 0 | universal (NULL) | - | yes |
| >Requested Procedure ID | (0040,1001) | 0 | universal (NULL) | - | yes |

| Attribute name | Tag | Туре | Matching | user input | return value displayed |
|------------------------------------|-------------|------|---------------------|---------------|------------------------------|
| >Scheduled Procedure ID | (0040,0009) | 0 | universal (NULL) | - | yes |
| Number of series related instances | (0020,1209) | 0 | universal (NULL) | - | yes |

| Attribute name | Tog | Type | Motobine | HOOK | roture. |
|---------------------------|-------------|------|-----------------------------------------------------|---------------|------------------------------|
| Attribute name | Tag | Type | Matching | user input | return value displayed |
| Image Level | | | | | |
| SOP Instance UID | (0008,0018) | U | single value | - | no |
| Image Number | (0020,0013) | R | universal (NULL) | - | yes |
| Storage Media File Set ID | (0008,0130) | 0 | universal (NULL) | - | no |
| Retrieve AE Title | (0008,0054) | 0 | universal (NULL) | - | no |
| Instance Date | (0008,0023) | 0 | universal (NULL) | - | no |
| Instance Time | (0008,0033) | 0 | universal (NULL) | - | no |
| Number of Frames | (0028,0008) | 0 | universal (NULL) | - | yes |
| Content Date | (0008,0023) | 0 | single value, range matching, universal | enter value | yes |
| Content Time | (0008,0033) | 0 | single value, range matching, universal | enter value | yes |
| Image comments | (0020,4000) | 0 | universal (NULL) | - | no |

| Attribute name | Tag | Type | Matching | user input | return value displayed |
|--------------------------------|-------------|------|--------------------------------------------|---------------|------------------------------|
| Referenced Request Sequence | (0040,A370) | 0 | sequence matching | - | yes |
| > Accession Number | (0008,0050) | 0 | single value, universal | - | yes |
| > Requested Procedure ID | (0040,1000) | Ο | single value, universal | - | yes |
| Concept Name Code Sequence | (0040,A043) | 0 | sequence matching | enter value | yes |
| > Code Value | (0008,0100) | 0 | single value, universal, wildcard | enter value | yes |
| > Coding Scheme Designator | (0008,0102) | 0 | single value, universal, wildcard | - | yes |
| > Coding Scheme Version | (0008,0103) | 0 | single value, universal, wildcard | - | yes |
| > Code Meaning | (0008,0104) | 0 | single value, universal, wildcard | - | yes |
| Template Identifier | (0040,DB00) | 0 | single value, universal, wildcard | - | yes |

| Attribute name | Tag | Туре | Matching | user input | return value displayed |
|---------------------------------------------------------|-------------|------|-----------------------------------------------------|---------------|------------------------------|
| Completion Flag | (0040,A491) | 0 | single value, universal, wildcard | enter value | yes |
| Verification Flag | (0040,A493) | 0 | single value, universal, wildcard | enter value | yes |
| Verifying Observer Sequence | (0040,A073) | 0 | sequence matching | enter value | yes |
| > Verifying Organization | (0008,A072) | 0 | single value, universal, wildcard | - | yes |
| > Verifying DateTime | (0008,A030) | 0 | single value, range matching, universal | enter value | yes |
| > Verifying Observer Name | (0008,A075) | 0 | single value, universal, wildcard | enter value | yes |
| > Verifying Observer Identification Code Sequence | (0040,A088) | 0 | sequence matching | | yes |
| >> Code Value | (0008,0100) | 0 | single value, universal, wildcard | - | yes |

| Attribute name | Tag | Туре | Matching | user input | return value displayed |
|--------------------------------|-------------|------|--------------------------------------------|---------------|------------------------------|
| >> Coding Scheme Designator | (0008,0102) | 0 | single value, universal, wildcard | - | yes |
| >> Coding Scheme Version | (0008,0103) | 0 | single value, universal, wildcard | - | yes |
| >> Code Meaning | (0008,0104) | 0 | single value, universal, wildcard | - | yes |

a. Only for Patient Root or Patient Study Only information model

b. Implicitly visualized in the UI if no study and series search attributes have been entered

c. Only for Study Root information model

d. Always a "*" is added to the user supplied string

e. Date range possible

f. Implicitly visualized in the UI if no series search attributes have been entered

The Find SCU interprets following status codes:

| C-FIND response status | | | | |
|------------------------|--------------------------------------------------------------------------------------------------------------------------------------|-------------------|----------------------------|--|
| Service Status | Meaning | Protocol Codes | Related Fields | |
| Refused | Out of Resources | A700 | (0000,0902) | |
| Failed | Identifier does not match SOP Class | A900 | (0000,0901) (0000,0902) | |
| | Unable to process | Cxxx | (0000,0901) (0000,0902) | |
| Cancel | Matching terminated due to Cancel request | FE00 | None | |
| Success | Matching is complete - No final Identifier is supplied | 0000 | None | |
| Pending | Matches are continuing – Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys | FF00 | Identifier | |
| | Matches are continuing – Warning that one or more Optional Keys were not supported for existence and/or matching for this identifier | FF01 | Identifier | |

Real-World Activity - Move SCU

Associated Real-World Activity - Move SCU "Import"

The operator selects a data entry in the Query UI and activates the "Import" function.

This will generate a retrieval request to the archival application which issues a C-MOVE service according to the Patient Root or Study Root query model. (The Storage Service Class Conformance Statement of the SCP describes the C-STORE service which is generated by processing the C-MOVE service.)

The transferred image data are processed as described in the storage class SCP descriptions.

The possibility to request the remote C-MOVE provider to move data to an application entity other than the C-MOVE SCU (the Somaris/7 DICOM application) is NOT used.

Proposed Presentation Contexts - Move SCU "Import"

The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | |
|------------------------------------------------|---------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------|-----|-------------|
| Abs | tract Syntax | Transfer S | Transfer Syntax | | Extended |
| Name | UID | Name List | UID List | | Negotiation |
| Patient Root Query/Retrieve Model - MOVE | 1.2.840.10008.5.1.4.1.2.1. 2 | DICOM Implicit VR Little Endian Transfer Syntax DICOM Explicit VR Little Endian Transfer Syntax DICOM Explicit VR Big Endian Transfer Syntax | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | See Note |
| Study Root Query/Retrieve Model - MOVE | 1.2.840.10008.5.1.4.1.2.2. 2 | DICOM Implicit VR Little Endian Transfer Syntax DICOM Explicit VR Little Endian Transfer Syntax DICOM Explicit VR Big Endian Transfer Syntax | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | See Note |

| Patient Study | 1.2.840.10008.5.1.4.1.2.3. | DICOM Implicit VR Little | 1.2.840.10008.1.2 | SCU | See Note |
|----------------|----------------------------|--------------------------|---------------------|-----|----------|
| Only Query/ | 2 | Endian Transfer Syntax | | | |
| Retrieve Model | | DICOM Explicit VR Little | 1.2.840.10008.1.2.1 | | |
| - MOVE | | Endian Transfer Syntax | | | |
| | | DICOM Explicit VR Big | 1.2.840.10008.1.2.2 | | |
| | | Endian Transfer Syntax | | | |

Note

C-MOVE Extended Negotiation will be NOT supported by the SCU.

C-MOVE on Patient level is not supported by the application.

SOP Specific Conformance Statement - Move SCU "Import"

At association establishment time the C-MOVE presentation context shall be negotiated. The C-STORE sub-operations must be done on a different association to transfer images to the own Storage Service Class SCP.

The Move SCU interprets following status codes:

| | C-MOVE response status | | |
|-------------------|------------------------------------------------------------|-------------------|----------------------------------------------------------|
| Service Status | Meaning | Protocol Codes | Related Fields |
| Refused | Out of Resources - Unable to calculate number of matches | A701 | (0000,0902) |
| | Out of Resources - Unable to perform suboperations | A702 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| Failed | Identifier does not match SOP Class | A900 | (0000,0901) (0000,0902) |
| Failed | Unable to process | Cxxx | (0000,0901) (0000,0902) |
| Cancel | Sub-operations terminated due to Cancel Indication | FE00 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| Warning | Sub-operations Complete - One or more Failures of Warnings | B000 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| Pending | Sub-operations are continuing | FF00 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |

| C-MOVE response status | | | | |
|------------------------|--------------------------------------------------|-------------------|----------------------------------------------------------|--|
| Service Status | Meaning | Protocol Codes | Related Fields | |
| Success | Sub-operations Complete - No Failures or Warning | 0000 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) | |

Association Acceptance Policy

The Query/Retrieve SCU and SCP establish an association by using the DICOM association services. During association establishment the Query/Retrieve application entities negotiate the supported SOP classes to exchange the capabilities of the SCU and the SCP.

The following DIMSE-C operations are supported as SCP:

- □ C-FIND
- □ C-GET
- □ C-MOVE
- □ C-FIND-CANCEL
- □ C-GET-CANCEL
- □ C-MOVE-CANCEL

Real-World Activity - Find SCP

Associated Real-World Activity - Find SCP

The associated Real-World activity is to respond to query requests to an SCU with the query model Patient Root, Study Root and Patient/Study Only. Relational retrieve operation is NOT supported. With a C-FIND-CANCEL request the running query can be cancelled at any time.

The SCP does support multiple C-FIND-requests over the same association, but not multiple C-MOVE requests.

Accepted Presentation Contexts - Find SCP

The Somaris/7 Query/Retrieve AE will accept Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | |
|----------------------------------------------------------|---------------------------------|----------------------------------------------------------------------------------|-----------------------------------------------------------------|-----|-------------|
| Abs | Abstract Syntax Transfer Syntax | | | | Extended |
| Name | UID | Name List | UID List | | Negotiation |
| Patient Root Query/Retrieve Model - FIND | 1.2.840.10008.5.1.4.1.2.1. 1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | See Note |
| Study Root Query/Retrieve Model - FIND | 1.2.840.10008.5.1.4.1.2.2. 1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | See Note |
| Patient/Study Only Query/ Retrieve Model - FIND | 1.2.840.10008.5.1.4.1.2.3. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | See Note |

Note

C-FIND Extended Negotiation will NOT be supported by the SCP.

The order of preference in accepting a Transfer syntax is:

- 1. Explicit VR Little Endian
- 2. Explicit VR Big Endian
- 3. Implicit VR Little Endian

SOP Specific Conformance Statement - Find SCP

The Somaris/7 DICOM Query/Retrieve SCP supports hierarchical gueries with all mandatory and optional search keys.

The following six notes apply to the handling of attribute Patient's Name (0010,0010) as Query/Retrieve SCP. The syntactical structure of Patient's Name (0010,0010) attribute is as follows:

<single byte group>=<ideographic group>=<phonetic group>

Each group may have up to five components, which are separated by carets "^".

- Matching of Patient's Name attribute (0010,0010) is done case-insensitive.
- 2. If a search string matches the complete value of a data base object's Patient's Name, a match will be returned.
- 3. If a search string matches an individual group (single byte, ideographic or phonetic) of a data base object's Patient's Name, a match will be returned.
- 4. If a search string matches two consecutive groups of a data base object's Patient's Name, a match will be returned.
- 5. Redundant group separators "=" or component separators "^" are treated as insignificant for matching.
- 6. Leading and trailing blanks within a component or a group of Patient's Name (0010,0010) are treated as insignificant for matching.

Except for the attribute Patient's Name (0010,0010) any other query attribute contents will be treated Case Sensitive.

With wildcard queries the symbol "?" is treated as "*" by the C-FIND-SCP application. As a consequence the query string of "?abc*" will be processed as "*abc*".

If the value for the patient level unique key "Patient ID" is not known, it may be returned with zero length. The attribute "Image Comments" will not be included in the C-FIND-RSP, if it is not set in the DB, even if it was requested as return key in the related C-FIND-RO

Usage of Storage Media File Set ID, Retrieve AE Title with C-FIND-RSP message:

- ☐ The C-FIND SCP may return the DICOM attributes StorageMediaFileSetID (0088,0130) and StorageMediaFileSetUID (0088,0140) as empty or not at all. The Storage Media File Set ID if existent can be returned at Study/Series/Image Level. Only on Image Level, the values of ONLINE, NEARLINE or OFFLINE are returned to indicate the Storage Location of the related instance.
- ☐ The C-FIND SCP may return the DICOM attributes Retrieve AE Title (0008,0054) as empty or not at all. The Retrieve AE Title if existent can only be returned at Image Level (for Patient Root and Study Root models) or Study Level (for Patient/Study Only model).

Relational Queries are NOT supported.

A remote DICOM AE can cancel the running query by sending a C-FIND-CANCEL. Matches are possibly continuing (more C-FIND response with status PENDING) until the cancel operation has completed.

The supported attributes on the various levels of the three information models are listed in the tables of the following sections.

Patient Root Information Model

| Patient level attributes, F | Patient level attributes, Patient Root Information ModelPatient level attributes | | | | |
|----------------------------------------|----------------------------------------------------------------------------------|------|--------------------------------------|--|--|
| Attribute name | Tag | Туре | Matching | | |
| Patient Name | (0010,0010) | R | single value, wildcard, universal | | |
| Patient ID | (0010,0020) | U | single value, wildcard, universal | | |
| Patient's Birth Date | (0010,0030) | 0 | single value, range, universal | | |
| Patient's Birth Time | (0010,0032) | 0 | single value, range, universal | | |
| Patient's Sex | (0010,0040) | 0 | single value, wildcard, universal | | |
| Ethnic Group | (0010,2160) | 0 | single value, wildcard, universal | | |
| Patient Comments | (0010,4000) | 0 | wildcard, universal | | |
| Number of Patient related Studies | (0020,1200) | 0 | universal | | |
| Number of Patient relates Series | (0020,1202) | 0 | universal | | |
| Number of Patient related Instances | (0020,1204) | 0 | universal | | |

| Study level attributes, Patient Root Information Model | | | | |
|--------------------------------------------------------|-------------|-----------|--------------------------------------|--|
| Attribute name | Tag | Usage SCU | Matching | |
| Study Instance UID | (0020,000D) | U | single value, list of UID | |
| Study ID | (0020,0010) | R | single value, wildcard, universal | |

| Study level at | Study level attributes, Patient Root Information Model | | | | | |
|--------------------------------------|--------------------------------------------------------|-----------|--------------------------------------|--|--|--|
| Attribute name | Tag | Usage SCU | Matching | | | |
| Study Date | (0008,0020) | R | single value, range, universal | | | |
| Study Time | (0008,0030) | R | single value, range, universal | | | |
| Accession Number | (0008,0050) | R | single value, wildcard, universal | | | |
| Referring Physician's Name | (0008,0090) | 0 | single value, wildcard, universal | | | |
| Study Description | (0008,1030) | 0 | single value, wildcard, universal | | | |
| Admitting Diagnoses Description | (0008,1080) | 0 | single value, wildcard, universal | | | |
| Patient's Age | (0010,1010) | 0 | single value, wildcard, universal | | | |
| Patient's Size | (0010,1020) | 0 | single value, universal | | | |
| Patient's Weight | (0010,1030) | 0 | single value, universal | | | |
| Additional Patient History | (0010,21B0) | 0 | wildcard, universal | | | |
| Name of Physician reading Study | (0008,1060) | 0 | single value, wildcard, universal | | | |
| Modalities in Study | (0008,0061) | 0 | multiple values, universal | | | |
| Number of Study related Series | (0020,1206) | 0 | universal | | | |
| Number of study related Instances | (0020,1208) | 0 | universal | | | |

| Series level att | ributes, Patient Ro | oot Information Mo | odel |
|--------------------------------|---------------------|--------------------|--------------------------------------|
| Attribute name | Tag | Usage SCU | Matching |
| Series Instance UID | (0020,000E) | U | single value, list of UID |
| Series Number | (0020,0011) | R | single value, universal |
| Modality | (0008,0060) | R | single value, wildcard, universal |
| Laterality | (0020,0060) | 0 | single value, wildcard, universal |
| Body Part Examined | (0018,0015) | 0 | single value, wildcard, universal |
| Patient Position | (0018,5100) | 0 | single value, wildcard, universal |
| Smallest Pixel Value in Series | (0028,0108) | 0 | single value, universal |
| Largest Pixel Value in Series | (0028,0109) | 0 | single value, universal |
| Protocol Name | (0018,1030) | 0 | single value, wildcard, universal |
| Series Date | (0008,0021) | 0 | single value, range, universal |
| Series Time | (0008,0031) | 0 | single value, range, universal |
| Series Description | (0008,103E) | 0 | single value, wildcard, universal |
| Operators Name | (0008,1070) | 0 | single value, wildcard, universal |
| Performing Physician's Name | (0008,1050) | 0 | single value, wildcard, universal |

| Series level attributes, Patient Root Information Model | | | | |
|---------------------------------------------------------|-------------|-----------|------------------------------|--|
| Attribute name | Tag | Usage SCU | Matching | |
| Performed Procedure Step Start Date | (0040,0244) | 0 | universal | |
| Performed Procedure Step Start Time | (0040,0245) | 0 | universal | |
| Number of series related Instances | (0020,1209) | 0 | universal | |
| Referenced Request Sequence | (0040,A370) | 0 | sequence matching | |
| >Requested Procedure ID | (0040,1001) | 0 | single value, uni- versal | |

| Image level attributes, Patient Root Information Model | | | | |
|--------------------------------------------------------|-------------|-----------|--------------------------------------|--|
| Attribute name | Tag | Usage SCU | Matching | |
| SOP Instance UID | (0008,0018) | U | single value, list of UID | |
| SOP Class UID | (0008,0016) | U | single value | |
| Instance Number | (0020,0013) | R | single value, universal | |
| Content Date | (0008,0023) | 0 | single value, range, universal | |
| Content Time | (0008,0033) | 0 | single value, range, universal | |
| Modality | (0008,0060) | 0 | single value, wildcard, universal | |
| Image comments | (0020,4000) | 0 | universal | |
| Concept Name Code Sequence | (0040,A043) | 0 | sequence matching | |
| > Code Value | (0008,0100) | 0 | single value, universal, wildcard | |
| > Coding Scheme Designator | (0008,0102) | 0 | single value, universal, wildcard | |
| > Coding Scheme Version | (0008,0103) | 0 | single value, universal, wildcard | |
| > Code Meaning | (0008,0104) | 0 | single value, universal, wildcard | |
| Template Identifier | (0040,DB00) | 0 | single value, universal, wildcard | |
| Completion Flag | (0040,A491) | 0 | single value, universal, wildcard | |
| Verification Flag | (0040,A493) | 0 | single value, universal, wildcard | |

| Image level attributes, Patient Root Information Model | | | | | |
|--------------------------------------------------------|-------------|-----------|-----------------------------------------------|--|--|
| Attribute name | Tag | Usage SCU | Matching | | |
| Verifying Observer Sequence | (0040,A073) | 0 | sequence matching | | |
| > Verifying Organization | (0008,A072) | 0 | single value, universal, wildcard | | |
| > Verifying DateTime | (0008,A030) | 0 | single value, range matching, universal | | |
| > Verifying Observer Name | (0008,A075) | 0 | single value, universal, wildcard | | |
| > Verifying Observer Identification Code Sequence | (0040,A088) | 0 | sequence matching | | |
| >> Code Value | (0008,0100) | 0 | single value, universal, wildcard | | |
| >> Coding Scheme Designator | (0008,0102) | 0 | single value, universal, wildcard | | |
| >> Coding Scheme Version | (0008,0103) | 0 | single value, universal, wildcard | | |
| >> Code Meaning | (0008,0104) | 0 | single value, universal, wildcard | | |

Study Root Information Model

| Study level attributes, Study Root Information Model | | | | | |
|------------------------------------------------------|-------------|------|--------------------------------------|--|--|
| Attribute name | Tag | Туре | Matching | | |
| Patient Name | (0010,0010) | R | single value, wildcard, universal | | |
| Patient ID | (0010,0020) | R | single value, wildcard, universal | | |
| Patient's Birth Date | (0010,0030) | 0 | single value, range, universal | | |
| Patient's Birth Time | (0010,0032) | 0 | single value, range, universal | | |
| Patient's Sex | (0010,0040) | 0 | single value, wildcard, universal | | |
| Patient Comments | (0010,4000) | 0 | wildcard, universal | | |
| Number of Patient related Studies | (0020,1200) | 0 | universal | | |
| Number of Patient relates Series | (0020,1202) | 0 | universal | | |
| Number of Patient related Instances | (0020,1204) | 0 | universal | | |
| Study Instance UID | (0020,000D) | U | single value, list of UIDs | | |
| Study ID | (0020,0010) | R | single value, wildcard, universal | | |
| Study Date | (0008,0020) | R | single value, range, universal | | |
| Study Time | (0008,0030) | R | single value, range, universal | | |
| Accession Number | (0008,0050) | R | single value, wildcard, universal | | |

| Study level attributes, Study Root Information Model | | | | | |
|------------------------------------------------------|-------------|------|--------------------------------------|--|--|
| Attribute name | Tag | Туре | Matching | | |
| Referring Physician's Name | (0008,0090) | 0 | single value, wildcard, universal | | |
| Study Description | (0008,1030) | 0 | single value, wildcard, universal | | |
| Admitting Diagnoses Description | (0008,1080) | 0 | single value, wildcard, universal | | |
| Patient's Age | (0010,1010) | 0 | single value, wildcard, universal | | |
| Patient's Size | (0010,1020) | 0 | single value, universal | | |
| Patient's Weight | (0010,1030) | 0 | single value, universal | | |
| Occupation | (0010,2180) | 0 | single value, wildcard, universal | | |
| Additional Patient History | (0010,21B0) | 0 | wildcard, universal | | |
| Name of Physician reading study | (0008,1060) | 0 | single value, wildcard, universal | | |
| Modalities in Study | (0008,0061) | 0 | multiple values, universal | | |
| Number of Study related Series | (0020,1206) | 0 | universal | | |
| Number of Study related Instances | (0020,1208) | 0 | universal | | |

| Series level a | ttributes, Study | Root Information M | odel |
|--------------------------------|------------------|--------------------|--------------------------------------|
| Attribute name | Tag | Туре | Matching |
| Series Instance UID | (0020,000E) | U | single value, list of UID |
| Series Number | (0020,0011) | R | single value, universal |
| Modality | (0008,0060) | R | single value, wildcard, universal |
| Laterality | (0020,0060) | 0 | single value, wildcard, universal |
| Body Part Examined | (0018,0015) | 0 | single value, wildcard, universal |
| Patient Position | (0018,5100) | 0 | single value, wildcard, universal |
| Smallest Pixel Value in Series | (0028,0108) | 0 | single value, universal |
| Largest Pixel Value in Series | (0028,0109) | 0 | single value, universal |
| Protocol Name | (0018,1030) | 0 | single value, wildcard, universal |
| Series Date | (0008,0021) | 0 | single value, range, universal |
| Series Time | (0008,0031) | 0 | single value, range, universal |
| Series Description | (0008,103E) | 0 | single value, wildcard, universal |
| Operators Name | (0008,1070) | 0 | single value, wildcard, universal |

| Series level attributes, Study Root Information Model | | | | | |
|-------------------------------------------------------|-------------|------|--------------------------------------|--|--|
| Attribute name | Tag | Туре | Matching | | |
| Performing Physician's Name | (0008,1050) | 0 | single value, wildcard, universal | | |
| Performed Procedure Step Start Date | (0040,0244) | 0 | universal | | |
| Performed Procedure Step Start Time | (0040,0245) | Ο | universal | | |
| Number of Series related Instances | (0020,1209) | 0 | universal | | |

| Image level attributes, Study Root Information Model | | | | |
|------------------------------------------------------|-------------|------|--------------------------------------|--|
| Attribute name | Tag | Туре | Matching | |
| SOP Instance UID | (0008,0018) | U | single value, list of UID | |
| SOP Class UID | (0008,0016) | U | single value | |
| Instance Number | (0020,0013) | R | single value, universal | |
| Content Date | (0008,0023) | 0 | single value, range, universal | |
| Content Time | (0008,0033) | 0 | single value, range, universal | |
| Modality | (0008,0060) | 0 | single value, wildcard, universal | |
| Image Comments | (0020,4000) | 0 | universal | |
| Referenced Request Sequence | (0040,A370) | 0 | sequence matching | |
| > Accession Number | (0008,0050) | 0 | single value, universal | |

| Image level at | Image level attributes, Study Root Information Model | | | |
|------------------------------------------------------|------------------------------------------------------|------|-----------------------------------------------|--|
| Attribute name | Tag | Туре | Matching | |
| > Requested Procedure ID | (0040,1000) | 0 | single value, universal | |
| Concept Name Code Sequence | (0040,A043) | 0 | sequence matching | |
| > Code Value | (0008,0100) | 0 | single value, universal, wildcard | |
| > Coding Scheme Designator | (0008,0102) | 0 | single value, universal, wildcard | |
| > Coding Scheme Version | (0008,0103) | 0 | single value, universal, wildcard | |
| > Code Meaning | (0008,0104) | 0 | single value, universal, wildcard | |
| Template Identifier | (0040,DB00) | 0 | single value, universal, wildcard | |
| Completion Flag | (0040,A491) | 0 | single value, universal, wildcard | |
| Verification Flag | (0040,A493) | 0 | single value, universal, wildcard | |
| Verifying Observer Sequence | (0040,A073) | Ο | sequence matching | |
| > Verifying Organization | (0008,A072) | 0 | single value, universal, wildcard | |
| > Verifying DateTime | (0008,A030) | 0 | single value, range matching, universal | |
| > Verifying Observer Name | (0008,A075) | 0 | single value, universal, wildcard | |
| > Verifying Observer Identification Code Sequence | (0040,A088) | 0 | sequence matching | |

| Image level attributes, Study Root Information Model | | | | |
|------------------------------------------------------|-------------|------|--------------------------------------|--|
| Attribute name | Tag | Type | Matching | |
| >> Code Value | (0008,0100) | 0 | single value, universal, wildcard | |
| >> Coding Scheme Designator | (0008,0102) | 0 | single value, universal, wildcard | |
| >> Coding Scheme Version | (0008,0103) | Ο | single value, universal, wildcard | |
| >> Code Meaning | (0008,0104) | 0 | single value, universal, wildcard | |

Patient Study Only Information Models

| Patient instance | level, Patient Study (| Only Information | on Model |
|-------------------------------------|------------------------|------------------|--------------------------------------|
| Attribute name | Tag | Туре | Matching |
| Patient Name | (0010,0010) | R | single value, wildcard, universal |
| Patient ID | (0010,0020) | U | single value, wildcard, universal |
| Patient's Birth Date | (0010,0030) | 0 | single value, range, universal |
| Patient's Birth Time | (0010,0032) | 0 | single value, range, universal |
| Patient's Sex | (0010,0040) | 0 | single value, wildcard, universal |
| Ethnic Group | (0010,2160) | 0 | single value, wildcard, universal |
| Patient comments | (0010,4000) | 0 | wildcard, universal |
| Number of Patient related Studies | (0020,1200) | 0 | universal |
| Number of Patient relates Series | (0020,1202) | 0 | universal |
| Number of Patient related Instances | (0020,1204) | 0 | universal |

| Study level attributes, Patient Study Only Information Model | | | | | | |
|--------------------------------------------------------------|-------------|---|--------------------------------------|--|--|--|
| Attribute name Tag Type Matching | | | | | | |
| Study Instance UID | (0020,000D) | U | single value, list of UIDs | | | |
| Study ID | (0020,0010) | R | single value, wildcard, universal | | | |

| Study level attrib | Study level attributes, Patient Study Only Information Model | | | | |
|------------------------------------|--------------------------------------------------------------|------|--------------------------------------|--|--|
| Attribute name | Tag | Туре | Matching | | |
| Study Date | (0008,0020) | R | single value, range, universal | | |
| Study Time | (0008,0030) | R | single value, range, universal | | |
| Accession Number | (0008,0050) | R | single value, wildcard, universal | | |
| Referring Physician's Name | (0008,0090) | 0 | single value, wildcard, universal | | |
| Study Description | (0008,1030) | 0 | single value, wildcard, universal | | |
| Admitting Diagnoses Description | (0008,1080) | 0 | single value, wildcard, universal | | |
| Patient's Age | (0010,1010) | 0 | single value, wildcard, universal | | |
| Patient's Size | (0010,1020) | 0 | single value, universal | | |
| Patient's Weight | (0010,1030) | 0 | single value, universal | | |
| Occupation | (0010,2180) | 0 | single value, wildcard, universal | | |
| Additional Patient History | (0010,21B0) | 0 | wildcard, universal | | |
| Name of Physician reading Study | (0008,1060) | 0 | single value, wildcard, universal | | |
| Modalities in Study | (0008,0061) | 0 | multiple values, universal | | |
| Number of Study related Series | (0020,1206) | 0 | universal | | |

| Study level attributes, Patient Study Only Information Model | | | | |
|--------------------------------------------------------------|-------------|---|-----------|--|
| Attribute name Tag Type Matching | | | | |
| Number of study related Instances | (0020,1208) | 0 | universal | |

| | C-FIND-RSP outes: | message | will | contain | the | following |
|-----|----------------------|---------------|------|------------|--------|-----------|
| □ s | pecific Charact | er Set (000 | 8,00 | 05) (If th | ere is | s a |
| sp | pecific characte | er set in use | e). | | | |

Note

- □ Query/Retrieve Level (0008,0052) from the C FIND RQ.
- ☐ Retrieve AE Title (0008,0054) at study, series and image level.

This value is a list of AE titles from which the images can be retrieved. Will be NULL except for the lowest level of the query model (Image level for Patient Root or Study Root and Study level for Patient/Study Only).

- ☐ Storage-Media File-set ID (0088,0130) at level study, series and image. If Storage-Media File-set ID is not present a NULL value will be returned.
- □ attributes requested by C_FIND_RQ and supported by the SCP

Note

See also → page A.2–71, SOP Specific Conformance Statement - Find SCP

The Find SCP returns following status codes:

| C-FIND return status | | | | | |
|----------------------|--------------------------------------------------------------------------------------------------------------------------------------|-------------------|----------------------------|--|--|
| Service Status | Meaning | Protocol Codes | Related Fields | | |
| Refused | Out of Resources | A700 | (0000,0902) | | |
| Failed | Identifier does not match SOP Class | A900 | (0000,0901) (0000,0902) | | |
| | Unable to process | C001 | (0000,0901) (0000,0902) | | |
| Cancel | Matching terminated due to Cancel request | FE00 | None | | |
| Success | Matching is complete - No final Identifier is supplied | 0000 | None | | |
| Pending | Matches are continuing – Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys | FF00 | Identifier | | |
| | Matches are continuing – Warning that one or more Optional Keys were not supported for existence and/or matching for this identifier | FF01 | Identifier | | |

Real World Activity - Get SCP

Associated Real-World Activity - Get SCP

The associated Real-World activity is to respond to retrieve requests initiated from a foreign SCU. The SCP supports the query model Patient Root, Study Root and Patient/Study Only. The Storage Service Class Conformance Statement describes the C-STORE service which is generated by C-GET service. Relational retrieve operation is NOT supported.

Multiple C-GET requests over the same association are NOT supported.

Proposed Presentation Contexts - GET SCP

The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | |
|-------------------------|---------------------------------|---------------------------|---------------------|------|-------------|
| Abst | Abstract Syntax Transfer Syntax | | | Role | Extended |
| Name | UID | Name List | UID List | | Negotiation |
| Patient Root | 1.2.840.10008.5.1.4.1.2.1. | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCP | See Note |
| Query/Retrieve | 3 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| Model - GET | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |
| Study Root | 1.2.840.10008.5.1.4.1.2.2. | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCP | See Note |
| Query/Retrieve | 3 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| Model - GET | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |
| Patient/Study | 1.2.840.10008.5.1.4.1.2.3. | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCP | See Note |
| Only Query/ | 3 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| Retrieve Model - GET | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |

Note

C-GET Extended Negotiation will be NOT supported by the SCP.

The order of preference in accepting a Transfer syntax is:

- 1. Explicit VR Little Endian
- 2. Explicit VR Big Endian
- 3. Implicit VR Little Endian

SOP Specific Conformance Statement - Get SCP

At association establishment time the C-GET presentation context must be negotiated along with the C-STORE sub-operations which must be accomplished on the same association as the C-GET operation. Relational retrieve operation is NOT supported.

All unique keys have to be supplied according to the selected Query/Retrieve Level. The related tables in the C-FIND SCP section will give information about "U" marked key attributes.

Note

In DICOM wildcard queries the symbol '?' is treated as '*' by Find SCP.

So a wildcard query with "?abc*" is actually treated as "*abc*"

The Get SCP returns following status codes:

| | C-MOVE return status | | | | | |
|-------------------|------------------------------------------------------------|-------------------|----------------------------------------------------------|--|--|--|
| Service Status | Meaning | Protocol Codes | Related Fields | | | |
| Refused | Out of Resources - Unable to calculate number of matches | A701 | (0000,0902) | | | |
| | Out of Resources - Unable to perform suboperations | A702 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) | | | |
| Failed | Identifier does not match SOP Class | A900 | (0000,0901) (0000,0902) | | | |
| | Unable to process | C001 | (0000,0901) (0000,0902) | | | |
| Cancel | Sub-operations terminated due to Cancel Indication | FE00 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) | | | |
| Warning | Sub-operations Complete - One or more Failures of Warnings | B000 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) | | | |
| Pending | Sub-operations are continuing | FF00 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) | | | |
| Success | Sub-operations Complete - No Failures or Warning | 0000 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) | | | |

Real-World Activity - Move SCP

Associated Real-World Activity - Move SCP

The associated Real-World activity is to respond to retrieve requests to an SCU. The SCP supports the query model Patient Root, Study Root and Patient/Study Only. The Storage Service Class Conformance Statement describes the C-STORE service which is generated by the C-MOVE service. Relational retrieve operation is NOT supported.

Multiple C-MOVE requests over the same association are NOT supported.

Accepted Presentation Contexts - Move SCP

The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | |
|----------------------------------------------------------|---------------------------------------------------------------|------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|------|-------------|
| Abst | tract Syntax | Transfer S | yntax | Role | Extended |
| Name | UID | Name List | UID List | | Negotiation |
| Patient Root Query/Retrieve Model - MOVE Study Root | 1.2.840.10008.5.1.4.1.2.1. 2 1.2.840.10008.5.1.4.1.2.2. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian Implicit VR Little Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 1.2.840.10008.1.2 | SCP | See Note |
| Query/Retrieve Model - MOVE | 2 | Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | 001 | occ Note |
| Patient/Study Only Query/ Retrieve Model - MOVE | 1.2.840.10008.5.1.4.1.2.3. | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCP | See Note |

Note

C-MOVE Extended Negotiation will be NOT supported by the SCP.

The order of preference in accepting a Transfer syntax is:

- 1. Explicit VR Little Endian
- 2. Explicit VR Big Endian
- 3. Implicit VR Little Endian

SOP Specific Conformance Statement - Move SCP

At association establishment time the C-MOVE presentation context shall be negotiated. The C-STORE sub-operations is done on a different association, specified in the C-MOVE request, to transfer images to another SCP of the Storage Service Class. Relational retrieve operation is NOT supported.

All unique keys have to be supplied according to the selected Query/Retrieve Level. The related tables in the C-FIND SCP section will give information about "U" marked key attributes.

Note

In DICOM wildcard queries the symbol '?' is treated as '*' by Find SCP.
So a wildcard query with "?abc*" is actually treated as "*abc*"

The Move SCP returns following status codes:

| | C-MOVE return status | | |
|-------------------|------------------------------------------------------------|-------------------|----------------------------------------------------------|
| Service Status | Meaning | Protocol Codes | Related Fields |
| Refused | Out of Resources - Unable to calculate number of matches | A701 | (0000,0902) |
| | Out of Resources - Unable to perform suboperations | A702 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| | Identifier does not match SOP Class | A900 | (0000,0901) (0000,0902) |
| Failed | Unable to process | C001 | (0000,0901) (0000,0902) |
| Cancel | Sub-operations terminated due to Cancel Indication | FE00 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| Warning | Sub-operations Complete - One or more Failures of Warnings | B000 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| Pending | Sub-operations are continuing | FF00 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |
| Success | Sub-operations Complete - No Failures or Warning | 0000 | (0000,1020) (0000,1021) (0000,1022) (0000,1023) |

Print AE Specification

The print management SCU (HCS) invokes print management DIMSE services to transfer images from the local AE to the remote SCP AE to print the images with the defined film format and size on a selected network DICOM hardcopy printer. See DICOM part 4 annex H. This is done in a "full-page" print mode.

Somaris/7 DICOM implementation provides Standard Conformance to the following DICOM V3.0 Basic Grayscale Print Management Meta SOP Classes as an SCU:

| Basic Gray Scale Print Management Meta SOP-Classes | | | | |
|-------------------------------------------------------------|------------------------|--|--|--|
| SOP Class Name | SOP Class UID | | | |
| Basic Grayscale Print Management Meta SOP Class | 1.2.840.10008.5.1.1.9 | | | |
| Basic Film Session SOPClass | 1.2.840.10008.5.1.1.1 | | | |
| - Basic Film Box SOP Class | 1.2.840.10008.5.1.1.2 | | | |
| Basic Grayscale Image Box SOP Class | 1.2.840.10008.5.1.1.4 | | | |
| - Printer SOP Class | 1.2.840.10008.5.1.1.16 | | | |
| - Print Job SOP Class | 1.2.840.10008.5.1.1.14 | | | |
| - Presentation LUT SOP Class | 1.2.840.10008.5.1.1.23 | | | |

| Color Print Management Meta SOP-Classes | | | | |
|-----------------------------------------------------------|-------------------------|--|--|--|
| SOP Class Name | SOP Class UID | | | |
| Basic Color Print Management Meta SOP Class | 1.2.840.10008.5.1.1.18 | | | |
| Basic Film Session SOP Class | 1.2.840.10008.5.1.1.1 | | | |
| - Basic Film Box SOP Class | 1.2.840.10008.5.1.1.2 | | | |
| Basic Color Image BoxSOP Class | 1.2.840.10008.5.1.1.4.1 | | | |
| - Printer SOP Class | 1.2.840.10008.5.1.1.16 | | | |
| - Print Job SOP Class | 1.2.840.10008.5.1.1.14 | | | |

Association Establishment Policies

General

Whenever a film is completely set up and printed by command or automatism, the job is prepared for processing. As soon as the queue is ready to process the job is activated and carried out according to the processing data. The related Print application will initiate an association to the print destination and process the printing of the related information.

The default PDU size will be 28 KB.

Number of Associations

The Somaris/7 DICOM Print application initiates one association at a time for print device configured.

Asynchronous Nature

The Somaris/7 DICOM implementation does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

Implementation Class UID of

□ "1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS_S7VA20A".

Association Initiation Policy

Triggered by the Print job queue the Print Management SCU establishes an association by using the DICOM association services. With the help of the N-GET request for the printer SOP Class the Status is determined before printing.

In case no problem is encountered with the N-CREATE/N-SET Services for the related Basic Print SOP Classes the film sheet is set up for printing and the image(s) is(are) transferred to the printer device.

After the last film is printed from the queue, the Print application will leave open the association for another 60 seconds. If a new film job is ready for printing within this time limit, the job will be processed immediately over the still open association. If there is no new job, the association is closed when the time_out has elapsed. This is done to optimize automatic printing.

During the "idle time" (no open association to printer) the Print application will issue a cyclic camera status request (using N-GET of Printer SOP Class) every 5 minutes.

Real-World Activity

Associated Real-World Activity - Printing a Printer Job Queue Entry Whenever a film sheet is prepared by the user, it is forwarded to the Printer Job queue. As soon as the associated Printer device is available the job is activated and an association is set up.

The film sheet is internally processed, converted to a Standard/ 1-1 page and then the page image is sent. Status is controlled by awaiting any N-EVENT message all through the transfer until the last image or film sheet is sent.

If the response from the remote application contains a status other than Success or Warning the association is aborted.

Proposed Presentation Contexts (Presentation Context Table) The Somaris/7 DICOM application will propose Presentation Contexts as shown in the following table:

| Presentation | context - | Print SCII | |
|--------------|-----------|------------|--|
| | | | |

| Abst | Abstract Syntax | | Transfer Syntax | | Transfer Syntax | | Extended |
|----------------------------------------------------------------|-------------------------|----------------------------------------------------------------------------------|-----------------------------------------------------------------|-----|-----------------|--|----------|
| Name | UID | Name List | UID List | | Negotiation | | |
| Basic Grayscale Print Management Meta SOP class | 1.2.840.10008.5.1.1.9 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None | | |
| Basic Color Print Management Meta SOP class | 1.2.840.10008.5.1.1.18 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None | | |
| Basic film session SOP class | 1.2.840.10008.5.1.1.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None | | |
| Basic film box SOP class | 1.2.840.10008.5.1.1.2 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None | | |
| Basic grayscale image box SOP class | 1.2.840.10008.5.1.1.4 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None | | |
| Basic color image box SOP class | 1.2.840.10008.5.1.1.4.1 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.2 | SCU | None | | |

| Printer SOP | 1.2.840.10008.5.1.1.16 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU | None |
|---------------|------------------------|---------------------------|---------------------|-----|------|
| class | | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |
| Print Job SOP | 1.2.840.10008.5.1.1.14 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU | None |
| class | | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |
| Presentation | 1.2.840.10008.5.1.1.23 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU | None |
| LUT SOP class | | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | | |
| | | Explicit VR Big Endian | 1.2.840.10008.1.2.2 | | |

SOP Specific Conformance Statement - Meta SOP Classes

The Somaris/7 DICOM Print Management SCU conforms to the DICOM Basic Grayscale Print Management Meta SOP Class and Basic Color Print Management Meta SOP Class.

and Basic Color Print Management Meta SOP Class.

The application uses a setting platform to define the properties of the connected DICOM SCP, e.g.:

maximum number of print jobs in the queue
maximum number of print copies
supported film sizes of the connected DICOM SCP
supported film formats of the DICOM SCP
lookup table definition

The printing is only suspended in the case of a failure return status of the SCP.

Basic Film Session SOP Class

The Basic Film Session information object definition describes all the user defined parameter which are common for all the films of a film session. The Basic Film Session refers to one or more Basic Film Boxes and that are printed on one hardcopy printer.

The Somaris/7 DICOM Print application supports the following DIMSE Service Elements for the Basic Film Session SOP class as SCU:

□ N-CREATE

□ N-DELETE

The Basic Film Session SOP class N-CREATE-RQ (SCU) uses following attributes:

| Attribute name | Tag | Usage SCU | Supported Values |
|------------------|-------------|-----------|----------------------------------|
| Number of Copies | (2000,0010) | U | 1 |
| Medium Type | (2000,0030) | U | BLUE FILM CLEAR FILM PAPER |
| Film Destination | (2000,0040) | U | MAGAZINE PROCESSOR |

The number of copies sent to the DICOM printer is always 1, the job is sent n times for n copies.

The Affected SOP Instance UID received with N-CREATE-RSP message from SCP will be kept internally and used for later requests (e.g. N-DELETE-RQ) on the Basic Film Session SOP Class - see table below:

| Attribute name | Tag | Source of information |
|----------------------------|----------------|---------------------------------------------------------------------|
| Requested SOP Instance UID | (0000,1000) | Affected SOP Instance UID of N- CREATE-RSP on Basic Film Session |
| | -> (0000,1001) | |

The N-DELETE-RQ on the Basic Film Session SOP Class is used to delete the complete Basic Film Session SOP Instance hierarchy.

The Basic Film Session SOP class interprets the following status codes (from N-CREATE-RSP, N-DELETE-RSP messages):

| Service Status | Meaning | Protocol Codes |
|-------------------------------------------------------------------------|--------------------------------------------------------------|-----------------------|
| Failure Film session SOP instances hid does not contain film box SOP in | | C600 |
| | Unable to create print job, print queue is full | C601 |
| | Image size is larger than images box size | C603 |
| Warning | Memory allocation not supported | B600 |
| | Film session printing is not supported | B601 |
| | Film box does not contain image box (empty page) | B602 |
| Success | Film belonging to the film session are accepted for printing | 0000 |

Basic Film Box SOP Class

The Basic Film Box information object definition describes all the user defined parameter of one film of the film session. The Basic Film Box information description defines the presentation parameters which are common for all images on a given sheet of film.

The Basic Film Box refers to one or more Image Boxes.

Supported as SCU are:

□ N-CREATE

□ N-ACTION

□ N-DELETE

The Basic Film Box SOP class N-CREATE-RQ message uses following attributes (the used values for each attribute depend on the DICOM Printer configuration within the Somaris/7 DICOM print management SCU):

| Attribute name | Tag | Usage SCU | Supported Values |
|-------------------------------------|-------------|-----------|---------------------------|
| Image Display Format | (2010,0010) | М | STANDARD\C-R |
| Referenced Film Session Sequence | (2010,0500) | М | |
| >Referenced SOP Class UID | (0008,1150) | М | 1.2.840.10008.5.1. 1.1 |
| >Referenced SOP Instance UID | (0008,1155) | М | |
| Film Orientation | (2010,0040) | М | PORTRAIT LANDSCAPE |

| Attribute name | Tag | Usage SCU | Supported Values |
|-----------------------------------------|-------------|-----------|-----------------------------------------------------------------------------------------------------|
| Film Size ID | (2010,0050) | М | 8INX10IN 10INX12IN 10INX14IN 11INX14IN 14INX14IN 14INX17IN 24CMX24CM 24CMX30CM |
| Magnification Type | (2010,0060) | М | BILINEAR CUBIC NONE REPLICATE |
| Border Density | (2010,0100) | U | BLACK WHITE |
| Max Density | (2010,0130) | U | 0 < Value |
| Min Density | (2010,0120) | U | 0 < Value < 50 |
| Illumination | (2010,015E) | U | 0 < Value Required if Presentation LUT is present |
| Reflective Ambient Light | (2010,0160) | U | 0 < Value Required if Presentation LUT is present |
| Referenced Presentation LUT Sequence | (2050,0500) | U | |

For Page Mode printing the Image Display format used is Standard\1,1. For Image Mode Printing the Image Display format used is Standard\C,R where C is the number of Columns and R is the number of Rows as specified in the Hardcopy Layout.

The N-CREATE-RSP message from the Print SCP includes the Referenced Image Box Sequence with its SOP Class / Instance UID pairs which will be kept internally to be further used for the subsequent Basic Image Box SOP Class N-SET-RQ messages.

When all Image Boxes (including parameters) for the film sheet have been set, the Somaris/7 DICOM print manager will issue a N-ACTION-RQ message with the SOP Instance UID of the Basic Film Box (returned in N-CREATE-RSP of Basic Film Box SOP class) and the Action Type ID of 1.

The affected SOP Instance UID received with N-CREATE-RSP message will be kept internally and used for later requests (e.g. N-DELETE-RQ) on the Basic Film Box SOP Class - see table below:

| Attribute name | Tag | Source of information |
|----------------------------|----------------|-------------------------------------------------------------|
| Requested SOP Instance UID | (0000,1000) | Affected SOP Instance UID of N-CREATE-RSP on Basic Film Box |
| | -> (0000,1001) | |

The Basic Film Box SOP class interprets the following status codes:

| Service Status | Meaning | Protocol Codes |
|----------------|-------------------------------------------------------------------------|-----------------------|
| Failure | Unable to create print job; print queue is full | C602 |
| | Image size is larger than image box size | C603 |
| Warning | Film box does not contain image box (empty page) | B603 |
| | Requested MinDensity or MaxDensity outside of printer's operating range | B605 |
| Success | Film accepted for printing | 0000 |

Basic Grayscale Image Box SOP Class

The Basic Grayscale Image Box information object definition is the presentation of an image and image related data in the image area of a film. The Basic Image Box information describes the presentation parameters and image pixel data which apply to a single image of a sheet of film.

The Grayscale Image Box SOP class uses only the N-SET-RQ with the following attributes:

| Attribute name | Tag | Usage SCU | Supported Values |
|-----------------------------------|-------------|-----------|---------------------|
| Image Position | (2020,0010) | М | 1 |
| Basic Grayscale Image Sequence | (2020,0110) | M | |
| >Samples Per Pixel | (0028,0002) | М | 1 |
| >Photometric Interpretation | (0028,0004) | М | MONOCHROME2 |
| >Rows | (0028,0010) | М | |
| >Columns | (0028,0011) | М | |
| >Pixel Aspect Ratio | (0028,0034) | М | |
| >Bits Allocated | (0028,0100) | М | 8, 16 |
| >Bits Stored | (0028,0101) | М | 8, 12 |
| >High Bit | (0028,0102) | М | 7, 11 |
| >Pixel Representation | (0028,0103) | М | 0 |
| >Pixel Data | (7FE0,0010) | М | |

The Grayscale Image Box SOP class interprets following status codes:

| Service Status | Meaning | Protocol Codes |
|----------------|-------------------------------------------------------------------------|----------------|
| Failure | Image contains more pixel than printer can print in Image box | C603 |
| | Insufficient memory in printer to store the image | C605 |
| Warning | Requested MinDensity or MaxDensity outside of printer's operating range | B605 |
| Success | | 0000 |

Basic Color Image Box SOP Class

The Basic Color Image Box information object definition is the presentation of an image and image related data in the image area of a film. The Basic Image Box information describes the presentation parameters and image pixel data which apply to a single image of a sheet of film.

The Color Image Box SOP class uses only the N-SET-RQ with the following attributes:

| Attribute name | Tag | Usage SCU | Supported Values |
|-----------------------------|-------------|-----------|---------------------|
| Image Position | (2020,0010) | М | 1 |
| Basic Color Image Sequence | (2020,0111) | М | |
| >Samples Per Pixel | (0028,0002) | М | 3 |
| >Photometric Interpretation | (0028,0004) | М | RGB |
| >Planar Configuration | (0028,0006) | М | 0 |
| >Rows | (0028,0010) | М | |
| >Columns | (0028,0011) | М | |
| >Pixel Aspect Ratio | (0028,0034) | М | |
| >Bits Allocated | (0028,0100) | М | 8 |
| >Bits Stored | (0028,0101) | М | 8 |
| >High Bit | (0028,0102) | М | 7 |
| >Pixel Representation | (0028,0103) | М | 0 |
| >Pixel Data | (7FE0,0010) | М | |

The Color Image Box SOP class interprets following status codes:

| Service Status | Meaning | Protocol Codes | |
|----------------|---------------------------------------------------------------|----------------|--|
| Failure | Image contains more pixel than printer can print in Image box | C603 | |
| | Insufficient memory in printer to store the image | C605 | |
| Warning | Image size larger than image box size. | B604 | |
| Success | | 0000 | |

Presentation LUT SOP Class

The objective of the Presentation LUT is to realize image hardcopy printing tailored for specific modalities, applications, and user preferences.

The output of the Presentation LUT is Presentation Values (P-Values). P-Values are approximately related to human perceptual response. They are intended to facilitate common input for hardcopy. P-Values are intended to be independent of the specific class or characteristics of the hardcopy device.

Attributes of the N CREATE RQ on the Presentation LUT SOP Class Attribute name Tag **Usage SCU** Supported Values U

(2050.0020)

The affected SOP Instance UID received with N-CREATE-RSP message wil be kept internally and is used for later requests on the Basic Film Box (N-CREATE-RQ) and on the Presentation LUT (N-DELETE-RQ) - see below.

| Attribute name | Tag | Source of information |
|----------------------------|----------------|---------------------------------------------------------------|
| Requested SOP Instance UID | (0000,1000) | Affected SOP Instance UID of N-CREATE-RSP on Presentation LUT |
| | -> (0000,1001) | |

The Presentation LUT SOP class interprets the following status codes:

| Meaning | Protocol Codes |
|----------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------|
| Requested Min Density or Max Density outside the HCD soperating range. HCD will use its respective minimum or maximum density value instead. | B605 |
| Presentation LUT successfully created | 0000 |
| | Requested Min Density or Max Density outside the HCD soperating range. HCD will use its respective minimum or maximum density value instead. |

Presentation LUT Shape

IDENTITY

Printer SOP Class

The Printer SOP Class is the possibility to monitor the status of the hardcopy printer in a synchronous and an asynchronous way.

The SCU uses the mandatory N-EVENT Report DIMSE service to monitor the changes of the printer status in an asynchronous way.

It can directly ask the Printer (SCP) for its status or can receive Events from the Print SCP asynchronously:

■ N GET as SCU

■ N EVENT REPORT as SCU

In both cases the following information is supported:

| Event type name | Event | Attributes | Tag | Usage SCU |
|-----------------|-------|---------------------|-------------|-----------|
| Normal | 1 | | | |
| Warning | 2 | Printer Status Info | (2110,0020) | U |
| Failure | 3 | Printer Status Info | (2110,0020) | U |

| Attribute name | Tag | Usage SCP | supported values |
|---------------------|-------------|-----------|---------------------------------|
| Printer Status | (2110,0010) | М | NORMAL FAILURE WARNING |
| Printer Status Info | (2110,0020) | М | see \rightarrow page A.2–118. |

For a detailed description of how Somaris/7 reacts to the various messages please refer to \rightarrow page A.2–118, *DICOM Print SCU - detailed status displays*.

Print Joh SOP Class

The Print Job SOP Class is the possibility to monitor the execution of the print process.

The Somaris/7 DICOM Print application supports the optional N-EVENT-REPORT DIMSE service to receive the changes of the print job status in an asynchronous way.

It can receive events from the Print SCP asynchronously:

□ N-EVENT-REPORT

Note

Somaris/7 does not support receiving N-EVENT from the camera during print sessions, normally this is configurable in the camera

The following information is supported:

| Event type name | Event | Attributes | Tag | Usage SCU |
|-----------------|-------|--------------------------|-------------|----------------------------------------------------------------|
| Pending | 1 | Execution Status Info | (2100,0030) | U |
| | | Print Job ID | (2100,0010) | - (Print Queue Management SOP Class not supported) |
| | | Film Session Label | (2000,0050) | U |
| | | Printer Name | (2110,0030) | U |
| Printing | 2 | Execution Status Info | (2100,0030) | U |
| | | Print Job ID | (2100,0010) | - (Print Queue Management SOP Class not supported) |
| | | Film Session Label | (2000,0050) | U |
| | | Printer Name | (2110,0030) | U |

| Event type name | Event | Attributes | Tag | Usage SCU |
|-----------------|-------|-----------------------|-------------|----------------------------------------------------------------|
| Done | 3 | Execution Status Info | (2100,0030) | U |
| | | Print Job ID | (2100,0010) | - (Print Queue Management SOP Class not supported) |
| | | Film Session Label | (2000,0050) | U |
| | | Printer Name | (2110,0030) | U |
| Failure | 4 | Execution Status Info | (2100,0030) | U |
| | | Print Job ID | (2100,0010) | - (Print Queue Management SOP Class not supported) |
| | | Film Session Label | (2000,0050) | U |
| | | Printer Name | (2110,0030) | U |

For a detailed description of how Somaris/7 reacts to the various messages please refer to \rightarrow page A.2–118, *DICOM Print SCU - detailed status displays*.

DICOM Print SCU - detailed status displays

The following tables document the behavior of the Somaris/7 DICOM Print AE in response to messages received for the printer SOP class and the print job SOP class.

Definitions of camera symbols:

- ☐ Idle: Camera is installed and ready; idle icon is displayed.
- ☐ Interact: The user has to react in near future, but not immediately.
- ☐ Example: A camera was low in 8x10 clear sheets: LOW 8x10 CLR was sent by N-EVENT-REPORT.
- ☐ Queue Stopped: The user has to react immediately. Either the camera needs immediate interaction or a job has been aborted.

Example: A camera is out of 8x10 clear sheets, or camera is down, or a film job is aborted.

Note

Different camera symbols are displayed according to the Printer Status Info.

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|--------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------|-----------------------------------------|
| NORMAL | Camera is ready. | Camera is ready. | <none>/idle</none> |
| BAD RECEIVE MGZ | There is a problem with the film receive magazine. Films from the printer cannot be transported into the magazine. | Problem with receive magazine. | <none>/interact</none> |
| BAD SUPPLY MGZ | There is a problem with a film supply magazine. Films from this magazine cannot be transported into the printer. | Problem with supply magazine. | <none>/interact</none> |
| CALIBRATING | Printer is performing self calibration, it is expected to be available for normal operation shortly. | Self calibration. Please wait. | <none>/idle</none> |
| CALIBRATION ERR | An error in the printer calibration has been detected, quality of processed films may not be optimal. | Problem in calibration. Film quality may not be optimal. | <none>/interact</none> |
| CHECK CHEMISTRY | A problem with the processor chemicals has been detected, quality of processed films may not be optimal. | Problem with chemistry. Film quality may not be optimal. | <none>/interact</none> |
| CHECK SORTER | There is an error in the film sorter. | Error in film sorter. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------|--------------------------------------------------------------|
| CHEMICALS EMPTY | There are no processing chemicals in the processor, films will not be printed and processed until the processor is back to normal. | Camera chemistry empty. Please check. | <none>/interact</none> |
| CHEMICALS LOW | The chemical level in the processor is low, if not corrected, it will probably shut down soon. | Camera chemistry low. Please check. | <none>/interact</none> |
| COVER OPEN | One or more printer or processor covers, drawers, doors are open. | Camera cover, drawer or door open. | <none>/interact</none> |
| ELEC CONFIG ERR | Printer configured improperly for this job. | Camera configured improperly for this job. Queue stopped. | Queue for this camera will be STOPPED/Queue stopped |
| ELEC DOWN | Printer is not operating due to some unspecified electrical hardware problem. | Camera electrical hardware problem. | <none>/interact</none> |
| ELEC SW ERROR | Printer not operating for some unspecified software error. | Camera software problem. Queue stopped. | Queue for this camera will be STOPPED/queue stopped |
| EMPTY 8x10 | The 8x10 inch film supply magazine is empty. | 8x10 film supply empty. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|-----------------------------------------------------|------------------------------------------|-----------------------------------------|
| EMPTY 8x10 BLUE | The 8x10 inch blue film supply magazine is empty. | 8x10 blue film supply empty. | <none>/interact</none> |
| EMPTY 8x10 CLR | The 8x10 inch clear film supply magazine is empty. | 8x10 clear film supply empty. | <none>/interact</none> |
| EMPTY 8x10 PAPR | The 8x10 inch paper supply magazine is empty. | 8x10 paper supply empty. | <none>/interact</none> |
| EMPTY 10x12 | The 10x12 inch film supply magazine is empty. | 10x12 film supply empty. | <none>/interact</none> |
| EMPTY 10x12 BLUE | The 10x12 inch blue film supply magazine is empty. | 10x12 blue film supply empty. | <none>/interact</none> |
| EMPTY 10x12 CLR | The 10x12 inch clear film supply magazine is empty. | 10x12 clear film supply empty. | <none>/interact</none> |
| EMPTY 10x12 PAPR | The 10x12 inch paper supply magazine is empty. | 10x12 paper supply empty. | <none>/interact</none> |
| EMPTY 10x14 | The 10x14 inch film supply magazine is empty. | 10x14 film supply empty. | <none>/interact</none> |
| EMPTY 10x14 BLUE | The 10x14 inch blue film supply magazine is empty. | 10x14 blue film supply empty. | <none>/interact</none> |
| EMPTY 10x14 CLR | The 10x14 inch clear film supply magazine is empty. | 10x14 clear film supply empty. | <none>/interact</none> |
| EMPTY 10x14 PAPR | The 10x14 inch paper supply magazine is empty. | 10x14 paper supply empty. | <none>/interact</none> |
| EMPTY 11x14 | The 11x14 inch film supply magazine is empty. | 11x14 film supply empty. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|-----------------------------------------------------|------------------------------------------|-----------------------------------------|
| EMPTY 11x14 BLUE | The 11x14 inch blue film supply magazine is empty. | 11x14 blue film supply empty. | <none>/interact</none> |
| EMPTY 11x14 CLR | The 11x14 inch clear film supply magazine is empty. | 11x14 clear film supply empty. | <none>/interact</none> |
| EMPTY 11x14 PAPR | The 11x14 inch paper supply magazine is empty. | 11x14 paper supply empty. | <none>/interact</none> |
| EMPTY 14x14 | The 14x14 inch film supply magazine is empty. | 14x14 film supply empty. | <none>/interact</none> |
| EMPTY 14x14 BLUE | The 14x14 inch blue film supply magazine is empty. | 14x14 blue film supply empty. | <none>/interact</none> |
| EMPTY 14x14 CLR | The 14x14 inch clear film supply magazine is empty. | 14x14 clear film supply empty. | <none>/interact</none> |
| EMPTY 14x14 PAPR | The 14x14 inch paper supply magazine is empty. | 14x14 paper supply empty. | <none>/interact</none> |
| EMPTY 14x17 | The 14x17 inch film supply magazine is empty. | 14x17 film supply empty. | <none>/interact</none> |
| EMPTY 14x17 BLUE | The 14x17 inch blue film supply magazine is empty. | 14x17 blue film supply empty. | <none>/interact</none> |
| EMPTY 14x17 CLR | The 14x17 inch clear film supply magazine is empty. | 14x17 clear film supply empty. | <none>/interact</none> |
| EMPTY 14x17 PAPR | The 14x17 inch paper supply magazine is empty. | 14x17 paper supply empty. | <none>/interact</none> |
| EMPTY 24x24 | The 24x24 inch film supply magazine is empty. | 24x24 film supply empty. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|---------------------------------------------------------------------------------------|------------------------------------------|-----------------------------------------|
| EMPTY 24x24 BLUE | The 24x24 inch blue film supply magazine is empty. | 24x24 blue film supply empty. | <none>/interact</none> |
| EMPTY 24x24 CLR | The 24x24 inch clear film supply magazine is empty. | 24x24 clear film supply empty. | <none>/interact</none> |
| EMPTY 24x24 PAPR | The 24x24 inch paper supply magazine is empty. | 24x24 paper supply empty. | <none>/interact</none> |
| EMPTY 24x30 | The 24x30 inch film supply magazine is empty. | 24x30 film supply empty. | <none>/interact</none> |
| EMPTY 24x30 BLUE | The 24x30 inch blue film supply magazine is empty. | 24x30 blue film supply empty. | <none>/interact</none> |
| EMPTY 24x30 CLR | The 24x30 inch clear film supply magazine is empty. | 24x30 clear film supply empty. | <none>/interact</none> |
| EMPTY 24x30 PAPR | The 24x30 inch paper supply magazine is empty. | 24x30 paper supply empty. | <none>/interact</none> |
| EMPTY A4 PAPR | The A4 paper supply magazine is empty. | A4 paper supply empty. | <none>/interact</none> |
| EMPTY A4 TRANS | The A4 transparency supply magazine is empty. | A4 transparency supply empty. | <none>/interact</none> |
| EXPOSURE FAILURE | The exposure device has failed due to some unspecified reason. | Exposure device has failed. | <none>/interact</none> |
| FILM JAM | A film transport error has occurred and a film is jammed in the printer or processor. | Film jam. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|-------------------------------------------------------------------------------------|------------------------------------------|-----------------------------------------|
| FILM TRANSP ERR | There is a malfunction with the film transport, there may or may not be a film jam. | Film transport problem. | <none>/interact</none> |
| FINISHER EMPTY | The finisher is empty. | Finisher is empty. | <none>/interact</none> |
| FINISHER ERROR | The finisher is not operating due to some unspecified reason. | Finisher problem. | <none>/interact</none> |
| FINISHER LOW | The finisher is low on supplies. | Finisher low. | <none>/interact</none> |
| LOW 8x10 | The 8x10 inch film supply magazine is low. | 8x10 film supply low. | <none>/interact</none> |
| LOW 8x10 BLUE | The 8x10 inch blue film supply magazine is low. | 8x10 blue film supply low. | <none>/interact</none> |
| LOW 8x10 CLR | The 8x10 inch clear film supply magazine is low. | 8x10 clear film supply low. | <none>/interact</none> |
| LOW 8x10 PAPR | The 8x10 inch paper supply magazine is low. | 8x10 paper supply low. | <none>/interact</none> |
| LOW 10x12 | The 10x12 inch film supply magazine is low. | 10x12 film supply low. | <none>/interact</none> |
| LOW 10x12 BLUE | The 10x12 inch blue film supply magazine is low. | 10x12 blue film supply low. | <none>/interact</none> |
| LOW 10x12 CLR | The 10x12 inch clear film supply magazine is low. | 10x12 clear film supply low. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|---------------------------------------------------|------------------------------------------|-----------------------------------------|
| LOW 10x12 PAPR | The 10x12 inch paper supply magazine is low. | 10x12 paper supply low. | <none>/interact</none> |
| LOW 10x14 | The 10x14 inch film supply magazine is low. | 10x14 film supply low. | <none>/interact</none> |
| LOW 10x14 BLUE | The 10x14 inch blue film supply magazine is low. | 10x14 blue film supply low. | <none>/interact</none> |
| LOW 10x14 CLR | The 10x14 inch clear film supply magazine is low. | 10x14 clear film supply low. | <none>/interact</none> |
| LOW 10x14 PAPR | The 10x14 inch paper supply magazine is low. | 10x14 paper supply low. | <none>/interact</none> |
| LOW 11x14 | The 11x14 inch film supply magazine is low. | 11x14 film supply low. | <none>/interact</none> |
| LOW 11x14 BLUE | The 11x14 inch blue film supply magazine is low. | 11x14 blue film supply low. | <none>/interact</none> |
| LOW 11x14 CLR | The 11x14 inch clear film supply magazine is low. | 11x14 clear film supply low. | <none>/interact</none> |
| LOW 11x14 PAPR | The 11x14 inch paper supply magazine is low. | 11x14 paper supply low. | <none>/interact</none> |
| LOW 14x14 | The 14x14 inch film supply magazine is low. | 14x14 film supply low. | <none>/interact</none> |
| LOW 14x14 BLUE | The 14x14 inch blue film supply magazine is low. | 14x14 blue film supply low. | <none>/interact</none> |
| LOW 14x14 CLR | The 14x14 inch clear film supply magazine is low. | 14x14 clear film supply low. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|---------------------------------------------------|------------------------------------------|-----------------------------------------|
| LOW 14x14 PAPR | The 14x14 inch paper supply magazine is low. | 14x14 paper supply low. | <none>/interact</none> |
| LOW 14x17 | The 14x17 inch film supply magazine is low. | 14x17 film supply low. | <none>/interact</none> |
| LOW 14x17 BLUE | The 14x17 inch blue film supply magazine is low. | 14x17 blue film supply low. | <none>/interact</none> |
| LOW 14x17 CLR | The 14x17 inch clear film supply magazine is low. | 14x17 clear film supply low. | <none>/interact</none> |
| LOW 14x17 PAPR | The 14x17 inch paper supply magazine is low. | 14x17 paper supply low. | <none>/interact</none> |
| LOW 24x24 | The 24x24 inch film supply magazine is low. | 24x24 film supply low. | <none>/interact</none> |
| LOW 24x24 BLUE | The 24x24 inch blue film supply magazine is low. | 24x24 blue film supply low. | <none>/interact</none> |
| LOW 24x24 CLR | The 24x24 inch clear film supply magazine is low. | 24x24 clear film supply low. | <none>/interact</none> |
| LOW 24x24 PAPR | The 24x24 inch paper supply magazine is low. | 24x24 paper supply low. | <none>/interact</none> |
| LOW 24x30 | The 24x30 inch film supply magazine is low. | 24x30 film supply low. | <none>/interact</none> |
| LOW 24x30 BLUE | The 24x30 inch blue film supply magazine is low. | 24x30 blue film supply low. | <none>/interact</none> |
| LOW 24x30 CLR | The 24x30 inch clear film supply magazine is low. | 24x30 clear film supply low. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|-----------------------------------------------------------------------------------------------------------|------------------------------------------|-----------------------------------------|
| LOW 24x30 PAPR | The 24x30 inch paper supply magazine is low. | 24x30 paper supply low. | <none>/interact</none> |
| LOW A4 PAPR | The A4 paper supply magazine is low. | A4 paper supply low. | <none>/interact</none> |
| LOW A4 TRANS | The A4 transparency supply magazine is low. | A4 transparency supply low. | <none>/interact</none> |
| NO RECEIVE MGZ | The film receive magazine no available. | Film receiver not available. | <none>/interact</none> |
| NO RIBBON | The ribbon cartridge needs to be replaced. | Replace ribbon cartridge. | <none>/interact</none> |
| NO SUPPLY MGZ | The film supply magazine specified for this job is not available. | Film supply not available. | <none>/interact</none> |
| CHECK PRINTER | The printer is not ready at this time, operator intervention is required to make the printer available. | Check camera. | <none>/interact</none> |
| CHECK PROC | The processor is not ready at this time, operator intervention is required to make the printer available. | Check processor. | <none>/interact</none> |
| PRINTER DOWN | The printer is not operating due to some unspecified reason. | Camera down. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------|-----------------------------------------|
| PRINTER INIT | The printer is not ready at this time, it is expected to become available without intervention, For example, it may be in a normal warm-up state. | Camera initializing. | <none>/idle</none> |
| PRINTER OFFLINE | The printer has been disabled by an operator or service person. | Camera off-line. | <none>/interact</none> |
| PROC DOWN | The processor is not operating due to some unspecified reason. | Processor down. | <none>/interact</none> |
| PROC INIT | The processor is not ready at this time, it is expected to become available without intervention. For example, it may be in a normal warm-up state. | Processor initializing. | <none>/idle</none> |
| PROC OVERFLOW FL | Processor chemicals are approaching the overflow full mark. | Processor chemicals overflow. | <none>/interact</none> |
| PROC OVERFLOW HI | Processor chemicals have reached the overflow full mark. | Processor chemicals near overflow. | <none>/interact</none> |
| QUEUED | Print job in Queue | - | <none>/idle</none> |
| RECEIVER FULL | The Film receive magazine is full. | Receiver full. | <none>/interact</none> |

| Printer Status info/Execution Status info | Description | Message string visible in the Status Bar | Other action for UI/camera symbol |
|-------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------|--------------------------------------------------------------|
| REQ MED NOT INST | The requested film, paper, or other media supply magazine is installed in the printer, but may be available with operator intervention. | Install media supply. | <none>/interact</none> |
| REQ MED NOT AVAI | The requested film, paper, or other media requested is not available on this printer. | Media supply not available on this camera. Queue stopped. Change camera. | Queue for this camera will be STOPPED/queue stopped |
| RIBBON ERROR | There is an unspecified problem with the print ribbon. | Error with print ribbon. | <none>/interact</none> |
| SUPPLY EMPTY | The printer is out of film. | Camera out of film. | <none>/interact</none> |
| SUPPLY LOW | The film supply is low. | Film supply low. | <none>/interact</none> |
| UNKNOWN | There is an unspecified problem. | Unspecified problem with camera. | <none>/interact</none> |

| Printer Status Infos: Additional Agfa printer status infos | | | | |
|------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| WARMING UP | Printer is in the warm-up stage. Spooling of print jobs to disk is still possible. | Camera is warming up. | <none>idle</none> | |
| OFFLINE | Printer is switched off-line. Spooling of print jobs to disk is still possible. | Camera is switched off- line. | <none>/interact</none> | |
| NONE | General printer warning, no specific information is available. Spooling of print jobs to disk is still possible. | - | <none>/idle</none> | |

| Printer Status Infos: Additional Kodak infos for Pacs Link (formerly Imation cameras) | | | | |
|---------------------------------------------------------------------------------------|-----------------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| SUPPLY MGZ ERR | The supply magazine has an error. | Film supply has an error. | <none>/interact</none> | |

| Printer Status Infos: Additional Kodak infos for Kodak 190 | | | | |
|------------------------------------------------------------|-------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| PRINTER STOPPED | The printer has stopped | Camera has stopped. | <none>/interact</none> | |

| Printer Status Infos: Additional Kodak infos for Kodak 190 | | | | |
|------------------------------------------------------------|--------------|------------------------------------------|--------------------------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| FATAL ERROR | Fatal error. | Fatal error. Queue stopped. | Queue for this camera will be STOPPED/queue stopped | |

| F | Printer Status Infos: Additional Kodak infos for 2180/1120 | | | | |
|---------------------|------------------------------------------------------------|------------------------------------------|--------------------------------------------------------------|--|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | | |
| PRINTER NOT RDY | Printer not ready. | Camera not ready. | <none>/interact</none> | | |
| CHECK PROCESSOR | Check processor. | Check processor. | <none>/interact</none> | | |
| NO TONER | No toner. | No toner. | <none>/interact</none> | | |
| FATAL | Fatal error. | Fatal error. Queue stopped. | Queue for this camera will be STOPPED/queue stopped | | |

| Printer Status Infos: Additional Codonics infos | | | | |
|-------------------------------------------------|------------------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| STANDARD | Printer is ready. | Camera is ready. | <none>/Normal</none> | |
| LOAD A-SIZE | Load A-size media. | Load A-size media. | <none>/interact</none> | |
| LOAD A- DVPAPER | Load A-size black and white paper. | Load A-size black and white paper. | <none>/interact</none> | |
| LOAD A- CVPAPER | Load A-size color paper. | Load A-size color paper. | <none>/interact</none> | |
| LOAD A- CVTRANS | Load A-size transparencies. | Load A-size transparencies. | <none>/interact</none> | |
| LOAD A4-SIZE | Load A4-size media. | Load A4-size media. | <none>/interact</none> | |

| Printer Status Infos: Additional Codonics infos | | | | |
|-------------------------------------------------|--------------------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| LOAD A4- DVPAPER | Load A4-size black and white paper. | Load A4-size black and white paper. | <none>/interact</none> | |
| LOAD A4- CVPAPER | Load A4-size color paper. | Load A4-size color paper. | <none>/interact</none> | |
| LOAD A4- CVTRANS | Load A4-size transparencies. | Load A4-size transparencies. | <none>/interact</none> | |
| LOAD LA-SIZE | Load LA-size media. | Load LA-size media. | <none>/interact</none> | |
| LOAD LA- DVPAPER | Load LA-size black and white paper. | Load LA-size black and white paper. | <none>/interact</none> | |
| LOAD LA- CVPAPER | Load LA-size color paper. | Load LA-size color paper. | <none>/interact</none> | |
| LOAD LA- CVTRANS | Load LA-size transparencies. | Load LA-size transparencies. | <none>/interact</none> | |
| LOAD LA4- SIZE | Load LA4-size media. | Load LA4-size media. | <none>/interact</none> | |
| LOAD LA4- DVPAPER | Load LA4-size black and white paper. | Load LA4-size black and white paper. | <none>/interact</none> | |
| LOAD LA4- CVPAPER | Load LA4-size color paper. | Load LA4-size color paper. | <none>/interact</none> | |
| LOAD LA4- CVTRANS | Load LA4-size transparencies. | Load LA4-size transparencies. | <none>/interact</none> | |
| LOAD XLA- SIZE | Load XLA-size media. | Load XLA-size media. | <none>/interact</none> | |
| LOAD XLA- DVPAPER | Load XLA-size black and white paper. | Load XLA-size black and white paper. | <none>/interact</none> | |

| Printer Status Infos: Additional Codonics infos | | | | |
|-------------------------------------------------|-------------------------------------------------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| LOAD XLA- CVPAPER | Load XLA-size color paper. | Load XLA-size color paper. | <none>/interact</none> | |
| LOAD XLA- CVTRANS | Load XLA-size transparencies. | Load XLA-size transparencies. | <none>/interact</none> | |
| LOAD XLA4- SIZE | Load XLA4-size media. | Load XLA4-size media. | <none>/interact</none> | |
| LOAD XLA4- DVPAPE | Load XLA4-size black and white paper. | Load XLA4-size black and white paper. | <none>/interact</none> | |
| LOAD XLA4- CVPAPE | Load XLA4-size color paper. | Load XLA4-size color paper. | <none>/interact</none> | |
| LOAD XLA4- CVTRAN | Load XLA4-size transparencies. | Load XLA4-size transparencies. | <none>/interact</none> | |
| LOAD XLW- SIZE | Load XLW-size media. | Load XLW-size media. | <none>/interact</none> | |
| LOAD XLW- DVPAPER | Load XLW-size black and white paper. | Load XLW-size black and white paper. | <none>/interact</none> | |
| LOAD XLW- CVPAPER | Load XLW-size color paper. | Load XLW-size color paper. | <none>/interact</none> | |
| LOAD 8X10- SIZE | Load 8x10 media. | Load 8x10 media. | <none>/interact</none> | |
| LOAD 8X10- DVFILM | Load 8x10 black and white film. | Load 8x10 black and white film. | <none>/interact</none> | |
| SUPPLY MISSING | The film supply magazine specified for this job is not available. | Film supply not available. | <none>/interact</none> | |

| Printer Status Infos: Additional Codonics infos | | | | |
|-------------------------------------------------|-------------------------------|------------------------------------------|--------------------------------------------|--|
| Printer Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | |
| RIBBON MISSING | Ribbon is missing. | Ribbon is missing. | <none>/interact</none> | |
| RIBBON EMPTY | Ribbon is empty. | Ribbon is empty. | <none>/interact</none> | |
| TOP COVER OPEN | Top cover of printer is open. | Top cover of camera is open. | <none>/interact</none> | |

| Ac | Additional DICOM Execution Status Information - evaluation | | | | |
|--------------------------|------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|--------------------------------------------------------------|--|--|
| Execution Status info | Description | Message string visible in the Status Bar | Other action for syngo/camera symbol | | |
| INVALID PAGE DES | The specified page layout cannot be printed or other page description errors have been detected. | Film Job cannot be printed on this camera. Queue stopped. Please redirect film job. | Queue for this camera will be STOPPED/Queue stopped | | |
| INSUFFICIEN T MEMORY | There is not enough memory available to complete this job. | Not enough memory available in camera. Queue stopped. Please continue queue or change camera. | Queue for this camera will be STOPPED/Queue stopped | | |
| NONE | General printer warning, no specific information is available. Spooling of print jobs to disk is still possible. | - | <none>/idle</none> | | |

Additional DICOM Execution Status Information

Printer Status Info and Execution Status Info are defined terms and can therefore be extended or reduced by camera manufacturers. Therefore syngo shall be flexible.

If any other printer status info or execution status info is received, syngo will react as shown in the following table

| Printer | Printer/ | Description | Message string | Other action for |
|-----------|----------------------|------------------------------------------|-------------------------------------------|-----------------------------|
| Status / | Execution | | visible in the | syngo/camera |
| Execution | Status Info | | Status Bar | symbol |
| WARNING | <any other=""></any> | <not defined="" info="" status=""></not> | Camera info: <status info=""></status> | <none>/ interact</none> |

Additional DICOM Execution Status Information

Printer Status Info and Execution Status Info are defined terms and can therefore be extended or reduced by camera manufacturers. Therefore syngo shall be flexible.

If any other printer status info or execution status info is received, syngo will react as shown in the following table

| Printer | Printer/ | Description | Message string | Other action for |
|-----------|----------------------|------------------------------------------|--------------------------------------------------------------|------------------------------------------------------------------|
| Status / | Execution | | visible in the | syngo/camera |
| Execution | Status Info | | Status Bar | symbol |
| FAILURE | <any other=""></any> | <not defined="" info="" status=""></not> | Camera info: <status info=""> Queue stopped.</status> | Queue for this camera will be STOPPED/ Queue stopped |

Modality Worklist AE Specification

The Modality Worklist SCU (patient registration in conjunction with the network application) requests that the remote SCP performs a match of all keys specified in the query against the information in its worklist database.

SIEMENS Somaris/7 DICOM implementation provides Standard Conformance to the following DICOM V3.0 SOP Class as an SCU:

| SOP Class Name | SOP Class UID |
|-----------------------------------------------|------------------------|
| Modality Worklist Information Model - FIND | 1.2.840.10008.5.1.4.31 |

Association Establishment Policies

General

It is possible to configure a cyclic update of the modality scheduler database through a background worklist request with date/time and modality information.

In addition the user can request worklist update with "Update Worklist". No duplicate entries will be added in the Scheduler DB. Entries are uniquely identified by the Study Instance UID (0020,000D) for the Requested Procedure and the SPS ID (0040,0009) in the SPS Sequence (0040,0100).

An interactive worklist query can be issued with search criteria entered in the patient based Query dialog from the patient browser.

The default PDU size used will be 28 KB.

Number of Associations

The Somaris/7 DICOM application initiates one association at a time to query worklist entry data.

Asynchronous Nature

The Somaris/7 DICOM implementation does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

Implementation Class UID of

1.3.12.2.1107.5.1.4

and an Implementation Version Name of

☐ "SIEMENS_S7VA20A".

Association Initiation Policy

The network application will (if configured) query the worklist periodically or by user request. Ever then it establishes an association by using the DICOM association services. During association establishment the negotiation of SOP classes to exchange the capabilities of the SCU and the SCP is not supported.

The following DIMSE-C operation is supported as SCU:

□ C-FIND

Real-World Activity

Associated Real-World Activity - Query (Update) Worklist A network application will perform worklist queries with the C-FIND request at regular intervals. In addition it can be triggered by immediate request. The received worklist items will be compared with the contents of the local scheduler database. New items will be inserted into scheduler database.

After each broad query all RP/SPS that were cancelled or rescheduled to another modality at the RIS will be automatically removed from the scheduler DB if

- the Examination of this procedure has not been started or finished yet
- the corresponding configuration item "Automatic removal of cancelled/rescheduled Request" was checked in the Service UI under DICOM / HIS/RIS Node

No automatic clean-up of the scheduler DB is performed after a Patient based Query since the worklist received does not give the complete list of all currently scheduled procedures for the modality.

Proposed Presentation Contexts - Query (Update) Worklist The Somaris/7 DICOM application will propose Presentation

Contexts as shown in the following table:

| | | - p | | | |
|--------------------------------------------------------|----------------------------|-------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------|----------|-------------|
| Abstract Syntax | | Transfer S | Role | Extended | |
| Name | UID | Name List | UID List | | Negotiation |
| Modality Worklist Information Model - FIND | 1.2.840.10008.5.1.4. 31 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1 .2 1.2.840.10008.1 .2.1 1.2.840.10008.1 .2.2 | SC U | None |

SOP Specific Conformance Statement - Update Worklist

Search Key Attributes of the Worklist C-FIND

The Somaris/7 DICOM worklist SCU supports "broad worklist gueries" with all required search keys.

The following tables describe the "broad query" search keys that the SCU supports, which is a query for all tasks scheduled for the own modality or own modality application entity defined with the following search keys:

| Search Key Attributes in a broad worklist query | | | | | | | |
|-------------------------------------------------|-------------|----------------------|-----------------------------------------------------------------------------------------------------|--|--|--|--|
| Attribute name ^a | Tag | Matching Key Type | query value | | | | |
| Scheduled Procedure Step | | | | | | | |
| Scheduled Procedure Step Sequence | (0040,0100) | R | | | | | |
| >Scheduled Station AE Title | (0040,0001) | R | Configurable ^b : own AET or "*" | | | | |
| >Scheduled Procedure Step Start Date | (0040,0002) | R | Configurable ^c : <act. date="">-<act Date> or range from UI</act </act.> | | | | |
| >Scheduled Procedure Step Start Time | (0040,0003) | R | Configurable ^c : <zero length=""> or range from UI</zero> | | | | |
| >Modality | (0008,0060) | R | Configurable ^b : own modality or "*" | | | | |

a. No other attributes of Register Dialog used for C-FIND

b. This depend on user configuration (Options->Configuration->Patient Registration) if the "own AET" is provided or not.. use HIS/RIS Tabcard for configuration.

c. It depends on user configuration (Options->Configuration->Patient Registration) if the actual Date with a full time range or an interactive input dialog for date/time specification is used.(<startDate>-<endDate>

Note

Since various SCP implementations depend on the Specific Character Set attribute to be sent with the C-FIND in order to deliver the attribute filled with the C-FIND-RSP (see below), as a default the attribute will be sent even if empty. In case this turns out to be a problem for a specific SCP in cooperation with Siemens this can be configured not to include this attribute.

Return Key Attributes of the Worklist C-FIND-RSP

The Somaris/7 DICOM worklist SCU supports worklist queries with return key attributes of all types. The following tables describe the return keys that the SCU supports.

An "x" in the UI column will indicate that the attribute is visualized when browsing the Worklist results with Patient Browser and/or during Patient Registration. The Patient Browser display is additionally influenced by the related Browser configuration.

A tag in the IOD column will indicate that the related attribute is included into the SOP Instances of the IODs created during processing of this worklist request.

A tag in the MPPS column will indicate that the related attribute is included into the SOP Instances of the MPPS objects created during processing of this worklist request. (See also the tables 'Performed Procedure Step N-CREATE Attributes' (pg. 150) and 'Performed Procedure Step N-SET Attributes' (pg. 154)).

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|-----------------------------------------|-------------|-----------------------|----|-------------|-------------|---------------------------------------------------------------------------------------------|
| SOP Common | | | | | | |
| Specific Character Set ^a | (0008,0005) | 1C | - | (0008,0005) | (0008,0005) | |
| Scheduled Procedure Step | | | | | | |
| Scheduled Procedure Step Sequence | (0040,0100) | 1 | | | | |
| >Modality | (0008,0060) | 1 | х | (0008,0060) | (0008,0060) | |
| >Requested Contrast Agent | (0032,1070) | 2C | х | (0032,1070) | | |
| >Scheduled Station AE Title | (0040,0001) | 1 | Х | | (0040,0241) | *"Scheduled Station AE Title" is taken as default for "Performed Station AE Title" |
| >Scheduled Procedure Step Start Date | (0040,0002) | 1 | x | | | |
| >Scheduled Procedure Step Start Time | (0040,0003) | 1 | x | | | |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|------------------------------------------|-------------|-----------------------|----|---------------------------------|---------------------------------|--------------------------------------------------------------------------------------------------------------------|
| >Scheduled Procedure Step End Date | (0040,0004) | 3 | - | | | |
| >Scheduled Procedure Step End Time | (0040,0005) | 3 | - | | | |
| >Scheduled Performing Physician's Name | (0040,0006) | 1 | х | (0008,1050) | (0008,1050) | *"Scheduled Perform- ing Physician's Name" is taken as default for "Performing Physician's Name" |
| >Scheduled Procedure Step Description | (0040,0007) | 1C | X | (0040,0007) (0040,0254) | (0040,0007) (0040,0254) | *"Scheduled Procedure Step Description" is taken as default for "Performed Procedure Step Description" |
| >Scheduled Protocol Code Sequence** | (0040,0008) | 1C | - | (0040,0008) (0040,0260) * | (0040,0008) (0040,0260) * | ** Uses universal sequence match |
| | | | | | | *"Scheduled Protocol Code Sequence" is taken as default for "Performed Protocol Code Sequence" |
| >>Code Value | (0008,0100) | 1C | х | | | |
| >>Coding Scheme Designator | (0008,0102) | 1C | х | | | |
| >>Coding Scheme Version | (0008,0103) | 3 | х | | | |
| >>Code Meaning | (0008,0104) | 3 | х | | | |
| >Scheduled Procedure Step ID | (0040,0009) | 1 | x | (0040,0009) (0040,0253) | (0040,0009) (0040,0253) | *"Scheduled Procedure Step ID" is taken as default for "Performed Procedure Step ID" |
| >Scheduled Station Name | (0040,0010) | 2 | х | | | |
| >Scheduled Procedure Step Location | (0040,0011) | 2 | X | | (0040,0242) | *"Scheduled Procedure Step Location" is taken as default for "Per- formed Procedure Step Location" |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|-------------------------------------------|-----------------|-----------------------|----|---------------------------------|---------------------------------|------------------------------------------------------------------------------------------------------|
| >Pre-Medication | (0040,0012) | 2C | х | | | |
| >Scheduled Procedure Step Status | (0040,0020) | 3 | x | | | |
| >Comments on the Scheduled Procedure Step | (0040,0400) | 3 | - | | | |
| Requested Procedure | | | | | | |
| Referenced Study Sequence** | (0008,1110) | 2 | - | (0008,1110) | (0008,1110) | ** Uses universal sequence match |
| >Referenced SOP Class UID | (0008,1150) | 1C | - | | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | - | | | |
| Study Instance UID | (0020,000D) | 1 | - | (0020,000D) | (0020,000D) | |
| Requested Procedure Description | (0032,1060) | 1C | x | (0032,1060) | (0032,1060) | |
| Requested Procedure Code | (0032,1064) | 1C | - | (0032,1032) | (0032,1032) | ** Uses universal |
| Sequence** | | | | (0032,1064) | | sequence match |
| | | | | | | *"Requested Proce- dure Code Sequence" is taken as default for "Procedure Code Sequence" |
| >Code Value | (0008,0100) | 1C | х | | | |
| >Code Scheme Designator | (0008,0102) | 1C | х | | | |
| >Code Scheme Version | (0008,0103) | 3 | х | | | |
| >Code Meaning | (0008,0104) | 3 | х | | | |
| Requested Procedure ID | (0040,1001) | 1 | х | (0040,1001) (0020,0010) * | (0040,1001) (0020,0010) * | *"Requested Proce- dure ID" is taken as default for "Study ID" |
| Reason for the Requested Procedure | (0040,1002) | 3 | - | | | |
| Requested Procedure Priority | (0040,1003) | 2 | х | | | |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|---------------------------------------------------|-------------|-----------------------|----|-------------|-------------|-------------------------------------------------|
| Patient Transport Arrangements | (0040,1004) | 2 | - | | | |
| Requested Procedure Location | (0040,1005) | 3 | - | | | |
| Confidentiality Code | (0040,1008) | 3 | - | | | |
| Reporting Priority | (0040,1009) | 3 | - | | | |
| Names of Intended Recipients of results | (0040,1010) | 3 | - | (0008,1048) | | |
| Requested Procedure Comments | (0040,1400) | 3 | х | | | |
| Reason for the Requested Procedure Code Sequence | (0040,100A) | 3 | - | | | |
| Imaging Service Request | | | | | | |
| Accession Number | (0008,0050) | 2 | х | (0008,0050) | (0008,0050) | |
| Referring Physician's Name | (0008,0090) | 2 | х | (0008,0090) | | |
| Requesting Physician | (0032,1032) | 2 | х | (0032,1032) | (0032,1032) | |
| Requesting Service | (0032,1033) | 3 | х | (0032,1033) | | |
| Reason for the Imaging Service Request | (0040,2001) | 3 | - | | | |
| Issuing Date of Imaging Service Request | (0040,2004) | 3 | - | | | |
| Issuing Time of Imaging Service Request | (0040,2005) | 3 | - | | | |
| Placer Order Number / Imaging Service Request* | (0040,2016) | 3 | - | | (0040,2016) | *Old tag (0040,2006) is retired and not used |
| Filler Order Number / Imaging Service Request* | (0040,2017) | 3 | - | | (0040,2017) | *Old tag (0040,2007) is retired and not used |
| Order entered by | (0040,2008) | 3 | - | | | |
| Order Enterer's Location | (0040,2009) | 3 | - | | | |
| Order Callback Phone Number | (0040,2010) | 3 | - | | | |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|----------------------------------|-------------|-----------------------|----|-------------|-------------|----------------------------------|
| Imaging Service Request Comments | (0040,2400) | 3 | х | | | |
| Visit Identification | | | | | | |
| Institution Name | (0008,0080) | 3 | Х | (0008,0080) | | |
| Institution Address | (0008,0081) | 3 | - | (0008,0081) | | |
| Institution Code Sequence** | (0008,0082) | 3 | - | | | ** Uses universal sequence match |
| >Code Value | (0008,0100) | 1C | | | | |
| >Code Scheme Designator | (0008,0102) | 1C | | | | |
| >Code Scheme Version | (0008,0103) | 3 | | | | |
| >Code Meaning | (0008,0104) | 3 | | | | |
| Admission ID | (0038,0010) | 2 | х | | | |
| Issuer of Admission ID | (0038,0011) | 3 | - | | | |
| Visit Status | | | | | | |
| Visit Status ID | (0038,0008) | 3 | - | | | |
| Current Patient Location | (0038,0300) | 2 | х | | | |
| Patient's Institution Residence | (0038,0400) | 3 | - | | | |
| Visit Comments | (0038,4000) | 3 | - | | | |
| Visit Relationship | | | | | | |
| Referenced Study Sequence** | (0008,1110) | 3 | - | | | ** Uses universal sequence match |
| >Referenced SOP Class UID | (0008,1150) | 1C | - | | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | - | | | |
| Referenced Patient Sequence** | (0008,1120) | 2 | - | | (0008,1120) | ** Uses universal sequence match |
| >Referenced SOP Class UID | (0008,1150) | 1C | - | | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | - | | | |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|----------------------------------------|-------------|-----------------------|----|-------------|-------------|----------------------------------|
| Visit Admission | | | | | | |
| Referring Physician's Name | (0008,0090) | 2 | х | (0008,0090) | | |
| Referring Physician's Address | (0008,0092) | 3 | - | | | |
| Referring Physician's Phone Numbers | (0008,0094) | 3 | - | | | |
| Admitting Diagnoses Description | (0008,1080) | 3 | x | (0008,1080) | | |
| Admitting Diagnosis Code Sequence** | (0008,1084) | 3 | | | | ** Uses universal sequence match |
| >Code Value | (0008,0100) | 1C | | | | |
| >Code Scheme Designator | (0008,0102) | 1C | | | | |
| >Code Scheme Version | (0008,0103) | 3 | | | | |
| >Code Meaning | (0008,0104) | 3 | | | | |
| Route of Admissions | (0038,0016) | 3 | - | | | |
| Admitting Date | (0038,0020) | 3 | - | | | |
| Admitting Time | (0038,0021) | 3 | - | | | |
| Patient Identification | | | | | | |
| Patient's Name | (0010,0010) | 1 | х | (0010,0010) | (0010,0010) | |
| Patient ID | (0010,0020) | 1 | х | (0010,0020) | (0010,0020) | |
| Issuer of Patient ID | (0010,0021) | 3 | - | (0010,0021) | | |
| Other Patient IDs | (0010,1000) | 3 | х | (0010,1000) | | |
| Other Patient Names | (0010,1001) | 3 | х | (0010,1001) | | |
| Patient's Birth Name | (0010,1005) | 3 | - | (0010,1005) | | |
| Patient's Mother's Birth Name | (0010,1060) | 3 | - | (0010,1060) | | |
| Medical Record Locator | (0010,1090) | 3 | - | (0010,1090) | | |
| Patient Demographic | | | | | | |
| Patients Birth Date | (0010,0030) | 2 | x | (0010,0030) | (0010,0030) | |
| Patients Birth Time | (0010,0032) | 3 | - | (0010,0032) | | |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|----------------------------------------------------|-----------------|-----------------------|----|-----------------|-------------|----------------------------------|
| Patient's Sex | (0010,0040) | 2 | х | (0010,0040) | (0010,0040) | |
| Patient's Insurance Plan Code Sequence** | (0010,1050) | 3 | - | (0010,1050) | | ** Uses universal sequence match |
| >Code Value | (0008,0100) | 1C | | | | |
| >Code Scheme Designator | (0008,0102) | 1C | | | | |
| >Code Scheme Version | (0008,0103) | 3 | | | | |
| >Code Meaning | (0008,0104) | 3 | | | | |
| Patient's Age | (0010,1010) | 3 | х | (0010,1010) | | |
| Patient's Size | (0010,1020) | 3 | х | (0010,1020) | | |
| Patient's Weight | (0010,1030) | 2 | х | (0010,1030) | | |
| Patient's Address | (0010,1040) | 3 | х | (0010,1040) | | |
| Military Rank | (0010,1080) | 3 | х | (0010,1080) | | |
| Branch of Service | (0010,1081) | 3 | - | (0010,1081) | | |
| Country of Residence | (0010,2150) | 3 | - | (0010,2150) | | |
| Region of Residence | (0010,2152) | 3 | - | (0010,2152) | | |
| Patient's Telephone Numbers | (0010,2154) | 3 | - | (0010,2154) | | |
| Ethnic Group | (0010,2160) | 3 | х | (0010,2160) | | |
| Occupation | (0010,2180) | 3 | - | (0010,2180) | | |
| Patient's Religious Preference | (0010,21F0) | 3 | - | (0010,21F0) | | |
| Patient Comments | (0010,4000) | 3 | х | (0010,4000) | | |
| Patient Data Confidentality Constraint Description | (0040,3001) | 2 | x | (0040,3001) | | |
| Patient Medical | | | | | | |
| Medical Alerts | (0010,2000) | 2 | х | (0010,2000) | | |
| Contrast Allergies | (0010,2110) | 2 | х | (0010,2110) | | |
| Pregnancy Status | (0010,21C0) | 2 | x | (0010,21C0) | | |
| Smoking Status | (0010,21A0) | 3 | х | (0010,21A0) | | |

| Attribute name | Tag | Return Key Type | UI | IOD | MPPS | Notes |
|----------------------------------------|-----------------|-----------------------|----|-----------------|------|-------------------------------------|
| Last Menstrual Date | (0010,21D0) | 3 | х | (0010,21D0) | | |
| Additional Patient History | (0010,21B0) | 3 | х | (0010,21B0) | | |
| Special Needs | (0038,0050) | 2 | х | (0038,0050) | | |
| Patient State | (0038,0500) | 2 | х | (0038,0500) | | |
| Patient Relationship | | | | | | |
| Referenced Study Sequence** | (0008,1110) | 3 | - | | | ** Uses universal sequence match |
| >Referenced SOP Class UID | (0008,1150) | 1C | - | | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | - | | | |
| Referenced Visit Sequence** | (0008,1125) | 3 | - | | | ** Uses universal sequence match |
| >Referenced SOP Class UID | (0008,1150) | 1C | - | | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | - | | | |
| Referenced Patient Alias Sequence** | (0038,0004) | 3 | - | | | ** Uses universal sequence match |
| >Referenced SOP Class UID | (0008,1150) | 1C | - | | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | - | | | |

a. The Specific Character Set distributed via Modality Worklist by the hospital network shall reflect the global hospital context, e.g. a hospital with german context shall not restrict the worklist entries to ISO_IR 6 but supply a character set adequate to hold all german language text input.

Associated Real-World Activity - Get Worklist

With "Get Worklist" in the patient based Worklist Query dialog the entered attributes are used to form a worklist request identifier. With the response data the Patient Registration dialog can be updated to perform examination in advance. The response data are additionally placed in the scheduler database.

Proposed Presentation Contexts - Get Worklist

The same Presentation contexts as with "Update Worklist" will be proposed.

SOP Specific Conformance - Get Worklist

Search Key Attributes of the Worklist C-FIND

The Somaris/7 DICOM worklist SCU supports "narrow worklist queries" with all required search keys.

The following table describes the search keys that the SCU supports for a patient based worklist query, which is defined by the following search keys

| Attribute name | Tag | Matching Key Type | query value |
|-------------------------------------------|-------------|----------------------|----------------------------------|
| Scheduled Procedure Step | | | |
| Scheduled Procedure Step Sequence | (0040,0100) | R | |
| >Scheduled Performing Physician's Name | (0040,0006) | R | inserted in UI or zero length |
| Requested Procedure | | | |
| Requested Procedure ID | (0040,1001) | Ο | inserted in UI or zero length |
| Imaging Service Request | | | |
| Accession Number | (0008,0050) | 0 | inserted in UI or zero length |
| Referring Physician's Name | (0008,0090) | Ο | inserted in UI or zero length |
| Visit Status | | | |

| Attribute name | Tag | Matching Key Type | query value |
|-----------------------------|-------------|----------------------|----------------------------------|
| Current Patient Location | (0038,0300) | 0 | inserted in UI or zero length |
| Patient Identification | | | |
| Patient's Name ^a | (0010,0010) | R | inserted in UI or zero length |
| Patient ID | (0010,0020) | R | inserted in UI or zero length |

a. Please be aware that although in the UI there are different entries for First Name and Last Name of a patient these are combined into one query attribute. Querying just for the Last Name (without wildcard) or not providing a first letter for the First Name may prevent expected matches.

Return Key Attributes used from the Worklist C-FIND-RSP

Please see table \rightarrow page A.2–144.

Status Codes of the Worklist C-FIND

The worklist SCU interprets following status codes:

| Service Status | Meaning | Status Codes (0000,0900) | Related Fields |
|-------------------|--------------------------------------------------------|-----------------------------|----------------------------|
| Refused | Out of Resources | A700 | (0000,0902) |
| Failed | Identifier does not match SOP Class | A900 | (0000,0901) (0000,0902) |
| | Unable to process | Сххх | (0000,0901) (0000,0902) |
| Cancel | Matching terminated due to Cancel request | FE00 | None |
| Success | Matching is complete - No final Identifier is supplied | 0000 | None |

| Service Status | Meaning | Status Codes (0000,0900) | Related Fields | |
|-------------------|--------------------------------------------------------------------------------------------------------------------------------------|-----------------------------|----------------|--|
| Pending | Matches are continuing - Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys | FF00 | Identifier | |
| | Matches are continuing - Warning that one or more Optional Keys were not supported for existence and/or matching for this identifier | FF01 | Identifier | |

Modality Performed Procedure Step AE Specification

The Modality Performed Procedure Step SCU (Patient Registration and MPPS UI) provide information about a performed real-world Procedure to a remote SCP (Information System).

SIEMENS Somaris/7 DICOM implementation provides Standard Conformance to the following DICOM V3.0 SOP Class as an SCU:

| SOP Class Name | SOP Class UID |
|--------------------------------------|-------------------------|
| Modality Performed Procedure Step | 1.2.840.10008.3.1.2.3.3 |

Association Establishment Policies

General

The creation of a MPPS Instance is done automatically by the Somaris/7 DICOM application whenever a patient is registered for image acquisition through the Patient Registration dialog. Exception: In case of Emergency Patients the MPPS is created only when the user explicitly sends a corresponding message from the MPPS user interface.

Further updates on the MPPS data can be done interactively from the related MPPS user interface. The MPPS "Complete" or "Discontinued" states can only be set from user interface.

The default PDU size used will be 28 KB

Number of Associations

The Siemens DICOM application initiates one association at a time to create or set MPPS instance.

Asynchronous Nature

The Siemens DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

Implementation Identifying Information

Implementation Class UID of

"1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS S7VA20A".

Association Initiation Policy

The Somaris/7 DICOM Application Entity acts as a Service Class User (SCU) for the Modality Performed Procedure Step Service Class (to notify a RIS about status of a procedure while it is performed).

To do so, it will issue a:

- □ N-CREATE DIMSE according to the CREATE Modality Performed Procedure Step SOP Instance operation or a
- □ N-SET DIMSE to update the contents and state of the MPPS according to the SET Modality Performed Procedure Step Information operation

Real World Activity

Associated Real-World Activity - Patient Registered A patient is registered by the Patient Registration "Exam" action. From this event the trigger to create a MPPS Instance is derived. The related Instance is then immediately communicated to the configured RIS system. An association is established and the MPPS Instance is sent.

Proposed Presentation Contexts - Patient Registered The Siemens DICOM application will propose Presentation Contexts as shown in the following table:

| | Presentation Context Table | | | | |
|--------------------------------------------|-----------------------------|-------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------|------|-------------|
| Abs | stract Syntax | Transfer : | Syntax | Role | Extended |
| Name | UID | Name List | UID List | | Negotiation |
| Modality Performed Procedure Step | 1.2.840.10008.3.1.2.3 .3 | Implicit VR Little Endian Explicit VR Little Endian Explicit VR Big Endian | 1.2.840.10008.1. 2 1.2.840.10008.1. 2.1 1.2.840.10008.1. 2.2 | SCU | None |

SOP Specific Conformance Statement - Patient Registered Attributes used for the Performed Procedure Step N-CREATE

The Siemens DICOM Modality Performed Procedure Step SCU informs the remote SCP when the examination of a scheduled procedure step will be performed (i.e. the patient is registered). The N-CREATE message is sent when the examination is started with successful registration of patient data. The following table describes the supported attributes for a N-CREATE message.

| Performed Procedure Step N-CREATE Attributes | | | | |
|---------------------------------------------------|-------------|------------------|----------------------------|--|
| Attribute name | Tag | Required Type | Value | |
| SOP Common | | | | |
| Specific Character Set | (0008,0005) | 1C | from MWL or created | |
| Performed Procedure Step Relationship | | | | |
| Scheduled Step Attribute Sequence | (0040,0270) | 1 | | |
| >Study Instance UID | (0020,000D) | 1 | from MWL or created | |
| >Referenced Study Sequence | (0008,1110) | 2 | from MWL or zero length | |
| >>Referenced SOP Class UID | (0008,1150) | 1C | | |
| >>Referenced SOP Instance UID | (0008,1155) | 1C | | |
| >Accession Number | (0008,0050) | 2 | from MWL or user input | |
| >Placer Order Number / Imaging Service Request | (0040,2016) | 3 | from MWL or zero length | |
| >Filler Order Number / Imaging Service Request | (0040,2017) | 3 | from MWL or zero length | |
| >Requested Procedure ID | (0040,1001) | 2 | from MWL or user input | |
| >Requested Procedure Description | (0032,1060) | 2 | from MWL or zero length | |

| Performed Procedure Step N-CREATE Attributes | | | | |
|----------------------------------------------|-------------|------------------|-----------------------------------|--|
| Attribute name | Tag | Required Type | Value | |
| >Scheduled Procedure Step ID | (0040,0009) | 2 | from MWL or zero length | |
| >Scheduled Procedure Step Description | (0040,0007) | 2 | from MWL or zero length | |
| >Scheduled Protocol Code Sequence | (0040,0008) | 2 | from MWL or zero length | |
| >>Code Value | (0008,0100) | 1C | | |
| >>Coding Scheme Designator | (0008,0102) | 1C | | |
| >>Code Scheme Version | (0008,0103) | 3 | | |
| >>Code Meaning | (0008,0104) | 3 | | |
| Patient's Name | (0010,0010) | 2 | from MWL or user input | |
| Patient ID | (0010,0020) | 2 | from MWL or user input or created | |
| Patients Birth Date | (0010,0030) | 2 | from MWL or user input | |
| Patient's Sex | (0010,0040) | 2 | from MWL or user input | |
| Referenced Patient Sequence | (0008,1120) | 2 | from MWL or zero length | |
| >Referenced SOP Class UID | (0008,1150) | 1C | | |
| >Referenced SOP Instance UID | (0008,1155) | 1C | | |
| Performed Procedure Step Information | | | | |
| Performed Procedure Step ID | (0040,0253) | 1 | from SPS ID or created | |
| Performed Station AE Title | (0040,0241) | 1 | own AE Title | |
| Performed Station Name | (0040,0242) | 2 | own hostname | |

| Performed Procedure Step N-CREATE Attributes | | | | |
|----------------------------------------------|-------------|------------------|----------------------------------------------------|--|
| Attribute name | Tag | Required Type | Value | |
| Performed Location | (0040,0243) | 2 | from SPS Location or zero length | |
| Performed Procedure Step Start Date | (0040,0244) | 1 | created | |
| Performed Procedure Step Start Time | (0040,0245) | 1 | created | |
| Performed Procedure Step Status | (0040,0252) | 1 | IN PROGRESS | |
| Performed Procedure Step Description | (0040,0254) | 2 | from SPS Description or zero length | |
| Performed Procedure Type Description | (0040,0255) | 2 | zero length | |
| Procedure Code Sequence | (0008,1032) | 2 | from Requested Procedure Code or zero length | |
| >Code Value | (0008,0100) | 1C | | |
| >Code Scheme Designator | (0008,0102) | 1C | | |
| >Code Scheme Version | (0008,0103) | 3 | | |
| >Code Meaning | (0008,0104) | 3 | | |
| Performed Procedure Step End Date | (0040,0250) | 2 | zero length | |
| Performed Procedure Step End Time | (0040,0251) | 2 | zero length | |
| Image Acquisition Results | | | | |
| Modality | (0008,0060) | 1 | CT | |
| Study ID | (0020,0010) | 2 | from Requested Procedure ID or created | |

| Performed Procedure Step N-CREATE Attributes | | | | |
|-------------------------------------------------|-------------|------------------|---------------------------------------------------------------------------------------------|--|
| Attribute name | Tag | Required Type | Value | |
| Performed Protocol Code Sequence | (0040,0260) | 2 | from Scheduled Protocol Code SQ or zero length | |
| >Code Value | (0008,0100) | 1C | | |
| >Code Scheme Designator | (0008,0102) | 1C | | |
| >Code Scheme Version | (0008,0103) | 3 | | |
| >Code Meaning | (0008,0104) | 3 | | |
| Performed Series Sequence | (0040,0340) | 2 | | |
| >Performing Physicians's Name | (0008,1050) | 2C | from MWL or user input | |
| >Protocol Name | (0008,1030) | n/a | User-defined description of the conditions under which the Series was performed | |
| >Operator's Name | (0008,1070) | 2C | user input | |
| >Series Instance UID | (0020,000E) | 1C | created | |
| >Series Description | (0008,103E) | 2C | zero length | |
| >Retrieve AE Title | (0008,0054) | 2C | zero length | |
| >Referenced Image Sequence | (0008,1140) | 2C | zero length | |
| >Referenced Standalone SOP Instance Sequence | (0040,0220) | 2C | zero length | |
| Radiation Dose Module | | | | |
| DistanceSourceToDetector | (0040,0301) | 3 | created | |
| DistanceSourceToEntrance | (0040,0306) | 3 | zero length | |

| Performed Procedure Step N-CREATE Attributes | | | | |
|----------------------------------------------|-------------|------------------|---------|--|
| Attribute name | Tag | Required Type | Value | |
| TotalNumberOfExposures | (0040,0301) | 3 | created | |
| CommentsOnRadiationDose | (0040,0310) | 3 | created | |
| Exposure Dose Sequence | (0040,030E) | 3 | created | |
| Billing And Material Management Code | Module | | | |
| Billing Supplies and Devices Sequence | (0040,0324) | 3 | created | |
| Quantity Sequence | (0040,0293) | 3 | created | |

Status Codes of the Performed Procedure Step N-CREATE

The Performed Procedure Step SCU interprets the following status values:

| | N-SET Response Status | | | | |
|-----------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------|-----------------------------|--|--|--|
| Service Status | Meaning | Status Codes (0000,0900) | | | |
| Failure | Processing Failure | 0110 | | | |
| | No such attribute | 0105 | | | |
| | Invalid attribute value | 0106 | | | |
| | Duplicate SOP Instance | 0111 | | | |
| | No such SOP Instance | 0112 | | | |
| | No such SOP class | 0118 | | | |
| | Class instance conflict | 0119 | | | |
| | Missing attribute | 0120 | | | |
| | Missing attribute value | 0121 | | | |
| | Resource limitation | 0213 | | | |
| Success | Successful Operation | 0000 | | | |
| Associated Real-World Activity - UI-Update | The associated Real-World activity is to mation to an SCP by using the DICOM cedure Step Service either implicitly (6 (MPPS UI). | Modality Performed Pro- | | | |
| Proposed Presentation Contexts -UI- Update | The same Presentation Contexts as in → page A.2–157 will b proposed. | | | | |

SOP Specific Conformance Statement - UI-Update

Attributes used for the Performed Procedure Step N-SET

The Somaris/7 DICOM Performed Procedure Step SCU informs the remote SCP about the performed examination and its status. The N-Set message is sent once for each reconstruction of acquisition data with status "IN_PROGRESS". The N-SET message is sent on user input with status "COMPLETED" or "DISCONTINUED", respectively. This is intended to be used when the examination is finished with status "COMPLETED" or when the examination could not be completed with status "DISCONTINUED" (already sent automatically when examination is ended via End Exam and no acquisition was done). The following table describes the supported attributes for a N-SET message.

| Performed Procedure Step N-SET Attributes | | | | |
|-------------------------------------------|-------------|------------------|-------------------------------------------------------------------------------------------------------|--|
| Attribute name | Tag | Required Type | Value | |
| Performed Procedure Step Informat | ion | | | |
| Performed Procedure Step Status | (0040,0252) | 3 | IN_PROGRESS or COMPLETED or DISCONTINUED (see above) | |
| Performed Procedure Step Description | (0040,0254) | 3 | from SPS Description or user input | |
| Performed Procedure Type Description | (0040,0255) | 3 | zero length | |
| Procedure Code Sequence | (0008,1032) | 3 | from Requested Procedure Code or empty on user change (see IHE Technical Framework Y3, App. C Note 6) | |
| >Code Value | (0008,0100) | 1C | | |

| Performed Procedure Step N-SET Attributes | | | | | | |
|-------------------------------------------|-------------|------------------|--------------------------------------------------------------------------------------------------|--|--|--|
| Attribute name | Tag | Required Type | Value | | | |
| >Code Scheme Designator | (0008,0102) | 1C | | | | |
| >Code Scheme Version | (0008,0103) | 3 | | | | |
| >Code Meaning | (0008,0104) | 3 | | | | |
| Performed Procedure Step End Date | (0040,0250) | 1 | created | | | |
| Performed Procedure Step End Time | (0040,0251) | 1 | created | | | |
| Image Acquisition Results | | | | | | |
| Performed Protocol Code Sequence | (0040,0260) | 3 | from Scheduled Protocol Code SQ or user input | | | |
| >Code Value | (0008,0100) | 1C | | | | |
| >Code Scheme Designator | (0008,0102) | 1C | | | | |
| >Code Scheme Version | (0008,0103) | 1C | | | | |
| >Code Meaning | (0008,0104) | 1C | | | | |
| Performed Series Sequence | (0040,0340) | 1 | | | | |
| >Performing Physicians's Name | (0008,1050) | 2C | from MWL or user input [empty if Study Split is used] | | | |
| >Protocol Name | (0018,1030) | 1C | name of Scan Protocol (Siemens standard or user defined) [dummy if Study Split is used] | | | |
| >Operator's Name | (0008,1070) | 2C | user input [empty if Study Split is used] | | | |
| >Series Instance UID | (0020,000E) | 1C | created [dummy if Study Split is used] | | | |

| Performed Procedure Step N-SET Attributes | | | | | | | |
|-------------------------------------------------|-------------|------------------|------------------------------------------------------------------------------------------------------------|--|--|--|--|
| Attribute name | Tag | Required Type | Value | | | | |
| >Series Description | (0008,103E) | 2C | range name with generated reconstruction information or user input [empty if Study Split is used] | | | | |
| >Retrieve AE Title | (0008,0054) | 2C | from Storage Commitment RSP or zero length | | | | |
| >Referenced Image Sequence | (0008,1140) | 2C | created [empty if Study Split is used] | | | | |
| >>Referenced SOP Class UID | (0008,1150) | 1C | | | | | |
| >>Referenced SOP Instance UID | (0008,1155) | 1C | | | | | |
| >Referenced Standalone SOP Instance Sequence | (0040,0220) | 2C | zero length | | | | |
| Radiation Dose | | | | | | | |
| Total Number of Exposures | (0040,0301) | 3 | created (number of scans) | | | | |
| Distance Source to Detector (SID) | (0018,1110) | 3 | created | | | | |
| Distance Source to Entrance | (0040,0306) | 3 | created | | | | |
| Comments on Radiation Dose | (0040,0310) | 3 | created (one line per scan) ^a | | | | |
| Exposure Dose Sequence | (0040,030E) | 3 | created | | | | |
| >Radiation Mode | (0018,115A) | 3 | created | | | | |
| >KVp | (0018,0060) | 3 | created | | | | |
| >X-ray Tube Current in uA | (0018,8151) | 3 | created | | | | |
| >Exposure Time | (0018,1150) | 3 | created | | | | |

| Performed Procedure Step N-SET Attributes | | | | | | | |
|-------------------------------------------|-------------|------------------|-------------------------------------------------------------------------------|--|--|--|--|
| Attribute name | Tag | Required Type | Value | | | | |
| >Filter Type | (0018,1160) | 3 | "WEDGE" | | | | |
| Billing and Material Management (| Code | | | | | | |
| Film Consumption Sequence | (0040,0321) | 3 | created or zero length | | | | |
| >Number of Films | (2100,0170) | 3 | | | | | |
| >Medium Type | (2000,0030) | 3 | | | | | |
| >Film Size ID | (2010,0050) | 3 | | | | | |
| Billing Supplies and Devices Sequence | (0040,0324) | 3 | user input in Examination Card (example values below) or MPPS window | | | | |
| >Billing Item Sequence | (0040,0296) | 3 | | | | | |
| >>Code Value | (0008,0100) | 1C | e.g. contrast media code | | | | |
| >>Code Scheme Designator | (0008,0102) | 1C | e.g. contrast media catalog | | | | |
| >>Code Meaning | (0008,0104) | 1C | e.g. contrast media name | | | | |
| >Quantity Sequence | (0040,0293) | 1C | | | | | |
| >>Quantity | (0040,0294) | 3 | e.g. volume of contrast media | | | | |
| >>Measuring Units Sequence | (0040,0295) | 3 | | | | | |
| >>>Code Value | (0008,0100) | 1C | e.g. cm3 | | | | |
| >>>Code Scheme Designator | (0008,0102) | 1C | e.g. UCUM | | | | |
| >>>Code Scheme Version | (0008,0103) | 1C | e.g. 1.4 | | | | |
| >>>Code Meaning | (0008,0104) | 1C | e.g. cm3 | | | | |

a. The following format is used:

RangeName: kV=xxx mAs=xxx CTDIvol=xxx DLP=xxx DualTube: kV=xxx/yyy mAs=xxx CTDIvol=xxx DLP=xxx CTDIvol and DLP is not provided for Topogram scans.

Status Codes of the Performed Procedure Step N-SET

The Performed Procedure Step SCU interprets the following status values:

| N-SET Response Status | | | | | | |
|-----------------------|------------------------------------------------------------------------------------|-----------------------------|--|--|--|--|
| Service Status | Meaning | Status Codes (0000,0900) | | | | |
| Failure | Processing Failure: Performed Procedure Step Object may no longer be updated | 0110 | | | | |
| | No such attribute | 0105 | | | | |
| | Invalid attribute value | 0106 | | | | |
| | No such SOP Instance | 0112 | | | | |
| | Invalid object instance | 0117 | | | | |
| | No such SOP class | 0118 | | | | |
| | Class instance conflict | 0119 | | | | |
| | Missing attribute value | 0121 | | | | |
| | Resource limitation | 0213 | | | | |
| Success | MPPS instance set | 0000 | | | | |

CHAPTER A.3

Communication Profiles

Supported Communication Stacks

The Somaris/7 DICOM application provides DICOM V3.0 TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard

The product target operating system is Windows XP.

OSI Stack

Not supported.

TCP/IP Stack

The Somaris/7 DICOM application uses the TCP/IP stack from the Windows XP system. It uses the MergeCOM-3 subroutine library from Merge Technologies Inc. that is based on a Berkeley socket interface.

API

The Somaris/7 DICOM application uses the MergeCOM library that is based on a TCP/IP socket interface.

Physical Media Support

The Somaris/7 DICOM application is indifferent to the physical medium over which TCP/IP executes. It inherits it from the target operating system upon which it executes.

Point-to-Point Stack

Not supported.

CHAPTER A.4

Extensions/Specializations/ Privatizations

Standard Extensions

Standard Extensions of all SOP Classes

The following tables list the data dictionary of all DICOM IOD attributes where the DICOM standard definitions are extended:

| Attribute Name | Tag | Private Creator | Type | Notes |
|----------------|-------------|------------------------|------|------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Image Type | (0008,0008) | - | 1 | see → page A.4–6, Image Type additional Defined Terms: |
| | | | | Defined Terms for value 3: OTHER MPR PROJECTION IMAGE UNDEFINED |
| | | | | Defined Terms for value 4: CSA * ^a CT_SOM4 * CT_SOM5 * ^b CT_SOM6 * CT_SOM7 * ECAT * SHS * |
| | | | | Somaris/7 based SOMATOM products provide a value 5 with the Defined Terms: ADD CTL FINISHED IN_WORK MRTD OTOM OTOP PBF PBV PKET PMON STD TTP TTS AVG, MIP, CBF, |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|------------------|-------------|-----------------|------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| | | | | CBV, CBVP,CBVA BV, BVP, TTSA MTTA, PMBP RSQP,CSQP, RSDP,CSDP, ALP |
| | | | | Somaris/7 based SOMATOM products provide a value 6 with the Defined Terms: DNRG SNRG Somaris/7 based SOMATOM products provide a value 7 with the Defined Terms: DET_A DET_B DET_AB |
| Patient Position | (0018,5100) | - | 2C | Additional Defined Terms for the Magnetom Open: HLS HLP FLS FLP HLDL HLDR FLDL FLDL FLDR |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-----------------------|-------------|-----------------|------|---------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Body Part Examined | (0018,0015) | | 3 | Additinal Defined Terms for the Somaris/7 based SOMATOM products: SPINE SPECIAL UNKNOWN SERVICE see → page A.4–14, Body Part Examined for further explanation |

a. For terms beginning with the stated prefix, e. g. "CSA", and ending with a "*" see → page A.4–6, Image Type.

All SOP classes may contain additional type 3 attributes which DICOM standard defines in a different DICOM IOD or DICOM SOP class (attributes from Normalized SOP classes).

This is the case for example for

- ☐ Rescale Slope (0028,1053)
- ☐ Rescale Intercept (0028,1052)

which are also used in the MR IOD.

b. For private extensions see page → page A.4–52, Private Elements for Storage SOP Classes.

Image Type

The Image Type (0008,0008) attribute identifies important image identification characteristics. These characteristics are:

1. Pixel Data Characteristics: ☐ Is the image an ORIGINAL Image; an image whose pixel values are based on original or source data, or ☐ is the image a DERIVED Image; an image whose pixel values have been derived in some manner from the pixel value of one or more other images. 2. Patient Examination Characteristics: ☐ Is the image a PRIMARY Image; an image created as a direct result of the Patient examination, or ☐ is the image a SECONDARY Image; an image created after the initial Patient examination. 3. Modality Specific Characteristics (SOP Specific Characteristics). 4. Implementation specific identifiers; other implementation specific identifiers shall be documented in an implementation's conformance claim. The Image Type attribute is multi-valued and shall be provided in the following manner: ☐ Value 1 shall identify the Pixel Data Characteristics: Enumerated Values for the Pixel Data Characteristics are: ☐ ORIGINAL = identifies an Original Image ■ DERIVED = identifies a Derived Image ☐ Value 2 shall identify the Patient Examination Characteristics: Enumerated Values for the Patient Examination Characteristics are: ☐ PRIMARY = identifies a Primary Image ☐ SECONDARY = identifies a Secondary Image ☐ Value 3 shall identify any Image IOD specific specialization, the following terms are defined in addition to the DICOM standard definitions: ☐ OTHER = converted non-Axial and non-Localizer CT images; images of no special type (new syntax)

☐ UNDEFINED = images of no special type (old syntax)

- $\hfill \square$ **Value 4** is implementation specific. The following terms are defined:
 - ☐ original *syngo* generated data set types:

CSA 3D EDITOR = object created by 3D Editor

CSA 3D FLY PATH = object created by Fly Through Path **CSA 3D FLY VRT** = object created by Fly Through Volume Rendering Technique

CSA 3D FUSION = object created by Fusion

CSA AVERAGE = image was created by Average

CSA BLACK IMAGE = SC Image with black pixels, only graphics information is of interest

CSA BOOKMARK = InSpace generated SC image containing bookmark information

CSA RESAMPLED = derived image created by zooming or panning original image

CSA REPORT = *syngo* Reporting (documentation of diagnosis)

CSA RESULT = *syngo* Reporting (post processing results)

CSA MIP = image created by Maximum Intensity Projection **CSA MIP THIN** = image created by Maximum Intensity

Projection Projection

CSA MIP THIN CV = image created by Maximum Intensity Projection (curved cut)

CSA MPR = image created by Multi Planar Reconstruction

CSA MPR CV = image created by Multi Planar

Reconstruction (curved cut)

CSA MPR THICK = image created by Multi Planar Reconstruction

CSA MPR THICK CV = image created by Multi Planar Reconstruction (curved cut)

CSA MPR THIN = image created by Multi Planar Reconstruction **CSA PSSD** = SC image as Perspective Shaded Surface Display

CSA SSD = SC Image as Shaded Surface Display

CSA SUBTRACT = image was created by Subtraction

CSA VRT = SC Image created by Volume Rendering Technique

ECAT ACF = CTI PET Attenuation Correction

ECAT NORMAL = CTI PET Normalization

ECAT 3D SINO = CTI PET 3D Sinogram Short

ECAT 3D SINO FLT = CTI PET 3D Sinogram Float

additional image types generated by Somaris/7 or Somaris/5 based SOMATOM products:

CT SOM7 PCT= PerfusionCT Image

CT SOM5 AVE = Averaged Image

CT SOM5 ICD = Interventional Cine Display Image

CT SOM5 MON = Monitoring Image

CT SOM5 MUL = Multiscan Image

CT_SOM5 PAR = Parameter Image

CT_SOM5 PROT = Protocol Image

CT SOM5 ROT = ROT Image

CT SOM5 RTD = Real Time Display Image

CT_SOM5 RTD DUAL = Common Somaris/7 Dual Tube Real Time Display Image

CT_SOM7 RTD DUAL = Specific Somaris/7 Dual Tube Real Time Display Image

CT SOM5 SEQ = Sequence Image

CT_SOM5 SEQ DUAL = Somaris/7 Dual Tube Sequence Image

CT SOM5 SPI = Spiral Image

CT_SOM5 SPI DUAL = Somaris/7 Dual Tube Spiral Image

CT_SOM7 SPI DUAL = Specific Somaris/7 Dual Tube Spiral Image

CT SOM5 SPO = Spiral Oblique Image

CT SOM5 STA = Static Image

CT_SOM5 SUB = Subtracted Image

CT SOM5 TOP = Topogram

CT SOM5 DPAN = Dental Panorama Rebuilt Tomogram

CT SOM5 DPAR = Dental Paraxial Rebuilt Tomogram

CT SOM5 DFLM = Dental Filming Image

CT_SOM5 DYB = Dynamic Evaluation Averaged Baseline Image

CT_SOM5 DYF = Dynamic Evaluation Fused Multislice Image

CT SOM5 PEVI = Pulmo Evaluation Image

CT SOM5 OEVA = Osteo Evaluated Tomogram

CT_SOM5 MIP = Maximum Intensity Projection image created by a CT application

CT_SOM5 MPR = Multi Planar Reconstruction image created by a CT application

CT_SOM5 REP = Lung Care Report Image

CT_SOM7 VPCT = Volume Perfusion CT

CT SOM7 FLU= Fluoroscopic

□ converted images

CT_SOM4 NONE = converted SOMARIS image

CT_SOM4 CONV = converted SOMARIS Convolution

Kernel file

CT_SOM4 DART = converted SOMARIS Dental Artificial image

CT SOM4 DEVA = converted SOMARIS Dental

Evaluation image

CT_SOM4 DGRA = converted SOMARIS Dental Graphics image

CT_SOM4 DMEA = converted SOMARIS Dynamic

Measurement image

CT SOM4 DPAN = converted SOMARIS Dental

Panorama image

CT_SOM4 DPAR = converted SOMARIS Dental Paraxial image

CT SOM4 EBT = converted SOMARIS Evolution image

CT_SOM4 HIS = converted SOMARIS Histogram

Graphics image

CT_SOM4 HISC = converted SOMARIS Histogram

Graphics image

CT SOM4 MUL = converted SOMARIS Multiscan image

CT SOM4 OEVA = converted SOMARIS Osteo

Evaluation image

CT SOM4 OTOM = converted SOMARIS Osteo

Tomogram image

CT SOM4 OTOP = converted SOMARIS Osteo

Topogram image

CT SOM4 PLOT = converted SOMARIS Plot image

CT SOM4 QUAL = converted SOMARIS Quality image

CT SOM4 R2D = converted SOMARIS 2D Rebuild image

CT SOM4 R3D = converted SOMARIS 3D Rebuild image

CT SOM4 R3DE = converted SOMARIS 3D Rebuild image

CT SOM4 RMAX = converted SOMARIS Maximum

Intensity Projection image

CT_SOM4 RMIN = converted SOMARIS Minimum Intensity Projection image

CT_SOM4 ROT = converted SOMARIS Rotation Mode image

CT_SOM4 RRAD = converted SOMARIS Radiographic Projection image

CT_SOM4 RVIT = converted SOMARIS Vessel Image Tool image

CT_SOM4 RVRT = converted SOMARIS Volumetric Rendering image

CT_SOM4 SAVE = converted SOMARIS Evolution Screen Save image

CT_SOM4 SCAN = converted SOMARIS Standard Mode image

CT_SOM4 SEQ = converted SOMARIS Sequence Mode image

CT_SOM4 SER = converted SOMARIS Serial Mode image

CT_SOM4 SIN = converted SOMARIS Sinogram image

 $\textbf{CT_SOM4 SINC} = \textbf{converted SOMARIS Sinogram image}$

CT_SOM4 SPI = converted SOMARIS Spiral Mode image CT_SOM4 STA = converted SOMARIS Static Mode image

CT_SOM4 TAB = converted SOMARIS Correction Table image

CT_SOM4 TOP = converted SOMARIS Topogram image CT_SOM4 GTOP = converted SOMARIS Topo Graphics image

CT_SOM4 PEVG = converted SOMARIS Pulmo Evaluation image

CT_SOM4 PEVI = converted SOMARIS Pulmo Evaluation image

CT_SOM4 PUL = converted SOMARIS Pulmo Respiration curve

CT_SOM4 PROT = converted SOMARIS Protocol image

CT_SOM4 TEXT = converted SOMARIS Text image

CT_SOM4 ICD = converted SOMARIS Interventional Cine image

SHS DENT = converted MagicView Dental Tomogram image

SHS DPAN = converted MagicView Dental Panorama image

SHS DPAR = converted MagicView Dental Paraxial image

SHS 3D_CURVED = converted MagicView image

SHS 3D_MIP = converted MagicView Maximum Intensity Projection image

SHS 3D_MPR = converted MagicView Multi Planar Reconstruction image

SHS 3D_SSD = converted MagicView Shaded Surface Display image

SHS 3D_VRT = converted MagicView Volumetric Rendering image

■ Value 5 is specific for the Somaris/7 or Somaris/5 based SOMATOM products. In special cases (3D postprocessing) values mentioned for a lower index may appear for value 5 or higher. This will refer to 3D postprocessing base image types.

The following terms are defined:

ADD = Additional Scan

ALP = Arterial Liver Perfusion

AVG= Average

BF= NeuroVPCT Blood Flow

BFI = NeuroVPCT Blood Flow - IncGamma

BV= NeuroVPCT Blood Volume

BVP = NeuroVPCT Blood Volume - Patlak

CBF = NeuroVPCT Blood Flow

CBFA = NeuroVPCT Blood Flow - Alternate

CBFI = NeuroVPCT Blood Flow - IncGamma

CBV= NeuroVPCT Blood Volume

CBVA = NeuroVPCT Blood Volume - Alternate

CBVP = NeuroVPCT Blood Volume - Patlak

CSQP = Patlak ChiSquare

CTL = Control Scan

HPI = Hepatic Perfusion Index

FINISHED = Lung Care Report Image (finished)

IN WORK = Lung Care Report Image (not finished)

MIP = Maximum Intensity Projection

MRTD = Multiscan Real Time Display Image

MTTA = Mean Transit Time - Alternate

OTOM = Osteo Scanned Tomogram

OTOP = Osteo Scanned Topogram

PBF = Perfusion Blood Flow Image

PBV = Perfusion Blood Volume Image

PKET = Peak Enhancement Parameter Image

PMBA = Permeability - Alternate

PMBP = Permeability

PMON = Premonitoring Scan

PEAK = NeuroVPCT Peak Enhancement

PVP = Portal Venous Liver Perfusion

RSDP = Patlak Residual

RSQP = Patlak RSquare

RT3D CONFIG = InSpace Configuration Image

STD = Standard image of corresponding Type 4

TTDA = Time to Drain - Alternate

TTP = Time to Peak Parameter Image

TTS = Time to Start Parameter Image

TTSA = NeuroVPCT Time to Start Parameter Image

□ Value 6 is specific for the Somaris/7 based SOMATOM products. For Dual Tube scans there is a more detailed distinction here.

The following terms are defined:

DNRG = Dual Energy

SNRG = Single Energy

□ Value 7 is specific for the Somaris/7 based SOMATOM products. For Dual Energy scans there is a more detailed distinction here.

The following terms are defined:

DET A = Only data of detector A used

DET B = Only data of detector B used

DET_AB = Data is derived from detector A and detector B

Body Part Examined

The Body Part Examined (0018,0015) attribute provides a textual description of the part of the body examined. The Somaris/7 based SOMATOM products extend the Defined Terms:

- □ SPINE = Summary term used instead of the Defined Terms CSPINE, TSPINE, LSPINE, and SSPINE
- ☐ SPECIAL = Image was acquired with acquisition modes that are not mapped to a certain part of the body
- ☐ SERVICE = Image was acquired for maintenance purpose
- ☐ **UNKNOWN** = No information about the body part available

See \rightarrow page A.4–39, Somaris/7 Attribute Interpretation for a mapping of the organ characteristics used for examination to the Body Part Examined terms.

In addition, the user interface permits the definition of new terms by the user. So in fact any syntactically correct value may be present as a value of this attribute. It is recommended, though, to use the DICOM defined terms when appropriate.

RGB color images

| The Somaris/7 DICOM application extends the CT Image IOD by the use of RGB color image description with the unsigned integer 24 bit color image plane pixel format: □ samples per pixel (attribute 0028, 0002) = 3 □ photometric interpretation (attribute 0028,0004) = "RGB" □ pixel representation (attribute 0028, 0103) = 0 □ bits allocated (attribute 0028, 0100) = 8 □ bits stored (attribute 0028,0101) = 8 □ high bit (attribute 0028,0102) = 7 □ planar configuration (attribute 0028,0006) = 0 |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| This format is used for Functional Imaging, i.e. images that meaningfully use all common CT Image attributes - however the pixel values do not represent a scaled Hounsfield value but a different value (depending on the type of image). Thus window related attributes must not be used to interpret the pixel values as scaled HU. The values used by Somaris/7 are: □ window center (attribute 0028, 1050) = 128 □ window width (attribute 0028, 1051) = 256 □ rescale intercept (attribute 0028, 1050) = 0 □ rescale slope (attribute 0028, 1051) = 1 |

The following types of images may use this format:

Somaris/7 Image Type (0008,0008) for objects created by Somaris/7

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|------------------------------------------------------|-----|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------|
| Averaged Image | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 AVE | none | AVE |
| Parameter Image (Patlak Blood Volume) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PKBV | PAR |
| Parameter Image (Peak enhanceme nt) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PKET | PAR |
| Parameter Image (Perfusion Blood Flow) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PBF | PAR |
| Parameter Image (Perfusion Blood Volume) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PBV | PAR |
| Parameter Image (Permeabili ty) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PMB | PAR |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|----------------------------------------------------------------|-----|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------|
| Parameter Image (Time to Peak) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | TTP | PAR |
| Parameter Image (Time to Start) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | TTS | PAR |
| Parameter Image (Patlak Residual) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PKER | PAR |
| Parameter Image (Patlak RSquare) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PKR2 | PAR |
| Parameter Image (Arterial Liver Perfusion) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | ALP | PAR |
| Parameter Image (Portal Venous Liver Perfusion) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | PVP | PAR |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|---------------------------------------------------------------------|-----|----------------------|----------------------|----------------------------------------|----------------------|----------------------|----------------------------|
| Parameter Image (Hepatic Perfusion Index) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 PAR | HPI | PAR |
| PerfusionCT Image (Maximum Intensity Projection) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | MIP | VPCT |
| PerfusionCT Image (Average) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | AVG | VPCT |
| PerfusionCT Image (NeuroVPC T Blood Flow) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CBF | VPCT |
| PerfusionCT Image (NeuroVPC T Blood Flow - IncGamma) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CBFI | VPCT |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image |
|----------------------------------------------------------------------|-----|----------------------|----------------------|----------------------------------------|----------------------|----------------------|---------------------|
| PerfusionCT Image (NeuroVPC T Blood Flow - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CBFA | Text VPCT |
| PerfusionC T Image (BodyVPCT Blood Flow) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | BF | VPCT |
| PerfusionC T Image (BodyVPCT Blood Flow - IncGamma) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | BFI | VPCT |
| PerfusionC T Image (BodyVPCT Blood Flow - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | BFA | VPCT |
| PerfusionC T Image (Peak enhanceme nt) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | PEAK | VPCT |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|-------------------------------------------------------------------------|-----|----------------------|----------------------|----------------------------------------|----------------------|----------------------|----------------------------|
| PerfusionC T Image (NeuroVPC T Blood Volume) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CBV | VPCT |
| PerfusionC T Image (NeuroVPC T Blood Volume - Patlak) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CBVP | VPCT |
| PerfusionC T Image (NeuroVPC T Blood Volume - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CBVA | VPCT |
| PerfusionC T Image (BodyVPCT Blood Volume) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | BV | VPCT |
| PerfusionC T Image (BodyVPCT Blood Volume - Patlak) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | BVP | VPCT |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|-----------------------------------------------------------------------|-----|----------------------|----------------------|----------------------------------------|----------------------|----------------------|----------------------------|
| PerfusionC T Image (BodyVPCT Blood Volume - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | BVA | VPCT |
| PerfusionC T Image (Time to Start) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | TTS | VPCT |
| PerfusionC T Image (Time to Start - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | TTSA | VPCT |
| PerfusionC T Image (Time to Peak) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | TTP | VPCT |
| PerfusionC T Image (Time To Drain Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | TTDA | VPCT |

| | | | | <u> </u> | | | |
|-------------------------------------------------------------------|-----|----------------------|----------------------|----------------------------------------|----------------------|----------------------|----------------------------|
| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
| PerfusionC T Image (Mean Transit Time - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | MTTA | VPCT |
| PerfusionC T Image (Permeabili ty) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | PMBP | VPCT |
| PerfusionC T Image (Permeabili ty - Alternate) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | PMBA | VPCT |
| PerfusionC T Image (Patlak RSquare) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | RSQP | VPCT |
| PerfusionC T Image (Patlak ChiSquare) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | CSQP | VPCT |
| PerfusionC T Image (Patlak Residual) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | RSDP | VPCT |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|-------------------------------------------------------------------|-----|----------------------|----------------------|----------------------------------------|----------------------|----------------------|----------------------------|
| PerfusionC T Image (Arterial Liver Perfusion) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | ALP | VPCT |
| PerfusionC T Image (Portal Venous Liver Perfusion) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | PVP | VPCT |
| PerfusionC T Image (Hepatic Perfusion Index) | СТ | DERIVED | SECOND ARY | AXIAL, SAGITTA L, CORONA L | CT_SOM 7 VPCT | HPI | VPCT |
| Screenshot Image | SC | DERIVED | SECOND ARY | OTHER | VPCT | none | VPCT |
| Max Intensity Projection Image (Perfusion) | СТ | DERIVED | SECOND ARY | AXIAL | CT_SOM 5 MIP | | MIP |
| LungCARE send-to- Filming images | СТ | DERIVED | SECOND ARY | OTHER | CSA MIP THIN | LC VALID WINDOW | MIP |

| Image Type Description | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text |
|-------------------------------------------|-----|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------|
| LungCARE send-to- Filming images | СТ | DERIVED | SECOND ARY | OTHER | CSA MPR THICK | LC VALID WINDOW | MPR |
| LungCARE send-to- Filming images | СТ | DERIVED | SECOND ARY | OTHER | CSA VRT THIN | LC VALID WINDOW | VRT |
| LungCARE send-to- Filming images | CT | DERIVED | SECOND ARY | OTHER | CSA PRVT | LC VALID WINDOW | PVRT |
| LungCARE send-to- Filming images | СТ | DERIVED | SECOND ARY | OTHER | CSA MPR | LC VALID WINDOW | MPR |

CT images with Color Lookup Table

Images deriving from the Application Neuro PBV are created as CT Images with the Photometric Interpretation (0028,0004) set to "MONOCHROME2". Additionally there is a Color Lookup Table stored in the Private Header.

| Som | Somaris/7 Image Type (0008,0008) for objects <u>created</u> by Somaris/7 | | | | | | | | | |
|------------------------------------------------------|--------------------------------------------------------------------------|--------------------------|----------------------|--------------------------|--------------------------|--------------------------|-----------------------------------------|--|--|--|
| Image Type Description [known creating applications] | IOD | 0008,000 8 Value 1 | 0008,0008 Value 2 | 0008,000 8 Value 3 | 0008,000 8 Value 4 | 0008,000 8 Value 5 | Somaris /7 Image Text or Lists | | | |
| Neuro PBV (Perfused Blood Volume) | CT | DERIVE D | SECONDA RY | AXIAL | NEUROP BV | none | none | | | |
| Neuro PBV (Perfusion Weighted Map) | СТ | DERIVE D | SECONDA RY | AXIAL | NEUROP M | none | none | | | |

Specializations

Images created by Somaris/7

The following table lists the Somaris/7 image types and the corresponding combinations of the Image Type Attribute values:

| Som | Somaris/7 Image Type (0008,0008) for objects created by Somaris/7 | | | | | | | | | |
|---------------------------------------------------------|-------------------------------------------------------------------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|--|--|--|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists | | | |
| 3D MPR | SC | DERIVED | SECONDA RY | LOCALIZ ER | CSA MPR | none | MPR | | | |
| 3D MPR THICK | SC | DERIVED | SECONDA RY | LOCALIZ ER | CSA MPR THICK | none | MPR THICK | | | |
| 3D MIP | SC | DERIVED | SECONDA RY | LOCALIZ ER | CSA MIP | none | MIP | | | |
| 3D MIP THIN | SC | DERIVED | SECONDA RY | LOCALIZ ER | CSA MIP THIN | none | MIP THIN | | | |
| Averaged Image [Average, DynEva, Perfusion] | CT | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 AVE | none | AVE | | | |
| Interventional Cine Display Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 ICD | none | ICD | | | |
| Monitoring Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 MON | none | MON | | | |
| Premonitoring Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 MON | PMON | MON | | | |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | , |
|---------------------------------------------------------------------------------------|--------|-----------------------------------------|----------------------|---------------|---------------------|-------------|-----|
| Image Type Description [known creating applications] | | • • • • • • • • • • • • • • • • • • • • | 0008,0008 Value 2 | • | • | | |
| Multiscan Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 MUL | none | MUL |
| Parameter Image (Arterial Liver Perfusion) [BodyPerfusio n] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | ALP | PAR |
| Parameter Image (Hepatic Perfusion Index) [BodyPerfusio n] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | HPI | PAR |
| Parameter Image (Portal Venous Liver Perfusion) [BodyPerfusio n] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PVP | PAR |
| Protocol Image, Time Density Curve Image [BodyPerfusio n] ^a | SC | DERIVED | SECONDA RY | OTHER | none | none | AC |

| Som | Somaris/7 Image Type (0008,0008) for objects <u>created</u> by Somaris/7 | | | | | | | | | |
|----------------------------------------------------------------|--------------------------------------------------------------------------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|--|--|--|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists | | | |
| Parameter Image (Patlak Blood Volume) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PKBV | PAR | | | |
| Parameter Image (Patlak Residual) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PKER | PAR | | | |
| Parameter Image (Patlak RSquare) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PKR2 | PAR | | | |
| Parameter Image (Peak enhancement) [DynEva, Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PKET | PAR | | | |
| Parameter Image (Perfusion Blood Flow) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PBF | PAR | | | |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | |
|----------------------------------------------------------------|--------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| Parameter Image (Perfusion Blood Volume) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | PBV | PAR |
| Parameter Image (Permeability) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | РМВ | PAR |
| Parameter Image (Time to Peak) [DynEva, Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | TTP | PAR |
| Parameter Image (Time to Start) [Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PAR | TTS | PAR |
| Protocol Image | SC | DERIVED | SECONDA RY | OTHER | CT_SOM 5 PROT | none | PROT (List only) |
| Report Image (not finished) | SC | DERIVED | SECONDA RY | OTHER | CT_SOM 5 REP | IN_WORK | REP |
| Report Image (finished) | SC | DERIVED | SECONDA RY | OTHER | CT_SOM 5 REP | FINISHED | REP |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | , |
|------------------------------------------------------------------|--------|----------------------|----------------------------|----------------------|----------------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| ROT Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 ROT | none | ROT |
| Real Time Display Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 RTD | none | RTD |
| Real Time Display Image (Dual Tube), Som5 compatible | CT | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 RTD DUAL | STD | RTD DUAL |
| Real Time Display Image (Cardio) | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 7 RTD DUAL | STD | RTD DUAL |
| Multiscan Real Time Display Image | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 RTD | MRTD | RTD |
| Sequence Image | СТ | ORIGINA L | PRIMARY / SECONDA RY | AXIAL | CT_SOM 5 SEQ | none | SEQ |
| Sequence Image (Dual Tube) | СТ | ORIGINA L | PRIMARY/ SECONDA RY | AXIAL | CT_SOM 5 SEQ DUAL | STD | SEQ DUAL |
| Additional Scan Image | СТ | ORIGINA L | PRIMARY / SECONDA RY | AXIAL | CT_SOM 5 SEQ | ADD | SEQ |

| Som | naris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | , |
|------------------------------------------------------|---------|----------------------|----------------------------|----------------------|-------------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| Control Scan Image | СТ | ORIGINA L | PRIMARY / SECONDA RY | AXIAL | CT_SOM 5 SEQ | CTL | SEQ |
| Spiral Image | СТ | ORIGINA L | PRIMARY / SECONDA RY | AXIAL | CT_SOM 5 SPI | none | SPI |
| Spiral Image (Dual Tube) | СТ | ORIGINA L | PRIMARY/ SECONDA RY | AXIAL | CT_SOM 5 SPI DUAL | STD | SPI DUAL |
| Spiral Image (Cardio) | СТ | ORIGINA L | PRIMARY/ SECONDA RY | AXIAL | CT_SOM 7 SPI DUAL | STD | SPI DUAL |
| Spiral Oblique Image | СТ | DERIVED | PRIMARY/ SECONDA RY | AXIAL | CT_SOM 5 SPO | none | SPO |
| Static Image | СТ | ORIGINA L | PRIMARY | OTHER | CT_SOM 5 STA | none | STA |
| Subtracted Image | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 SUB | none | SUB |
| Topogram | СТ | ORIGINA L | PRIMARY | LOCALIZ ER | CT_SOM 5 TOP | none | TOP |
| Osteo Scanned Tomogram | СТ | ORIGINA L | PRIMARY | AXIAL | CT_SOM 5 SEQ | ОТОМ | SEQ |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | • |
|------------------------------------------------------|--------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| Osteo Scanned Topogram | СТ | ORIGINA L | PRIMARY | LOCALIZ ER | CT_SOM 5 TOP | OTOP | TOP |
| Osteo Evaluated Tomogram | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 OEVA | none | OEVA |
| Pulmo Evaluated Tomogram | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 PEVI | none | PEVI |
| Calcium Scoring Table [Calcium Scoring] | SC | DERIVED | SECONDA RY | OTHER | CT_SOM 5 TAB | none | TAB |
| Dental Filming Image | СТ | DERIVED | SECONDA RY | OTHER | CT_SOM 5 DFLM | none | DFLM |
| Dental Panorama Rebuild Tomogram | CT | DERIVED | SECONDA RY | OTHER | CT_SOM 5 DPAN | none | DPAN |
| Dental Paraxial Rebuild Tomogram | СТ | DERIVED | SECONDA RY | OTHER | CT_SOM 5 DPAR | none | DPAR |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | , |
|---------------------------------------------------------------------------------------|--------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| Dental/ Volume Maximum Intensity Projection Image [Dental, Volume, DynEva, Perfusion] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 MIP | none | MIP |
| Dental Panorama Reference Image [Dental, Volume] | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 MPR | none | MPR |
| Dental Paraxial Reference Image | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 MPR | none | MPR |
| Dental Reference Image | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 MPR | none | MPR |
| Dynamic Evaluation Averaged Baseline | CT | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 DYB | none | DYB |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | ' |
|--------------------------------------------------------------|--------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| Dynamic Evaluation Fused Multislice | СТ | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 DYF | none | DYF |
| Volume reformatted images (sagittal and coronal) | СТ | DERIVED | SECONDA RY | OTHER | CT_SOM 5 MPR | none | MPR |
| Various result images [CalciumScori ng, Colon] | СТ | DERIVED | SECONDA RY | AXIAL | CSA MPR | none | MPR |
| Various result images | СТ | DERIVED | SECONDA RY | AXIAL | CSA MPR THICK | none | MPR |
| Various result images | СТ | DERIVED | SECONDA RY | AXIAL | CSA MIP | none | MIP |
| Various result images [CalciumScori ng] | СТ | DERIVED | SECONDA RY | AXIAL | CSA MIP THIN | none | MIP |
| Various result images [Colon] | SC | DERIVED | SECONDA RY | OTHER | CSA PSSD | none | PSSD |
| Various result images | СТ | DERIVED | SECONDA RY | AXIAL | CSA VRT | none | VRT |
| Various result images [Colon] | SC | DERIVED | SECONDA RY | OTHER | CSA VRT | none | VRT |

| Som | aris/7 | 7 Image Typ | e (0008,0008 | 3) for object | s <u>created</u> by | y Somaris/7 | , |
|------------------------------------------------------|--------|----------------------|----------------------|----------------------|--------------------------|----------------------|----------------------------------------|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists |
| InSpace bookmark [InSpace] | SC | DERIVED | SECONDA RY | OTHER | CSA BOOKMA RK | RT3D CONFIG | ВООК |
| InSpace result images [InSpace] | SC | DERIVED | SECONDA RY | OTHER | CSA 3DPRO- JECTION | none | 3DPR |
| LungCARE save images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | MIP | LC VALID WINDOW | MIP |
| LungCARE save images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | MPR | LC VALID WINDOW | MPR |
| LungCARE save images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | VRT | LC VALID WINDOW | VRT |
| LungCARE save images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | PVRT | LC VALID WINDOW | PVRT |
| LungCARE save images [LungCARE] | СТ | DERIVED | SECONDA RY | OTHER | MPR | LC VALID WINDOW | MPR |
| LungCARE save images [LungCARE] | СТ | DERIVED | SECONDA RY | OTHER | CT_SOM 5 SPI | LC VALID WINDOW | SPI |
| LungCARE report images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | MIP | LC VALID WINDOW | none |

| Som | Somaris/7 Image Type (0008,0008) for objects <u>created</u> by Somaris/7 | | | | | | | |
|-------------------------------------------------------------------------------------|--------------------------------------------------------------------------|----------------------|----------------------|-------|-------------------------|--------------------|------|--|
| Image Type Description [known creating applications] | | 0008,0008 Value 1 | 0008,0008 Value 2 | • | 0008,0008 Value 4 | = | | |
| LungCARE report images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | MPR | LC VALID WINDOW | none | |
| LungCARE report images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | VRT | LC VALID WINDOW | none | |
| LungCARE report images [LungCARE] | SC | DERIVED | SECONDA RY | OTHER | PVRT | LC VALID WINDOW | none | |
| Various Graphics [DynEva, Osteo, Pulmo, Volume, Perfusion, Argus] | SC | DERIVED | SECONDA RY | OTHER | CSA BLACK IMAGE | none | none | |
| Dual Energy Mixed Series [Dual Energy] | SC | DERIVED | SECONDA RY | AXIAL | CT_SOM 5 SPI DUAL | STD | none | |

a. More detailed attribute information will be provided in a future version.

Note

Some applications will create Structured Reports. However, when running in specific (service configured) settings outside of the system they will appear as a SC image, labelled as Type 3 "OTHER" and Type 4 "CSA REPORT". This private extension is not published in detail here because the intended use is real DICOM SR.

Structured Reports created by Somaris/7

The following table lists the Somaris/7 image types and the corresponding combinations of the Image Type Attribute values.

| Som | Somaris/7 Image Type (0008,0008) for objects <u>created</u> by Somaris/7 | | | | | | | |
|-----------------------------------------------------------------------------|--------------------------------------------------------------------------|----------------------|----------------------|----------------------|----------------------|----------------------|----------------------------------------|--|
| Image Type Description [known creating applications] | IOD | 0008,0008 Value 1 | 0008,0008 Value 2 | 0008,0008 Value 3 | 0008,0008 Value 4 | 0008,0008 Value 5 | Somaris/7 Image Text or Lists | |
| LungCARE SR reports [LungCARE, Calcium Scoring, Circulation] | SR | ORIGINAL | PRIMARY | OTHER | CSA REPORT | none | none | |

LungCARE, Calcium Scoring and Circulation create Comprehensive Structured Reports.

The following is valid for Calcium Scoring created Structured Reports:

CaScoring creates structured reports generally following the templates defined in DICOM supplement 97 "CT/MR Cardiovascular Analysis Report" (Version: 0.11; Working Draft). Some changes of these templates that could not be considered may occur until document is released as final version.

The following is valid for Circulation created Structured Reports:

Circulation creates structured reports generally following the DICOM template TID3900.

Somaris/7 Attribute Interpretation

For an overview of selected attribute filling for different Image Types please refer to the Annex.

The table below explains how common attributes of created and uncompressed ORIGINAL and AXIAL images are set by Somaris/7 image reconstruction. This table does not intend to be a substitute for DICOM 3.0 attribute definitions. However, in addition to these it will provide an impression of which attributes are in common use - and how they are related to Somaris/7 examination:

| Tag | Name | Explanation |
|-----------|--------------------------------|-----------------------------------------------------------------------------------------------------------|
| 0002,0012 | Implementation Class UID | Value: 1.3.12.2.1107.5.1.4 |
| 0002,0013 | Implementation Version Name | Value: SIEMENS_S7VA20A |
| 0008,0005 | Specific Character Set | |
| 0008,0008 | Image Type | see \rightarrow page A.4–26, Images created by Somaris/7 |
| 0008,0016 | SOP Class UID | Value: 1.2.840.10008.5.1.4.1.1.2 |
| 0008,0018 | SOP Instance UID | created |
| 0008,0020 | Study Date | For existing studies their date and time |
| 0008,0030 | Study Time | entries are copied into the corresponding entries in a new image's header. |
| | | If a new study is created the date and time entries from the first series of this new study will be used. |

| Tag | Name | Explanation |
|-----------|----------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 0008,0021 | Series Date | For existing series their date and time |
| 0008,0031 | Series Time | entries are copied into the corresponding entries in a new image's header. |
| | | If a new series is created the current real-world date and time will be used. |
| 0008,0022 | Acquisition Date | Acquisition Date and Time is defined |
| 0008,0032 | Acquisition Time | as the real-world beginning of the accumulation of data which contribute to a particular image. Due to the multislice technology several images may have the same Acquisition Date and Time. |
| 0008,0023 | Image (Content) Date | For all images which result from a |
| 0008,0033 | Image (Content) Time | reconstruction of acquired data, the time stamp is the same as Acquisition Date and Time. This is true for images that were reconstructed immediately after data acquisition as well as for images that were reconstructed at any time later. For all other images created by any other means the time stamp is derived from a point in time during the creation process of these images. |
| 0008,0050 | Accession Number | Input entered from MWL or during patient registration. May be null-length. |
| | | NOTE: When Study Split is used, the value corresponding to the Requested Procedure Description selected in Exam UI will be sent |
| 0008,0060 | Modality | Value: CT |
| 0008,0070 | Manufacturer | Value: SIEMENS |

| Tag | Name | Explanation |
|-----------|------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 0008,0080 | Institution Name | Hospital name read from configuration data or user input entered during patient registration or examination setup |
| 0008,0081 | Institution Address | Hospital address read from configuration data. Default format is 4 lines containing Street, City, District and Country. |
| 0008,0090 | Referring Physician's Name | Input from MWL or entered during patient registration or examination setup. May be null-length. |
| 0008,1010 | Station Name | Name of the computer the examination is controlled with |
| 0008,1030 | Study Description | Is derived from information entered during patient registration or examination setup. Concatenated from Body Region and selected Scan Protocol name, separated by a "^" |
| | | NOTE: When Study Split is used, the Requested Procedure Description selected in Exam UI will be sent |
| 0008,1050 | Performing Physician's Name | Input entered from MWL or during patient registration. |
| 0008,1070 | Operator's Name | According to user input |
| 0008,1080 | Admitting Diagnoses Description | Input entered from MWL or during patient registration. |
| 0008,103E | Series Description | According to user input (system proposes <range name=""> <slice thickness=""> <kernel>)</kernel></slice></range> |
| 0008,1090 | Manufacturer's Model Name | The Siemens product name |

| Tag | Name | Explanation |
|------------|--------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 0008,1140 | Referenced Image Sequence | |
| >0008,1150 | Referenced SOP Class UID | SOP Class UID of Topogram used for planning |
| >0008,1155 | Referenced SOP Instance UID | SOP Instance UID of Topogram used for planning |
| 0008,2112 | Source Image Sequence | |
| >0008,1150 | Referenced SOP Class UID | SOP Class UID of Rawdata used for reconstruction |
| >0008,1155 | Referenced SOP Instance UID | SOP Instance UID of Rawdata used for reconstruction |
| 0008,2111 | Derivation Description | Lossy Image Compression will be set to 01 for Extended Field of View because of low image quality. Derivation Description: "Reconstruction field larger than scan field" |
| 0009,00xx | Private Creator | Value: SIEMENS CT VA1 DUMMY |
| 0010,0010 | Patient's Name | Input from MWL or entered during patient registration or examination setup. |
| 0010,0020 | Patient ID | Input from MWL or entered or generated during patient registration or examination setup. |
| 0010,0030 | Patient's Birth Date | Input from MWL or entered during patient registration or examination setup. May be calculated from age. |
| 0010,0040 | Patient's Sex | Input from MWL or entered during patient registration or examination setup. |

| Tag | Name | Exp | lanation |
|-----------|----------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 0010,1010 | Patient's Age | Input from MWL or entered during patient registration or examination setup. May be calculated from Patient Birth Date | |
| 0018,0010 | Contrast/Bolus Agent | If present: The attribute is filled with the value entered in the contrast card. If nothin was entered the attribute is filled with the fixed term "APPLIED". | |
| 0018,0015 | Body Part Examined | the organ chara directly entered overrid. The organ char contained in the for examina registration or emay vary for diff protocol (use S Protocols colum values for spe | camined is derived from acteristic of the scan or by the operator, which es this default. Cacteristic parameter is scan protocol selected ation during patient examination setup and erent scans within one system/Run/List Scan n "OrgChar" for a list of cific scan protocols). Ceristic Body Part HEAD NECK SHOULDER CHEST ABDOMEN SPINE PELVIS EXTREMITY HEART SERVICE UNKNOWN |

| Tag | Name | Explanation |
|-----------|--------------------------------|-----------------------------------------------------------------------------------------------------------------|
| 0018,0022 | Scan Options | This field is used for storing cardiac information. |
| 0018,0050 | Slice Thickness | Resulting slice thickness. Not necessarily equal to collimated slice. |
| 0018,0060 | KVP | Voltage selected for scan. |
| 0018,0090 | Data Collection Diameter | |
| 0018,1000 | Device Serial Number | Serial number of the CT system's gantry. |
| 0018,1020 | Software Version(s) | Software Version of Somaris/7. Only one value is used. |
| 0018,1030 | Protocol Name | Name of Scan Protocol selected during patient registration or examination setup |
| 0018,1100 | Reconstruction Diameter | Field of View selected for reconstruction |
| 0018,1110 | Distance Source to Detector | |
| 0018,1111 | Distance Source to Patient | |
| 0018,1120 | Gantry/Detector Tilt | |
| 0018,1130 | Table Height | Table height as entered for scan |
| 0018,1140 | Rotation Direction | |
| 0018,1150 | Exposure Time | The exposure time for the respective image. For spiral scan or multiscan this corresponds to the rotation time. |
| 0018,1151 | X-ray Tube Current | Tube current used for scanning |

| Tag | Name | Explanation | |
|-----------|-----------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------|--|
| 0018,1152 | Exposure | (Effective) mAs. For spiral images the product of exposure time and x-ray tube current is corrected with Table Feed per Rotation. Expressed in mAs. | |
| 0018,1160 | Filter Type | 0: No Filter, 1: Teflon Filter | |
| 0018,1170 | Generator Power | | |
| 0018,1190 | Focal Spots | Size of the focal spot actually used to generate x-ray radiation for a particular image. This is a single value expressed in mm. | |
| 0018,1200 | Date of Last Calibration | Date of last Base Calibration | |
| 0018,1201 | Time of Last Calibration | Time of last Base Calibration | |
| 0018,1210 | Convolution Kernel | 4 character code of convolution kernel | |
| 0018,5100 | Patient Position | As entered for scan | |
| 0018,9306 | Single Collimation Width | As entered for scan | |
| 0018,9307 | Total Collimation Width | As entered for scan | |
| 0018,9309 | Table Speed | The distance in mm that the table moves in one second during the gathering of data that resulted in this image. | |
| 0018,9310 | Table Feed per Rotation | As entered for scan | |
| 0018,9311 | CT Pitch Factor | As entered for scan | |
| 0018,9323 | Exposure Modulation Type | As entered for scan | |

| Tag | Name | Explanation |
|-----------|---------------------------------|---------------------------------------------------------------------------------------------------------------------------------|
| 0018,9324 | Estimated Dose Saving | calculated by the system |
| 0018,9345 | CTDIvol | calculated by the system |
| 0019,00xx | Private Creator | Value: SIEMENS CT VA0 COAD |
| 0019,xxB0 | Feed per Rotation | Movement of table during one rotation in mm |
| 0020,000D | Study Instance UID | From MWL or created |
| | | NOTE: When Study Split is used, the value corresponding to the Requested Procedure Description selected in Exam UI will be sent |
| 0020,000E | Series Instance UID | From MWL or created |
| 0020,0010 | Study ID | From MWL or created |
| | | NOTE: When Study Split is used, the value corresponding to the Requested Procedure Description selected in Exam UI will be sent |
| 0020,0011 | Series Number | created |
| 0020,0012 | Acquisition Number | Scan number within this examination. |
| 0020,0013 | Instance Number | created; direction within range selectable by user |
| 0020,0032 | Image Position (Patient) | The x, y, and z coordinates of the center of the first pixel, in mm. |
| 0020,0037 | Image Orientation (Patient) | |
| 0020,0052 | Frame of Reference UID | created |
| 0020,1040 | Position Reference Indicator | null-length |

| Tag | Name | Explanation | |
|-----------|-------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|
| 0020,1041 | Slice Location | Topogram: Slice Location is defined as the relative table position of the very beginning of the Topogram image expressed in mm. Tomogram: Slice Location is defined as the relative position of the intersection of the image's slice with the z-axis expressed in mm. This positions is relative to the current reference point and corresponds to the table position. | |
| 0020,4000 | Image Comments | According to user input. 2 lines supported. | |
| 0021,00xx | Private Creator | Value: SIEMENS MED | |
| 0021,xx11 | Target | Center x/y as entered for reconstruction | |
| 0028,0002 | Samples per Pixel | Value: 1 | |
| 0028,0004 | Photometric Interpretation | Value: MONOCHROME2 | |
| 0028,0010 | Rows | | |
| 0028,0011 | Columns | | |
| 0028,0030 | Pixel Spacing | | |
| 0028,0100 | Bits Allocated | Value: 16 | |
| 0028,0101 | Bits Stored | Value: 12 | |
| 0028,0102 | High Bit | Value: 11 | |
| 0028,0103 | Pixel Representation | Value: 0 | |
| 0028,0106 | Smallest Image Pixel Value | Calculated by examination | |
| 0028,0107 | Largest Image Pixel Value | Calculated by examination | |

| Tag | Name | Explanation | |
|------------|-----------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|
| 0028,1050 | Window Center | Last windowing center saved. First window followed by second window. | |
| 0028,1051 | Window Width | Last windowing width saved. First window followed by second window. | |
| 0028,1052 | Rescale Intercept | Value -1024 for images without extended CT scale | |
| 0028,1053 | Rescale Slope | Value 1 for images without extended CT scale | |
| 0028,1055 | Window Center & Width Explanation | Value: "WINDOW1", "WINDOW2" | |
| 0028,2110 | Lossy Image Compression | Lossy Image Compression will be set to 01 for Extended Field of View because of low image quality. Derivation Description: "Reconstruction field larger than scan field" | |
| 0029,xxxx | (private data) | see \rightarrow page A.4–52, Privatizations | |
| 0029,xx40 | Application Header Sequence | see → page A.4–52, Privatizations | |
| >0029,00xx | Private Creator | "SIEMENS MEDCOM HEADER" | |
| >0029,xx41 | Application Header Type | "SOM 5 TPOS" | |
| >0029,xx42 | Application Header ID | "SOM 5 NULLPOSITION" | |
| >0029,xx43 | Application Header Version | "VB10A 20030626" | |
| >0029,xx44 | Application Header Info | Table Null Position coded as a null terminated character buffer | |

| Tag | Name | Explanation |
|-----------|------------------------------------|---------------------------------------------------------------------------------------------------------------------------------|
| 0032,1060 | Requested Procedure Description | Input entered during patient registration or examination setup. |
| | | NOTE: When Study Split is used, the value corresponding to the Requested Procedure Description selected in Exam UI will be sent |
| 7FE0,0010 | Pixel Data | |

OOG, Overlays, High Bits

Graphics in Somaris/7 images are stored as Object Oriented Graphics (OOG) in private attributes (→ page A.4–63, *MED-COM OOG Module*). Non *syngo* based systems are not expected to interpret this information.

In order to allow display access to graphics information for DICOM based systems private OOG information is converted on export into DICOM Overlay information stored in group 6000. This is the recommended way for a DICOM based system to access overlay graphics information.

The following attibutes are generated:

Overlay Rows (6000,0010)

Overlay Columns (6000,0011)

Number of Frames in Overlay (6000,0015)

Overlay Description (6000,0022) = "Siemens MedCom Object Graphics"

Overlay Type (6000.0040) = "G"

Origin (6000.0050) = 1.1

Image Frame Origin (6000,0051)

Overlay Bits Allocated (6000.0100)

Bit Position (6000,0102)

Overlay Data (6000,3000)

However, DICOM Overlay information stored in group 6000 is not supported by all systems that might be used to store Somaris/7 images. These systems may not be able to display Somaris/7 generated overlays. For a special group of these systems there is another way to provide graphical overlay information. A remote node can be configured to have overlay graphics converted into unused pixel data above High Bit (0028,0102) for images that fulfil the following condition:

- \Box bits allocated (attribute 0028, 0100) = 16
- \Box bits stored (attribute 0028,0101) = 12
- ☐ high bit (attribute 0028,0102) = 11

Some systems are known to support this coding; they are able to display this information with the images. Please note, however, that the proper and recommended way to store overlays with DICOM is the use of group 6000.

Privatizations

Private Elements for Storage SOP Classes

The following private attributes are defined for all Siemens *syngo* based applicarions:

Registry of DICOM Data Elements

| Tag | Private Owner Code | Name | VR | VM |
|-------------|---------------------------|---------------------------|----|----|
| (0029,xx08) | SIEMENS CSA HEADER | CSA Image Header Type | CS | 1 |
| (0029,xx09) | SIEMENS CSA HEADER | CSA Image Header Version | LO | 1 |
| (0029,xx10) | SIEMENS CSA HEADER | CSA Image Header Info | ОВ | 1 |
| (0029,xx18) | SIEMENS CSA HEADER | CSA Series Header Type | CS | 1 |
| (0029,xx19) | SIEMENS CSA HEADER | CSA Series Header Version | LO | 1 |
| (0029,xx20) | SIEMENS CSA HEADER | CSA Series Header Info | ОВ | 1 |
| (0029,xx08) | SIEMENS CSA NON- IMAGE | CSA Data Type | CS | 1 |
| (0029,xx09) | SIEMENS CSA NON- IMAGE | CSA Data Version | LO | 1 |
| (0029,xx10) | SIEMENS CSA NON- IMAGE | CSA Data Info | ОВ | 1 |
| (0029,xx08) | SIEMENS CSA REPORT | syngo Report Type | CS | 1 |
| (0029,xx09) | SIEMENS CSA REPORT | syngo Report | LO | 1 |
| (0029,xx15) | SIEMENS CSA REPORT | SR Variant | US | 1 |
| (0029,xx17) | SIEMENS CSA REPORT | SC SOP Instance UID | UI | 1 |
| (0029,xx10) | SIEMENS CSA ENVELOPE | syngo Report Data | ОВ | 1 |
| (0029,xx11) | SIEMENS CSA ENVELOPE | syngo Report Presentation | ОВ | 1 |

| Tag | Private Owner Code | Name | VR | VM |
|-------------|--------------------------|-----------------------------|----|----|
| (0029,xx08) | SIEMENS MEDCOM HEADER | MedCom Header Type | CS | 1 |
| (0029,xx09) | SIEMENS MEDCOM HEADER | MedCom Header Version | LO | 1 |
| (0029,xx10) | SIEMENS MEDCOM HEADER | MedCom Header Info | ОВ | 1 |
| (0029,xx20) | SIEMENS MEDCOM HEADER | MedCom History Information | ОВ | 1 |
| (0029,xx31) | SIEMENS MEDCOM HEADER | PMTF Information 1 | LO | 1 |
| (0029,xx32) | SIEMENS MEDCOM HEADER | PMTF Information 2 | UL | 1 |
| (0029,xx33) | SIEMENS MEDCOM HEADER | PMTF Information 3 | UL | 1 |
| (0029,xx34) | SIEMENS MEDCOM HEADER | PMTF Information 4 | CS | 1 |
| (0029,xx35) | SIEMENS MEDCOM HEADER | PMTF Information 5 | UL | 1 |
| (0029,xx40) | SIEMENS MEDCOM HEADER | Application Header Sequence | SQ | 1 |
| (0029,xx41) | SIEMENS MEDCOM HEADER | Application Header Type | CS | 1 |
| (0029,xx42) | SIEMENS MEDCOM HEADER | Application Header ID | LO | 1 |
| (0029,xx43) | SIEMENS MEDCOM HEADER | Application Header Version | LO | 1 |
| (0029,xx44) | SIEMENS MEDCOM HEADER | Application Header Info | ОВ | 1 |

| Tag | Private Owner Code | Name | VR | VM |
|-------------|--------------------------|----------------------------------------|----|----|
| (0029,xx50) | SIEMENS MEDCOM HEADER | Workflow Control Flags | LO | 8 |
| (0029,xx51) | SIEMENS MEDCOM HEADER | Archive Management Flag Keep Online | CS | 1 |
| (0029,xx52) | SIEMENS MEDCOM HEAD | Archive Management Flag Do Not Archive | CS | 1 |
| (0029,xx53) | SIEMENS MEDCOM HEADER | Image Location Status | CS | 1 |
| (0029,xx54) | SIEMENS MEDCOM HEADER | Estimated Retrieve Time | DS | 1 |
| (0029,xx55) | SIEMENS MEDCOM HEADER | Data Size of Retrieved Images | DS | 1 |
| (0029,xx70) | SIEMENS MEDCOM HEADER | Siemens Link Sequence | SQ | 1 |
| (0029,xx71) | SIEMENS MEDCOM HEADER | Referenced Tag | AT | 1 |
| (0029,xx72) | SIEMENS MEDCOM HEADER | Referenced Tag Type | CS | 1 |

| Tag | Private Owner Code | Name | VR | VM |
|-------------|---------------------------|-----------------------------------|----|----|
| (0029,xx73) | SIEMENS MEDCOM HEADER | Referenced Value Length | UL | 1 |
| (0029,xx74) | SIEMENS MEDCOM HEADER | Referenced Object Device Type | CS | 1 |
| (0029,xx75) | SIEMENS MEDCOM HEADER | Referenced Object Device Location | ОВ | 1 |
| (0029,xx76) | SIEMENS MEDCOM HEADER | Referenced Object ID | ОВ | 1 |
| (0029,xx60) | SIEMENS MEDCOM HEADER2 | Series Work Flow Status | LO | 1 |
| (0029,xx08) | SIEMENS MEDCOM OOG | MEDCOM OOG Type | CS | 1 |
| (0029,xx09) | SIEMENS MEDCOM OOG | MEDCOM OOG Version | LO | 1 |
| (0029,xx10) | SIEMENS MEDCOM OOG | MEDCOM OOG Info | ОВ | 1 |
| (7FE1,xx10) | SIEMENS CSA NON- IMAGE | CSA Data | ОВ | 1 |

The next subsections will explain in which IODs these private data elements are used.

All syngo Supported Image SOP Classes

Extended Image IOD Module Table

| IE | Module | Reference | Usag e | Note |
|--------------|----------------------|--------------------------------------------|-----------|-----------------------------------------|
| Patient | Patient | [1] PS 3.3 C.7.1.1 | М | |
| Study | General Study | [1] PS 3.3 C.7.2.1 | М | |
| | Patient Study | [1] PS 3.3 C.7.2.2 | U | |
| Series | General Series | [1] PS 3.3 C.7.3.1 | М | |
| Equipme t | n General Equipment | [1] PS 3.3 C.7.5.1 | U | |
| Image | General Image | [1] PS 3.3 C.7.6.1 | М | |
| | Image Pixel | [1] PS 3.3 C.7.6.3 | М | |
| | IOD specific modules | [1] PS 3.3 C.8.2.1 | M/U | depends on the IOD |
| | CSA Image Header | → page A.4–57, CSA Image Header Module | U | |
| | CSA Series Header | → page A.4–57, CSA Series Header Module | U | |
| | MEDCOM Header | → page A.4–58, MEDCOM Header Module | U | private <i>syngo</i> information |
| | MEDCOM OOG | → page A.4–63, MEDCOM OOG Module | U | if object graphics is attached to image |
| | SOP Common | [1] PS 3.3 C.12.1 | М | |

CSA Image Header Module

The table in this section contains private IOD Attributes that describe the CSA Image Header.

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-----------------------------|-------------|-----------------------|------|------------------------------------------------------------------------------------------------------------------|
| CSA Image Header Type | (0029,xx08) | SIEMENS CSA HEADER | 1 | CSA Image Header identification characteristics. Defined Terms: NUM 4 = NUMARIS/4 SOM 5 = Somaris/5 or Somaris/7 |
| CSA Image Header Version | (0029,xx09) | SIEMENS CSA HEADER | 3 | Version of CSA Image Header Info (0029,xx10) format. |
| CSA Image Header Info | (0029,xx10) | SIEMENS CSA HEADER | 3 | Manufacturer Model dependent information. |

CSA Series Header Module

The table in this section contains private IOD Attributes that describe the CSA Series Header.

| Attribute Name | Tag | Private Creator | Туре | Notes |
|------------------------------|-------------|-----------------------|------|-------------------------------------------------------------------------------------------------------------------|
| CSA Series Header Type | (0029,xx08) | SIEMENS CSA HEADER | 1 | CSA Series Header identification characteristics. Defined Terms: NUM 4 = NUMARIS/4 SOM 5 = Somaris/5 or Somaris/7 |
| CSA Series Header Version | (0029,xx19) | SIEMENS CSA HEADER | 3 | Version of CSA Series Header Info (0029,xx20) format. |
| CSA Series Header Info | (0029,xx20) | SIEMENS CSA HEADER | 3 | Manufacturer Model dependent information. |

MEDCOM Header Module

The table in this section contains private IOD Attributes that describe MEDCOM Header.

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-------------------------------|-------------|-----------------------------|------|---------------------------------------------------------------------------------------------------------------------------------------------------|
| MedCom Header Type | (0029,xx08) | SIEMENS MEDCOM HEADER | 1C | MedCom Header identification characteristics. Defined Terms: MEDCOM 1 Required if MedCom Header Info (0029,xx10) present. |
| MedCom Header Version | (0029,xx09) | SIEMENS MEDCOM HEADER | 2C | Version of MedCom Header Info (0029,xx10) format. Required if MEDCOM Header Info (0029,xx10) present. |
| MedCom Header Info | (0029,xx10) | SIEMENS MEDCOM HEADER | 3 | Manufacturer model dependent information. The value of the attribute MedCom Header Info (0029,xx10) can be build up in each user defined format. |
| MedCom History Information | (0029,xx20) | SIEMENS MEDCOM HEADER | 3 | MedCom defined Patient Registration history information. See → page A.4–63, MEDCOM History Information. |
| PMTF Information 1 | (0029,xx31) | SIEMENS MEDCOM HEADER | 3 | Transformation Information |
| PMTF Information 2 | (0029,xx32) | SIEMENS MEDCOM HEADER | 3 | Transformation Information |
| PMTF Information 3 | (0029,xx33) | SIEMENS MEDCOM HEADER | 3 | Transformation Information |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|----------------------------------------------|-------------|-----------------------------|------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| PMTF Information 4 | (0029,xx34) | SIEMENS MEDCOM HEADER | 3 | Transformation Information |
| Application Header Sequence | (0029,xx40) | SIEMENS MEDCOM HEADER | 3 | Sequence of Application Header Items. Zero or more Items shall be included in this sequence. Encoded as a sequence of items. |
| >Application Header Type | (0029,xx41) | SIEMENS MEDCOM HEADER | 1C | Application Header identification characteristics. Required if Sequence is sent. |
| >Application Header ID | (0029,xx42) | SIEMENS MEDCOM HEADER | 3 | Identification of an application header. |
| >Application Header Version | (0029,xx43) | SIEMENS MEDCOM HEADER | 3 | Version of Application Header Info (0029,xx43) format. |
| >Application Header Info | (0029,xx44) | SIEMENS MEDCOM HEADER | 3 | Application dependent information. |
| Workflow Control Flags | (0029,xx50) | SIEMENS MEDCOM HEADER | 3 | Eight free definable flags. |
| Archive Management Flag Keep Online | (0029,xx51) | SIEMENS MEDCOM HEADER | 3 | Flag to control remote archive management system to keep the image always online (also when already archived). Enumerated Values: 00 = remote control not required, 01 = keep image online. |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-------------------------------------------------|-------------|-----------------------------|------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Archive Management Flag Do Not Archive | (0029,xx52) | SIEMENS MEDCOM HEADER | 3 | Flag to control remote archive management system not to archive the related image. Enumerated Values: 00 = remote control not required, 01 = don't archive image. |
| Image Location Status | (0029,xx53) | SIEMENS MEDCOM HEADER | 3 | Image location status to control retrieving. Defined Terms: ONLINE = retrieving has to be done as usual, NEARLINE = move request to SCP and delay according to value of Estimated Retrieve Time (0029,xx54), OFFLINE = invoking a retrieve operation initiates an operator request, INVALID = invoking a retrieve operation would always result in an error. |
| Estimated Retrieve Time | (0029,xx54) | SIEMENS MEDCOM HEADER | 3 | Estimated retrieve time in seconds. A value less then zero (< 0) indicates location is OFFLINE or INVALID. |
| Data Size of Retrieved Images | (0029,xx55) | SIEMENS MEDCOM HEADER | 3 | Data size of images in MByte. |
| Siemens Link Sequence | (0029,xx70) | SIEMENS MEDCOM HEADER | 3 | Sequence of Link items. Each item identifies the location of the location of one missing tag. One or more items can be located in this sequence. |
| Referenced Tag | (0029,xx71) | SIEMENS MEDCOM HEADER | 1 | The referenced tag. The value of this tag is in the Child Data Object (CDO). Currently it is always Pixel Data (7FE0,0010). |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-----------------------------------------|-------------|-----------------------------|------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Referenced Tag Type | (0029,xx72) | SIEMENS MEDCOM HEADER | 1 | The value representation (type) of the missing tag (e.g. OW). Enumerated values are all DICOM defined Value Representations. |
| Referenced Value Length | (0029,xx73) | SIEMENS MEDCOM HEADER | 1 | The length of the referenced tag value in bytes. |
| Referenced Object Device Type | (0029,xx74) | SIEMENS MEDCOM HEADER | 1 | The Device Type that stores the Child Data Object (CDO) with the referenced tag value. Currently it should be "SHMEM". In future, "SDM", "LOID" or "FILE" are also imaginable. Defined Terms are SHMEM = Shared Memory SDM = Series Data Management LOID = Database FILE |
| Referenced Object Device Location | (0029,xx75) | SIEMENS MEDCOM HEADER | 2 | The Location of the device that stores the Child Data Object (CDO) with the referenced tag value. For the "SHMEM" case, it is the shared memory directory. Can be empty, then the default directory will be taken. In future, for "SDM" this will be the SDM_ID, for FILE it will be the directory name and for "LOID" it will be the database name. |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|----------------------------|-------------|------------------------------|------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Referenced Object ID | (0029,xx76) | SIEMENS MEDCOM HEADER | 1 | The ID of the object that contains the Child Data Object (CDO) with the referenced tag value. In case of "SHMEM", it is the shared memory ID. In future, for "SDM" this will be a Sirius OID, for "FILE" the file name, for "DB" the LOID. |
| Series Work Flow Status | (0029,xx60) | SIEMENS MEDCOM HEADER2 | 3 | syngo Patient Browser specific flags used for clinical work: - com = completed - rea = read - ver = verified |

MEDCOM History Information

The value of the attribute MEDCOM History Information (0029,xx20) is defined in the following way:

| Part | Name | Type | Bytes | Notes |
|---------|------------------------|--------|-------|----------------------|
| header | Identifier | string | 32 | always "CSA HISTORY" |
| | Version | string | 32 | e.g. "V1.10" |
| n items | Class Name | string | 64 | |
| | Modification String | string | 1024 | |

MEDCOM OOG Module

The table in this section contains private IOD Attributes that describe MEDCOM Object Oriented Graphics (OOG). This module is used when object graphics is drawn on the image and need to be stored as graphic object properties (Line, Circle, Rectangle, Arrow, and so on). iven the condition that the module contents was not removed by other modalities, the graphic objects remain re-animatable if such an image was transferred and is then retrieved back.

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-----------------------|-------------|------------------------|------|---------------------------------------------------------------------------------------------------|
| MedCom OOG Type | (0029,xx08) | SIEMENS MEDCOM OOG | 1 | MEDCOM Object Oriented Graphics (OOG) identification characteristics. Defined Terms: MEDCOM OOG 1 |
| MedCom OOG Version | (0029,xx09) | SIEMENS MEDCOM OOG | 3 | Version of MEDCOM OOG Info (0029,xx10) format. |
| MedCom OOG Info | (0029,xx10) | SIEMENS MEDCOM OOG | 3 | MEDCOM Object Oriented Graphics (OOG) data. |

The graphics objects are also stored in one Image overlay plane for compatibility with other products, which don't support the MedCom OOG module. Any system which does not support this MedCom OOG module has to remove these private attributes when modifying the image overlay plane content.

syngo Report Data

The module contains private IOD Attributes that describe *syngo* reports. This module is used when *syngo* report data are added to DICOM SR and DICOM SC objects.

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-------------------------------------|-------------|-------------------------|------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| syngo Report Type | (0029,xx08) | SIEMENS CSA REPORT | 1 | syngo report characteristics, e.g. report creating application. Defined Terms: CT_CASCORING CT_CIRCULATION CT_LUNGCARE INSPACE MR_ARGUS This attribute value will be used to identify the corresponding application during generic extension dll management. A restricted character set is used: only A-Z and underscore are supported. |
| syngo Report Version | (0029,xx09) | SIEMENS CSA REPORT | 3 | Version of <i>syngo</i> Report Data (0029,xx10) format. |
| <i>syngo</i> Report Data | (0029,xx10) | SIEMENS CSA ENVELOPE | 3 | Application specific report related data |
| <i>syngo</i> Report Presentation | (0029,xx11) | SIEMENS CSA ENVELOPE | 3 | Application specific report related data |

| Attribute Name | Tag | Private Creator | Type | Notes |
|------------------------|-------------|-----------------------|------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| SR Variant | (0029,xx15) | SIEMENS CSA REPORT | | DICOM SR Variant. Enumerated Values: 0 = Basic Text SR (1.2.840.1008.5.1.4.1.1.88.11) 1 = Enhanced SR (1.2.840.1008.5.1.4.1.1.88.22) 2 = Comprehensive SR (1.2.840.1008.5.1.4.1.1.88.33) 3 = Mammography CAD SR (1.2.840.1008.5.1.4.1.1.88.50) 4 = Key Object Selection Document (1.2.840.1008.5.1.4.1.1.88.59) 5 = Chest CAD SR (1.2.840.1008.5.1.4.1.1.88.65) |
| SC SOP Instance UID | (0029,xx17) | SIEMENS CSA REPORT | 3 | DICOM SOP Instance UID of syngo based SC image representing the syngo report object. This UID will be used to identify the Resulting SC object after SR to SC conversion. |

Private Elements for CT Image Storage SOP Class

The following private attributes are defined by Somaris/7.

Registry of DICOM Data Elements

| Tag | Private Owner Code | Name | VR | VM | Notes |
|-------------|------------------------|------------------------------------|----|----|------------------------------------------------------------------------------------------------|
| (0019,xx90) | SIEMENS CT VA0 COAD | Osteo Offset | DS | 1 | Offset of the water equivalent material of the Siemens Osteo phantom to real water |
| (0019,xx92) | SIEMENS CT VA0 COAD | Osteo Regression Line Slope | DS | 1 | Slope of the regression line for the ESP (=European Spine Phantom) standardization |
| (0019,xx93) | SIEMENS CT VA0 COAD | Osteo Regression Line Intercept | DS | 1 | Intercept of the regression line for the ESP (= European Spine Phantom) standardization |
| (0019,xx96) | SIEMENS CT VA0 COAD | Osteo Phantom Number | IS | 1 | Number of the Siemens Osteo phantom |
| (0019,xxB0) | SIEMENS CT VA0 COAD | Feed per Rotation | DS | 1 | Som/4 style Feed per Rotation (Backwards Compatibility) |

| Tag | Private Owner Code | Name | VR | VM | Notes |
|-------------|------------------------|-------------------------------|----|----|---------------------------------------------------------------------------------------------------------------------|
| (0019,xxBD) | SIEMENS CT VA0 COAD | Pulmo Trigger Level | IS | 1 | Spirometer trigger level used for the scan, given in percent of VC (= Vital Capacity) of the patient |
| (0019,xxBE) | SIEMENS CT VA0 COAD | Expiratoric Reserve Volume | DS | 1 | ERV (= Exspiratoric Reserve Volume) achieved by the patient |
| (0019,xxBF) | SIEMENS CT VA0 COAD | Vital Capacity | DS | 1 | VC (= Vital Capacity) achieved by the patient |
| (0019,xxC0) | SIEMENS CT VA0 COAD | Pulmo Water | DS | 1 | Density of the water insert of the Siemens Pulmo phantom |
| (0019,xxC1) | SIEMENS CT VA0 COAD | Pulmo Air | DS | 1 | Density of the air holes of the Siemens Pulmo phantom |
| (0019,xxC2) | SIEMENS CT VA0 COAD | Pulmo Date | DA | 1 | Date of the evaluation of the Siemens Pulmo phantom |
| (0019,xxC3) | SIEMENS CT VA0 COAD | Pulmo Time | TM | 1 | Time of the evaluation of the Siemens Pulmo phantom |
| (0021xx11) | SIEMENS MED | Target | DS | 2 | Som/4 style Target (Backwards Compatibility) |

| Tag | Private Owner Code | Name | VR | VM | Notes |
|-------------|-------------------------|---------------------------------|----|----|-------|
| (0009,00xx) | SIEMENS CT VA1 DUMMY | Private Creator Data Element | LO | 1 | 1 |

Private SOP class CSA Non-Image

This chapter includes the definition of the Siemens AG B Med CSA defined private Non-Image Object (called CsaNonImage IOD). The focus of this private Non-Image Object is to address the requirement for non-image data sets found in *syngo* based applications.

The MedCom Non-Image Information Object Definition specifies data sets that are converted from a non-DICOM format to a modality independent DICOM format.


Examples of such manufacturer model dependent data sets are:

| Raw Data |
|----------------------|
| CT Admin Data |
| MR Spectroscopy Data |

□ etc.

CSA Non-Image IOD Entity Relationship Model

The E-R model in [DICOM] A.1.2 depicts those components of the DICOM Information Model which directly refer to the CSA Non-Image IOD. The frame of reference IE, overlay IE, modality look up table IE, VOI lookup table IE and curve IE are not components of the CSA Non-Image IOD.

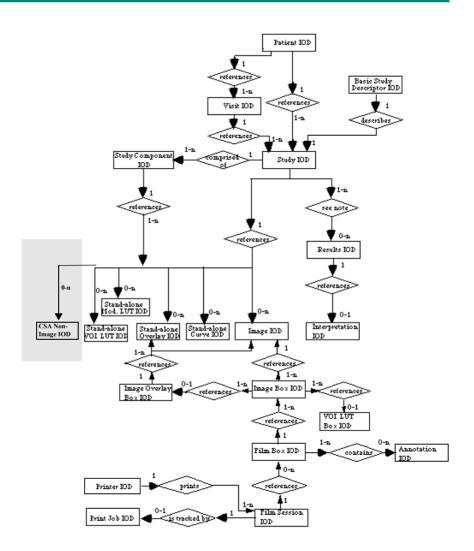

CSA Non-Image IOD Module Table

| IE | Module | Reference | Usage |
|-----------|-------------------|--------------------------------------------|-------|
| Patient | Patient | [1] PS 3.3 C.7.1.1 | M |
| Study | General Study | [1] PS 3.3 C.7.2.1 | M |
| | Patient Study | [1] PS 3.3 C.7.2.2 | U |
| Series | General Series | [1] PS 3.3 C.7.3.1 | M |
| Equipment | General Equipment | [1] PS 3.3 C.7.5.1 | U |
| CSA | CSA Image Header | → page A.4–57, CSA Image Header Module | U |
| | CSA Series Header | → page A.4–57, CSA Series Header Module | U |
| | MEDCOM Header | → page A.4–58, MEDCOM Header Module | U |
| | CSA Non-Image | → page A.4–73, CSA Non- Image Module | M |
| | SOP Common | [1] PS 3.3 C.12.1 | M |

CSA Non-Image Module

The table in this section contains private IOD Attributes that describe CSA Non-Images.

| Attribute Name | Tag | Private Creator | Туре | Notes |
|------------------------------|-------------|--------------------|------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Image Type | (0008,0008) | - | 3 | Image identification characteristics. See → page A.4–76, CT Extensions of the Non-Image Object |
| Acquisition Date | (0008,0022) | - | 3 | The date the acquisition of data that resulted in this data set started. |
| Acquisition Time | (0008,0032) | - | 3 | The time the acquisition of data that resulted in this data set started. |
| Conversion Type | (0008,0064) | - | 3 | Describes the kind of image conversion. Defined Terms: DV = Digitized Video, DI = Digital Interface, DF = Digitized Film, WSD = Workstation. |
| Referenced Image Sequence | (0008,1140) | - | 3 | A sequence which provides reference to a set of Image SOP Class/Instance identifying other images significantly related to this data set. Encoded as sequence of items: (0008,1150) and (0008,1155). |
| Derivation Description | (0008,2111) | - | 3 | A text description of how this data set was derived. |
| Source Image Sequence | (0008,2112) | - | 3 | A Sequence which identifies the set of Image SOP Class/Instance pairs of the Images which were used to derive this data set. Zero or more Items may be included in this Sequence. Encoded as sequence of items: (0008,1150) and (0008,1155). |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|----------------------------|-------------|--------------------|------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Patient Position | (0018,5100) | - | 3 | Patient position descriptor relative to the equipment. |
| Acquisition Number | (0020,0012) | - | 3 | A number identifying the gathering of data over a period of time which resulted in this data set. |
| Instance Number | (0020,0013) | - | 3 | A number that identifies this data set. |
| Frame of Reference UID | (0020,0052) | - | 3 | Uniquely identifies the frame of reference for a Series. |
| Image Comments | (0020,4000) | - | 3 | User-defined comments about the image. |
| Quality Control Image | (0028,0300) | - | 3 | Indicates whether or not this image is a quality control or phantom image. If this Attribute is absent, then the image may or may not be a quality control or phantom image. Enumerated Values: YES, NO. |
| Burned In Annotation | (0028,0301) | - | 3 | Indicates whether or not image contains sufficient burned in annotation to identify the patient and date the image was acquired. If this Attribute is absent, then the image may or may not contain burned in annotation. Enumerated Values: YES, NO. |
| Lossy Image Compression | (0028,2110) | - | 3 | Specifies whether an Image has undergone lossy compression. Enumerated Values: 00 = Image has NOT been subjected to lossy compression, 01 = Image has been subjected to lossy compression. |

| Attribute Name | Tag | Private Creator | Туре | Notes |
|-------------------------------------|-------------|--------------------------|------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Lossy Image Compression Ratio | (0028,2112) | - | 3 | Describes the approximate lossy compression ratio(s) that have been applied to this image. May be multi valued if successive lossy compression steps have been applied. |
| CSA Data Type | (0029,xx08) | SIEMENS CSA NON-IMAGE | 1 | CSA Data identification characteristics. Defined Terms: RAW DATA NUM 4 = NUMARIS/4 Raw Data SPEC NUM 4 = NUMARIS/4 Spectroscopy RAW DATA SOM 5 = Somaris/5 Raw Data RAW DATA SOM 7 = Somaris/7 Raw Data BSR REPORT = BSR Study Report Data |
| | | | | COL REPORT SOM5 = syngo Colonography Report Data |
| CSA Data Version | (0029,xx09) | SIEMENS CSA NON-IMAGE | 3 | Version of CSA Data Info (0029,xx10) format and CSA Non- Image Data (7FE1,xx10) format. |
| CSA Data Info | (0029,xx10) | SIEMENS CSA NON-IMAGE | 3 | Information to describe the CSA Data (7FE1,xx10). The value of the attribute CSA Data Info (0029,xx10) can be build up in each user defined format. |
| CSA Data | (7FE1,xx10) | SIEMENS CSA NON-IMAGE | 1 | Binary data as byte stream. |

CT Extensions of the Non-Image Object

Somaris/7 uses the following defined term for Image Type

(0008,0008):

☐ Value 1: ORIGINAL☐ Value 2: PRIMARY

☐ Value 3: AXIAL, LOCALIZER, OTHER

☐ Value4: a CT_SOM5 * enumeration

☐ Value5: Somaris/7 specific enumeration

The following table lists the Somaris/7 non-image types and the corresponding combinations of the Image Type Attribute values.

| Description | 0008,000 8 Value 1 | 0008,000 8 Value 2 | 0008,000 8 Value 3 | 0008,0008 Value 4 | 0008,000 8 Value 5 | | |
|-------------------|--------------------------|----------------------------|--------------------------|----------------------|--------------------------|--|--|
| CAD Results | DERIVED | SECONDA RY | OTHER | CAD MARKS | none | | |
| Colonography Data | DERIVED | SECONDA RY | OTHER | CT_SOM5 COL | none | | |
| Coronary Tree | DERIVED | SECONDA RY | OTHER | CT_CIRCULATION | none | | |
| Raw Data | Same entri | Same entries as for images | | | | | |

Private SOP Classes

| SOP Class Name | SOP Class UID |
|----------------|---------------------|
| CSA Non-Image | 1.3.12.2.1107.5.9.1 |

Private Transfer Syntaxes

Not applicable.

Configuration

AE Title / Presentation Address Mapping

To ensure unique identification the hostname should be part of the AE Titles (see examples below, hostname = station4). The string can be up to 16 characters long and must not contain any extended characters, only 7 bit ASCII characters (excluding control characters) are allowed according to DICOM standard. An example name is HRI_station4. Another common setting is the use of the hostname in capital letters, e.g. "STATION4".

Note

The current implementation does not support the full DICOM standard syntax. Spaces and special characters (like &<>") in the AE title string are not supported.

DICOM Verification

The Verification Service uses the AE configuration of the DICOM Service that is checked with the C-ECHO message. E.g. Verification will use the Storage AE, if initiated to check the configuration of a remote DICOM node.

DICOM Storage/StorageCommitment/QR AE Title

There is a common AE Title for Storage, Storage Commitment and Query/Retrieve. It can be configured via Service UI in Configuration/DICOM/General.

The port is set to the fixed value of 104.

DICOM Modality Worklist and MPPS AE Title

There is a common AE Title for Modality Worklist and MPPS. It can be configured via Service UI in Configuration/DICOM/General.

DICOM Print AE Title

There is a separate AE Title for Print application. It can be configured via Service UI in Configuration/DICOM/General.

No input of AETs starting with a numeric character is possible.

Remote AE Titles and Presentation Addresses

Remote AETs, host names, IP addresses and port numbers can be configured using the Service application. For each AET a list of supported services can be configured.

Configurable Parameters

The Application Entity Titles, host names and port numbers for remote AE are configured using the Somaris/7 Service/
Installation Tool. For each AET the list of services supported can be configured.

Storage, Storage Commitment and Query/Retrieve

- □ a quality factor which determines the proposed transfer syntax in case that an user has initiated the C-STORE. By convention, 0 means: Only Uncompressed Transfer Syntax(es) are proposed, 100 means: Lossless Transfer Syntax is proposed, and any other value between 1 and 99 means that an JPEG Lossy Transfer Syntax is proposed. One Uncompressed Transfer Syntax will be proposed in any case. This parameter is general for all destination nodes.
- □ a "compression type supported" which determines the proposed transfer syntax in case that the C-STORE was initiated as a sub-operation of an incoming C-MOVE-RQ. By convention, 0 means: Only Uncompressed Transfer Syntax(es) are proposed, 1 means: Lossless Transfer Syntax is proposed, and 2 means that an JPEG Lossy Transfer Syntax is proposed. One uncompressed transfer syntax will be proposed in any case. This parameter can be set for each configured destination node.

Note:

By default association requests are accepted by the SCP regardless of the value of DICOM Application Context Name set in the requests.

This behavior can be changed by modifying the value of the entry ACCEPT_ANY_CONTEXT _NAME in the configuration file mergecom.pro of MergeCOM-3 Tool Kit. If the value is FALSE, association requests are accepted only when DICOM Application Context Name is set to "1.2.840.10008.3.1.1.1" (see DICOM specification PS 3.7-2003, A.2.1)

Additional configurable parameters for Storage Commitment are:

When acting as SCU:

- ☐ Flag to indicate whether the association will be kept open to receive the response or to close the association and be prepared to receive the response on another association.
- ☐ Time-out which defines how long the association of N-ACTION is kept to receive a N-EVENT-REPORT on the same association. The same value is used to wait for a N-EVENT-REPORT on another association (applicability of transaction UID).

 (default 1 h)

When acting as SCP:

flag to indicate whether an archive system is installed.

Print

| | e Service application can be used to configure the SCP COM Printer). |
|-----|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| | ese parameters are mandatory to be set: AET Host name P address Port number |
| can | ese parameters have defaults as per configuration file and a be changed: Default camera Pixel size Additional or changed film sheet formats (e.g. inch 14x14, nch 14x17,) |
| | List with mapping pixel size to each film sheet format Minimal density Stored printed film jobs Media type |
| | Film destination |

Modality Worklist

The Service application can be used to set the AETs, port numbers, host names, IP addresses, capabilities, and time-outs for the remote nodes (SCPs).

Additional configurable parameters for Basic Worklist Query are:

- □ Query Waiting time the time to wait for the C-FIND-RSP after sending the C-FIND-RQ (default 20 s)
- ☐ Max. Query Match Number the maximum number of entries accepted in one worklist (default is 200)
- ☐ Query Interval the time between two C-FIND-RQs to the Hospital Information System (default is 60 min)
- ☐ Broad Worklist Query behaviour: two values are defined: Set the AE Title search attribute to the own AE Title, and the Modality search attribute to "*".
 - Set the Modality search attribute to the own modality and the AE Title search attribute to "*".

The sending of an empty Specific Character Set in a C-FIND-RQ can be omitted by configuration. However, this is not available in the Service application but must be set by a Siemens service technician.

Modality Performed Procedure Step

The Service application can be used to configure the Modality Worklist SCP to be a Modality Performed Procedure Step SCP also.

Additional configurable parameters for Modality Performed Procedure Step are:

- □ Default Catalog Name (0008,0102) used for items entered in MPPS UI
- ☐ Default Catalog Version (0008,0103) used for items entered in MPPS UI

Default Parameters

| This configuration tool also uses some default parameters: |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| ☐ maximal PDU size set to 262144 Bytes |
| ☐ time-out for accepting/rejecting an association request: 60 s |
| □ time-out for responding to an association open/close request: 60 s |
| ☐ time-out for accepting a message over network: 60 s |
| $\hfill \Box$ time-out for waiting for data between TCP/IP-packets: 60 s |
| |
| The Time-outs for waiting for a service Request/Response message from the remote node are as follows: |
| ☐ for Storage SCP/SCU: 600 s |
| □ for Storage Commitment SCU: time-out for Response to N-ACTION: 600 s time-out for N-EVENT-REPORT: configurable, see → page A.5–3, Configurable Parameters. |
| ☐ for Query/Retrieve SCP/SCU: 600 s |
| □ for Modality Worklist SCU: configurable, see → page A.5–3, Configurable Parameters. |
| ☐ for Print Management SCU: time-out for Response to N-SET-RQ: 240 s time-out for Response to other Requests: 60 s |
| |



Support of Extended Character Sets

The Somaris/7 DICOM application supports the following character sets as defined in the four tables below:

The following is Single-Byte Character Sets without Code

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|---------------------|
| Default repertoire | none | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 1 | ISO_IR 100 | ISO_IR 100 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 2 | ISO_IR 101 | ISO_IR 101 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 3 | ISO_IR 109 | ISO_IR 109 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 4 | ISO_IR 110 | ISO_IR 110 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Cyrillic | ISO_IR 144 | ISO_IR 144 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Arabic | ISO_IR 127 | ISO_IR 127 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Greek | ISO_IR 126 | ISO_IR 126 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Hebrew | ISO_IR 138 | ISO_IR 138 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 5 | ISO_IR 148 | ISO_IR 148 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Japanese | ISO_IR 13 | ISO_IR 13 | JIS X 0201:Katakana |

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|-------------------|
| | | ISO_IR 14 | JIS X 0201:Romaji |

The following table shows Single-Byte Character Sets with Code Extension

| Character Set Description | Defined Term | Standard for Code Extension | ESC Sequence | ISO Registration Number | Character Set |
|------------------------------|--------------------|-----------------------------------|--------------------|-------------------------------|-------------------|
| Default repertoire | ISO 2022 IR 6 | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.1 | ISO 2022 IR 100 | ISO 2022 | ESC 02/13 04/01 | ISO_IR 100 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.2 | ISO 2022 IR 101 | ISO 2022 | ESC 02/13 04/02 | ISO_IR 101 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.3 | ISO 2022 IR 109 | ISO 2022 | ESC 02/13 04/03 | ISO_IR 109 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.4 | ISO 2022 IR 110 | ISO 2022 | ESC 02/13 04/04 | ISO_IR 110 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Cyrillic | ISO 2022 IR 144 | ISO 2022 | ESC 02/13 04/12 | ISO_IR 144 | Supplementary set |

| Character Set Description | Defined Term | Standard for Code Extension | ESC Sequence | ISO Registration Number | Character Set |
|------------------------------|--------------------|-----------------------------------|--------------------|-------------------------------|----------------------------|
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Arabic | ISO 2022 IR 127 | ISO 2022 | ESC 02/13 04/07 | ISO_IR 127 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Greek | ISO 2022 IR 126 | ISO 2022 | ESC 02/13 04/06 | ISO_IR 126 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Hebrew | ISO 2022 IR 138 | ISO 2022 | ESC 02/13 04/08 | ISO_IR 138 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.5 | ISO 2022 IR 148 | ISO 2022 | ESC 02/13 04/13 | ISO_IR 148 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Japanese | ISO 2022 IR 13 | ISO 2022 | ESC 02/13 04/09 | ISO_IR 13 | JIS X 0201:Katakana |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 14 | JIS X 0201- 1976:Romaji |

The following table shows Multi-Byte Character Sets without Code Extension:

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|------------------|
| Unicode | ISO_IR 192 | ISO 10646 | Unicode in UTF-8 |

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|-------------------------------------------------------------|
| Chinese | GB18030 | GB18030 | GB 18030-2000 (China Association for Standardization) |

The following table shows Multi-Byte Character Sets with Code Extension:

| Character Set Description | Defined Term | Standard for Code Extension | ESC Sequence | ISO Registration Number | Character Set |
|---------------------------------|--------------------|-----------------------------------|--------------------------|-------------------------------|------------------------------------------------------------|
| Japanese | ISO 2022 IR 87 | ISO 2022 | ESC 02/04 04/02 | ISO_IR 87 | JIS X 0208: Kanji |
| | ISO 2022 IR 159 | ISO 2022 | ESC 02/04 02/08 04/04 | ISO_IR 159 | JIS X 2012: Supplementary Kanji set |
| Chinese ^a | ISO 2022 IR 58 | ISO 2022 | ESC 02/04 04/01 | ISO_IR 58 | GB2312-80 (China Association for Standardization) |

a. This Character Set is an extension of DICOM for the chinese language.

When there is a mismatch between the SCS tags (0008,0005) and the characters in an IOD coming into the system, then the following measures are taken to make the characters DICOM conform:

☐ Try to import with ISO-IR 100. If ISO-IR 100 also fails convert each illegal character to'?'.

There are now 3 categories of character sets which have to be differentiated because of their different encoding formats:

| ISO_IR 100, etc. =>Encoded in ISO 2022 |
|--------------------------------------------------------------------------------------------------------------------------------|
| ISO_IR 192: Encoded in UTF-8 |
| GB18030: Encoded in GB18030 |
| is not possible to recognize the following mismatches auto atically on receiving or importing: |
| An attribute value is encoded in ISO_IR 192 <=> (0008,0005) contains a conventional ISO character set as primary character set |
| An attribute value is encoded in GB18030 <=> (0008,0005) contains a conventional ISO character set as primary character set |
| An attribute value is encoded in ISO 2022 <=> (0008,0005) contains ISO_IR 192 |
| An attribute value is encoded in ISO 2022 <=> (0008,0005) contains GB18030 |

☐ Conventional ISO character sets: ISO IR 6. ISO 2022 IR 6.

An IOD that contains one of the above mentioned inconsistencies is not DICOM conform. As these kinds of inconsistencies cannot be recognized by the system, the IOD will not be rejected but the character data might be corrupted

Older versions of *syngo* do not support the newly introduced character sets ISO_IR 192 and GB18030 and their special encodings. That means, an IOD which contains one of these new character sets in (0008,0005) will be rejected by an older *syngo* system.

DICOM Conformance Statement

Part II - Media Storage

This part contains the Conformance Statement to all "Offline Media Application Profiles (incl. private extensions)" supported by the Somaris/7 archive options.


| by the Somans/7 archive options. | |
|----------------------------------------------|--|
| The application profiles supported shall be: | |
| ☐ Standard Application Profiles | |

Augmented Application Profilessyngo private Application Profiles

For general introductory topics see → page 5, *Introduction*.

Implementation Model

Application Data Flow Diagram

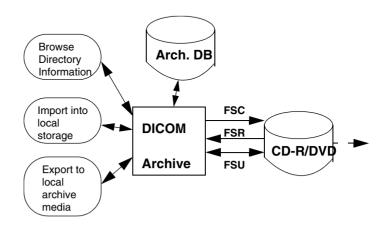

The DICOM archive application will serve as an interface to the CD-R/DVD off-line medium device. It serves interfaces to include the off-line media directory into the browser and to copy SOP instances to a medium or retrieve SOP Instances from medium into local storage.

The DICOM Archive application will support 120 mm CD-R/DVD media (see Table → page B.2–1).

The FSU role will update new SOP Instances only to media with pre-existing File-sets conforming to the Application Profiles supported.

The contents of the DICOMDIR will be temporarily stored in Archive-Database.

Functional definitions of AE's

The Somaris/7 DICOM off-line media storage application consists of the DICOM Archive application entity serving all interfaces to access off-line media. The DICOM Archive application is capable of

- creating a new File-set onto an unwritten medium (Export to...)
- updating an existing File-set by writing new SOP Instances onto the medium
- importing SOP Instances from the medium onto local storage
- → reading the File-set's DICOMDIR information temporarily into database and pass it to display applications.

Sequencing of Real-World Activities

The DICOM Archive application will not perform updates before the Directory information of the DICOMDIR is completely read.

When performing updates, the SOP instances are checked for existence before updating. Duplicate instances will be avoided.

File Meta Information Options

The Implementation Class UID is:

□ "1.3.12.2.1107.5.1.4"

and an Implementation Version Name of

☐ "SIEMENS_S7VA20A"



AE Specifications

DICOM Archive Specification

The DICOM Archive provides Standard conformance to Media Storage Service Class (Interchange Option).

| Application Profiles Supported | Real World Activity | Role | SC Option |
|------------------------------------------------------------------------------|-------------------------------|---------|-------------|
| PRI-SYNGO-CD | Browse Directory Information | FSR | Interchange |
| PRI-SYNGO-DVD AUG-GEN-CD | Import into local Storage | FSR | Interchange |
| AUG-CTMR-CD ^a STD-XA1K-CD ^a | Export to local archive media | FSC,FSU | Interchange |
| STD-GEN-CD | Browse Directory Information | FSR | Interchange |
| STD-CTMR-CD STD-XABC-CD STD-XA1K-CD TD-US-zz-yF-xxxxxx ^b | Import into local Storage | FSR | Interchange |

a. With no private SOP Class used, the PRI-SYNGO-CD profile definitions are appropriate to describe the augmentation of the related -STD Profiles.

xxxxxx can take 2 values: CDR and DVD

On synngo-based products the Private Extended *syngo* Profile (e.g. PRI-SYNGO-CD) will be preferrably used by the system. The General Purpose Interchange Profile (STD-GEN-CD), Ultrasound Profile (STD-US-xxx), CT and MR Image Profile (STD-CTMR-xxx), Waveform Interchange (STD-WVFM-xxx), Basic Cardiac Profile (STD-XABC-CD) and 1024 X-Ray Angiographic Profile (STD-XA1K-CD) will be supported with read capability of the related media.

b. All combinations of the following values for zz, yF and xxxxxx are suported:

^{&#}x27;yF' can take two values: SF for Single Frame and MF for Multi Frame.

^{&#}x27;zz' can take three values: ID (Image Display), SC(Spatial Calibration) and CC (Combined Calibration)

File Meta Information for the Application Entity

The Source Application Entity Title is set by configuration. See → Chapter B.5, *Configuration* for details.

Real-World Activities for this Application Entity

Real-World Activity: Browse Directory Information

The DICOM Archive application acts as FSR using the interchange option when requested to read the media directory.

The DICOM archive application will read the DICOMDIR and insert that directory entries, that are valid for the application profiles supported, into a local database. The database can then be used for browsing media contents.

Note

Icon Image SQ is also supported in DICOMDIR. But only those Icon Images with Bits Allocated (0028,0100) equal to 8 and size 64 by 64 or 128 by 128 pixels are imported into database and are visible in PatientBrowser.

Application Profiles for the RWA: Browse Directory Information

See table → page B.2–1 for the Application Profiles listed that invoke this Application Entity for the Browse Directory Information RWA.

Real-World Activity: Import into local Storage

The DICOM Archive application acts as FSR using the interchange option when requested to read SOP Instances from the medium into the local storage.

The SOP Instance selected from the media directory will be copied into the local storage. Only SOP Instances, that are valid for the application profile supported and are listed as supported by the Storage SCP Conformance section (→ page A.2–4), can be retrieved from media storage. This is due to the fact that the Browse Directory Information will filter all SOP Instances not matching the Application profiles supported.

During operation no "Attribute Value Precedence" is applied to the SOP Instances. Detached Patient Management is not supported (please refer to DICOM part 11, Media Storage Application Profiles).

For media conforming to the STD-GEN-CD Profile the following SOP classes will be supported as a FSR:

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID |
|--------------------------------|--------------------------------|------------------------------------------------------------------|
| CR Image | 1.2.840.10008.5.1.4.1.1.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| DX Image - For Presentation | 1.2.840.10008.5.1.4.1.1.1.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1.1.1 .1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| MG Image - For Presentation | 1.2.840.10008.5.1.4.1.1.1.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID |
|---------------------------------|-----------------------------|------------------------------------------------------------------|
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1.1.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| IOX Image - For Presentation | 1.2.840.10008.5.1.4.1.1.3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1.1.3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| CT Image | 1.2.840.10008.5.1.4.1.1.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1.3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| US-MF image | 1.2.840.10008.5.1.4.1.1.3.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| MR Image | 1.2.840.10008.5.1.4.1.1.4 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1.6 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| US Image | 1.2.840.10008.5.1.4.1.1.6.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| SC Image | 1.2.840.10008.5.1.4.1.1.7 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID |
|------------------------------------------------|-----------------------------------|------------------------------------------------------------------|
| 12-lead ECG Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.1 .1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| General ECG Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.1 .2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Ambulatory ECG Wave- form Storage | 1.2.840.10008.5.1.4.1.1.9.1 .3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Hemodynamic Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.2 .1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Cardiac Elec-trophysiology Waveform Storage | 1.2.840.10008.5.1.4.1.1.9.3 .1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Basic Voice Audio Wave- form Storage | 1.2.840.10008.5.1.4.1.1.9.4 .1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| X-Ray Angigraphic Image | 1.2.840.10008.5.1.4.1.1.12. 1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| X-Ray Radiofluoroscopic- Image | 1.2.840.10008.5.1.4.1.1.12. 2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| NM Image | 1.2.840.10008.5.1.4.1.1.20 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Basic Text SR | 1.2.840.10008.5.1.4.1.1.88. 11 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID |
|--------------------------------|-----------------------------------|------------------------------------------------------------------|
| Enhanced SR | 1.2.840.10008.5.1.4.1.1.88. 22 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Comprehensive SR | 1.2.840.10008.5.1.4.1.1.88. 33 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Mammography CAD SR | 1.2.840.10008.5.1.4.1.1.88. 50 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Chest CAD SR | 1.2.840.10008.5.1.4.1.1.88. 65 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Procedure Log | 1.2.840.10008.5.1.4.1.1.88. 40 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| X-Ray Radiation Dose SR | 1.2.840.10008.5.1.4.1.1.88. 67 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| PET Image | 1.2.840.10008.5.1.4.1.1.12 8 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Image | 1.2.840.10008.5.1.4.1.1.48 1.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Dose | 1.2.840.10008.5.1.4.1.1.48 1.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Structure Set | 1.2.840.10008.5.1.4.1.1.48 1.3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID |
|-----------------------------------|-----------------------------------|------------------------------------------------------------------|
| RT Beams Treatment Record | 1.2.840.10008.5.1.4.1.1.48 1.4 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Plan | 1.2.840.10008.5.1.4.1.1.48 1.5 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Brachy Treatment Record | 1.2.840.10008.5.1.4.1.1.48 1.6 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Treatment Summary Record | 1.2.840.10008.5.1.4.1.1.48 1.7 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Ion Plan | 1.2.840.10008.5.1.4.1.1.48 1.8 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| RT Ion Beams Treatment Record | 1.2.840.10008.5.1.4.1.1.48 1.9 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |
| Csa Non-Image | 1.3.12.2.1107.5.9.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 |

RWA: Import into local Storage

invoke this Application Entity for the Import to Local Storage RWA.

Real-World Activity: Export to local Archive Media

The DICOM Archive application acts as FSU (for media with existing DICOM file-set) or FSC (media not initialized) using the interchange option when requested to copy SOP Instances from the local storage to local Archive medium.

The DICOM Archive application will receive a list of SOP Instances to be copied to the local archive medium. According to the state of the medium inserted (new medium, Medium with DICOM file-set) the validity of the SOP Instances according to the applicable profile is checked. Only valid SOP Instances are accepted.

When the DICOM archive application is requested to copy SOP Instances the preferred application profile according configuration will be used to validate and copy the referred SOP Instances. When creating a new file set no Descriptor File will be allocated and the related ID is not used.

The DICOM archive application will not close the medium.

Application Profiles for the RWA: Export to local Archive Media

See table \rightarrow page B.2-1 for the Application Profiles listed that invoke this Application Entity for the Export to local Archive Media RWA.

DICOM Conformance Statement

CHAPTER **B.3**

Augmented and Private Profiles

Augmented Application Profiles

AUG-GEN-CD

With no private Siemens Non-Images stored onto Medium, the definitions of the PRI-SYNGO-CD Profile are applicable to denote the augmentations for the STD-GEN-CD Standard Profile.

Storage of Private Information Objects will only be supported with reference to a Private Application Profile (see next section).

The Siemens non-image is typically used for raw data and 3D private data.

AUG-CTMR-xxxx

With no private Siemens Non-Images stored onto Medium, the definitions of the PRI-SYNGO-CD Profile are applicable to denote the augmentations for the and STD-CTMR-CD Standard Profiles.

Storage of Private Information Objects will only be supported with reference to a Private Application Profile (see next section).

AUG-XA1K-CD

With no private Siemens Non-Images stored onto Medium, the definitions of the PRI-SYNGO-CD Profile are applicable to denote the augmentations for the STD-XA1K-CD Standard Profile.

Storage of Private Information Objects will only be supported with reference to a Private Application Profile (see next section).

syngo[®] private offline Media Application Profile

This section will contain a *syngo* specific Application Profile.

The Structure of this Application Profile is defined in Part 11 of the 2000 DICOM Standard.

It is needed to describe the requirements for Offline Media Storage of the private IOD (Non-Image IOD).

Class and Profile Identification

This document defines an Application Profile class for "syngo® speaking1" modalities or applications.

The identifier for this class shall be PRI-SYNGO. This class is intended to be used for interchange of extended and private Information Objects via CD-R offline media between dedicated acquisition or workstation modalities built from a common *syngo* architecture.

^{1. &#}x27;syngo' and 'We speak syngo' are registered trademarks of Siemens AG

The specific application profiles in this class are shown in the table below:

| Application Profile | Identifier | Description |
|------------------------------------------|---------------|-------------------------------------------------------------------------------------------------------------------------|
| "syngo speaking" System on CD-R | PRI-SYNGO-CD | Handles interchange of Composite SOP Instances and privately defined SOP Instances (Siemens Non- Image IOD) |
| " <i>syngo</i> speaking" System on DVD R | PRI-SYNGO-DVD | Handles interchange of Composite SOP Instances and privately defined SOP Instances (Siemens Non- Image IOD) |

Equipment claiming conformance for this *syngo* Application Profile Class shall make a clear statement on handling of the private defined SOP Instances.

Clinical Context

This application profile facilitates the interchange of original acquired and derived images and private data related to them. Typical media interchange would be from in-lab acquisition equipment to dedicated workstations and archive systems with specific extensions to handle the private data objects (in both directions).

Additionally, images used to prepare procedures, multi-modality images (e.g. integrated US) and images derived from primary diagnostic images, such as annotations, quantitative analysis images, reference images, screen capture images may be interchanged via this profile.

Roles and Service Class Options

This Application Profile uses the Media Storage Service Class defines in PS 3.4 with the Interchange Option.

The Application Entity shall support one or more of the roles of File Set Creator (FSC), File Set Reader (FSR), and File Set Updater (FSU), defined in PS3.10.

The Application Entity acting as File Set Creator generates a File Set under the PRI-SYNGO Application Profiles.

File Set Creators shall be able to generate the Basic Directory SOP Class in the DICOMDIR file with all the subsidiary Directory Records related to the Image SOP Classes and Private SOP Classes stored in the File Set.

In case of the PRI-SYNGO-CD and PRI-SYNGO-DVD profile, the FSC shall offer the ability to allow multi-session (additional information may be subsequently added to the disc). For both profiles a multi-session media can be finalized.

File Set Creator

Note

A multiple volume (a logical volume that can cross multiple phydical media) is not supported by this Application Profile Class. If a set of Files, e.g., a Study, cannot be written entirely on one CD-R, the FSC will create multiple independent DICOM File Sets such that each File Set can reside on a single CD-R medium controlled by its individual DICOMDIR file. The user of the FSC can opt to use written labels on the discs to reflect that there is more than one disc for this set of files (e.g. a study).

File Set Reader

The role of the File Set Reader shall be used by Application Entities which receive the transferred File Set.

File Set Readers shall be able to read all the defined SOP Instances files defined for the specific Application Profiles to which a conformance claim is made, using all the defined Transfer Syntaxes.

File Set Updater

The role of the File Set Updater shall be used by Application Entities, which receive a transferred File Set and update it by the addition of processed information.

File Set Updaters shall be able to read and update the DICOM-DIR file. File Set Updaters do not have to read the image/private information objects. File Set Updaters shall be able to generate any of the SOP Instance files defined for the specific Application Profiles to which a conformance claim is made, and to read and update the DICOMDIR file.

In case of the PRI-SYNGO-CD and PRI-SYNGO-DVD profile, the FSU shall offer the ability to allow multi-session (additional information can be subsequently added to the disc).

Note (for CD-R and DVD-R)

If the disc has not been finalized, the File Set Updater will be able to update information assuming there is enough space on the disc to write a new DICOMDIR file, the information, and the fundamental CD-R/DVD-R control structures. CD-R/DVD-R control structures are the structures that inherent to CD-R/DVD-R standards; see PS 3.12

PRI-SYNGO Profiles

SOP Classes and Transfer Syntaxes

These Application Profiles are based on the Media Storage Service Class with the Interchange Option. In the table below Transfer Syntax UID "RLE Lossless" applies ony for decompression.

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------------|-------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| Basic Directory | 1.2.840.10008.1.3.10 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | M |
| CR Image | 1.2.840.10008.5.1.4.1.1 .1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| CR Image | 1.2.840.10008.5.1.4.1.1 .1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| CR Image | 1.2.840.10008.5.1.4.1.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| CR Image | 1.2.840.10008.5.1.4.1.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| CR Image | 1.2.840.10008.5.1.4.1.1 .1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| CR Image | 1.2.840.10008.5.1.4.1.1 .1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|---------------------------------------|---------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| CR Image | 1.2.840.10008.5.1.4.1.1 .1 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| CR Image | 1.2.840.10008.5.1.4.1.1 .1 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | 0 | 0 |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | М | 0 |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | Ο | 0 |

| | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|---------------------------------------|-----------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| Object Definitions | | | | | |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| DX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.1 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | 0 | 0 |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.1.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.1.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1 .1.1.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.1.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.1.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| DX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.1.1 | RLE Lossless 1.2.840.10008.1.2.5 | O | 0 | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|---------------------------------------|---------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | М | 0 |
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | JPEG Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | JPEG Lossy 1.2.840.10008.1.2.91 | 0 | Ο | Ο |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|----------------------------------------|-----------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| MG Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.2 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.2.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.2.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1 .1.2.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.2.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.2.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |
| MG Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.2.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|----------------------------------------|---------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | М | - |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | М | 0 |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | Ο |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | JPEG Lossless 1.2.840.10008.1.2.90 | 0 | 0 | Ο |
| IOX Image - For Presentatio n | 1.2.840.10008.5.1.4.1.1 .1.3 | JPEG Lossy 1.2.840.10008.1.2.91 | 0 | Ο | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|----------------------------------|-----------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.3.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | М | 0 |
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.3.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1 .1.3.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.3.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.3.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| IOX Image - For Processing | 1.2.840.10008.5.1.4.1.1. 1.3.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------------|-------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| CT Image | 1.2.840.10008.5.1.4.1.1 .2 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | 0 | 0 |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | - | M | - |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | - | M | - |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | - | M | - |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------------|---------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | - | M | - |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | RLE Lossless 1.2.840.10008.1.2.5 | - | M | - |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | - | M | - |
| US-MF image (retired) | 1.2.840.10008.5.1.4.1.1 .3 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | - | M | - |
| US-MF image | 1.2.840.10008.5.1.4.1.1 .3.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| US-MF image | 1.2.840.10008.5.1.4.1.1 .3.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| US-MF image | 1.2.840.10008.5.1.4.1.1 .3.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| US-MF image | 1.2.840.10008.5.1.4.1.1 .3.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| US-MF image | 1.2.840.10008.5.1.4.1.1 .3.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |

| Object | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|-----------------------|---------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| Definitions | | | | | |
| US-MF image | 1.2.840.10008.5.1.4.1.1 .3.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | M | Ο |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | М | 0 |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| MR Image | 1.2.840.10008.5.1.4.1.1 .4 | JPEG 2000 Lossy 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | - | М | - |

| Information | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|-----------------------|---------------------------------|---------------------------------------------------------------------------------|-----|------|-----|
| Object Definitions | OOI Olass old | manister Symax and SIB | 100 | 1011 | 100 |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | - | М | - |
| US Image (retired) | 1.2.840.10008.5.1.4.1 .1.6 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | - | M | - |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | - | M | - |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | RLE Lossless 1.2.840.10008.1.2.5 | - | M | - |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | - | M | - |
| US Image (retired) | 1.2.840.10008.5.1.4.1.1 .6 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | - | M | - |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | Ο | M | 0 |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------------|---------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | M | 0 |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | M | 0 |
| US Image | 1.2.840.10008.5.1.4.1.1 .6.1 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | M | 0 |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |

| Information Object | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|---------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| Definitions | | | | | |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| SC Image | 1.2.840.10008.5.1.4.1.1 .7 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | 0 | 0 |
| Waveform Storage SOP Classes | 1.2.840.10008.5.1.4.1.1. 9.1.1 1.2.840.10008.5.1.4.1.1. 9.1.2 1.2.840.10008.5.1.4.1.1. 9.1.3 1.2.840.10008.5.1.4.1.1. 9.2.1 1.2.840.10008.5.1.4.1.1. 9.3.1 1.2.840.10008.5.1.4.1.1. | Uncompressed | M | M | 0 |

| Information | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|---------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| Object Definitions | SOF Class OID | Hallsler Sylitax and Old | rsc | ron | F30 |
| Waveform Storage SOP Classes | 1.2.840.10008.5.1.4.1.1. 9.1.1 1.2.840.10008.5.1.4.1.1. 9.1.2 1.2.840.10008.5.1.4.1.1. 9.1.3 1.2.840.10008.5.1.4.1.1. 9.2.1 1.2.840.10008.5.1.4.1.1. 9.3.1 1.2.840.10008.5.1.4.1.1. 9.4.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.1 | 0 | M | 0 |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|-----------------------------------------|----------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | М | 0 |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | M | 0 |
| X-Ray Angiograph ic Image | 1.2.840.10008.5.1.4.1.1 .12.1 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | M | 0 |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | М | 0 |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | М | 0 |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |

| Information Object | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|-----------------------------------------|----------------------------------|------------------------------------------------------------------------------------------|-----|-----|-----|
| Definitions | | | | | |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | М | 0 |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | M | 0 |
| X-Ray Radioflour oscopic Image | 1.2.840.10008.5.1.4.1.1 .12.2 | JPEG 2000 Lossless 1.2.840.10008.1.2.91 | 0 | M | 0 |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------|-----------------------------------|------------------------------------------------------------------|-----|-----|-----|
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| NM Image | 1.2.840.10008.5.1.4.1.1 .20 | JPEG 2000 Lossless 1.2.840.10008.1.2.91 | Ο | 0 | 0 |
| Basic Text SR | 1.2.840.10008.5.1.4.1.1. 88.11 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| Basic Text SR | 1.2.840.10008.5.1.4.1.1. 88.11 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| Enhanced SR | 1.2.840.10008.5.1.4.1.1. 88.22 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| Enhanced SR | 1.2.840.10008.5.1.4.1.1. 88.22 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| Comprehe nsive SR | 1.2.840.10008.5.1.4.1.1. 88.33 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| Comprehe nsive SR | 1.2.840.10008.5.1.4.1.1. 88.33 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| Mammogra phy CAD SR | 1.2.840.10008.5.1.4.1.1. 88.50 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------------|-----------------------------------|---------------------------------------------------------------------------|-----|-----|-----|
| Mammogra phy CAD SR | 1.2.840.10008.5.1.4.1.1. 88.50 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | М | 0 |
| Chest CAD SR | 1.2.840.10008.5.1.4.1.1. 88.65 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| Chest CAD SR | 1.2.840.10008.5.1.4.1.1. 88.65 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| Procedure Log | 1.2.840.10008.5.1.4.1.1 .88.40 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| Procedure Log | 1.2.840.10008.5.1.4.1.1 .88.40 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| X-Ray Radiation Dose SR | 1.2.840.10008.5.1.4.1.1 .88.67 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | М | 0 |
| X-Ray Radiation Dose SR | 1.2.840.10008.5.1.4.1.1 .88.67 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | М | 0 |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | М | 0 |

| | 000 01 1110 | T (0 : 1111D | | EOD | |
|-----------------------|-----------------------------------|---------------------------------------------------------------------------------|---------|------------|-----|
| Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | М | - |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | Ο | 0 |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| PET Image | 1.2.840.10008.5.1.4.1.1 .128 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | 0 | 0 |
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | M | 0 |
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70 | 0 | M | 0 |
| RT Image | 1.2.840.10008.5.1.4.1.1 .481.1 | JPEG Lossless Process 14 NON-HIER 1.2.840.10008.1.2.4.57 | - | M | - |
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |

| Object | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|---------------------------------|-----------------------------------|---------------------------------------------------------------------------------|-----|-----|-----|
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 | 0 | 0 | 0 |
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | RLE Lossless 1.2.840.10008.1.2.5 | 0 | 0 | 0 |
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | JPEG 2000 Lossless 1.2.840.10008.1.2.90 | 0 | 0 | 0 |
| RT Image | 1.2.840.10008.5.1.4.1.1. 481.1 | JPEG 2000 Lossy 1.2.840.10008.1.2.91 | 0 | 0 | 0 |
| RT Dose | 1.2.840.10008.5.1.4.1.1. 481.2 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| RT Dose | 1.2.840.10008.5.1.4.1.1. 481.2 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| RT Structure Set | 1.2.840.10008.5.1.4.1.1. 481.3 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| RT Structure Set | 1.2.840.10008.5.1.4.1.1. 481.3 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| RT Beams Treatment Record | 1.2.840.10008.5.1.4.1.1. 481.4 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| RT Beams Treatment Record | 1.2.840.10008.5.1.4.1.1. 481.4 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|--------------------------------------|-----------------------------------|------------------------------------------------------------------|-----|-----|-----|
| RT Plan | 1.2.840.10008.5.1.4.1.1. 481.5 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | М | 0 |
| RT Plan | 1.2.840.10008.5.1.4.1.1. 481.5 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| RT Brachy Treatment Record | 1.2.840.10008.5.1.4.1.1. 481.6 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| RT Brachy Treatment Record | 1.2.840.10008.5.1.4.1.1. 481.6 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| RT Treatment Summary Record | 1.2.840.10008.5.1.4.1.1. 481.7 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| RT Treatment Summary Record | 1.2.840.10008.5.1.4.1.1. 481.7 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | М | 0 |
| RT Ion Plan | 1.2.840.10008.5.1.4.1.1. 481.8 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | M | M | 0 |
| RT Ion Plan | 1.2.840.10008.5.1.4.1.1. 481.8 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |

| Information Object Definitions | SOP Class UID | Transfer Syntax and UID | FSC | FSR | FSU |
|----------------------------------------|-----------------------------------|------------------------------------------------------------------|-----|-----|-----|
| RT Ion Beams Treatment Record | 1.2.840.10008.5.1.4.1.1. 481.9 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | М | М | 0 |
| RT Ion Beams Treatment Record | 1.2.840.10008.5.1.4.1.1. 481.9 | Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2 | 0 | M | 0 |
| Csa Non- Image | 1.3.12.2.1107.5.9.1 | Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1 | 0 | M | 0 |

FSC, FSR, FSU - denote the requirement for those roles

O - Optional

M - Mandatory

Physical Media and Formats

The PRI-SYNGO-CD Profile requires the 120mm CD-R physical media with the ISO/IEC 9660 Media Format, as defined in PS3 12

The PRI-SYNGO-DVD Profile requires the 120mm DVD-R physical media with the UDF 2.01 Media Format, as defined in PS3 12

Directory Information in DICOMDIR

Conforming Application Entities shall include in the DICOMDIR File the Basic Directory IOD containing Directory Records at the Patient and subsidiary levels appropriate to the SOP Classes in the File Set. All DICOM files in the File Set incorporating SOP instances defined for the specific Application Profile shall be referenced by Directory Records.

Note

DICOMDIRs with no directory information are not allowed by this Application Profile

Privately defined IODs will be referenced by "PRIVATE" Directory Records.

Basic Directory IOD Specification

This Application Profile makes use of optional attributes of the Basic Directory IOD to support recognition of Patient's Storage Service request results in spanning multiple volumes (file sets). Therefore the File Set Descriptor File can be used and is then referenced by optional Basic Directory IOD attributes. If existent, the specified Descriptor File may be used by FSR applications. Any FSU, FSC shall make a clear statement if the Descriptor File mechanism is used according to the specialization defined in this Application Profile.

The Descriptor Files shall have the following contents:

One single line without any control characters and according to the Basic Character Set having the following defined text:

"MULTIVOLUME: xx of vy"

xx, yy are replaced by the actual number of the volume (xx) and the total number of volumes in the set (yy).

If used, the Descriptor File shall have the File ID "README" and reside in the same directory level as the DICOMDIR. It is referenced by the attribute [0004,1141] File Set Descriptor File ID having the defined contents of "README".

Additional Keys

File Set Creators and Updaters are required to generate the mandatory elements specified in PS 3.3, Annex F of the DICOM Standard. Table below: PRI-SYNGO-CD Additional DICOMDIR Keys specifies the additional associated keys. At each directory record level other additional data elements can be added, but it is not required that File Set Readers be able to use them as keys. Refer to the Basic Directory IOD in PS 3.3.

| Attribute Name | Tag | Directory Record Level | Туре | Notes |
|-----------------------------------|-------------|---------------------------|------|-----------------------------------------|
| Date Of Birth | (0010,0030) | PATIENT | 2C | required, if present in SOP Instance |
| Patient's Sex | (0010,0040) | PATIENT | 2C | required, if present in SOP Instance |
| Series Date | (0008,0021) | SERIES | 3 | |
| Series Time | (0008,0031) | SERIES | 3 | |
| Institution Name | (0008,0080) | SERIES | 2C | required, if present in SOP Instance |
| Institution Address | (0008,0081) | SERIES | 2C | required, if present in SOP Instance |
| Series Description | (0008,103E) | SERIES | 3 | |
| Performing Physician's Name | (0008,1050) | SERIES | 2C | required, if present in SOP Instance |
| Image Type | (8000,0008) | IMAGE | 1C | required, if present in SOP Instance |
| SOP Class UID | (0008,0016) | IMAGE | 3 | |
| SOP Instance UID | (0008,0018) | IMAGE | 3 | |
| Image Date | (0008,0023) | IMAGE | 3 | |

| Attribute Name | Tag | Directory Record Level | Туре | Notes |
|-------------------------------------|--------------|---------------------------|------|-----------------------------------------|
| Image Time | (0008,0033) | IMAGE | 3 | |
| Content Date | (0008,0023) | IMAGE | 3 | |
| Content Time | (0008,0033) | IMAGE | 3 | |
| Referenced Image Sequence | (0008,1140) | IMAGE | 1C | required, if present in SOP Instance |
| > Referenced SOP Class UID | (0008,1150) | | | |
| > Referenced SOP Instance UID | (0008,1155) | | | |
| Image Position (Patient) | (00020,0032) | IMAGE | 2C | required, if present in SOP Instance |
| Image Orientation (Patient) | (0020,0037) | IMAGE | 2C | required, if present in SOP Instance |
| Frame Of Reference UID | (0020,0052) | IMAGE | 2C | required, if present in SOP Instance |
| Rows | (0028,0010) | IMAGE | 3 | |
| Columns | (0028,0011) | IMAGE | 3 | |
| Pixel Spacing | (0028,0030) | IMAGE | 1C | С |
| Calibration Image | (0050,0004) | IMAGE | 2C | required, if present in SOP Instance |
| Icon Image Sequence | (0088,0200) | IMAGE | 3 | required for Image SOP Classes |
| > Samples per Pixel | (0028,0002) | | | 1 |

| Attribute Name | Tag | Directory Record Level | Туре | Notes |
|---------------------------------|-------------|---------------------------|------|--------------------------------------|
| > Photometric Interpretation | (0028,0004) | | | MONOCHROME2 |
| > Rows | (0028,0010) | | | 128 for XA IOD, 64 otherwise |
| > Columns | (0028,0011) | | | 128 for XA IOD, 64 otherwise |
| > Bits Allocated | (0028,0100) | | | 8 |
| > Bits Stored | (0028,0101) | | | 8 |
| > High Bit | (0028,0102) | | | 7 |
| > Pixel Representation | (0028,0103) | | | 0 (unsigned) |
| > Pixel Data | (7FE0,0010) | | | Icon Image pixel data |
| Curve Number | (0020,0024) | CURVE | 1C | required, if present in SOP Instance |

Private Directory Record Keys

Private Directory Records are supproted by this Application Profile Class at the following level:

IMAGE

The PRIVATE Directory Records will have required elements in addition to the mandatory elements specified in PS 3.3.

The following table will list the additional required keys for PRI-VATE Directory Records.

| Attribute Name | Tag | Directory Record Level | Туре | Notes |
|-----------------------|-----------------|---------------------------|------|-----------------------------------------------|
| Private Record UID | (0004, 1432) | PRIVATE | 1 | See Conformance Statement |
| SOP Class UID | (0008, 0016) | PRIVATE | 1C | required, if present in SOP Instance |
| SOP Instance UID | (0008, 0018) | PRIVATE | 1C | required, if present in SOP Instance |
| Image Type | (0008, 0008) | PRIVATE | 3 | identification characteristics |
| Acquisition Date | (0008, 0022) | PRIVATE | 3 | |
| Acquisition Time | (0008, 0032) | PRIVATE | 3 | |
| Acquisition Number | (0020, 0012) | PRIVATE | 3 | |
| CSA Data Type | (0029, xx08) | PRIVATE | 1 | private owner code = SIEMENS CSA NON-IMAGE |
| CSA Data Version | (0029, xx09) | PRIVATE | 3 | private owner code = SIEMENS CSA NON-IMAGE |

Icon Images

Directory Records of type SERIES or IMAGE may include Icon Images. The Icon Image Pixel data shall be as specified in PS 3.3 "Icon Imge Key Definition", and restricted such, that Bits Allocated (0028,0100) and Bits Stored (0028,0101) shall be equal to 8, an d Rows (0028,0010) and Columns (0028, 0011) shall be equal to 128 for XA Images and 64 for all other images. The Photometric interpretation (0028,0004) shall always be restricted to "MONOCHROME2".

PRIVATE Directory Records will not contain Icon Image information.

Other Parameters

This section defines other parameters common to all specific Application Profiles in the PRI-SYNGO class which need to be specified in order to ensure interoperable media interchange.

Multiframe JPEG Format

The JPEG encoding of pixel data shall use Interchange Format (with table specification) for all frames.

DICOM Conformance Statement

CHAPTER **B.4**

Extensions, Specializations and Privatizations of

SOP Classes and Transfer Syntaxes

The SOP Classes listed refer in majority to those created by the equipment to which this Conformance Statement is related to. For SOP classes not listed in this section, please refer to the Storage section of the DICOM Conformance Statement of the product. This will include all SOP Instances that can be received and displayed and therefore will be included into offline media storage even though these SOP Instances are not created by the equipment serving the Media Storage Service.

SOP Specific Conformance Statement for Basic Directory

Extension, Specialization for SIEMENS Non-Image Objects

According to the PRI-SYNGO Application Profile Class the usage of the Private Creator UIDs and further optional keys for the Directory Records referring to SIEMENS Non-Image Objects are listed in the following table.

| Attribute | Tag | Value used |
|--------------------|-------------|---------------------|
| Private Record UID | (0004,1432) | 1.3.12.2.1107.5.9.1 |
| SOP Class UID | (0008,0016) | 1.3.12.2.1107.5.9.1 |

For the Non-Images no Icon Image Sequence will be generated.

Configuration

AE Title Mapping

DICOM Media Storage AE Title

The DICOM Storage application (Image Manager) provides the application entity title:

CsalmageManager



Support of Extended Character Sets

The Somaris/7 DICOM application supports the following character sets as defined in the four tables below:

The following table shows Single_Byte Character Sets without Code Extenension:

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|---------------------|
| Default repertoire | none | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 1 | ISO_IR 100 | ISO_IR 100 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 2 | ISO_IR 101 | ISO_IR 101 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 3 | ISO_IR 109 | ISO_IR 109 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 4 | ISO_IR 110 | ISO_IR 110 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Cyrillic | ISO_IR 144 | ISO_IR 144 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Arabic | ISO_IR 127 | ISO_IR 127 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Greek | ISO_IR 126 | ISO_IR 126 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Hebrew | ISO_IR 138 | ISO_IR 138 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Latin alphabet No. 5 | ISO_IR 148 | ISO_IR 148 | Supplementary set |
| | | ISO_IR 6 | ISO 646 |
| Japanese | ISO_IR 13 | ISO_IR 13 | JIS X 0201:Katakana |

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|-------------------|
| | | ISO_IR 14 | JIS X 0201:Romaji |

The following table shows Single-Byte Character with Code Extension:

| Character Set Description | Defined Term | Standard for Code Extension | ESC Sequence | ISO Registration Number | Character Set |
|---------------------------------|--------------------|-----------------------------------|--------------------|-------------------------------|-------------------|
| Default repertoire | ISO 2022 IR 6 | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.1 | ISO 2022 IR 100 | ISO 2022 | ESC 02/13 04/01 | ISO_IR 100 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.2 | ISO 2022 IR 101 | ISO 2022 | ESC 02/13 04/02 | ISO_IR 101 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.3 | ISO 2022 IR 109 | ISO 2022 | ESC 02/13 04/03 | ISO_IR 109 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.4 | ISO 2022 IR 110 | ISO 2022 | ESC 02/13 04/04 | ISO_IR 110 | Supplementary set |

| Character Set Description | Defined Term | Standard for Code Extension | ESC Sequence | ISO Registration Number | Character Set |
|---------------------------------|--------------------|-----------------------------------|--------------------|-------------------------------|----------------------------|
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Cyrillic | ISO 2022 IR 144 | ISO 2022 | ESC 02/13 04/12 | ISO_IR 144 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Arabic | ISO 2022 IR 127 | ISO 2022 | ESC 02/13 04/07 | ISO_IR 127 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Greek | ISO 2022 IR 126 | ISO 2022 | ESC 02/13 04/06 | ISO_IR 126 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Hebrew | ISO 2022 IR 138 | ISO 2022 | ESC 02/13 04/08 | ISO_IR 138 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Latin alphabet No.5 | ISO 2022 IR 148 | ISO 2022 | ESC 02/13 04/13 | ISO_IR 148 | Supplementary set |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 6 | ISO 646 |
| Japanese | ISO 2022 IR 13 | ISO 2022 | ESC 02/13 04/09 | ISO_IR 13 | JIS X 0201:Katakana |
| | | ISO 2022 | ESC 02/08 04/02 | ISO_IR 14 | JIS X 0201- 1976:Romaji |

The following table shows Multi-Byte Character Sets without Code Extension:

| Character Set Description | Defined Term | ISO Registration Number | Character Set |
|------------------------------|--------------|----------------------------|-------------------------------------------------------------|
| Unicode | ISO_IR 192 | ISO 10646 | Unicode in UTF-8 |
| Chinese | GB18030 | GB18030 | GB 18030-2000 (China Association for Standardization) |

The following table shows Multi-Byte Character Sets with Code Extension:

| Character Set Description | Defined Term | Standard for Code Extension | ESC Sequence | ISO Registration Number | Character Set |
|---------------------------------|--------------------|-----------------------------------|--------------------------|-------------------------------|------------------------------------------------------------|
| Japanese | ISO 2022 IR 87 | ISO 2022 | ESC 02/04 04/02 | ISO_IR 87 | JIS X 0208: Kanji |
| | ISO 2022 IR 159 | ISO 2022 | ESC 02/04 02/08 04/04 | ISO_IR 159 | JIS X 2012: Supplementary Kanji set |
| Chinese ^a | ISO 2022 IR 58 | ISO 2022 | ESC 02/04 04/01 | ISO_IR 58 | GB2312-80 (China Association for Standardization) |

a. This Character Set is an extension of DICOM for the chinese language.

When there is a mismatch between the SCS tags (0008,0005) and the characters in an IOD coming into the system, then the following measures are taken to make the characters DICOM conform:

| | ry to import with ISO-IR 100. If ISO-IR 100 also fails convert ach illegal character to'?'. |
|------|-----------------------------------------------------------------------------------------------------------------------------|
| | re are now 3 categories of character sets which have to be rentiated because of their different encoding formats: |
| IS | Conventional ISO character sets: ISO_IR 6, ISO 2022 IR 6, SO_IR 100, etc. >Encoded in ISO 2022 |
| □ 18 | SO_IR 192: Encoded in UTF-8 |
| □ G | B18030: Encoded in GB18030 |
| | is not possible to recognize the following mismatches utomatically on receiving or importing: |
| C | n attribute value is encoded in ISO_IR 192 <=> (0008,0005) ontains a conventional ISO character set as primary character et |
| C | n attribute value is encoded in GB18030 <=> (0008,0005) ontains a conventional ISO character set as primary haracter set |
| | n attribute value is encoded in ISO 2022 <=> (0008,0005) ontains ISO_IR 192 |
| | n attribute value is encoded in ISO 2022 <=> (0008,0005) ontains GB18030 |
| ۸n I | OD that contains one of the above mentioned inconsisten |

An IOD that contains one of the above mentioned inconsistencies is not DICOM conform. As these kinds of inconsistencies cannot be recognized by the system, the IOD will not be rejected but the character data might be corrupted

Older versions of *syngo* do not support the newly introduced character sets ISO_IR 192 and GB18030 and their special encodings. That means, an IOD which contains one of these new character sets in (0008,0005) will be rejected by an older *syngo* system.

| Media Storage – Support of Extended Character Sets | DICOM Conformance Statement |
|----------------------------------------------------|-----------------------------|
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| | |



Annex

Siemens Private Non-Image IOD

Please see section

→ page A.4–70, Private SOP class CSA Non-Image.

Siemens Standard Extended Modules

Please see section

→ page A.4–56, All syngo Supported Image SOP Classes.

Siemens Standard Extended Modules

Please see section

→ page A.4–52, Registry of DICOM Data Elements and → page A.4–67. Registry of DICOM Data Elements.

Standard Extensions of all SOP Classes

Please see section

→ page A.4–1, Standard Extensions of all SOP Classes.

DICOM Print SCU - detailed status displays

Please see section

→ page A.2–118, DICOM Print SCU - detailed status displays.

