

The functional information gathered by *syngo* iFlow was very valuable for choosing the right treatment strategy for the next step.

Courtesy of

Jianping Gu, MD, Wensheng Lou, MD, Department of Interventional Radiology, Nanjing No. 1 Hospital, China

Supported by syngo iFlow

System & Software Artis zee VC14 syngo X Workplace VB21

Case Description

Patient history

A 75-year-old male patient with 8-year history of hypertension suffered from low extremity arterial occlusive disease. Implantation of a stent 3 years prior, presenting with severe pain and numbness in his right lower limb.

Diagnosis

The middle and upper segments of the right superficial femoral artery were occluded, while the lower segment still showed sufficient perfusion due to collateral flow. The popliteal, peroneal, anterior tibial, and posterior tibial arteries were not obstructed but presented with localized plaque formations. A severe stenosis existed at the bifurcation of the posterior tibial artery.

Treatment

A balloon dilatation and subsequent thrombolysis therapy were performed at the upper segment of the right superficial femoral artery. 2 stents (6x150 mm, Protege, EV3) were implanted into the right superficial femoral artery to reopen the vessel.

General comments

This case demonstrated the capabilities of *syngo* iFlow to evaluate the perfusion and circulation in and around distal arteries during the interventional procedure. The functional information was very valuable for choosing the right treatment strategy for the next step.



Real-time assessment of revascularization of peripheral vascular disease

Acquisition protocol	DSA 2 f/s
Injection protocol	
Catheter position	Not specified
Contrast medium (CM)	320 mg iodine/mL
Dilution (CM/Saline):	No
Injection volume	8 mL
Injection rate	3 mL/s
Duration of injection	2.6 s
X-ray delay	No
Power injector used	Yes

Clinical Images

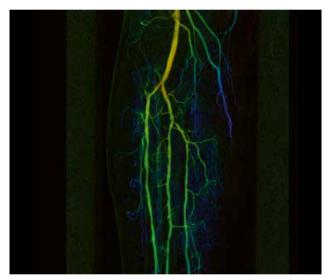


Figure 1: *syngo* iFlow image before treatment of the stenosed superficial femoral artery

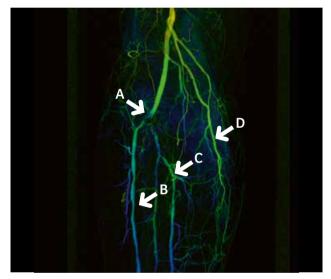


Figure 2: During stenting of the superficial femoral artery an occlusion of the popliteal bifurcation was caused by an embolus (A)

syngo iFlow shows the hemodynamic changes in the distal vessels. Time-to-peak (TTP) analysis based on the syngo iFlow images showed an increased flow in a collateral branch (D) \downarrow 2.5 s. At the same time blood flow within the anterior (B) \uparrow 2.5 s and posterior tibial arteries (C) \uparrow 1.5 s slowed down

Clinical Images

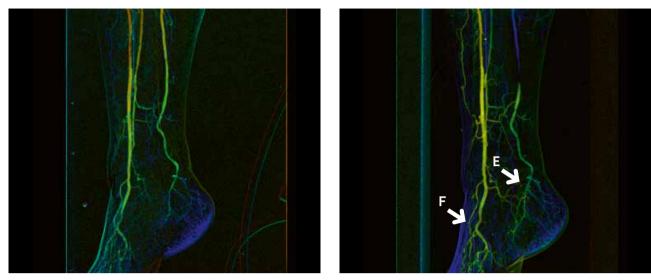


Figure 3: TTP shows that (E) the blood flow in the malleolar artery was almost the same as before (TTP \uparrow 0.5 s) and that (F) the blood flow in the dorsalis pedis artery was improved (TTP \downarrow 3.5 s)

syngo iFlow measurement shows improved distal blood flow after intervention despite the embolus

Siemens Healthineers Headquarters

Siemens Healthcare GmbH

Henkestr. 127

91052 Erlangen, Germany Phone: +49 9131 84-0 siemens-healthineers.com The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.

All rights reserved.