### **Study Protocol**

# Transarterial embolization of ependymom metastases in the liver

**Oncology Interventions** 

### **Case Description**

### Patient history

60-year-old female patient with history of extraneural myxopapillary ependymoma, initially diagnosed in 2009.

#### Diagnosis

Liver-, lung- and lymphnode metastases since 2015.

#### Treatment

First session of transarterial embolisation of livermetastases with bland embospheres (100-300 um).

We placed the catheter in the A. mesenterica superior for arteriography and indirect mesentericoportography, followed by arteriography of the celiac trunc.

A cone beam CT run in arterial phase was done with microcatheter placement in the proper hepatic artery. Embolisation was performed from tumorfeeding branch of the right hepatic artery. After embolization a final DSA showed a decreased perfusion in the embolized liver arteries.

#### **General comments**

With the help of the *syngo* DynaCT 3D vessel map, we can ensure the proper position of the catheter for selective embolisation. It enhances the identification of aberrrant vessels, we easily could verify that there are no vessel branches to the bowel or stomach distal to the embolization position.

#### **Tips and tricks**

To gather best *syngo* DynaCT image quality, proper breathhold of the patient is vital.

Explain the importance of his cooperation and give breathhold commands understandable and vigorous to the patient.



#### Courtesy of

PD Dr. Jan Hinrichs, MD Institute for Diagnostic and Interventional Radiology, Medical School Hannover, Germany

Supported by syngo DynaCT

## System & Software

Artis pheno VE 10

## Transarterial embolization of ependymom metastases in the liver

Acquisition protocol	5s DCT Body
Injection protocol	
Catheter position	Microcatheter in proper hepatic artery
Contrast medium (CM)	300 mg iodine/mL
Dilution (CM/Saline):	60%
Injection volume	24 mL
Injection rate	2.0 mL/s
Duration of injection	12 s
X-ray delay	5 sec
Power injector used	No
Reconstructions	
Name	DCT Body
VOI size	Full
Slice matrix	512×512
Kernel type	HU
Image characteristics	Normal
Reconstruction mode	Nat
Viewing preset	DynaCT Body

## **Clinical Images**





Figure 1: DSA scene in embolisation position



Figure 2: *syngo* DynaCT VRT visualization



Figure 3: *syngo* DynaCT imaging MIP 10 mm Shows tumor and feeding arteries in axial, coronal and sagital

#### **Siemens Healthineers Headquarters**

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen, Germany Phone: +49 913184-0 siemens-healthineers.com The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.

All rights reserved.