

SIEMENS MAGNETOM Avanto syngo MR B17

\\USER\PLM AW Ortho\20min_protocols_NYU_27.05.09\elbow\t2_tse_fs_sag
TA: 3:20 PAT: Off Voxel size: 0.5×0.5×2.5 mm Rel. SNR: 1.00 SIEMENS: tse

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	23
Dist. factor	10 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	H >> F
Rotation	95.70 deg
Phase oversampling	50 %
FoV read	120 mm
FoV phase	100.0 %
Slice thickness	2.5 mm
TR	5360 ms
TE	55 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Elliptical filter
Coil elements	FS

Contrast

MTC	Off
Magn. preparation	None
Flip angle	120 deg
Fat suppr.	SPAIR
Fat sat. mode	Weak
Water suppr.	None
Restore magn.	On
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	256
Phase resolution	100 %
Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	Off

B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
Special sat.	None
Set-n-Go Protocol	Off
Table position	H
Table position	1 mm
Inline Composing	Off
Tim CT mode	Off

System

Body	Off
FS	On
Positioning mode	ISO
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Sagittal
Rotation	95.70 deg
A >> P	120 mm
F >> H	120 mm
R >> L	64 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	2D
Compensate T2 decay	Off

SIEMENS MAGNETOM Avanto syngo MR B17

Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	199 Hz/Px
Flow comp.	No
Allowed delay	0 s
Echo spacing	11.1 ms
Define	Turbo factor
Turbo factor	11
Echo trains per slice	35
RF pulse type	Low SAR
Gradient mode	Fast