

NOTE

This protocol was provided by
**Prof. Michael Recht, NYU
 School of Medicine, Dep. of
 Radiology, NY, USA, on behalf of
 the MSK Advisory Board**

contrast:
 sequence type:
 slice orientation:

acquisition time [min:s]:
 resolution [mm³):
 matrix [Px2]:
 bandwidth (Hz/Px):
 turbo factor:
 TR [ms]:
 TE [ms]:
 TI [ms]:
 field of view [mm]:

2D SCAN

PD TSE	PD TSE	PD TSE	PD TSE	T1 TSE	T1 TIRM	PD TSE
transversal	transversal	coronal	coronal	sagittal	sagittal	sagittal
02:37	03:02	02:39	02:56	01:48	01:59	03:02
0.4x0.3x3.0	0.4x0.3x3.0	0.4x0.3x3.0	0.4x0.3x3.0	0.5x0.4x3.0	0.5x0.4x3.0	0.4x0.4x3.0
512x348	320x256	512x448	512x348	384x288	384x288	320x320
257	100	257	257	372	250	256
9	7	9	9	4	12	9
4450	5630	4500	4860	700	4500	4630
47	30	52	52	11	38	43
				190		
140	110	140	140	140	140	140

3D SCAN

	NOT YET AVAILABLE
--	-------------------------

clinical requests

general
 injury ligaments
 coalition
 degenerative pain osteoarthritis

total scan time [min:s]
 2D scan → 18:03
 13:25
 13:27
 13:27

pd_tse_tra	pd_tse_fs_tra	pd_tse_cor	pd_tse_fs_cor	t1_tse_sag	t1_tirm_sag	pd_tse_fs_sag
pd_tse_tra	pd_tse_fs_tra		pd_tse_fs_cor	t1_tse_sag		pd_tse_fs_sag
	pd_tse_fs_tra	pd_tse_cor	pd_tse_fs_cor	t1_tse_sag		pd_tse_fs_sag
	pd_tse_fs_tra	pd_tse_cor	pd_tse_fs_cor	t1_tse_sag		pd_tse_fs_sag

MAGNETOM Verio Ankle Protocol

www.siemens.com/magnetom-world

SIEMENS