

SIEMENS MAGNETOM Product syngo MR B17

\\USER\knee_20min\20min_protocols_NYU_27.05.09\knee_ok\pd_tse_fs_tra
TA: 2:48 PAT: Off Voxel size: 0.5×0.4×3.0 mm Rel. SNR: 1.00 SIEMENS: tse

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	30
Dist. factor	10 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	76.00 deg
Phase oversampling	60 %
FoV read	140 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	5210 ms
TE	47.0 ms
Averages	1
Concatenations	1
Filter	Elliptical filter
Coil elements	K15

Contrast

MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	Fat sat.
Fat sat. mode	Weak
Water suppr.	None
Restore magn.	Off
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	320
Phase resolution	85 %
Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On

Mode

Inplane

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
Sat. region 1	
Thickness	53 mm
Position	L0.6 A1.8 H121.8
Orientation	T > C0.9 > S-0.3
Special sat.	None
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off
Tim CT mode	Off

System

Body	Off
K15	On
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	76.00 deg
A >> P	140 mm
R >> L	140 mm
F >> H	99 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	2D

SIEMENS MAGNETOM Product syngo MR B17

Compensate T2 decay	Off
Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	182 Hz/Px
Flow comp.	No
Allowed delay	30 s
Echo spacing	9.46 ms
Define	Turbo factor
Turbo factor	15
Echo trains per slice	30
RF pulse type	Fast
Gradient mode	Fast