# ADVIA Centaur PSA Assay Specifications





The Siemens Healthcare Diagnostics ADVIA Centaur® PSA assay is a two-site sandwich immunoassay employing direct chemiluminescence technology using a polyclonal goat anti-PSA antibody labeled with acridinium ester in the LITE reagent and a monoclonal mouse anti-PSA antibody which is covalently coupled to paramagnetic particles in the solid phase.

## Outstanding Assay Performance

- Equimolar performance across the range of PSA concentrations
- Excellent correlation with other equimolar PSA assays
- Standardized to WHO 96/670 PSA standard for consistent results

### **Clinical Utility**

- Clinically proven ranges for detection of prostate cancer in screening populations
- Clinically relevant standardized assay range and sensitivity for monitoring patients with prostate cancer

#### **ADVIA Centaur - Maximizing Satisfaction**

- Optimal productivity ADVIA Centaur/XP up to 240 tests per hour; ADVIA Centaur CP - up to 180 tests per hour
- Comprehensive menu including Allergy, Anemia, Cardiovascular, Metabolic, Fertility, Infectious Disease, Oncology, TDM, and Thyroid

### **Equimolarity**

Five samples with PSA-ACT concentrations ranging from 0-100% and a total PSA concentration of approximately 4.0 ng/mL were analyzed to show equimolar performance of the ADVIA Centaur PSA Assays.

Equimolarity				
% PSA-ACT	ADVIA Centaur PSA (ng/mL)			
0%	4.2			
20%	4.1			
50%	4.5			
80%	4.5			
100%	4.5			

**ADVIA Centaur® PSA Assay Specifications** 

Answers for life.

**SIEMENS** 

### **ADVIA Centaur PSA Assay Specifications**

### **PSA Performance Summary**

	Sample Type	Sample Volume	Assay Range	Analytical Sensitivity	Cut-off (suggested)	Calibration Interval	On-Board Stability
ADVIA Centaur	Serum	35 uL	0.01-100 ng/mL	0.01 ng/mL	4.0 ng/mL	28 days	28 days
ADVIA Centaur CP	Serum	35 uL	0.01-100 ng/mL	0.01 ng/mL	4.0 ng/mL	28 days	28 days

Ordering Information					
Catalog No.	Description	Contents			
118157	<ul> <li>5 ReadyPack primary reagent packs of ADVIA Centaur PSA LITE reagent and solid phase ADVIA Centaur PSA Master Curve Card</li> </ul>	500 tests			
118156	<ul> <li>1 ReadyPack primary reagent pack of ADVIA Centaur PSA LITE reagent and solid phase ADVIA Centaur PSA Master Curve Card</li> </ul>	100 tests			
118221	Calibrator Q ADVIA Centaur	6 vials of low calibrator 6 vials of high calibrator			
118220	Calibrator Q ADVIA Centaur	2 vials of low calibrator 2 vials of high calibrator			

ADVIA Centaur and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc. All other trademarks and brands are the property of their respective owners. Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

### Siemens Global Headquarters

Siemens AG Wittelsbacherplatz 2 80333 Muenchen Germany

### Global Siemens Healthcare Headquarters

Siemens AG Healthcare Sector Henkestrasse 127 91052 Erlangen, Germany Telephone: +49 9131 84 - 0 www.siemens.com/healthcare

### **Global Division**

Siemens Healthcare Diagnostics Inc. 1717 Deerfield Road Deerfield, IL 60015-0778 USA www.siemens.com/diagnostics

www.siemens.com/diagnostics