

UROSKOP Access VA14

SP

DICOM Conformance Statement UROSKOP Access

Copyright by SIEMENS AG Medical Solutions Erlangen

This page intentionally left blank.

Table of Contents

Table of Contents.....	3
List of Figures	10
List of Tables.....	11
1 Introduction	14
1.1 Purpose of the Document.....	14
1.2 Scope.....	14
1.3 Definitions and Abbreviations	14
1.3.1 Definitions	14
1.3.2 Abbreviation	15
1.4 References.....	15
1.5 Connectivity and Interoperability	15
2 Implementation Models	16
2.1 Verification	16
2.1.1 Application Data Flow Diagram.....	16
2.1.2 Functional Definitions of Application Entities.....	16
2.1.3 Sequencing of real World Activities.....	16
2.2 Storage	17
2.2.1 Application Data Flow Diagram.....	17
2.2.2 Functional Definitions of Application Entities.....	17
2.2.3 Sequencing of real World Activities.....	18
2.3 Storage Commitment Push Model.....	18
2.3.1 Application Data Flow Diagram.....	18
2.3.2 Functional Definitions of Application Entities.....	19
2.3.3 Sequencing of real World Activities.....	19
2.4 Query/Retrieve.....	19
2.4.1 Application Data Flow Diagram.....	19
2.4.2 Functional Definitions of Application Entities.....	21
2.4.3 Sequencing of real World Activities.....	21
2.5 Print.....	21
2.5.1 Application Data Flow Diagram.....	21
2.5.2 Functional Definitions of Application Entities.....	21
2.5.3 Sequencing of real World Activities.....	22

2.6 Modality Worklist.....	22
2.6.1 Application Data Flow Diagram.....	22
2.6.2 Functional Definitions of Application Entities.....	22
2.6.3 Sequencing of real World Activities.....	23
2.7 Modality Performed Procedure Step.....	23
2.7.1 Application Data Flow Diagram.....	23
2.7.2 Functional Definitions of Application Entities.....	23
2.7.3 Sequencing of real World Activities.....	24
3 AE Specifications.....	25
3.1 Verification AE Specification.....	25
3.1.1 Association Initiation by Real-World Activity.....	25
3.1.1.1 Real-World Activity - Verification SCU.....	25
3.1.1.1.1 Associated Real-World Activity - Verification SCU.....	25
3.1.1.1.2 Proposed Presentation Contexts - Verification SCU	25
3.1.1.1.3 SOP Specific Conformance Statement - Verification SCU	25
3.1.2 Association Acceptance Policy.....	26
3.2 Storage AE Specification.....	26
3.2.1 Association Establishment Policies	26
3.2.1.1 General	26
3.2.1.2 Number of Associations	26
3.2.1.3 Asynchronous Nature	27
3.2.1.4 Implementation Identifying Information.....	27
3.2.1.5 Maximum PDU Size offered.....	27
3.2.1.6 Dicom Application Context.....	27
3.2.2 Association Initiation by Real-World Activity.....	27
3.2.2.1 Real-World Activity - Storage SCU.....	27
3.2.2.1.1 Associated Real-World Activity - Storage SCU.....	27
3.2.2.1.2 Proposed Presentation Contexts - Storage SCU	27
3.2.2.1.3 SOP Specific Conformance Statement - Storage SCU	30
3.2.3 Association Acceptance Policy.....	31
3.2.3.1 Real-World Activity - Storage SCP	31
3.2.3.1.1 Associated Real-World Activity - Storage SCP	31
3.2.3.1.2 Accepted Presentation Contexts - Storage SCP.....	31
3.2.3.1.3 SOP Specific Conformance Statement – Storage SCP	33
3.2.3.1.4 Presentation Context Acceptance Criterion – Storage SCP.....	35
3.2.3.1.5 Transfer Syntax Selection Policies – Storage SCP	35

3.3 Storage Commitment AE Specification.....	36
3.3.1 Association Establishment Policies	36
3.3.1.1 General	36
3.3.1.2 Number of Associations	36
3.3.1.3 Asynchronous Nature	36
3.3.1.4 Implementation Identifying Information.....	36
3.3.2 Association Initiation Policy.....	37
3.3.2.1 Real-World Activity - Send Storage Commitment Request.....	37
3.3.2.1.1 Associated Real-World Activity - Send Storage Commitment Request.....	37
3.3.2.1.2 Proposed Presentation Contexts - Send Storage Commitment Request	37
3.3.2.1.3 SOP Specific Conformance Statement - Send Storage Commitment Request.....	37
3.3.2.2 Real World Activity - Send Storage Commitment Response	38
3.3.2.2.1 Associated Real World Activity - Send Storage Commitment Response ..	38
3.3.2.2.2 Accepted Presentation Contexts - Send Storage Commitment Response...	38
3.3.2.2.3 SOP Specific Conformance Statement - Send Storage Commitment Response	38
3.3.3 Association Acceptance Policy.....	38
3.3.3.1 Real World Activity - Receive Storage Commitment Request.....	39
3.3.3.1.1 Associated Real-World Activity - Receive Storage Commitment Request	39
3.3.3.1.2 Accepted Presentation Contexts - Receive Storage Commitment Request	39
3.3.3.1.3 SOP Specific Conformance Statement - Receive Storage Commitment Request	39
3.3.3.2 Real World Activity - Receive Storage Commitment Response.....	39
3.3.3.2.1 Associated Real World Activity - Receive Storage Commitment Response.....	39
3.3.3.2.2 Accepted Presentation Contexts - Receive Storage Commitment Response.....	40
3.3.3.2.3 SOP Specific Conformance Statement - Receive Storage Commitment Response	40
3.4 Query/Retrieve AE Specification	40
3.4.1 Association Establishment Policies	41
3.4.1.1 General	41
3.4.1.2 Number of Associations	41
3.4.1.3 Asynchronous Nature	41
3.4.1.4 Implementation Identifying Information.....	41
3.4.2 Association Initiation Policy.....	41
3.4.2.1 Real World Activity – Find SCU	41
3.4.2.1.1 Associated Real-World Activity – Find SCU	41
3.4.2.1.2 Proposed Presentation Contexts – Find SCU.....	42

3.4.2.1.3 SOP Specific Conformance Statement - Find SCU	42
3.4.2.2 Real World Activity - Move SCU	44
3.4.2.2.1 Associated Real-World Activity - Move SCU.....	44
3.4.2.2.2 Proposed Presentation Contexts - Move SCU	44
3.4.2.2.3 SOP Specific Conformance Statement - Move SCU	44
3.4.3 Association Acceptance Policy.....	45
3.4.3.1 Real World Activity - Find SCP.....	46
3.4.3.1.1 Associated Real-World Activity - Find SCP	46
3.4.3.1.2 Proposed Presentation Contexts - Find SCP	46
3.4.3.1.3 SOP Specific Conformance Statement - Find SCP.....	47
3.4.3.2 Real World Activity - Get SCP	54
3.4.3.2.1 Associated Real-World Activity - Get SCP	54
3.4.3.2.2 Proposed Presentation Contexts - Get SCP	54
3.4.3.2.3 SOP Specific Conformance Statement - Get SCP	55
3.4.3.3 Real World Activity - Move SCP.....	56
3.4.3.3.1 Associated Real-World Activity - Move SCP	56
3.4.3.3.2 Proposed Presentation Contexts - Move SCP	57
3.4.3.3.3 SOP Specific Conformance Statement - Move SCP.....	57
3.5 Print AE Specification	58
3.5.1 Association Establishment Policies	59
3.5.1.1 General	59
3.5.1.2 Number of Associations	59
3.5.1.3 Asynchronous Nature	59
3.5.1.4 Implementation Identifying Information.....	59
3.5.2 Association Initiation Policy.....	60
3.5.2.1 Real-World Activity	60
3.5.2.1.1 Associated Real-World Activity	60
3.5.2.1.2 Proposed Presentation Contexts.....	60
3.5.2.1.3 SOP Specific Conformance Statement	62
3.6 Modality Worklist AE Specification	79
3.6.1 Association Establishment Policies	80
3.6.1.1 General	80
3.6.1.2 Number of Associations	80
3.6.1.3 Asynchronous Nature	80
3.6.1.4 Implementation Identifying Information.....	80
3.6.2 Association Initiation Policy.....	80

3.6.2.1 Real World Activity	80
3.6.2.1.1 Associated Real-World Activity	80
3.6.2.1.2 Proposed Presentation Contexts.....	81
3.6.2.1.3 SOP Specific Conformance Statement	81
3.7 Modality Performed Procedure Step AE Specification	86
3.7.1 Association Establishment Policies	86
3.7.1.1 General	86
3.7.1.2 Number of Associations	86
3.7.1.3 Asynchronous Nature	86
3.7.1.4 Implementation Identifying Information.....	86
3.7.2 Association Initiation Policy.....	86
3.7.2.1 Real World Activity	87
3.7.2.1.1 Associated Real-World Activity	87
3.7.2.1.2 Proposed Presentation Contexts.....	87
3.7.2.1.3 SOP Specific Conformance Statement	87
4 Communication Profiles	92
4.1 Supported Communication Stacks.....	92
4.1.1 OSI Stack	92
4.1.2 TCP/IP Stack.....	92
4.1.2.1 API.....	92
4.1.2.2 Physical Media Support.....	92
5 Extensions/Specializations/Privatizations	93
5.1 Standard Extended/Specialized/Private SOPs	93
5.1.1 Standard Extensions of all SOP Classes	93
5.1.1.1 Image Type.....	94
5.1.2 Image Type - value 4	95
5.1.3 Private Elements for Storage SOP Classes	95
5.1.3.1 Registry of DICOM Data Elements	95
5.1.3.2 All UROSKOP Access Supported Image SOP Classes	96
5.1.3.2.1 CSA Image Header Module	97
5.1.3.2.2 CSA Series Header Module	97
5.1.3.2.3 MEDCOM Header Module.....	98
5.1.3.2.4 MEDCOM OOG Module	100
5.1.3.2.5 Edge Enhancement Module	100
6 Media Storage.....	102

6.1 Introduction.....	102
6.1.1 Purpose.....	102
6.1.2 Scope.....	102
6.1.3 Definitions, Abbreviations.....	102
6.1.3.1 Definitions	102
6.1.3.2 Abbreviations	102
6.1.4 References.....	103
6.1.5 Connectivity and Interoperability	103
6.2 Implementation Model.....	104
6.2.1 Application Data Flow Model	104
6.2.2 Functional Definitions of AEs	104
6.2.3 Sequencing of Real World Activities	105
6.2.4 File Meta Information Options	105
6.3 AE Specification	105
6.3.1 DICOM Archive Specification	105
6.3.1.1 File Meta Information for the Application Entity	105
6.3.1.2 Real-World Activities for this Application Entity.....	106
6.3.1.2.1 Real-World Activity: Browse Directory Information.....	106
6.3.1.2.2 Real-World Activity: Import into Local Storage	106
6.3.1.2.3 Real-World Activity: Export to local Archive Media.....	106
6.3.1.3 Application profiles	107
6.3.1.3.1 DICOMDIR Keys	107
6.3.1.3.2 STD-GEN-CD.....	110
6.3.1.3.3 STD-CTMR-xxxx	112
6.3.1.3.4 STD-XABC-CD	113
6.3.1.3.5 STD-XA1K-CD	114
6.3.1.3.6 STD-US-zz-yF-xxx(Single and multiframe)	114
6.4 Augmented and Private Profiles	115
6.4.1 Augmented Application Profiles	115
6.4.1.1 AUG-GEN-CD, AUG-CTMR-xxxx	115
6.4.2 Private Application Profiles.....	118
6.4.2.1 PRI-GEN-CD, PRI-CTMR-xxxx	118
6.5 Extensions, Specializations and Privatizations of SOP Classes and Transfer Syntaxes	120
7 Configuration.....	121
7.1 AE Title / Presentation Address Mapping	121
7.1.1 Storage, Storage Commitment and Query Retrieve.....	121

7.1.2 Print.....	122
7.1.3 Modality Worklist.....	122
7.2 Default Parameters.....	122
8 APPENDIX	124
8.1 Attributes for the generated XRF SOP	124

List of Figures

Figure 1: Application Data Flow Diagram Verification SCU	16
Figure 2: Application Data Flow Diagram C-STORE SCU	17
Figure 3: Application Data Flow Diagram C-STORE SCP	17
Figure 4: Application Data Flow Diagram Storage Commitment SCU.....	18
Figure 5: Application Data Flow Diagram Storage Commitment SCP	18
Figure 6: Application Data Flow Diagram QUERY/RETRIEVE SCU.....	20
Figure 7: Application Data Flow Diagram QUERY/RETRIEVE SCP	20
Figure 8: Application Data Flow Diagram PRINT SCU	21
Figure 9: Application Data Flow Diagram MODALITY WORKLIST SCU.....	22
Figure 10: Application Data Flow Diagram MODALITY PERFORMED PROCEDURE STEP SCU.....	23
Figure 11: Connectivity.....	104

List of Tables

Table 1: Siemens UROSOP Access DICOM Products.....	14
Table 2: Initiation presentation context verification	25
Table 3: Standard SOP Classes as an Storage SCU and an SCP	26
Table 4: Initiation presentation context Storage	28
Table 5: Acceptable presentation Contexts Storage.....	32
Table 6: Standard SOP Classes as Storage Commitment Push Model	36
Table 7: Initiation presentation context Storage Commitment Request	37
Table 8: Initiation presentation context Storage Commitment Response	38
Table 9: Presentation context accepted for Storage Commitment	39
Table 10: Presentation context accepted for Storage Commitment	40
Table 11: SOP Classes as an SCU / SCP	40
Table 12: Proposed presentation Context Find SCU	42
Table 13: query attributes	42
Table 14: Proposed Presentation Context Move SCU	44
Table 15: C_MOVE Response Status	45
Table 16: Proposed Presentation Contexts - Find SCP	46
Table 17: Patient level attributes, Patient Root Information Model	47
Table 18: Study level attributes, Patient Root Information Model	47
Table 19: Series level attributes, Patient Root Information Model.....	48
Table 20: Series level attributes, Patient Root Information Model.....	48
Table 21: Image level attributes, Patient Root Information Model.....	49
Table 22: Study level attributes, Study Root Information Model.....	49
Table 23: Series level attributes, Study Root Information Model.....	51
Table 24 Image level attributes, Study Root Information Model	51
Table 25: Patient instance level, Patient Study Only Information Model.....	52
Table 26: Study level attributes, Patient Study Only Information Model.....	52
Table 27: C-FIND return status.....	54
Table 28: Proposed Presentation Contexts Get SCP return status	55
Table 29: C-GET return status	56
Table 30: Proposed Presentation Contexts - Move SCP	57
Table 31: C-MOVE return status	58
Table 32: Basic Gray Scale Print Management Meta SOP-Classes.....	58
Table 33: Basic Color Print Management Meta SOP-Classes	59

Table 34: Proposed Presentation Contexts.....	60
Table 35: Used Basic Film Session N-CREATE attributes	63
Table 36: Attributes of the N-DELETE_RQ on the Basic Film Session SOP Class	63
Table 37: Basic Film Session SOP status.....	63
Table 38: Used Film Box N-CREATE_RQ attributes	64
Table 39: Attributes of the N-DELETE_RQ on the Basic Film Session SOP Class	65
Table 40: Basic Film Box SOP Status	65
Table 41: Used Basic Grayscale Image Box N_SET attributes	65
Table 42: Basic Grayscale Image Box SOP status.....	66
Table 43: Used Basic Color Image Box N-SET attributes.....	66
Table 44: Basic Color Image Box SOP status.....	67
Table 45: Attributes of the N-CREATE_RQ on the Presentation LUT SOP Class.....	67
Table 46: Attributes of the N-CREATE_RSP on the Presentation LUT SOP Class	67
Table 47: Attributes of the N-CREATE_RSP on the Presentation LUT SOP Class	68
Table 48: Status codes for the N-CREATE_RSP and N-DELETE_RSP	68
Table 49: Used Printer N-EVENT report.....	69
Table 50: Mandatory Printer N-Get Attributes	69
Table 51: Used Print Job N-EVENT report	69
Table 52: Printer Status Info within Printer SOP Class/Execution Status Info within Print Job SOP	70
Table 53: Printer Status Infos: Additional Agfa printer status infos.....	76
Table 54: Printer Status Infos: Additional Kodak infos for Pacs Link (formerly Imation cameras)	76
Table 55: Printer Status Infos: Additional Kodak infos for Kodak 190.....	77
Table 56: Printer Status Infos: Additional Kodak infos for 2180/1120	77
Table 57: Printer Status Infos: Additional Codonics infos	77
Table 58: Additional DICOM Execution Status Infos	79
Table 59: SOP Classes as an SCU	79
Table 60: Proposed presentation contexts.....	81
Table 61: Search Key Attributes in a broad worklist query	81
Table 62: Search Key Attributes in a patient based worklist query	82
Table 63: Modality Worklist C_FIND_RSP Return Key Attributes	83
Table 64: C-FIND Response Status	85
Table 65: SOP Classes as an SCU	86
Table 66: Proposed presentation contexts.....	87
Table 67: Performed Procedure Step N-CREATE Attributes.....	88

Table 68: N-SET Response Status	90
Table 69: Performed Procedure Step N-SET Attributes	90
Table 70: N-SET Response Status	91
Table 71: Standard Extensions of all SOP Classes	93
Table 72: Private Elements	95
Table 73: CSA Image IOD Modules.....	96
Table 74: CSA Image Header Module.....	97
Table 75: CSA Series Header Module	97
Table 76: MEDCOM Header Module.....	98
Table 77: MEDCOM History Information.....	100
Table 78: MEDCOM OOG Module	100
Table 79: MEDCOM OOG Module.....	100
Table 80: Siemens UROSKOP Access DICOM Products.....	102
Table 81: Application Profiles Supported.....	105
Table 82: Tags in DICOMDIR file	107
Table 83: SOP Classes conforming STD-GEN-CD.....	110
Table 84: SOP Classes STD-CTMR-xxxx.....	112
Table 85: STD-XABC-CD Supported SOP Classes	113
Table 86: STD-XA1K-CD Supported SOP Classes	114
Table 87: STD-US-ID-yF-xxx Supported SOP Classes	114
Table 88: Augmented Application profiles, Activities, and Roles for DICOM Archive.....	115
Table 89: AUG-GEN-CD, AUG-CTMR-xxxx Supported SOP Classes	115
Table 90: Private Application profiles, Activities, and Roles for DICOM Archive.....	118
Table 91: PRI-GEN-CD, PRI-CTMR-xxxx Supported SOP Classes	119
Table 92: DICOMDIR keys for CsaNonImage.....	119

1 Introduction

UROSKOP ACCESS is a xray-system for fluoroscopic and radiographic use. The DICOM features are realized within the imaging software.

1.1 Purpose of the Document

This UROSKOP Access Conformance Statement is written according to part PS 3.2 of [1].

The applications described in this conformance statement are realized within UROSKOP Access and with imaging software VA14. The UROSKOP Access DICOM network implementation acts as SCU and SCP for the DICOM Query/Retrieve service and the XRF storage Service, as SCU for the defined storage service IOD's and as SCU for the DICOM Print, Modality Worklist and MPPS service. This DICOM Conformance Statement refers to the UROSKOP ACCESS product using imaging software VA14.

Area of validity of the document

This DICOM Conformance Statement refers to the UROSKOP Access product using imaging software VA14.

1.2 Scope

This DICOM Conformance Statement refers to UROSKOP Access product using software VA14. The following table relates software names to the UROSKOP Access product.

Table 1: Siemens UROSKOP Access DICOM Products

Software Name	Siemens Product
VX43A_UROSKOP_ACCESS_VA14	UROSKOP Access

1.3 Definitions and Abbreviations

1.3.1 Definitions

DICOM	Digital Imaging and Communications in Medicine
DIMSE	DICOM Message Service Element
DIMSE-C	DICOM Message Service Element with Composite information objects
DIMSE-N	DICOM Message Service Element with Normalized information objects.

1.3.2 Abbreviation

ACR	American College of Radiology
AE	DICOM Application Entity
ASCII	American Standard Code for Information Interchange
FSC	File Set Creator
FSR	File Set Reader
FSU	File Set Updater
HIS	Hospital Information System
IOD	DICOM Information Object Definition
ISO	International Standard Organization
NEMA	National Electrical Manufacturers Association
O	Optional Key Attribute
R	Required Key Attribute
RIS	Radiology Information System
PDU	DICOM Protocol Data Unit
SCU	DICOM Service Class User (DICOM client)
SCP	DICOM Service Class Provider (DICOM server)
SOP	DICOM Service-Object Pair
U	Unique Key Attribute

1.4 References

- [1] Digital Imaging and Communications in Medicine (DICOM) 3.0, NEMA PS 3.1-3.15 2001

1.5 Connectivity and Interoperability

The implementation of the UROSKOP Access DICOM interface has been carefully tested to assure compliance with this Conformance Statement. But the Conformance Statement and the DICOM standard does not necessarily guarantee interoperability of Siemens modalities and modalities of other vendors.

The user must compare the relevant Conformance Statements and if a successful interconnection seems possible, the user is responsible to specify an appropriate test suite and to validate the required interoperability. A network environment may need additional functions outside of the scope of DICOM.

2 Implementation Models

2.1 Verification


The verification service class defines an application-level class of service which allows the operator to verify the ability of an application on a remote node to respond to DICOM messages. The DICOM Service Tool application supports the verification service to act as SCU.

The other direction - responding to Verification requests from remote applications - is handled by the Storage SCP application.

2.1.1 Application Data Flow Diagram

The UROSKOP Access DICOM network implementation is a Windows NT application and acts as SCU for the verification service.

Figure 1: Application Data Flow Diagram Verification SCU

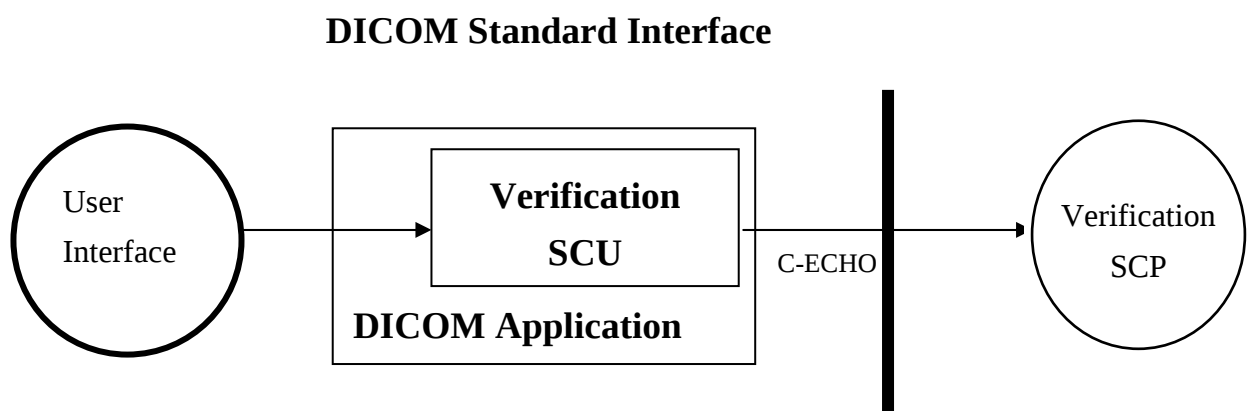

2.1.2 Functional Definitions of Application Entities

The DICOM Service Tool application opens an association to an application on the remote node and sends a verification message to verify that the remote application can respond to DICOM messages.

2.1.3 Sequencing of real World Activities

During configuration a verification button allows to check the current configuration.

2.2 Storage

The UROSKOP Access DICOM Application Entity originates associations for Storage of DICOM Composite Information Objects in Remote Application Entities.

2.2.1 Application Data Flow Diagram

The UROSKOP Access DICOM network implementation is a Windows NT application and acts as SCU and SCP for the storage network service and as SCP for the DICOM verification service.

Figure 2: Application Data Flow Diagram C-STORE SCU


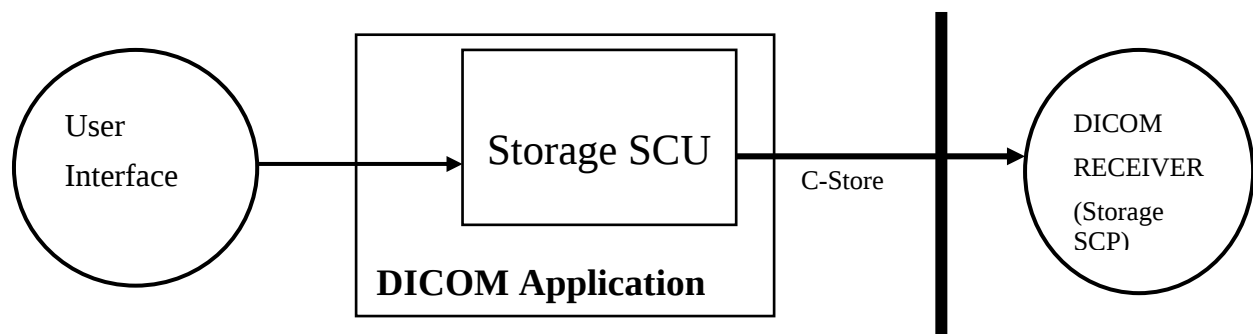


Figure 3: Application Data Flow Diagram C-STORE SCP

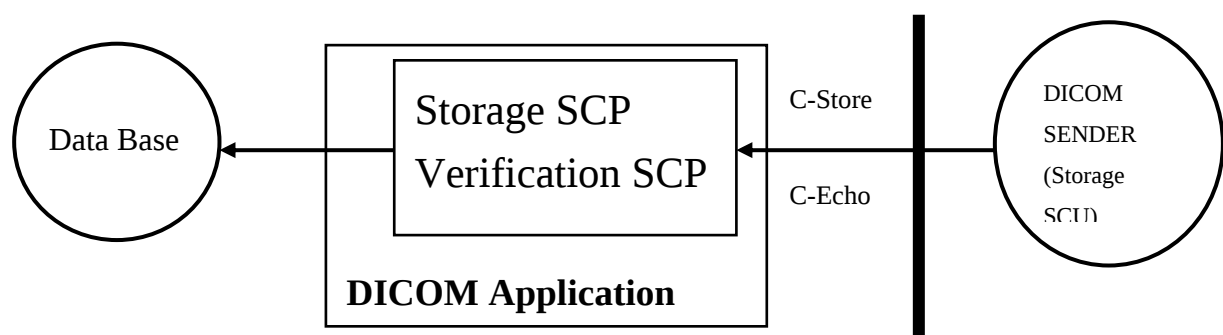

2.2.2 Functional Definitions of Application Entities

All SCP components of the UROSKOP Access DICOM application are operating as background daemon processes. They are automatically started at system start-up.

2.2.3 Sequencing of real World Activities

not applicable.

2.3 Storage Commitment Push Model

The Storage Commitment service class defines an application-level class of service, which facilitates the commitment to storage. It performs an additional task of commitment of composite objects apart from the network-based storage of images as defined by the Storage Service class.

2.3.1 Application Data Flow Diagram

Figure 4: Application Data Flow Diagram Storage Commitment SCU


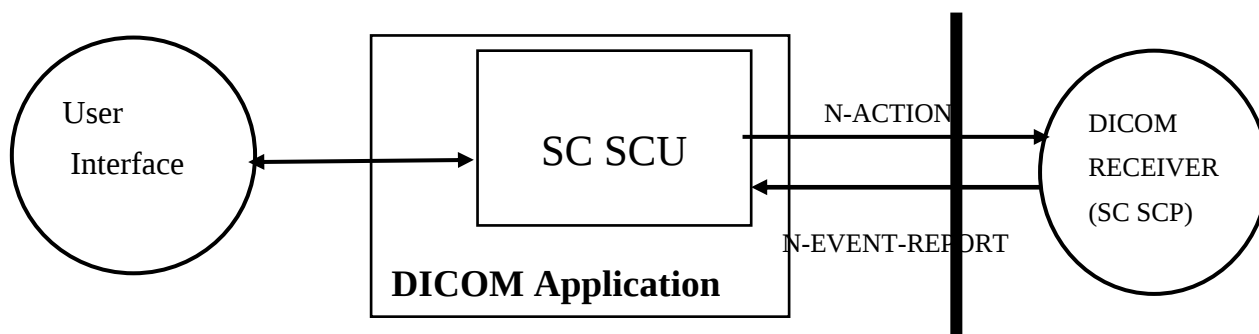


Figure 5: Application Data Flow Diagram Storage Commitment SCP

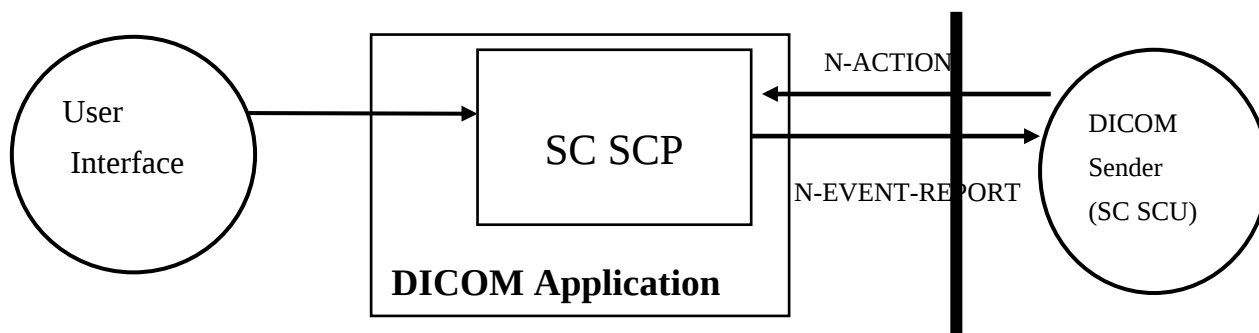

2.3.2 Functional Definitions of Application Entities

The Storage Commitment Push Model SCU (SC-SCU) uses the Storage Commitment Service Class to get commitment for one or more composite objects from the Storage Commitment Push Model SCP (SC-SCP). SC-SCU issues an N-ACTION request to SC-SCP containing a list of references to composite objects, requesting that the SC-SCP takes responsibility for storage commitment of the composite objects. If the SC-SCP has determined that all the composite objects exist and it has successfully completed storage commitment for the set of composite objects, it issues an N-EVENT-REPORT request with status successful and the list of stored images. SC-SCU now knows that the composite objects have been committed by SC-SCP and can delete its copies of composite objects. The information from the N-EVENT-REPORT is propagated back to the user interface.

If committed storage cannot be done for some reason for any of the list of composite objects the SC_SCP issues an N-EVENT-REPORT request with a status of completed-failures exist and would include both the successful and failed list. The UROSKOP Access SCU can be configured to receive the N-EVENT-REPORT on the same association on which N_ACTION was sent or on a different association. The UROSKOP Access SCP will send the N-EVENT-REPORT on the same association on which the N_ACTION was received if this association is still open. Otherwise, the N-EVENT-REPORT will be sent on a new association. It is configurable if UROSKOP Access is running on an archive system. If not, then UROSKOP Access SC SCP will verify if the composite objects are stored in the local database and answer accordingly. Otherwise there the archive system is asked to confirm the commitment, and the answer returned by the archive will be returned to the SC SCU.

2.3.3 Sequencing of real World Activities

not applicable.

2.4 Query/Retrieve

The query/retrieve service class defines an application-level class of services which facilitates the management of images and patient data against the well defined information model of DICOM and allows a DICOM AE to retrieve images from a remote DICOM node or to request a remote DICOM AE to initiate a transfer of images to another DICOM AE. The DICOM query/retrieve application supports the query/retrieve services to act as SCU and SCP.

2.4.1 Application Data Flow Diagram

The UROSKOP Access DICOM network implementation is a Windows NT application and acts as SCU and SCP for the query/retrieve network service.

Figure 6: Application Data Flow Diagram QUERY/RETRIEVE SCU


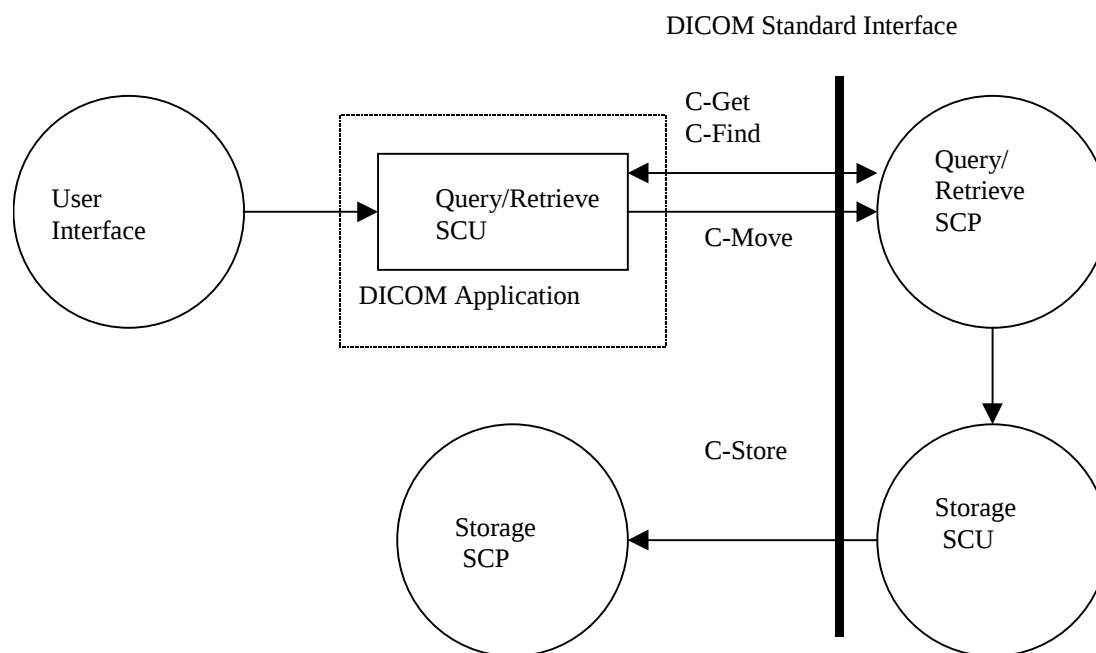


Figure 7: Application Data Flow Diagram QUERY/RETRIEVE SCP

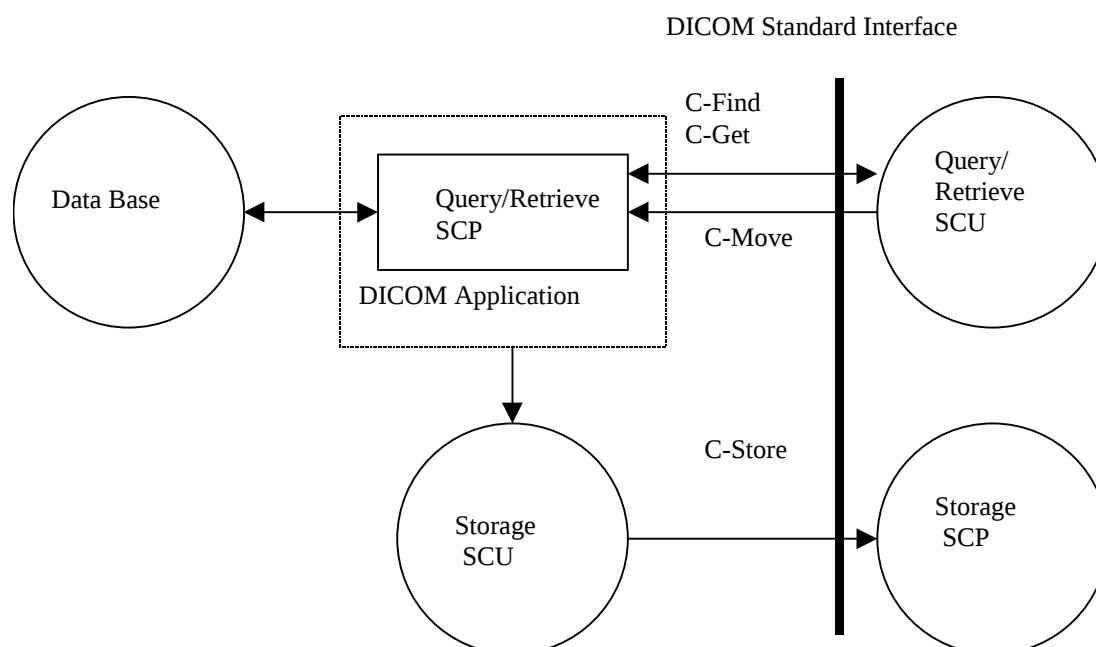

2.4.2 Functional Definitions of Application Entities

The query/retrieve SCU requests the query/retrieve SCP to perform a match to the keys specified in the request and a C-GET or a C-MOVE DIMSE service initiates a C-STORE sub operation to transfer an image from a Storage SCU to a Storage SCP.

The query/retrieve SCP responds to C-FIND DIMSE services and a C-GET or a C-MOVE involves the UROSKOP Access DICOM query/retrieve SCP application to initiate a C-STORE sub operation to a Storage SCP.

All components of the UROSKOP Access DICOM query/retrieve SCP application are operating as back-ground daemon processes. They are automatically started at system startup.

2.4.3 Sequencing of real World Activities

not applicable.


2.5 Print

The Print Management Service Classes define an application-level class of services which facilitate the printing of images on a hardcopy medium. The print management SCU and print management SCP are peer DICOM print management application entities. The DICOM print application supports the print management DIMSE services to act as SCU.

2.5.1 Application Data Flow Diagram

The UROSKOP Access DICOM network implementation is a Windows NT application and acts as SCU for the print management network service.

Figure 8: Application Data Flow Diagram PRINT SCU

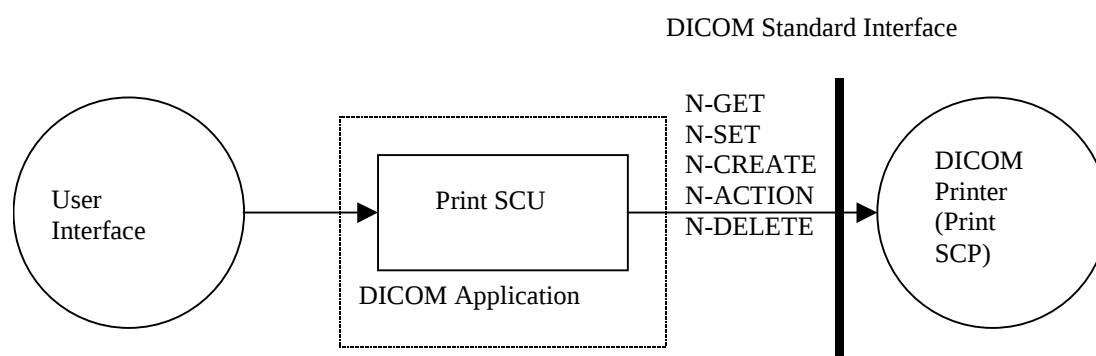

2.5.2 Functional Definitions of Application Entities

The user invokes a print job and the SCU uses the SOP classes of a film session, a film box

and image boxes for acquiring all the information which is required for a film session. The N-ACTION is used to print the film session. The DIMSE services of the Printer SOP Class and the Print Job SOP Class allow the SCU to control the print jobs and printer status information of the SCP.

2.5.3 Sequencing of real World Activities

not applicable.


2.6 Modality Worklist

The modality worklist service class defines an application-level class of service, which facilitates the transfer of worklists from the information system to the imaging modality. The worklist is queried by the AE and supplies the SCU with the scheduled tasks, which have to be performed on the modality. The DICOM worklist application supports the worklist service to act as SCU.

2.6.1 Application Data Flow Diagram

The UROSKOP Access DICOM network implementation is a Windows NT application and acts as SCU for the worklist network service.

Figure 9: Application Data Flow Diagram MODALITY WORKLIST SCU

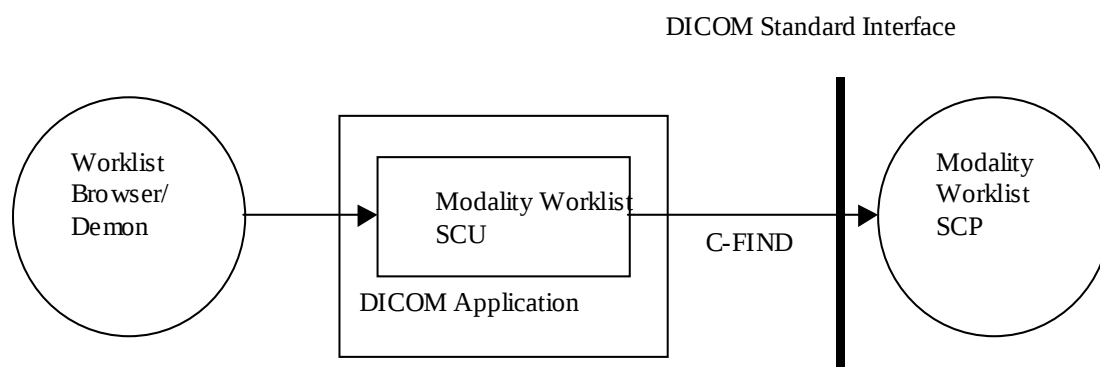

Note: It is configurable to get the worklist updates either automatically (in a configurable time interval) or manually (initiated by the user). There are two kinds of query the user can do: broad worklist query (all jobs for the own modality or own application entity) and patient based worklist query (where more search keys can be given, including Patient Name and Patient ID).

2.6.2 Functional Definitions of Application Entities

The modality worklist SCU requests the worklist SCP to perform a match to the keys

The modality worklist SCP responds to the C-FIND query and sends scheduled imaging service requests and patient demographic information from the information system to the modality.

not applicable.

The Modality Performed Procedure Step service class defines an application-level class of service, which facilitates the transfer of billing and radiation dose information from the imaging modality to the information system. The performed procedure step is sent by the AE and supplies the SCP with the performed tasks on the modality. The DICOM performed procedure step. Application supports the performed procedure step service to act as SCU.

The UROSKOP Access DICOM network implementation is a Windows NT application and acts as SCU for the performed procedure step network service.

The diagram illustrates the DICOM Standard Interface. It features three main components: a circle on the left labeled 'PPS Browser Daemon', a central rectangle labeled 'DICOM Application' containing a smaller rectangle labeled 'Performed Procedure Step SCU', and a circle on the right labeled 'Performed Proc. Step SCP'. A thick vertical line represents the interface. Arrows show the flow of data: one arrow points from the 'DICOM Application' to the 'PPS Browser Daemon', and two arrows (labeled 'N-CREATE' and 'N-Set') point from the 'Performed Procedure Step SCU' to the 'Performed Proc. Step SCP' across the interface line.

The modality performed procedure step SCU informs the performed procedure step SCP about the performed procedure steps examined at the modality using the N-CREATE and N-SET DIMSE services.

The modality performed procedure step SCP responds to the N-CREATE and N-SET and

confirms that it received the information from the modality.

2.7.3 Sequencing of real World Activities

not applicable.

3 AE Specifications

3.1 Verification AE Specification

3.1.1 Association Initiation by Real-World Activity

The UROSOP Access Service Tool application attempts to initiate a new association for •DIMSE C-ECHO service operations.

3.1.1.1 Real-World Activity - Verification SCU

3.1.1.1.1 Associated Real-World Activity - Verification SCU

The associated Real-World activity is a C-ECHO request initiated by the DICOM Service Tool application. If the process successfully establishes an association to a remote Application Entity, it will send the C-ECHO-Request via the open association to verify that the remote Application Entity is responding to DICOM messages.

3.1.1.1.2 Proposed Presentation Contexts - Verification SCU

The UROSOP Access application will propose Presentation Contexts as shown in the following table:

Table 2: Initiation presentation context verification

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Verification 1.2.840.10008.1.1		Implicit VR Little Endian	1.2.840.10008.1.2.	SCU	None
			1.2.840.10008.1.2.2		
		Explicit VR Big Endian	1.2.840.10008.1.2.1		
		Explicit VR Little Endian			

3.1.1.1.3 SOP Specific Conformance Statement - Verification SCU

The Application conforms to the definition of a Verification SCU in accordance to the DICOM Standard.

3.1.2 Association Acceptance Policy

The Verification SCP is part of the Storage SCP - see section 3.2.3

3.2 Storage AE Specification

UROSKOP Access provides Standard Conformance to the following DICOM V3.0 SOP Classes as both an SCU and SCP:

Table 3: Standard SOP Classes as an Storage SCU and an SCP

SOP Class Name	SOP Class UID
CR Image Storage	1.2.840.10008.5.1.4.1.1.1
CT Image Storage	1.2.840.10008.5.1.4.1.1.2
US Multi-Frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1
MR Image Storage	1.2.840.10008.5.1.4.1.1.4
US Image Storage	1.2.840.10008.5.1.4.1.1.6.1
SC (Secondary Capture) Image Storage	1.2.840.10008.5.1.4.1.1.7
XA (X-Ray Angiographic) Image Storage	1.2.840.10008.5.1.4.1.1.12.1
XA Bi-Plane (X-Ray Angiographic Bi-Plane) Image Storage (Retired)	1.2.840.10008.5.1.4.1.1.12.3
XRF (X-Ray Radiofluoroscopic) Image Storage	1.2.840.10008.5.1.4.1.1.12.2
NM Image Storage	1.2.840.10008.5.1.4.1.1.20

Note:

All color images are handled correctly in the sense of DICOM, but there is a limitation in the viewer. Only a color depth of 8 bit is supported for color images.

3.2.1 Association Establishment Policies

3.2.1.1 General

The configuration of the UROSKOP Access DICOM application defines the Application Entity Titles, the port numbers and of course the host name and net address.

3.2.1.2 Number of Associations

The UROSKOP Access DICOM application initiates several associations at a time, one for each transfer request being processed.

3.2.1.3 Asynchronous Nature

The UROSKOP Access DICOM software does not support asynchronous communication (multiple out-standing transactions over a single association).

3.2.1.4 Implementation Identifying Information

The Implementation Class UID UROSKOP Access DICOM software provides a single of

1.3.12.2.1107.5.12.3

and an Implementation Version Name of

- “SIEMENS_SWFVB10E”.

3.2.1.5 Maximum PDU Size offered

- The maximum PDU size offered by UROSKOP Access is configurable with default set to 28672 Bytes

3.2.1.6 Dicom Application Context

Dicom Application context name: 1.2.840.10008.3.1.1.1

3.2.2 Association Initiation by Real-World Activity

The UROSKOP Access DICOM application attempts to initiate a new association for

- DIMSE C-STORE service operations.

3.2.2.1 Real-World Activity - Storage SCU

3.2.2.1.1 Associated Real-World Activity - Storage SCU

The associated Real-World activities are:

- a user wants to send one or more composite objects to a remote node.
- The local C-MOVE SCP initiates C-STORE sub-operations as a reaction to a C-MOVE-RQ coming from a remote node.

For both cases, if the process successfully establishes an association to a remote Application Entity, it will transfer each image one after another via the open association. If the C-STORE Response from the remote Application contains an error status the association is aborted or released.

3.2.2.1.2 Proposed Presentation Contexts - Storage SCU

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table. Not all the listed transfer syntaxes will be proposed, please see the discussion after the table for the actual proposed transfer syntaxes.

Table 4: Initiation presentation context Storage

Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
CR Image Storage SOP class	1.2.840.10008.5.1.4.1.1.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
CT Image Storage SOP class	1.2.840.10008.5.1.4.1.1.2	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
MR Image Storage SOP class	1.2.840.10008.5.1.4.1.1.4	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
NM Image Storage SOP class	1.2.840.10008.5.1.4.1.1.20	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
SC Image Storage SOP class	1.2.840.10008.5.1.4.1.1.7	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
XA Image Storage SOP class	1.2.840.10008.5.1.4.1.1.12.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None

XA Bi-Plane Image Storage SOP class (Retired)	1.2.840.10008.5.1.4.1.1.12.3	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
XRF Image Storage SOP class	1.2.840.10008.5.1.4.1.1.12.2	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
US MF Image Storage SOP class	1.2.840.10008.5.1.4.1.1.13.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None
US Image Storage SOP class	1.2.840.10008.5.1.4.1.1.16.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCU	None

Note:

Uncompressed transfer syntaxes are proposed together in a single presentation context for each abstract syntax.

Not all the listed transfer syntaxes will be proposed all the time. For some abstract syntaxes only a list of uncompressed (UC) transfer syntaxes (one or more) will be proposed, for other abstract syntaxes also JPEG Lossless (LL) syntax will be proposed and/or a list of JPEG Lossy (LY) transfer syntaxes. The contents of this lists is configurable, e.g. UC could be configured to contain only Implicit Little Endian for instance.

Depending on the real world activity initiating the C-STORE, we have the following behaviors:

- if the C-STORE is initiated by a user, a configuration parameter called Quality Factor(Q) will be used to decide which transfer syntax lists will be proposed. Q can take values between 0 and 100. If Q=0, only UC will be proposed. If Q = 100, UC and LL will be proposed. Else UN and LY will be proposed.
- if the C-STORE is initiated by the C-MOVE SCP, there is another configuration parameter called Compression Types Supported (CTS) which will be used to decide what transfer syntaxes are proposed. CTS can take values between 0 and 2. If CTS=0, only UC will be proposed. If CTS=1, UC and LY will be proposed. Else UC and LL will be proposed.

It is not possible to send an image that is locally stored in a Lossy compressed format using an uncompressed or lossless transfer syntax.

3.2.2.1.3 SOP Specific Conformance Statement - Storage SCU

The DICOM images created by UROSKOP Access DICOM application conform to the DICOM IOD definitions (Standard extended IODs). But they will contain additional private elements which have to be discarded by a DICOM system when modifying the image.

When performing a save for a new operation, all non-defined UROSKOP Access attributes will be deleted. For association and DIMSE level time outs refer sec. 6.2

3.2.2.1.3.1 Image Pixel Attribute Description for Grayscale Images

The UROSKOP Access DICOM application supports the MONOCHROME2 and MONOCHROME1 photometric interpretation pixel format and graphic overlay with unsigned integer and 8 or 16 bits allocated. Possible values:

Pixel plane

samples per pixel (attribute 0028, 0002) = 1

photometric interpretation (attribute 0028,0004) = "MONOCHROME1"

- photometric interpretation (attribute 0028,0004) = "MONOCHROME2"
- pixel representation (attribute 0028, 0103) = 0
- bits allocated (attribute 0028, 0100) = 8, 16
- bits stored (attribute 0028,0101) = 8, 10, 12
- high bit (attribute 0028,0102) = 7, 9, 11

Overlay plane

- overlay type (attribute 60xx, 0040) = "G"
- overlay bits allocated (attribute 60xx, 0100) = 16
- overlay bit position (attribute 60xx, 0102) = 12, 13, 14, 15

Overlay plane

- overlay type (attribute 60xx, 0040) = "G"
- overlay bits allocated (attribute 60xx, 0100) = 1
- overlay bit position (attribute 60xx, 0102) = 0
- overlay data (attribute 60xx, 3000) = supported.

The UROSKOP Access DICOM application sends also the MONOCHROME1 and MONOCHROME2 photometric interpretation pixel format with binary 2's complement integer and 16 bits allocated.

Possible values:

Pixel plane

- samples per pixel (attribute 0028, 0002) = 1

- photometric interpretation (attribute 0028,0004) = "MONOCHROME1"
- photometric interpretation (attribute 0028,0004) = "MONOCHROME2"
- pixel representation (attribute 0028, 0103) = 1
- bits allocated (attribute 0028, 0100) = 16
- bits stored (attribute 0028,0101) = 16
- high bit (attribute 0028,0102) = 15

Overlay plane

- overlay type (attribute 60xx, 0040) = "G"
- overlay bits allocated (attribute 60xx, 0100) = 1
- overlay bit position (attribute 60xx, 0102) = 0
- overlay data (attribute 60xx, 3000) = supported.

3.2.3 Association Acceptance Policy

The UROSKOP Access DICOM application attempts to accept a new association for

- DIMSE C-ECHO
- DIMSE C-STORE

service operations.

3.2.3.1 Real-World Activity - Storage SCP

3.2.3.1.1 Associated Real-World Activity - Storage SCP

The daemon receiving process will accept an association and will receive images(see below) transmitted on that association and will store the images on disk in the own database.

3.2.3.1.2 Accepted Presentation Contexts - Storage SCP

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 5: Acceptable presentation Contexts Storage

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
CR Image Service class Storage	1.2.840.10008.5.1.4.1.1.1	Implicit VR Little Endian	12.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline	1.2.840.10008.1.2.4.50		
		JPEG Extended (2 & 4)	1.2.840.10008.1.2.4.51		
		JPEG Lossless Non-hierarchical	1.2.840.10008.1.2.4.70		
CT Image Storage Service class	1.2.840.10008.5.1.4.1.1.2	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline	1.2.840.10008.1.2.4.50		
		JPEG Extended (2 & 4)	1.2.840.10008.1.2.4.51		
		JPEG Lossless Non-hierarchical	1.2.840.10008.1.2.4.70		
MR Image Service class Storage	1.2.840.10008.5.1.4.1.1.4.	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline	1.2.840.10008.1.2.4.50		
		JPEG Extended (2 & 4)	1.2.840.10008.1.2.4.51		
		JPEG Lossless Non-hierarchical	1.2.840.10008.1.2.4.70		
NM Image Storage Service class	1.2.840.10008.5.1.4.1.1.20	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline	1.2.840.10008.1.2.4.50		
		JPEG Extended (2 & 4)	1.2.840.10008.1.2.4.51		
		JPEG Lossless Non-hierarchical	1.2.840.10008.1.2.4.70		
SC Image Storage Service class	1.2.840.10008.5.1.4.1.1.7	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline	1.2.840.10008.1.2.4.50		
		JPEG Extended (2 & 4)	1.2.840.10008.1.2.4.51		
		JPEG Lossless Non-hierarchical	1.2.840.10008.1.2.4.70		
XA Image Storage Service class	1.2.840.10008.5.1.4.1.1.12.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline	1.2.840.10008.1.2.4.50		
		JPEG Extended (2 & 4)	1.2.840.10008.1.2.4.51		
		JPEG Lossless Non-hierarchical	1.2.840.10008.1.2.4.70		

XA Bi-Plane Image Service class Storage (Retired)	1.2.840.10008.5.1.4.1.1.12. 3	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCP	None
XRF Image Service class Storage	1.2.840.10008.5.1.4.1.1.12. 2	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCP	None
US MF Image Service class Storage	1.2.840.10008.5.1.4.1.1.3.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCP	None
US Image Service class Storage	1.2.840.10008.5.1.4.1.1.6.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian JPEG Baseline JPEG Extended (2 & 4) JPEG Lossless Non-hierarchical	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51 1.2.840.10008.1.2.4.70	SCP	None

3.2.3.1.3 SOP Specific Conformance Statement – Storage SCP

The UROSKOP Access DICOM application conforms to the Full Storage Service Class at Level 2 – with the exception that private Sequences (private elements with VR=SQ) are not supported in Explicit VR Transfer syntax and will be ignored. When private Sequences are received in implicit VR then the whole sequence is stored as one binary element of VR=OW.

In the event of a successful C-STORE operation, the image has successfully been written on disk in the Siemens CSA image format.

The UROSKOP Access DICOM receiver returns the status Success upon successful operation otherwise one of the following status codes is returned and the association aborted:

- Refused (A700):

This error status indicates a lack of Resources (e.g. not enough disk space) on the UROSKOP Access modality.

- Error (A900 or C000):

An error occurred while processing the image which makes it impossible to proceed. The image will not be stored and the association aborted.

If an image with the same SOP Instance UID (as that image being received) is already present in the database then the received image will be ignored. So if a remote node sends twice the same image (same SOP Instance UID) then there will still be only one image (the first) in the

database of the DICOM receiver.

The Siemens DICOM receiver can receive all kinds of different image formats. But for Display of such images the following restrictions apply:

3.2.3.1.3.1 Image Pixel Attribute Acceptance Criterion for Grayscale Images

The UROSKOP Access DICOM application accepts the MONOCHROME1 and MONOCHROME2 photometric interpretation pixel format and graphic overlay with unsigned integer and 8 or 16 bits allocated. Accepted values:

Pixel plane

- samples per pixel (attribute 0028, 0002) = 1
- photometric interpretation (attribute 0028,0004) = "MONOCHROME1"
- photometric interpretation (attribute 0028,0004) = "MONOCHROME2"
- pixel representation (attribute 0028, 0103) = 0 (unsigned)
- bits allocated (attribute 0028, 0100) = 8, 16
- bits stored (attribute 0028,0101) = 8, 10, 12(in case of 16 bits allocated 8 bits stored is not allowed)
- high bit (attribute 0028,0102) = bits stored –1
- only aspect ratio 1:1 is allowed

Overlay plane

- overlay type (attribute 60xx, 0040) = "G"
- overlay bits allocated (attribute 60xx, 0100) = 16
- overlay bit position (attribute 60xx, 0102) = 12, 13, 14, 15(only bits above high bit are allowed)

Overlay plane

- overlay type (attribute 60xx, 0040) = "G"
- overlay bits allocated (attribute 60xx, 0100) = 1
- overlay bit position (attribute 60xx, 0102) = 0
- overlay data (attribute 60xx, 3000) = supported.

The UROSKOP Access DICOM application accepts also the MONOCHROME1 and MONOCHROME2 photometric interpretation pixel format with binary 2's complement integer and 16 bits allocated. Accepted values:

Pixel plane

- samples per pixel (attribute 0028, 0002) = 1
- photometric interpretation (attribute 0028,0004) = "MONOCHROME1"
- photometric interpretation (attribute 0028,0004) = "MONOCHROME2"
- pixel representation (attribute 0028, 0103) = 1 (signed)

- bits allocated (attribute 0028, 0100) = 16
- bits stored (attribute 0028,0101) = 16
- high bit (attribute 0028,0102) = 15
- only aspect ratio 1:1 is allowed

Overlay plane

- overlay type (attribute 60xx, 0040) = "G"
- bits allocated (attribute 60xx, 0100) = 1
- bit position (attribute 60xx, 0102) = 0
- overlay data (attribute 60xx, 3000) = supported.

For MOD LUT both the linear LUT (Rescale Slope/Intercept) and the MOD LUT SQ are supported and considered when pixel data is displayed. However the following limitations apply:

- The MOD LUT SQ will be ignored for 8 bit signed pixels
- the pixel format is not changed by applying the MOD LUT SQ, even if the output range of the LUT is larger than the input range.(e.g. 8 bit -> 16 bit)
- If the MOD LUT SQ contains multiple LUTs then only the first one is used.

For VOI LUT also both the linear LUT (Window Center/Width) and the VOI LUT SQ are supported (VOI LUT SQ with 8 or 16 bit LUT data).

The Display application supports only rectangular Shutters in this version. Images with other shutter types will be displayed without Shutter.

3.2.3.1.4 Presentation Context Acceptance Criterion – Storage SCP

The UROSKOP Access DICOM application will accept any number of verification or storage SOP classes that are listed above. In the event that the UROSKOP Access DICOM application runs out of resources, it will reject the association request.

3.2.3.1.5 Transfer Syntax Selection Policies – Storage SCP

The UROSKOP Access DICOM application supports

- the Implicit VR Little Endian, the Explicit VR Little Endian and Explicit VR Big Endian transfer syntaxes
- the JPEG Baseline and JPEG Extended transfer syntaxes (JPEG lossy).
- the JPEG Lossless Non-hierarchical transfer syntax.

Any proposed presentation context which includes one of these transfer syntaxes will be accepted Any proposed presentation context that does not include one of these transfer syntaxes will be rejected.

The order of preference in accepting a Transfer syntax is:

1. JPEG Extended Process 2 And 4.
2. JPEG Lossless Non-hierarchical

3. JPEG Baseline
4. Explicit VR Little Endian
5. Explicit VR Big Endian
6. Implicit VR Little Endian

3.3 Storage Commitment AE Specification

Table 6: Standard SOP Classes as Storage Commitment Push Model

SOP Class Name	SOP Class UID
Storage Commitment Push Model	1.2.840.10008.1.20.1

3.3.1 Association Establishment Policies

3.3.1.1 General

The configuration of UROSOP Access Storage Commitment AE defines the Application Entity Titles, the port numbers and of course the hostname and net address.

3.3.1.2 Number of Associations

UROSOP Access Storage Commitment AE initiates several association at a time, one for each storage commitment request being processed.

3.3.1.3 Asynchronous Nature

UROSOP Access Storage Commitment AE does not support asynchronous communication (multiple out-standing transactions over a single association).

3.3.1.4 Implementation Identifying Information

UROSOP Access Storage Commitment AE provides a single Implementation Class UID of **1.3.12.2.1107.5.12.3**

and an Implementation Version Name of

- “SIEMENS_SWFVB10E”.

3.3.2 Association Initiation Policy

UROSKOP Access Storage Commitment AE initiates an association in two cases: when acting as SCU, in order to send a request for storage commitment and when acting as SCP and the association on which the storage commitment request was received is not open anymore.

3.3.2.1 Real-World Activity - Send Storage Commitment Request

3.3.2.1.1 Associated Real-World Activity - Send Storage Commitment Request

The user has sent (or archived) images to another DICOM node, which is configured as storage commitment SCP. UROSKOP Access will automatically attempt to send a storage commitment request for this images.

3.3.2.1.2 Proposed Presentation Contexts - Send Storage Commitment Request

UROSKOP Access Storage Commitment AE will propose Presentation Contexts as shown in the following table:

Table 7: Initiation presentation context Storage Commitment Request

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Storage Commitment Push Model SOP Class	1.2.840.10008.1.201	Implicit VR Little Endian	12.840.10008.1.2	SCU	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		

3.3.2.1.3 SOP Specific Conformance Statement - Send Storage Commitment Request

The SCU sends the N-ACTION request message and waits for the N-ACTION response. After receiving this, the transaction is marked as "waiting".

Depending on a configuration value, the association will be closed or kept open. In the first case, there is another configurable value giving the number of minutes (by default 60) to wait for the corresponding N_EVENT_REPORT. In the second case, this time is the (also configurable) time-out for the association. For both cases, if the N_EVENT_REPORT does not arrive during the configured time, the transaction will be marked as failed.

If the N_EVENT_REPORT received has the status of "complete - failure exists", the transaction is marked as failed, else the transaction is marked as "completed"; In both cases, a message is shown to the user.

Storage Commitment is supported for all the SOP class UID's as mentioned in 'Acceptable

presentation contexts Storage'.

The Referenced Study Component Sequence is not supported.

Storage Media File-Set ID and UID Attributes will not be supported in the N-ACTION primitive invoked by the Storage Commitment SCU.

3.3.2.2 Real World Activity - Send Storage Commitment Response

3.3.2.2.1 Associated Real World Activity - Send Storage Commitment Response

Acting as an SC-SCP, UROSOP Access Storage Commitment AE received an Storage Commitment re request carried out the request, and is ready to send back the response, but the association is not open anymore. In this case it will initiate an association to send the N_EVENT_REPORT to the SCU.

3.3.2.2.2 Accepted Presentation Contexts - Send Storage Commitment Response

UROSOP Access Storage Commitment AE will accept Presentation Contexts as shown in the following table:

Table 8: Initiation presentation context Storage Commitment Response

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Storage Commitment Push Model SOP Class	1.2.840.10008.1.20.1	Implicit VR Little Endian	12.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		

3.3.2.2.3 SOP Specific Conformance Statement - Send Storage Commitment Response

Storage Media File-Set ID and UID Attributes will not be supported in the N-EVENT-REPORT primitive invoked by the Storage Commitment SCP.

3.3.3 Association Acceptance Policy

The UROSOP Access Storage Commitment AE accepts an association in two cases: when acting as SCP and an association request for Storage Commitment Push model arrives, and when acting as SCU if configured to receive N-EVENT-REPORT request on a separate association.

3.3.3.1 Real World Activity - Receive Storage Commitment Request

3.3.3.1.1 Associated Real-World Activity - Receive Storage Commitment Request

An association request arrives for UROSKOP Access Storage Commitment Push Model.

3.3.3.1.2 Accepted Presentation Contexts - Receive Storage Commitment Request

UROSKOP Access Storage Commitment AE will accept Presentation Contexts as shown in the following table:

Table 9: Presentation context accepted for Storage Commitment

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Storage Commitment Push Model SOP Class	1.2.840.10008.1.20..1	Implicit VR Little Endian	12.840.10008.1.2	SCP	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		

3.3.3.1.3 SOP Specific Conformance Statement - Receive Storage Commitment Request

Note:

Only one N-ACTION request per association is currently supported by the Storage Commitment SCP.

Storage Media File-Set ID and UID Attributes will not be supported in the N-EVENT-REPORT primitive invoked by the Storage Commitment SCP.

3.3.3.2 Real World Activity - Receive Storage Commitment Response

3.3.3.2.1 Associated Real World Activity - Receive Storage Commitment Response

UROSKOP Access Storage Commitment AE has sent a Storage Commitment Request and, being configured to receive response on a separate association, has closed the association, and now it gets an association request from the Storage Commitment SCP that want to send the results.

3.3.3.2.2 Accepted Presentation Contexts - Receive Storage Commitment Response

UROSKOP Access Storage Commitment AE will accept Presentation Contexts as shown in the following table:

Table 10: Presentation context accepted for Storage Commitment

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Storage Commitment Push Model SOP Class	1.2.840.10008.1.20.1	Implicit VR Little Endian	12.840.10008.1.2	SCU	None
		Explicit VR Big Endian	1.2.840.10008.1.2.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		

3.3.3.2.3 SOP Specific Conformance Statement - Receive Storage Commitment Response

If the N_EVENT_REPORT received has the status of "complete - failure exists", the transaction is marked as failed, else the transaction is marked as "completed"; In both cases, a message is shown to the user.

3.4 Query/Retrieve AE Specification

The Query/Retrieve SCU request that the remote SCP perform a match of all keys specified in the request, against the information in its database and the identified images will be moved or retrieved to the same or a different storage association.

The Query/Retrieve SCP responds to queries based on the records based on its database and images will be send to the requesting SCU or to a different storage destination.

UROSKOP Access DICOM products provide Standard Conformance to the following DICOM V3.0 SOP Classes as SCU and SCP:

Table 11: SOP Classes as an SCU / SCP

SOP Class Name	SOP Class UID
Patient Root Query/Retrieve Information Model – FIND	1.2.840.10008.5.1.4.1.2.1.1
Patient Root Query/Retrieve Information Model – MOVE	1.2.840.10008.5.1.4.1.2.1.2
Study Root Query/Retrieve Information Model- FIND	1.2.840.10008.5.1.4.1.2.2.1
Study Root Query/Retrieve Information Model – MOVE	1.2.840.10008.5.1.4.1.2.2.2

3.4.1 Association Establishment Policies

3.4.1.1 General

The configuration of the UROSKOP Access DICOM query/retrieve application defines the Application Entity Titles, the port numbers and of course the host name and net address.

3.4.1.2 Number of Associations

The UROSKOP Access DICOM Query/Retrieve AE initiates several associations at a time, one for each query/retrieve request being processed.

3.4.1.3 Asynchronous Nature

The UROSKOP Access DICOM AE does not support asynchronous communication (multiple out-standing transactions over a single association).

3.4.1.4 Implementation Identifying Information

The UROSKOP Access DICOM software provides a single Implementation Class UID of

1.3.12.2.1107.5.12.3

and an Implementation Version Name of

• “SIEMENS_SWFVB10E”.

3.4.2 Association Initiation Policy

The Query/Retrieve SCU and SCP establish an association by using the DICOM association services. During association establishment the Query/Retrieve application entities negotiate the supported SOP classes to exchange the capabilities of the SCU and the SCP.

The following DIMSE-C operations are supported as SCU:

- C-FIND
- C-MOVE

3.4.2.1 Real World Activity – Find SCU

3.4.2.1.1 Associated Real-World Activity – Find SCU

The associated Real-World activity is to initiate query request to an SCP with the query model Patient Root and Study Root.

3.4.2.1.2 Proposed Presentation Contexts – Find SCU

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 12: Proposed presentation Context Find SCU

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
PatientRoot Find Query/Retrieve	1.2.840.10008.5.1.4.1.2.1.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Study Root Query/Retrieve Find	1.2.840.10008.5.1.4.1.2.2.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

It is configurable which of the two query models (or both) are to be used by the Siemens Query SCU application.

Note:

C-FIND Extended Negotiation will NOT be supported by the SCU.

3.4.2.1.3 SOP Specific Conformance Statement - Find SCU

The UROSKOP Access DICOM Query/Retrieve SCU supports hierarchical queries with all mandatory search keys. On each level, the unique attributes of all previous levels are also sent (values are provided by the user by selection from a list). For instance for a query using the patient root model on Series level, the Patient ID of the current selected patient and the Study Instance UID of the current selected study are included in the message.

The Table 13: query attributes: describes the search keys for the two query models (Patient Root and Study Root) that the UROSKOP Access Query / Retrieve application supports as an SCU. Matching are either wildcard, which means that the user can supply a string containing wildcards, either universal, which means that the attribute is returned no matter what value it has.

Table 13: query attributes

Attribute name	Tag	Type	Matching	User input	Return value displayed
Patient level					
Patient name	(0010,0010)	R	wildcard	Enter value	yes
Patient ID	(0010,0020)	U	wildcard	Enter value	yes
Patient's birth date	(0010,0030)	O	Universal (Null)	-	yes
Patient's sex	(0010,0040)	O	Universal (Null)	-	yes

Number of Patient related studies	(0020,1200)	O	Universal (Null)		yes
Number of Patient relates series	(0020,1202)	O	Universal (Null)		yes
Number of Patient related instances	(0020,1204)	O	Univera (NULL)		-yes
Study level					
Patient name ^b	(0010,0010)	R	wildcard	Enter value	yes
Patient ID ^b	(0010,0020)	R	wildcard c	Enter value	yes
Study instance UID	(0020,000D)	U	single value	Select from list	
Study id	(0020,0010)	R	Universal (Null)	-	yes
Study date	(0008,0020)	R	Universal (Null)	-	yes
Study time	(0008,0030)	R	Universal (Null)	-	yes
ACCESSION number	(0008,0050)	R	Universal (Null)	-	yes
Study description	(0008,1030)	O	universal		yes
Referring physician's Name	(0008,0090)	O	Universal (NULL)		-yes
Name of physician reading study	(0008,8060)	O	Universal (NULL)		yes
Modalities in Study	(0008,0061)	O	Universal (NULL)		yes
Storage Media File Set ID	(0008,0130)	O	Universal (NULL)		yes
Retrieve AE Title	(0008,0054)	O	Universal (NULL)		-yes
Number of study related series	(0020,1206)	O	Universal (NULL)		yes
Number of study related instances	(0020,1208)	O	Universal (NULL)		yes

Table11: Patient root and Study Root query attributes

Attribute name	Tag	Type	Matching	User input	Return value displayed
Series level					
Series instance UID	(0020,000E)	U	single value	Select from list	-
Series number	(0020,0011)	R	Universal (NULL)	-	yes
Modality	(0008,0060)	R	Universal (NULL)	-	yes
Series date	(0008,0021)	O	Universal (NULL)	-	yes
Series time	(0008,0031)	O	Universal (NULL)	-	yes
Study ID	(0020,0010)	O	Universal (NULL)	-	yes
Series description	(0008,103E)	O	Universal (NULL)	-	yes
Storage Media File Set ID	(0008,0130)	O	Universal (NULL)	-	yes
Retrieve AE Title	(0008,0054)	O	Universal (NULL)	-	yes
Protocol name	(0018,1030)	O	Universal (NULL)	-	yes
Performed procedure step start date	(0040,0244)	O	Universal (NULL)	-	yes
Performed procedure step start time	(0040,0245)	O	Universal (NULL)	-	yes
Number of series related instances	(0020,1209)	O	Universal (NULL)	-	yes
Image level					
SOP instance UID	(0008,0018)	U	Universal (NULL)	-	yes
Instance Number	(0020,0013)	R	Universal (NULL)	-	yes
Storage Media File Set ID	(0008,0130)	O	Universal (NULL)	-	yes
Retrieve AE Title	(0008,0054)	O	Universal (NULL)	-	yes
Image date	(0008,0023)	O	Universal (NULL)	-	yes
Image time	(0008,0033)	O	Universal (NULL)	-	yes
Number of Frames	(0028,0008)	O	Universal (NULL)	-	yes
Image comments	(0020,4000)	O	Universal (NULL)	-	yes

- a. Only for Patient Root information model
- b. Only for Study Root information model
- c. Always a "*" is added to the string supplied by the user

3.4.2.2 Real World Activity - Move SCU

3.4.2.2.1 Associated Real-World Activity - Move SCU

The operator uses the UROSOP Access DICOM Query application to enter the query values and then initiates the retrieval of all matching DICOM composite objects (like images) from the remote node.

This will generate retrieval requests to a remote C-MOVE SCP using the C-MOVE operation with the query model Patient Root, Study Root and Patient/Study Only. The Storage Service Class Conformance Statement of the SCP must describe the C-STORE service which is generated by the C-MOVE service.

3.4.2.2.2 Proposed Presentation Contexts - Move SCU

The UROSOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 14: Proposed Presentation Context Move SCU

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
PatientRoot Query/Retrieve Move	1.2.840.10008.5.1.4.1.2.1.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Study Root Query/Retrieve Move	1.2.840.10008.5.1.4.1.2.2.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

Note: C-MOVE Extended Negotiation will be NOT supported by the SCU. C-MOVE on Patient level is not supported by the application.

3.4.2.2.3 SOP Specific Conformance Statement - Move SCU

At association establishment time the C-MOVE presentation context shall be negotiated. The C-STORE sub-operations must be done on a different association to transfer images to

another SCP of the Storage Service Class.

The Move SCU interprets following status codes:

Table 15: C_MOVE Response Status

Service Status	Meaning	Protocol Codes	Related Fields
Refused	Out of Resources - Unable to calculate number of matches Out of Resources - Unable to perform sub-operations	A701 A702	(0000,0902) (0000,1020) (0000,1021) (0000,1022) (0000,1023)
Failed	Identifier does not match SOP Class	A900	(0000,0901) (0000,0902)
Failed	Unable to process	Cxxx	(0000,0901) (0000,0902)
Cancel	Sub-operations terminated due to Cancel Indication	FE00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Warning	Sub-operations Complete - One or more Failures of Warnings	B000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Pending	Sub-operations are continuing	FF00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Success	Sub-operations Complete - No Failures or Warning	0000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)

3.4.3 Association Acceptance Policy

The Query/Retrieve SCU and SCP establish an association by using the DICOM association services. During association establishment the Query/Retrieve application entities negotiate the supported SOP classes to exchange the capabilities of the SCU and the SCP.

The following DIMSE-C operations are supported as SCP:

- C-FIND
- C-GET
- C-MOVE

- C-FIND-CANCEL
- C-GET-CANCEL
- C-MOVE-CANCEL

The SCP does support multiple C-FIND requests over the same association, but not multiple C-GET or C-MOVE Request.

3.4.3.1 Real World Activity - Find SCP

3.4.3.1.1 Associated Real-World Activity - Find SCP

The associated Real-World activity is to respond to query requests to an SCU with the query model Patient Root, Study Root and Patient/Study Only.

3.4.3.1.2 Proposed Presentation Contexts - Find SCP

The UROSOP Access DICOM AE will propose Presentation Contexts as shown in the following table:

Table 16: Proposed Presentation Contexts - Find SCP

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Patient Root Query/Retrieve Find	1.2.840.10008.5.1.4.1.2.1.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Study Root Query/Retrieve Find	1.2.840.10008.5.1.4.1.2.2.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Patient/Study Only Query/Retrieve Find	1.2.840.10008.5.1.4.1.2.3.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

Note:

C-FIND Extended Negotiation will be NOT supported by the SCP.

The order of preference in accepting a Transfer syntax is:

1. Explicit VR Little Endian
2. Explicit VR Big Endian
3. Implicit VR Little Endian

3.4.3.1.3 SOP Specific Conformance Statement - Find SCP

The UROSKOP Access DICOM Query/Retrieve SCP supports hierarchical with all mandatory and optional search keys. The query attributes will be treated Case Sensitive. The supported attributes on the various levels of the three information models are listed in following tables: Patient Root Information Model.

Table 17: Patient level attributes, Patient Root Information Model

Attribute name	Tag	Type	Matching
Patient name	(0010,0010)	R	single value, wildcard, universal
Patient id	(0010,0020)	U	single value, wildcard, universal
Patient's birth date	(0010,0030)	O	single value, range, universal
Patient's birth time	(0010,0032)	O	single value, range, universal
Patient's sex	(0010,0040)	O	single value, wildcard, universal
Ethnic group	(0010,2160)	O	single value, wildcard, universal
Patient comments	(0010,4000)	O	wildcard, universal
Number of Patient related studies	(0020,1200)	O	universal
Number of Patient relates series	(0020,1202)	O	universal
Number of Patient related instances	(0020,1204)	O	universal

Table 18: Study level attributes, Patient Root Information Model

Attribute name	Tag	Type	Matching
Study instance UID	(0020,000D)	U	single value, list of UIDs
Study id	(0020,0010)	R	single value, wildcard, universal
Study date	(0008,0020)	R	single value, range, universal
Study time	(0008,0030)	R	single value, range, universal
ACCESSion number	(0008,0050)	R	single value, wildcard, universal
Referring physician's name	(0008,0090)	O	single value, wildcard, universal

Attribute name	Tag	Type	Matching
Study description	(0008,1030)	O	single value, wildcard, universal
Admitting diagnoses description	(0008,1080)	O	single value, wildcard, universal
Patient's age	(0010,1010)	O	single value, wildcard, universal
Patient's size	(0010,1020)	O	single value, universal
Patient's weight	(0010,1030)	O	single value, universal
Occupation	(0010,2180)	O	single value, wildcard, universal
Additional patient history	(0010,21B0)	O	wildcard, universal
Name of physician reading study	(0008,1060)	O	single value, wildcard, universal
Modalities in Study	(0008,0061)	O	multiple values, universal
Number of study related series	(0020,1206)	O	universal
Number of study related instances	(0020,1208)	O	universal

Table 19: Series level attributes, Patient Root Information Model

Attribute name	Tag	Type	Matching
Series instance UID	(0020,000E)	U	single value, list of UID
Series number	(0020,0011)	R	single value, universal
Modality	(0008,0060)	R	single value, wildcard, universal
Laterality	(0020,0060)	O	single value, wildcard, universal

Table 20: Series level attributes, Patient Root Information Model

Attribute name	Tag	Type	Matching
Body part examined	(0018,0015)	O	single value, wildcard, universal
Patient position	(0018,5100)	O	single value, wildcard, universal
Smallest pixel value in series	(0028,0108)	O	single value, universal
Largest pixel value in series	(0028,0109)	O	single value, universal
Protocol name	(0018,1030)	O	single value, wildcard, universal

Attribute name	Tag	Type	Matching
Series date	(0008,0021)	O	single value, range, universal
Series time	(0008,0031)	O	single value, range, universal
Series description	(0008,103E)	O	single value, wildcard, universal
Operators name	(0008,1070)	O	single value, wildcard, universal
Performing physician's name	(0008,1050)	O	single value, wildcard, universal
Performed procedure step start date	(0040,0244)	O	universal
Performed procedure step start time	(0040,0245)	O	universal
Number of series related instances	(0020,1209)	O	universal

Table 21: Image level attributes, Patient Root Information Model

Attribute name	Tag	Type	Matching
SOP instance UID	(0008,0018)	U	single value, list of UID
Image number	(0020,0013)	R	single value, universal
Image date	(0008,0023)	O	single value, range, universal
Image time	(0008,0033)	O	single value, range, universal
Modality	(0008,0060)	O	single value, wildcard, universal
Image comments	(0020,4000)	O	universal

3.4.3.1.3.1 Study Root Information Model

Table 22: Study level attributes, Study Root Information Model

Attribute name	Tag	Type	Matching
Patient name	(0010,0010)	R	single value, wildcard, universal
Patient id	(0010,0020)	R	single value, wildcard, universal
Patient's birth date	(0010,0030)	O	single value, range, universal
Patient's birth time	(0010,0032)	O	single value, range, universal

Attribute name	Tag	Type	Matching
Patient's sex	(0010,0040)	O	single value, wildcard, universal
Patient comments	(0010,4000)	O	wildcard, universal
Number of Patient related studies	(0020,1200)	O	universal
Number of Patient relates series	(0020,1202)	O	universal
Number of Patient related instances	(0020,1204)	O	universal
Study instance UID	(0020,000D)	U	single value, list of UIDs
Study id	(0020,0010)	R	single value, wildcard, universal
Study date	(0008,0020)	R	single value, range, universal
Study time	(0008,0030)	R	single value, range, universal
ACCESSion number	(0008,0050)	R	single value, wildcard, universal
Referring physician's name	(0008,0090)	O	single value, wildcard, universal
Study description	(0008,1030)	O	single value, wildcard, universal
Admitting diagnoses description	(0008,1080)	O	single value, wildcard, universal
Patient's age	(0010,1010)	O	Single value, wildcard, universal
Patient's size	(0010,1020)	O	Single value, universal
Patient's weight	(0010,1030)	O	Single value, universal
Occupation	(0010,2180)	O	Single value, wildcard, universal
Additional patient history	(0010,21B0)	O	wildcard, universal
Name of physician reading study	(0008,8060)	O	Single value, wildcard, universal
Modalities in Study	(0008,0061)	O	multiple values, universal
Number of study related series	(0020,1206)	O	universal
Number of study related instances	(0020,1208)	O	universal

Table 23: Series level attributes, Study Root Information Model

Attribute name	Tag	Type	Matching
Series instance UID	(0020,000E)	U	single value, list of UID
Series number	(0020,0011)	R	single value, universal
Modality	(0008,0060)	R	single value, wildcard, universal
Laterality	(0020,0060)	O	single value, wildcard, universal
Body part examined	(0018,0015)	O	single value, wildcard, universal
Patient position	(0018,5100)	O	single value, wildcard, universal
Smallest pixel value in series	(0028,0108)	O	single value, universal
Largest pixel value in series	(0028,0109)	O	single value, universal
Protocol name	(0018,1030)	O	single value, wildcard, universal
Series date	(0008,0021)	O	single value, range, universal
Series time	(0008,0031)	O	single value, range, universal
Series description	(0008,103E)	O	single value, wildcard, universal
Operators name	(0008,1070)	O	single value, wildcard, universal
Performing physician's name	(0008,1050)	O	single value, wildcard, universal
Performed procedure step start date	(0040,0244)	O	universal
Performed procedure step start time	(0040,0245)	O	universal
Number of series related instances	(0020,1209)	O	universal

Table 24 Image level attributes, Study Root Information Model

Attribute name	Tag	Type	Matching
SOP instance UID	(0008,0018)	U	single value, list of UID
Image number	(0020,0013)	R	single value, universal
Image date	(0008,0023)	O	single value, range, universal

Attribute name	Tag	Type	Matching
Image time	(0008,0033)	O	single value, range, universal
Modality	(0008,0060)	O	single value, wildcard, universal
Image comments	(0020,4000)	O	universal

3.4.3.1.3.2 Patient Study Only Information models

Table 25: Patient instance level, Patient Study Only Information Model

Attribute name	Tag	Type	Matching
Patient name	(0010,0010)	R	single value, wildcard, universal
Patient id	(0010,0020)	U	single value, wildcard, universal
Patient's birth date	(0010,0030)	O	single value, range, universal
Patient's birth time	(0010,0032)	O	single value, range, universal
Patient's sex	(0010,0040)	O	single value, wildcard, universal
Ethnic group	(0010,2160)	O	single value, wildcard, universal
Patient comments	(0010,4000)	O	wildcard, universal
Number of Patient related studies	(0020,1200)	O	universal
Number of Patient relates series	(0020,1202)	O	universal
Number of Patient related instances	(0020,1204)	O	universal

Table 26: Study level attributes, Patient Study Only Information Model

Attribute name	Tag	Type	Matching
Study instance UID	(0020,000D)	U	single value, list of UIDs
Study id	(0020,0010)	R	single value, wildcard, universal
Study date	(0008,0020)	R	single value, range, universal
Study time	(0008,0030)	R	single value, range, universal
ACCESSION number	(0008,0050)	R	single value, wildcard, universal
Referring physician's name	(0008,0090)	O	single value, wildcard, universal

Attribute name	Tag	Type	Matching
Study description	(0008,1030)	O	single value, wildcard, universal
Admitting diagnoses description	(0008,1080)	O	single value, wildcard, universal
Patient's age	(0010,1010)	O	single value, wildcard, universal
Patient's size	(0010,1020)	O	single value, universal
Patient's weight	(0010,1030)	O	single value, universal
Occupation	(0010,2180)	O	single value, wildcard, universal
Additional patient history	(0010,21B0)	O	wildcard, universal
Name of physician reading study	(0008,8060)	O	single value, wildcard, universal
Modalities in Study	(0008,0061)	O	multiple values, universal
Number of study related series	(0020,1206)	O	universal
Number of study related instances	(0020,1208)	O	universal

ATTENTION! In DICOM wildcard queries the symbol '?' is treated as '*' by Find SCP. So a wildcard query with "?abc*" is actually treated as "*abc*"

If the value for the unique key Patient ID is not known, it may be returned empty.

The C_FIND_RSP message will contain the following attributes:

- Specific Character Set (0008,0005) (If there is a specific character set in use)
- Query/Retrieve Level (0008,0052) from the C_FIND_RQ
- Retrieve AE Title (0008,0054) at study, series and image level. This value is a list of AE titles from which the images can be retrieved. Might be NULL except for the lowest level of the query model (Image level for Patient Root or Study Root and Study level for Patient/Study Only).
- Storage-Media FileSet ID (0088,0130) at level study, series and image
If Storage-Media FileSet ID is not present a NULL value will be returned
- attributes requested by C_FIND_RQ and supported by the SCP.

Relational queries are not supported.

A Remote DICOM AE can cancel the query by sending a C_CANCEL_FIND_RQ message. If the Find SCP receives C_CANCEL_FIND_RQ before it has completed the processing of the matches it shall stop the database matching process and return a status of Cancelled to the Remote DICOM AE.

The Find SCP returns following status codes:

Table 27: C-FIND return status

Service Status	Meaning	Protocol Codes	Related Fields
Refused	Out of Resources	A700	(0000,0902)
Failed	Identifier does not match SOP Class Unable to process	A900 C001	(0000,0901) (0000,0902) (0000,0901) (0000,0902)
Cancel	Matching terminated due to Cancel request	FE00	None
Success	Matching is complete - No final Identifier is supplied	0000	None
Pending	Matches are continuing - Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys Matches are continuing - Warning that one or more Optional Keys were not supported for existence and/or matching for this identifier	FF00 FF01	Identifier Identifier

3.4.3.2 Real World Activity - Get SCP

3.4.3.2.1 Associated Real-World Activity - Get SCP

The associated Real-World activity is to respond to retrieve requests initiated from an SCU. The SCP supports the query model Patient Root, Study Root and Patient/Study Only. The Storage Service Class Conformance Statement describes the C-STORE service which is generated by the C-GET service.

3.4.3.2.2 Proposed Presentation Contexts - Get SCP

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 28: Proposed Presentation Contexts Get SCP return status

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Patient Root Query/Retrieve Get	1.2.840.10008.5.1.4.1.2.1.3	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Study Root Query/Retrieve Get	1.2.840.10008.5.1.4.1.2.2.3	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Patient/Study Only Query/Retrieve Get	1.2.840.10008.5.1.4.1.2.3.3	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

Note:

C-GET Extended Negotiation will be NOT supported by the SCP.

The order of preference in accepting a Transfer syntax is:

1. Explicit VR Little Endian
2. Explicit VR Big Endian
3. Implicit VR Little Endian

3.4.3.2.3 SOP Specific Conformance Statement - Get SCP

At association establishment time the C-GET presentation context must be negotiated along with the C-STORE sub-operations which must be accomplished on the same association as the C-GET operation.

Note:

In DICOM wildcard queries the symbol '?' is treated as '*' by Get SCP.

So a wildcard query with “?abc*” is actually treated as “*abc*”

The Get SCP returns following status codes:

Table 29: C-GET return status

Service Status	Meaning	Protocol Codes	Related Fields
Refused	Out of Resources - Unable to calculate number of matches	A701	(0000,0902) (0000,1020) (0000,1021)
	Out of Resources - Unable to perform sub-operations	A702	(0000,1022) (0000,1023)
Failed	Identifier does not match SOP Class	A900	(0000,0901) (0000,0902)
Failed	Unable to process	C001	(0000,0901) (0000,0902)
Cancel	Sub-operations terminated due to Cancel Indication	FE00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Warning	Sub-operations Complete - One or more Failures of Warnings	B000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Pending	Sub-operations are continuing	FF00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Success	Sub-operations Complete – No Failures or Warnings	0000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Success	Sub-operations Complete - No Failures or Warning	0000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)

3.4.3.3 Real World Activity - Move SCP

3.4.3.3.1 Associated Real-World Activity - Move SCP

The associated Real-World activity is to respond to retrieve requests to an SCU. The SCP supports the query model Patient Root, Study Root and Patient/Study Only. The Storage Service Class Conformance Statement describes the C-STORE service which is generated by the C-MOVE service.

3.4.3.3.2 Proposed Presentation Contexts - Move SCP

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in:

Table 30: Proposed Presentation Contexts - Move SCP

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Patient Root. Query/Retrieve Move -	1.2.840.10008.5.1.4.1.2.1.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Study Root. Query/Retrieve Move -	1.2.840.10008.5.14.1.2.2.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		
Patient/Study Only Query/ Retrieve Move	1.2.840.10008.5.1.4.1.2.3.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCP	See Note
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

Note: C-MOVE Extended Negotiation will be NOT supported by the SCP.

The order of preference in accepting a Transfer syntax is:

1. Explicit VR Little Endian
2. Explicit VR Big Endian
3. Implicit VR Little Endian

3.4.3.3.3 SOP Specific Conformance Statement - Move SCP

At association establishment time the C-MOVE presentation context shall be negotiated. The C-STORE sub-operations is done on a different association, specified in the C-MOVE request, to transfer images to another SCP of the Storage Service Class.

Table 31: C-MOVE return status

Service Status	Meaning	Protocol Codes	Related Fields
Refused	Out of Resources - Unable to calculate number of matches Out of Resources - Unable to perform sub-operations	A701 A702	(0000,0902) (0000,1020) (0000,1021) (0000,1022) (0000,1023)
Failed	Identifier does not match SOP Class	A900	(0000,0901) (0000,0902)
Failed	Unable to process	C001	(0000,0901) (0000,0902)
Cancel	Sub-operations terminated due to Cancel Indication	FE00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Warning	Sub-operations Complete One or more Failures of Warnings	B000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Pending	Sub-operations are continuing	FF00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Success	Sub-operations Complete – No Failures or Warning	0000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)

3.5 Print AE Specification

The print management SCU invokes print management DIMSE services to transfer images from the local AE to the remote SCP AE to print the images with the defined film format and size on a selected network DICOM hardcopy printer. See DICOM part 4 annex H.

UROSKOP Access DICOM products provide Standard Conformance to the following DICOM V3.0 Basic Grayscale Print Management Meta SOP Classes Basic Color Print Management Meta SOP Class and the optional Print Job SOP Class as an SCU:

Table 32: Basic Gray Scale Print Management Meta SOP-Classes

SOP Class Name	SOP Class UID	Usage SCU/SCP
Basic Grayscale Print Management Meta SOP Class	1.2.840.10008.5.1.1.9	M/M
- Basic Film Session SOP Class	1.2.840.10008.5.1.1.1	M/M
- Basic Film Box SOP Class	1.2.840.10008.5.1.1.2	M/M
- Basic Grayscale Image Box SOP Class	1.2.840.10008.5.1.1.4	M/M

- Printer SOP Class	1.2.840.10008.5.1.1.16	M/M
Print Job SOP Class	1.2.840.10008.5.1.1.14	U/U
Presentation LUT SOP Class	1.2.840.10008.5.1.1.23	U/U

Table 33: Basic Color Print Management Meta SOP-Classes

SOP Class Name	SOP Class UID	Usage SCU/SCP
Basic Color Print Management Meta SOP Class	1.2.840.10008.5.1.1.18	M/M
- Basic Film Session SOP Class	1.2.840.10008.5.1.1.1	M/M
- Basic Film Box SOP Class	1.2.840.10008.5.1.1.2	M/M
- Basic Color Image Box SOP Class	1.2.840.10008.5.1.1.4.1	M/M
- Printer SOP Class	1.2.840.10008.5.1.1.16	M/M
Print Job SOP Class	1.2.840.10008.5.1.1.14	U/U

3.5.1 Association Establishment Policies

3.5.1.1 General

The configuration of the UROSKOP Access DICOM print management SCU defines the Application Entity Titles, the port numbers and of course the host name and net address.

3.5.1.2 Number of Associations

The UROSKOP Access DICOM application initiates one/several association(s) at a time, one for each transfer request being processed.

3.5.1.3 Asynchronous Nature

The UROSKOP Access DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

3.5.1.4 Implementation Identifying Information

The UROSKOP Access DICOM software provides a single Implementation Class UID of

1.3.12.2.1107.5.12.3

and an Implementation Version Name of

- “SIEMENS_SWFVB10E”.

3.5.2 Association Initiation Policy

The Print Management SCU and SCP establish an association by using the DICOM association services. During association establishment the Print Management application entities negotiate the supported SOP classes to exchange the capabilities of the SCU and the SCP. If the SCU supports only mandatory SOP classes, the negotiation of optional capabilities is not necessary.

3.5.2.1 Real-World Activity

3.5.2.1.1 Associated Real-World Activity

The associated Real-World activity is to print over a network a set of images on a film sheet with one or more copies. The images are converted to Standard\1-1. If the response from the remote application contains a status other than Success or Warning the association is aborted.

3.5.2.1.2 Proposed Presentation Contexts

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 34: Proposed Presentation Contexts

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
Basic Grayscale Print Management Meta SOP class	1.2.840.10008.5.1.1.9	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Basic film session SOP class	1.2.840.10008.5.1.1.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Basic film box SOP class	1.2.840.10008.5.1.1.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		

Basic grayscale image box SOP class	1.2.840.10008.5.1.1.4	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Printer SOP class	1.2.840.10008.5.1.1.16	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Print Job SOP class	1.2.840.10008.5.1.1.14	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Presentation LUT SOP class	1.2.840.10008.5.1.1.23	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
Basic Color Print Management Meta SOP class	1.2.840.10008.5.1.1.18	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Basic film session SOP class	1.2.840.10008.5.1.1.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Basic film box SOP class	1.2.840.10008.5.1.1.2	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		

Basic color image box SOP class	1.2.840.10008.5.1.1.4.1	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Printer SOP class	1.2.840.10008.5.1.1.16	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		
Print Job SOP class	1.2.840.10008.5.1.1.14	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax,	1.2.840.10008.1.2.2		

3.5.2.1.3 SOP Specific Conformance Statement

The UROSKOP Access DICOM SCU conforms to the DICOM Basic Grayscale Print Management Class. The application uses a setting platform to define the properties of the connected DICOM SCP,

E.g.:

- Maximum number of print jobs in the queue
- Maximum number of print copies
- Supported film sizes of the connected DICOM SCP
- Supported film formats of the DICOM SCP
- Lookup table definition.

The printing is only suspended in the case of a failure return status of the SCP.

3.5.2.1.3.1 SOP Specific Conformance to Basic Film Session SOP Class

The Basic Film Session information object definition describes all the user-defined parameter, which are common for all the films of a film session. The Basic Film Session refers to one or more Basic Film Boxes that are printed on one hardcopy printer.

The UROSKOP Access DICOM Print application supports the following DIMSE Service Elements for the

Basic Film Session SOP class as SCU:

-N-CREATE

-N-DELETE

The Basic Film Session SOP class N-CREATE_RQ (SCU) uses following attributes:

Table 35: Used Basic Film Session N-CREATE attributes

Attribute name	Tag	Usage SCU	Supported Values
Number of Copies	(2000,0010)	U	
Medium Type	(2000,0030)	U	BLUE FILM CLEAR FILM PAPER
Film Destination	(2000,0040)	U	MAGAZINE PROCESSOR

The Affected SOP Instance UID received in N-CREATE_RSP message from SCP will be saved internally and used for later requests like N-DELETE_RQ on the Basic Film Session SOP Class - see table below:)

Table 36: Attributes of the N-DELETE_RQ on the Basic Film Session SOP Class

Attribute name	Tag	Source of information
Requested SOP Instance UID	(0008,0018)	Affected SOP Instance UID of N-CREATE_RSP on Basic Film Session

The N-DELETE_RQ on the Basic Film Session SOP Class is used to delete the complete Basic Film Session SOP Instance hierarchy.

The Basic Film Session SOP class interprets following status codes (from N-CREATE_RSP, N-DELETE_RSP messages):

Table 37: Basic Film Session SOP status

Service Status	Meaning	Protocol Codes
Failure	Film session SOP instances hierarchy does not contain film box SOP instances	C600
	Unable to create print job, print queue is full	C601
	Image size is larger than images box size	C603
Warning	Memory allocation not supported	B600
	Film session printing is not supported	B601
Warning	Film box does not contain image box (empty page)	B602
Success	Film belonging to the film session are accepted for printing	0000

3.5.2.1.3.2 SOP Specific Conformance to Basic Film Box SOP Class

The Basic Film Box information object definition describes all the user defined parameter of one film of the film session. The Basic Film Box information description defines the

presentation parameters, which are common for all images on a given sheet of film.

The Basic Film Box refers to one or more Image Boxes.

Supported as SCU are:

- N-CREATE
- N-ACTION
- N-DELETE

The Basic Film Box SOP class N-CREATE_RQ message uses following attributes (the used values for each attribute depend how the DICOM Printer is configured within the UROSKOP Access product):

Table 38: Used Film Box N-CREATE_RQ attributes

Attribute name	Tag	Usage SCU	Supported Values
Image Display Format	(2010,0010)	M	Standard\1-1
Referenced Film Session Sequence	(2010,0500)	M	
>Referenced SOP Class UID	(0008,1150)	M	1.2.840.10008.5.1.1.1
>Referenced SOP Instance UID	(0008,1155)	M	
Film Orientation	(2010,0040)	M	PORTRAIT
Film Size ID	(2010,0050)	M	8INX10IN 10INX12IN 10INX14IN 11INX14IN 11INX17IN 14INX14IN 14INX17IN 24CMX24CM 24CMX30CM
Magnification Type	(2010,0060)	M	BILINEAR CUBIC NONE REPLICATE
Max Density	(2010,0130)	U	
Min Density	(2010,0120)	U	
Illumination	(2010,015E)	U	> 0 Required if Presentation LUT is present
Reflective Ambient Light	(2010,0160)	U	> 0 Required if Presentation LUT is present
Referenced Presentation LUT SOP Class UID	(2050,0500)	U	

The N-CREATE_RSP message from the SCP then contains the References Image Box Sequence with its SOP Class and Instance UID which is stored internally and then used for

the Basic Image Box SOP Class N-SET RQ messages.

After all parameters for the Image boxes on the film sheet have been set then the UROSKOP Access DICOM print application SCU will issue a N_ACTION_RQ message with the SOP Instance UID of the Basic Film Box (returned in N-CREATE_RSP of Basic Film Box SOP class) and the Action Type ID set to 1.

The Affected SOP Instance UID received in N-CREATE_RSP message from SCP will be saved internally and can be used later for N-DELETE_RQ request on the Basic Film Box SOP Class - see table below:)

Table 39: Attributes of the N-DELETE_RQ on the Basic Film Session SOP Class

Attribute name	Tag	Source of information
Requested SOP Instance UID	(0008,0018)	Affected SOP Instance UID of N-CREATE_RSP on Basic Film Box

The Basic Film Box SOP class interprets following status codes from the N-CREATE_RSP, N-DELETE_RSP and N_ACTION_RSP messages:

Table 40: Basic Film Box SOP Status

Service Status	Meaning	Protocol Codes
Failure	Unable to create print job, print queue is full	C602
	Image size is larger than images box size	C603
Warning	Film box does not contain image box (empty page)	B603
	Requested MinDensity or MaxDensity outside of printer's operating range	B605
	Film accepted for printing	0000

3.5.2.1.3.3 SOP Specific Conformance to Basic Grayscale Image Box SOP Class

The Basic Grayscale Image Box information object definition is the presentation of an image and image related data in the image area of a film. The Basic Image Box information describes the presentation parameters and image pixel data, which apply to a single image of a sheet of film.

The Grayscale Image Box SOP class uses only the N_SET_RQ with the following attributes

Table 41: Used Basic Grayscale Image Box N_SET attributes

Attribute name	Tag	Usage SCU	Supported Values
Image Position	(2020,0010)	M	1
Basic Grayscale Image Sequence	(2020,0110)	M	
>Samples Per Pixel	(0028,0002)	M	1
>Photometric Interpretation	(0028,0004)	M	MONOCHROME1/2 for grayscale images
>Rows	(0028,0010)	M	

>Columns	(0028,0011)	M	
>Pixel Aspect Ratio	(0028,0034)	M	
>Bits Allocated	(0028,0100)	M	8
>Bits Stored	(0028,0101)	M	8
>High Bit	(0028,0102)	M	7
>Pixel Representation	(0028,0103)	M	0
>Pixel Data	(7FE0,0010)	M	

The Grayscale Image Box SOP class interprets following status codes:

Table 42: Basic Grayscale Image Box SOP status

Service Status	Meaning	Protocol Codes
Warning	Requested MinDensity or MaxDensity outside of printer's operating range	B605
Failure	Image contains more pixel than printer can print in Image box	C603
	Insufficient memory in printer to store the image	C605
Success		0000

3.5.2.1.3.4 SOP Specific Conformance to Basic Color Image Box SOP Class

Color Image Box information object definition is the presentation of an image and image related data in the image area of a film. The Basic Image Box information describes the presentation parameters and image pixel data which apply to a single image of a sheet of film.

The Color Image Box SOP class uses only the N_SET_RQ with the following attributes

Table 43: Used Basic Color Image Box N-SET attributes

Attribute name	Tag	UsageSCU	SupportedValues
Image Position	(2020,0010)	M	1
Basic Color Image Sequence	(2020,0111)	M	
>Samples Per Pixel	(0028,0002)	M	3
>Photometric Interpretation	(0028,0004)	M	RGB
>Planar Configuration	(0028,0006)	M	0
>Rows	(0028,0010)	M	
>Columns	(0028,0011)	M	
>Pixel Aspect Ratio	(0028,0034)	M	
>Bits Allocated	(0028,0100)	M	8
>Bits Stored	(0028,0101)	M	8
>High Bit	(0028,0102)	M	7
>Pixel Representation	(0028,0103)	M	0
>Pixel Data	(7FE0,0010)	M	

The Color Image Box SOP class interprets following status codes:

Table 44: Basic Color Image Box SOP status

Service Status	Meaning	Protocol Codes
Warning	Image size larger than image box size.	B604
Failure	Image contains more pixel than printer can print in Image box	C603
	Insufficient memory in printer to store the image	C605
Success		0000

3.5.2.1.3.5 SOP Specific Conformance to Presentation LUT SOP Class

The Presentation LUT Information Object is the Presentation LUT. The objective of the Presentation LUT is to realize image hardcopy printing tailored for specific modalities, applications, and user preferences. The output of the Presentation LUT is Presentation Values (P-Values). P-Values are approximately related to human perceptual response. They are intended to facilitate common input for hardcopy. P-Values are intended to be independent of the specific class or characteristics of the hardcopy device.

Table 45: Attributes of the N-CREATE_RQ on the Presentation LUT SOP Class

Attribute name	Tag	Usage SCU	Supported Values
Presentation LUT Shape	(2050,0020).	U	IDENTITY

Table 46: Attributes of the N-CREATE_RSP on the Presentation LUT SOP Class

Attribute of Presentation LUT	Type	Value list	Note
Affected SOP Instance UID	CString	-	Instance UID After receiving the N-CREATE_RSP, the Affected SOP Instance UID has to be saved and used for later actions (e.g. N-CREATE_RQ ,N-DELETE_RQ) on the Presentation LUT SOP Class. N-CREATE_RQ on the Basic Film Box requires this UID to be set , if the Presentation LUT SOP class is to be supported by the SCU N-CREATE_RQ on the Basic Film Box is required to include the attributes: Illumination, and Reflective Ambient Light, if the Presentation LUT SOP class is to be supported by the SCU

Table 47: Attributes of the N-CREATE_RSP on the Presentation LUT SOP Class

Attribute of Presentation LUT	Source of information on this attribute	Type	Value list	Note
Requested SOP Instance UID	Affected SOP Instance UID of N-CREATE_RSP on Presentation LUT	CString	-	-

The N-DELETE_RQ on the Presentation LUT SOP Class is used to delete the complete Presentation LUT SOP Instance hierarchy.

There is no attribute contained in the response message from the SCP which is interesting for the SCU for further use.

The Presentation LUT SOP class interprets the following status codes from the N-CREATE_RSP, N-DELETE_RSP messages:

Table 48: Status codes for the N-CREATE_RSP and N-DELETE_RSP

Service Status	Meaning	Protocol Codes
Warning	Requested Min Density or Max Density outside the HCD's operating range. HCD will use its respective minimum or maximum density value instead.	B605
Success	Presentation LUT Successfully created	0000

3.5.2.1.3.5 SOP Specific Conformance to Printer SOP Class

The Printer SOP Class is the possibility to monitor the status of the hardcopy printer in a synchronous and an asynchronous way.

The UROSKOP Access uses the mandatory N-EVENT Report DIMSE service to monitor the changes of the printer status in an asynchronous way.

It can directly ask the Print SP for its status or can receive Events from the Print SCP asynchronously:

N-GET as SCU

N-EVENT-REPORT as SCU

In both cases the following information is supported:

Table 49: Used Printer N-EVENT report

Event type name	Event	Attributes	Tag	Usage SCU
Normal	1			
Warning	2	Printer Status Info	(2110,0020)	U
Failure	3	Printer Status Info	(2110,0020)	U

Table 50: Mandatory Printer N-Get Attributes

Attribute name	Tag	Usage SCP
Printer Status	(2110,0010)	NORMAL FAILURE WARNING
Printer Status Info	(2110,0020)	see 0

3.5.2.1.3.6 SOP Specific Conformance to Print Job SOP Class

The Print Job SOP Class is the possibility to monitor the execution of the print process.

The UROSKOP Access DICOM Print application supports the optional N-EVENT Report DIMSE service to receive the changes of the print job status in an asynchronous way

It can receive Events from the Print SCP asynchronously:

- N-EVENT-REPORT

The following information is supported:

Table 51: Used Print Job N-EVENT report

Event type name	Event	Attributes	Tag	Usage SCU
Normal	1	Execution Status Info	(2100,0030)	U
		Print Job ID	(2100,0010)	- (Print Queue Management SOP Class not supported)
		Film Session Label	(2000,0050)	U
		Printer Name	(2110,0030)	U
Printing	2	Execution Status Info	(2100,0030)	U
		Print Job ID	(2100,0010)	- (Print Queue Management SOP Class not supported)
		Film Session Label	(2000,0050)	U
		Printer Name	(2110,0030)	U

Done	3	Execution Status	(2100,0030)	U
		Print Job ID	(2100,0010)	.- (Print Queue Management SOP Class not supported)
		Film Session Label	(2000,0050)	U
		Printer Name	(2110,0030)	U
Failure	4	Execution Status Info	(2100,0030)	U
		Print Job ID	(2100,0010)	- (Print Queue Management SOP Class not supported)
		Film Session Label	(2000,0050)	U
		Printer Name	(2110,0030)	U

For a detailed description of how UROSKOP Access reacts to the various messages please refer to the section: "Behavior of SCU when receiving information" .

3.5.2.1.3.7 Behavior of SCU when receiving information

The following tables document the behavior of the UROSKOP Access system in response to messages received for the printer SOP class and the print job SOP class.

Table 52: Printer Status Info within Printer SOP Class/Execution Status Info within Print Job SOP

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
NORMAL	Camera is ready.	Camera is ready.	<None>/idle
BAD RECEIVE MGZ	There is a problem with the film receive magazine. Films from the printer cannot be transported into the magazine.	Problem with receive magazine.	<None>/interact
BAD SUPPLY MGZ	There is a problem with a film supply magazine. Films from this magazine cannot be transported into the printer.	Problem with supply magazine.	<None>/interact
CALIBRATION	Printer is performing self calibration, it is expected to be available for normal operation shortly.	Self calibration. Please wait.	<None>/idle
CALIBRATION ERR	An error in the printer calibration has been detected, quality of processed films may not be optimal.	Problem in calibration. Film quality may not be optimal.	<None>/interact
CHECK CHEMISTRY	A problem with the processor chemicals has been detected, quality of processed films may not be optimal.	Problem with chemistry. Film quality may not be optimal.	<None>/interact

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
CHECK SORTER	There is an error in the film sorter	Error in film sorter.	<None>/interact
CHEMICALS EMPTY	There are no processing chemicals in the processor, films will not be printed and processed until the processor is back to normal.	Camera chemistry empty. Please check.	<None>/interact
CHEMICALS LOW	The chemical level in the processor is low, if not corrected, it will probably shut down soon.	Camera chemistry low. Please check.	<None>/interact
COVER OPEN	One or more printer or processor covers, drawers, doors are open.	Camera cover, drawer or door open.	<None>/interact
ELEC CONFIG ERR	Printer configured improperly for this job.	Camera configured improperly for this job. Queue stopped.	Queue for this camera will be STOPPED/Queue stopped
ELEC DOWN	Printer is not operating due to some unspecified electrical hardware problem.	Camera electrical hardware problem.	<None>/interact
ELEC SW ERROR	Printer not operating for some unspecified software error.	Camera software problem. Queue stopped.	Queue for this camera will be STOPPED/queue stopped
EMPTY 8x10	The 8x10 inch film supply magazine is empty.	8x10 film supply empty.	<None>/interact
EMPTY 8x10 BLUE	The 8x10 inch blue film supply magazine is empty.	8x10 blue film supply empty.	<None>/interact
EMPTY 8x10 CLR	The 8x10 inch clear film supply magazine is empty.	8x10 clear film supply empty.	<None>/interact
EMPTY 8x10 PAPR	The 8x10 inch paper supply magazine is empty.	8x10 paper supply empty.	<None>/interact
EMPTY 10x12	The 10x12 inch film supply magazine is empty.	10x12 film supply empty.	<None>/interact
EMPTY 10x12 BLUE	The 10x12 inch blue film supply magazine is empty.	10x12 blue film supply empty.	<None>/interact
EMPTY 10x12 CLR	The 10x12 inch clear film supply magazine is empty.	10x12 clear film supply empty.	<None>/interact
EMPTY 10x12 PAPR	The 10x12 inch paper supply magazine is empty.	10x12 paper supply empty.	<None>/interact
EMPTY 10x14	The 10x14 inch film supply magazine is empty.	10x14 film supply empty.	<None>/interact
EMPTY 10x14 BLUE	The 10x14 inch blue film supply magazine is empty.	10x14 blue film supply empty.	<None>/interact
EMPTY 10x14 CLR	The 10x14 inch clear film supply magazine is empty.	10x14 clear film supply empty.	<None>/interact

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
EMPTY 10x14 PAPR	The 10x14 inch paper supply magazine is empty.	10x14 paper supply empty.	<None>/interact
EMPTY 11x14	The 11x14 inch film supply magazine is empty.	11x14 film supply empty.	<None>/interact
EMPTY 11x14 BLUE	The 11x14 inch blue film supply magazine is empty.	11x14 blue film supply empty.	<None>/interact
EMPTY 11x14 CLR	The 11x14 inch clear film supply magazine is empty.	11x14 clear film supply empty.	<None>/interact
EMPTY 11x14 PAPR	The 11x14 inch paper supply magazine is empty.	11x14 paper supply empty.	<None>/interact
EMPTY 14x14	The 14x14 inch film supply magazine is empty.	14x14 film supply empty.	<None>/interact
EMPTY 14x14 BLUE	The 14x14 inch blue film supply magazine is empty.	14x14 blue film supply empty.	<None>/interact
EMPTY 14x14 CLR	The 14x14 inch clear film supply magazine is empty.	14x14 clear film supply empty.	<None>/interact
EMPTY 14x14 PAPR	The 14x14 inch paper supply magazine is empty.	14x14 paper supply empty.	<None>/interact
EMPTY 14x17	The 14x17 inch film supply magazine is empty.	14x17 film supply empty.	<None>/interact
EMPTY 14x17 BLUE	The 14x17 inch blue film supply magazine is empty.	14x17 blue film supply empty.	<None>/interact
EMPTY 14x17 CLR	The 14x17 inch clear film supply magazine is empty.	14x17 clear film supply empty.	<None>/interact
EMPTY 14x17 PAPR	The 14x17 inch paper supply magazine is empty.	14x17 paper supply empty.	<None>/interact
EMPTY 24x24	The 24x24 inch film supply magazine is empty.	24x24 film supply empty.	<None>/interact
EMPTY 24x24 BLUE	The 24x24 inch blue film supply magazine is empty.	24x24 blue film supply empty.	<None>/interact
EMPTY 24x24 CLR	The 24x24 inch clear film supply magazine is empty.	24x24 clear film supply empty.	<None>/interact
EMPTY 24x24 PAPR	The 24x24 inch paper supply magazine is empty.	24x24 paper supply empty.	<None>/interact
EMPTY 24x30	The 24x30 inch film supply magazine is empty.	24x30 film supply empty.	<None>/interact
EMPTY 24x30 BLUE	The 24x30 inch blue film supply magazine is empty.	24x30 blue film supply empty.	<None>/interact
EMPTY 24x30 CLR	The 24x30 inch clear film supply magazine is empty.	24x30 clear film supply empty.	<None>/interact
EMPTY 24x30 PAPR	The 24x30 inch paper supply magazine is empty.	24x30 paper supply empty.	<None>/interact
EMPTY A4 PAPR	The A4 paper supply magazine is empty.	A4 paper supply empty.	<None>/interact

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
EMPTY A4 TRANS	The A4 transparency supply magazine is empty.	A4 transparency supply empty.	<None>/interact
EXPOSURE FAILURE	The exposure device has failed due to some unspecified reason.	Exposure device has failed.	<None>/interact
FILM JAM	A film transport error has occurred and a film is jammed in the printer or processor.	Film jam.	<None>/interact
FILM TRANSP ERR	There is a malfunction with the film transport, there may or may not be a film jam.	Film transport problem.	<None>/interact
FINISHER EMPTY	The finisher is empty.	Finisher is empty.	<None>/interact
FINISHER ERROR	The finisher is not operating due to some unspecified reason.	Finisher problem.	<None>/interact
FINISHER LOW	The finisher is low on supplies	Finisher low.	<None>/interact
LOW 8x10	The 8x10 inch film supply magazine is low.	8x10 film supply low.	<None>/interact
LOW 8x10 BLUE	The 8x10 inch blue film supply magazine is low.	8x10 blue film supply low.	<None>/interact
LOW 8x10 CLR	The 8x10 inch clear film supply magazine is low.	8x10 clear film supply low.	<None>/interact
LOW 8x10 PAPR	The 8x10 inch paper supply magazine is low.	8x10 paper supply low.	<None>/interact
LOW 10x12	The 10x12 inch film supply magazine is low.	10x12 film supply low.	<None>/interact
LOW 10x12 BLUE	The 10x12 inch blue film supply magazine is low.	10x12 blue film supply low.	<None>/interact
LOW 10x12 CLR	The 10x12 inch clear film supply magazine is low.	10x12 clear film supply low.	<None>/interact
LOW 10x12 PAPR	The 10x12 inch paper supply magazine is low.	10x12 paper supply low.	<None>/interact
LOW 10x14	The 10x14 inch film supply magazine is low.	10x14 film supply low.	<None>/interact
LOW 10x14 BLUE	The 10x14 inch blue film supply magazine is low.	10x14 blue film supply low.	<None>/interact
LOW 10x14 CLR	The 10x14 inch clear film supply magazine is low.	10x14 clear film supply low.	<None>/interact
LOW 10x14 PAPR	The 10x14 inch paper supply magazine is low.	10x14 paper supply low.	<None>/interact
LOW 11x14	The 11x14 inch film supply magazine is low.	11x14 film supply low.	<None>/interact

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSOP Access /camera symbol
LOW 11x14 BLUE	The 11x14 inch blue film supply magazine is low.	11x14 blue film supply low.	<None>/interact
LOW 11x14 CLR	The 11x14 inch clear film supply magazine is low.	11x14 clear film supply low.	<None>/interact
LOW 11x14 PAPR	The 11x14 inch paper supply magazine is low.	11x14 paper supply low.	<None>/interact
LOW 14x14	The 14x14 inch film supply magazine is low.	14x14 film supply low.	<None>/interact
LOW 14x14 BLUE	The 14x14 inch blue film supply magazine is low.	14x14 blue film supply low.	<None>/interact
LOW 14x14 CLR	The 14x14 inch clear film supply magazine is low.	14x14 clear film supply low.	<None>/interact
LOW 14x14 PAPR	The 14x14 inch paper supply magazine is low.	14x14 paper supply low.	<None>/interact
LOW 14x17	The 14x17 inch film supply magazine is low.	14x17 film supply low.	<None>/interact
LOW 14x17 BLUE	The 14x17 inch blue film supply magazine is low.	14x17 blue film supply low.	<None>/interact
LOW 14x17 CLR	The 14x17 inch clear film supply magazine is low.	14x17 clear film supply low.	<None>/interact
LOW 14x17 PAPR	The 14x17 inch paper supply magazine is low.	14x17 paper supply low.	<None>/interact
LOW 24x24	The 24x24 inch film supply magazine is low.	24x24 film supply low.	<None>/interact
LOW 24x24 BLUE	The 24x24 inch blue film supply magazine is low.	24x24 blue film supply low.	<None>/interact
LOW 24x24 CLR	The 24x24 inch clear film supply magazine is low.	24x24 clear film supply low.	<None>/interact
LOW 24x24 PAPR	The 24x24 inch paper supply magazine is low.	24x24 paper supply low.	<None>/interact
LOW 24x30	The 24x30 inch film supply magazine is low.	24x30 film supply low.	<None>/interact
LOW 24x30 BLUE	The 24x30 inch blue film supply magazine is low.	24x30 blue film supply low.	<None>/interact
LOW 24x30 CLR	The 24x30 inch clear film supply magazine is low.	24x30 clear film supply low.	<None>/interact
LOW 24x30 PAPR	The 24x30 inch paper supply magazine is low.	24x30 paper supply low.	<None>/interact
LOW A4 PAPR	The A4 paper supply magazine is low.	A4 paper supply low.	<None>/interact
LOW A4 TRANS	The A4 transparency supply magazine is low.	A4 transparency supply low.	<None>/interact
NO RECEIVE MGZ	The film receive magazine is not available.	Film receiver not available.	<None>/interact

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
NO RIBBON	The ribbon cartridge needs to be replaced.	Replace ribbon cartridge.	<None>/interact
NO SUPPLY MGZ	The film supply magazine specified for this job is not available.	Film supply not available.	<None>/interact
CHECK PRINTER	The printer is not ready at this time, operator intervention is required to make the printer available.	Check camera.	<None>/interact
CHECK PROC	The processor is not ready at this time, operator intervention is required to make the printer available.	Check processor.	<None>/interact
PRINTER DOWN	The printer is not operating due to some unspecified reason.	Camera down.	<None>/interact
PRINTER INIT	The printer is not ready at this time, it is expected to become available without intervention, For example, it may be in a normal warm-up state.	Camera initializing.	<None>/idle
PRINTER OFFLINE	The printer has been disabled by an operator or service person.	Camera off-line.	<None>/interact
PROC DOWN	The processor is not operating due to some unspecified reason.	Processor down.	<None>/interact
PROC INIT	The processor is not ready at this time, it is expected to become available without intervention. For example, it may be in a normal warm-up state.	Processor initializing.	<None>/idle
PROC OVERFLOW FL	Processor chemicals are approaching the overflow full mark.	Processor chemicals overflow.	<None>/interact
PROC OVERFLOW HI	Processor chemicals have reached the overflow full mark.	Processor chemicals near overflow.	<None>/interact
QUEUED	Print job in Queue	-	<None>/idle
RECEIVER FULL	The Film receive magazine is full.	Receiver full.	<None>/interact
REQ MED NOT INST	The requested film, paper, or other media supply magazine is installed in the printer, but may be available with operator intervention.	Install media supply.	<None>/interact

Printer Status info/Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
REQ MED NOT AVAI	The requested film, paper, or other media requested is not available on this printer.	Media supply not available on this camera. Queue stopped. Change camera.	Queue for this camera will be STOPPED/queue stopped
RIBBON ERROR	There is an unspecified problem with the print ribbon.	Error with print ribbon.	<None>/interact
SUPPLY EMPTY	The printer is out of film.	Camera out of film.	<None>/interact
SUPPLY LOW	The film supply is low.	Film supply low.	<None>/interact
UNKNOWN	There is an unspecified problem.	Unspecified problem with camera.	<None>/interact

Table 53: Printer Status Infos: Additional Agfa printer status infos

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
WARMING UP	Printer is in the warm-up stage. Spooling of print jobs to disk is still possible.	Camera is warming up.	<None>idle
OFFLINE	Printer is switched off-line. Spooling of print jobs to disk is still possible.	Camera is switched off-line.	<None>/interact
NONE	General printer warning, no specific information is available. Spooling of print jobs to disk is still possible.	-	<None>/idle

Table 54: Printer Status Infos: Additional Kodak infos for Pacs Link (formerly Imation cameras)

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
SUPPLY MGZ ERR	The supply magazine has an error.	Film supply has an error.	<None>/interact
-	-	-	-

Table 55: Printer Status Infos: Additional Kodak infos for Kodak 190

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
PRINTER STOPPED	The printer has stopped	Camera has stopped.	<None>/interact
FATAL ERROR	Fatal error.	Fatal error. Queue stopped.	Queue for this camera will be STOPPED/queue stopped

Table 56: Printer Status Infos: Additional Kodak infos for 2180/1120

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
PRINTER NOT RDY	Printer not ready.	Camera not ready.	<None>/interact
CHECK PROCESSOR	Check processor.	Check processor.	<None>/interact
NO TONER	No toner.	No toner.	<None>/interact
FATAL	Fatal error.	Fatal error. Queue stopped.	Queue for this camera will be STOPPED/queue stopped

Table 57: Printer Status Infos: Additional Codonics infos

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
STANDARD	Printer is ready.	Camera is ready.	<None>/Normal
LOAD A-SIZE	Load A-size media.	Load A-size media.	<None>/interact
LOAD A-DVPAPER	Load A-size black and white paper.	Load A-size black and white paper.	<None>/interact
LOAD A-CVPAPER	Load A-size color paper.	Load A-size color paper.	<None>/interact
LOAD A-CVTRANS	Load A-size transparencies.	Load A-size transparencies.	<None>/interact
LOAD A4-SIZE	Load A4-size media.	Load A4-size media.	<None>/interact
LOAD A4-DVPAPER	Load A4-size black and white paper.	Load A4-size black and white paper.	<None>/interact
LOAD A4-CVPAPER	Load A4-size color paper.	Load A4-size color paper.	<None>/interact
LOAD A4-CVTRANS	Load A4-size transparencies.	Load A4-size transparencies.	<None>/interact
LOAD LA-SIZE	Load LA-size media.	Load LA-size media.	<None>/interact

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSOP Access /camera symbol
LOAD LA-DVPAPER	Load LA-size black and white paper.	Load LA-size black and white paper.	<None>/interact
LOAD LA-CVPAPER	Load LA-size color paper.	Load LA-size color paper.	<None>/interact
LOAD LA-CVTRANS	Load LA-size transparencies.	Load LA-size transparencies.	<None>/interact
LOAD LA4-SIZE	Load LA4-size media.	Load LA4-size media.	<None>/interact
LOAD LA4-DVPAPER	Load LA4-size black and white paper.	Load LA4-size black and white paper.	<None>/interact
LOAD LA4-CVPAPER	Load LA4-size color paper.	Load LA4-size color paper.	<None>/interact
LOAD LA4-CVTRANS	Load LA4-size transparencies.	Load LA4-size transparencies.	<None>/interact
LOAD XLA-SIZE	Load XLA-size media.	Load XLA-size media.	<None>/interact
LOAD XLA-DVPAPER	Load XLA-size black and white paper.	Load XLA-size black and white paper.	<None>/interact
LOAD XLA-CVPAPER	Load XLA-size color paper.	Load XLA-size color paper.	<None>/interact
LOAD XLA-CVTRANS	Load XLA-size transparencies.	Load XLA-size transparencies.	<None>/interact
LOAD XLA4-SIZE	Load XLA4-size media.	Load XLA4-size media.	<None>/interact
LOAD XLA4-DVPAPE	Load XLA4-size black and white paper.	Load XLA4-size black and white paper.	<None>/interact
LOAD XLA4-CVPAPE	Load XLA4-size color paper.	Load XLA4-size color paper.	<None>/interact
LOAD XLA4-CVTRAN	Load XLA4-size transparencies.	Load XLA4-size transparencies.	<None>/interact
LOAD XLW-SIZE	Load XLW-size media.	Load XLW-size media.	<None>/interact
LOAD XLW-DVPAPER	Load XLW-size black and white paper.	Load XLW-size black and white paper.	<None>/interact
LOAD XLW-CVPAPER	Load XLW-size color paper.	Load XLW-size color paper.	<None>/interact
LOAD 8X10-SIZE	Load 8x10 media.	Load 8x10 media.	<None>/interact
LOAD 8X10-DVFILM	Load 8x10 black and white film.	Load 8x10 black and white film.	<None>/interact
SUPPLY MISSING	The film supply magazine specified for this job is not available.	Film supply not available.	<None>/interact

Printer Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
RIBBON MISSING	Ribbon is missing.	Ribbon is missing.	<None>/interact
RIBBON EMPTY	Ribbon is empty.	Ribbon is empty.	<None>/interact
TOP COVER OPEN	Top cover of printer is open.	Top cover of camera is open.	<None>/interact

Table 58: Additional DICOM Execution Status Infos

Execution Status info	Description	Message string visible in the Status Bar	Other action for UROSKOP Access /camera symbol
INVALID PAGE DES	The specified page layout cannot be printed or other page description errors have been detected.	Film Job cannot be printed on this camera. Queue stopped. Please redirect film job.	Queue for this camera will be STOPPED/queue stopped
INSUFFIC MEMORY	There is not enough memory available to complete this job.	Not enough memory available in camera. Queue stopped. Please continue queue or change camera.	Queue for this camera will be STOPPED/queue stopped
NONE	General printer warning, no specific information is available. Spooling of print jobs to disk is still possible.	-	<None>/idle

3.6 Modality Worklist AE Specification

The Modality worklist SCU requests that the remote SCP performs a match of all keys specified in the query against the information in its worklist database.

UROSKOP Access DICOM products provide Standard Conformance to the following DICOM

V3.0 SOP Class as an SCU:

Table 59: SOP Classes as an SCU

SOP Class Name	SOP Class UID
Modality Worklist Information Model - FIND	1.2.840.10008.5.1.4.31

3.6.1 Association Establishment Policies

3.6.1.1 General

The configuration of the UROSKOP Access DICOM modality worklist application defines the Application Entity Titles, the port numbers and of course the host name and net address.

3.6.1.2 Number of Associations

The UROSKOP Access DICOM application initiates one/several association(s) at a time, one for each transfer request being processed.

3.6.1.3 Asynchronous Nature

The UROSKOP Access DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

3.6.1.4 Implementation Identifying Information

The UROSKOP Access DICOM software provides a single Implementation Class UID of
1.3.12.2.1107.5.12.3

and an Implementation Version Name of

- “SIEMENS_SWFVB10E”.

3.6.2 Association Initiation Policy

The Modality Worklist SCU establishes an association by using the DICOM association services.

The following DIMSE-C operation is supported as SCU:

- C-FIND

3.6.2.1 Real World Activity

3.6.2.1.1 Associated Real-World Activity

The associated Real-World activity is to initiate query requests to an SCP by using the DICOM Worklist Information Model.

3.6.2.1.2 Proposed Presentation Contexts

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 60: Proposed presentation contexts

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
Modality Worklist Information Model- FIND	1.2.840.10008.5.1.4.31	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

3.6.2.1.3 SOP Specific Conformance Statement

Search Key Attributes of the Worklist C-FIND

The UROSKOP Access DICOM worklist SCU supports worklist queries with all required search keys and several optional search keys.

The following table describes the search keys that the SCU supports for a broad worklist query, which is a query for all tasks scheduled for the own modality or own modality application entity, defined with the following search keys:

Table 61: Search Key Attributes in a broad worklist query

Attribute name	Tag	Matching Key Type	query value
Scheduled Procedure Step			
Scheduled Procedure Step Sequence	(0040,0100)	R	
>Scheduled Station AE Title	(0040,0001)	R	Configurable1: own AET or "*"
>Scheduled Procedure Step Start Date	(0040,0002)	R	Configurable: inserted in UI2 or zero length

1 One and only one of the attributes "Modality" and "AE Title" is set to "*". There is a configuration parameter telling which of them. The other one is always set to the "own" value (i.e. own modality respectively own AE Title).

2 <startDate>-<endDate>

Attribute name	Tag	Matching Key Type	query value
>Scheduled Procedure Step Start Time	(0040,0003)	R	Configurable: inserted in UI3 or today
Modality	(0008,0060)	R	Configurable own modality or “*”

The following table describes the search keys that the SCU supports for a patient based worklist query, which is defined by the following search keys:

Table 62: Search Key Attributes in a patient based worklist query

Attribute name	Tag	Matching Key Type	query value
Scheduled Procedure Step			
Scheduled Procedure Step Sequence	(0040,0100)	R	
>Scheduled Performing Physician's Name	(0040,0006)	R	inserted in UI or zero length
Requested Procedure			
Requested Procedure ID	(0040,1001)	O	inserted in UI or zero length
Imaging Service Request			
Accession Number	(0008,0050)	O	inserted in UI or zero length
Referring Physician's Name	(0008,0090)	O	inserted in UI or zero length
Visit Status			
Current Patient Location	(0038,0300)	O	inserted in UI or zero length
Patient Identification			
Patient's Name	(0010,0010)	R	inserted in UI or zero length
Patient ID	(0010,0020)	R	inserted in UI or zero length

Return Key Attributes used from the Worklist C_FIND_RSP

The UROSKOP Access DICOM worklist SCU supports worklist queries with return key attributes of all types. The following table describes the return keys that the SCU supports. Most attributes can be shown in the User Interface (Patient Registration or Patient Browser), but in Patient Browser it is configurable which of them are shown really.

3<startTime>-<endTime>

Table 63: Modality Worklist C_FIND_RSP Return Key Attributes

Attribute name	Tag	Return Key Type	Displayed in User Interface
SOP Common			
Specific Character Set	(0008,0005)	1C	-
Scheduled Procedure Step			
Scheduled Procedure Step Sequence	(0040,0100)	1	-
>Scheduled Station AE Title	(0040,0001)	1	yes
>Scheduled Procedure Step Start Date	(0040,0002)	1	yes
>Scheduled Procedure Step Start Time	(0040,0003)	1	yes
>Scheduled Procedure Step End Date	(0040,0004)	3	-
>Scheduled Procedure Step End Time	(0040,0005)	3	-
>Modality	(0008,0060)	1	yes
>Scheduled Performing Physician's Name	(0040,0006)	1	yes
>Scheduled Procedure Step Description	(0040,0007)	1C	yes
>Scheduled Station Name	(0040,0010)	2	yes
>Scheduled Procedure Step Location	(0040,0011)	2	yes
>Scheduled Action Item Code Sequence	(0040,0008)	1C	-
>>Code Value	(0008,0100)	1C	yes
>>Coding Scheme Designator	(0008,0102)	1C	yes
>>Coding Scheme Version	(0008,0102)	3	yes
>>Code Meaning	(0008,0104)	3	yes
>Pre-Medication	(0040,0012)	2C	yes
>Scheduled Procedure Step ID	(0040,0009)	1	yes
>Requested Contrast Agent	(0032,1070)	2C	yes
>Scheduled Procedure Step Status	(0040,0020)	3	yes
>Comments on the Scheduled Procedure Step	(0040,0400)	3	-
Requested Procedure			
Requested Procedure ID	(0040,1001)	1	yes
Requested Procedure Description	(0032,1060)	1C	yes
Requested Procedure Code Sequence	(0032,1064)	1C	-
>Code Value	(0008,0100)	1C	yes
>Code Scheme Designator	(0008,0102)	1C	yes
>Code Scheme Version	(0008,0103)	3	yes
>Code Meaning	(0008,0104)	3	yes
Study Instance UID	(0020,000D)	1	-
Referenced Study Sequence	(0008,1110)	2	-
>Referenced SOP Class UID	(0008,1150)	1C	-

Attribute name	Tag	Return Key Type	Displayed in User Interface
>Referenced SOP Instance UID	(0008,1155)	1C	-
Requested Procedure Priority	(0040,1003)	2	yes
Patient Transport Arrangements	(0040,1004)	2	-
Reason for the Requested Procedure	(0040,1002)	3	-
Confidentiality Code	(0040,1008)	3	-
Reporting Priority	(0040,1009)	3	-
Names of Intended Recipients of results	(0040,1010)	3	-
Requested Procedure Comments	(0040,1400)	3	yes
Requested Procedure Location	(0040,1005)	3	-
Imaging Service Request			
ACCESSION Number	(0008,0050)	2	yes
Requesting Physician	(0032,1032)	2	yes
Referring Physician's Name	(0008,0090)	2	yes
Reason for the Imaging Service Request	(0040,2001)	3	-
Imaging Service Request Comments	(0040,2400)	3	yes
Requesting Service	(0032,1033)	3	yes
Issuing Date of Imaging Service Request	(0040,2004)	3	-
Issuing Time of Imaging Service Request	(0040,2005)	3	-
Placer Order Number / Imaging Service Request	(0040,2016)	3	-
Filler Order Number / Imaging Service Request	(0040,2017)	3	-
Order entered by...	(0040,2008)	3	-
Order Enterer's Location	(0040,2009)	3	-
Order Callback Phone Number	(0040,2010)	3	-
Visit Identification			
Admission ID	(0038,0010)	2	yes
Issuer of Admission ID	(0038,0011)	3	-
Visit Status			
Current Patient Location	(0038,0300)	2	yes
Visit Relationship			
Referenced Patient Sequence	(0008,1120)	2	-
>Referenced SOP Class UID	(0008,1150)	2	-
>Referenced SOP Instance UID	(0008,1155)	2	-
Visit Admission			
Institution Name	(0008,0000)	3	yes
Admitting Diagnoses Description	(0008,1080)	3	yes
Patient Identification			
Patient's Name	(0010,0010)	1	yes

Attribute name	Tag	Return Key Type	Displayed in User Interface
Patient ID	(0010,0020)	1	yes
Patient Demographic			
Patients Birth Date	(0010,0030)	2	yes
Patient's Sex	(0010,0040)	2	yes
Patient's Size	(0010,1020)	3	yes
Patient's Weight	(0010,1030)	2	yes
Confidentiality constraint on patient data	(0040,3001)	2	yes
Patient's Address	(0010,1040)	3	yes
Military Rank	(0010,1080)	3	yes
Ethnic Group	(0010,2160)	3	yes
Patient Comments	(0010,4000)	3	yes
Patient Medical			
Patient State	(0038,0500)	2	yes
Pregnancy Status	(0010,21C0)	2	yes
Medical Alerts	(0010,2000)	2	yes
Contrast Allergies	(0010,2110)	2	yes
Special Needs	(0038,0050)	2	yes
Smoking Status	(0010,21A0)	3	yes
Last Menstrual Date	(0010,21D0)	3	yes
Additional Patient History	(0010,21B0)	3	yes

Status Codes of the Worklist C-FIND

The worklist SCU interprets following status codes:

Table 64: C-FIND Response Status

Service Status	Meaning	Status Codes (0000,0900)
Refused	Out of Resources SOP class not supported	A700 0122
Failed	Identifier does not match SOP Class Unable to process	A900 C001
Cancel	Matching terminated due to Cancel request	FE00
Success	Matching is complete - No final Identifier is supplied	0000
Pending	Matches are continuing - Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys. Matches are continuing - Warning that one or more Optional Keys were not supported for existence and/or matching for this identifier	FF00 FF01

3.7 Modality Performed Procedure Step AE Specification

The Modality Performed Procedure Step SCU informs the remote SCP about the performed examinations at the modality.

UROSOP Access DICOM products provide Standard Conformance to the following DICOM V3.0 SOP Class as an SCU:

Table 65: SOP Classes as an SCU

SOP Class Name	SOP Class UID
Modality Performed Procedure Step	1.2.840.10008.3.1.2.3.3

3.7.1 Association Establishment Policies

3.7.1.1 General

The configuration of the UROSOP Access DICOM Performed Procedure Step application defines the Application Entity Titles, the port numbers and of course the host name and net address.

3.7.1.2 Number of Associations

The UROSOP Access DICOM application initiates one/several association(s) at a time, one for each transfer request being processed.

3.7.1.3 Asynchronous Nature

The UROSOP Access DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

3.7.1.4 Implementation Identifying Information

The UROSOP Access DICOM software provides a single Implementation Class UID of

1.3.12.2.1107.5.12.3

and an Implementation Version Name of

• “SIEMENS_SWFVB10E”.

3.7.2 Association Initiation Policy

The Modality Performed Procedure Step SCU establishes an association by using the DICOM association services.

The following DIMSE-N operations are supported as SCU:

- N-CREATE
- N-SET

3.7.2.1 Real World Activity

3.7.2.1.1 Associated Real-World Activity

The associated Real-World activity is to send examination information to a SCP by using the DICOM Modality Performed Procedure Step Service.

3.7.2.1.2 Proposed Presentation Contexts

The UROSKOP Access DICOM application will propose Presentation Contexts as shown in the following table:

Table 66: Proposed presentation contexts

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended
Name	UID	Name List	UID List		Negotiation
Modality Performed Procedure Step	1.2.840.10008.3.1.2.3.3	DICOM Implicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2	SCU	None
		DICOM Explicit VR Little Endian Transfer Syntax	1.2.840.10008.1.2.1		
		DICOM Explicit VR Big Endian Transfer Syntax	1.2.840.10008.1.2.2		

3.7.2.1.3 SOP Specific Conformance Statement

Attributes used for the Performed Procedure Step N-CREATE

The UROSKOP Access DICOM performed procedure step SCU informs the remote SCP when the examination of a scheduled procedure step will be performed. The N-CREATE message is sent when the examination is started. The following table describes the supported attributes for a N-CREATE message.

Table 67: Performed Procedure Step N-CREATE Attributes

Attribute name	Tag	Required Type	Value
SOP Common			
Specific Character Set	(0008,0005)	1C	from MWL or created
Performed Procedure Step Relationship			
Scheduled Step Attribute Sequence	(0040,0270)	1	
>Study Instance UID	(0020,000D)	1	from MWL or created
>Referenced Study Sequence	(0008,1110)	2	from MWL or zero length
>>Referenced SOP Class UID	(0008,1150)	1C	
>>Referenced SOP Instance UID	(0008,1155)	1C	
>ACCESSion Number	(0008,0050)	2	from MWL or user input
>Placer Order Number / Imaging Service Request	(0040,2016)	3	from MWL or zero length
>Filler Order Number / Imaging Service Request	(0040,2017)	3	from MWL or zero length
>Requested Procedure ID	(0040,1001)	2	from MWL or user input
>Requested Procedure Description	(0032,1060)	2	from MWL or zero length
>Scheduled Procedure Step ID	(0040,0009)	2	from MWL or zero length
>Scheduled Procedure Step Description	(0040,0007)	2	from MWL or zero length
>Scheduled Action Item Code Sequence	(0040,0008)	2	from MWL or zero length
>>Code Value	(0008,0100)	1C	
>>Coding Scheme Designator	(0008,0102)	1C	
>>Code Scheme Version	(0008,0103)	3	
>>Code Meaning	(0008,0104)	3	
Patient's Name	(0010,0010)	2	from MWL or user input
Patient ID	(0010,0020)	2	from MWL or user input or created
Patients Birth Date	(0010,0030)	2	from MWL or user input
Patient's Sex	(0010,0040)	2	from MWL or user input
Referenced Patient Sequence	(0008,1120)	2	from MWL or zero length
>Referenced SOP Class UID	(0008,1150)	1C	
>Referenced SOP Instance UID	(0008,1155)	1C	
Performed Procedure Step Information			
Performed Procedure Step ID	(0040,0253)	1	from SPS ID or created
Performed Station AE Title	(0040,0241)	1	own AE Title
Performed Station Name	(0040,0242)	2	own hostname
Performed Location	(0040,0243)	2	from SPS Location or zero length
Performed Procedure Step Start Date	(0040,0244)	1	created

Attribute name	Tag	Required Type	Value
Performed Procedure Step Start Time	(0040,0245)	1	created
Performed Procedure Step Status	(0040,0252)	1	IN PROGRESS
Performed Procedure Step Description	(0040,0254)	2	from SPS Description or zero length
Performed Procedure Type Description	(0040,0255)	2	zero length
Procedure Code Sequence	(0008,1032)	2	from Requested Procedure Code or zero length
>Code Value	(0008,0100)	1C	
>Code Scheme Designator	(0008,0102)	1C	
>Code Scheme Version	(0008,0103)	3	
>Code Meaning	(0008,0104)	3	
Performed Procedure Step End Date	(0040,0250)	2	zero length
Performed Procedure Step End Time	(0040,0251)	2	zero length
Image Acquisition Results			
Modality	(0008,0060)	1	RF
Study ID	(0020,0010)	2	from Requested Procedure ID or created
Performed Action Item Code Sequence	(0040,0260)	2	from Scheduled Action Item Code SQ or zero length
>Code Value	(0008,0100)	1C	
>Code Scheme Designator	(0008,0102)	1C	
>Code Scheme Version	(0008,0103)	3	
>Code Meaning	(0008,0104)	3	
Performed Series Sequence	(0040,0340)	2	
>Performing Physicians's Name	(0008,1050)	2C	from MWL or user input
>Operator's Name	(0008,1070)	2C	user input
>Series Instance UID	(0020,000E)	1C	created
>Series Description	(0008,103E)	2C	zero length
>Retrieve AE Title	(0008,0054)	2C	zero length
>Referenced Image Sequence	(0008,1140)	2C	zero length
>Referenced Standalone SOP Instance Sequence	(0040,0220)	2C	zero length

Status Codes of the Performed Procedure Step N-CREATE

The Performed Procedure Step SCU interprets the following status values:

Table 68: N-SET Response Status

Service Status	Meaning	Status Codes (0000,0900)
Failure	Processing Failure	0110
	No such attribute	0105
	Invalid attribute value	0106
	Duplicate SOP Instance	0111
	No such SOP Instance	0112
	No such SOP class	0118
	Class instance conflict	0119
	Missing attribute	0120
	Missing attribute value	0121
	Resource limitation	0213
Success	Successful Operation	0000

Attributes used for the Performed Procedure Step N-SET

The UROSKOP Access DICOM performed procedure step SCU informs the remote SCP about the performed examination and its status. The N-SET message is only sent once when the examination is finished with status "COMPLETED" or when the examination could not be completed with status "DISCONTINUED". The following table describes the supported attributes for a N-SET message.

Table 69: Performed Procedure Step N-SET Attributes

Attribute name	Tag	Required Type	Value
Performed Procedure Step Information			
Performed Procedure Step Status	(0040,0252)	3	created
Performed Procedure Step Description	(0040,0254)	3	from SPS Description or user input
Performed Procedure Type Description	(0040,0255)	3	user input
Procedure Code Sequence	(0008,1032)	3	from Requested Procedure Code
>Code Value	(0008,0100)	1C	
>Code Scheme Designator	(0008,0102)	1C	
>Code Scheme Version	(0008,0103)	3	
>Code Meaning	(0008,0104)	3	
Performed Procedure Step End Date	(0040,0250)	3	created
Performed Procedure Step End Time	(0040,0251)	3	created
Image Acquisition Results			
Performed Action Item Code Sequence	(0040,0260)	3	from Scheduled Action Item Code SQ or
>Code Value	(0008,0100)	1C	

Attribute name	Tag	Required Type	Value
>Code Scheme Designator	(0008,0102)	1C	
>Code Scheme Version	(0008,0103)	3	
>Code Meaning	(0008,0104)	3	
Performed Series Sequence	(0040,0340)	3	
>Performing Physicians's Name	(0008,1050)	2C	from MWL or user input
>Protocol Name	(0018,1030)	1C	ASPIA
>Operator's Name	(0008,1070)	2C	user input
>Series Instance UID	(0020,000E)	1C	created
>Series Description	(0008,103E)	2C	n.a
>Retrieve AE Title	(0008,0054)	2C	from Storage Commitment RSP or zero length
>Referenced Image Sequence	(0008,1140)	2C	created
>>Referenced SOP Class UID	(0008,1150)	1C	
>>Referenced SOP Instance UID	(0008,1155)	1C	
>Referenced Standalone SOP Instance Sequence	(0040,0220)	2C	zero length
Total Time of Fluoroscopy	(0040,0300)	n.a	
Total Number of Exposures	(0040,0301)	n.a	
Image Area Dose Product	(0018,115E)	n.a	

Status Codes of the Performed Procedure Step N-SET

The Performed Procedure Step SCU interprets the following status values:

Table 70: N-SET Response Status

Service Status	Meaning	Status Codes (0000,0900)
Failure	Processing Failure: Performed Procedure Step Object may no longer be updated No such attribute Invalid attribute value No such SOP Instance Invalid object instance No such SOP class Class instance conflict Missing attribute value Resource limitation	0110 0105 0106 0112 0117 0118 0119 0121 0213
Success	Successful Operation	0000

4 Communication Profiles

4.1 Supported Communication Stacks

The UROSOP Access DICOM application provide DICOM V3.0 TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

4.1.1 OSI Stack

not supported.

4.1.2 TCP/IP Stack

The UROSOP Access DICOM application uses the TCP/IP stack from the Windows NT system. It uses the MergeCOM-3 subroutine library from Merge Technologies Inc. that is based on a TCP/IP socket interface.

4.1.2.1 API

The UROSOP Access DICOM application uses the Merge COM library.

4.1.2.2 Physical Media Support

The UROSOP Access DICOM application is indifferent to the physical medium over which TCP/IP executes.

5 Extensions/Specializations/Privatizations

5.1 Standard Extended/Specialized/Private SOPs

5.1.1 Standard Extensions of all SOP Classes

The following tables list the data dictionary of all DICOM IOD attributes where the DICOM standard definitions are extended:

Table 71: Standard Extensions of all SOP Classes

Attribute Name	Tag	Private Creator	Type	Notes
Image Type	(0008,0008)	-	1	additional Defined Terms: Defined Terms for value 3: OTHER Defined Terms for value 4: CSA 3D EDITOR CSA 3D FLY PATH CSA 3D FLY VRT CSA 3D FUSION CSA AVERAGE CSA BLACK IMAGE CSA RESAMPLED CSA MIP CSA MPR CSA MPR CURVED CSA MPR THICK CSA SSD CSA SUBTRACT CT_SOM4 * ECAT ACF ECAT NORMAL ECAT 3D SINO ECAT 3D SINO FLT SHS *

Attribute Name	Tag	Private Creator	Type	Notes
Patient Position	(0018,5100)	-	2C	additional Defined Terms for the Magnetom Open: HLS HLP FLS FLP HLDL HLDR FLDL FLDR

All SOP classes may contain additional type 3 attributes which DICOM standard defines in a different DICOM IOD or DICOM SOP class (attributes from Normalized SOP classes).

This is the case for example for

- Rescale Slope (0028,1053)
- Rescale Intercept (0028,1052)

which are also used in the MR IOD.

5.1.1.1 Image Type

The Image Type (0008,0008) attribute identifies important image identification characteristics. These characteristics are:

1. Pixel Data Characteristics:

- is the image an ORIGINAL Image; an image whose pixel values are based on original or source data, or
- is the image a DERIVED Image; an image whose pixel values have been derived in some manner from the pixel value of one or more other images.

2. Patient Examination Characteristics:

- is the image a PRIMARY Image; an image created as a direct result of the Patient examination, or is the image a SECONDARY Image; an image created after the initial Patient examination.

3. Modality Specific Characteristics (SOP Specific Characteristics).

4. Implementation specific identifiers; other implementation specific identifiers shall be documented in an implementation's conformance claim.

The Image Type attribute is multi-valued and shall be provided in the following manner:

- o Value 1 shall identify the Pixel Data Characteristics; Enumerated Values for the Pixel Data Characteristics are:

- ORIGINAL = identifies an Original Image
- DERIVED = identifies a Derived Image

- o Value 2 shall identify the Patient Examination Characteristics; Enumerated Values for the Patient Examination Characteristics are:
 - RIMARY = identifies a Primary Image
 - SECONDARY = identifies a Secondary Image
- o Value 3 shall identify any Image IOD specific specialization, the following terms are defined in addition to the DICOM standard definitions:
 - OTHER = is also used for converted non-Axial and non-Localizer CT images
 - MPR = for 3D MPR images
 - PROJECTION IMAGE = for 3D MIP and SSD images

5.1.2 Image Type - value 4

n.a.

5.1.3 Private Elements for Storage SOP Classes

The following private attributes are defined by UROSOP Access DICOM applications.

5.1.3.1 Registry of DICOM Data Elements

Table 72: Private Elements

Tag	Private Owner Code	Name	VR	VM
(0029,xx02)				
(0029,xx08)	SIEMENS CSA HEADER	CSA Image Header Type	CS	1
(0029,xx09)	SIEMENS CSA HEADER	CSA Image Header Version	LO	1
(0029,xx10)	SIEMENS CSA HEADER	CSA Image Header Info	OB	1
(0029,xx18)	SIEMENS CSA HEADER	CSA Series Header Type	CS	1
(0029,xx19)	SIEMENS CSA HEADER	CSA Series Header Version	LO	1
(0029,xx20)	SIEMENS CSA HEADER	CSA Series Header Info	OB	1
(0029,xx08)	SIEMENS CSA NON-IMAGE	CSA Data Type	CS	1
(0029,xx09)	SIEMENS CSA NON-IMAGE	CSA Data Version	LO	1
(0029,xx10)	SIEMENS CSA NON-IMAGE	CSA Data Info	OB	1
(0029,xx08)	SIEMENS MEDCOM HEADER	MedCom Header Type	CS	1
(0029,xx09)	SIEMENS MEDCOM HEADER	MedCom Header Version	LO	1
(0029,xx10)	SIEMENS MEDCOM HEADER	MedCom Header Info	OB	1
(0029,xx20)	SIEMENS MEDCOM HEADER	MedCom History Information	OB	1
(0029,xx31)	SIEMENS MEDCOM HEADER	PMTF Information 1	LO	1
(0029,xx32)	SIEMENS MEDCOM HEADER	PMTF Information 2	UL	1

(0029,xx33)	SIEMENS MEDCOM HEADER	PMTF Information 3	UL	1
(0029,xx34)	SIEMENS MEDCOM HEADER	PMTF Information 4	CS	1
(0029,xx35)	SIEMENS MEDCOM HEADER	PMTF Information 5	UL	1
(0029,xx40)	SIEMENS MEDCOM HEADER	Application Header Sequence	SQ	1
(0029,xx41)	SIEMENS MEDCOM HEADER	Application Header Type	CS	1
(0029,xx42)	SIEMENS MEDCOM HEADER	Application Header ID	LO	1
(0029,xx43)	SIEMENS MEDCOM HEADER	Application Header Version	LO	1
(0029,xx44)	SIEMENS MEDCOM HEADER	Application Header Info	OB	1
(0029,xx50)	SIEMENS MEDCOM HEADER	Workflow Control Flags	LO	8
(0029,xx51)	SIEMENS MEDCOM HEADER	Archive Management Flag Keep Online	CS	1
(0029,xx52)	SIEMENS MEDCOM HEADER	Archive Management Flag Do Not Archive	CS	1
(0029,xx53)	SIEMENS MEDCOM HEADER	Image Location Status	CS	1
(0029,xx54)	SIEMENS MEDCOM HEADER	Estimated Retrieve Time	DS	1
(0029,xx55)	SIEMENS MEDCOM HEADER	Data Size of Retrieved Images	DS	1
(0029,xx08)	SIEMENS MEDCOM OOG	MEDCOM OOG Type	CS	1
(0029,xx09)	SIEMENS MEDCOM OOG	MEDCOM OOG Version	LO	1
(0029,xx10)	SIEMENS MEDCOM OOG	MEDCOM OOG Info	OB	1

Tag	Private Owner Code	Name	VR	VM
(7FE1,xx10)	SIEMENS CSA NON-IMAGE	CSA Data	OB	1

The next subsections will explain in which IODs these private data elements are used.

5.1.3.2 All UROSKOP Access Supported Image SOP Classes

extended Image IOD Module Table

Table 73: CSA Image IOD Modules

IE	Module	Reference	Usage	Note
Patient	Patient	part 3 C.7.1.1	M	
Study	General Study	part 3 C.7.2.1	M	
	Patient Study	part 3 C.7.2.2	U	
Series	General Series	part 3 C.7.3.1	M	
Equipment	General Equipment	part 3 C.7.5.1	U	

Image	General Image	part 3 C.7.6.1	M	depends on the IOD private GG information private UROSKOP Access information if object graphics is attached to image
	Image Pixel	part 3 C.7.6.3	M	
	IOD specific modules	part 3 C.8.<module>	M/U	
	CSA Image Header	5.1.3.2.2	U	
	CSA Series Header	5.1.3.2.3	U	
	MEDCOM Header	5.1.3.2.4	U	
	MEDCOM OOG	5.1.3.2.5	U	
	SOP Common	part 3 C.12.1	M	

5.1.3.2.1 CSA Image Header Module

The table in this section contains private IOD Attributes that describe the CSA Image Header.

Table 74: CSA Image Header Module

Attribute Name	Tag	Private Creator	Type	Notes
CSA Image Header Type	(0029,xx08)	SIEMENS CSA HEADER	1	CSA Image Header identification characteristics. Defined Terms: <NUM 4 = NUMARIS/4 > <SOM 5 = SOMARIS/5 >
CSA Image Header Version	(0029,xx09)	SIEMENS CSA HEADER	3	Version of CSA Image Header Info (0029,xx10) format.
CSA Image Header Info	(0029,xx10)	SIEMENS CSA HEADER	3	Product dependent information.

5.1.3.2.2 CSA Series Header Module

The table in this section contains private IOD Attributes that describe the CSA Series Header.

Table 75: CSA Series Header Module

Attribute Name	Tag	Private Creator	Type	Notes
CSA Series Header Type	(0029,xx18)	SIEMENS CSA HEADER	1	CSA Series Header identification characteristics. Defined Terms: <NUM 4 = NUMARIS/4> <SOM 5 = Somaris/5
CSA Series Header Version	(0029,xx19)	SIEMENS CSA HEADER	3	Version of CSA Series Header Info (0029,xx20) format.

Attribute Name	Tag	Private Creator	Type	Notes
CSA Series Header Info	(0029,xx20)	SIEMENS CSA HEADER	3	Product dependent information.

5.1.3.2.3 MEDCOM Header Module

The table in this section contains private IOD Attributes that describe MEDCOM Header.

Table 76: MEDCOM Header Module

Attribute Name	Tag	Private Creator	Type	Notes
MedCom Header Type	(0029,xx08)	SIEMENS MED-COM HEADER	1C	MedCom Header identification characteristics. Defined Terms: MEDCOM 1 Required if MedCom Header Info (0029,xx10) present.
MedCom Header Version	(0029,xx09)	SIEMENS MED-COM HEADER	2C	Version of MedCom Header Info (0029,xx10) format. Required if MEDCOM Header Info (0029,xx10) present.
MedCom Header Info	(0029,xx10)	SIEMENS MED-COM HEADER	3	Manufacturer model dependent information. The value of the attribute MedCom Header Info (0029,xx10) can be build up in each user defined format.
MedCom History Information	(0029,xx20)	SIEMENS MED-COM HEADER	3	MedCom defined Patient Registration history information. See 5.1.3.2.4.1.
PMTF Information 1	(0029,xx31)	SIEMENS MED-COM HEADER	3	Transformation Information
PMTF Information 2	(0029,xx32)	SIEMENS MED-COM HEADER	3	Transformation Information
PMTF Information 3	(0029,xx33)	SIEMENS MED-COM HEADER	3	Transformation Information
PMTF Information 4	(0029,xx34)	SIEMENS MED-COM HEADER	3	Transformation Information
Application Header Sequence	(0029,xx40)	SIEMENS MED-COM HEADER	3	Sequence of Application Header Items. Zero or more Items shall be included in this sequence. Encoded as a sequence of items.
>Application Header Type	(0029,xx41)	SIEMENS MED-COM HEADER	1C	Application Header identification characteristics. Required if Sequence is sent.

Attribute Name	Tag	Private Creator	Type	Notes
>Application Header ID	(0029,xx42)	SIEMENS MED-COM HEADER	3	Identification of an application header.
>Application Header Version	(0029,xx43)	SIEMENS MED-COM HEADER	3	Version of Application Header Info (0029,xx43) format.
>Application Header Info	(0029,xx44)	SIEMENS MED-COM HEADER	3	Application dependent information.
Workflow Control Flags	(0029,xx50)	SIEMENS MED-COM HEADER	3	Eight free definable flags.
Archive Management Flag Keep Online	(0029,xx51)	SIEMENS MED-COM HEADER	3	Flag to control remote archive management system to keep the image always online (also when already archived). Enumerated Values: 00 = remote control not required, 01 = keep image online.
Archive Management Flag Do Not Archive	(0029,xx52)	SIEMENS MED-COM HEADER	3	Flag to control remote archive management system not to archive the related image. Enumerated Values: 00 = remote control not required, 01 = don't archive image.
Image Location Status	(0029,xx53)	SIEMENS MED-COM HEADER	3	Image location status to control retrieving. Defined Terms: ONLINE = retrieving has to be done as usual, NEARLINE = move request to SCP and delay according to value of Estimated Retrieve Time (0029,xx54), OFFLINE = invoking a retrieve operation initiates an operator request, INVALID = invoking a retrieve operation would always result in an error.
Estimated Retrieve Time	(0029,xx54)	SIEMENS MED-COM HEADER	3	Estimated retrieve time in seconds. A value less than zero (< 0) indicates location is OFFLINE or INVALID.
Data Size of Retrieved Images	(0029,xx55)	SIEMENS MED-COM HEADER	3	Data size of images in MByte.

5.1.3.2.3.1 MEDCOM History Information

The value of the attribute MEDCOM History Information (0029,xx20) is defined in the following way:

Table 77: MEDCOM History Information

Part	Name	Type	Bytes	Notes
header	Identifier	string	32	always "CSA HISTORY"
	Version	string	32	e.g. "V1.10"
> n items	Class Name	string	64	
	Modification String	string	1024	

5.1.3.2.4 MEDCOM OOG Module

The table in this section contains private IOD Attributes that describe MEDCOM Object Oriented Graphics (OOG). This module is used when object graphics is drawn on the image and stores the properties of the graphics objects (Line, Circle, Rectangle, Arrow, and so on). So the graphics objects will remain reanimatable even if such an image is transferred via DICOM C-Store SOP class.

Table 78: MEDCOM OOG Module

Attribute Name	Tag	Private Creator	Type	Notes
MedCom OOG Type	(0029,xx08)	SIEMENS MED-COM OOG	1	MEDCOM Object Oriented Graphics (OOG) identification characteristics. Defined Terms: MEDCOM OOG 1
MedCom OOG Version	(0029,xx09)	SIEMENS MED-COM OOG	3	Version of MEDCOM OOG Info (0029,xx10) format.
MedCom OOG Info	(0029,xx10)	SIEMENS MED-COM OOG	3	MEDCOM Object Oriented Graphics (OOG) data.

5.1.3.2.5 Edge Enhancement Module

Table 79: MEDCOM OOG Module

Attribute Name	Tag	Private Creator	Type	Notes
Standard Edge Enhancement Sequence Standard formula according	(0029,xx00)	CARDIO-D.R. 1.0	3	Standard formula according Dynaview Extentions (one or more items possible)
>Convolution Kernel Size	(0029,xx01)	CARDIO-D.R. 1.0	1C	x-/y-size value pair. Each value shall be greater or equal to 3. Required if Sequence is present
>Convolution Kernel	(0029,xx02)	Coefficients CARDIO-D.R. 1.0	1C	Row-by-row list of the kernel Coefficients. Required if Sequence is present

Attribute Name	Tag	Private Creator	Type	Notes
>Edge Enhancement Gain	(0029,xx03)	CARDIO-D.R. 1.0	1C	Applied Filter gain Factor. Range is from 0 to 100 Percent. Required if Sequence is present

The graphics objects are also stored in one Image overlay plane for compatibility with other products which don't support the MedCom OOG module. Any system which does not support this MedCom OOG module has to remove these private attributes when modifying the image overlay data.

6 Media Storage

6.1 Introduction

6.1.1 Purpose

This DICOM Conformance Statement is written according to part PS 3.2 of [1]. The applications described in this conformance statement are the SIEMENS UROSKOP Access DICOM off-line media applications. The UROSKOP Access DICOM off-line media storage service implementation acts as FSC, FSU and/or FSR for the specified application profiles and the related SOP

Class instances.

6.1.2 Scope

This DICOM Conformance Statement refers to SIEMENS <xxx> product. The following table relates software names to SIEMENS UROSKOP Access products.

Table 80: Siemens UROSKOP Access DICOM Products

Software Name	Siemens Product
VX43A_UROSKOP_ACCESS_VA14	UROSKOP Access

6.1.3 Definitions, Abbreviations

6.1.3.1 Definitions

DICOM Digital Imaging and Communications in Medicine

DIMSE DICOM Message Service Element

DIMSE-C DICOM Message Service Element with Composite information objects

6.1.3.2 Abbreviations

ACR American College of Radiology

AE DICOM Application Entity

ASCII American Standard Code for Information Interchange

DB Database

DCS DICOM Conformance Statement

FSC File Set Creator
FSR File Set Reader
FSU File Set Updater
IOD DICOM Information Object Definition
ISO International Standard Organization
R Required Key Attribute
NEMA National Electrical Manufacturers Association
O Optional Key Attribute
PDU DICOM Protocol Data Unit
RWA Real-World Activity
U Unique Key Attribute

6.1.4 References

[2] Digital Imaging and Communications in Medicine (DICOM) 3.0, NEMA PS 3.10-12, 1998


6.1.5 Connectivity and Interoperability

The implementation of the Siemens DICOM interface has been carefully tested to assure correspondence with this Conformance Statement. But the Conformance Statement and the DICOM standard does not guarantee interoperability of Siemens modalities and modalities of other vendors. The user must compare the relevant Conformance Statements and if a successful interconnection should be possible, the user is responsible to specify an appropriate test suite and to validate the interoperability, which is required. A network environment may need additional functions out of the scope of DICOM.

6.2 Implementation Model

6.2.1 Application Data Flow Model

Figure 11: Connectivity

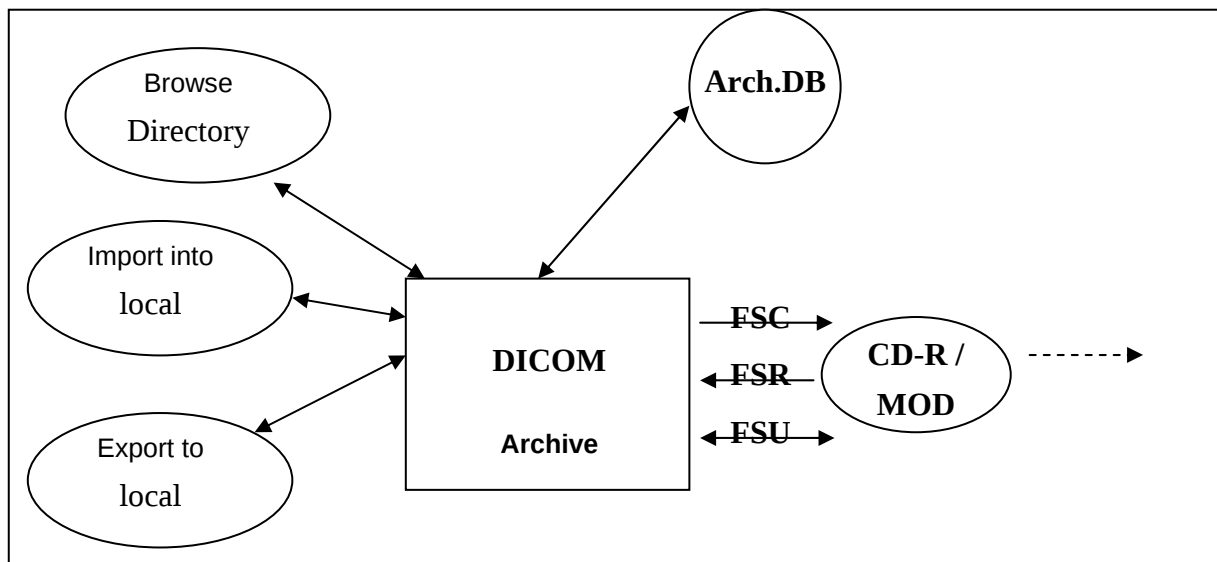

The DICOM archive application will serve as an interface to device. It serves interfaces to include the off-line media directory into the browser and to copy SOP instances to a medium or retrieve SOP Instances from medium into local storage. The DICOM Archive application will support CD-R and MOD media (Table 81: Application Profiles Supported). The FSU role will update new SOP Instances only to media with pre-existing File-sets conforming to the Application Profiles supported. The contents of the DICOMDIR will be temporarily stored in Archive-Database.

6.2.2 Functional Definitions of AEs

The UROSKOP Access DICOM offline media storage application consists of the DICOM Archive application entity serving all interfaces to ACCESS offline media. The DICOM Archive application is capable of

- creating a new File-set onto an unwritten medium (Export to...).
- updating an existing File-set by writing new SOP Instances onto the medium (Export to...).
- importing SOP Instances from the medium onto local storage
- reading the File-sets DICOMDIR information into temporary database and pass it to display applications.

6.2.3 Sequencing of Real World Activities

The DICOM Archive application will not perform updates before the Directory information of the DICOMDIR is completely read.

6.2.4 File Meta Information Options

The Implementation Class UID is:

1.3.12.2.1107.5.12.3

and an Implementation Version Name of

• “SIEMENS_SWFVB10E”.

6.3 AE Specification

6.3.1 DICOM Archive Specification

The DICOM Archive provides Standard conformance to Media Storage Service Class (Interchange Option). In addition Augmented conformance is provided to store extra data attributes important for the full feature support of the UROSKOP Access product SW. Details are listed in following Table:

Table 81: Application Profiles Supported

Application Profiles Supported	Real-World Activity	Role	SC Option
STD-GEN-CD	Browse Directory Information	FSR	Interchange
STD-CTMR-CD	Import into local Storage	FSR	Interchange
STD-XABC-CD			
STD-XA1K-CD			
TD-US-zz-yF-xxxxxx ^a			
STD-WVFM-GEN-FD			

^a – All combinations of the following values for zz, yF and xxxxxx are supported: 'yF' can take two values: SF for Single Frame and MF for Multi Frame. 'zz' can take three values: ID (Image Display), SC(Spatial Calibration) and CC (Combined Calibration) xxxxxx can take 8 values: FLOP and CDR

6.3.1.1 File Meta Information for the Application Entity

The Source Application Entity Title is set by configuration. See Chapter "Configuration" for details.

6.3.1.2 Real-World Activities for this Application Entity

6.3.1.2.1 Real-World Activity: Browse Directory Information

The DICOM Archive application acts as FSR using the interchange option when requested to read the media directory.

The DICOM archive application will read the DICOMDIR and insert those directory entries, that are valid for the application profiles supported, into a local database. The database can then be used for browsing media contents.

Note:

IconImageSQ is also supported in DICOMDIR. But only those Icon Images with Bits Allocated (0028,0100) equal to 8 and size of 64x64 or 128x128 pixels are imported into database and are visible in the Browser.

6.3.1.2.1.1 Application Profiles for the RWA: Browse Directory Information

Table 81: Application Profiles Supported for the Application Profiles listed that invoke this Application Entity for the Browse Directory Information RWA.

6.3.1.2.2 Real-World Activity: Import into Local Storage

The DICOM Archive application acts as FSR using the interchange option when requested to read SOP Instances from the medium into the local storage. The SOP Instance selected from the media directory will be copied into the local storage. Only SOP Instances, that are supported, can be retrieved from media storage.

6.3.1.2.2.1 Application Profiles for the RWA: Import into Local Storage

for the Application Profiles listed that invoke this Application Entity for the Copy to Local Storage RWA.

6.3.1.2.3 Real-World Activity: Export to local Archive Media

The DICOM Archive application acts as FSU (for media with existing DICOM file-set) or FSC.

(media not initialized) using the interchange option when requested to copy SOP Instances from the local storage to local Archive medium. The DICOM Archive application will receive a list of SOP Instances to be copied to the local archive medium. According to the state of the medium inserted (new medium, Medium with DI-COM file-set) the SOP Instances are either updated or created on the media. Only valid SOP Instances are accepted. The DICOM archive application will not close the CD-R medium.

6.3.1.2.3.1 Application Profiles for the RWA: Export to local Archive Media

Table 81: Application Profiles Supported for the Application Profiles listed that invoke this Application Entity for the Copy to local Archive RWA.

Note:

If the image to be archived also has an Icon Image in the database then there will be a Icon Image SQ be generated in DICOMDIR file for this image. The Icon Image SQ will contain the following attributes:

- Samples Per Pixel (0028,0002) = 1
- Photometric Interpretation (0028,0004) = "MONOCHROME1/2"
- Rows (0028,0010), Columns (0028,0011)
- = 128,128 for XA IOD images
- = 64,64 for other images
- Bits Allocated (0028,0100) = 8
- Bits Stored (0028,0101) = 8
- High Bit (0028,0102) = 7
- Pixel Representation (0028,0103) = 0 (unsigned int)
- Planar Configuration (0028, 0006) is not set
- Pixel Aspect Ration (0028,0034) is not set (aspect ratio is 1/1)

6.3.1.3 Application profiles

6.3.1.3.1 DICOMDIR Keys

The DICOMDIR file will contain the following attributes for the levels Patient - Study - Series- Image/Curve (valid for all Application profiles described in this section):

Table 82: Tags in DICOMDIR file

Attribute	Name	Tag	Type Notes
File-Set identification			
File-set ID	(0004,1130)	2	volume label of media
Directory information			
Record of the Root Directory Entry	(0004,1200)	1	
Offset of the Last Directory Record of the Root Directory Entity	(0004,1202)	1	
File-set Consistency Flag)	(0004,1212)	1	0000H
Directory Record Sequence	(0004,1220)	2	
> Offset of the Next Directory Record	(0004,1400)	1C	
> Record In-use flag	(0004,1410)	1C	FFFFH

> Offset of Referenced Lower-Level Directory Entity	(0004,1420)	1C	
> Directory Record Type	(0004,1430)	1C	PATIENT, STUDY, SERIES, IMAGE, CURVE, PRIVATE
> Referenced File ID	(0004,1500)	1C	contains the filename on media for the Directory Records of Type IMAGE and PRIVATE
> Referenced SOP Class UID in File	(0004,1510)	1C	for the Directory Records of Type IMAGE and PRIVATE
> Referenced SOP Instance UID in File	(0004,1511)	1C	for the Directory Records of Type IMAGE and PRIVATE
> Referenced Transfer Syntax UID in File	(0004,1512)	1C	for the Directory Records of Type IMAGE and PRIVATE
> Record Selection Keys	see below		
Patient Keys			
Specific Character Set	(0008,0005)	1C	
Patient's Name	(0010,0010)	2	
Patient ID	(0010,0020)	1	
Date Of Birth	(0010,0030)	3	Type 2 in STD-XA* profiles
Patient's Sex	(0010,0040)	3	Type 2 in STD-XA* profiles
Study Keys			
Specific Character Set	(0008,0005)	1C	
Study Date	(0008,0020)	1	
Study Time	(0008,0030)	1	
Accession Number	(0008,0050)	2	
Study Description	(0008,1030)	2	
Study Instance UID	(0020,000D)	1C	
Study ID	(0020,0010)	1	Will be generated automatically, if not present. Value = "-"
Series Keys			Directory Record Type SERIES
Specific Character Set	(0008,0005)	1C	
Series Date	(0008,0021)	3	
Series Time	(0008,0031)	3	
Modality	(0008,0060)	1	
Institution name	(0008,0080)	3	Type 2 in STD-XA* profiles
Institution Address	(0008,0081)	3	Type 2 in STD-XA* profiles
Series Description	(0008,103E)	3	
Performing Physician	(0008,1050)	3	Type 2 in STD-XA* profiles
Series Instance UID	(0020,000E)	1	
Series Instance UID	(0020,000E)	1	

Image Keys			Directory Record Type IMAGE
Specific Character Set	(0008,0005)	1C	
Image Type	(0008,0008)	3	identification characteristics Type 1 in STD-XA* profiles
SOP Class UID	(0008,0016)	3	
SOP Instance UID	(0008,0018)	3	
Image Date	(0008,0023)	3	
Image Time	(0008,0033)	3	
Referenced Image Sequence	(0008,1140)	3	Type 1C in STD-CTMR profile, required if present in image
> Referenced SOP Class UID	(0008,1150)		
> Referenced SOP Instance UID	(0008,1155)		
Image Number	(0020,0013)	1	
Image Position (Patient)	(0020,0032)	3	Type 1C in STD-CTMR profile, required if present in image
Image Orientation (Patient)	(0020,0037)	3	Type 1C in STD-CTMR profile, required if present in image
Frame Of Reference UID	(0020,0052)	3	Type 1C in STD-CTMR profile, required if present in image
Rows	(0028,0010)	3	
Rows	(0028,0010)	3	
Pixel Spacing	(0028,0030)	3	Type 1C in STD-CTMR profile, required if present in image
Calibration Image	(0050,0004)	3	Type 2 in STD-XA* profiles, for XA IOD
Icon Image Sequence	(0088,0200)	3	required for XA Application profiles, optional for the others
> Samples per Pixel	(0028,0002)	1	
> Photometric Interpretation	(0028,0004)		MONOCHROME1 MONOCHROME2
> Rows	(0028,0010)		128 for XA IOD, 64 otherwise Type 1 in STD-CTMR profile
> Columns	(0028,0011)		128 for XA IOD, 64 otherwise Type 1 in STD-CTMR profile
> Bits Allocated	(0028,0100)		8
> Bits Stored	(0028,0101)		8
> High Bit	(0028,0102)		7
> Pixel Representation	(0028,0103)		0 (unsigned)
> Pixel Data	(7FE0,0010)		Icon Image pixel data

See also Table 92: DICOMDIR keys for CsaNonImage for the DICOMDIR attributes set for CsaNonImage IOD.

6.3.1.3.2 STD-GEN-CD

For media conforming to the STD-GEN-CD Profile the following SOP classes will be support-ed as an FSR, FSC,FSU

Table 83: SOP Classes conforming STD-GEN-CD

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	M	M	O
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	O	M	O
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	M	M	O
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	O	M	O
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	M	M	O
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
CR Image	1.2.840.10008.5.1.4.1.1.1	Explicit VR Little Endian	M	M	O

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
		Uncompressed 1.2.840.10008.1.2.1			
CR Image	1.2.840.10008.5.1.4.1.1.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
CR Image	1.2.840.10008.5.1.4.1.1.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
CR Image	1.2.840.10008.5.1.4.1.1.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
CT image	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	M	M	O
CT Image	1.2.840.10008.5.1.4.1.1.2	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
CT Image	1.2.840.10008.5.1.4.1.1.2	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
CT Image	1.2.840.10008.5.1.4.1.1.2	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
MR Image	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	M	M	O
MR Image	1.2.840.10008.5.1.4.1.1.4	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
MR Image	1.2.840.10008.5.1.4.1.1.4	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
MR Image	1.2.840.10008.5.1.4.1.1.4	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	M	M	O
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Big Endian Uncompressed	O	M	O

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
		1.2.840.10008.1.2.2			
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	M	M	O
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
NM Image	1.2.840.10008.5.1.4.1.1.20	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	M	M	O
NM Image	1.2.840.10008.5.1.4.1.1.20	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	O	M	O
NM Image	1.2.840.10008.5.1.4.1.1.20	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O
NM Image	1.2.840.10008.5.1.4.1.1.20	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O

FSC, FSR, FSU – denote the requirements for those roles

O – Optional

M - Mandatory

6.3.1.3.3 STD-CTMR-xxxx

Table 84: SOP Classes STD-CTMR-xxxx

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
CT Image	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
CT Image	1.2.840.10008.5.1.4.1.1.2	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
MR Image	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
Copyright Siemens AG 2001. All rights reserved. For internal use only. MR Image	1.2.840.10008.5.1.4.1.1.4	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
SC Image (grayscale)	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
SC Image (grayscale)	1.2.840.10008.5.1.4.1.1.7	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
SC Image (palette color)	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes

Detached Patient Management is not supported for import and therefore no precedence of values from those Instances can be supported.

6.3.1.3.4 STD-XABC-CD

For media conforming to the STD-XABC-CD Profile the following SOP classes will be supported

as an FSR,FSC,FSU

Table 85: STD-XABC-CD Supported SOP Classes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
XA Image	1.2.840.10008.5.1.4.1.1.12.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes

Detached Patient Management is not supported for import and therefore no precedence of values from those Instances can be supported.

The FSC will create only images with Rows = Columns = 512 on the media.

6.3.1.3.5 STD-XA1K-CD

For media conforming to the STD-XABC-CD Profile the following SOP classes will be supported as an FSR,FSC,FSU

Table 86: STD-XA1K-CD Supported SOP Classes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
XA Image	1.2.840.10008.5.1.4.1.1.12.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
SC Image	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1			

Standalone IOD's (Standalone Overlay, Standalone Curve, etc.) are not supported by either FSR/ FSC/FSU. Detached Patient Management is not supported for import and therefore no precedence of values from those Instances can be supported.

6.3.1.3.6 STD-US-zz-yF-xxx(Single and multiframe)

For media conforming to the STD-US-zz-yF-FLOP, STD-US-zz-yF-MOD128, STD-US-zz--yF-MOD230, STD-US-zz-yF-MOD540, STD-US-zz-yF-MOD650, STD-US-zz-yF-MOD12, STD-US-zz-yF-MOD23, STD-US-zz-yF-CDR Profiles the following SOP classes and transfer

syntaxes will be supported as an FSR, FSC,FSU

Note:

- 'yF' in the application profile name must be replaced by SF for Single Frame and MF for Multi Frame
- 'zz' in the application profile name must be replaced by any one of ID, SC, CC (Image Display, Spatial Calibration, Combined Calibration)

Table 87: STD-US-ID-yF-xxx Supported SOP Classes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
Ultrasound Multiframe Image	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes

Only the following Photometric Interpretations are supported by FSR/FSC/FSU:

- MONOCHROME1/2
- PALETTE COLOR
- RGB

This restriction also applies for FSR.

6.4 Augmented and Private Profiles

6.4.1 Augmented Application Profiles

When configuring a compressed Transfer Syntax the STD-CTMR and STD-GEN application profile classes will be extended to store instances of the following SOP classes in compressed format:

Table 88: Augmented Application profiles, Activities, and Roles for DICOM Archive

Application Profiles Supported	Real World Activity	Role	SC Option
AUG-GEN-CD	Browse Directory Information	FSR	Interchange
AUG-CTMR-MOD650	Import into local Storage	FSR	Interchange
AUG-CTMR-MOD12	Export to local archive media	FSC,FSU	Interchange
AUG-CTMR-MOD23			
AUG-CTMR-CD			

6.4.1.1 AUG-GEN-CD, AUG-CTMR-xxxx

For media conforming to the AUG-GEN-CD or AUG-CTMR-MOD650, AUG-CTMR-MOD12, AUG-CTMR-MOD23, AUG-CTMR-CD Profile the following SOP classes will be supported as an FSR, FSC,FSU

Table 89: AUG-GEN-CD, AUG-CTMR-xxxx Supported SOP Classes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	M	M	O
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	O	M	O
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	O	M	O

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
X-Ray Angiographic Image	1.2.840.10008.5.1.4.1.1.12.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	O	O	O
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	M	M	O
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	No	Yes	no
X-Ray Radiofluoroscopic Image	1.2.840.10008.5.1.4.1.1.12.2	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	No	Yes	no
Secondary Capture Image	1.2.840.10008.5.1.4.1.1.7	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes
CR Image	1.2.840.10008.5.1.4.1.1.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
CR Image	1.2.840.10008.5.1.4.1.1.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
CR Image	1.2.840.10008.5.1.4.1.1.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	No	Yes	no
CR Image	1.2.840.10008.5.1.4.1.1.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes
CT image	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
CT Image	1.2.840.10008.5.1.4.1.1.2	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
CT Image	1.2.840.10008.5.1.4.1.1.2	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	no	yes	no
CT Image	1.2.840.10008.5.1.4.1.1.2	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes
MR Image	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
MR Image	1.2.840.10008.5.1.4.1.1.4	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
MR Image	1.2.840.10008.5.1.4.1.1.4	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	no	yes	no
MR Image	1.2.840.10008.5.1.4.1.1.4	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	no	yes	no
Ultrasound Image	1.2.840.10008.5.1.4.1.1.6.1	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
Ultrasound Multi-frame Image	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	no	yes	no
Ultrasound Multi-frame	1.2.840.10008.5.1.4.1.1.3.1	JPEG Lossy (baseline or	yes	yes	yes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
Image		extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51			
NM Image	1.2.840.10008.5.1.4.1.1.20	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes
NM Image	1.2.840.10008.5.1.4.1.1.20	JPEG Lossless Process 14 (selection value 1) 1.2.840.10008.1.2.4.70	yes	yes	yes
NM Image	1.2.840.10008.5.1.4.1.1.20	Explicit VR Big Endian Uncompressed 1.2.840.10008.1.2.2	no	yes	no
NM Image	1.2.840.10008.5.1.4.1.1.20	JPEG Lossy (baseline or extended) 1.2.840.10008.1.2.4.50 1.2.840.10008.1.2.4.51	yes	yes	yes

6.4.2 Private Application Profiles

The following Private Application Profiles supported to store private objects in addition to the Standard SOP classes allowed for the corresponding Standard Application Profile

Table 90: Private Application profiles, Activities, and Roles for DICOM Archive

Application Profiles Supported	Real World Activity	Role	SC Option
PRI-GEN-CD	Browse Directory Information	FSR	Interchange
PRI-CTMR-MOD650	Import into local Storage	FSR	Interchange
PRI-CTMR-MOD12	Export to local archive media	FSC,FSU	Interchange
PRI-CTMR-MOD23			
PRI-CTMR-CD			

6.4.2.1 PRI-GEN-CD, PRI-CTMR-xxxx

For media conforming to the PRI-GEN-CD or PRI-CTMR-MOD650, PRI-CTMR-MOD12, PRI-CTMR-MOD23, PRI-CTMR-CD Profile the following SOP classes will be supported as an FSR, FSC,FSU in addition to the Standard SOP classes allowed for the corresponding Standard Application Profiles.

Table 91: PRI-GEN-CD, PRI-CTMR-xxxx Supported SOP Classes

Information Object Definition	SOP Class UID	Transfer Syntax UID	FSC	FSR	FSU
CsaNonImage	1.3.12.2.1107.5.9.1	Explicit VR Little Endian Uncompressed 1.2.840.10008.1.2.1	yes	yes	yes

The DICOMDIR file will contain the Directory record as described Table 82: Tags in DICOMDIR file but with the following Private Keys instead of the Image Keys:

Table 92: DICOMDIR keys for CsaNonImage

Attribute Name	Tag	Private Creator	Type	Notes
Directory Record Type -	(0004,1430)		1	PRIVATE
Private Record UID -	(0004,1432)		1	1.3.12.2.1107.5.9.1
Private keys				
SOP Class UID -	(0008,0016)		3	1.3.12.2.1107.5.9.1
SOP Instance UID	(0008,0018)		3	
Image Type -	(0008,0008)		3	identification characteristics
Acquisition Date -	(0008,0022)		3	The date the acquisition of data that resulted in this data set started.
Acquisition Time -	(0008,0032)		3	The time the acquisition of data that resulted in this data set started.
Acquisition Number -	(0020,0012)		3	A number identifying the single continuous gathering of data over a period of time which resulted in this data set.
CSA Data Type	(0029,xx08)	SIEMENS CSA NON-IMAGE	1.	CSA Data identification characteristics
CSA Data Version	(0029,xx09)	SIEMENS CSA NON-IMAGE	3	Version of CSA Non-Image Data

No Icon Image SQ will be stored for CsaNonImage objects.

6.5 Extensions, Specializations and Privatizations of SOP Classes and Transfer Syntaxes

n.a.

7 Configuration

7.1 AE Title / Presentation Address Mapping

To ensure unique identification the hostname should be part of the AE Titles, abbreviated AET's. The string can be up to 16 characters long and must not contain any extended characters, only 7 bit ASCII characters (excluding control characters) are allowed according to the DICOM standard. An example name is HRI_station4.

Local AE Titels and Presentation Addresses

The local AET's can be configured using the Service application. The following AETs can be entered:

- One common AET for Storage AE, Storage Commitment AE and Query/Retrieve AE.
- One AET for Modality Worklist AE.
- One AET for Print AE

Storage and Query/Retrieve SCP listen on port 104

Remote AE Titles and Presentation Addresses

For remote AET's, host names, IP addresses and port numbers can be configured using the Service application. For each AET a list of supported services can also be configured.

7.1.1 Storage, Storage Commitment and Query Retrieve

The Service application can be used to set the AET's, port numbers, host names, IP addresses and capabilities for the remote nodes' (SCP's). The user can select transfer syntaxes, compression types and query models for each SCP separately.

Additional configurable parameters for Storage Commitment are:

When acting as SCU:

- flag to indicate whether the association will be kept open to receive the response or to close the association and be prepared to receive the response on another association.
- timeout which defines how long the association of N-ACTION is kept to receive a N-EVENT-REPORT request on the same association.
- timeout for N-EVENT-REPORT (applicability of transaction UID) (default 1 h)
- a quality factor which determines the proposed transfer syntax in case that an user has initiated the C-STORE. By convention, 0 means: Only Uncompressed Transfer Syntax(es) are proposed, 100 means: Lossless Transfer Syntax is proposed, and any other value between 1 and 99 means that an JPEG Lossy Transfer Syntax is proposed. One Uncompressed Transfer Syntax will be proposed in any case. This parameter is general for all destination nodes.
- a "compression type supported" which determines the proposed transfer syntax in case that the C-STORE was initiated as a suboperation of an incoming C-MOVE-RQ. By

convention, 0 means: Only Uncompressed Transfer Syntax(es) are proposed, 1 means: Lossless Transfer Syntax is proposed, and 2 means that an JPEG Lossy Transfer Syntax is proposed. One uncompressed transfer syntax will be proposed in any case. This parameter can be set for each configured destination node.

When acting as SCP:

- flag to indicate if an archive system is installed

7.1.2 Print

The Service application can be used to configure the SCP.

AET, host name, IP address and port number can be set.

7.1.3 Modality Worklist

The Service application can be used to set the AETs, port numbers, host names, IP addresses, capabilities and time-outs for the remote nodes' (SCP's)

Additional configurable parameters for Modality Worklist Query are:

- Query Waiting time - the time to wait for the C-FIND-RSP after sending the C-FIND-RQ (default 20 sec.)
- Max Query Match Number - the maximum number of entries accepted in one worklist (default is 200)
- Query Interval: the time between two C-FIND-RQ to the Hospital Information system (default is 60 min.)
- Broad Worklist Query behavior: two values are defined:
Set the AE Title search attribute to the own AE Title, and the Modality search attribute to "*".
Set the Modality search attribute to the own modality and the AE Title search attribute to "*".

7.2 Default Parameters

- maximal PDU size is set to 28672 Bytes
- time-out for accepting/rejecting an association request: 60 s
- time-out for responding to an association open/close request: 60 s
- time-out for accepting a message over network: 60 s
- time-out for waiting for data between TCP/IP-packets: 60 s

The Time-outs for waiting for a Request/Response message from the remote node are as following:

- for Storage SCP/SCU: 600 s
- for Storage Commitment SCU:
time-out for Response to N-ACTION: 600 s
time-out for N-EVENT-REPORT: configurable,
- for Query/Retrieve SCP/SCU: 600 s
- for Modality Worklist SCU: configurable, for Print Management SCU:
time-out for Response to N-SET-RQ: 240 s
time-out for Response to other Requests: 60 s

8 APPENDIX

8.1 Attributes for the generated XRF SOP

Tag	Type		Fluoro MF	FL_ATB	DR_Single	DR_MF
0002,0000 Group Length	UL	1	192	192	192	192
0002,0001 File Meta Information Version	OB	1	00 01	00 01	00 01	00 01
0002,0002 Media Storage SOP Class UID	UI	1	1.2.840.10008.5.1.4.1.1.1.2.2	1.2.840.10008.5.1.4.1.1.1.2.2	1.2.840.10008.5.1.4.1.1.1.2.2	1.2.840.10008.5.1.4.1.1.1.2.2
0002,0003 Media Storage SOP Instance UID	UI	1				
0002,0010 Transfer Syntax UID	UI	1	1.2.840.10008.1.2.1	1.2.840.10008.1.2.1	1.2.840.10008.1.2.1	1.2.840.10008.1.2.1
0002,0012 Implementation Class UID	UI	1	1.3.12.2.1107.5.9.200001.01	1.3.12.2.1107.5.9.200001.01	1.3.12.2.1107.5.9.200001.01	1.3.12.2.1107.5.9.200001.01
0002,0013 Implementation Version Name	SH	1	*)	*)	*)	*)
0008,0005 Specific Character Set	CS	1-n	ISO_IR 100	ISO_IR 100	ISO_IR 100	ISO_IR 100
0008,0008 Image Type	CS	1-n	ORIGINAL\PRIMARY	ORIGINAL\PRIMARY	ORIGINAL\PRIMARY	ORIGINAL\PRIMARY
0008,0016 SOP Class UID	UI	1	1.2.840.10008.5.1.4.1.1.1.2.2	1.2.840.10008.5.1.4.1.1.1.2.2	1.2.840.10008.5.1.4.1.1.1.2.2	1.2.840.10008.5.1.4.1.1.1.2.2
0008,0018 SOP Instance UID	UI	1				
0008,0020 Study Date	DA	1	20021212	20021212	20021212	20021212
0008,0021 Series Date	DA	1	20021212	20021212	20021212	20021212
0008,0022 Acquisition Date	DA	1	20021212	20021212	20021212	20021212
0008,0023 Content Date	DA	1	20021212	20021212		20021212
0008,0030 Study Time	TM	1	81205.9	81205.9	81205.9	81205.9
0008,0031 Series Time	TM	1	81621.508	81253.038	81234.131	81318.805
0008,0032 Acquisition Time	TM	1	81617.642	81250.094	81236.625	81312.286
0008,0033 Content Time	TM	1	81621.518	81253.038		81318.815
0008,0050 ACCESSion Number	SH	1				
0008,0060 Modality	CS	1	RF	RF	RF	RF
0008,0070 Manufacturer	LO	1	SIEMENS	SIEMENS	SIEMENS	SIEMENS
0008,0080 Institution Name	LO	1				
0008,0090 Referring Physician's Name	PN	1				
0008,1030 Study Description	LO	1				
0008,103E Series Description	LO	1	CFC_imatestflu	CFC_FL_Pediatry	EXP_DR_Pediatry	EXP_DR_Retro_Pyelogr_1
0008,1090 Manufacturer's	LO	1	UROSKOP ACCESS	UROSKOP ACCESS	UROSKOP ACCESS	UROSKOP ACCESS

Model Name						
0010,0010 Patient's Name	PN	1				
0010,0020 Patient ID	LO	1				
0010,0030 Patient's Birth Date	DA	1				
0010,0040 Patient's Sex	CS	1				
0010,1010 Patient's Age	AS	1				
0018,0060 KVP	DS	1				
0018,1000 Device Serial Number	LO	1				
0018,1020 Software Versions	LO	1-n	*)	*)	*)	*)
0018,1030 Protocol Name	LO	1	*)	*)	*)	*)
0018,1063 Frame Time	DS	1	40			250
0018,1147 Field of View Shape	CS	1	ROUND	ROUND	ROUND	ROUND
0018,1152 Exposure	IS	1				
0018,1150 Exposure Time	IS	1				
0018,1151 X-ray Tube Current	IS	1				
0018,1155 Radiation Setting	CS	1	SC	SC	GR	GR
0018,115A Radiation Mode	CS	1	CONTINUOUS	CONTINUOUS	CONTINUOUS	PULSED
0018,115E Image Area Dose Product	DS	1	0.02	0.02	0	0.72
0018,1166 Grid	CS	1-n	IN	IN	IN	IN
0018,1244 Preferred Playback Sequencing	US	1	0			0
0018,1600 Shutter Shape	CS	03-Jan	CIRCULAR	CIRCULAR	CIRCULAR	CIRCULAR
0018,1610 Center of Circular Shutter	IS	2	513\513	513\513	513\513	513\513
0018,1612 Radius of Circular Shutter	IS	1	550	550	550	550
0020,000D Study Instance UID	UI	1				
0020,000E Series Instance UID	UI	1				
0020,0010 Study ID	SH	1				
0020,0011 Series Number	IS	1				
0020,0013 Instance Number	IS	1				
0020,0020 Patient Orientation	CS	2	L\F	L\F	L\F	L\F
0020,4000 Image Comments	LT	1				
0028,0002 Samples per Pixel	US	1	1	1	1	1
0028,0004 Photometric Interpretation	CS	1	MONOCHROME1/2	MONOCHROME1/2	MONOCHROME1/2	MONOCHROME1/2
0028,0008 Number of Frames	IS	1				

*) actual Software Version

Copyright © Siemens AG 2005. All rights reserved.