SIEMENS

FLUOROSPOT® T.O.P. VD10A

AX

DICOM Conformance Statement

V2.1 15-May-00

© Siemens AG 2000 All rights reserved

Siemens AG, Medical Engineering Group, Siemensstr. 1, D-91301 Forchheim, Germany

Headquarters: Berlin and Munich Siemens AG, Wittelsbacher Platz 2, D-80333 Munich, Germany

Manual Layout medigration GmbH, D-91058 Erlangen, Germany

Printed in the Federal Republic of Germany Release 05.00

Table of Contents

1	Int	troduction	6
	1.1	Overview	6
	1.2	Scope and Field	6
	1.3	Audience	
	1.4	Remarks	
	1.5	Definitions, Terms and Abbreviations	
	1.6	References	7
2	lm	age Storage	8
	2.1	Implementation Model	8
	2.1		
	2.1 2.1		
		3	
	2.2		
	2.2	.1 Association Establishment Policies	
		2.2.1.2 Number of Associations	
		2.2.1.3 Asynchronous Nature	
	2	2.2.1.4 Implementation Identifying Information	10
	2.2	12 7 Goodan in madron by read real real real real real real real real	
		2.2.2.1 Associated Real-World Activity	10
		2.2.2.2 Proposed Presentation Context (Presentation Context Table)	
		2.2.2.3 SOP specific Conformance Statement	
	2.2	,	
	2.3		
	2.3		
	2.3		
	2.3	.3 Physical Media Support	11
	2.4	Extensions / Specializations / Privatizations	11
	2.4	·	
	2.4	.2 Private Transfer Syntaxes	11
	2.5	Configuration	12
	2.5		
	2.5		
		2.5.2.1 Number of Simultaneous Associations	
		2.5.2.2 Maximum PDU Size (configurable in Service)	
	2	2.5.2.3 Time Out (configurable in Service)	
	2.6	Support of Extended Character Sets	12
3	W	orklist Management	13
	3.1	Implementation Model	13

3.1.1 Application Data Flow Diagram	13
3.1.2 Functional Definition "Worklist Update"	
3.1.3 Sequencing of Real-World Activities	
3.2 AE Specification	
3.2.1 Association Establishment Policies	
3.2.1.1 General	
3.2.1.2 Number of Associations	
3.2.1.3 Asynchronous Nature	
3.2.1.4 Implementation Identifying Information	
3.2.2.1 Associated Real-World Activity	
3.2.2.2 Proposed Presentation Context (Presentation Context Table	
3.2.2.3 SOP Specific Conformance Statement	
3.2.3 Association Acceptance Policy	
·	
3.3 Communication Profiles	
3.3.2 TCP/IP Stack	
3.3.3 Physical Media Support	
3.4 Extensions/Specializations/Privatizations	
3.4.1 Standard Extended Basic Worklist Management	
3.4.2 Private Transfer Syntaxes	16
3.5 Configuration	
3.5.1 AE Title/Presentation Address Mapping	
3.5.2 Configurable Parameters	
3.5.2.1 Number of Simultaneous Associations	
3.5.2.2 Maximum PDU Size (configurable in Service)	17
3.5.2.3 Time Out (configurable in Service)	17
3.6 Support of Extended Character Sets	17
3.6 Support of Extended Character Sets	
3.6 Support of Extended Character Sets	
Print Management	18
Print Management	
Print Management 4.1 Implementation Model	18 18
Print Management 4.1 Implementation Model	18 18 18
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print"	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print"	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print" 4.1.3 Sequencing of Real-World Activities. 4.2 AE Specification. 4.2.1 Association Establishment Policies. 4.2.1.1 General. 4.2.1.2 Number of Associations.	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print" 4.1.3 Sequencing of Real-World Activities. 4.2 AE Specification. 4.2.1 Association Establishment Policies. 4.2.1.1 General. 4.2.1.2 Number of Associations 4.2.1.3 Asynchronous Nature.	
4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print". 4.1.3 Sequencing of Real-World Activities. 4.2 AE Specification. 4.2.1 Association Establishment Policies. 4.2.1.1 General 4.2.1.2 Number of Associations 4.2.1.3 Asynchronous Nature 4.2.1.4 Implementation Identifying Information 4.2.2 Association Initiation by Real-World Activity 4.2.2.1 Associated Real-World Activity	
4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print". 4.1.3 Sequencing of Real-World Activities. 4.2 AE Specification. 4.2.1 Association Establishment Policies. 4.2.1.1 General 4.2.1.2 Number of Associations 4.2.1.3 Asynchronous Nature 4.2.1.4 Implementation Identifying Information 4.2.2 Association Initiation by Real-World Activity 4.2.2.1 Associated Real-World Activity	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram 4.1.2 Functional Definition "Print" 4.1.3 Sequencing of Real-World Activities 4.2 AE Specification 4.2.1 Association Establishment Policies 4.2.1.1 General 4.2.1.2 Number of Associations 4.2.1.3 Asynchronous Nature 4.2.1.4 Implementation Identifying Information 4.2.2 Association Initiation by Real-World Activity 4.2.2.1 Associated Real-World Activity 4.2.2.2 Proposed Presentation Context (Presentation Context Table 4.2.3 SOP specific Conformance Statement 4.2.3 Association Acceptance Policy 4.3 Communication Profiles 4.3.1 Supported Communication Stacks (part 8) 4.3.2 TCP/IP Stack	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram. 4.1.2 Functional Definition "Print" 4.1.3 Sequencing of Real-World Activities. 4.2 AE Specification. 4.2.1 Association Establishment Policies. 4.2.1.1 General	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	
4.1 Implementation Model 4.1.1 Application Data Flow Diagram	

		E Title/Presentation Address Mapping	
		onfigurable Parameters	
		Number of Simultaneous Associations	
		Time Out (configurable in Service)	
		,	
•		ort of Extended Character Sets	
5	Verificat	tion	23
	5.1 Imple	ementation Model	. 23
	5.1.1 A	pplication Data Flow Diagram	. 23
		unctional Definition Verification	
		equencing of Real-World Activities	
;		pecification	
		ssociation Establishment Policies	
		General	
		Number of Associations	
	5.2.1.3	Implementation Identifying Information	
		ssociation Initiation by Real-World Activity	
	5.2.2.1	Associated Real-World Activity	. 25
	5.2.2.2	,	
	5.2.2.3	SOP Specific Conformance Statement	
		ssociation Acceptance Policy	
;	5.3 Comı	munication Profiles	. 25
		upported Communication Stacks (part 8)	
		CP/IP Stackhysical Media Supporthysical Media Support	
		•	
;		nsions/Specializations/Privatizations	
		tandard Extended Basic Worklist Management	
		rivate Transfer Syntaxes	
;		iguration	
		E Title/Presentation Address Mapping	
		onfigurable Parameters	
		Maximum PDU Size (configurable in Service)	
		Time Out (configurable in Service)	
	5.6 Supp	ort of Extended Character Sets	27
4/	nnex A: S	iemens DICOM Storage Standard Extended IOD Description	28
•	Table A.1:	Standard Secondary Capture Elements	. 28
	Table A.2:	Elements of the XRF IOD Supported in SC Images	. 29
		Standard X-Ray Radiofluoroscopic and Standard X-Ray Angiographic	
	Elements	Standard X-Nay Nadiondoroscopic and Standard X-Nay Angiographic	.30
		Other Elements Supported (all Images)	
		Retired Elements Supported (all Images)	
		Private Elements, Displayed on the User Interface	
		iemens Worklist Request Identifier Description	
		Worklist Request Identifier	
4	nnex C: Si	iemens Basic Grayscale Print Management Description	35

FLUOROSPOT® T.O.P. VD10A

Table C.1:	Basic Film Session	. 35
Table C.2:	Basic Film Box	.35
Table C.3:	Basic Grayscale Image Box	.35
Table C.4:	Printer	.35
Table C.5:	Error Mapping	.36

1 Introduction

1.1 Overview

The Conformance Statement describes the DICOM interface for the Siemens FLUOROSPOT® T.O.P. VD10A in terms of part 2 of [DICOM].

This introduction describes the application's implemented DICOM functionality in general terms.

1.2 Scope and Field

The Siemens product FLUOROSPOT® T.O.P is a Multipurpose System for digital R/F, Angiography and Interventional Procedures. The FLUOROSPOT® T.O.P. is designed to be integrated into an environment of medical DICOM-based devices. FLUOROSPOT® T.O.P. supports the storage of images utilizing the DICOM "Storage Service Class", the retrieval of worklists from an Information System utilizing the DICOM "Basic Worklist Management Service Class" and the print of images on a printer utilizing the DICOM "Print Management Service". For testing it is utilizing the DICOM "Verification Service Class"

1.3 Audience

This document is intended for hospital staff, health system integrators, software designers or implementers. It is assumed that the reader has a working understanding of DICOM.

1.4 Remarks

DICOM, by itself, does not guarantee interoperability. However, the Conformance Statement facilitates a first-level validation for interoperability between different applications supporting the same DICOM functionality as SCU and SCP, respectively.

This Conformance Statement is not intended to replace validation with other DICOM equipment to ensure proper exchange of information intended.

The scope of this Conformance Statement is to facilitate communication with Siemens and other vendors' Medical equipment. The Conformance Statement should be read and understood in conjunction with the DICOM 3.0 Standard [DICOM]. However, by itself it is not guaranteed to ensure the desired interoperability and a successful interconnectivity.

The user should be aware of the following important issues:

- The comparison of different conformance statements is the first step towards assessing interconnectivity between Siemens and non-Siemens equipment.
- Test procedures should be defined and tests should be performed by the user to validate the connectivity desired. DICOM itself and the conformance parts do not specify this.
- The standard will evolve to meet the users' future requirements. Siemens is actively involved in developing the standard further and therefore reserves the right to make changes to its products or to discontinue its delivery.

1.5 Definitions, Terms and Abbreviations

Definitions, terms and abbreviations used in this document are defined within the different parts of the DICOM standard.

Additional Abbreviations and terms are as follows:

CSE Customer Service Engineer

DSA Digital Subtraction Angiography

RIS Radiology Information System

SIREGRAPH CF Overtable System by SIEMENS

SIT Siemens Installation Tool, configuration tool used by CSE

SUN OS UNIX operating system from SUN Microsystems, Inc., Palo Alto, USA

1.6 References

[DICOM] Digital Imaging and Communications in Medicine (DICOM), NEMA PS 3.1-3.14, 1999

2 Image Storage

2.1 Implementation Model

2.1.1 Application Data Flow Diagram

Image Send is performed on the user's request for each study completed or for specific images selected. Upon request, an association will be initiated, one of the images selected will be sent to the remote node and the association will be closed. This procedure will be repeated until all images selected are sent.

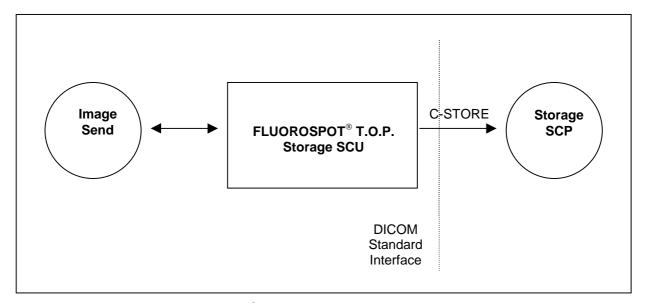


Figure 1: FLUOROSPOT® T.O.P. DICOM Storage Implementation Model

2.1.2 Functional Definition "Image Send"

The FLUOROSPOT® T.O.P. DICOM Application Entity acts as a Service Class User (SCU) for the

• Storage Service Class (to store images in a remote DICOM Node)

Upon the completion of an imaging procedure the FLUOROSPOT® T.O.P. will build a DICOM standard extended data set and initiates sequential associations for each image to be sent. Depending on the configuration the SC IOD, XRF IOD or the XA IOD are used for the data sets.

For more information regarding the DICOM standard extended data sets, please refer to Section 2.2.2.3 "SOP specific Conformance Statement" and Annex A.

The Image Send Functionality is initiated through the user interface. The remote destination has to be defined during the configuration procedure.

Optionally an automatic send functionality can be performed. All images acquired are sent to the dedicated node (e.g. an digital archive) automatically.

If any other SCP response status than "Success" or "Warning" is received by FLUOROSPOT® T.O.P., a notification to check for network problems will appear on the user interface. During the transmission of images to the remote node, a yellow "busy-light" will be displayed. After completion the light turns off automatically.

2.1.3 Sequencing of Real-World Activities

not applicable

2.2 AE Specification

The FLUOROSPOT® T.O.P. Application Entity provides Standard Conformance to the following DICOM SOP Class as a SCU:

SOP Class Name	SOP Class UID	
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	
X-Ray RadioFluoroscopic Image Storage	1.2.840.10008.5.1.4.1.1.12.2	

2.2.1 Association Establishment Policies

2.2.1.1 **General**

Application Context Name (ACN)	1.2.840.10008.3.1.1.1		
Maximum Length PDU offered	configurable in Service, default 28 KB		

2.2.1.2 Number of Associations

FLUOROSPOT® T.O.P. will attempt to initiate one association at a time, one for each Image to be sent.

2.2.1.3 Asynchronous Nature

Asynchronous communication, i.e. multiple outstanding transactions over a single association, is not supported.

2.2.1.4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.3.4	
Implementation Version Name	"SIEMENS_FLT_VD"	

2.2.2 Association Initiation by Real-World Activity

FLUOROSPOT® T.O.P. will attempt to initiate a new association for:

• DICOM Image Send (C-STORE)

2.2.2.1 Associated Real-World Activity

Image Send attempts to send an Image Object to a remote node. If the FLUOROSPOT® T.O.P. AE establishes an association to a remote AE, it will transfer one image at a time via the open association. If the C-STORE response from the remote node contains a status other than "Success" or "Warning", the association is aborted. The image remains on the FLUOROSPOT® T.O.P. with the status "Aborted". Image Send can be restarted at any time by user interaction.

The DICOM target nodes will be configured by a CSE with the SIT service tool. For each node it is configurable, whether the SC IOD or the modality specific (XRF or XA) IOD is used.

2.2.2.2 Proposed Presentation Context (Presentation Context Table)

The DICOM Interface of the FLUOROSPOT® T.O.P. will propose the following presentation contexts:

Presentation Context Table					
Abstract Syntax		Transfer Syntax			Extended
Name	UID	Name List	UID List	Role	Negotiation
Secondary Captured Image Storage Service Class	1.2.840.10008.5.1.4.1.1.7	DICOM Implicit VR Little Endian DICOM Explicit VR Little Endian	1.2.840.10008.1.2 1.2.840.10008.1.2.1	SCU	None
X-Ray Angiographic Image Storage Service Class	1.2.840.10008.5.1.4.1.1.12.1	DICOM Implicit VR Little Endian DICOM Explicit VR Little Endian JPEG Lossless, Non- Hierarchical (Process 14)	1.2.840.10008.1.2 1.2.840.10008.1.2.1 1.2.840.10008.1.2.4.70	SCU	None

X-Ray RadioFluoroscopic	1.2.840.10008.5.1.4.1.1.12.1	DICOM Implicit VR	1.2.840.10008.1.2	SCU	None	ı
Image Storage Service		Little Endian				ı
Class		DICOM Explicit VR	1.2.840.10008.1.2.1			l
		Little Endian				ı
		JPEG Lossless, Non-	1.2.840.10008.1.2.4.70			l
		Hierarchical (Process				ı
		14)				ı

2.2.2.3 SOP specific Conformance Statement

The DICOM images created by the DICOM interface of the FLUOROSPOT® T.O.P. conform to the DICOM IOD definitions (Standard extended IODs). The objects contain additional elements, so-called "retired" elements from earlier versions of the standard. Please refer to Annex A for a complete listing of all supported DICOM elements.

The receiving DICOM nodes are responsible for data consistency when modifying images. All unknown private data elements have to be removed upon modification.

2.2.3 Association Acceptance Policy

not applicable

2.3 Communication Profiles

2.3.1 Supported Communication Stacks (part 8)

The DICOM Interface of the FLUOROSPOT® T.O.P. provides DICOM TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

2.3.2 TCP/IP Stack

The DICOM Interface of the FLUOROSPOT® T.O.P. uses the TCP/IP stack from the SUN-OS Solaris 2.x system upon which it executes. It uses a subroutine library, based on a Berkeley socket interface.

2.3.3 Physical Media Support

The DICOM Interface of the FLUOROSPOT® T.O.P. is indifferent to the physical medium over which TCP/IP executes. It inherits this from the SUN-OS UNIX System upon which it executes.

2.4 Extensions / Specializations / Privatizations

2.4.1 Standard Extended / Specialized / Private SOPs

For extended IODs see Appendix A

2.4.2 Private Transfer Syntaxes

None

2.5 Configuration

2.5.1 AE Title/Presentation Address Mapping

The Application Entity Title, Host name and Port number are defined via the SIT service tool. The SIT service tool is intended to be used by a CSE only.

2.5.2 Configurable Parameters

The Application Entity Titles, Host names and Port numbers are configured using the SIT.

Maximum PDU Size and time outs are configured using the SIT.

Within the SIT it is configurable for each node, whether the SC IOD or the modality specific (XRF or XA) IOD is used. It is also configurable, if JPEG Compression should be used for Multi-Frame Cine Loops (8 Bit, 512 Matrix Size).

The system type (XRF or XA) and the tag Modality (0008,0060) are configurable in the system configuration.

2.5.2.1 Number of Simultaneous Associations

FLUOROSPOT® T.O.P. supports for one service only one association at a time.

2.5.2.2 Maximum PDU Size (configurable in Service)

max PDU size:

default 28 KB

2.5.2.3 Time Out (configurable in Service)

time-out until a SCP has to accept/reject an association request: default 30 sec
 time-out for accepting a message over network: default 15 sec
 time-out for waiting for data between packets: default 15 sec

2.6 Support of Extended Character Sets

ISO-IR 100 (ISO 8859-1:1987 Latin Alphabet N 1. supplementary set)

3 Worklist Management

3.1 Implementation Model

3.1.1 Application Data Flow Diagram

Worklist Update is performed as a result of an operator request, or automatically at certain time intervals. Each request results in an initiation for an association. Under normal conditions the association will be closed after receiving a "Success" response from the Information System.

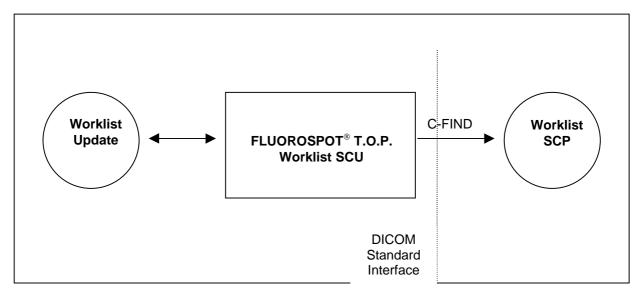


Figure 2: FLUOROSPOT® T.O.P. DICOM Worklist Implementation Model

3.1.2 Functional Definition "Worklist Update"

The FLUOROSPOT® T.O.P. DICOM Application Entity acts as a Service Class User (SCU) for the

Basic Worklist Management Service Class (to request a Worklist from a RIS)

The request for a Worklist Update is initiated by user interaction, i.e. pressing the button "Update", or automatically at certain time intervals. These intervals are configurable, from one to 60 minutes.

Upon initiation of the request, the FLUOROSPOT® T.O.P. will build an Identifier for the C-FIND request, will initiate an association to send the request and will wait for Worklist responses. After retrieval of all responses, FLUOROSPOT® T.O.P. will access the local data base to add or update patient demographic data.

FLUOROSPOT® T.O.P. requests all items for a specific day (actual date) scheduled for this modality (DF or RF, to be configured) or scheduled for the station (AET). The choice is configurable within the SIT service tool (CSE only).

Optionally there is the possibility to query patient data using the following tags: Accession Number, Requested Procedure ID, Patient ID (all Single Value Matching) and Patient Name (Wild Card Matching.)

If any other SCP response status than "Success" or "Pending" is received by FLUOROSPOT® T.O.P., a notification to check for network problems will appear on the user interface.

The FLUOROSPOT® T.O.P. Worklist Request Identifier is described in Annex B "Siemens Worklist Request Identifier Description".

3.1.3 Sequencing of Real-World Activities

not applicable

3.2 AE Specification

The FLUOROSPOT® T.O.P. Application Entity provides Standard Conformance to the following DICOM SOP Class as a SCU:

SOP Class Name	SOP Class UID	
Modality Worklist Information Model - FIND	1.2.840.10008.5.1.4.31	

3.2.1 Association Establishment Policies

3.2.1.1 **General**

Application Context Name (ACN)	1.2.840.10008.3.1.1.1		
Maximum Length PDU offered	configurable in Service, default 28 KB		

3.2.1.2 Number of Associations

FLUOROSPOT® T.O.P. will attempt to initiate one association for a Worklist Update request.

3.2.1.3 Asynchronous Nature

Asynchronous communication, i.e. multiple outstanding transactions over a single association, is not supported.

3.2.1.4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.3.4
Implementation Version Name	"SIEMENS_FLT_VD"

3.2.2 Association Initiation by Real-World Activity

FLUOROSPOT® T.O.P. will attempt to initiate a new association for:

DICOM Worklist Update (C-FIND)

3.2.2.1 Associated Real-World Activity

Worklist Update attempts to download a Worklist from a remote node. If the FLUOROSPOT® T.O.P. AE establishes an association to a remote AE, it will transfer all worklist items via the open association.

If the C-FIND responses from the remote node contain a status other than "Success" or "Pending", the association is aborted. The next attempt to establish an association will take place automatically after the configured time interval for Worklist Update (one to 60 minutes), or at the user's request by pressing the Update button.

3.2.2.2 Proposed Presentation Context (Presentation Context Table)

The DICOM Interface of the FLUOROSPOT® T.O.P. will propose the following presentation contexts:

Presentation Context Table						
Abstract Syntax Transfer Syntax				Extended		
Name	UID	Name List UID List		Role	Negotiation	
Modality Worklist	1.2.840.10008.5.1.4.31	DICOM Implicit VR Little Endian	1.2.840.10008.1.2	scu	None	
Information Model FIND		DICOM Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU	None	
		DICOM Explicit VR Big Endian	1.2.840.10008.1.2.2	SCU	None	

3.2.2.3 SOP Specific Conformance Statement

Please refer to Annex B for the description of the Siemens Worklist Request Identifier.

3.2.3 Association Acceptance Policy

not applicable

3.3 Communication Profiles

3.3.1 Supported Communication Stacks (part 8)

The DICOM Interface of the FLUOROSPOT® T.O.P. provides DICOM TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

3.3.2 TCP/IP Stack

The DICOM Interface of the FLUOROSPOT® T.O.P. uses the TCP/IP stack from the SUN-OS Solaris 2.x system upon which it executes. It uses a subroutine library, based on a Berkeley socket interface.

3.3.3 Physical Media Support

The DICOM Interface of the FLUOROSPOT® T.O.P. is indifferent to the physical medium over which TCP/IP executes. It inherits this from the SUN-OS UNIX System upon which it executes.

3.4 Extensions/Specializations/Privatizations

3.4.1 Standard Extended Basic Worklist Management

Please refer to Annex B for the description of the Siemens Worklist Request Identifier.

3.4.2 Private Transfer Syntaxes

None

3.5 Configuration

3.5.1 AE Title/Presentation Address Mapping

The Application Entity Title, Host name and Port number are defined via the SIT service tool. The SIT service tool is intended to be used by a CSE only.

3.5.2 Configurable Parameters

The Application Entity Titles, Host names and Port numbers are configured using the SIT.

Maximum PDU Size and time outs are configured using the SIT.

Within the SIT is configurable, if all data for the given modality or only the data for the specific AET is received.

The tag Modality (0008,0060) for the worklist query is configurable in the system configuration. Default is "RF"

It is configurable in the system configuration whether a popup for special query (Accession Number etc.) is shown or only broad queries (by Date and Modality) are performed.

3.5.2.1 Number of Simultaneous Associations

FLUOROSPOT® T.O.P. supports for one service only one association at a time.

3.5.2.2 Maximum PDU Size (configurable in Service)

max PDU size: default 28 KB

3.5.2.3 Time Out (configurable in Service)

time-out until a SCP has to accept/reject an association request: default 30 sec
 time-out for accepting a message over network: default 15 sec
 time-out for waiting for data between packets: default 15 sec

3.6 Support of Extended Character Sets

ISO-IR 100 (ISO 8859-1:1987 Latin Alphabet N 1. supplementary set)

4 Print Management

4.1 Implementation Model

4.1.1 Application Data Flow Diagram

Image Print is performed on the user's request for specific images selected. Upon request, an association will be initiated, a complete film sheet will be sent to the remote node and the association will be closed. This procedure will be repeated until all film sheets are sent.

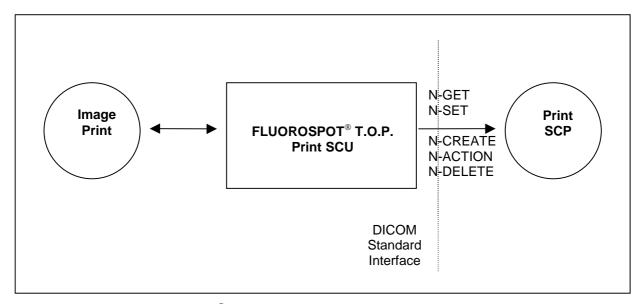


Figure 3: FLUOROSPOT® T.O.P. DICOM Print Management Implementation Model

4.1.2 Functional Definition "Print"

The FLUOROSPOT® T.O.P. DICOM Application Entity acts as a Service Class User (SCU) for the

• Print Management Service Class (to print images in a remote DICOM Node)

Upon the completion of an imaging procedure the FLUOROSPOT® T.O.P. will invoke Basic Grayscale Management Print Management DIMSE services to initiate sequential associations for each film sheet to be sent.

For more information regarding the DICOM standard extended data sets, please refer to Annex C

The print functionality is initiated through the user interface. The remote destination has to be defined during the configuration procedure.

If any other SCP response status than "Success" or "Warning" is received by FLUOROSPOT® T.O.P., a notification will appear on the user interface and an automatic retry will be started.

During the transmission of film sheets to the remote node, a yellow "busy-light" will be displayed. After completion the light turns off automatically.

4.1.3 Sequencing of Real-World Activities

not applicable

4.2 AE Specification

The FLUOROSPOT® T.O.P. Application Entity provides Standard Conformance to the following DICOM SOP Class as a SCU:

SOP Class Name	SOP Class UID
Basic Grayscale Print Management Meta	1.2.840.10008.5.1.1.9

This includes the following SOP Classes

SOP Class Name	SOP Class UID
Basic Film Session SOP Class	1.2.840.10008.5.1.1.1
Basic Film Box SOP Class	1.2.840.10008.5.1.1.2
Basic Grayscale Image Box SOP Class	1.2.840.10008.5.1.1.4
Printer SOP Class	1.2.840.10008.5.1.1.16

4.2.1 Association Establishment Policies

4.2.1.1 **General**

Application Context Name (ACN)	1.2.840.10008.3.1.1.1
Maximum Length PDU offered	configurable in Service, default 28 KB

4.2.1.2 Number of Associations

FLUOROSPOT® T.O.P. will attempt to initiate one association at a time, one for each film sheet to be sent.

4.2.1.3 Asynchronous Nature

Asynchronous communication, i.e. multiple outstanding transactions over a single association, is not supported.

4.2.1.4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.3.4
Implementation Version Name	"SIEMENS_FLT_VD"

4.2.2 Association Initiation by Real-World Activity

FLUOROSPOT® T.O.P. will attempt to initiate a new association for DICOM Print Management. The association is closed after successful print of the film sheet or an error message by the SCP.

4.2.2.1 Associated Real-World Activity

Print Management attempts to print a film sheet with one ore more copies on a remote printer. The FLUOROSPOT® T.O.P. prepares a complete film sheet (with header, footer, format division) and uses a 1:1 format to print it on the printer.

If the response from the remote application contains the status "Failure", the session is aborted and after a few seconds a new association is started for retrial.

Warnings will be ignored by the FLUOROSPOT® T.O.P, errors will be shown in the user interface.

4.2.2.2 Proposed Presentation Context (Presentation Context Table)

The DICOM Interface of the FLUOROSPOT® T.O.P. will propose the following presentation contexts:

Presentation Context Table						
Abstract Syntax Transfer Syntax			fer Syntax		Extended	
Name	UID	Name List	Name List UID List		Negotiation	
Basic Grayscale Print	1.2.840.10008.5.1.1.9	DICOM Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None	
Management Meta		DICOM Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU	None	
		DICOM Explicit VR Big Endian	1.2.840.10008.1.2.2	SCU	None	

4.2.2.3 SOP specific Conformance Statement

The Print Management of FLUOROSPOT® T.O.P. conforms to the DICOM Basic Print Management SOP definitions.

The following DIMSE Services are used (in order of usage), to implement the Print Management:

Printer N_GET
Basic Film Session N_CREATE
Basic Film Box N_CREATE
Basic Grayscale Image Box N_SET
Basic Film Box N_ACTION
Printer N_GET
Basic Film Session N_DELETE

Please refer to Annex C for a complete listing of all supported DICOM elements .

The Print Management is restricted to printers listed in the configuration of the FLUOROSPOT® T.O.P.

4.2.3 Association Acceptance Policy

not applicable

4.3 Communication Profiles

4.3.1 Supported Communication Stacks (part 8)

The DICOM Interface of the FLUOROSPOT® T.O.P. provides DICOM TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

4.3.2 TCP/IP Stack

The DICOM Interface of the FLUOROSPOT® T.O.P. uses the TCP/IP stack from the SUN-OS Solaris 2.x system upon which it executes. It uses a subroutine library, based on a Berkeley socket interface

4.3.3 Physical Media Support

The DICOM Interface of the FLUOROSPOT® T.O.P. is indifferent to the physical medium over which TCP/IP executes. It inherits this from the SUN-OS UNIX System upon which it executes.

4.4 Extensions / Specializations / Privatizations

4.4.1 Standard Extended / Specialized / Private SOPs

None

4.4.2 Private Transfer Syntaxes

None

4.5 Configuration

4.5.1 AE Title/Presentation Address Mapping

The Application Entity Title, Host name and Port number are defined via the SIT service tool. The SIT service tool is intended to be used by a CSE only.

4.5.2 Configurable Parameters

The Application Entity Titles, Host names and Port numbers are configured using the SIT.

Maximum PDU Size and time outs are configured using the SIT.

The tag Magnification Type (2010,0060) is configured using the SIT. Default and recommended is "BILINEAR"

The tag Max Density (2010,0130) is configured using the SIT. Default and recommended is "300"

Depending on the configuration within the SIT, images are sent "linear" or "non-linear" (i.e. with FLUOROSPOT [®] T.O.P. specific LUT). Default an recommended is "non-linear".

The size of the film sheet in pixel (rows, columns) and the selectable film sizes (e.g. "14x17") depend on the configured printer (SIT).

4.5.2.1 Number of Simultaneous Associations

FLUOROSPOT® T.O.P. supports for print management only one association at a time.

4.5.2.2 Maximum PDU Size (configurable in Service)

max PDU size: default 28 KB

4.5.2.3 Time Out (configurable in Service)

time-out until a SCP has to accept/reject an association request: default 15 sec
 time-out for accepting a message over network: default 15 sec
 time-out for waiting for data between packets: default 15 sec

4.6 Support of Extended Character Sets

ISO-IR 100 (ISO 8859-1:1987 Latin Alphabet N 1. supplementary set)

5 Verification

5.1 Implementation Model

5.1.1 Application Data Flow Diagram

Verification is performed by a CSE to verify the ability of a foreign DICOM application on a remote node to respond to DICOM messages.

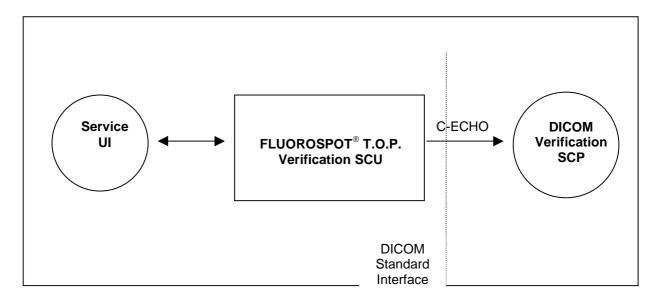


Figure 4: FLUOROSPOT® T.O.P. DICOM Verification Implementation Model

5.1.2 Functional Definition Verification

The Service UI Tool opens an association when a test of a remote application is requested during a configuration session. This can be done after adding or change a remote node in the configuration to verify the communication.

5.1.3 Sequencing of Real-World Activities

not applicable

5.2 AE Specification

The FLUOROSPOT® T.O.P. Application Entity provides Standard Conformance to the following DICOM SOP Class as a SCU:

SOP Class Name	SOP Class UID		
Verification	1.2.840.10008.1.1		

5.2.1 Association Establishment Policies

5.2.1.1 General

Application Context Name (ACN)	1.2.840.10008.3.1.1.1
Maximum Length PDU offered to a Storage SCP	configurable in Service, default 28 KB
Maximum Length PDU offered to a Worklist SCP	configurable in Service, default 28 KB
Maximum Length PDU offered to a Print SCP	configurable in Service, default 28 KB

5.2.1.2 Number of Associations

FLUOROSPOT® T.O.P. will attempt to initiate one association for verification.

5.2.1.3 Asynchronous Nature

Asynchronous communication, i.e. multiple outstanding transactions over a single association, is not supported.

5.2.1.4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.3.4
Implementation Version Name	"SIEMENS_FLT_VD"

5.2.2 Association Initiation by Real-World Activity

FLUOROSPOT® T.O.P. will attempt to initiate a new association for:

DICOM Verification (C-ECHO)

5.2.2.1 Associated Real-World Activity

The associated Real-World Activity is C-ECHO request initiated by the CSE to check whether a remote node is correctly configured. Any response other than "Success" implies an error in configuration either at the local or at the remote node.

5.2.2.2 Proposed Presentation Context (Presentation Context Table)

The DICOM Interface of the FLUOROSPOT® T.O.P. will propose the following presentation contexts:

Presentation Context Table						
Abstract Syntax Transfer Syntax			Extended			
Name	UID	Name List UID List		Role	Negotiation	
Verification	1.2.840.10008.1.1	DICOM Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None	
	1.2.0 10.10000.11.1	DICOM Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU	None	
		DICOM Explicit VR Big Endian (not for Storage)	1.2.840.10008.1.2.2	SCU	None	

5.2.2.3 SOP Specific Conformance Statement

DICOM Explicit VR Big Endian is not used for testing a Storage SCP

5.2.3 Association Acceptance Policy

not applicable

5.3 Communication Profiles

5.3.1 Supported Communication Stacks (part 8)

The DICOM Interface of the FLUOROSPOT® T.O.P. provides DICOM TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

5.3.2 TCP/IP Stack

The DICOM Interface of the FLUOROSPOT® T.O.P. uses the TCP/IP stack from the SUN-OS Solaris 2.x system upon which it executes. It uses a subroutine library, based on a Berkeley socket interface.

5.3.3 Physical Media Support

The DICOM Interface of the FLUOROSPOT® T.O.P. is indifferent to the physical medium over which TCP/IP executes. It inherits this from the SUN-OS UNIX System upon which it executes.

5.4 Extensions/Specializations/Privatizations

5.4.1 Standard Extended Basic Worklist Management

None

5.4.2 Private Transfer Syntaxes

None

5.5 Configuration

5.5.1 AE Title/Presentation Address Mapping

The Application Entity Title, Host name and Port number are defined via the SIT service tool. The SIT service tool is intended to be used by a CSE only.

5.5.2 Configurable Parameters

The Application Entity Titles, Host names and Port numbers are configured using the SIT.

Maximum PDU Size and time outs depend on the service to be tested. For each service they are configured using the SIT.

5.5.2.1 Number of Simultaneous Associations

FLUOROSPOT® T.O.P. supports for one service only one association at a time.

5.5.2.2 Maximum PDU Size (configurable in Service)

•	max PDU size (testing a Storage SCP):	default 28 KB
•	max PDU size (testing a Worklist SCP):	default 28 KB
•	max PDU size (testing a Print SCP):	default 28 KB

5.5.2.3 Time Out (configurable in Service)

time-out until a SCP has to accept/reject an association request: default 30 sec
 time-out for accepting a message over network: default 15 sec
 time-out for waiting for data between packets: default 15 sec

5.6 Support of Extended Character Sets

ISO-IR 100 (ISO 8859-1:1987 Latin Alphabet N 1. supplementary set)

Annex A: Siemens DICOM Storage Standard Extended IOD Description

Table A.1: Standard Secondary Capture Elements

Module	Attribute Name	TAG	Туре	Comments	Visibility
Patient	Patient's Name	0010,0010	2	RIS or User Input	Patient List, New Patient, Image View, Film
	Patient ID	0010,0020	2	RIS or user input	Patient List, New Patient, Image View, Film
	Patient's Birth Date	0010,0030	2	RIS or user input	Patient List, New Patient, Image View, Film
	Patient's Sex	0010,0040	2	RIS or user input	New Patient
	Other Patient ID's	0010,1000	3	RIS	RIS data
	Ethnic Group	0010,2160	3	RIS	RIS data
	Patient Comments	0010,4000	3	RIS	RIS data
General Study	Study Instance UID	0020,000D	1	RIS or system generated	Trio data
Ochicial Olddy	Study Date	0008,0020	2	Trib or system generated	Patient List
	Study Time	0008,0020	2		Patient List
	Referring Physician's Name	0008,0090	2	RIS	RIS data
	Study ID	0020,0010	2	System generated	KIS data
	Accession Number	0008,0050	2	RIS or User Input	Patient List*, New Patient, Film*
	Study Description	0008,1030	3	RIS (copied from 0032,1060) or User Input	Patient List*, New Patient, Film*
	Physician (s) of Record	0008,1048	3	RIS (copied from 0040,1010)	RIS data
	Referenced Study Sequence	0008,1110	3	RIS	
	>Referenced SOP Class UID	0008,1150	1C	RIS	
	>Referenced SOP Instance UID	0008,1155	1C	RIS	
Patient Study	Admitting Diagnosis Description	0008,1080	3	RIS	RIS data
	Patient's Weight	0010,1030	3	RIS	RIS data
	Additional Patient History	0010,21B0	3	RIS	RIS data
General Series	Modality	0008,0060	1	Configurable, default: "RF"	
	Series Instance UID	0020,000E	1		
	Series Number	0020,0011	2		
	Laterality	0020,0060	2C	always zero length	
	Performing Physician's Name	0008,1050	3	RIS (copied from 0040,0006) or User input	Patient List, New Patient, Film
	Series Description	0008,103E	3	Organ Program Name from Acquisition System	Image Report
	Operator's Name	0008,1070	3	Same as 0008,1050	
	Requested Attribute Sequence	0040,0275	3		
	>Requested Procedure ID	0040,1001	1C	RIS or User Input	Patient List*, New Patient, Film*
	>Scheduled Procedure Step ID	0040,0009	1C	RIS (copied from 0040,0100)	
	>Scheduled Procedure Step Description	0040,007	3	RIS (copied from 0040,0100)	
	>Scheduled Action Item Code Sequence	0040,0008	3	RIS (copied from 0040,0100)	
	>>Code Value	0008,0100	1C		
	>>Code Scheme Designator	0008,0102	1C		
	>>Code Meaning	0008,0104	3		
General Equipment	Manufacturer	0008,0070	2	"SIEMENS "	
• •	Institution Name	0008,0080	3	User input	Film
	Station Name	0008,1010	3	Service input (PLA)	
	Institutional Department Name	0008,1040	3	Same as 0008,0080	
	Manufacturer's Model Name	0008,1090	3	"FLUOROSPOT_TOP01"	Film
	Device Serial Number	0018,1000	3	Host ID	
	Software Versions	0018,1020	3		
SC Equipment	Conversion Type	0008,0064	1	Defined Term "DI" is used	
	Modality	0008,0060	3	See "General Series"	
General Image	Image Number	0020,0013	2		

	Patient Orientation	0020,0020	2C	always zero length	
	Image Date	0008,0023	2C		Image View, Film
	Image Time	0008,0033	2C		Image View, Film
	Image Type	0008,0008	3		
	Acquisition Number	0020,0012	3	Same as 0020,0013	
	Acquisition Date	0008,0022	3	Same as 0008,0023	
	Acquisition Time	0008,0032	3	Same as 0008,0033	
	Image Comments	0020,4000	3	User input	Image View, Film
Image Pixel	Samples per Pixel	0028,0002	1	Always "1"	
	Photometric Interpretation	0028,0004	1	"MONOCHROME2"	
	Rows	0028,0010	1	1024 or 2048	Image View, Film
	Columns	0028,0011	1	1024 or 2048	Image View, Film
	Bits Allocated	0028,0100	1	16	
	Bits Stored	0028,0101	1	12	
	High Bit	0028,0102	1	11	
	Pixel Representation	0028,0103	1	0	
	Pixel Data	7FE0,0010	1		
Overlay Plane	Overlay Rows	6000,0010	1	Same as 0028,0010	
-	Overlay Columns	6000,0011	1	Same as 0028,0011	
	Overlay Type	6000,0040	1	"G "	
	Origin	6000,0050	1	1,1	
	Overlay Bits Allocated	6000,0100	1	16	
	Bit Position	6000,0102	1	15	
VOI LUT	Window Center	0028,1050	3	-1024 5120	
	Window Width	0028,1051	1C	8 4096	
SOP Common	SOP Class UID	0008,0016	1		
	SOP Instance UID	0008,0018	1		
	Specific Character Set	0008,0005	1C	"ISO_IR 100"	

 $^{^{\}star}$ It is configurable by the user if Accession No., Requested Procedure ID or Organ (=Study Description), respectively, are shown in the Patient List and on Film.

Table A.2: Elements of the XRF IOD Supported in SC Images

Module	Attribute Name	Tag	Туре	Comments	Visibility
Cine	Cine Rate	0018,0040	3	(0.5 is rounded to 1)	Image Report
X-Ray Image	Pixel Intensity Relationship	0028,1040	1	"LIN "	<u> </u>
	Scan Options	0018,0022	3	Only with tomo and peri images	
X-Ray Acquisition	KVP	0018,0060	2		Image Report
	Radiation Setting	0018,1155	1		-
	Exposure Time	0018,1150	2C		
	Exposure	0018,1152	2C	Rounded to integer	Image Report
	Grid	0018,1166	3		
	Average Pulse Width	0018,1154	3		Image Report
	Radiation Mode	0018,115A	3		
	Intensifier Size	0018,1162	3		
	Field of View Shape	0018,1147	3	"ROUND "	
	Field of View Dimension(s)	0018,1149	3		Image Report
X-Ray Table	Table Motion	0018,1134	3	"STATIC"	
XRF Positioner	Distance Source to Patient	0018,1111	3	not with SIREGRAPH CF	
	Distance Source to Detector	0018,1110	3	not with SIREGRAPH CF	
XRF Tomo Acquisition	Tomo Layer Height	0018,1460	1	Only with Tomo Images; not with SIREGRAPH CF	Image View, Film
	Tomo Time	0018,1480	3	Only with Tomo Images; not with SIREGRAPH CF	

Table A.3: Standard X-Ray Radiofluoroscopic and Standard X-Ray Angiographic Elements

Module	Attribute Name	TAG	Туре	Comments	Visibility
Patient	Patient's Name	0010,0010	2	RIS or User Input	Patient List, New Patient,
	Patient ID	0010,0020	2	RIS or user input	Image View, Film Patient List, New Patient, Image View, Film
	Patient's Birth Date	0010,0030	2	RIS or user input	Patient List, New Patient, Image View, Film
	Patient's Sex	0010,0040	2	RIS or user input	New Patient
	Other Patient ID's	0010,0040	3	RIS	RIS data
	Ethnic Group	0010,1000	3	RIS	RIS data
	Patient Comments	0010,2100	3	RIS	RIS data
General Study	Study Instance UID	0020,000D	1	RIS or system generated	THO data
Contrair Citacy	Study Date	0008,0020	2	The or System generated	Patient List
	Study Time	0008,0030	2		Patient List
	Referring Physician's Name	0008,0090	2	RIS	RIS data
	Study ID	0020,0010	2	System generated	1110 00010
	Accession Number	0008,0050	2	RIS or User Input	Patient List*, New Patient, Film*
	Study Description	0008,1030	3	RIS (copied from 0032,1060) or User Input	Patient List*, New Patient, Film*
	Physician (s) of Record	0008,1048	3	RIS (copied from 0040,1010)	RIS data
	Referenced Study Sequence	0008,1110	3	RIS	
	>Referenced SOP Class UID	0008,1150	1C	RIS	
	>Referenced SOP Instance UID	0008,1155	1C	RIS	
Patient Study	Admitting Diagnosis Description	0008,1080	3	RIS	RIS data
	Patient's Weight	0010,1030	3	RIS	RIS data
	Additional Patient History	0010,21B0	3	RIS	RIS data
General Series	Modality	0008,0060	1	Configurable, default: "RF"	
	Series Instance UID	0020,000E	1		
	Series Number	0020,0011	2		
	Laterality	0020,0060	2C	always zero length	
	Series Date	0008,0021	3		
	Series Time Performing Physician's Name	0008,0031	3	RIS (copied from 0040,0006) or User input	Patient List, New Patient, Film
	Series Description	0008,103E	3	Organ Program Name from Acquisition System	Image Report
	Operator's Name	0008,1070	3	Same as 0008,1050	
	Requested Attribute Sequence	0040,0275	3	- Carrio do 0000, 1000	
	>Requested Procedure ID	0040,1001	1C	RIS or User Input	Patient List*, New Patient, Film*
	>Scheduled Procedure Step ID	0040,0009	1C	RIS (copied from 0040,0100)	
	>Scheduled Procedure Step Description	0040,007	3	RIS (copied from 0040,0100)	
	>Scheduled Action Item Code Sequence	0040,0008	3	RIS (copied from 0040,0100)	
	>>Code Value	0008,0100	1C		
	>>Code Scheme Designator	0008,0102	1C		
	>>Code Meaning	0008,0104	3		
General Equipment	Manufacturer	0008,0070	2	"SIEMENS "	
	Institution Name	0008,0080	3	User input	Film
	Station Name	0008,1010	3	Service input (PLA)	
	Institutional Department Name	0008,1040	3	Same as 0008,0080	F:!
	Manufacturer's Model Name	0008,1090	3	"FLUOROSPOT_TOP01"	Film
	Device Serial Number	0018,1000	3	Host ID	
General Image	Software Versions Image Number	0018,1020 0020,0013	2		
General Image	Patient Orientation	0020,0013	2C	always zero length	
	Image Date	0020,0020	2C 2C	aiways zeio lengin	Image View, Film
	Image Time	0008,0023	2C		Image View, Film
 	Image Type	0008,003	3		iiiaye view, riiiii

	A consistion Number	0000 0040	2	C 0000 0040	
	Acquisition Number	0020,0012	3	Same as 0020,0013	
	Acquisition Date	0008,0022		Same as 0008,0023	
	Acquisition Time Image Comments	0008,0032	3	Same as 0008,0033	Imaga Viau, Film
Image Pixel	Samples per Pixel	0020,4000	3 1	User input Always "1"	Image View, Film
illiage Fixel	Photometric Interpretation	0028,0002	1	"MONOCHROME2"	
	Rows	0028,0004	1	512,1024 or 2048	Image View, Film
	Columns	0028,0010	1	512,1024 or 2048 512,1024 or 2048	Image View, Film
	Bits Allocated	0028,0011	1		illiage view, Fillii
	Bits Stored	0028,0100		8 or 16 8 or 12	
			1		
	High Bit	0028,0102		7 or 11	
	Pixel Representation	0028,0103 7FE0,0010	1	0	
Contract / Dalue	Pixel Data		1	A lever ve me de leverth	
Contrast / Bolus	Contras / Bolus Agent	0018,0010	2	Always zero length	
Cine Mode	Frame Time	0018,1063	0	(0.5 's maximals d.(s.4)	Lancard Dancard
NA 101 F	Cine Rate	0018,0040	3	(0.5 is rounded to 1)	Image Report
Multi-Frame	Number of Frames	0028,0008	1	Only with multi-frame	
	Frame Increment Pointer	0028,0009	1	00181063H Only with multi-frame	
Mask Module	Mask Subtraction Sequence	0028,6100	1	Only with DSA multi-frame	
	> Mask Operation	0028,6101	1	"AVG_SUB"	
	> Mask Frame Numbers	0028,6110	1C		
	> Mask Sub-pixel Shift	0028,6114	3		
	Recommended Viewing Mode	0028,1090	2	Only with DSA multi-frame	
Display Shutter	Shutter Shape	0018,1600	1	"RECTANGULAR\CIRCUL AR" or "CIRCULAR"	
	Shutter Left Vertical Edge	0018,1602	1C	not if 0028,0100 = 8	
	Shutter Right Vertical Edge	0018,1604	1C	not if 0028,0100 = 8	
	Shutter Upper Horizontal Edge	0018,1606	1C	not if 0028,0100 = 8	
	Shutter Lower Horizontal Edge	0018,1608	1C	not if 0028,0100 = 8	
	Center of Circular Shutter	0018,1610	1C	,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,	
	Radius of Circular Shutter	0018,1612	1C		
X-Ray Image	Frame Increment Pointer	0028,0009	1C	See "Multi-Frame Module"	
- · · · · · · · · · · · · · · · · · · ·	Image Type	0008,0008	1	See "General Image Module"	
	Pixel Intensity Relationship	0028,1040	1	"LIN " or "LOG" (DSA multi-frame)	
	Samples per Pixel	0028,0002	1	See "Image Pixel Module"	
	Photometric Interpretation	0028,0004	1	See "Image Pixel Module"	
	Bits Allocated	0028,0100	1	See "Image Pixel Module"	
	Bits Stored	0028,0101	1	See "Image Pixel Module"	
	High Bit	0028,0102	1	See "Image Pixel Module"	
	Pixel Representation	0028,0103	1	See "Image Pixel Module"	
				Only with tomo and peri	
	Scan Options	0018,0022	3	images	
X-Ray Acquisition	KVP	0018,0060	2		Image Report
	Radiation Setting	0018,1155	11		
	Exposure Time	0018,1150	2C		
	Exposure	0018,1152	2C		Image Report
	Grid	0018,1166	3		
	Average Pulse Width	0018,1154	3		Image Report
	Radiation Mode	0018,115A	3		
	Intensifier Size	0018,1162	3		
	Field of View Shape	0018,1147	3	"ROUND "	
	Field of View Dimension(s)	0018,1149	3		Image Report
X-Ray Table	Table Motion	0018,1134	3	"STATIC"	
XRF Positioner	Distance Source to Detector	0018,1110	3		
	Distance Source to Patient	0018,1111	3		
	Column Angulation	0018,1450	3	Only with XRF IOD	
XRF Tomo Acquisition	Tomo Layer Height	0018,1460	1	Only with Tomo Images; not with SIREGRAPH CF	Image View, Film
	Tomo Time	0018,1480	3	Only with Tomo Images; not with SIREGRAPH CF	
XA Positioner	Distance Source to Patient	0018,1111	3	Not with SIREGRAPH CF	
	Distance Source to Detector	0018,1110	3	Not with SIREGRAPH CF	
	Positioner Motion	0018,1450	2C	"STATIC" Only with XA IOD	
	Positioner Primary Angle	0018,1510	2	Always zero length; Only with XA IOD	

	Positioner Secondary Angle	0018,1511	2	Always zero length; Only with XA IOD	
Overlay Plane	Overlay Rows	6000,0010	1	Same as 0028,0010; not if 0028,0100 = 8	
	Overlay Columns	6000,0011	1	Same as (0028,0011); not if 0028,0100 = 8	
	Overlay Type	6000,0040	1	"G"; not if 0028,0100 = 8	
	Origin	6000,0050	1	1,1; not if 0028,0100 = 8	
	Overlay Bits Allocated	6000,0100	1	16; not if 0028,0100 = 8	
	Bit Position	6000,0102	1	15; not if 0028,0100 = 8	
Multi-Frame Overlay	Number of Frames in Overlay	6000,0015	1	= number of frames in multi-frame; only with multi-frame	
	Image Frame Origin		3	1; only with multi-frame	
Modality LUT	Modality LUT Sequence	0028,3000	3	Only with DSA multi-frame	
	> LUT Description	0028,3002	1C	4096,0,16	
	> Modality LUT Type	0028,3004	1C	"US"	
	> LUT Data	0028,3006	1C		
VOI LUT	Window Center	0028,1050	3	-1024 5120	
	Window Width	0028,1051	1C	8 4096	
SOP Common	SOP Class UID	0008,0016	1		
	SOP Instance UID	0008,0018	1		
	Specific Character Set	0008,0005	1C	"ISO_IR 100"	

^{*} It is configurable by the user if Accession No., Requested Procedure ID or Organ (=Study Description), respectively, are shown in the Patient List and on Film.

Comment to special image types:

DSA Images: Logarithmic raw data is sent; Subtraction has to be done by SCP Multi-Frame Images: Scenes are sent as multi-frame objects (max. object size 150 MB) 8 Bit Loops (0028,0100 = 8): Always 512x512, no rectangular shutter, no overlay graphic

Table A.4: Other Elements Supported (all Images)

Module	Attribute	Tag	Type Comments		Visibility
Image Plane	Pixel Spacing	0028,0030		Used for distance measurement	
Visit Status	Current Patient Location	0038,0300		RIS or user input	Patient List, New Patient, Film

Table A.5: Retired Elements Supported (all Images)

Module	Attribute Name	Tag	Type	Comments	Visibility
	DATA Set Type	0008,0040	RET	0	
	DATA Set Subtype	0008,0041	RET	"IMA DFR " or "IMA TOMO"	
	Comments	0018,4000	RET	Same as 0020,4000	

Table A.6: Private Elements, Displayed on the User Interface

Attribute	Private Creator	Tag	Comments	Visibility
Edge Enhancement [%] ("E:")	SIEMENS DFR.01 MANIPULATED	0017,xx29	[0100]	Image View, Film
Harmonization [%] ("H:")	SIEMENS DFR.01 MANIPULATED	0017,xx30	[0100]	Image View, Film
Landmark	SIEMENS DFR.01 MANIPULATED	0017,xx71	[0100]	Image View, Film
Pixel Shift horizontal	SIEMENS DFR.01 MANIPULATED	0017,xx77	[-20.0 +20.0]	Image View, Film
Pixel Shift vertical	SIEMENS DFR.01 MANIPULATED	0017,xx78	[-20.0 +20.0]	Image View, Film
Left Marker	SIEMENS DFR.01 MANIPULATED	0017,xx83	Text, i.e. "L"	Image View, Film
Right Marker	SIEMENS DFR.01 MANIPULATED	0017,xx84	Text, i.e. "R"	Image View, Film
Image Name Extension 1	SIEMENS DFR.01 MANIPULATED	0017,xxA2	Image Number 1 st line	Image View, Film
Image Name Extension 2	SIEMENS DFR.01 MANIPULATED	0017,xxA3	Image Number 2 nd line	Image View, Film

Page 32 of 36 © Siemens AG B Med V2.1

Annex B: Siemens Worklist Request Identifier Description

Table B.1: Worklist Request Identifier

Module Name	Matching	Return	Tag	М	R	DICOM
Attribute Name	Key Type	Key				Part 3
	' ' ' '	Type				Tables
Scheduled Procedure Step		7.				C.4-10
Scheduled Procedure Step Sequence	R	1	0040,0100			
> Scheduled Station AETitle	R	1	0040,0001	F		
> Scheduled Procedure Step Start Date	R	1	0040,0002	R		
> Modality	R	1	0008,0060	F		
> Scheduled Performing Physician's Name	R	2	0040,0006		Х	
> Scheduled Procedure Step Description	0	1C	0040,0007		Х	
> Scheduled Action Item Code Sequence >> Code Value	0	1C 1C	0040,0008 0008,0100		X	
>> Code Value >> Coding Scheme Designator	0	1C	0008,0100		X X	
>> Code Meaning	Ö	3	0008,0102		X	
> Pre-Medication	Ö	2C	0040,0012		X	
> Scheduled Procedure Step ID	Ö	1	0040,0009		Х	
> Requested Contrast Agent	Ö	2C	0032,1070		Х	
Requested Procedure						C.4-11
Requested Procedure ID	0	1	0040,1001	S	×	
Requested Procedure Description	Ö	1C	0032,1060		Х	
Study Instance UID	0	1	0020,000D		Х	
Referenced Study Sequence	0	2	0008,1110		Х	
> Referenced SOP Class UID	0	1C	0008,1150		Х	
> Referenced SOP Instance UID	0	1C	0008,1155		Х	
Patient Transport Arrangements	0	2	0040,1004		Х	
Requested Procedure Comments	0	3	0040,1400		Х	
Names of Intended Recipients of Results	0	3	0040,1010		Х	
Imaging Service Request	_	_				C.4-12
Accession Number	0	2	0008,0050	S	X	
Requesting Physician	0	2	0032,1032		Х	
Referring Physician's Name	0	2 3	0008,0090		X	
Image Service Request Comments Requesting Service	0	3	0040,2400 0032,1032		X	
Visit Identification		<u> </u>	0002,1002		^	C.3-2
Admission ID	0	2	0038,0010		Х	C.3-2
Visit Status	0		0038,0010		^	C.3-3
	0	2	0020 0200		.,	C.3-3
Current Patient Location	0		0038,0300	-	Х	C 2 4
Visit Admission	0	2	0000 1000		.,	C.3-4
Admitting Diagnosis Description Patient Identification	U	3	0008,1080		Х	C.2-2
	R	1	0010 0010	W		0.2-2
Patient Name Patient ID	R R	1 1	0010,0010 0010,0020	S	×	
Other Patients IDs	0	3	0010,0020	٦	х	
Patient Demographic	<u> </u>		30.10,1000			C.2-3
Patient beinographic Patient's Birth Date	0	2	0010,0030		x	0.2-3
Patient's Sex	0	2	0010,0030		X	
Patient's Weight	Ö	2	0010,0040		x	
Confidentiality Constraint on Patient Data	Ö	2	0040,3001		X	
Ethnic Group	Ö	3	0010,2160		Х	
Patient Comments	0	3	0010,4000		Х	
Patient Medical						C.2-4
Patient State	0	2	0038,0500		х	
Pregnancy Status	0	2	0010,21C0		х	
Medical Alerts	0	2	0010,2000		х	
Contrast Allergies	0	2	0010,2110		Х	
Special Needs	0	2	0038,0050		Х	
Additional Patient History	0	3	0010,21B0		Χ	

Matching Types
M: Matching Key
F: Fixed Single Value (Value configured by CSE)

R: Range matching for start date only (always for actual date, i.e. 19980811-19980811)

S: Single Value matching W: Wildcard matching

R: Return Keys

x: Universal Matching

The Information Model Module Table defines the fixed structure of the Worklist Request Identifier.

The table should be read as follows:

- First column: Module Name and attributes supported to build a FLUOROSPOT® T.O.P. Worklist Request Identifier.
- Second column: Requested [R] or optional [O] Matching key. FLUOROSPOT® T.O.P.
 assumes that the Modality Worklist SCP supports Matching Keys as defined in DICOM PS
 3.4 K.2.2.1.1, i.e. that a SCP is able to at least perform Fixed Single Value, Wildcard and
 Date Range matching.
- Third column: Mandatory, conditional and optional Return key.
- Fourth column: Appropriate DICOM tag for this attribute.
- Next two columns: FLUOROSPOT® T.O.P. only expects a "Fixed Value Matching" for
 "Modality" or Scheduled Station AETitle, respectively, "Range Matching" for "Scheduled
 Procedure Step Start Date".
 Accession Number, Requested Procedure ID, Patient ID and Patient Name can be queried
 through the user interface. Empty fields are treated as Return Keys.
- Last column: Reference to the DICOM standard

Annex C: Siemens Basic Grayscale Print Management Description

Table C.1: Basic Film Session

DIMSE	Attribute Name	Tag	Туре	Comments
N-CREATE	Number of Copies	2000,0010	U/M	1 9; User input

After use the Basic Film Session is closed with N-DELETE

Table C.2: Basic Film Box

DIMSE	Attribute Name	Tag	Туре	Comments
N-CREATE	Image Display Format	2010,0010	M/M	"STANDARD\1,1"
	Film Orientation	2010,0040	M/M	"PORTRAIT"
	Referenced Film Session Service	2010,0500	M/M	
	>Referenced SOP Class UID	0008,1150	M/M	
	> Referenced SOP Instance UID	0008,1155	M/M	
	Film Size ID	2010,0050	U/M	"14INx17IN", "11INx14IN" or "8INx10IN"; user input
	Magnification Type	2010,0060	U/M	"REPLICATE", "BILINEAR" or "CUBIC"; configurable in service
	Max Density	2010,0130	U/M	200 300; configurable in service
	Trim	2010,0140	U/U	"NO"

N-ACTION is used to start printing

Table C.3: Basic Grayscale Image Box

DIMSE	Attribute Name	Tag	Type	Comments
N-SET	Image Position	2020,0010	M/M	1
	Basic Grayscale Image Sequence	2020,0110	M/M	
	>Samples Per Pixel	0028,0002	M/M	1
	>Photometric Interpretation	0028,0004	M/M	"MONOCHROME2"
	>Rows	0028,0010	M/M	
	>Columns	0028,0011	M/M	
	>Pixel Aspect Ratio	0028,0034	MC/M	1,1
	>Bits Allocated	0028,0100	M/M	8
	>Bits Stored	0028,0101	M/M	8
	>High Bit	0028,0102	M/M	7
	>Pixel Representation	0028,0103	M/M	0
	>Pixel Data	7FE0,0010	M/M	
	Polarity	2020,0020	U/M	"NORMAL"

Table C.4: Printer

DIMSE	Attribute Name	Tag	Type	Comments
N-GET	Printer Status	2110,0010	U/M	
	Printer Status Info	2110,0020	U/M	

Table C.5: Error Mapping

Error	SOP Class	Message on User Interface	
No Network Connection	Association Negotiation	Can not connect to printer	
C601	Basic Film Session	Supply Empty	
C602	Basic Film Box	Supply Empty	
C605	Basic Grayscale Image Box	Supply Empty	
"SUPPLY EMPTY"	Printer (Printer Status Info), if Printer Status = "FAILURE"	Supply Empty	
"RECEIVER FULL"	Printer (Printer Status Info) , if Printer Status = "FAILURE"	Receive Full	
"NO RECEIVE MGZ"	Printer (Printer Status Info) , if Printer Status = "FAILURE"	Receive Full	
"FILM JAM"	Printer (Printer Status Info) , if Printer Status = "FAILURE"	Film Jam	
All other Errors / Failures		DICOM Error	

Warnings are ignored by the Print Management.