Case 10

A Rare Anomaly of the Middle Cerebral Artery Detected by Three-Dimensional **Subtraction CT-Angiography**

Jacqui Fielding

Department of Radiology, Angliss Hospital, Upper Ferntree Gully, Melbourne VIC, Australia

HISTORY

A 66-year-old woman with a history of hypertension and recent headaches presented at the hospital with an acute onset of vertigo, fatigue and severe headache. On examination and questioning of the patient, it was found that her mother died at an early age from complications of a cerebral aneurysm. The patient was referred for a brain CT and brain CTA to rule out sub-arachnoid hemorrhage and aneurysm.

DIAGNOSIS

The non-enhanced brain CT shows no sign of subarachnoid bleeding. Using the functionality of digital subtraction CT-Angiography, automatically subtracting a non-contrast from a contrast enhanced study, the complete cerebrovascular tree could be demonstrated and two progressed aneurysms in the middle cerebral arteries could be detected.

COMMENTS

In this 66-year-old woman, preoperative angiography and 3-dimensional computed tomography angiography revealed a double aneurysm at the right middle cerebral artery (Circle of Willis).

Both aneurysms were clipped and superficial temporal artery-DMCA* anastomosis was performed. She was discharged with no neurologic deficits.


EXAMINATION PROTOCOL

Scanner	SOMATOM Emotion 16
Scan area	Circle of Willis
Scan length	80 mm
Scan direction	caudo-cranial
Scan time	7 s
Tube voltage	100 kV
Tube current	100 Eff. mAs
Dose modulation	CARE Dose off
CTDI _{vol}	15 mGy
Rotation time	0.6 s
Pitch	0.8
Slice collimation	16 x 0.6 mm
Slice width	0.75 mm
Reconstruction increment	0.5 mm
Reconstruction kernel	Н31
Contrast	Ultravist 370
Volume	50 ml
Flow rate	4 ml/s
lodine delivery rate	4 ml/s
Start delay	CARE Bolus


^{*}Duplication of the middle cerebral artery.




1 View into the brain via Volume Rendering Technique (VRT), showing the position of the double aneurysm in the Circle of Willis.

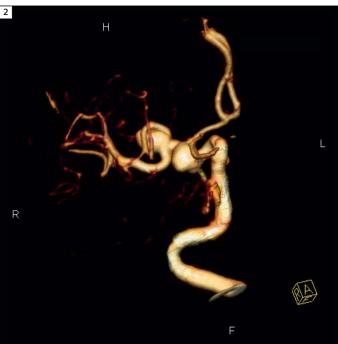

2 Detailed VRT image, showing both aneurysms, located consecutively in an aortic brain vessel.

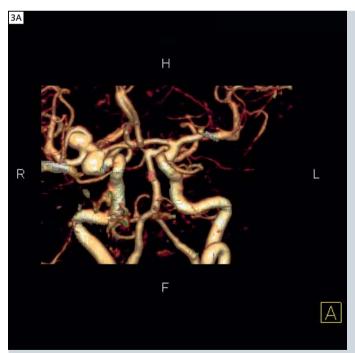

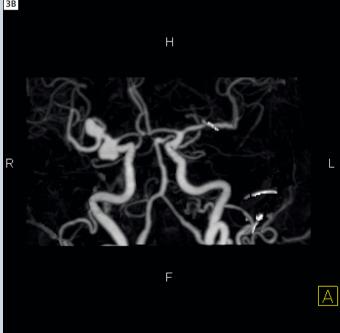

3 View on the two aneurysms in the Circle of Willis, virtually separated from brain tissue in VRT (Fig. 3A) and in comparison in MIP (Fig. 3B).