

syngo.via

Keyboard Shortcuts

Legal Manufacturer

Siemens AG
Wittelsbacherplatz 2
DE-80333 Muenchen
Germany

Global Siemens Healthcare Headquarters

Siemens AG
Healthcare Sector
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
www.siemens.com/healthcare

Global Siemens Headquarters

Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

www.siemens.com/magnetom-world



SIEMENS

syngo.via

Keyboard Shortcuts



To maximize efficiency, *syngo.via* offers a number of shortcuts. By using keyboard shortcuts, you can, for example, use measurement tools or move through the dataset without having to switch tools.

General

- Ctrl + A Select all segments
- Ctrl + G Select all graphics
- Ctrl + Z Undo
- Ctrl + Y Redo
- Ctrl + C Copy selection to Clipboard
- Ctrl + X Move selection to Clipboard
- Ctrl + V Paste
- Ctrl + arrow keys Move focus to next segment
- Del Delete selected graphic
- Esc End current mouse action, cancel selection
- Print Create a screenshot

Navigation

- 2D or MPR*
 - Arrow key up/down Scroll through space
 - Arrow key left/right Scroll through second dimension (for example, time)
 - Page up/down Move focus a predefined number of images
 - Home Go to first image in image stack
 - End Go to last image in image stack
- Volume*
 - Arrow keys Rotate volume

Tools

- S Create snapshot
- R 3D reference point
- Alt + R Open Report Editor

Measurement Tools

- L Pixel lens
- D Distance line
- C ROI circle
- M Marker

Function keys

- F1 Open Online Help
- F5 Color LUT grayscale
- F10 Invert color LUT
- F11 Show image area only, hide other UI elements
- F9 Toggle alpha blending toggling (1st, 2nd dataset, 50/50)

Layout

- T Show/hide image text
- G Show/hide graphics
- Alt + L Open Layout Editor

- From Series Navigator to segment*
 - Drag + left mouse button Replace target segment content with dragged data
 - Drag + right mouse button Open context menu with drag/options
- In Series Navigator*
 - Double-click left mouse button View images in floating segment