

Siemens Healthcare Diagnostics, a global leader in clinical diagnostics, provides healthcare professionals in hospital, reference, and physician office laboratories and point-of-care settings with the vital information required to accurately diagnose, treat, and monitor patients. Our innovative portfolio of performance-driven solutions and personalized customer care combine to streamline workflow, enhance operational efficiency, and support improved patient outcomes.

All associated marks are trademarks of Siemens Healthcare Diagnostics Inc.
All other trademarks and brands are the property of their respective owners. Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Order No. A91DX-CAI-130843-GC1-4A00 12-2013 | All rights reserved © 2013 Siemens Healthcare Diagnostics Inc.

Global Siemens Headquarters

Siemens AG Wittelsbacherplatz 2 80333 Muenchen Germany

Global Siemens Healthcare Headquarters

Siemens AG
Healthcare Sector
Henkestrasse 127
91052 Erlangen
Telephone: +49 9131 84-0
Germany
www.siemens.com/healthcare

Global Division

Siemens Healthcare Diagnostics Inc. 511 Benedict Avenue Tarrytown, NY 10591-5005 USA www.siemens.com/diagnostics

White Paper

TSH Receptor Autoantibodies in Graves' Disease:

Their Utility in Diagnosis and Treatment Monitoring

www.siemens.com/diagnostics Answers for life.

TSH Receptor Autoantibodies in Graves' Disease: Their Utility in Diagnosis and Treatment Monitoring

Summary/Abstract

A number of thyroid diseases involve the production of autoantibodies directed against thyroid tissue. In Graves' disease, a major cause of hyperthyroidism, the presence of stimulatory autoantibodies (TSAb, or TSI) activate the thyrotropin receptor (TSHR) leading to unregulated production and secretion of thyroid hormone. Since there are two types of TSHR antibodies that can exist, stimulating (TSAb/TSI) and blocking antibodies (TBAb), it is important to be able to identify the type prevalent in the patient, in order to make a proper diagnosis. TRAb assays which can detect 1) both blocking and stimulating antibodies (TBII) or 2) stimulating only antibodies (TSI). Both assays provide different information and have proven valuable in the diagnosis and management of Graves' disease and its associated conditions.

Background

Thyroid-stimulating hormone (TSH), also known as thyrotropin, exerts its action on the thyroid to bring about the synthesis and release of the thyroid hormones T_3 and T_4 . Secretion of TSH from the pituitary gland, in turn, is stimulated by the release of thyrotropin-releasing hormone (TRH) by the hypothalamus (Figure 1). Rising thyroid hormone levels inhibit synthesis and secretion of TSH and, to a lesser extent, the release of TRH. Falling thyroid hormone levels elicit release of TSH. Disturbances in the production, secretion, or regulation of thyroid hormones can lead to either hypothyroidism (inadequate thyroid hormone levels) or hyperthyroidism (excess hormone levels). $^{1-3}$

The synthesis of the thyroid hormones requires iodine, which is ingested as iodide. Iodide is concentrated within thyroid follicular cells, where it is converted to organic iodine by thyroid peroxidase (TPO). T₃ (monoiodotyrosine + diiodotyrosine) and T4 (diiodotyrosine + diiodotyrosine) are formed in the follicular cells, where they remain, incorporated within thyroglobulin, until they are cleaved from it. The free hormones are then released from the follicular cells into the circulation, where they bind to serum proteins, primarily thyroxine-binding globulin (with high affinity and low capacity), which carries about 75% of the hormones. Only 0.3% of T₃ and about 0.03% of T₄ are free. It is the free hormones that are biologically active; free T₃ is the more active hormone, whereas free T₄ has minimal activity but acts as a reservoir for free T₃. Eighty percent of circulating T₃ is produced by extrathyroidal monodeiodination of T₄.1 (See Figure 2.)

What Is Graves' Disease?

Graves' disease (GD), also known as toxic diffuse goiter, is one of the most common causes of hyperthyroidism³ and by far the most common autoimmune cause of hyperthyroidism in pregnancy.⁴ Hyperthyroidism, distinguished by some experts as a category within the broader concept of thyrotoxicosis, is a condition characterized by excess synthesis and secretion of thyroid hormones by the thyroid. Thyrotoxicosis denotes any clinical state arising from elevated concentrations of free thyroid hormones in plasma and tissues and clinical manifestations of excessive thyroid hormone metabolism.⁵.⁶ (Other authorities use thyrotoxicosis and hyperthyroidism synonymously.)

Graves' disease is caused by the presence of autoantibodies to the TSH receptor. These stimulatory antibodies (TSI) activate TSH receptors on the thyroid follicular cells and lead to unregulated production and secretion of thyroid hormone.

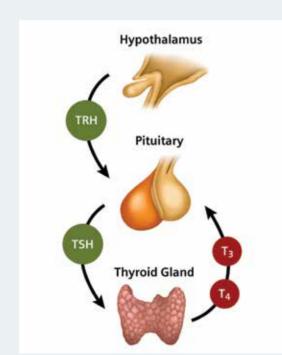


Figure 1. The hypothalamic–pituitary–thyroid axis. In euthyroid subjects, TRH stimulates production and release of TSH, which in turn stimulates production and release of the thyroid hormones T₃ and T₄. Rising thyroid hormone levels have a major inhibitory effect on TSH and a lesser inhibitory effect on TRH production in a negative feedback loop.

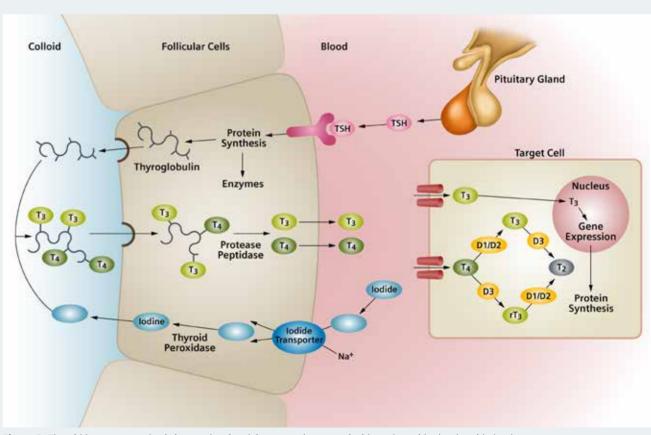


Figure 2. Thyroid hormone synthesis is complex, involving steps that occur inside and outside the thyroid gland.

The classic signs of GD are goiter (thyroid swelling) with hyperthyroidism, orbitopathy (characteristic eye disease), and pretibial myxedema (swelling above the tibia as a result of mucopolysaccharide deposition). Many patients, however, present with only some of these signs or different ones that may be associated with a number of other conditions as well and therefore are not sufficient for diagnosis. These common signs and symptoms include nervousness, impaired concentration, tachycardia, impaired visual focusing, fatigue, heat intolerance, excessive sweating, weight loss, and poor sleep. Because these signs and symptoms are neither sensitive nor specific, laboratory testing must complement the clinical examination to arrive at a diagnosis. 3.6

How is Graves' Disease Treated?

Treatment of thyrotoxicosis is determined by the underlying cause. Standard treatments for GD consist of antithyroid drugs (ATD) (to inhibit the iodination and coupling steps of thyroid hormone synthesis), iodine-131 (radioiodine) therapy, and surgery (thyroidectomy).³

What are the Important Conditions Associated with Graves' Disease?

Graves' Ophthalmopathy

About 30% to 50% of patients with GD experience the associated ophthalmopathy.⁷ It is also referred to as Graves' orbitopathy (GO). For many patients, the most distressing complication of GD is the ophthalmopathy and its associated symptoms: exposure keratitis (inflammation of the cornea associated with pain and impaired vision); and proptosis (forward displacement/bulging of the eyeball out of the socket), resulting in dry eyes, eyelid retraction, and the inability to close the eye (associated with a high risk of eye infection and ulceration).⁸ About 5% of affected patients have disease that results in vision impairment or threatens sight.⁷

Thyrotoxicosis in pregnancy

Thyrotoxicosis is present in about 0.1% to 0.4% of pregnancies,4 GD being the cause in 85% of these cases.9 In women in their reproductive years with GD, several presentations are possible. They may 1) be undiagnosed for hyperthyroidism until pregnant, though previously symptomatic; 2) be on antithyroid therapy before conception because of a prior diagnosis of hyperthyroidism; 3) have recurring hyperthyroidism following antithyroid therapy-induced remission; 4) have been treated with iodine-131 or surgery; or 5) have given birth to an infant with thyroid dysfunction. Affected patients experience the most severe symptoms during the first trimester and in the postpartum period.4 Treatment during pregnancy is necessary to prevent complications in mother and child, both in utero and postpartum (Table 1). ATD, the preferred treatment option in pregnancy, reduced the rate of fetal loss from 45% in untreated hyperthyroid women (before thiouracil drugs came into use) to 7.2%.4

Neonatal hyperthyroidism

Of the children born to mothers with GD. 1% to 5% will develop hyperthyroidism.4 Neonatal hyperthyroidism due to maternal GD develops as a result of the transmission of maternal TSI to the fetus in utero.9,10 The duration of neonatal hyperthyroidism is dependent on the half-life (2 to 8 weeks) of the TSI.4 Affected neonates develop the symptoms associated with hyperthyroidism within 1 to 3 days after birth, with congestive heart failure among the most serious possible manifestations. Other possible manifestations in babies born to mothers with very high levels of TSI—which may be the case if the mother has undergone ablative therapy before pregnancy but has persistent, high TSI levels—include accelerated bone maturation, craniosynostosis, and small size for gestational age (Table 1).4 Early diagnosis and treatment are essential to ameliorate and prevent short- and longterm complications associated with hyperthyroidism.¹⁰

Table 1. Maternal, fetal, and neonatal complications associated with maternal GD (adapted from references 4, 10, and 11).

Maternal Complications	Fetal/Neonatal Complications	
Placenta abruptio	Intrauterine growth restriction	Goiter
Miscarriage	Small size for gestational age	Tachycardia
Preterm delivery	Hyperthyroidism	Low birth weight
Congestive heart failure	Hypothyroidism	Prematurity
Thyroid storm	Accelerated bone maturity	Stillbirth
Pre-eclampsia	Craniosynostosis	Heart failure

What Is the Role of Laboratory Testing in Graves' Disease?

Diagnosis

Laboratory tests play important roles in the initial diagnosis of GD. The typical profile of Graves' hyperthyroidism is low TSH and high free T4. In some cases, the profile is low TSH, normal free T4, and high free T3 levels. A sensitive TSH test is the recommended screening test in patients at risk for hyperthyroidism. The TSH test should be done in conjunction with a free T4 test and, in select cases, a free T3 test. TBII and TSI assays are also recognized as suitable tools for the diagnosis of GD and for differentiating this condition from other thyrotoxicoses in various diagnostically challenging scenarios. The presence of TSI is strongly indicative of GD: TSI are detected in 77.8% to 96.0% of GD patients.

Treatment Monitoring

Following the initial diagnosis of GD, clinicians continue to use TSH and free T₄ assays to assess response to treatment and for long-term follow-up.^{5,14} Established and reported uses of TSI measurement for treatment monitoring and prognosis are examined below.

What is the difference between the TRAb assays currently available?

There are two different types of TRAb assays currently available. 1) TBII are receptor assays that measure thyroid binding inhibiting immunoglobulins. These are the antibodies that block binding of TSH to an in vitro TSHR preparation and do not differentiate between stimulating and blocking antibodies in serum samples. Their inability to differentiate between the functional properties of TRAb prevent them from accurately predicting GD phenotype in every patient. Third generation TBII assays have made slight improvements to this issue. 2) TSI are currently cell bioassays and measure the ability of TRAb to stimulate

or inhibit TSHR activity. Therefore they are capable of differentiating between the stimulating and blocking antibodies. The TSI assay shows close correlation with the results of RAIU, thyroid hormone levels and the presence of GO. TBII assays lack this relationship, making TSI a better biologic marker and preferred for diagnostic purposes ^{15, 16}

What is the Utility of TSI in Graves' Disease?

The ability of TSI assays to differentiate stimulating antibodies from all other antibodies that bind to the TSH receptor offers a unique and important view into the dynamics of the autoimmune process that drives GD. The utility reported specifically for TSI assays is examined in this section.

The more firmly established applications of TBII and TSI assays are in the differential diagnosis of GD in diagnostically challenging cases, in assessing maternal GD in pregnancy with its attendant risk of neonatal hyperthyroidism, and as a guide for determining the duration of ATD. (Table 2).^{5,7}

Graves' disease diagnosis

Although most cases of GD can be diagnosed on the basis of clinical signs (along with results of TSH, T4, and T3 assays), the value of TRAb testing becomes evident in clinically ambiguous, challenging cases: euthyroid or hypothyroid Graves' ophthalmopathy (GO), unilateral exophthalmos, subclinical hyperthyroidism, painless thyroiditis, GD in elderly patients, thyrotoxicosis associated with hyperemesis gravidarum, thyrotoxicosis resulting from amiodarone therapy, and GD in endemic iodine-deficient areas with a high incidence of toxic multinodular goiter. ^{17,18}

Some experts propose TRAb testing in all thyrotoxic patients because of its usefulness for differential diagnosis.⁷ Others advocate a more targeted use of such tests.^{17,18} That TRAb testing is currently not more

Table 2. Indications for TRAb/TSI testing (adapted from references 5 and 7).

Indication	Comments
Diagnosis of GD	Aids in differentiating GD from other forms of thyrotoxicosis
Ophthalmopathy	Useful for differential diagnosis of GD in patients with unilateral orbitopathy or orbitopathy with euthyroid or hypothyroid status
Pregnancy	Useful in women who are currently on antithyroid drug therapy, have had either radioactive iodine or surgery for thyrotoxicosis, or have had children with neonatal thyrotoxicosis
Therapy guidance	Helps identify patients on ATD who are more likely to remit

frequently employed is likely a reflection of tradition and concerns over cost and clinical reliability, but the major limitations of early TRAb assays have been overcome in later generations of assays such as third generation TBII and the TSI assay.^{7,17,19}

TSI assays have been shown by many studies to offer better sensitivities and specificities than TRAb (TBII) assays. The results can vary depending on the populations tested, i.e., active versus treated GD. A study by Daud et al.,²⁰ 33 patients with active GD and 24 patients with treated GD were tested with a TSI assay, a TRAb ELISA, and a TRAb radioreceptor assay. The overall sensitivities were 100%, 84.2%, and 71.9%, respectively.

Another study included active (untreated) Graves' patients and showed that the TSI assay had superior sensitivity at 80% compared to the TRAb method at 63%; specificity was 100% for both methods. Using a different cutoff, the authors obtained a sensitivity for the TSI assay of 86%.²¹

Grippa et al. tested sera from 56 GD patients, 140 normal individuals, 10 pregnant women, and 38 patients with other diseases using a TSI assay. Clinical sensitivity and specificity were 95% and 99%, respectively. The investigators also compared results from this TSI assay and a TRAb assay on a subset of samples: 30 Graves' disease samples and 68 non-Graves' disease samples consisting of 11 non-GD hyperthyroid, 14 Hashimoto's disease, 3 rheumatoid arthritis, and 40 clinically healthy normal samples. Clinical sensitivity and specificity were 97% and 99% for the TSI assay and 67% and 99% for the TRAb assay, respectively.²²

For reasons of economy and efficiency, TSI tests have been proposed as an alternative approach that can help distinguish GD from other forms of thyrotoxicosis and reduce the use of thyroid scintigraphy. A TSI assay utilization study used an evidence-based economic model to evaluate whether the inclusion of TSI assays for the diagnosis of hyperthyroidism affected time to diagnosis and annual costs in a population of 100,000 managed-

TSI and Hyperthyroidism

TRAb that stimulate receptor activity are known as thyroid-stimulating immunoglobulins (TSI) or thyroid-stimulating antibodies (TSAb). The binding of TSI to the TSH receptor leads to increased/excess production of thyroid hormones. These antibodies are associated with the development of hyperthyroidism, particularly, Graves' disease.

care enrollees. The study found that including TSI in all hyperthyroidism testing algorithms would reduce net diagnostic costs by up to 43%, reduce net costs of misdiagnosis and treatment of unexplained symptoms by up to 85%, reduce overall net costs by up to 47%, and reduce time to diagnosis by up to 46%.²³

Graves' Ophthalmopathy

GO is a common finding in GD, afflicting up to half of GD patients A diagnosis of GD is more difficult when hyperthyroidism develops after GO, which is true for up to 18% of GD patients (in most cases, within a year).²⁴ GO can also occur in euthyroid and hypothyroid patients, with a reported prevalence ranging from less than 3% to more than 30%.²⁵

Euthyroid and hypothyroid patients present their own diagnostic challenges: typically, asymmetrical (that is, unilateral) ocular and other clinical manifestations that are more subtle than those of hyperthyroid GO patients.²⁴

A study by Ponto et al. found that among individuals with GO, 98% (106/108) were positive for TSI. The presence of TSI correlated with GO activity (determined by clinical activity score [CAS] evaluating classic inflammatory signs) and severity (determined by clinical severity score). TSI levels were higher in moderate-to-severe ophthalmopathy than in mild ophthalmopathy; and all patients with active GO were positive for TSI, whereas 84% were positive for TBII. Patients who had four of the seven symptoms assessed by the CAS all had higher levels of TSI compared to individuals without these symptoms.¹⁶

Khoo et al., examining 100 nonsmokers newly diagnosed with GD and not yet treated, found that TSI levels but not TBII levels correlated positively with GO prevalence. A total of 43 patients had GO. When all patients were stratified into quartiles by increasing TSI levels, the prevalence of GO in these groups was 20%, 36%, 52%, and 64%. The authors also found a higher correlation with GO when TSI was elevated and TPOAb was absent.²⁶

Lytton et al., who have called TSI "functional biomarkers for GO," also found an association between TSI and the activity and severity of GO. TSI was present in 97% of patients with Graves' ophthalmology and 95% of patients with GD. The correlation (r) between TSI presence and CAS in GO was 0.87 and 0.7, respectively, compared with TRAb assays that demonstrated lower correlations of 0.17 and 0.54. This study also demonstrated higher clinical sensitivity and specificity for the TSI assay compared to the TRAb assay: 97% versus 77% sensitivity and 89% versus 43% specificity.²⁷

A study found that ordering TSI in all hyperthyroidism testing algorithms would reduce net costs of misdiagnosis by up to 85%.²³

Jang et al., investigating the relationship between TSI values and each component of the CAS, found that TSI strongly correlated with all seven components. Compared to two TBII assays, the TSI assay showed significantly higher correlation with CAS and thus greater utility for assessing both inflammatory activity and severity of GO.²⁸

Pregnancy and neonatal hyperthyroidism

Maternal thyroid disorders may affect fetal and neonatal thyroid function through placental transfer of autoantibodies. In the case of Graves' disease, TSH receptor-stimulating antibodies and antithyroid drugs readily cross the placenta with the possibility of causing hyperthyroidism in the neonate. The drugs are cleared from the neonatal circulation within the first few days, but the antibodies will disappear more slowly, stimulating the thyroid and causing a life-threatening hyperthyroid condition in the neonate.29 Data indicate that 2% to 10% of pregnant women with active Graves' disease will have hyperthyroid newborns. The best predictor of neonatal hyperthyroidism is a high level of TSI in the pregnant woman measured in the last trimester. The following are several guidelines from a European Thyroid Association symposium that have been published for the measurement of TSH-receptor antibodies in pregnancy: 1) A euthyroid pregnant woman (with or without thyroid hormone replacement therapy) who has previously received radioiodine therapy or undergone thyroid surgery for Graves' disease should be measured for TSH-receptor antibodies/TSI early in pregnancy. If the level is high the fetus should be followed carefully for signs of hyperthyroidism. TSI should be measured again in the last trimester to evaluate the risk for neonatal hyperthyroidism. 2) A pregnant woman who takes antithyroid drugs for Graves' disease to keep thyroid function normal should be measured for TSH receptor antibodies in the last trimester. If antibody levels are high, evaluation for neonatal hyperthyroidism is needed. Assays specifically measuring antibodies stimulating the thyroid have clear theoretical advantage over other TSH receptor antibody assays.²⁹ Measurement of TSI in the pregnant woman or neonate is also helpful in distinguishing a transient autoimmune form of neonatal hyperthyroidism from other, non-immune forms of congenital hyperthyroidism.³⁰

Once a diagnosis of neonatal hyperthyroidism has been made, Polak et al. advise confirming the presence of TSI to verify the autoimmune nature of the hyperthyroidism. Postpartum cord blood may be used for thyroid function tests; and cord blood TSI may be predictive of congenital hyperthyroidism. The presence of TSI with a low TSH is consistent with a diagnosis of transient hyperthyroidism due to maternal GD.¹⁰

Another clinical challenge is distinguishing GD from postpartum thyroiditis in women who had been euthyroid. TPOAb positivity does not settle the question, but TSI measurement is clearly valuable for supporting a diagnosis of GD in this context.¹⁸

The guidelines of the American Thyroid Association and American Association of Clinical Endocrinologists Taskforce on Hyperthyroidism and Other Causes of Thyrotoxicosis⁵ recommend the following):

- Diagnosed with GD for the first time during pregnancy
- Measure TRAb at diagnosis and if high, again at gestational weeks 22–26.
- Have a history of GD for which they are on antithyroid therapy during pregnancy
- Measure TRAb at diagnosis and if high, again at weeks 22–26, or measure for the first time at weeks 22–26.
- Have undergone surgical or radioiodine ablative therapy
- Measure TRAb at diagnosis and if high, again at weeks 22–26, or measure for the first time at weeks 22–26.
- Have had a child with hyperthyroidism diagnosed in utero or postpartum.

The same guidelines advise that TRAb testing is unnecessary for women who have completed a course of ATD and have achieved euthyroid status.⁵

TSI may persist in serum even in GD patients who have become euthyroid following ATD or who are hypothyroid after surgical or radioiodine thyroid ablation. TSI can cause fetal hyperthyroidism after crossing the placenta, with serious implications for mother and child. Fetal indications for measuring TSI include tachycardia, goiter detected by ultrasound, and intrauterine growth restriction.

Neonatal hyperthyroidism, also known as congenital hyperthyroidism, is rare, with a prevalence of 0.6% to 1% in babies born to mothers with GD. The most common type of neonatal hyperthyroidism is a transient occurrence caused by the transplacental passage of maternal TSI.¹⁰

The drug methimazole (MMI), a common ATD used to treat GD, also crosses the placenta and affects the fetal thyroid, creating a risk for fetal hypothyroidism. Maternal TSI measurements can help determine when MMI can be discontinued, thereby removing the threat to normal fetal thyroid function.⁵

After birth, MMI is metabolized more rapidly in the newborn than the maternal TRAb which, with a half-life of 2 to 8 weeks, persist longer.⁴ Elevated TRAb in late pregnancy can thus signal the need to carefully monitor the newborn, in whom neonatal hyperthyroidism could appear a few days after birth.⁵ Newborns in whom hyperthyroidism is suspected should have T₄, T₃, and TSH testing to confirm the diagnosis and receive testing for TSI to verify the autoimmune nature of the hyperthyroidism.¹⁰ If untreated, neonatal hyperthyroidism carries the risk of dramatic effects, including impaired growth and development, mental deficiency, and death.^{4,10}

Prognosis

Various studies report that TSI testing is a valuable tool in predicting recurrence of GD following antithyroid drug (ATD) therapy,^{7,17,18,32} useful as a guide for continuing ATD,¹¹ and valuable for predicting the course of GO.³² Such utility would have important implications for patient management, allowing a more personalized treatment program for individual patients. Investigators who call for further evidence to confirm a prognostic role for TRAb assays cite the need for clarification because of variables in study populations, assay architecture, and study designs, as well as the molecular heterogeneity of TRAb themselves.^{7,18} Despite a lack of consensus on all prognostic applications reported for TRAb/TSI assays, recommendations for their use in certain contexts appear in recently published guidelines.

Hyperthyroidism of Graves' disease. Various studies have investigated the prognostic value of TSI assays on their own merit or compared to TBII assays. Nearly three decades ago, Madec et al. observed that high post-ATD TSI values, but not pretreatment values, predicted relapse of GD. Out of 55 patients who were treated and followed for 2 years, 16 had persistently high TSI levels; of these, 13 (81.3%) relapsed. Low or undetectable posttreatment TSI levels did not predict remission: of the remaining 39 patients with low or undetectable post-ATD TSI, 41% relapsed.³³

In another small study, 55 untreated hyperthyroid GD patients who received ATD for 12 to 48 months were followed for 12 to 120 months. TSI (by their in-house method) was high in 21 out of 27 patients who relapsed and low in 27 of 28 patients who went into remission, for a sensitivity of 77.8%, a specificity of 96.4%, and positive and negative predictive values of 95.4% and 81.8%, respectively. These statistics were slightly better than those for a commercially available TRAb assay and comparable to those for a commercially available TSI assay. Thyroid volume contributed additional prognostic value.³⁴

Takasu et al., following the TSAb levels of 98 GD patients over 10 years who received ATD, also reported a correlation between disappearance of TSAb and remission of Graves' hyperthyroidism. Out of 73 patients whose TSAb disappeared, 60 (i.e., 82%) achieved remission of their GD. In contrast, out of 10 patients who had persistently elevated TSAb levels, none achieved remission. The remaining patients experienced more complex disease courses.³⁵

In the context of radioiodine therapy for GD, preand post-treatment thyroid size are the most important prognostic factors in terms of thyroid function after therapy: a large thyroid correlates with less responsiveness, and a large reduction in thyroid volume is associated with better response. But TSI measurements also reportedly have prognostic value: patients who remained hyperthyroid after treatment had higher TSI levels than patients who became hypothyroid or euthyroid. In addition, elevation of TSI post-treatment correlates with the development of hypothyroidism (which the authors postulated is the result of an augmented immune response to the increased availability of TSH receptor escaping from damaged thyroid cells. TSI measurement within a year of radioiodine therapy thus could help predict patient clinical status.36

Graves' ophthalmopathy. In support of the prognostic utility of TSI measurements in GO, Eckstein et al. tabulated odds ratios for the probabilities of a patient's TRAb levels indicating mild or severe courses of GO when compared to various TRAb cutoffs for 3-month intervals spanning a 2-year period.³¹ Furthermore, the Recommendations of the Thyroid Section of the German Society of Endocrinology recognize the use of TRAb/TSI measurements for predicting the course of GO and suggest that they may help the physician in therapy-related decisions.³⁷

As already noted, one study by Jang et al., ³⁸ found the performance of a current TBII assay and TSI assay to be comparable, but a separate study of theirs reported additional value of TSI because it correlated with all seven components of the CAS, whereas TBII correlated with only one component. ²⁷

Conclusion

TSI has demonstrated utility in the diagnosis and management of GD. The recommended uses, as indicated by current guidelines,⁵ center on the diagnosis of hyperthyroidism in clinically ambiguous cases, including GO in euthyroid patients; in pregnant women with treated or newly diagnosed GD, with a view to addressing the maternal GD and the risk of fetal and neonatal thyroid dysfunction; and in guiding the duration of ATD.

Some investigators envision a more expansive use of TRAb/TSI testing that would encompass routine use of the test for the initial diagnosis of GD. Concerns over such a view are usually associated with assay availability, cost, labor intensiveness (particularly with regard to bioassays), and poor performance of early TRAb assays, as well as physicians' customary practice in test ordering.^{7,17,19}

Incorporation of TSI assays into routine clinical use can improve GD patient management through their particular value in assessing disease activity and severity and in providing prognostic value with regard to treatment.³⁹ In the special case of pregnancy and the serious implications of maternal GD for maternal, fetal, and neonatal health, there is a desire for the additional characterization of autoantibodies that TSI assays offer.⁴⁰ Research supports the prospect of significant cost savings, reduction of patients' exposure to radioactive tests, and improved diagnostic efficiency through the incorporation of TSI assays into routine diagnostic testing schemes.^{11,23}

Although there is much literature on the value of TRAb assays as a clinical tool for diagnosis and monitoring of Graves' disease, TSI which is not as readily available has clear advantages as pointed out throughout this paper. Currently, orders for TRAb assays exceed those for TSI assays, even though TSI assays specifically measure the stimulating antibodies that cause the hyperthyroidism of GD. This situation may be explained by the lesser availability of TSI assays, even in many high-volume laboratories. There is a need for an automated TSI assay that is less labor intensive and therefore less costly, 17 as well as faster than those available today.

References

- The Merck manual for health care professionals [Internet].
 Whitehouse Station (NJ, USA): Merck Sharp & Dohme Corp.
 (subsidiary of Merck & Co., Inc.); c2010-2013.
 Thyroid disorders / Overview of thyroid function; reviewed/modified 2012 May [cited 2013 Feb 19]. Available from:
 http://www.merckmanuals.com/professional/endocrine_and_metabolic_disorders/thyroid_disorders/overview_of_thyroid_function.html
- The Merck manual for health care professionals [Internet].
 Whitehouse Station (NJ, USA): Merck Sharp & Dohme Corp.
 (subsidiary of Merck & Co., Inc.); c2010-2013.
 Thyroid disorders / Hypothyroidism (myxedema); reviewed/modified 2012 May [cited 2013 Feb 19]. Available from: http://www.merckmanuals.com/professional/endocrine_and_metabolic_disorders/thyroid_disorders/hypothyroidism.html
- The Merck manual for health care professionals [Internet].
 Whitehouse Station (NJ, USA): Merck Sharp & Dohme Corp.
 (subsidiary of Merck & Co., Inc.); c2010-2013.
 Thyroid disorders / Hyperthyroidism (thyrotoxicosis); reviewed/modified 2012 May [cited 2013 Feb 19]. Available from: http://www.merckmanuals.com/professional/endocrine_and_metabolic_disorders/thyroid_disorders/hyperthyroidism.html
- Patil-Sisodia K, Mestman JH. Graves hyperthyroidism and pregnancy: a clinical update. Endocr Pract. 2010 Jan-Feb;16(1):118-29.
- Bahn RS [Chair], Burch HB, Cooper DS, Garber JR, Greenlee MC, Klein I, Laurberg P, McDougall IR, Montori VM, Rivkees SA, Ross DS, Sosa JA, Stan MN; American Thyroid Association; American Association of Clinical Endocrinologists. Hyperthyroidism and other causes of thyrotoxicosis: management guidelines of the American Thyroid Association and American Association of Clinical Endocrinologists. Thyroid. 2011 Jun;21(6):593-646. Epub 2011 Apr 21.
- Winter WE, Schatz D, Bertholf RL. The thyroid: pathophysiology and thyroid function testing. In: Burtis CA, Ashwood ER, Bruns DE, editors. Tietz textbook of clinical chemistry and molecular diagnostics. 5th ed. St. Louis: Saunders (Elsevier); 2012.
- 7. Kamath C, Adlan MA, Premawardhana LD. The role of thyrotrophin receptor antibody assays in Graves' disease. J Thyroid Res. 2012;2012:525936. Epub 2012 Apr 19. 21. Lounsbury R, Gupta M. Clinical evaluation of thyroid stimulating autoantibody [TSAb] reporter assay using chimeric TSH receptor transfected CHO cell line: comparison with radio receptor binding.
- 8. ACP Medicine; a publication of the American College of Physicians [Internet]. Hamilton (Ontario, Canada): Decker Intellectual Properties; c2012. Ladenson PW. Thyroid; [cited 2013 Aug 12]. Available from: http://www.acpmedicine.com/acpmedicine/secured/searchReader.action?bookId=ACP&chapId=1229&partId=part04&type=chapID&start=0&heading=chapter&from=Chapter&hitStart=0#
- De Groot L, Abalovich M, Alexander EK, Amino N, Barbour L, Cobin RH, Eastman CJ, Lazarus JH, Luton D, Mandel SJ, Mestman J, Rovet J, Sullivan S. Management of thyroid dysfunction during pregnancy and postpartum: an Endocrine Society clinical practice quideline. J Clin Endocrinol Metab. 2012 Aug;97(8):2543-65.
- Polak M, Legac I, Vuillard E, Guibourdenche J, Castanet M, Luton D. Congenital hyperthyroidism: the fetus as a patient. Horm Res. 2006;65(5):235-42. Epub 2006 Mar 30.

- Barbesino G, Tomer Y. Clinical utility of TSH receptor antibodies.
 J Clin Endocrinol Metab. 2013 Jun;98(6):2247-55.
 Epub 2013 Mar 28.
- Macchia E, Concetti R, Borgoni F, Cetani F, Fenzi GF, Pinchera A. Assays of TSH-receptor antibodies in 576 patients with various thyroid disorders: their incidence, significance and clinical usefulness. Autoimmunity. 1989;3(2):103-12.
- Takasu N, Oshiro C, Akamine H, Komiya I, Nagata A, Sato Y, Yoshimura H, Ito K. Thyroid-stimulating antibody and TSH-binding inhibitor immunoglobulin in 277 Graves' patients and in 686 normal subjects. J Endocrinol Invest. 1997 Sep;20(8):452-61.
- 14. Beastall GH, Beckett GJ, Franklyn J, Fraser WD, Hickey J, John R, Kendall-Taylor P, Nevens B, Vanderpump M. UK guidelines for the use of thyroid function tests [Internet]. London: Association for Clinical Biochemistry; British Thyroid Association; British Thyroid Foundation; 2006 Jul [cited 2013 Apr 11]. 86 p. Available from: http://www.british-thyroid-association.org/info-for-patients/Docs/TFT guideline final version July 2006.pdf
- Gupta Manjula. Thyrotropin-receptor antibodies in thyroid diseases: Advances in detection techniques and clinical applications. Clinica Chimica Acta 293 (2000) 1-29.
- Ponto KA, Kanitz M, Olivo PD, Pitz S, Pfeiffer N, Kahaly GJ. Clinical relevance of thyroid-stimulating immunoglobulins in Graves' ophthalmopathy. Ophthalmology. 2011 Nov;118(11):2279-85.
- Matthews DC, Syed AA. The role of TSH receptor antibodies in the management of Graves' disease. Eur J Intern Med. 2011 Jun;22(3):213-6. Epub 2011 Mar 22.
- 18. Winter WE, Jialal I, Devaraj S. Thyrotropin receptor antibody assays: clinical utility. Am J Clin Pathol. 2013 Feb;139(2):140-2.
- Tozzoli R, Bagnasco M, Giavarina D, Bizzaro N. TSH receptor autoantibody immunoassay in patients with Graves' disease: improvement of diagnostic accuracy over different generations of methods. Systematic review and meta-analysis. Autoimmun Rev. 2012 Dec;12(2):107-13. Epub 2012 Jul 7.
- 20. Daud S, Hsu A, Gupta M. TRAbs in Graves' disease as detected by TSAb, TBI-ELISA and TBI-RRA. Thyroid 2008;19(Suppl 1):S40
- 21. Lounsbury R, Gupta M. Clinical evaluation of thyroid stimulating autoantibody [TSAb] reporter assay using chimeric TSH receptor transfected CHO cell line: comparison with radio receptor binding assay [poster][Internet]. Cleveland (OH): Cleveland Clinic; 2009 [cited 2013 Sep 4]. Available from: http://thyretain.com/ physicians home/resources/resource posters/
- 22. Grippa L, Stockwell J, Kelly R, Houtz JA, Brown J, Scholl D, Kohn LD, Li Y. Detection of serum thyroid stimulating immunoglobulin (TSI) in Graves' disease (GD) using a novel cell-based reporter assay [Internet]. Poster session presented at: 25th Annual Clinical Virology Symposium (CVS); 2009 Apr 19–22; Daytona Beach, FL, US. Available from http://thyretain.com/physicians_home/ resources/resource_posters/
- 23. McKee A, Peyerl F. TSI assay utilization: impact on costs of Graves' hyperthyroidism diagnosis. Am J Manag Care. 2012 Jan 1;18(1):e1-14.

- Eckstein AK, Lösch C, Glowacka D, Schott M, Mann K, Esser J, Morgenthaler NG. Euthyroid and primarily hypothyroid patients develop milder and significantly more asymmetrical Graves ophthalmopathy. Br J Ophthalmol. 2009 Aug;93(8):1052-6. Epub 2009 Feb 15.
- 25. Lazarus JH. Epidemiology of Graves' orbitopathy (GO) and relationship with thyroid disease. Best Pract Res Clin Endocrinol Metab. 2012 Jun;26(3):273-9.
- 26. Khoo DH, Ho SC, Seah LL, Fong KS, Tai ES, Chee SP, Eng PH, Aw SE, Fok AC. The combination of absent thyroid peroxidase antibodies and high thyroid-stimulating immunoglobulin levels in Graves' disease identifies a group at markedly increased risk of ophthalmopathy. Thyroid. 1999 Dec;9(12):1175-80.
- Lytton SD, Ponto KA, Kanitz M, Matheis N, Kohn LD, Kahaly GJ.
 A novel thyroid stimulating immunoglobulin bioassay is a functional indicator of activity and severity of Graves' orbitopathy. J Clin Endocrinol Metab. 2010 May;95(5):2123-31.

 Epub 2010 Mar 17.
- Jang SY, Shin DY, Lee EJ, Choi YJ, Lee SY, Yoon JS.
 Correlation between TSH receptor antibody assays and clinical manifestations of Graves' orbitopathy. Yonsei Med J. 2013 Jul;54(4):1033-9.
- Laurberg P, Nygaard B, Glinoer D, Grussendorf M, Orgiazzi J. Guidelines for TSH-receptor antibody measurements in pregnancy: results of an evidence-based symposium organized by the European Thyroid Association. Eur J Endocrinol. 1998 Dec;139(6):584-6.
- Polak M. Hyperthyroidism in early infancy: pathogenesis, clinical features and diagnosis with a focus on neonatal hyperthyroidism. Thyroid. 1998 Dec;8(12):1171-7.
- 31. Laurberg P, Bournaud C, Karmisholt J, Orgiazzi J. Management of Graves' hyperthyroidism in pregnancy: focus on both maternal and foetal thyroid function, and caution against surgical thyroidectomy in pregnancy. Eur J Endocrinol. 2009 Jan;160(1):1-8. Epub 2008 Oct 10.
- Eckstein A, Esser J, Mann K, Schott M. Clinical value of TSH receptor antibodies measurement in patients with Graves' orbitopathy. Pediatr Endocrinol Rev. 2010 Mar;7 Suppl 2:198-203.
- 33. Madec AM, Laurent MC, Lorcy Y, Le Guerrier AM, Rostagnat-Stefanutti A, Orgiazzi J, Allannic H. Thyroid stimulating antibodies: an aid to the strategy of treatment of Graves' disease? Clin Endocrinol (Oxf). 1984 Sep;21(3):247-55.
- 34. Giuliani C, Cerrone D, Harii N, Thornton M, Kohn LD, Dagia NM, Bucci I, Carpentieri M, Di Nenno B, Di Blasio A, Vitti P, Monaco F, Napolitano G. A TSHR-LH/CGR chimera that measures functional thyroid-stimulating autoantibodies (TSAb) can predict remission or recurrence in Graves' patients undergoing antithyroid drug (ATD) treatment. J Clin Endocrinol Metab.
 2012 Jul:97(7):E1080-7. Epub 2012 Apr 6.
- 35. Takasu N, Matsushita M. Changes of TSH-stimulation blocking antibody (TSBAb) and thyroid stimulating antibody (TSAb) over 10 years in 34 TSBAb-positive patients with hypothyroidism and in 98 TSAb-positive Graves' patients with hyperthyroidism: reevaluation of TSBAb and TSAb in TSH-receptor-antibody (TRAb)-positive patients. J Thyroid Res. 2012;2012:182176. Epub 2012 May 10.

- Chiovato L, Fiore E, Vitti P, Rocchi R, Rago T, Dokic D, Latrofa F, Mammoli C, Lippi F, Ceccarelli C, Pinchera A. Outcome of thyroid function in Graves' patients treated with radioiodine: role of thyroid-stimulating and thyrotropin-blocking antibodies and of radioiodine-induced thyroid damage. J Clin Endocrinol Metab. 1998 Jan;83(1):40-6
- 37. Eckstein A, Mann K, Kahaly GJ, Grussendorf M, Reiners C, Feldkamp J, Quadbeck B, Bockisch A, Schott M. [Role of TSH receptor autoantibodies for the diagnosis of Graves' disease and for the prediction of the course of hyperthyroidism and ophthalmopathy. Recommendations of the Thyroid Section of the German Society of Endocrinology]. [Article in German] Med Klin (Munich). 2009 May 15;104(5):343-8. Epub 2009 May 16.
- 38. Jang SY, Shin DY, Lee EJ, Lee SY, Yoon JS. Relevance of TSH-receptor antibody levels in predicting disease course in Graves' orbitopathy: comparison of the third-generation TBII assay and Mc4-TSI bioassay. Eye (Lond). 2013 Aug;27(8):964-71. Epub 2013 Jun 7
- 39. Lytton SD, Kahaly GJ. Bioassays for TSH-receptor autoantibodies: an update. Autoimmun Rev. 2010 Dec;10(2):116-22. Epub 2010 Aug 31.
- Leschik JJ, Diana T, Olivo PD, König J, Krahn U, Li Y, Kanitz M, Kahaly GJ. Analytical performance and clinical utility of a bioassay for thyroid-stimulating immunoglobulins. Am J Clin Pathol. 2013 Feb;139(2):192-200.

10