

www.siemens.com/molecular

kPCR PLX BK Virus (BKV) DNA Assay

VERSANT kPCR Molecular System

Assay Description

The kPCR PLX BK Virus (BKV) DNA assay is an in vitro diagnostic test for the detection and quantification of BKV-specific DNA in human EDTA plasma specimens. The assay is based on real-time PCR technology, utilizing polymerase chain reaction (PCR) for the amplification of specific target sequences and target-specific probes for the detection of the amplified DNA. The kPCR PLX family of assays is configured for use with the VERSANT® kPCR Molecular System and the VERSANT Sample Preparation 1.2 Reagents Kit.

Clinical Background

- The human polyomaviruses BK Virus (BKV) establishes ubiquitous infections worldwide.
- Primary infections generally occur in early childhood and are typically subclinical, followed by a lifelong persistence.
- Whereas polyomavirus infections remain unapparent in immunocompetent individuals, reactivation of BKV in association with immunosuppression may lead to serious diseases.
- BKV reactivation is most common in bone marrow and renal transplant patients, resulting in hemorrhagic cystitis (HC) and polyomavirus associated nephropathy (PVAN), respectively.

VERSANT kPCR Molecular System with VERSANT MiPLX Software Solution

Advancing flexibility, workflow, customization, and consolidation in the molecular laboratory.

- Comprehensive and flexible solution for molecular diagnostics.
- Consolidation of quality assays for infectious-disease testing on a single, automated platform.
- Multiplexing of up to six assays from one sample in one run.
- Standardized extraction method for isolation of quality nucleic acids from a wide variety of specimen types.
- Flexible open-channel capabilities enables customization of laboratory-developed and third-party assays.
- Efficient workflow with powerful productivity due to ease of use, minimum hands-on time, maximum walkaway time, and fast turnaround time.

One Solution—More Choice.



Answers for life.

kPCR PLX BK Virus (BKV) DNA Assay


Performance Characteristics	
Description	Value
Limit of detection (LoD)	114 copies/mL
Linear range	2.00 E2 to 1.00 E7 copies/mL
Clinical specificity (negative samples)	≥95%

Product Specifications	
Description	Value
Reagent storage conditions	-20°C
Specimen volume	700 µL
kPCR PLX Universal Negative Control (UNC)	Full process negative control that is extracted and amplified along with the samples


Ordering Information		
Catalog Number	kPCR PLX Assay Menu*	Quantity
10718587	kPCR PLX Adenovirus DNA	72 tests
10718585	kPCR PLX BK Virus (BKV) DNA	72 tests
10718580	kPCR PLX Cytomegalovirus (CMV) DNA	72 tests
10718581	kPCR PLX Epstein-Barr Virus (EBV) DNA	72 tests
10718582	kPCR PLX Herpes Simplex Virus 1 and 2 (HSV) DNA	72 tests
10718584	kPCR PLX Human Herpes Virus 6 (HHV-6) DNA	72 tests
10718586	kPCR PLX JC Virus (JCV) DNA	72 tests
10718598	kPCR PLX Parvovirus B19 DNA	72 tests
10718583	kPCR PLX Varicella-Zoster Virus (VZV) DNA	72 tests
10629800	VERSANT Sample Preparation 1.2 Reagents (Boxes 1 and 2)	96 tests

*kPCR PLX assays include all controls and calibrators in the same kit. CE marked for IVD use.

VERSANT MiPLX Software with Dynamic Protocols allows you to run 1, 2, 4, or 6 assays from one extracted sample in one run.

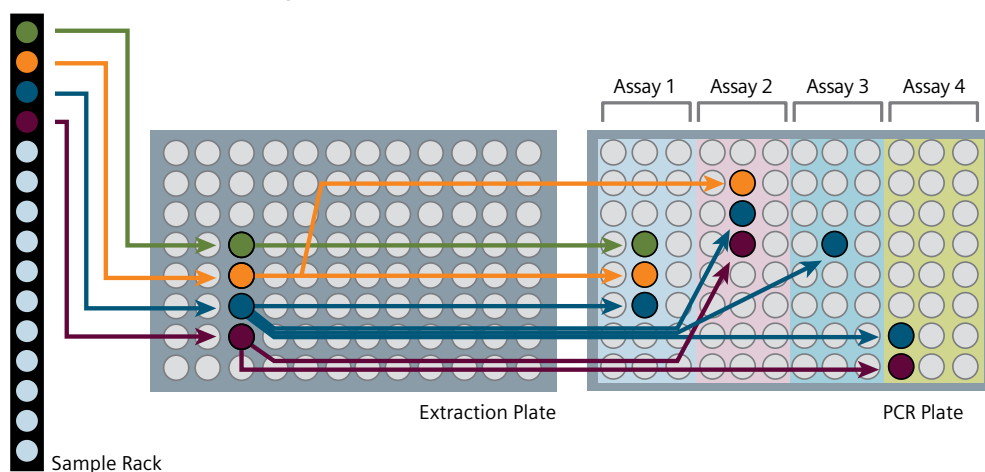

VERSANT MiPLX Software Solution

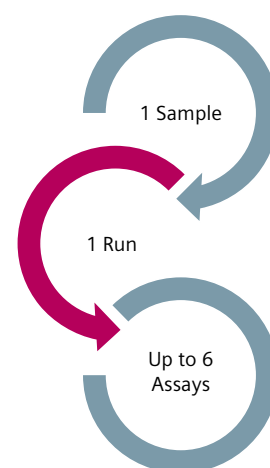

Optimize your laboratory's workflow with the ability to run the kPCR PLX BK Virus (BKV) DNA assay along with other kPCR PLX assays from a single patient specimen.

For customer support, please contact your local technical support provider or distributor or visit www.siemens.com/diagnostics.

VERSANT and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc. or its affiliates. All other trademarks and brands are the property of their respective owners.

kPCR PLX assays are manufactured by Altona Diagnostics GmbH and distributed by Siemens Healthcare Diagnostics Inc.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Global Siemens Headquarters

Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

Global Siemens Healthcare Headquarters

Siemens AG, Healthcare
Henkestrasse 127
91052 Erlangen
Germany
Telephone: +49 9131 84-0
www.siemens.com/healthcare

Global Division

Siemens Healthcare Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591-5005
USA
www.siemens.com/diagnostics

Order No. A91DX-MM-150114-XC1-4A00
03-2015 | All rights reserved
© 2015 Siemens Healthcare Diagnostics Inc.