

Artis zee - Study Protocol

Atrial Fibrillation with LA Injection and Rapid Pacing

Electrophysiology

Supported by

- syngo DynaCT Cardiac
- syngo iPilot
- syngo InSpace3D
- syngo InSpaceEP

Courtesy of

Prof. Hein Heidbuechel, MD,
Joris Ector, MD,
Stijn de Buck,
Department of Cardiology,
University Hospital Gasthuisberg,
Leuven,
Belgium

System & Software

Artis zee with 30 x 40 FD syngo X Workplace

Case Description

Left atrium segmentation for ablation treatment

Diagnosis

Paroxysmal atrial fibrillation

Treatment

Ablation therapy

Tips and Tricks

- Prevent cardiac motion during image acquisition through rapid right ventricular pacing.
- Breathhold maneuver: apnea was induced during acquisition to reduce respiratory motion artifacts.
- Metallic cables should be removed from the field of view.
- Arms can be kept in normal position beside the body.

Protocol content:

- Rapid pacing, started at 240 bpm.
- After verification of good capture, a contrast solution is injected in the left atrium
- After 2 seconds, actual image acquisition is started automatically.
- After the acquisition is completed, pacing is stopped.


"Visualization of the left atrium, ostia of the pulmonary veins and atrial appendage commonly used for guidance in atrial fibrillation procedures."

Atrial Fibrillation with LA Injection and Rapid Pacing

Acquisition Protocol	5sDCT Card
Number of Projections:	248
System Dose:	0.54 μGy/f*
Increment in Degrees:	0.8 °/f
Injection Protocol	
Catheter Position:	Left Atrium
Contrast Medium (CM):	370 mg lodine/ml
Test Bolus:	w/o
Dilution:	60 %
Dilution: Injection Volume:	60 % 100 ml (60 ml CM/ 40 ml Saline)
	100 ml (60 ml CM/
Injection Volume:	100 ml (60 ml CM/ 40 ml Saline)
Injection Volume: Injection Rate:	100 ml (60 ml CM/ 40 ml Saline) 20 ml/s

Reconstruction	
VOI Size:	Full
Slice Matrix:	256 x 256
Kernel Type:	HU
Image Characteristics:	Normal
Reconstruction Mode:	Nat Fill
Viewing Preset:	Golden
Secondary Recon	No

^{*}For software version VC14 and higher, 0.36 uGy/f is used for all 5-second protocols. (Except for low-dose protocols)

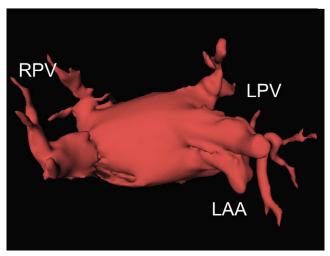

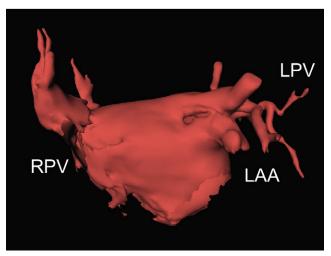


Image segmentation of the left atrium (acquisition during rapid ventricular pacing); left: superior view, right: anterior view.

The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.

All rights reserved.

Siemens Healthcare Headquarters

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen Germany

Phone: +49 9131 84-0 siemens.com/healthcare

PDF only. | © Siemens Healthcare GmbH, 2016