

SIEMENS



Transducer Disinfectants

ACUSON SC2000™ Ultrasound System, PRIME Edition

Approved Pre-Cleaners and High Level Disinfectants for Transducers

Pre-Cleaners

	10V4	4V1c	4Z1c	6C1HD	8V3	9L4	AUX CW	CW2	CW5	V5Ms	V7M	Z6Ms
Acrylan	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	⊙
Alkazyme	⊙	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	⊙
Alkydiol Spray 40	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Ampholysine Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	⊙
ANIOSYME DD1	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	✓
Asepti-Wipe	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	⊙
Asepti-zyme	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓
Asphene 381 Spray	⊙	✓	⊙	⊙	✓	✓	✓	✓	✓	⊙	⊙	⊙
Cavi wipes & Cavicide	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	⊙
Cidalkan	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Cidezyme	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	⊙
CIDEZYME XTRA	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cleanisept Wipe	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cleanisept Wipe forte	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Empower	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓
Endozyme	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	⊙
Engarde	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Enzol	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Erinox	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Esculase	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙
Exidine 4	⊙	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙
Firstsinald	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	⊙	⊙	⊙
Hexanios G+R	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙
Incidine	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Instru-Zyme	⊙	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓
Liv des special 45	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	⊙
Manu-Klenz	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Medallion	⊙	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	⊙
Medi-Prep	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Metrizyme	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
M-Ytidesinfektion	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙
Neodiol +	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	⊙	⊙	⊙
Orthozyme	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	⊙
pH7Q	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	⊙	⊙
PI-Spray	⊙	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
PI-Spray II	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Preven's D51	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙

Pre-Cleaners

	10V4	4V1c	4Z1c	6C1HD	8V3	9L4	AUX CW	CW2	CW5	V5Ms	V7M	Z6Ms
Protex Spray/Wipes	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Protex Ultra wipes	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙
Quat-Stat	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
RBS-50	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Rivascop	✓	✓	⊙	⊙	✓	✓	⊙	⊙	⊙	⊙	⊙	⊙
Sani-Cloth AF3	✓	✓	⊙	⊙	✓	✓	⊙	⊙	⊙	⊙	⊙	✓
Sani-Cloth HB	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓
Sani-Cloth Plus	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙
SaniZide	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙
Savogerm	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Sekulyse	⊙	✓	⊙	✓	✓	✓	✓	✓	✓	✓	⊙	✓
Sekupoudre	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	✓	⊙
Sekusept	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
SONO Ultrasound Wipes	✓	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	✓
Super Sani-Cloth	✓	✓	✓	✗	✓	✓	✓	✓	✓	✓	⊙	✓
TheraCide	✓	✓	⊙	⊙	✓	⊙	✓	✓	✓	⊙	⊙	⊙
Transeptic	✓	✓	✓	✗	✓	✓	✓	✓	✓	✓	✓	✓
Trionic D Wipe	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Virex II 256	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓
WEX-CIDE	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙

Key

✓	Approved
✗	Not Approved
⊙	Not Tested

High-Level Disinfectants

	10V4	4V1c	4Z1c	6C1HD	8V3	9L4	AUX CW	CW2	V5M5 (-S/N 327XXXX)	V5M5 (S/N 328XXXX ~)	V7M	Z6M5
Aidal Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Alkacide	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Asepti-steryl 14	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Asepti-steryl 28	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
BM-28 Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cetylcode-G	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
CIDEX	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙
CIDEX 7	⊙	✓	⊙	⊙	✓	✓	✓	✓	⊙	⊙	⊙	✓
CIDEX OPA	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
CIDEX PA	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	✓	⊙
CIDEX PLUS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Clorox Bleach germicidal cleaner	✓	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Deson	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Dispatch	✓	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓
Endosporine	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Erihyd Forte	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Formac 14	⊙	✓	✓	✓	✓	⊙	✓	✓	⊙	⊙	⊙	✓
Formac 28	⊙	✓	✓	✓	✓	⊙	✓	✓	⊙	⊙	⊙	⊙
Gigasept	⊙	✓	✓	✓	✓	⊙	✓	✓	⊙	⊙	⊙	⊙
GIGASEPT FF	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Glutaraldehyde NS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Glutarcode	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Glutaster Basica	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Instrunet Esporicida	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Korsolex	⊙	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	⊙	✓
Maxicide NS 14 Day	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MedSci	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Metricide	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MetriCide OPA Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MetriCide 28	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Microcide PA	⊙	✓	⊙	⊙	✓	⊙	✓	✓	⊙	⊙	⊙	⊙
MILTON	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Nova Plus NS	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	✓	⊙
Nu-Cidex	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	✓	⊙

High-Level Disinfectants

	10V4	4V1c	4Z1c	6C1HD	8V3	9L4	AUX CW	CW2	V5Ms (-S/N 327XXXX)	V5Ms (S/N 328XXXX ~)	V7M	Z6Ms
Omnicide 14NS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Omnicide 28	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Opal	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Oxivir-TB	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	⊙
Pennysaver +	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Procide 14 N.S.	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Procide 28	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙	⊙	⊙	✓
Procide Plus	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Rapicide	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Resert XL HLD	⊙	✓	⊙	⊙	✓	⊙	⊙	✗	✗	⊙	⊙	✓
SBS Glutar 2%	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Sekusept Easy	⊙	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	✓	⊙	✓
Sekusept Extra	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙
Sporadyne	⊙	✓	⊙	⊙	✓	⊙	✓	✓	⊙	⊙	⊙	⊙
Sporicidin	⊙	✓	⊙	⊙	✓	⊙	✓	✓	⊙	⊙	⊙	⊙
Steranios 2% NG	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙
TD-100 Auto Processor	⊙	⊙	⊙	⊙	⊙	⊙	⊙	✓	✓	✓	✓	✓
Tristel	⊙	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	⊙	⊙
Trophon EPR, NanoNebulant	✓	⊙	⊙	✗	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙
Trophon EPR, Sonex-HL	✓	⊙	⊙	✗	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙
VIRKON Triple Salt	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✗	✓	✓
Wavicide-01	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙

Key

✓	Approved
✗	Not Approved
⊙	Not Tested

Important Notes

Not a Claim of Efficacy	These disinfectant compatibility tables only indicate that we have evaluated for compatibility with the materials in the manufacturing of Siemens Medical Solutions ultrasound transducers and not for their efficacy in bringing about the appropriate level of disinfection. The manufacturer of the solution should be consulted for information regarding the efficacy of each solution.
Use Only Approved Products	Siemens Medical Solutions can take no responsibility for damage caused by using non-approved cleaning products. Be certain you only use products approved and listed in this brochure.

Notes on Some Very Common Chemicals

Bleach	Bleach (Sodium Hypochlorite) must not be used to soak transducers even in diluted solutions. Soaking transducers in bleach will cause damage. Wiping probes with a cloth dampened with <10% bleach solution is an acceptable pre-cleaning process.
Isopropyl Alcohol	Transducers must not be soaked in Isopropyl Alcohol (IPA). This will cause damage to the transducer. Wiping transducers with a cloth dampened with Isopropyl Alcohol is an acceptable pre-cleaning process.

ACUSON SC2000 is a trademark of
Siemens Medical Solutions USA, Inc.

Siemens Healthcare Headquarters

Siemens Healthcare GmbH
Henkestraße 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens.com/healthcare

Legal Manufacturer

Siemens Medical Solutions USA, Inc.
Ultrasound
685 E. Middlefield Road
Mountain View, CA 94043
USA
Phone: 1-888-826-9702
www.siemens.com/ultrasound