

SIEMENS



Transducer Disinfectants

ACUSON S2000™ Ultrasound System

Approved Pre-Cleaners and High Level Disinfectants for Transducers

Answers for life.

Pre-Cleaners

	10V4	14L5	14L5 BV*	14L5 SP	18L6 HD	4C1	4P1	4V1	4V1c	6C1 HD	6C2	7CF1	7CF2	8V3	9EVF4	9L4	12L4	CW2	CW5	EC9-4	EV-8C4	MC9-4	V5Ms (~S/N 327XXXX)	V5Ms (S/N 328XXXX~)
Acrylan	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓
Alkazyne	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	✓	✓	✓	✓
Alkydiol Spray 40	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙	⊙
Ampholysine Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙
Asepti-Wipe	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
Asepti-zyme	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Asphene 381 Spray	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
Cavi wipes & Cavicide	✓	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
Cidalkan	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
CIDEZYME XTRA	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cleanisept Wipe	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cleanisept Wipe forte	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Empower	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Endozyme	✓	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
Engarde	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	⊙	⊙	⊙	⊙
Enzol	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Erinox	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	✓	⊙	✓	⊙	✓	✓
Esculase	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓
Exidine 4	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓
Firstsinald	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙	⊙
Hexanios G+R	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓
Incidine	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
Instru-Zyme	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓
Liv des special 45	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓
Manu-Klenz	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	✓	⊙	✓	⊙	⊙	⊙
Medallion	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	⊙	⊙	⊙	⊙	⊙
Medi-Prep	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	✓	⊙	✓	⊙	⊙	⊙
MetriZyme	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
M-Ytidesinfektion	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓
Neodiol +	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙	⊙
Orthozyme	✓	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
pH7Q	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓
PI-Spray	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	⊙	✓	✓
PI-Spray II	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Preven's D51	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙

Pre-Cleaners

	10V4	14L5	14L5 BV*	14L5 SP	18L6 HD	4C1	4P1	4V1	4V1c	6C1 HD	6C2	7CF1	7CF2	8V3	9EVF4	9L4	12L4	CW2	CW5	EC9-4	EV-8C4	MC9-4	V5Ms (~S/N 327XXXX)	V5Ms (S/N 328XXXX~)
Protex Spray/Wipes	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Protex Ultra wipes	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Quat-Stat	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	⊙	⊙	⊙	⊙
RBS-50	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	✓	⊙	✓	⊙	⊙	⊙
Rivascop	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	⊙	⊙	✓	⊙	✓	⊙	⊙
Sani-Cloth AF3	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙
Sani-Cloth HB	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	⊙	✓	✓	✓
Sani-Cloth Plus	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	⊙	✓	✓	✓
SaniZide	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
Savogerm	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
Sekulyse	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓
Sekupoudre	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	✓	✓
Sekusept	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	⊙	⊙	⊙	⊙
SONO ultrasound wipes	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙
Super Sani-Cloth	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	✗	✓	✓	✓	✓	✓	✓	✗	✓	✓	✓	⊙	✓	✓	✓
TheraCide	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	⊙	⊙
Transeptic	✓	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	✗	✓	✓	✓	✓	✓	✓	✗	✓	✓	✓	✓	✓	✓	✓
Trionic D wipe	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓
Virex II 256	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	✓	✓
WEX-CIDE	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	✓	⊙	✓	⊙	✓	✓

Key

✓	Approved
✗	Not Approved
⊙	Not Tested

* Transducer for the ACUSON S2000 ABVS

High-Level Disinfectants

	10V4	14L5	14L5 BV*	14L5 SP	18L6 HD	4C1	4P1	4V1	4V1c	6C1 HD	6C2	7CF1	7CF2	8V3	9EVF4	9L4	12L4	CW2	CW5	EC9-4	EV-8C4	MC9-4	V5Ms (~S/N 327XXXX)	V5Ms (S/N 328XXXX-)
Aidal Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Alkacide	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	✓	✓	✓
Asepti-steryl 14	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Asepti-steryl 28	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
BM-28 Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cetylcode-G	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
CIDEX	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
CIDEX 7	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
CIDEX OPA	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
CIDEX PA	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	✓	✓	✓
CIDEX PLUS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Clorox Bleach germicidal cleaner	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓
Deson	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓
Dispatch	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓
Endosporine	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Erihyd Forte	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Formac 14	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	✓	⊙	✓	⊙	⊙	⊙
Formac 28	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙	⊙
Gigasept	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙	⊙
Gigasept FF	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓
Glutaraldehyde NS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Glutarcode	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Glutaster Basica	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
Instrunet Esporicida	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Korsolex	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	⊙	⊙	⊙
Maxicide NS 14 Day	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MedSci	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
MetriCide	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MetriCide OPA Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MetriCide 28	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓
Microcide PA	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙	⊙
mikrozyd PAA Wipes	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	✓	✓	✓
MILTON	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	⊙	✓	✓	✓	✓	✓	✓	✓
Nova Plus NS	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	✓	✓	✓
Nu-Cidex	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	✓	✓	⊙	✓	⊙	✓	✓	✓

High-Level Disinfectants

	10V4	14L5	14L5 BV*	14L5 SP	18L6 HD	4C1	4P1	4V1	4V1c	6C1 HD	6C2	7CF1	7CF2	8V3	9EVF4	9L4	12L4	CW2	CW5	EC9-4	EV-8C4	MC9-4	V5Ms (-S/N 327XXXX)	V5Ms (S/N 328XXXX-)
Omnicide 14NS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Omnicide 28	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	⊙	✓	✓
Opal	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Oxivir-TB	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙
Pennysaver +	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Procide 14 N.S.	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Procide 28	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	⊙	⊙	⊙
Procide Plus	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓
Rapicide	✓	✓	⊙	✓	⊙	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Resert XL HLD	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	✓	⊙	✓	⊙	⊙	✗	✓	✓	✗	✗
SBS Glutar 2%	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Sekusept Easy	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	⊙	✓	⊙	⊙	⊙	⊙	⊙	✓	✓
Sekusept Extra	⊙	⊙	✓	⊙	✓	✓	⊙	✓	✓	✓	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	✓	✓
Sporadyne	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
Sporicidin	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	⊙	✓	⊙	⊙	⊙
Steranios 2% NG	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
TD-100 Auto Processor	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	⊙	✓	✓	
Tristel Sporicidal Wipes	⊙	⊙	⊙	⊙	⊙	✓	⊙	✓	✓	⊙	✓	⊙	⊙	✓	⊙	⊙	⊙	✓	✓	✓	✓	⊙	✓	✓
Trophon EPR, NanoNebulant	✓	✓	⊙	✓	✗	⊙	✓	⊙	⊙	✗	⊙	✓	✓	⊙	✓	✓	⊙	⊙	⊙	✓	✓	✓	⊙	⊙
Trophon EPR, Sonex-HL	✓	✓	⊙	✓	✗	⊙	✓	⊙	⊙	✗	⊙	✓	✓	⊙	✓	✓	⊙	⊙	⊙	✓	✓	✓	⊙	⊙
VIRKON Triple Salt	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	⊙	✓	✓	✓	✓	✓	✓	✓	✗
Wavicide-01	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

Key

✓	Approved
✗	Not Approved
⊙	Not Tested

* Transducer for the ACUSON S2000 ABVS

Important Notes

Not a claim of efficacy	These disinfectant compatibility tables only indicate that we have evaluated for compatibility with the materials used in the manufacturing of Siemens medical ultrasound transducers and not for their efficacy in bringing about the appropriate level of disinfection. The manufacturer of the solution should be consulted for information regarding the efficacy of each solution.
Use Only Approved Products	Siemens Medical Solutions can take no responsibility for damage caused by using non-approved cleaning products. Be certain you only use products approved in the attached matrix.

Notes on some very common chemicals

Bleach	Bleach (Sodium Hypochlorite) must not be used to soak transducers even in dilute solutions. Soaking transducers in bleach will cause damage. Wiping probes with a cloth dampened with < 10 % bleach solution is an acceptable pre-cleaning process.
Isopropyl Alcohol (IPA)	Transducer must not be soaked in Isopropyl Alcohol. This will cause damage to the transducer. Wiping transducers with a cloth dampened with Isopropyl Alcohol is an acceptable pre-cleaning process.
V5M Care and Handling	To extend the useful life of the transducer, the TEE transducer should be withdrawn from the stomach acid environment when not actively acquiring a transgastric view during the operation procedure. The TEE transducer should be returned to the neutral position and moved out of the stomach and "parked" in the esophagus until such time as additional transgastric views are required.

ACUSON S2000 is a trademark of
Siemens Medical Solutions USA, Inc.

Siemens Healthcare Headquarters

Siemens Healthcare GmbH
Henkestraße 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens.com/healthcare

Legal Manufacturer

Siemens Medical Solutions USA, Inc.
Ultrasound
685 E. Middlefield Road
Mountain View, CA 94043
USA
Phone: 1-888-826-9702
www.siemens.com/ultrasound