

# siemens.com/healthcare

# ADVIA Centaur Enhanced Liver Fibrosis (ELF) Test

**Specification Sheet** 







The ADVIA Centaur® systems' Enhanced Liver Fibrosis (ELF™) test is an in vitro diagnostic multivariate index assay intended to provide a single ELF score by combining in an algorithm the quantitative measurements of hyaluronic acid (HA), amino-terminal propeptide of type III procollagen (PIIINP), and tissue inhibitor of metalloproteinase 1 (TIMP-1) in human serum using the ADVIA Centaur XP, ADVIA Centaur XPT, and ADVIA Centaur CP systems.

The ELF test is indicated, in conjunction with other laboratory findings and clinical assessments, as an aid in the diagnosis and assessment of the severity of liver fibrosis in patients with signs and symptoms of chronic liver disease.

- First direct marker panel of liver fibrosis
- Standardized component assays (HA, PIIINP, TIMP1)
- Uses a routine blood serum sample
- Clinically validated for liver fibrosis assessment in mixed, HCV, and NAFLD patient groups

The ELF test was clinically validated on the Immuno 1 system in an international multicentre study with a mix of patient groups and was found to be accurate to differentiate mild, moderate and severe fibrosis<sup>1</sup>, and subsequently has been shown to be at least as good as biopsy at predicting liver disease-related outcomes<sup>2-3</sup>.

#### Calculating the ELF Score

To calculate the ELF score for the ADVIA Centaur systems, first obtain results for the ADVIA Centaur HA, PIIINP, and TIMP-1 assays and then use the following equation/algorithm to calculate the ELF score:

## ADVIA Centaur XP/XPT:

ELF score =  $2.278 + 0.851 \ln(C_{HA}) + 0.751 \ln(C_{P3NP}) + 0.394 \ln(C_{TIMP1})$ 

# **ADVIA Centaur CP:**

ELF score =  $2.494 + 0.846 \ln(C_{HA}) + 0.735 \ln(C_{P3NP}) + 0.391 \ln(C_{TIMP1})$ 

Concentrations (C) of each assay are in ng/mL

### Interpretation of Results

Interpretation of the ELF score is as follows:

< 7.7	None to mild
≥ 7.7 to < 9.8	Moderate
≥ 9.8	Severe

# **Siemens ELF Test**


Assay Summary					
Test	Sample Type	Sample Volume	Assay Range	Onboard Stability	Calibration Interval
TIMP-1	Serum	25 μL	3.5 - 1,300 ng/mL	60 days	28 days
PIIINP	Serum	20 μL	0.5 - 150 ng/mL	60 days	28 days
НА	Serum	20 μL	1.6 - 1,000 ng/mL	60 days	14 days

Ordering Information				
Catalog No.	Contents	No. of Tests		
10492065	TIMP-1 ReadyPack®	50		
10493157	HA ReadyPack	50		
10492440	PIIINP ReadyPack	50		
10492344	ADVIA Centaur ELF Calibrator	$2 \times 2.0$ mL low calibrator $2 \times 2.0$ mL high calibrator		
10492342	ADVIA Centaur ELF QC	3 x 2.0 mL Control 1 3 x 2.0 mL Control 2 3 x 2.0 mL Control 3		
10492364	ADVIA Centaur Multi-Diluent 13	2 x 5.0 mL ReadyPack ancillary reagent packs		

The ELF tests are CE- marked. Not available for sale in the U.S.



- 1. Rosenberg et al. "Serum Markers Detect the Presence of Liver Fibrosis: A Cohort Study" Gastroenterology 2004;127:1704-1703.
- 2. Parkes et al. "Enhanced Liver Fibrosis Test Can Predict Clinical Outcome in Patients with Chronic Liver Disease" <u>Gut</u> 2010;59:1245-1251
- 3. Pinzani. "The ELF panel: a new crystal ball in hepatology?" Gut 2010;59:1165-1167

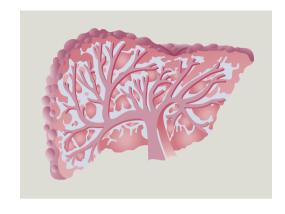

The ADVIA Centaur Immunoassay Systems from Siemens Healthcare offers easy consolidation and automation of routine, specialty, and STAT testing in any laboratory. Make your lab even more efficient by consolidating your liver disease testing onto the ADVIA Centaur system.

For additional information on the Siemens ELF test, go to siemens.com/elf.

Centaur, ELF, ReadyPack, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

#### **Local Contact Information**

#### **Siemens Healthcare**

Laboratory Diagnostics 511 Benedict Avenue Tarrytown, NY 10591-5005 USA

Phone: +1 914-631-8000 siemens.com/healthcare

#### Siemens Healthcare Headquarters

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen Germany Phone: +49 9131 84-0 siemens.com/healthcare

Order No. A91DX-CAI-160434-XC1-4A00 | 06-2016 | © Siemens Healthcare Diagnostics Inc., 2016