



The BUG Hospital Tübingen is one of the major trauma centers in Southwestern Germany. In 2012, the hospital was looking for a new high-end CT to replace the older system.

Metal artifact reduction

with Dual Energy and SOMATOM Definition Edge

Abstract

Dual Energy has been one of the most important innovations in Computed Tomography in recent years and has made its path into clinical routine. With previous CT generations, it was difficult to evaluate tissue and bone structures close to metal implants such as nails, screws, and plates. With SOMATOM Definition Edge and its Dual Energy functionality, this has changed.

The BGU Hospital in Tübingen, Germany was one of the first customers to install SOMATOM Definition Edge in combination with Dual Energy in order to reduce metal artifacts. This combination makes possible better evaluation of clinical cases and an earlier removal of the patients' metal implants.



In mid-2012, the hospital decided to go for SOMATOM Definition Edge. The Siemens high-end single source CT is equipped with the renowned Stellar detector technology and several more dose reduction features.

Introduction

Closely located to the University Hospital Tübingen, the BGU Hospital is one of the largest trauma hospitals in Southwestern Germany. Its portfolio mainly includes healthcare service for all kind of accidents and stretches from fractures to spine surgery and polytrauma. The radiology is being run by Prof. Claussen (Head of Radiology Department, University Hospital Tübingen and Radiology BGU Hospital) and his team. The chief radiologist of the radiology department at the BGU hospital, Dr. Luz, is an experienced user of SOMATOM Definition Edge and its Dual Energy application monoenergetic. In the beginning of 2012, the hospital was looking for a high-end CT system to replace their outdated previous system which had served several years.

The main criterion was that it should, on the one hand, include the latest technology in terms of dose reduction and fast iterative reconstruction. On the other hand, it should offer additional value for their specialist portfolio as a regional trauma center. Being a trauma center means that a large group of patients have to undergo CT scans more than once – for example, in follow-up scans after surgery – and that many patients receive metal implants for some weeks in order to stabilize difficult fractures or degenerated bone tissue.



Challenge

“With the technology of our previous CT system, CT staff was pessimistic most of the time,” says Dr. Luz. “The metal implants caused artifacts that did not allow an exact evaluation of the surrounding tissue. This means that in order to stabilize a fracture, we generally left the implants in the body too long instead of removing them too early.” Every radiologist faces this issue. Metal implants such as screws, nails, or plates cause streak artifacts that derive from multiple mechanisms. One of the major effects is beam hardening. The “broad polychromatic spectrum of the X-radiation is attenuated differently, depending on the energy of the radiation, the object type, and projection direction, and consequently causes variable increases in the mean energy of the spectrum,”¹ especially in thick structures such as bones and metal.

The result are images where you see dark zones, surrounded by zones of white and even fine streaks disturbing the diagnostic image quality far away from the origin metal object. It is hard to differentiate the tissue around the metal object which stabilizes a critical, complex fracture or fixes a weakness in the spine. This is why radiologists tend to leave the metal in the body longer rather than removing it too early. The BGU hospital was therefore looking for a solution to get better image quality without putting the patients at additional risk with dose.



“SOMATOM Definition Edge allows us to scan patients at an extremely low dose with excellent image quality,” says Dr. Oliver Luz (Chief radiologist BGU Hospital Tübingen).


Solution

In mid-2012, the BGU Hospital Tübingen decided to go for SOMATOM Definition Edge. This is Siemens’ high-end single source CT system, which was introduced to the market at RSNA 2011 and was available for delivery by the middle of 2012. Besides the fast pitch of 1.7 that makes possible up to 230mm/s* acquisition in trauma scanning, the system comes with the newest detector technology. The Stellar detector is the first fully integrated detector and thus results in a higher signal-to-noise ratio. The combination of Edge technology and 0.5 mm slices offers excellent image quality at reduced dose. Add SAFIRE – and the dose can be reduced by up to 60%.^{*} This is also the experience of the BGU Hospital.

“SOMATOM Definition Edge allows us to scan at an extremely low dose with excellent image quality,” says Dr. Luz. The challenge of metal artifacts is solved with the Dual Energy functionality of the system. As easy as a spiral scan, the system performs a dose optimized Dual Energy scan generating images of different keV levels. Here in Dual Energy scan mode, all dose features that apply to the system in standard scan modes come into effect. This means that Adaptive Dose Shield^[2,3], CARE Dose4DTM, and SAFIRE^[4] assist to apply only the right dose to patients without putting them at additional risk. The result of the Dual Energy scan is better image quality with higher diagnostic outcome.

*In clinical practice, the use of SAFIRE may reduce CT patient dose depending on the clinical task, patient size, anatomical location, and clinical practice. A consultation with a radiologist and a physicist should be made to determine the appropriate dose to obtain diagnostic image quality for the particular clinical task. The following test method was used to determine a 54 to 60% dose reduction when using the SAFIRE reconstruction software. Noise, CT numbers, homogeneity, low-contrast resolution and high contrast resolution were assessed in a Gammex 438 phantom. Low dose data reconstructed with SAFIRE showed the same image quality compared to full dose data based on this test. Data on file.

Without Dual Energy, the images showed metal artifacts like in the 70 keV image (A). Now it is possible to generate images at different keV levels with reduced metal artifacts (B). This offers better image quality for diagnostic evaluation.

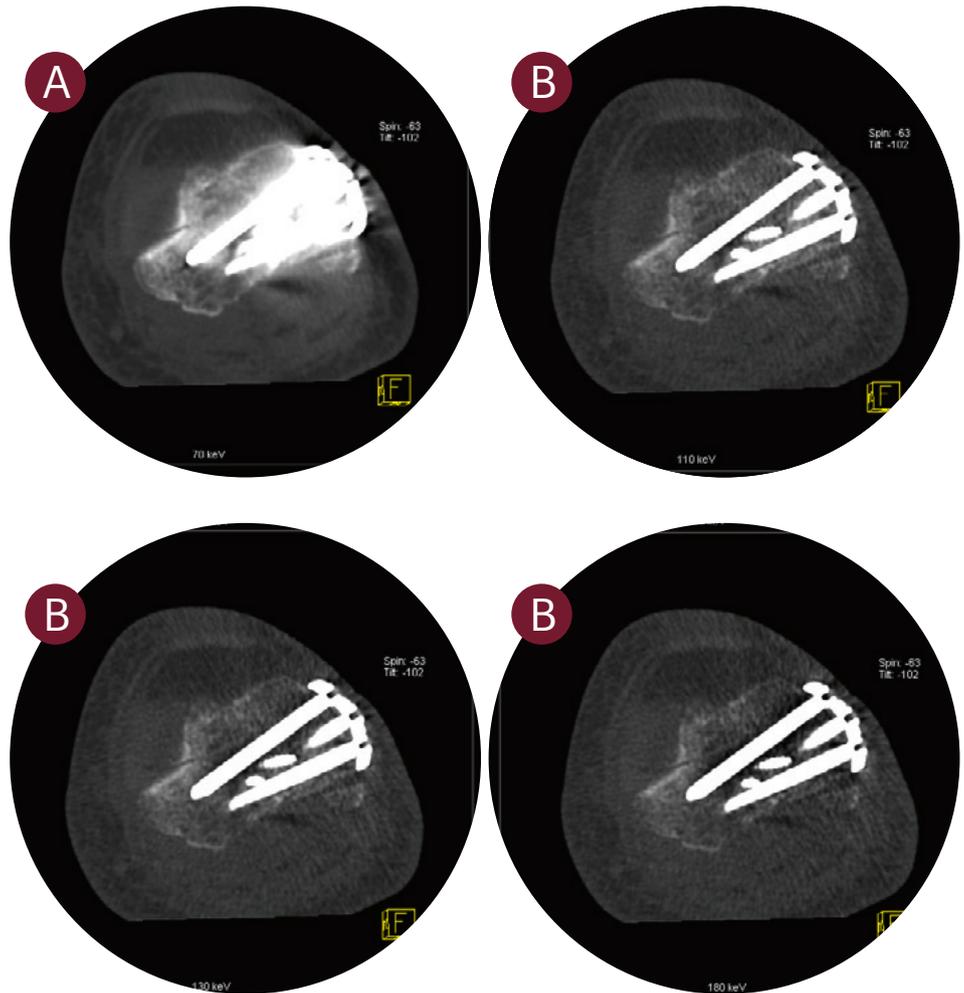

Outcomes

SOMATOM Definition Edge has fulfilled what the BGU hospital expected from a new high-end imaging system. With the new Stellar detector technology, the dose in standard procedures has been lowered, and with the fast iterative reconstruction (up to 20 frames/s) the system is faster than before. For their special needs to reduce metal artifacts, Dual Energy monoenergetic imaging now enables radiologists to view the images at different keV levels and thus with higher image quality. Bone tissue around nails, screws, and plates can be better evaluated for further treatment decisions and an earlier removal of the stabilizing implant. "With SOMATOM Definition Edge, CT staff has now become optimistic. On average, we see that we suggest to our

surgeons to remove the metal implants up to three to four weeks earlier than before with our previous system," concludes Dr. Luz. Besides the fact that it is more convenient for the patient to get the implant removed earlier, there is another advantage that comes with this. The patient can start rehab earlier, has faster recovery from the injury, and can therefore get back to work earlier. Not only the patients but also the hospital benefits from SOMATOM Definition Edge: the new technology attracts additional patients and helps to prevent unnecessary operation procedures which might have been undertaken with the previous CT technology when metal artifacts might have caused misleading interpretation of diagnostic images.

Conclusion

The BGU Hospital Tübingen was looking for a new high-end CT system and found their wishes fulfilled in SOMATOM Definition Edge. The new Stellar detector technology and many of the dose reduction features have reduced radiation exposure of patients. Nevertheless, image quality has improved. Additionally, the system adds value to the clinical portfolio with its Dual Energy functionality.

As easy as a spiral scan, the system generates images at different keV levels that now can show surrounding tissue even with large metal implants such as huge nails, screws, or plates. Diagnostic evaluation of images has changed from previously pessimistic evaluation to an optimistic earlier removal of metal implants, sending patients to rehab earlier than before.

References

- [1] Kalender, W. (2011): Computed Tomography, P. 131, 3rd. edition
- [2] Deak PD, Langner O, Lell M, Kalender WA. Effects of adaptive section collimation on patient radiation dose in multisection spiral CT. Radiology. 2009 Jul; 252(1):140-7.
- [3] Christner JA, Zavaletta VA, Eusemann CD, Walz-Flannigan AI, McCollough CH. Dose reduction in helical CT: dynamically adjustable z-axis X-ray beam collimation. AJR Am J Roentgenol. 2010 Jan; 194(1):W49-55.
- [4] Baumueller S, Winklehner A, Karlo C, Goetti R, Flohr T, Russi EW, Frauenfelder T, Alkadhi H. Low-dose CT of the lung: potential value of iterative reconstructions. Eur Radiol. 2012 Dec;22(12):2597-606.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and are subject to change without prior notice. Some of the features and products described herein may not be available in the United States. The information in this document contains general technical descriptions of specifications and options as well as standard and optional features that do not always have to be present in individual cases. Siemens reserves the right to modify the design, packaging, specifications and options described herein without prior notice. Please contact your local Siemens sales representative for the most current information.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced. The statements contained herein are based on the actual experience of data on file to support these claims. However, these statements do not suggest or constitute a warranty that all product experiences will yield similar results. Results may vary based on the particular circumstances of individual sites and users.

International version.
Not for distribution or use in the U.S.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84 0
siemens.com/healthineers