

Evaluation of absorbed dose using sequential quantitative SPECT/CT following ^{177}Lu DOTATATE metastatic neuroendocrine tumor therapy

.....

Data courtesy of Dale L. Bailey PhD, Kathy P. Willows PhD, and Paul J. Roach MB BS FRACP
Department of Nuclear Medicine
Royal North Shore Hospital & University of Sydney
Sydney, Australia


History

A 49-year-old male presented with extensive liver metastases. ^{68}Ga -DOTATATE PET/CT imaging was used to confirm that the mass was the result of a neuroendocrine tumor (NET). The patient was subsequently referred for radionuclide therapy with ^{177}Lu DOTATATE. High tracer uptake within the liver metastases demonstrated on the PET/CT suggested a high concentration of somatostatin receptors. As such, the patient underwent 2 cycles of radionuclide therapy, each with 7.4 GBq of ^{177}Lu DOTATATE. A follow-up ^{68}Ga -DOTATATE PET/CT showed a significant amount of central necrosis within the large liver lesions and a slight reduction in size in the smaller metastatic lesions. In view of the partial response to therapy, along with substantial volume of residual functioning tracer avid tumor in the liver and absence of bone or nodal or soft tissue metastases, the patient was treated with a third therapeutic dose of 7.4 GBq ^{177}Lu DOTATATE.

The therapy dose was delivered as an intravenous infusion along with the standard regime of amino

acid administration prior to and during therapy. Thirty minutes after commencing administration of the first therapeutic dose of ^{177}Lu DOTATATE, the patient underwent a SPECT/CT study, which was followed by subsequent SPECT/CT studies performed at 4-, 24-, and 120-hours post injection in order to assess tracer concentration in tumor and critical organs, such as the kidneys, across 4 time points. The exams were performed on a Symbia Intevo™ scanner using xSPECT Quant™, which involved using a NIST-traceable ^{75}Se point source of known activity for calibration in order to accurately quantify the studies.

xSPECT Quant acquisition was performed using 60 stops with 20 seconds per stop. The study was reconstructed using OSCGM reconstruction incorporating attenuation and scatter correction. The volume of interest (VOI) generated using fused SPECT/CT images over the tumor and critical organs, such as the kidneys, were evaluated to assess volumes and absolute tracer concentration within the VOIs.

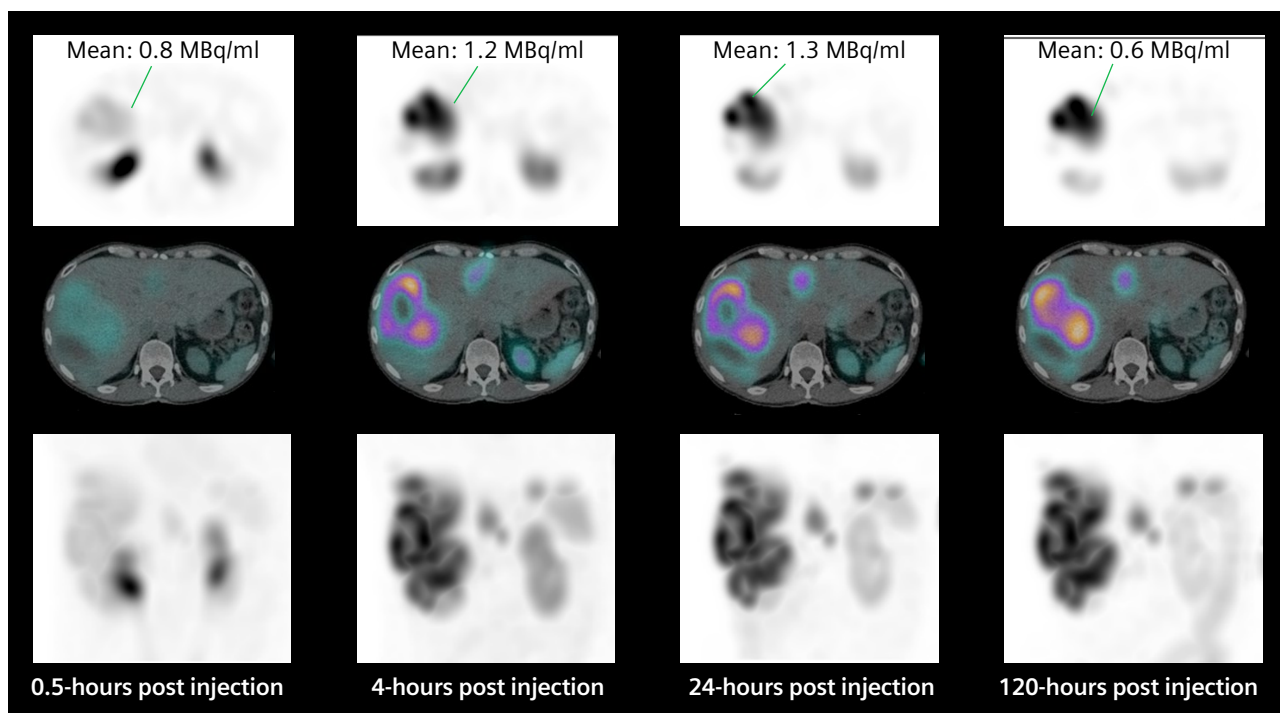

1 Sequential 4-time point SPECT/CT following 7.4 GBq ^{177}Lu DOTATATE administration in a patient with multiple liver metastases resulting from a NET. Transverse multiplanar reconstruction (MPR), fused, and sequential maximum intensity projection (MIP) show progressive increase in uptake within the liver lesions with maximum uptake on the 24-hour study and very slow washout. The 120-hour study shows substantial retention of tracer within large metastatic lesions in the right lobe of the liver as well as within the smaller metastatic lesions in the left lobe, reflecting the slow washout. Fused axial SPECT/CT images show a large highly tracer-avid metastatic lesion with central necrosis in the right hepatic lobe. Multiple similar lesions are seen on coronal SPECT images. Tracer concentration measured with xSPECT Quant within a segment of functioning tumor shows gradual increase in tracer concentration to a maximum at 24 hours with subsequent slow washout with almost half of peak concentration remaining after 120 hours.

Diagnosis

Visual inspection of sequential 4-time point SPECT/CT (Figures 1 and 2) demonstrated high tracer uptake in multiple liver metastases with progressive increase in uptake until 24 hours and very slow clearance as evident in substantial remaining tracer uptake by 120 hours. This suggests high initial uptake and slow washout in the tumor, which indicates a possibility of high tumor dose delivery due to increased tumor tracer retention time. However, the kidneys showed normal cortical uptake. Fast clearance suggests lower renal tracer retention time with the possibility of low renal dose.

xSPECT Quant enabled the evaluation of changes in tracer concentration in the kidneys and tumor. xSPECT Quant and CT data fed into research dosimetry

software enable automated generation of VOIs, for the tumor as well as critical organs, such as the kidneys, liver, and spleen, in order to estimate absolute tracer concentration in each VOI in kBq/mL. Tumor, kidney, and liver volumes can be derived from VOI generated from fused SPECT/CT images with high accuracy with corresponding tracer concentrations attained from xSPECT Quant leading to the estimation of absolute tracer concentration in kBq within the VOI. Such estimations from SPECT/CT data across 4 time points may be used to determine absorbed dose to the tumor and critical organs like the kidneys, liver, and spleen.


Furthermore, multiple time point images subsequently converted to percentage-of-administered radioactivity concentration enable time activity curves to be generated,

mono or bi-exponential curve fitting to be performed, as well as subsequent Medical Internal Radiation Dose (MIRD) or voxel-based dosimetry.

The multi-time point xSPECT Quant data was processed by research dosimetry software to yield dose estimates. The 4 time point studies were accurately registered to each other and VOIs generated and aligned across multiple time points to accurately assess quantitative changes in absolute tracer concentration across time and yield time activity curves.

Comments

Most well-differentiated NETs express somatostatin receptors, in particular SST2 receptors, which results in high uptake of radiolabeled somatostatin analogs, such as ^{177}Lu DOTATATE, that

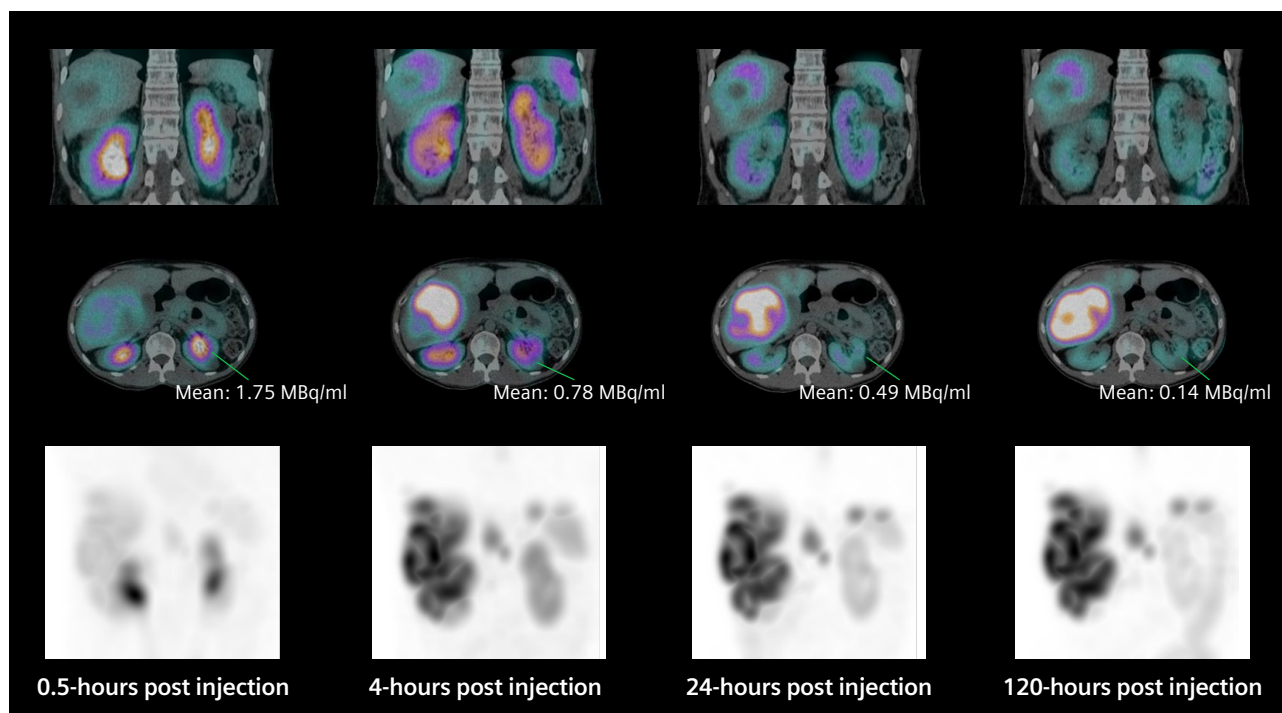


2 Coronal and transverse SPECT/CT fused images at the plane of both kidneys show normal cortical uptake with significant pelvicalyceal stasis at 0.5 hours post-therapy administration; 4-, 24-, and 120-hour studies show fast clearance of tracer from bilateral renal cortex as well as fast washout from both renal pelvises. Tracer concentration measurement from one region of left renal cortex shows highest cortical tracer concentration at 0.5 hours post injection with fast clearance. Cortical tracer concentration decreased by 75% within 24 hours post injection and less than 10% of initial cortical tracer concentration was retained by 120 hours. The clearance pattern from both kidneys suggested normal initial uptake and fast washout with minimal tracer retention, which reflects good renal function and potentially low renal dose from therapy administration.

are used for treatment of patients with advanced metastatic NETs with peptide receptor radionuclide therapy (PRRT). Apart from β emission, ^{177}Lu DOTATATE is also a gamma emitter, which enables planar and SPECT imaging to be performed following therapeutic dose. This allows for the demonstration of tumor uptake and estimation of tumor and critical organ dose along with a prediction of therapeutic efficacy, in addition to the need and feasibility of multiple therapies. The standard therapy approach with ^{177}Lu DOTATATE is for 4 cycles to be administered at 8–12 week intervals with each therapy dose being 7–7.4 GBq. The absorbed dose to the tumor is restricted by the maximum acceptable absorbed dose to the kidneys and bone marrow, hence limiting the total number of administered therapy cycles.

A reliable dosimetry method is mandatory in order to ensure optimal administration of therapy dose without exceeding toxicity limits. Therapeutic dosimetry might help better predict the maximal absorbed dose to the tumors while minimizing radiation burden to the critical organs. In order to achieve a reliable dosimetry estimate, accurate and precise activity quantification is essential.

Although the majority of the experience of dosimetry for ^{177}Lu -DOTATATE therapy is based on sequential whole-body planar scintigraphic acquisition, the overlap between liver and renal activity on planar images may lead to errors in defining renal and liver volumes leading to inaccurate quantification of tracer concentration for dosimetry.

Often a SPECT or SPECT/CT acquisition for 3D delineation and segmentation of liver and kidneys is performed as part of the dosimetry acquisition in order to accurately define the relative activity of the liver and kidney volumes, thereby improving the accuracy of the time activity curves. Sequential SPECT/CT acquisitions instead of planar along with the use of 3D-voxel-based dosimetry has been shown to improve the dosimetric accuracy and in some cases have shown significant difference in tumor and critical organ dose between 2D and 3D dosimetry. A study comparing planar and SPECT/CT based 3D dosimetry for ^{111}In -, ^{90}Y -, and ^{177}Lu -labelled somatostatin analogs in metastatic NET showed consistently higher dose with 3D dosimetry compared to planar 2D dosimetry.¹ The mean dose

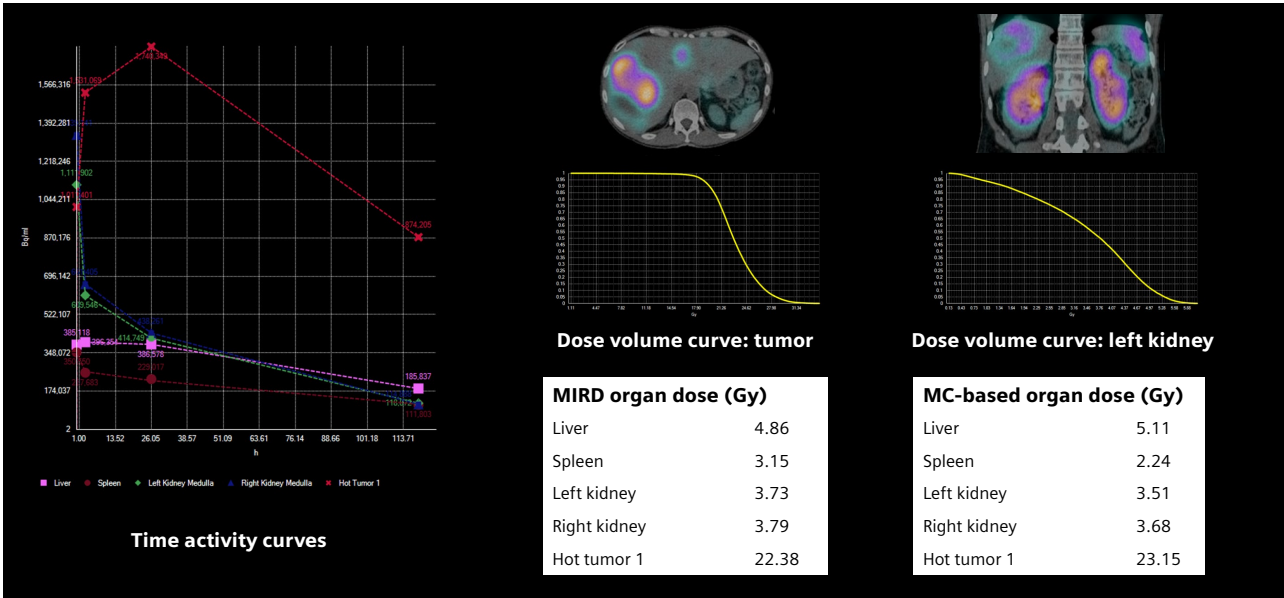

3 Time activity curves generated from VOIs drawn over the left and right kidney, whole liver, spleen, and 1 large Liver metastases are aligned across multiple time points. The time activity curves yield retention times, which reflect an estimate of the number of disintegrations occurring within the VOI throughout the entire course of the tracer transit through the organ or tumor considering both the physical and biological half-life. Combining the time activity integrals with dose conversion factors yields the effective dose in Gy as shown in the dosimetry analysis tables. Dose volume curves for the tumor and left kidney are also depicted as representative curves demonstrating the overall high and uniform dose to the majority of the tumor volume. A bi-exponential curve fit method was used. MIRD organ dose tables show an overall dose of 3.73 Gy to the left kidney and 3.79 Gy to the right kidney, which may be considered low for such therapy administrations. One large tumor represented by the curve shows dose of 22.38 Gy, which represents a high tumor dose reflective of the potential for lasting suppression of tumor growth. A comparison of MIRD-based and voxel-based (Monte Carlo) dosimetry shows comparable values. However, dose volume histograms generated by voxel-based dosimetry gives a better understanding of the distribution of absorbed dose within a tumor and kidney as shown in the figure.

From the dosimetry tables, it is clear that the left and right renal dose (3.73 Gy and 3.79 Gy) are within expected levels considering the initial therapeutic dose administered. The tumor dose in 1 large liver lesion is calculated to be 22.38 Gy, which suggests the possibility of significant cumulative dose to the tumor and favorable response. In view of the low renal dose from this therapy calculated by dosimetry and the overall low cumulative renal dose, the possibility of further therapies for this patient exists. The decision for submitting the patient to further therapies would depend on the tumor response based on subsequent functional imaging results, the patient's symptoms, and serum tumor markers. High tumor dose calculated by dosimetry suggests the possibility of lasting suppression of tumor activity as well as possible shrinkage of small lesions and an increase in necrosis levels of large lesions.

was 0.45 mGy/MBq with 3D dosimetry compared to 0.31 mGy/MBq for planar.

Conclusion

The present study demonstrates the value of 3D-voxel-based dosimetry derived from Monte Carlo calculations using sequential SPECT/CT studies. It also uses standard MIRD dosimetry with automated organ and tumor segmentations obtained from the SPECT/CT studies. Organ segmentations for liver, spleen, and kidneys were based on CT data, while tumor segmentations were based on SPECT data with VOI generated using

a threshold of 50% of maximum uptake. Time activity curves generated from automatically segmented organs or tumor were fitted using bi-exponential curve fit, from which total tracer residence time was generated from the area under the curve for dosimetric evaluation.

xSPECT Quant enables accurate estimation of tracer concentration of ¹⁷⁷Lu within a voxel or VOI secondary to system calibration with a NIST-traceable Selenium 75-point source of known concentration along with CT attenuation and scatter correction. Tracer concentration within an individual voxel or VOI defined from

sequential SPECT/CT studies is used to generate accurate time activity curves for dosimetry calculation.

In the present study, the renal dose (left kidney: 3.73 Gy, right kidney: 3.79 Gy by MIRD) is comparable between MIRD and voxel-based dosimetric methods. The tumor dose measured in the largest liver metastases (22.4 Gy MIRD) is significantly higher than renal dose, as is expected based on the tracer uptake and clearance differences between the tumor and renal cortex.

An accurate estimation of renal dose is key in the determination of the maximum allowable number of ¹⁷⁷Lu

DOTATATE therapy cycles based on the accepted cumulative renal dose threshold of 23 Gy or a higher threshold of 40 Gy in patients with good renal function and without any risk factors like hypertension or diabetes. In a recent study, hybrid dosimetry (3 planar with single SPECT/CT) was performed on 51 patients with metastatic NET undergoing multiple therapies with ^{177}Lu DOTATATE (7.4 GBq per therapy cycle).² Most patients were treated based on a renal dose of 23 Gy with a median number of therapy cycles being 5 (range 3–7). However, 5 patients with good renal function without any other risk factors were treated up to a threshold of 40 Gy for cumulative renal dose and the median number of therapies in this group was 7 (range 5–8). Significant renal toxicity with glomerular filtration rate (GFR) < 30 ml/min/1.73 m² was not seen in any patient. The absence of significant renal toxicity in patients treated with multiple therapies irrespective of their initial renal functional status is encouraging for offering these patients multiple therapies. However, the decrease in GFR with progressive number of therapies was higher in the group with lower initial GFR levels. Thus, repeated monitoring of

renal status and accurate renal dosimetry following every therapy cycle is key to ensuring absence of significant long term renal impairment in these patients, especially those with associated hypertension or diabetes.

Since the patient had undergone 2 previous therapies with similar levels of renal doses and the current therapy cycle delivers a total renal absorbed dose of more than 3.5 Gy, planning of additional therapies needs to be based on correct assessment of total renal dose threshold for this patient. In view of normal renal function, normal renal cortical thickness on CT and low initial uptake and fast clearance of tracer from the renal cortex, a higher threshold of 40 Gy for cumulative absorbed renal dose for this patient is justified. Additional therapies with dosimetry for each therapy cycle can therefore be advocated, which may further improve tumor control. In view of the large tumor burden and high tracer avidity of the tumor, the amount of circulating tracer available for the kidney will be limited thereby reducing the risk of renal toxicity (tumor sink effect). However, large

tumor burden is an adverse prognostic indicator irrespective of therapy type and somatostatin receptor avidity of the tumor and the objective of repeated radionuclide therapy would be primarily to decrease hormone release related symptoms, tumor mass effects, and improve quality of life. ●

xSPECT Quant is not commercially available in some countries. Due to regulatory reasons, its future availability cannot be guaranteed. Please contact your local Siemens organization for further details.

The research dosimetry software is a work in progress software and not available for clinical use. Its future availability cannot be guaranteed. Please contact your local Siemens organization for further details.

References

- ¹ Baechler S, Hobbs R, Boubaker A, et al. Three-dimensional radiobiological dosimetry of kidneys for treatment planning in peptide receptor radionuclide therapy. *Med Phys*. 2012;39(10):6118-6128.
- ² Sundlov A, Sjogreen-Gleisner K, Svensson J, et al. Individualised ^{177}Lu -DOTATATE treatment of neuroendocrine tumours based on kidney dosimetry. *Eur J Nucl Med Mol Imaging*. 2017;44:1480-1489.

Examination protocol

Scanner: Symbia Intevo

SPECT		CT	
Injected dose	7.4 GBq ^{177}Lu DOTATATE	Tube voltage	130 kV
Acquisition	0.5 hours, 4 hours, 24 hours, and 120 hours post injection 60 stops per detector (120 stops total) 20 sec/stop OSCGM reconstruction	Tube current	27 eff mAs
		Slice thickness	3 mm

Legal information: On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and is subject to change without prior notice. Some/all of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features, which do not always have to be present in individual cases.

Siemens reserves the right to modify the design, packaging, specifications, and options described herein without prior notice.

Please contact your local Siemens sales representative for the most current information.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

“Siemens Healthineers” is considered a brand name. Its use is not intended to represent the legal entity to which this product is registered. Please contact your local Siemens organization for further details.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Published by

Siemens Medical Solutions USA, Inc.
40 Liberty Boulevard
Malvern, PA 19355
USA
Phone: +1-888-826-9702
siemens-healthineers.com

Global Business Line

Siemens Medical Solutions USA, Inc.
Molecular Imaging
2501 North Barrington Road
Hoffman Estates, IL 60192
USA
siemens.com/mi