

PET/CT detection of temporal arteritis


By Yu Iwabuchi, MD Data courtesy of Keio University Hospital, Tokyo, Japan

History

A 63-year-old man with a history of coronary artery disease and atherosclerotic narrowing of carotid vessels presented with occasional chest pain along with a fever of long duration. In view of the possibility of diffuse cardiovascular disease, the patient underwent PET/CT performed on a Biograph™ mCT. Following a low-dose non-contrast CT, a whole-body PET scan was performed from the top of head to the upper thigh and PET data was reconstructed using ultraHD•PET (time-of-flight [ToF] and point spread function [PSF] reconstruction combined) at a 200 x 200 matrix.

Diagnosis

The PET/CT images showed a clear uptake of the tracer on the artery such as the aorta, carotid artery, and subclavian artery, suggesting vasculitis. These findings were difficult to point out in a contrast-enhanced CT performed on the same day (Figure 1).

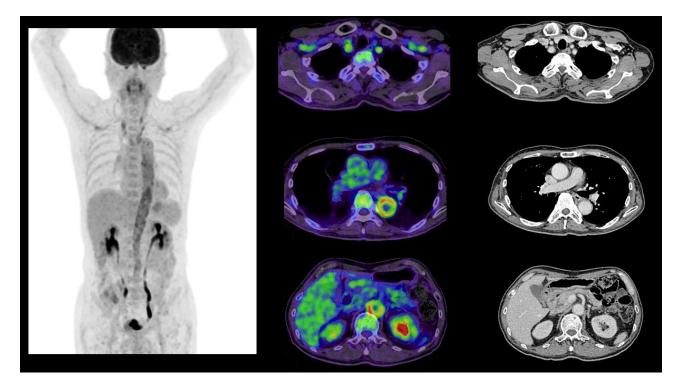

1 The PET/CT shows uptake of tracer in the walls of the descending aorta until the aortic bifurcation as well as the aortic arch. Bilateral carotid and subclavian arteries also show uptake. The pattern of uptake is suggestive of extensive vasculitis. The contrast CT performed as part of the PET/CT study does not show any abnormality including thickening or irregularity of the aortic or carotid or subclavian arterial wall.

Data courtesy of Keio University Hospital, Tokyo, Japan.

Interestingly, the PET/CT study also demonstrated a characteristic accumulation of the tracer on the bilateral temporal arteries, which implies temporal arteritis (Figure 2). The diagnosis was subsequently confirmed by the result of the biopsy of a temporal artery.

Comments

Cardiovascular diseases are often presented with symptoms related to heart as well as vascular manifestations. Vasculitis is often present together with coronary artery disease. It is often difficult to diagnose vasculitis at an early stage by using the conventional morphological imaging such as CT. Abnormal vascular hypermetabolism seen in PET/CT is often reflective of vasculitis at an early stage before irreversible morphological change of the artery. Vascular uptake in PET/CT provides the physician an overview of the inflammation within the entire cardiovascular system, especially arteries, which is key to systemic evaluation of cardiovascular disease. Although uptake in large arteries are well delineated on PET/CT, it is often difficult to visualize low level of uptake as is often seen in inflammation of smaller vessels such as temporal arteries.¹ However, with the development of PET reconstructions in recent years such as ToF and PSF, detection capability of small lesions has increased. In the present case, an incidental finding of a small lesion of the temporal artery detected due to the improved spatial resolution and lesion contrast obtained because of ToF and PSF helped identify peripheral cardiovascular disease (temporal arteritis). Improved lesion contrast achieved by ToF and PSF along with overall high count rate capability of the Biograph mCT enable high resolution PET imaging with higher matrix reconstructions which result in smaller voxels for improved definition of low intensity focal uptake within arterial walls reflecting arteritis.


Conclusion

In a patient with diffuse cardiovascular disease, PET/CT demonstrated an incidental finding of focal hypermetabolism in bilateral temporal arteries which suggested presence of temporal arteritis.

References

¹ M. Brodmann, R. W. Lipp, A. Passath, G. Seinost et al. The role of 2-18F-fluoro-2-deoxy-D-glucose positron emission tomography in the diagnosis of giant cell arteritis of the temporal arteries. Rheumatology 2004;43:241–242.

The statements by Siemens Healthineers' customers described herein are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption) there can be no guarantee that other customers will achieve the same results.

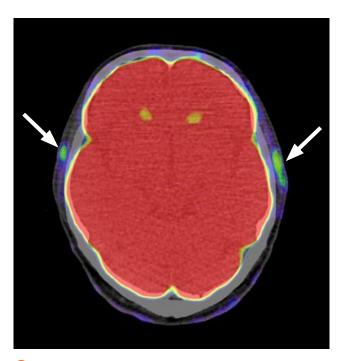

2 Axial section through the brain of the PET/CT study with adjustment of PET window settings in order to demonstrate uptake of tracer in bilateral temporal arteries (see arrows) suggestive of temporal arteritis.

Data courtesy of Keio University Hospital, Tokyo, Japan.

Examination protocol

Scanner: Biograph mCT

PET/CT	
Injected dose	6.1 mCi
Scan delay	90 min
Acquisition	1.2 mm/sec whole-body ultraHD·PET reconstruction
СТ	
Tube voltage	100 kV
Tube voltage Tube current	100 kV 68 eff mAs
5	
Tube current	68 eff mAs

Legal information: On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and is subject to change without prior notice. Some/all of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features, which do not always have to be present in individual cases.

Siemens reserves the right to modify the design, packaging, specifications, and options described herein without prior notice.

Please contact your local Siemens sales representative for the most current information.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

"Siemens Healthineers" is considered a brand name. Its use is not intended to represent the legal entity to which this product is registered. Please contact your local Siemens organization for further details.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen Germany Phone: +49 913184-0 siemens-healthineers.com

Published by

Siemens Medical Solutions USA, Inc. Molecular Imaging 2501 North Barrington Road Hoffman Estates, IL 60192 USA siemens.com/mi

MI-3714 | PDF only | © Siemens Healthcare GmbH, 06.2018