


Detection of bladder invasion and pelvic lymph node metastases in a patient with primary prostate cancer

By Mario Jreige, MD; Marie Nicod Lalonde, MD; Niklaus Schaefer, MD; and John O. Prior, MD, PhD Data courtesy of Centre Hospitalier Universitaire Vaudois, Lausanne, Switzerland


History

A 62-year-old man with primary prostate cancer diagnosed on clinical examination and transrectal ultrasound underwent ⁶⁸Ga PSMA PET/CT for initial staging. The PET/CT study was performed on Biograph Vision™. ⁶⁸Ga PSMA (2.8 mCi, 105 MBg) was injected intravenously and a PET/CT

study was performed after 1 hour and 5 minutes. An initial whole-body CT (100 kV, 104 eff mAs, 64 x 0.6 mm collimation) was performed followed by continuous bed motion (FlowMotion™ technology) PET acquisition with a constant table speed of 1.0 mm/sec. The PET study was reconstructed at 440 x 440 matrix using time-of-flight (ToF) and point-spread function (PSF).

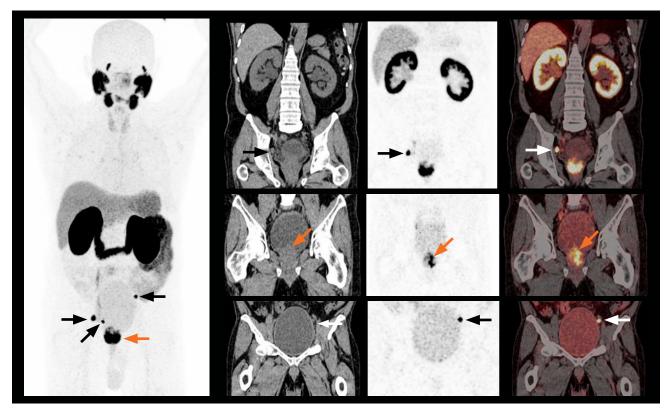


1 Maximum intensity projection (MIP) and thin MIP images of 68 Ga PSMA PET show high uptake in the prostatic mass with irregular extension into the bladder (orange arrows) reflecting bladder neck infiltration of large primary prostate cancer. Three tracer-avid pelvic lymph node metastases (black/white arrows) are also well visualized. Right internal iliac (top row) and left common iliac (bottom row) lymph node metastases (black/white arrows) are sharply delineated on the coronal images. The bladder is enlarged secondary to infiltration from prostate cancer (middle row) causing outflow obstruction along with mild dilatation of right ureter. Salivary glands, liver, kidneys, and intestines show physiological uptake of tracer as expected. No skeletal lesion is visualized.

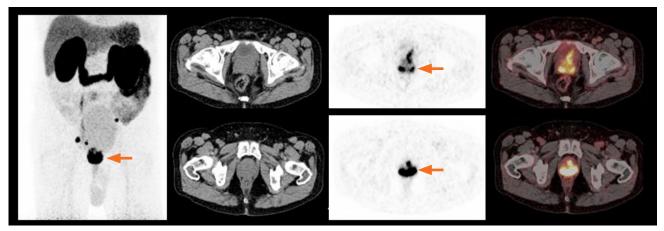

2 Axial slices through PET, CT, and fused images at the level of the prostate show high uptake of ⁶⁸Ga PSMA in the primary prostate tumor (arrow) along with extension of the tracer-avid tumor into the adjacent bladder wall. The primary prostate tumor measures 2.67 cm x 3.57 cm x 3.01 cm in its maximum antero-posterior, lateral, and supero-inferior dimensions with a SUV_{max} of 37.3 and a SUV_{beak} of 22.3.

Data courtesy of Centre Hospitalier Universitaire Vaudois, Lausanne, Switzerland.

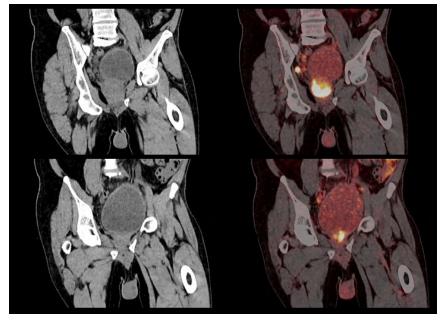

3 Oblique coronal slices of CT and fused PET/CT images through the prostate and bladder highlight the intensely tracer-avid, primary tumor (SUV_{max} 37.3, SUV_{peak} 22.9) involving the whole of the enlarged prostate with infiltration into the adjacent bladder wall with irregular margins. The enlarged (1.36 cm) right external iliac lymph node metastases is also well visualized with a high contrast to background with a SUV_{max} of 41.7.

Data courtesy of Centre Hospitalier Universitaire Vaudois, Lausanne, Switzerland.

Findings

68Ga PSMA PET/CT (Figures 1 to 6) shows primary prostate cancer involving the majority of the enlarged prostate gland measuring nearly 3 cm in all dimensions with invasion into the bladder neck and external urinary sphincter (T4) with resulting bladder outflow obstruction. The PET/CT also revealed 3 highly tracer-avid pelvic lymph node metastases in the right meso-rectal, right external iliac and left common iliac nodes. No distant metastases were visualized. Bladder invasion by primary tumor and the


presence of pelvic nodal metastases puts the patient in stage IV. The primary prostatic tumor showed high intensity of uptake of 68Ga PSMA with a SUV_{max} of 37.3. The pelvic nodal metastases were also highly traceravid with the SUV_{max} ranging from 36 to 56. The meso-rectal and left common iliac nodes measured 8-9 mm in diameter but were intensely tracer-avid with high lesion contrast, with the highest SUV_{max} being in the left common iliac node. Physiological uptake of 68Ga PSMA in the salivary glands, liver, kidney, and intestines were within normal limits.

The enlarged bladder shows low level of tracer uptake with mild dilatation of left ureter reflecting bladder outflow obstruction due to tumor invasion of the bladder neck.

68 Ga PET/CT was instrumental in visualization of bladder neck invasion of tumor and pelvic nodal metastases as well as ruling out PSMA-avid skeletal or other distant metastases.

Comments

The presence of lymph node metastases in primary prostate cancer patients is associated with adverse prognosis. The

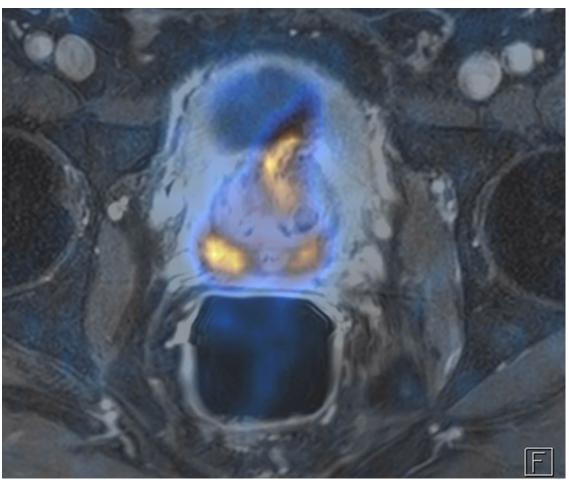


4 The axial section at the level of a primary prostate tumor of a fused 68Ga PSMA PET and pelvic MRI data shows 68Ga PSMA-avid primary tumor in the peripheral zone bilaterally as well as in the transitional zone with extension into the adjacent bladder wall. Data courtesy of Centre Hospitalier Universitaire Vaudois, Lausanne, Switzerland.

presence of bladder neck invasion from primary prostate tumor along with multiple pelvic lymph node metastases in this patient put him in stage IV, which requires systemic therapy. Therapy options in such patients include radical prostatectomy with pelvic lymphadenectomy or radiotherapy along with neoadjuvant or concurrent anti androgen therapy (ADT). Detection of lymph node and distant metastases is the most critical factor in accurate staging of primary prostate cancer, which influences further therapy choices. The detection rate of pelvic lymph nodal metastases and distant metastases at initial diagnosis is around 20% and 10% respectively and is impacted by serum prostate-specific antigen (PSA) levels.1 The effectiveness of definitive radiation therapy depends on accurate definition of extent of direct extraprostatic extension

and detection of pelvic lymph node metastases in order to ensure adequate dose delivery to target volumes. ⁶⁸Ga PSMA PET/CT has been shown to improve the success rate of radiation therapy in primary prostate cancer by identifying patients with occult metastases and thereby modifying target volume delineations and dose plans in order to adequately cover local disease.²


Radiation target volumes that were created using only CT were compared to ⁶⁸Ga PSMA positive lesions in a study involving 73 patients with high risk primary prostate cancer (PSA>10 ng/ml) to determine whether PSMA-positive lesions were localized inside or outside of irradiated volumes.² Twenty-five out of 73 patients (34%) showed ⁶⁸Ga PSMA-positive lymph node metastases, while only 7 of 73 (9.5%) showed enlarged

nodes suspicious for metastases on CT. 16.5% of patients had at least one PSMApositive lesion not covered by target volume based on CT. 5.5% of patients had at least one centimeter of primary prostate tumor not covered by CT-based radiation target volume. In these patients, PET/CT had a major impact on radiation therapy planning leading to a change in the target volume. The majority of PSMApositive lymph nodes not covered by CT based plan were in perirectal or common iliac nodal groups. The median size of PSMA-positive lymph nodes missed on the CT was 6 mm. In all patients with nodal lesions or periprostatic extensions identified by PSMA PET/CT, radiation plans were modified with dose escalation to PSMA-avid lesions. Similar impact on therapy plan is expected in this patient to manage the infiltration of bladder neck by primary tumor as well as lymph nodal

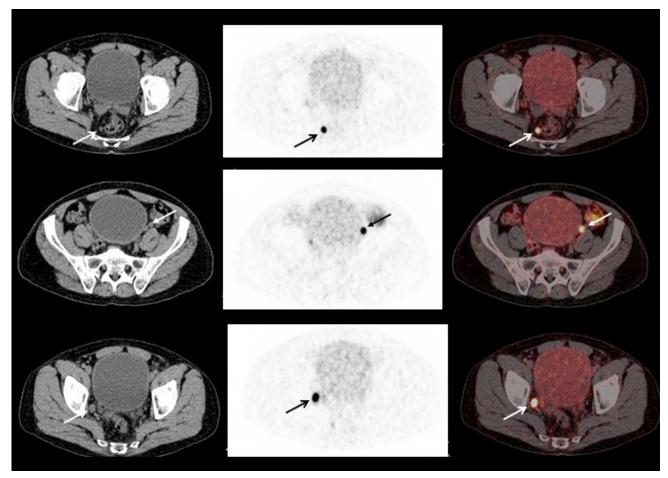

Axial CT, PET, and fused slices show 68 Ga PSMA-avid right mesorectal (top row); left common iliac (middle row) and right external iliac (bottom row) lymph node metastases (arrows). The mesorectal node measures 9 mm in diameter with a SUV_{max} of 36.1 and the common iliac node measures 8 mm with a SUV_{max} of 56.03. The right external iliac node is the largest, measuring 1.36 cm in its maximum diameter and has a SUV_{max} of 41.7.

 ${\tt Data\ courtesy\ of\ Centre\ Hospitalier\ Universitaire\ Vaudois, Lausanne,\ Switzerland.}$

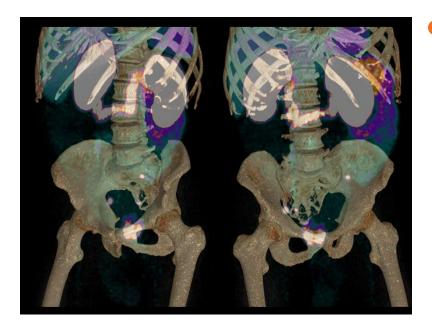

Volume rendering of fused PET/CT images show the relationship of the lymph node metastases with the pelvis.

Data courtesy of Centre Hospitalier Universitaire Vaudois, Lausanne, Switzerland.

metastases. The meso-rectal and left common iliac nodes were 8 mm in diameter but highly PSMA-avid and the radiation fields need to include these nodes along with the right external iliac nodal metastases with dose escalation to PSMA positive lesions.

Although the avidity of prostatic malignant cells for PSMA is the key driver for ⁶⁸Ga PSMA PET/CT due to its ability to accurately define a primary tumor and peri-prostatic extensions and detect small lymph node metastases, PET/CT image quality, resolution, and lesion contrast is of utmost importance in order to ensure the highest sensitivity of the study. Technological improvements in PET/CT such as incorporation of SIPM technology for improved ToF performance for

improved lesion detectability may potentially further improve sensitivity of PET/CT for detection of nodal and distant metastases with direct impact in staging and management. In this context, Biograph Vision with a 214 picosecond ToF timing resolution and 3.2 mm crystal size is able to provide very high lesion contrast and high resolution imaging that sharply defines primary prostate tumor and the bladder neck invasion as well as tracer avid lymph node metastases, the smallest of which in this case was 8 mm in diameter. The reconstruction matrix for the whole-body PET study was 440 x 440, which reflects the high resolution of Biograph Vision enabled by the small 3.2 x 3.2 mm crystal size. Smaller crystal size enables higher resolution acquisition, which reduces

partial volume effects that often impact the definition of small lesions and improves quantitative accuracy. The high SUV_{max} of the small nodal metastases as small as 8 mm as in the left common iliac node is testament to the high lesion contrast generated by 214 picosecond ToF timing resolution as well as reduced partial volume effects enabled by high resolution imaging on Biograph Vision.

Conclusion

High lesion contrast and resolution enabled by Biograph Vision was instrumental in sharp definition of invasion of bladder neck by aggressive primary prostate cancer along with multiple, small pelvic lymph node metastases by 68Ga PSMA PET/CT.

References

- ¹ Schmidt-Hegemann et al. Detection level and pattern of positive lesions using PSMA PET/CT for staging prior to radiation therapy. *Radiation Oncology* (2017) 12:176.
- ² Calais et al. Potential impact of ⁶⁸Ga-PSMA-11 PET/CT on prostate cancer definitive radiation therapy planning, *J Nucl Med.* 2018 Apr 13.

The tracer used in this case is a research radiopharmaceutical. It is neither recognized to be safe and effective by the FDA nor commercially available in the United States or in all countries worldwide. Siemens does not make any claims regarding its use.

Biograph Vision is not commercially available in all countries. Due to regulatory reasons, its future availability cannot be guaranteed. Please contact your local Siemens organization for further details.

The outcomes achieved by the Siemens customers described herein were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g. hospital size, case mix, level of IT adoption) there can be no guarantee that others will achieve the same results.

Examination protocol

Scanner: Biograph Vision

P	ET		СТ	
	njected	2.8 mCi	Tube voltage	100 kV
d	ose	(105 MBq)	Tube current	104 eff mAs
Α	Acquisition FlowMotion 1.0 mm/sec	Slice collimation	64 x 0.6 mm	
			Slice thickness	3 mm

Legal information: On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and is subject to change without prior notice. Some/all of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features, which do not always have to be present in individual cases.

Siemens reserves the right to modify the design, packaging, specifications, and options described herein without prior notice.

Please contact your local Siemens sales representative for the most current information.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

"Siemens Healthineers" is considered a brand name. Its use is not intended to represent the legal entity to which this product is registered. Please contact your local Siemens organization for further details.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen Germany +49 9131 84-0 siemens-healthineers.com

Published by

Siemens Medical Solutions USA, Inc. Molecular Imaging 2501 North Barrington Road Hoffman Estates, IL 60192 +1 847-304-7700 siemens.com/mi

MI-4164.DD.CP | PDF only | © Siemens Healthcare GmbH, 11.2018