**Clinical Results Clinical Results** 

#### Case 2

# **Coronary CTA with Reduced Contrast** and Radiation Dose of 70 kV

By Yining Wang, MD, Jian Cao, MD,

Department of Radiology, Peking Union Medical College, Beijing, P.R. China

### History

A 61-year-old female patient was referred to the hospital complaining of chest pain and shortness of breath. A coronary computed tomography CT Angiography (cCTA) was requested to rule out coronary artery disease.

### Diagnosis

The CT images demonstrated a mild stenosis, from soft plaque, in the proximal left anterior descending artery (LAD), and a myocardial bridge in the middle LAD with no evidence of stenosis. The circumflex (Cx) was small in caliber but showed no evidence of stenosis. The right coronary artery (RCA) appeared normal.


#### Comments

cCTA is a valuable non-invasive imaging examination with high diagnostic accuracy. Technological advances allow not only dose reduction but also improvement in the image acquisition. The SOMATOM Definition Flash scanner holds several technical advantages, including the latest Stellar Detector and sinogram-affirmed iterative reconstruction (SAFIRE), the first model-based and raw data-based iterative reconstruction application. Both of them make it possible to use lower tube voltage in cCTA examinations with excellent image quality.

The use of a lower tube voltage (70 kV) scanning protocol leads to a significant increase of mean attenuation and mean contrast enhancement of the coronary arteries as well as to significantly higher image noise. The contrast enhancement urges us to minimize the amount of contrast media and the image noise can be solved perfectly with the application of SAFIRE technique. Dual Source CT Flash mode with a very high pitch spiral scanning, can not only shorten the acquisition time, but also reduce both the radiation exposure and the necessary amount of contrast media (in this case, 0.39 s, 0.19 mSv and 45 mL).

## **Examination Protocol**

| Scanner        | SOMATOM Definition Flash |                          |                                |
|----------------|--------------------------|--------------------------|--------------------------------|
| Scan area      | Heart                    | Slice collimation        | 128 × 0.6 mm                   |
| Scan length    | 115 mm                   | Slice width              | 0.75 mm                        |
| Scan direction | Cranio-caudal            | Temporal resolution      | 75 ms                          |
| Scan time      | 0.39 s                   | Reconstruction increment | 0.5 mm                         |
| Tube voltage   | 70 kV                    | Reconstruction kernel    | 126f                           |
| Tube current   | 270 eff.mAs              | Patient heart rate       | 57-69 bpm                      |
| CTDIvol        | 0.78 mGy                 | Contrast                 |                                |
| DLP            | 13.7 mGy cm              | Volume                   | 45 mL                          |
| Effective dose | 0.19 mSv                 | Flow rate                | 3.5 mL/s                       |
| Rotation time  | 0.28 s                   | Start delay              | Test Blous Peak Trigger + 21 s |
| Pitch          | 3.4                      |                          |                                |

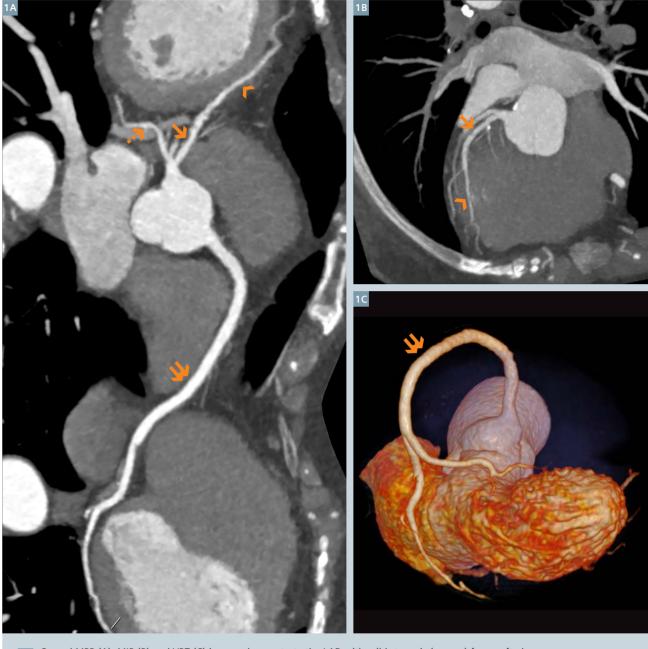

Curved MPR (A), MIP (B) and VRT (C) images demonstrate the LAD with mild stenosis (arrows) from soft plaque, and a myocardium bridge (arrowheads) in the middle LAD with no evidence of stenosis. The Cx (dashed arrow) and the RCA (double arrows) appear to ne normal, although the Cx is small in caliber.