
Artificial Intelligence for MRI

Heinrich von Busch, Ph.D.

Siemens Healthineers, Erlangen, Germany

The developments of the last few years in computational hardware as well as algorithms, in particular the advent of methods suitable for training complex neural networks (Deep Learning), have opened up new possibilities for machine learning [1] and automation in many applications. Magnetic Resonance imaging, postprocessing and interpretation, in fact radiology in general, are no exception. In the following, we will attempt to provide a brief overview of the current state of development.

In 2016, computer expert Geoffrey Hinton suggested that "we should stop training radiologists now" [2], as the discipline would soon be made obsolete by intelligent new algorithms that outperform human readers. His statement attracted considerable attention and gave rise to countless passionate debates in the community. From a 2019 perspective, however, the idea that radiology could become extinct anytime soon appears less than plausible for a number of reasons, among them the appreciation that many aspects of radiology go way beyond well-defined, "sandbox" problem-solving. Put simply, in the absence of "general artificial intelligence", which could indeed replace humans in any complex cognitive task (and is as yet not even visible on the horizon), there is no algorithm that could *replace* a radiologist. That said, artificial intelligence (AI) promises to *augment* workflows in radiology in many ways, by providing supportive tools particularly for highly standardized and repetitive tasks, starting from the identification and delineation of anatomical structures and organs and the corresponding extraction of quantitative parameters. The concept, in fact, is not entirely new. Tools

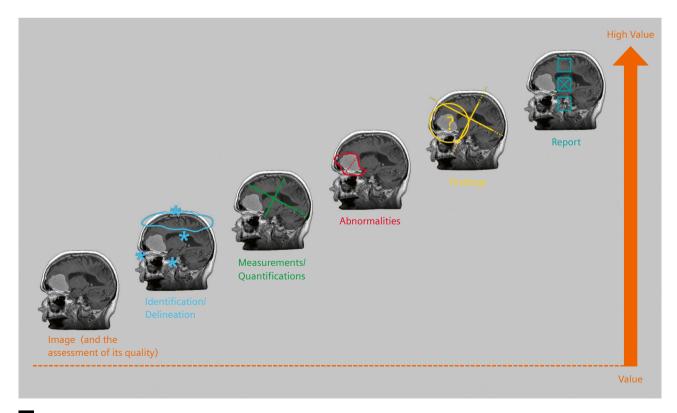

1 Steps in image analysis and interpretation

Product News

based on machine learning algorithms applied to medical images have been in wide clinical use for many years, as in MRI exam planning support provided by Siemens' Dot Engines. What is new is that the latest technical advances promise to improve the performance of such tools to a level that will extend their usefulness to many more use cases than could be covered previously.

Before addressing the state of the art in AI tools for radiology, let us review a case of machine problem-solving that exemplifies many of the characteristic traits that fuel the excitement about the latest level of AI – strategy games. Chess, in particular, has long been considered a touchstone of cognitive ability, so that the victory of IBM's Deep Blue over the reigning world chess champion Garri Kasparov in 1997 was seen as a milestone in machine intelligence. Fast forward to 2017 and the latest edition of Google DeepMind's software AlphaGo Zero [3] for the board game Go, and consider the leap forward. Not only does Go exhibit far more options per move than chess, but the inner workings of the machines involved could hardly be more different. Whereas Deep Blue relied on custom integrated circuits specifically designed for the ultrafast calculation of nothing but chess moves and was trained on a large library of matches played by human chess masters, AlphaGo Zero used general neural network hardware and learned Go starting from just the rules of the game by playing itself. With this approach, it took only a few days to surpass the level of the best human players.

While this was clearly an impressive achievement, it is important to place it in perspective in relation to "real-life" tasks and realize that games such as Chess and Go have a well-defined, limited set of input parameters, a clear figure of merit, and easily allow the generation of large volumes of data that an algorithm can be trained on. Image interpretation in radiology, by contrast, is characterized by very large sets of input parameters (such as high-resolution 3D image data), significant variability both in subjects and in the imaging process itself (particularly so in MRI), an often less-than-perfect reference standard, and, importantly, only limited access to training cases. Based on these observations, it is tempting to hypothesize that wellstandardized, high-volume image interpretation tasks, such as those performed in screening, lend themselves most readily to AI support. Indeed, automated tools aiding the reading of chest CT scans and X-ray mammograms are among the first high-profile AI products for radiology on the market.


Among the many use cases of AI related to MR imaging, which range from exam planning to image reconstruction, to image interpretation, and to clinical decision support, let us now focus specifically on image analysis and interpretation and how AI tools may help in the process at different levels (Fig. 1). Going from an image series as the input to the radiology report as the result involves a number of steps. The first step is the assessment of the quality of the image. Next, there are the localization and delineation of anatomical structures in the image. Such results may already be highly useful in their own right because they relieve clinical caregivers of tedious work, e.g., in automated scan planning, or organ contouring for radiotherapy. Then the measurement and quantification of the morphological structures that were identified may follow, as in organ volumetry, which in fact is often only feasible in routine reading if supported by automation. Yet another step may identify abnormalities in the image, followed by an assessment of their clinical significance. It is here that we cross into the territory of computer-aided detection (CADe) and computer-aided diagnosis (CADx). Then there is the step of compiling the findings and results for a clinical report. Typically, this is the highest level of abstraction that is achievable based on information from images alone; in many cases the report, and certainly the recommendations for clinical follow-up, will require additional input beyond imaging, e.g., from patient anamnesis and laboratory exams.

Let us turn now to how AI tools will become available to the radiologist. It is our conviction at Siemens Healthineers that the success of AI in clinical practice will hinge not only on the stand-alone performance and robustness of the algorithms, but in equal measure on their seamless integration into the radiology workflow. One example of our vision is the AI-Rad Companion¹ as our platform for inserting AI into routine, PACS-based reading. The first application on this platform will be a comprehensive package of visualizations, evaluations, and quantifications for thorax CT². Based on our teamplay infrastructure, results will be automatically generated as soon as the exam is complete, and added to the case as extra DICOM image series. In this way, they are readily accessible, along with the original images, in any viewer in which the reader may choose to open the case. In the chest CT example, the analyses cover the lung², the aorta², the heart², and the spine³, so as to address not only the organ or organs that may have been the original focus for prescribing the exam, but other anatomies that are visible on the image as well. Figure 2 shows examples of the output. For the lung, evaluations comprise the detection of suspicious lung nodules including measurements of their size, as well as the visualization and quantification of volumes of low X-ray attenuation within the lung, both for the lung as a whole and for each lung lobe separately.


³Work in progress: the application is currently under development and is not for sale in the U.S. and in other countries. Its future availability cannot be ensured.

¹The AI-Rad Companion Engine has 510(k) clearance.

²CT Chest, Heart, and Aorta are 510(k) pending.

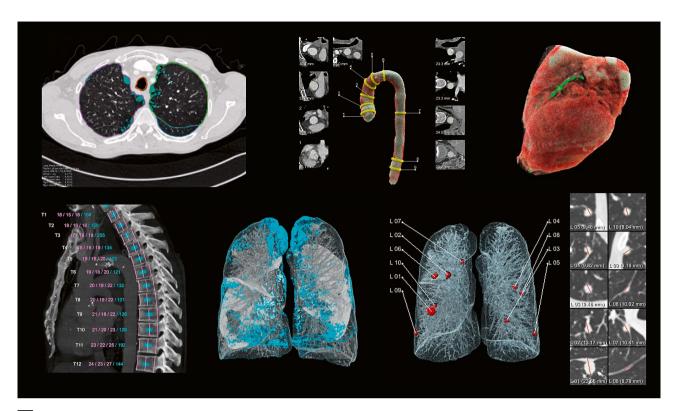

2 Chest CT² results from the Al-Rad Companion platform

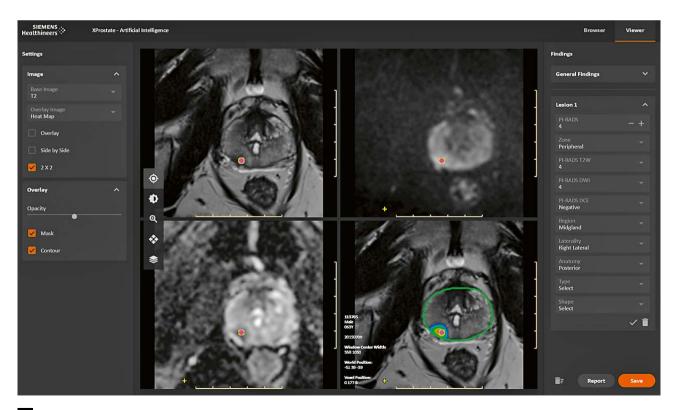

3 Prototype for AI evaluation of multiparametric prostate MRI³

For the heart, a visualization and quantification of the volume of the heart and in particular of coronary calcifications is prepared. For the aorta, a segmentation is performed and visualized, along with measurements of the diameters at different positions, and for the spine³, the vertebral bodies are automatically segmented, visualized, labeled, and measured, both in height and in bone density. All of these evaluations are designed to be robust to the input variability presented by different CT scanners and acquisition protocols, and to work with and without the use of contrast agents.

It is worth noting that chest CT imaging is just the first area for which we plan to bring to the market AI functionalities that are designed to save time and support consistent quality in the reading process. For a glimpse of our vision of the future, let us discuss another application, currently work in progress, that targets the AI analysis of multiparametric prostate MRI and is intended to cover all the analysis steps described above for image reading up to the prepopulation of a structured report. Again, the aim is to save time and support consistent quality. Figure 3 shows a screenshot of a minimal user interface geared specifically to prostate MRI reading. In this prototype, AI algorithms provide an automatic segmentation of the overall prostate and of the peripheral zone of the prostate, a map of lesions identified as suspicious, and a proposal for their PI-RADS classification, for the reader to check and edit if necessary before the results are compiled in a structured report.

There are plenty of further application scenarios involving AI beyond those presented here, such as in more efficient image reconstruction, in patient triage, and in clinical decision support. What they all have in common is a supporting role, often in well-defined, repetitive tasks. Even though they will be powerful instruments in some rather well-standardized contexts, it is to be expected that these

applications will become just a new set of tools in the hands of the radiologist, assisting but not replacing the human reader. In fact, their use and usefulness may become so ubiquitous within a few years that, just as with chess programs, we will no longer even call them "AI".

References

- 1 Russell SJ, Norvig P. Artificial Intelligence: A Modern Approach. 3rd ed. Upper Saddle River: Prentice Hall; 2010.
- 2 Hinton G. On Radiology. [video file]. 2016 Nov 24. Available from:
 - https://www.youtube.com/watch?v=2HMPRXstSvQ
- 3 Silver D, Schrittwieser J, Simonyan K et al. Mastering the game of Go without human knowledge. Nature [Internet]. 2017 Oct 19; 550: 354–359. Available from: https://www.nature.com/articles/nature24270



Contact

Heinrich von Busch, Ph.D. Product Owner Artificial Intelligence for Oncology Siemens Healthcare GmbH SHS DS DAT AI Hartmannstr. 17 91052 Erlangen, Germany Mobile: +49 162 2883167 heinrich.von_busch@siemens-healthineers.com

We are looking forward to hearing from you

Founded in 1993 the MAGNETOM Flash magazine has been around almost as long as MAGNETOM MRI scanners.

In times of online media and digitalization we would value your feedback to ensure that the magazine continues to fit your interests and professional needs.

Please take a moment to complete a brief reader survey at **www.siemens.com/magnetom-world**

