

	Wipe	Contains	Suitable for use?
	Sani-Cloth Chlor 1000 (orange)	Active ingredients: <i><1% Sodium hypochlorite <1% anionic surfactant</i>	YES
	Clinell Chlorox Wipes	Active ingredients: <i><1% Sodium Hypochlorite</i>	YES
	Sani-Cloth Detergent Multi surface (yellow)	No alcohol or disinfectant Active ingredient: <i>Dimethyl oxazolidine</i>	Shouldn't cause problems
	Sani-Cloth 70 (red)	Active ingredient: <i>70% Isopropyl alcohol</i>	Shouldn't cause problems
	Super Sani-Cloth Plus (red/pink)	Active ingredients: <i>Isopropyl alcohol Alkyl dimethyl ethyl benzyl ammonium</i>	DO NOT USE
	Clinell Universal Wipes (green)	Active ingredients: <i>Benzalkonium Chloride Didecyldimethyl ammonium chloride</i>	DO NOT USE

	<p>Clinell Detergent Wipes (yellow)</p>	<p>No alcohol or disinfectant</p>	<p>Active ingredients: <i>Caprylyl/Capryl Glucoside</i> <i>Lauryl Polyglucose Butoxydiglycol</i></p> <p>Shouldn't cause problems</p>
	<p>Clinell Sporcidal Wipes (red/orange)</p>	<p>Active ingredients:</p>	<p><i>Sodium Percarbonate</i> <i>Citric Acid</i></p> <p>Shouldn't cause problems</p>