

Cleaning and Disinfecting the Ultrasound System

v1 2020-03-27

Complete instructions, including warnings and cautions, for cleaning and disinfecting your ultrasound system are available in the Instructions for Use for your system.

- ACUSON Sequoia
- ACUSON Redwood
- ACUSON Juniper
- ACUSON P500
- ACUSON S Family
- ACUSON SC2000
- ACUSON NX2 and ACUSON NX2 Elite
- ACUSON NX3 and ACUSON NX3 Elite
- ACUSON Freestyle

Warnings

- To avoid electric shock and damage to the system, always power off and disconnect the equipment from the AC power source before cleaning and disinfecting.
- The use of any disinfectants other than those specified in the Instructions for Use may damage the ultrasound system and accessory surfaces and, as a result, may create electrical hazards for the patients and/or users.

Cautions

- Do not clean the system with chlorinated or aromatic solvents, acidic or basic solutions, isopropyl alcohol or strong cleaners such as ammoniated products, as these can damage the surface of the system. Use the recommended cleaning procedure.
Isopropyl alcohol exception: It is acceptable to use isopropyl alcohol only when recommended for cleaning the trackball assembly; refer to the Instructions for Use for your ultrasound system.
- Do not use spray cleaners on the ultrasound system as this may force cleaning fluid into the system and damage electronic components. It is also possible for the solvent fumes to build up and form flammable gases or damage internal components.
- Do not pour any fluid onto the ultrasound system surfaces, as fluid seepage into the electrical circuitry may cause excessive leakage current or system failure.
- To avoid the possibility of static shock and damage to the ultrasound system, avoid the use of aerosol spray cleaners on the monitor screens.
- Prior to cleaning or disinfecting the system, you must disconnect and remove transducers from the ultrasound system. Unintended contact with disinfectants approved only for use with the system can result in damage to the transducers.

Cleaning the System


To clean the ultrasound system, use a clean gauze pad or lint-free cloth, lightly moistened with a mild detergent, to wipe the surface of the ultrasound system to ensure areas are free of gel and any other visible residue.

- Gently wipe the glass surfaces, for example, the touch screen and monitor.
- Take particular care to clean the areas near the trackball, touchpad, soft keys, rotary knobs, and the slide controls.
- Ensure that fluids do not seep into any openings on the ultrasound system, for example, the seams on the control panel, the keyboard, or a DVD slot.

Disinfecting the System

To disinfect the ultrasound system, use an approved disinfectant wipe to disinfect the ultrasound system and accessories. Follow the disinfectant manufacturer's instructions.

See also: For a list of additional disinfectant wipes and detailed instructions for cleaning the trackball assembly, transducer holders, gel warmer, and transducer ports, refer to the Instructions for Use for your ultrasound system.

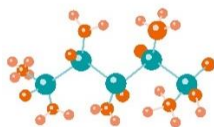

ACUSON Sequoia ultrasound system


On EPA List	Agent	System	Touch Display
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients		
	Cleanisept	X	X
	Cleanisept Wipes Forte	X	
	Clinell Universal Wipes	Interim	
Y	Clorox Healthcare Hydrogen Peroxide Wipe		
	IPA (less than 14%)		
	Lightly moistened cloth with a mild detergent		
	Mikrozid Sensitive Wipes	X	
	Protex Ultra		
	Rinse with water		
	Sani-Cloth		
	Sani-Cloth Active	X	
	Sani-Cloth AF	X	
Y	Sani-Cloth AF3	X	X
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X	X
	Sani-Cloth HB	X	X
	Sani-Cloth Plus	X	X
Y	Super Sani-Cloth	X	X
	Transeptic Spray (interim)	Interim	
	Tristel Trio Wipes WW DEMO ONLY		
	Tuffie5 Universal Sanitizing Wipe	X	

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound

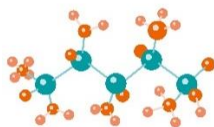

ACUSON Redwood ultrasound system


On EPA List	Agent	System	Touch Display
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients		
	Cleanisept	X	
	Cleanisept Wipes Forte	X	
	Clinell Universal Wipes	Interim	
Y	Clorox Healthcare Hydrogen Peroxide Wipe		
	IPA (less than 14%)		
	Lightly moistened cloth with a mild detergent		
	Mikrozid Sensitive Wipes	X	
	Protex Ultra		
	Rinse with water		
	Sani-Cloth		
	Sani-Cloth Active	X	
	Sani-Cloth AF		
Y	Sani-Cloth AF3	X	
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X	
	Sani-Cloth HB	X	
	Sani-Cloth Plus	X	
Y	Super Sani-Cloth	X	
	Transeptic Spray (interim)	Interim	
	Tristel Trio Wipes WW DEMO ONLY		
	Tuffie5 Universal Sanitizing Wipe	X	

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound

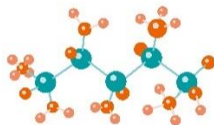

ACUSON Juniper ultrasound system


On EPA List	Agent	System	Monitor
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients		
	Cleanisept	X	
	Cleanisept Wipes Forte	X	
	Clinell Universal Wipes	Interim	
Y	Clorox Healthcare Hydrogen Peroxide Wipe		
	IPA (less than 14%)		X
	Lightly moistened cloth with a mild detergent		
	Mikrozid Sensitive Wipes		
	Protex Ultra		
	Rinse with water		
	Sani-Cloth		
	Sani-Cloth Active	X	
	Sani-Cloth AF	X	
Y	Sani-Cloth AF3	X	
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X	
	Sani-Cloth HB	X	
	Sani-Cloth Plus	X	
Y	Super Sani-Cloth	X	
	Transeptic Spray (interim)	Interim	
	Tristel Trio Wipes WW DEMO ONLY		
	Tuffie5 Universal Sanitizing Wipe		

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound

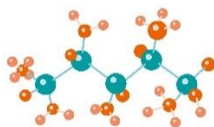

ACUSON P500 ultrasound system


On EPA List	Agent	System	Monitor
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients		
	Cleanisept		
	Cleanisept Wipes Forte	X	
	Clinell Universal Wipes	Interim	
Y	Clorox Healthcare Hydrogen Peroxide Wipe		
	IPA (less than 14%)		X
	Lightly moistened cloth with a mild detergent		
	Mikrozid Sensitive Wipes		
	Protex Ultra		
	Rinse with water		
	Sani-Cloth		
	Sani-Cloth Active		
	Sani-Cloth AF	X	
Y	Sani-Cloth AF3	X	
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X	
	Sani-Cloth HB	X	
	Sani-Cloth Plus	X	
Y	Super Sani-Cloth	X	
	Transeptic Spray (interim)	Interim	
	Tristel Trio Wipes WW DEMO ONLY		
	Tuffie5 Universal Sanitizing Wipe		

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-healthineers.com/ultrasound

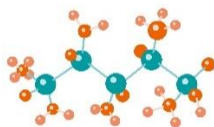

ACUSON Freestyle ultrasound system


On EPA List	Agent	System	Monitor	Deluxe Roll Stand
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients			
	Cleanisept			
	Cleanisept Wipes Forte	X		
	Clinell Universal Wipes	Interim		
Y	Clorox Healthcare Hydrogen Peroxide Wipe	X	X	X
	IPA (less than 14%)			
	Lightly moistened cloth with a mild detergent			
	Mikrozid Sensitive Wipes			
	Protex Ultra	X	X	
	Rinse with water			
	Sani-Cloth	X		
	Sani-Cloth Active			
	Sani-Cloth AF			
Y	Sani-Cloth AF3	X	X	X
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X		X
	Sani-Cloth HB	X		
	Sani-Cloth Plus	X		
Y	Super Sani-Cloth	X		X
	Transeptic Spray (interim)	X		
	Tristel Trio Wipes WW DEMO ONLY			
	Tuffie5 Universal Sanitizing Wipe			

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-healthineers.com/ultrasound

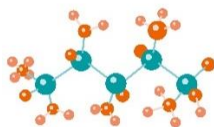

ACUSON S Family ultrasound systems


On EPA List	Agent	System	Monitor
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients	X	
	Cleanisept		
	Cleanisept Wipes Forte	X	
	Clinell Universal Wipes	Interim	
Y	Clorox Healthcare Hydrogen Peroxide Wipe		
	IPA (less than 14%)		
	Lightly moistened cloth with a mild detergent		X
	Mikrozid Sensitive Wipes		
	Protex Ultra		
	Rinse with water		
	Sani-Cloth	X	
	Sani-Cloth Active		
	Sani-Cloth AF		
Y	Sani-Cloth AF3		
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X	
	Sani-Cloth HB	X	
	Sani-Cloth Plus	X	
Y	Super Sani-Cloth	X	
	Transeptic Spray (interim)	Interim	
	Tristel Trio Wipes WW DEMO ONLY		
	Tuffie5 Universal Sanitizing Wipe		

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound

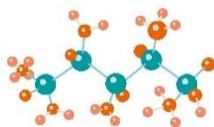

ACUSON SC2000 ultrasound system


On EPA List	Agent	System
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients	X
	Cleanisept	
	Cleanisept Wipes Forte	X
	Clinell Universal Wipes	Interim
Y	Clorox Healthcare Hydrogen Peroxide Wipe	
	IPA (less than 14%)	
	Lightly moistened cloth with a mild detergent	
	Mikrozyd Sensitive Wipes	
	Protex Ultra	
	Rinse with water	
	Sani-Cloth	X
	Sani-Cloth Active	
	Sani-Cloth AF	
Y	Sani-Cloth AF3	
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X
	Sani-Cloth HB	X
	Sani-Cloth Plus	X
Y	Super Sani-Cloth	X
	Transeptic Spray (interim)	Interim
	Tristel Trio Wipes WW DEMO ONLY	
	Tuffie5 Universal Sanitizing Wipe	

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound

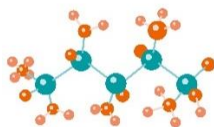

ACUSON NX2 / NX2 Elite ultrasound system


On EPA List	Agent	System
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients	
	Cleanisept	
	Cleanisept Wipes Forte	X
	Clinell Universal Wipes	Interim
Y	Clorox Healthcare Hydrogen Peroxide Wipe	
	IPA (less than 14%)	
	Lightly moistened cloth with a mild detergent	
	Mikrozid Sensitive Wipes	
	Protex Ultra	
	Rinse with water	
	Sani-Cloth	
	Sani-Cloth Active	
	Sani-Cloth AF	X
Y	Sani-Cloth AF3	X
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X
	Sani-Cloth HB	X
	Sani-Cloth Plus	
Y	Super Sani-Cloth	X
	Transeptic Spray (interim)	Interim
	Tristel Trio Wipes WW DEMO ONLY	
	Tuffie5 Universal Sanitizing Wipe	

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound

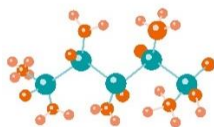

ACUSON NX3 / NX3 Elite ultrasound system

On EPA List	Agent	System
	Any bleach wipe with <1% sodium hypochlorite and no other active ingredients	
	Cleanisept	
	Cleanisept Wipes Forte	X
	Clinell Universal Wipes	Interim
Y	Clorox Healthcare Hydrogen Peroxide Wipe	
	IPA (less than 14%)	
	Lightly moistened cloth with a mild detergent	
	Mikrozid Sensitive Wipes	
	Protex Ultra	
	Rinse with water	
	Sani-Cloth	
	Sani-Cloth Active	
	Sani-Cloth AF	X
Y	Sani-Cloth AF3	X
Y	Sani-Cloth Bleach Germicidal Disposal Wipes	X
	Sani-Cloth HB	X
	Sani-Cloth Plus	
Y	Super Sani-Cloth	X
	Transeptic Spray (interim)	Interim
	Tristel Trio Wipes WW DEMO ONLY	
	Tuffie5 Universal Sanitizing Wipe	

System	System encompasses all system surfaces, including the touch panel, excluding the ultrasound display monitor.
X	Agent approved for use.
Interim	Temporarily approved for use.
Y	On USA EPA List-N: products that meet EPA's criteria for use against against SARS-CoV-2.

Unrestricted

685 East Middlefield Road
Mountain View, California 94043

Tel.: (650) 969 - 9112
Fax: (650) 968 - 2217
www.siemens-
healthineers.com/ultrasound