

All instructions in the operator manual regarding cleaning and, when applicable, regarding disinfecting and sterilization must be always observed.

For information regarding cleaning of RF coils, see (→ Operator Manual Coils).

## **Cleaning the LCD monitor/ video display**

The LCD monitor of the *syngo* Acquisition Workplace and the video display are cleaned in the same way.

- ✧ Clean the LCD monitor/video display at least every two months.
- ✧ Prior to cleaning, switch off the LCD monitor/video display and disconnect the main power plug. But if the monitor/display is under hot running conditions, please wait till the monitor/display is chilled (this can take up to one hour).
- ✧ Clean the monitor/video display using a microfiber cloth.
- ✧ If the LCD monitor/video display cannot be effectively cleaned with the microfiber cloth: use window cleaner. Do *not* use window cleaner on the monitor housing.
- ✧ Immediately remove any water drops from the LCD monitor/video display.
- ✧ Avoid scratching the surface area of the LCD monitor/video display.
- ✧ Avoid impacts to the LCD monitor/video display.



---

The LCD monitor/video display are highly sensitive to mechanical damage.

---

**Cleaning the camera lens**

- ✧ Carefully clean the camera lens with a lint-free cloth and lens cleaner.

**Cleaning the data carrier**

- ✧ Clean contaminated data carriers with a clean cloth (cotton or microfiber).
- ✧ Follow the manufacturer's notes when cleaning CD/DVD data carriers.

**Cleaning the plugs and connectors**

- ✧ Dampen a soft cloth with water or a diluted household cleaner solution. Do *not* use organic solvents such as alcohol or acetone.
- ✧ Carefully wipe the plugs and connectors with the cloth. Do *not* touch the contacts.

**Cleaning the patient table and the straps**

- ✧ Clean the patient table using a liquid household agent.
- ✧ Wash the straps of the patient table at a temperature of 60 °C.
- ✧ Use commercial disinfecting materials. However, do *not* use solutions with alcohol or acetone.

**Cleaning and disinfecting the receptors**

- ✧ Do *not* use cleaners or disinfectants containing alcohol or ether.
- ✧ Do *not* use hard or sharp objects (e.g. knives or tweezers) to remove residue.
- ✧ Clean the receptors with a commercially available cleaner. Follow manufacturer's instructions.
- ✧ Use a dampened cloth for cleaning.



---

Do not submerge the receptors in cleaning liquid.

---

- ✧ Disinfect the receptors with a commercially available disinfecting agent. Follow the manufacturer's instructions.

**Disinfecting system components**

---

Disinfecting sprays damage electronic components. For this reason, components may only be cleaned with a semi-dry cloth.

---

- ✧ Disinfect the system components with commercial disinfecting materials. However, do not use solutions with alcohol or acetone. Follow manufacturer's instructions.



---

Siemens does not test alcohol-containing, phenol-alkaline and alkaline-based disinfectants for harmful effects on surfaces. Avoid disinfecting system components with these materials.

---

**Care and cleaning of floors**

Do *not* use the following cleaning or care products:

- ❑ Sprays
- ❑ Silicon-based cleaning or care products
- ❑ Cleaning or care products with substances that release ammonia
- ❑ Cleaning or care products that destroy the anti-static properties of the floor covering
- ✧ Use commercially available cleaning or care products for the floor. Follow manufacturer's instructions.