Reduce the Dose without Compromising Diagnosis and Therapy. Really?

Image quality and dose are usually viewed as two sides of the same coin: the higher the dose, the better the quality of the image. But today it is possible to produce good images with a lower dose. Here we take a look at radiation hazards, dose values, and a selection of hard- and software solutions.

Text: Hildegard Kaulen | Illustrations: Jens Amende



iagnostic confidence isn't negotiable: what use would a CT scan or projection image be if the clinical questions remained unresolved? X-ray technology has become an indispensable tool for diagnosis and therapy. But X-rays also entail the risk of damaging the genetic material or causing burns. For these reasons, X-ray imaging always involves a compromise between dose and diagnostic needs. Theoretically, every single X-ray quantum absorbed could produce a mutation leading to cancer decades later. Supposedly, it's a linear

relationship: the higher the dose, the greater the risk of cancer. In medical parlance this is called a stochastic risk.

The possibility of skin burns resulting from X-ray quanta hitting a small surface, by contrast, is known as a non-stochastic or deterministic risk. A threshold exists. But in CT scanning this threshold is usually not reached, because the gantry constantly rotates around the patient and scans thin slices. For this reason, it is mainly the cancer risk that is relevant in a CT scan. This is different



in angiography: because the X-ray beam hits the body from one side, high patient entry doses may result at this point. This means that for most angiography cases, the deterministic risk is relevant.

Varying dose values

There are dose values for the energy, effect, and damage of radiation. The energy or absorbed dose represents the amount of energy absorbed per kilogram of body weight. It can be directly measured and is expressed in gray (Gy) units. The so-called equivalent dose, by contrast, takes account of the damage caused by different types of radiation, and cannot be measured directly. Here the unit of measurement is sievert (Sv) or millisievert (mSv). The damage caused by exposure to radiation is expressed by what is commonly called effective dose; this cannot be measured directly either. The effective dose takes account of the sensitivity of different organs and types of tissue to radiation and corresponds to the mean whole-body dose; it is also measured in Sv or mSv.

Since the effective dose enables different examinations to be compared, it is the one used to assess the risk. An important figure in this respect is natural background radiation, which essentially consists of terrestrial and cosmic radiation. Germany, for example, has natural background radiation of around 2.1 mSv per year. Because there are no studies, however, it's not clear what cancer risks are associated with the respective

Different requirements call for different dose reduction solutions in computed tomography and angiography. effective doses. All the calculations done so far have been on the basis of exceptional situations such as the atom bombs dropped on Hiroshima and Nagasaki, and the Chernobyl reactor disaster. There are limits for people who are exposed to radiation because their job involves handling X-ray equipment, closed radiation sources, or open radioactive substances. In Europe, the limit for the occupational effective dose is 20 mSv, and in the United States 50 mSv. These limits do not include medical examinations for diagnostic or treatment purposes.


The ALARA principle

The rule of thumb in medicine is the ALARA principle: the dose should be kept "as low as reasonably achievable". To this end, various hardand software solutions have been developed over the years, notably by Siemens Healthineers. On the hardware side they include improved X-ray tubes and detectors and using filters to pre-harden the X-ray beam; software solutions increasingly automate parameters such as tube current, exposure time, or focal spot size, as well as iterative reconstruction of image data. The dose can also be reduced by means of optimum patient positioning and proper staff training.

However, the requirements for CT and angiography are different. Angiography involves producing large-area projection images that need to have high spatial resolution for the edges of vessels to be visible. The resolution of individual CT images can be lower because the final image is put together from hundreds of individual projections. Another factor is that with angiography, staff carrying out the intervention also have to protect themselves, because some of the X-ray quanta emitted from the source don't leave the patient's body in the direction of the detector but are deflected in other directions as scattered radiation and may thus also hit the staff.

Hardware solutions

X-ray tubes reduce dose via the size of the focal spot. The focal spot is the area of the anode surface where the beam of electrons from the cathode is focused and where the X-rays are generated. The smaller the focal spot, the better the imaging result and thus the relation between image information and adapted dose. This would suggest that a dot-shaped focal spot would be

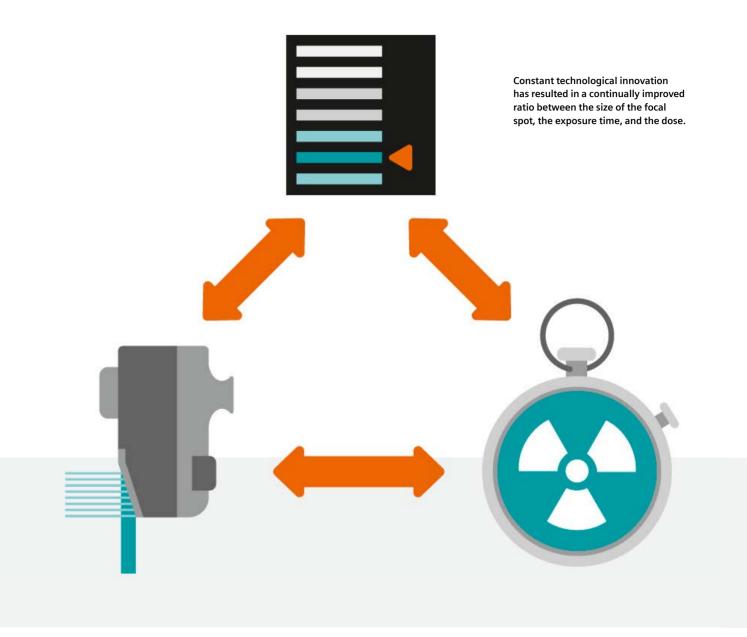

optimal. However, the focused power would generate so much heat that the anode could melt. For this reason, a compromise has to be found between the size of the focal spot, the exposure time, and the dose. An additional reason for using a smaller focal spot is the improved spatial resolution, which increases as the size of the focal spot decreases.

X-rays do not automatically have the optimal energy spectrum for the clinical question being investigated. This is why any quanta that will not be contributing to the image are eliminated by

automatically choosing the thickest possible prefilter. To ensure that the filtered X-rays remain strong enough for imaging, the X-ray source has to be very powerful to compensate for the prefiltered fraction. For this reason, among other things, so-called flat emitters have replaced the spiral-wound filaments that were used in the past, and the electron beam is actively focused in order to optimize the focal spot.

Because only a small number of X-rays ultimately reach the detector, the quanta also have to be converted into image signals as efficiently as

Dose in Imaging Dose in Imaging

possible. An important milestone in CT scanning was the Stellar detector from Siemens Healthineers, which digitizes the measurement signal over a shorter signal path. This means the signal can be transmitted without loss, halving the electronic noise in the low-dose range. This particularly benefits obese patients. A crucial development for angiography was the zen detector, which works with crystalline rather than amorphous silicon. This also reduces the electronic


Software solutions

The dose required for a CT can also be reduced if the scan parameters are automatically adjusted to the patient's anatomy and the clinical guestions being investigated. Uroliths, for example, provide a high level of contrast and can be identified with a lower dose than liver metastases in a homogeneous parenchyma. Smaller people can be scanned at a lower dose than larger individuals. Adjustments of this sort are made by modulating the tube current and voltage.

In a system with automatic tube current modulation, the current is adjusted to the density of the tissue and the patient's anatomy for every single slice. This way organs such as female breasts and the lenses of the eye that are particularly sensitive to radiation can be protected. Tube voltage is adapted for scans involving contrast media. A scan at 80 kV or 100 kV entails a significantly lower dose than one at 120 kV. Iterative reconstruction of image data also helps to indirectly reduce the radiation dose because the algorithm compensates for the elevated noise resulting from the lower dose.

In addition, angiography systems have been recently developed even further. The new imaging chain OPTIQ aims at keeping a constant image-to-noise ratio, which enables high dose savings. This new regulation is being combined with material-specific imaging, which optimizes the X-ray spectrum based on the material to be imaged. Again, this results in additional dose savings.[1]


Automatic adjustments to the clinical questions and the patient's anatomy through software enables the use of individually adapted doses.

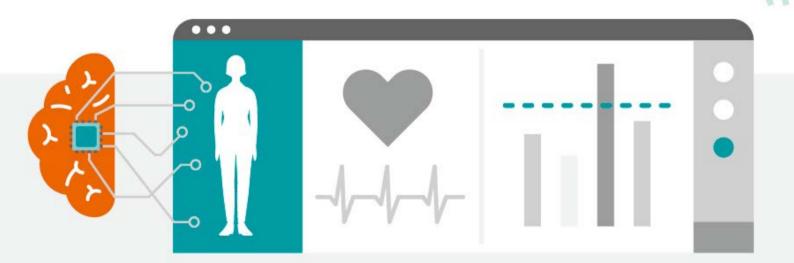

A few years ago, the University of Maastricht's Martijn Kemerink and his colleagues used a historical X-ray machine to determine how high the doses must have been at the time X-rays were discovered. According to their research, in 1895 Wilhelm Conrad Röntgen must have used around 74 mGy for the famous X-ray of the hand of his wife Berta.[2] These days the dose would be around 0.05 mGy - 1,500 times less. Back then, an exposure of 90 minutes was required; today it's 20 milliseconds. Kemerink and his associates found that the historical machine had plenty of room for improvement. And indeed, a lot has happened since then. •

Hildegard Kaulen, PhD, is a molecular biologist. Following time at Rockefeller University in New York and at Harvard Medical School, since the mid-1990s she has been a freelance journalist writing for respected daily newspapers and science magazines.

- [1] Dehairs, M., Bosmans H., and Marshall, N.W., "Implementation of a spatio-temporal figure of merit for new automatic dose rate control regimes in dynamic X-ray imaging", Phys. Med. Biol. 64, 045001 (2019)
- [2] Kemerink, Martijn, PhD, et al, Characteristics of a First-Generation X-Ray System, Radiology, Vol. 259, No. 2, May 2011, 534-539, https://pubs.rsna.org/ doi/10.1148/radiol.11101899. Last accessed February 18, 2020

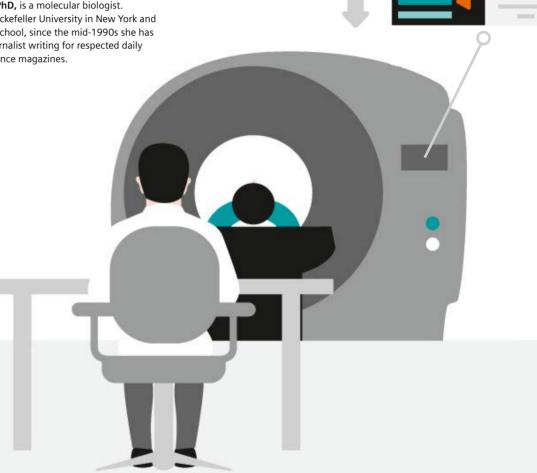

News & Stories · siemens-healthineers.com/news News & Stories · siemens-healthineers.com/news