## Simple-to-follow procedure for trained professionals

## CLINITEST® Rapid COVID-19 Antigen Test

siemens-healthineers.com



Do not use test kit after the expiration date.

Used materials should be discarded as biohazardous waste in accordance with local regulations.

Bring specimen and test components to room temperature if needed (15–30°C) and use the appropriate personal protective equipment.



Carefully insert the nasopharyngeal swab into patient's nostril, gently rotate the swab several times over the surface of the posterior nasopharynx, and gently remove.



Add 10 drops ( $\sim$ 300  $\mu$ L) of the sample extraction buffer into the extraction tube, keeping the tube upright using the stand provided.



Roll the swab 6+ times inside tube, squeeze against bottom and sides of tube, let stand for 1 minute, and squeeze several more times.



Safely remove and discard the swab, and insert dropper tip into the tube containing the specimen.



Lay cassette flat, label, and add four drops ( $\sim$ 100  $\mu$ L) of the test sample into the sample well. Results will be ready in 15 minutes.



Results Key





At Siemens Healthineers, our purpose is to enable healthcare providers to increase value by empowering them on their journey toward expanding precision medicine, transforming care delivery, and improving patient experience, all made possible by digitalizing healthcare.

An estimated 5 million patients globally benefit every day from our innovative technologies and services in the areas of diagnostic and therapeutic imaging, laboratory diagnostics, and molecular medicine, as well as digital health and enterprise services.

We are a leading medical technology company with over 120 years of experience and 18,000 patents globally. Through the dedication of more than 50,000 colleagues in 75 countries, we will continue to innovate and shape the future of healthcare.

CLINITEST is a trademark of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Not available for sale in the U.S. Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

## **Siemens Healthineers Headquarters**

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen, Germany Phone: +49 9131 84-0 siemens-healthineers.com

## Published by

Siemens Healthcare Diagnostics Inc. Point of Care Diagnostics 2 Edgewater Drive Norwood, MA 02062-4637 USA

Phone: +1 781-269-3000

1110110. 11 701 203 3000