



How to play audio files for your patients

Playing in-bore audio files for patients during their scan



To play the provided audio files during a scan, follow these steps:



Download the audio files to a playback device such as a smartphone or tablet.



Connect your device to the intercom's Aux input located in the control room.

- Use a 3.5 mm audio cable, or
- If your device does not have a headphone jack, use a suitable adapter (e.g., USB-C to 3.5 mm).



For maximum sound quality, please adjust the settings as follows:

- On your playback device: Headphones volume 100 %, Media Player volume 1–2 % max
- In your scanner's settings: Intercom volume 2–3 clicks up from zero (patient dependent)
- Directly at the "Select&Go" panel: Adjust volume for the patient



Open the desired audio file on your device and press "Play".