

Simple-to-follow procedure for trained professionals


CLINITEST® Rapid COVID-19 + Influenza Antigen Test

siemens-healthineers.com/clinitest-covid-19-flu-antigen-test



Specimen Collection

Nasopharyngeal

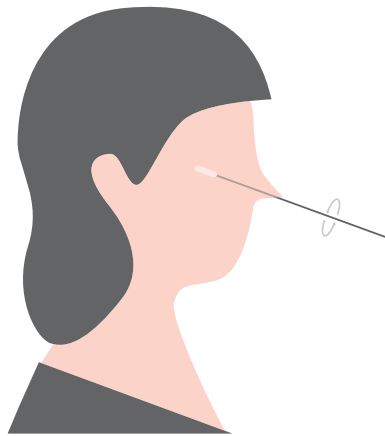


Nasopharyngeal Swab

Carefully insert the sterile swab into patient's nostril ensuring the tip reaches the posterior nasopharynx. Gently rotate the swab several times and gently remove.

Do not use test kit after the expiration date. Used materials should be discarded as biohazardous waste in accordance with local regulations.


Bring specimen and test components to room temperature if needed (15–30°C), and use the appropriate personal protective equipment.

Sample Preparation

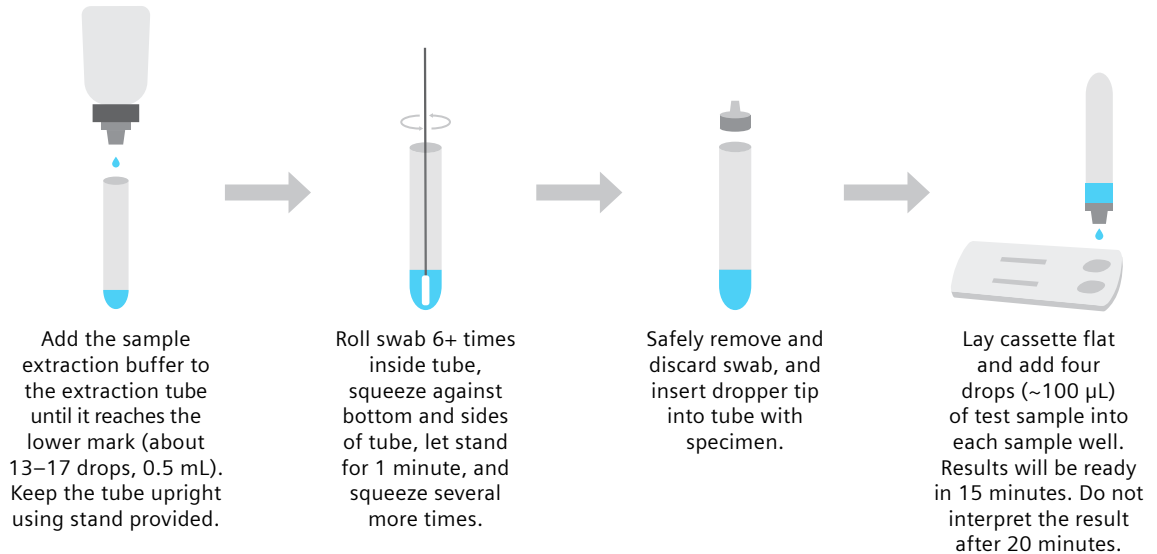


Results Interpretation

For COVID-19

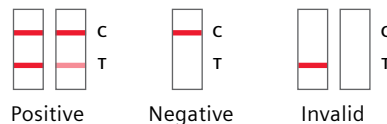

For Flu A/B

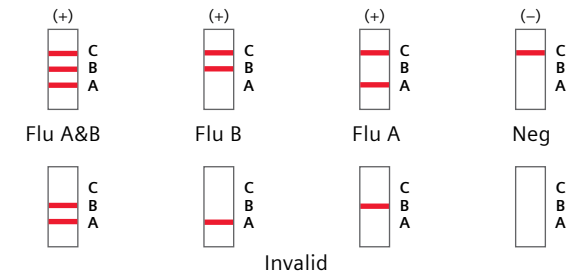

At Siemens Healthineers, we pioneer breakthroughs in healthcare. For everyone. Everywhere. By constantly bringing breakthrough innovations to market, we enable healthcare professionals to deliver high-quality care, leading to the best possible outcome for patients.

Our portfolio, spanning from in-vitro and in-vivo diagnostics to image-guided therapy and innovative cancer care, is crucial for clinical decision-making and treatment pathways. With our strengths in patient twinning, precision therapy, as well as digital, data, and artificial intelligence (AI), we are well positioned to take on the biggest challenges in healthcare. We will continue to build on these strengths to help fight the world's most threatening diseases, improving the quality of outcomes, and enabling access to care.

We are a team of 66,000 highly dedicated employees across more than 70 countries passionately pushing the boundaries of what's possible in healthcare to help improve people's lives around the world.

CLINITEST is a trademark of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Not available for sale in the U.S. Product availability may vary by country. Distributed by Siemens Healthineers.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Published by

Siemens Healthcare Diagnostics Inc.
Point of Care Diagnostics
2 Edgewater Drive
Norwood, MA 02062-4637
USA
Phone: +1 781-269-3000