

Table of contents
-------------------

\\user 2017 E 11
------------------

Clinical mMR
--------------

CihatEldeniz
--------------

A06A ARIA
-----------

AAHead_Scout		
t2_tse_tra_dark-fluid		ARIA
t2_fl2d_tra_hemo		ARIA
ep2d_diff_3scan_trace_p2		ARIA
TRA		SWI
AAHead_Scout		
SAG	T1	FL2D
TRA		T2
TRA	30	DIFFUSION
t2_tse_tra_dark-fluid		ARIA
t2_fl2d_tra_hemo		ARIA
ep2d_diff_3scan_trace_p2		ARIA
TRA		SWI
SAG	T1	MPRAGE
COR	T1FS	TSE

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\AAHead\_Scout

TA: 0:14 PM: REF Voxel size: 1.6×1.6×1.6 mmPAT: 3 Rel. SNR: 1.00 : fl

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
TE	1.37 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	3.15 ms
TE	1.37 ms
Flip angle	8 deg

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1

**Resolution - Common**

FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
Base resolution	160
Phase resolution	100 %
Slice resolution	69 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Trajectory	Cartesian

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	24
Accel. factor 3D	1

**Resolution - iPAT**

Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
Multi-slice mode	Sequential
Series	Ascending
Concatenations	1

**Geometry - AutoAlign**

Slab group	1
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off

**System - Miscellaneous**

Matrix Coil Mode	Auto (Triple)
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Flip angle	8 deg
Measurements	1
Time to center	6.2 s

**Inline - Inline**

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	Off

**Sequence - Part 1**

Introduction	On
Dimension	3D
Asymmetric echo	Weak
Contrasts	1
Multi-slice mode	Sequential
Bandwidth	540 Hz/Px

**Sequence - Part 2**

RF pulse type	Fast
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
------	-----

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\t2\_tse\_tra\_dark-fluid ARIA

TA: 2:08 PM: REF Voxel size: 0.5×0.5×4.0 mmPAT: 3 Rel. SNR: 1.00 : tir

**Properties**

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	9000.0 ms
TE	96.0 ms
Averages	1
Concatenations	2
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

**Contrast - Common**

TR	9000.0 ms
TE	96.0 ms
TD	0.0 ms
MTC	Off
Magn. preparation	Slice-sel. IR
TI	2500 ms
Flip angle	150 deg
Fat suppr.	Fat sat.
Fat sat. mode	Strong
Water suppr.	None
Restore magn.	Off
Freeze suppressed tissue	On

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

**Resolution - Common**

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	256
Phase resolution	85 %

**Resolution - Common**

Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	On

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	34
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	On

**Geometry - Common**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	9000.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	2

**Geometry - AutoAlign**

Slice group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Water suppr.	None
Restore magn.	Off
Special sat.	None

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slices	30
Slice thickness	4.0 mm
Dist. factor	30 %
FoV read	240 mm
FoV phase	100.0 %

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	240 mm
A >> P	240 mm
F >> H	155 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	9000.0 ms
Concatenations	2

**Physio - Cardiac**

Magn. preparation	Slice-sel. IR
TI	2500 ms

**Physio - Cardiac**

Fat suppr.	Fat sat.
Dark blood	Off
FoV read	240 mm
FoV phase	100.0 %
Phase resolution	85 %
Trajectory	Cartesian

**Physio - PACE**

Resp. control	Off
Concatenations	2

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	On
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	9.62 ms
Bandwidth	199 Hz/Px

**Sequence - Part 2**

Define	Turbo factor
Echo trains per slice	6
Phase correction	Automatic
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Hyperecho	Off
WARP	Off
Red. EC sensitivity	Off
Turbo factor	16

**Sequence - Assistant**

Mode	Off
Allowed delay	60 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\t2\_fl2d\_tra\_hemo ARIA

TA: 2:26 PM: REF Voxel size: 0.5×0.5×4.0 mmPAT: 2 Rel. SNR: 1.00 : fl\_r

**Properties**

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	598.0 ms
TE	35.00 ms
Averages	1
Concatenations	2
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

**Contrast - Common**

TR	598.0 ms
TE	35.00 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	256
Phase resolution	85 %
Phase partial Fourier	Off
Interpolation	On

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	On

**Geometry - Common**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	598.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	2

**Geometry - AutoAlign**

Slice group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slices	30
Slice thickness	4.0 mm
Dist. factor	30 %
FoV read	240 mm
FoV phase	100.0 %
Segments	1

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	598.0 ms
Concatenations	2
Segments	1

**Physio - Cardiac**

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	240 mm
FoV phase	100.0 %
Phase resolution	85 %

**Physio - PACE**

Resp. control	Off
Concatenations	2

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Soft Tissue**

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	1
Flow comp.	Slice/Read
Multi-slice mode	Interleaved
Bandwidth	200 Hz/Px

**Sequence - Part 2**

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
Allowed delay	60 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\ep2d\_diff\_3scan\_trace\_p2 ARIA

TA: 0:44 PM: REF Voxel size: 1.9×1.9×4.0 mmPAT: 2 Rel. SNR: 1.00 : epse

**Properties**

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	6100 ms
TE	72.0 ms
Averages	1
Concatenations	1
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	6100 ms
TE	72.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms

**Resolution - Common**

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	128
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2

**Resolution - iPAT**

Ref. lines PE	30
Matrix Coil Mode	Auto (Triple)
Reference scan mode	EPI/separate

**Resolution - Filter Image**

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

**Resolution - Filter Rawdata**

Raw filter	On
Elliptical filter	Off

**Geometry - Common**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slice group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T



**System - Miscellaneous**

Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	155 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	6100 ms
Concatenations	1

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Diff - Neuro**

Diffusion mode	3-Scan Trace
Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	2
b-value 1	0 s/mm <sup>2</sup>
b-value 2	1000 s/mm <sup>2</sup>
b-value 1	1
b-value 2	1
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

**Diff - Body**

Diffusion mode	3-Scan Trace
Diff. directions	3

**Diff - Body**

Diffusion Scheme	Bipolar
Diff. weightings	2
b-value 1	0 s/mm <sup>2</sup>
b-value 2	1000 s/mm <sup>2</sup>
b-value 1	1
b-value 2	1
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm <sup>2</sup>
Noise level	40

**Diff - Composing**

Inline Composing	Off
Distortion Corr.	Off

**Sequence - Part 1**

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.68 ms
Bandwidth	1698 Hz/Px

**Sequence - Part 2**

EPI factor	128
RF pulse type	Normal
Gradient mode	Fast

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\TRA SWI

TA: 2:15 PM: REF Voxel size: 0.6×0.6×2.0 mmPAT: 6 Rel. SNR: 1.00 : fl\_r

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	20.0 %
Slices per slab	80
FoV read	220 mm
FoV phase	87.5 %
Slice thickness	2.00 mm
TR	30.0 ms
TE	20.00 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	30.0 ms
TE	20.00 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

**Contrast - Dynamic**

Averages	1
Averaging mode	Short term
Reconstruction	Magn./Phase
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	220 mm
FoV phase	87.5 %
Slice thickness	2.00 mm
Base resolution	384
Phase resolution	100 %
Slice resolution	81 %
Phase partial Fourier	Off

**Resolution - Common**

Slice partial Fourier	Off
Interpolation	Off

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	30
Accel. factor 3D	2
Ref. lines 3D	30
Matrix Coil Mode	Auto (Triple)
Reference scan mode	GRE/separate

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	20.0 %
Slices per slab	80
FoV read	220 mm
FoV phase	87.5 %
Slice thickness	2.00 mm
TR	30.0 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slab group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None

**Geometry - Saturation**

Special sat.	None
--------------	------

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slabs	1
Slices per slab	80
Slice thickness	2.00 mm
Dist. factor	20 %
FoV read	220 mm
FoV phase	87.5 %
Segments	1

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	193 mm
A >> P	220 mm
F >> H	160 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	30.0 ms
Concatenations	1
Segments	1

**Physio - Cardiac**

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	220 mm
FoV phase	87.5 %
Phase resolution	100 %

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Soft Tissue**

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	1
Flow comp.	Yes
Multi-slice mode	Sequential
Bandwidth	100 Hz/Px

**Sequence - Part 2**

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
Allowed delay	0 s

\\user 2017 E 11\\Clinical mMR\\CihatEldeniz\\A06A ARIA\\AAHead\_Scout

TA: 0:14 PM: REF Voxel size: 1.6×1.6×1.6 mmPAT: 3 Rel. SNR: 1.00 : fl

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
TE	1.37 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	3.15 ms
TE	1.37 ms
Flip angle	8 deg

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1

**Resolution - Common**

FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
Base resolution	160
Phase resolution	100 %
Slice resolution	69 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Trajectory	Cartesian

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	24
Accel. factor 3D	1

**Resolution - iPAT**

Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	128
FoV read	260 mm
FoV phase	100.0 %
Slice thickness	1.6 mm
TR	3.15 ms
Multi-slice mode	Sequential
Series	Ascending
Concatenations	1

**Geometry - AutoAlign**

Slab group	1
Position	L0.0 A10.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off

**System - Miscellaneous**

Matrix Coil Mode	Auto (Triple)
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Flip angle	8 deg
Measurements	1
Time to center	6.2 s

**Inline - Inline**

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	Off

**Sequence - Part 1**

Introduction	On
Dimension	3D
Asymmetric echo	Weak
Contrasts	1
Multi-slice mode	Sequential
Bandwidth	540 Hz/Px

**Sequence - Part 2**

RF pulse type	Fast
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
------	-----

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\SAG T1 FL2D

TA: 0:41 PM: ISO Voxel size: 0.9×0.9×4.0 mmPAT: 2 Rel. SNR: 1.00 : fl

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	On
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	35
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Basis
Phase oversampling	40 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	240.0 ms
TE	2.46 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEP;NEP

**Contrast - Common**

TR	240.0 ms
TE	2.46 ms
MTC	Off
Magn. preparation	None
Flip angle	80 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

**Contrast - Dynamic**

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	256
Phase resolution	81 %
Phase partial Fourier	Off
Interpolation	Off

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	48
Matrix Coil Mode	Dual
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	On

**Geometry - Common**

Slice group	1
Slices	35
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	240.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slice group	1
AutoAlign	Head > Basis
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

**Geometry - Saturation**

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slices	35
Slice thickness	4.0 mm
Dist. factor	20 %
FoV read	220 mm
FoV phase	100.0 %
Segments	1

**System - Miscellaneous**

Positioning mode	ISO
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Dual
AutoAlign	Head > Basis
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	240.0 ms
Concatenations	1
Segments	1

**Physio - Cardiac**

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	220 mm
FoV phase	100.0 %
Phase resolution	81 %

**Physio - PACE**

Resp. control	Breath-hold
Concatenations	1

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Soft Tissue**

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	Off
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	360 Hz/Px

**Sequence - Part 2**

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
Allowed delay	40 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\TRA T2

TA: 1:32 PM: REF Voxel size: 0.4×0.4×5.0 mmPAT: 3 Rel. SNR: 1.00 : tse

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	26
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	Head > Brain
Phase oversampling	10 %
FoV read	220 mm
FoV phase	93.8 %
Slice thickness	5.0 mm
TR	6000.0 ms
TE	86.0 ms
Averages	2
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	6000.0 ms
TE	86.0 ms
MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	None
Water suppr.	None
Restore magn.	Off

**Contrast - Dynamic**

Averages	2
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	220 mm
FoV phase	93.8 %
Slice thickness	5.0 mm
Base resolution	256
Phase resolution	100 %
Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	On

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	26
Matrix Coil Mode	Dual
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slice group	1
Slices	26
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
FoV read	220 mm
FoV phase	93.8 %
Slice thickness	5.0 mm
TR	6000.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slice group	1
AutoAlign	Head > Brain
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Fat suppr.	None
Water suppr.	None
Restore magn.	Off
Special sat.	None

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm



**Geometry - Tim Planning Suite**

Inline Composing	Off
------------------	-----

**Geometry - Tim CT**

Tim CT mode	Off
Slices	26
Slice thickness	5.0 mm
Dist. factor	20 %
FoV read	220 mm
FoV phase	93.8 %

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Dual
AutoAlign	Head > Brain
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	6000.0 ms
Concatenations	1

**Physio - Cardiac**

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	220 mm
FoV phase	93.8 %
Phase resolution	100 %
Trajectory	Cartesian

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	On
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	9.52 ms
Bandwidth	219 Hz/Px

**Sequence - Part 2**

Define	Turbo factor
Echo trains per slice	7
Phase correction	Automatic
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Normal
Hyperecho	Off
WARP	Off
Red. EC sensitivity	Off
Turbo factor	15

**Sequence - Assistant**

Mode	TR
Max. TR	6500.0 ms
Allowed delay	60 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\TRA 30 DIFFUSION

TA: 5:52 PM: REF Voxel size: 1.0×1.0×2.0 mmPAT: 2 Rel. SNR: 1.00 : epse

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	80
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	10000 ms
TE	79.0 ms
Averages	1
Concatenations	1
Filter	Raw filter, Dynamic Field Corr., Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	10000 ms
TE	79.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

**Resolution - Common**

FoV read	230 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
Base resolution	116
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	On

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Matrix Coil Mode	Auto (Triple)
Reference scan mode	GRE/separate

**Resolution - Filter Image**

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	On
Unfiltered images	Off

**Resolution - Filter Rawdata**

Raw filter	On
Elliptical filter	Off

**Geometry - Common**

Slice group	1
Slices	80
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	10000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slice group	1
AutoAlign	Head > Brain
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**System - Miscellaneous**

Positioning mode	REF
------------------	-----

**System - Miscellaneous**

Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Coil Mode	Auto (Triple)
AutoAlign	Head > Brain
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	230 mm
R >> L	230 mm
F >> H	160 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	10000 ms
Concatenations	1

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Diff - Neuro**

Diffusion mode	MDDW
Diff. directions	30
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm <sup>2</sup>
b-value 2	1000 s/mm <sup>2</sup>
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
FA maps	On
Mosaic	On
Tensor	On
Noise level	40

**Diff - Body**

Diffusion mode	MDDW
Diff. directions	30
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm <sup>2</sup>
b-value 2	1000 s/mm <sup>2</sup>
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	On
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm <sup>2</sup>
Noise level	40

**Diff - Composing**

Inline Composing	Off
Distortion Corr.	Off

**Sequence - Part 1**

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.65 ms
Bandwidth	1796 Hz/Px

**Sequence - Part 2**

EPI factor	116
RF pulse type	Normal
Gradient mode	Fast

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\t2\_tse\_tra\_dark-fluid ARIA

TA: 2:08 PM: REF Voxel size: 0.5×0.5×4.0 mmPAT: 3 Rel. SNR: 1.00 : tir

**Properties**

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	9000.0 ms
TE	96.0 ms
Averages	1
Concatenations	2
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

**Contrast - Common**

TR	9000.0 ms
TE	96.0 ms
TD	0.0 ms
MTC	Off
Magn. preparation	Slice-sel. IR
TI	2500 ms
Flip angle	150 deg
Fat suppr.	Fat sat.
Fat sat. mode	Strong
Water suppr.	None
Restore magn.	Off
Freeze suppressed tissue	On

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

**Resolution - Common**

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	256
Phase resolution	85 %

**Resolution - Common**

Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	On

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	34
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	On

**Geometry - Common**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	9000.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	2

**Geometry - AutoAlign**

Slice group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Water suppr.	None
Restore magn.	Off
Special sat.	None

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slices	30
Slice thickness	4.0 mm
Dist. factor	30 %
FoV read	240 mm
FoV phase	100.0 %

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	240 mm
A >> P	240 mm
F >> H	155 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	9000.0 ms
Concatenations	2

**Physio - Cardiac**

Magn. preparation	Slice-sel. IR
TI	2500 ms

**Physio - Cardiac**

Fat suppr.	Fat sat.
Dark blood	Off
FoV read	240 mm
FoV phase	100.0 %
Phase resolution	85 %
Trajectory	Cartesian

**Physio - PACE**

Resp. control	Off
Concatenations	2

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	On
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	9.62 ms
Bandwidth	199 Hz/Px

**Sequence - Part 2**

Define	Turbo factor
Echo trains per slice	6
Phase correction	Automatic
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Hyperecho	Off
WARP	Off
Red. EC sensitivity	Off
Turbo factor	16

**Sequence - Assistant**

Mode	Off
Allowed delay	60 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\t2\_fl2d\_tra\_hemo ARIA

TA: 1:32 PM: REF Voxel size: 0.5×0.5×4.0 mmPAT: 2 Rel. SNR: 1.00 : fl\_r

**Properties**

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	745.0 ms
TE	20.00 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

**Contrast - Common**

TR	745.0 ms
TE	20.00 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	256
Phase resolution	85 %
Phase partial Fourier	Off
Interpolation	On

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	On

**Geometry - Common**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	745.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slice group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slices	30
Slice thickness	4.0 mm
Dist. factor	30 %
FoV read	240 mm
FoV phase	100.0 %
Segments	1

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	745.0 ms
Concatenations	1
Segments	1

**Physio - Cardiac**

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	240 mm
FoV phase	100.0 %
Phase resolution	85 %

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Soft Tissue**

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	1
Flow comp.	Slice/Read
Multi-slice mode	Interleaved
Bandwidth	200 Hz/Px

**Sequence - Part 2**

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
Allowed delay	60 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\ep2d\_diff\_3scan\_trace\_p2 ARIA

TA: 0:44 PM: REF Voxel size: 1.9×1.9×4.0 mmPAT: 2 Rel. SNR: 1.00 : epse

**Properties**

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	6100 ms
TE	72.0 ms
Averages	1
Concatenations	1
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	6100 ms
TE	72.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

**Contrast - Dynamic**

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms

**Resolution - Common**

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	128
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2

**Resolution - iPAT**

Ref. lines PE	30
Matrix Coil Mode	Auto (Triple)
Reference scan mode	EPI/separate

**Resolution - Filter Image**

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

**Resolution - Filter Rawdata**

Raw filter	On
Elliptical filter	Off

**Geometry - Common**

Slice group	1
Slices	30
Dist. factor	30 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	6100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slice group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T



**System - Miscellaneous**

Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	155 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	6100 ms
Concatenations	1

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Diff - Neuro**

Diffusion mode	3-Scan Trace
Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	2
b-value 1	0 s/mm <sup>2</sup>
b-value 2	1000 s/mm <sup>2</sup>
b-value 1	1
b-value 2	1
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

**Diff - Body**

Diffusion mode	3-Scan Trace
Diff. directions	3

**Diff - Body**

Diffusion Scheme	Bipolar
Diff. weightings	2
b-value 1	0 s/mm <sup>2</sup>
b-value 2	1000 s/mm <sup>2</sup>
b-value 1	1
b-value 2	1
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm <sup>2</sup>
Noise level	40

**Diff - Composing**

Inline Composing	Off
Distortion Corr.	Off

**Sequence - Part 1**

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.68 ms
Bandwidth	1698 Hz/Px

**Sequence - Part 2**

EPI factor	128
RF pulse type	Normal
Gradient mode	Fast

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\TRA SWI

TA: 2:15 PM: REF Voxel size: 0.6×0.6×2.0 mmPAT: 6 Rel. SNR: 1.00 : fl\_r

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	20.0 %
Slices per slab	80
FoV read	220 mm
FoV phase	87.5 %
Slice thickness	2.00 mm
TR	30.0 ms
TE	20.00 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	30.0 ms
TE	20.00 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

**Contrast - Dynamic**

Averages	1
Averaging mode	Short term
Reconstruction	Magn./Phase
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	220 mm
FoV phase	87.5 %
Slice thickness	2.00 mm
Base resolution	384
Phase resolution	100 %
Slice resolution	81 %
Phase partial Fourier	Off

**Resolution - Common**

Slice partial Fourier	Off
Interpolation	Off

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	30
Accel. factor 3D	2
Ref. lines 3D	30
Matrix Coil Mode	Auto (Triple)
Reference scan mode	GRE/separate

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	20.0 %
Slices per slab	80
FoV read	220 mm
FoV phase	87.5 %
Slice thickness	2.00 mm
TR	30.0 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	1

**Geometry - AutoAlign**

Slab group	1
AutoAlign	---
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

**Geometry - Saturation**

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None

**Geometry - Saturation**

Special sat.	None
--------------	------

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slabs	1
Slices per slab	80
Slice thickness	2.00 mm
Dist. factor	20 %
FoV read	220 mm
FoV phase	87.5 %
Segments	1

**System - Miscellaneous**

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Coil Mode	Auto (Triple)
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	193 mm
A >> P	220 mm
F >> H	160 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	30.0 ms
Concatenations	1
Segments	1

**Physio - Cardiac**

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	220 mm
FoV phase	87.5 %
Phase resolution	100 %

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Soft Tissue**

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	1
Flow comp.	Yes
Multi-slice mode	Sequential
Bandwidth	100 Hz/Px

**Sequence - Part 2**

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

**Sequence - Assistant**

Mode	Off
Allowed delay	0 s

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\SAG T1 MPRAGE

TA: 4:44 PM: FIX Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : tfl

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

**Routine**

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Basis
Phase oversampling	0 %
Slice oversampling	18.2 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	2300.0 ms
TE	2.26 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(3D), Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	2300.0 ms
TE	2.26 ms
Magn. preparation	Non-sel. IR
TI	900 ms
Flip angle	8 deg
Fat suppr.	None
Water suppr.	None

**Contrast - Dynamic**

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	7/8
Slice partial Fourier	7/8

**Resolution - Common**

Interpolation	Off
---------------	-----

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Matrix Coil Mode	Dual
Reference scan mode	Integrated

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	18.2 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	2300.0 ms
Multi-slice mode	Single shot
Series	Ascending
Concatenations	1

**Geometry - AutoAlign**

Slab group	1
AutoAlign	Head > Basis
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

**Geometry - Navigator****Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm

**Geometry - Tim Planning Suite**

Inline Composing	Off
------------------	-----

**System - Miscellaneous**

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Dual
AutoAlign	Head > Basis
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	On
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	2300.0 ms
Concatenations	1

**Physio - Cardiac**

Magn. preparation	Non-sel. IR
TI	900 ms
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	100.0 %
Phase resolution	100 %

**Physio - PACE**

Resp. control	Off
Concatenations	1

**Inline - Common**

Subtract	Off
Measurements	1

**Inline - Common**

StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	6.8 ms
Bandwidth	200 Hz/Px

**Sequence - Part 2**

RF pulse type	Normal
Gradient mode	Normal
Excitation	Non-sel.
RF spoiling	On
Incr. Gradient spoiling	Off
Turbo factor	182

**Sequence - Assistant**

Mode	Off
------	-----

\\user 2017 E 11\Clinical mMR\CihatEldeniz\A06A ARIA\COR T1FS TSE

TA: 1:49 PM: ISO Voxel size: 0.4×0.4×5.0 mmPAT: 2 Rel. SNR: 1.00 : tse

**Properties**

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

**Routine**

Slice group	1
Slices	26
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	Head > Brain
Phase oversampling	20 %
FoV read	230 mm
FoV phase	84.4 %
Slice thickness	5.0 mm
TR	800.0 ms
TE	8.5 ms
Averages	2
Concatenations	2
Filter	Distortion Corr.(2D), Prescan Normalize
Coil elements	HEA;HEP

**Contrast - Common**

TR	800.0 ms
TE	8.5 ms
TD	0.0 ms
MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	Fat sat.
Fat sat. mode	Strong
Water suppr.	None
Restore magn.	Off

**Contrast - Dynamic**

Averages	2
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

**Resolution - Common**

FoV read	230 mm
FoV phase	84.4 %
Slice thickness	5.0 mm
Base resolution	256
Phase resolution	75 %
Phase partial Fourier	Off
Trajectory	Cartesian

**Resolution - Common**

Interpolation	On
---------------	----

**Resolution - iPAT**

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Matrix Coil Mode	Dual
Reference scan mode	Self-calibration

**Resolution - Filter Image**

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

**Resolution - Filter Rawdata**

Raw filter	Off
Elliptical filter	Off

**Geometry - Common**

Slice group	1
Slices	26
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	230 mm
FoV phase	84.4 %
Slice thickness	5.0 mm
TR	800.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	2

**Geometry - AutoAlign**

Slice group	1
AutoAlign	Head > Brain
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Coronal

**Geometry - Saturation**

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Water suppr.	None
Restore magn.	Off
Special sat.	None

**Geometry - Navigator**

**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

**Geometry - Tim CT**

Tim CT mode	Off
Slices	26
Slice thickness	5.0 mm
Dist. factor	20 %
FoV read	230 mm
FoV phase	84.4 %

**System - Miscellaneous**

Positioning mode	ISO
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Coil Mode	Dual
AutoAlign	Head > Brain
Coil Select Mode	Default

**System - Adjustments**

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

**System - Adjust Volume**

Position	Isocenter
Orientation	Coronal
Rotation	0.00 deg
R >> L	195 mm
F >> H	230 mm
A >> P	155 mm
Reset	Off

**System - Tx/Rx**

Frequency 1H	123.214369 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

**Physio - Signal1**

1st Signal/Mode	None
TR	800.0 ms
Concatenations	2

**Physio - Cardiac**

Magn. preparation	None
Fat suppr.	Fat sat.
Dark blood	Off
FoV read	230 mm

**Physio - Cardiac**

FoV phase	84.4 %
Phase resolution	75 %
Trajectory	Cartesian

**Physio - PACE**

Resp. control	Off
Concatenations	2

**Inline - Common**

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

**Inline - MIP**

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save Original Images	On

**Inline - Composing**

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

**Sequence - Part 1**

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	On
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	8.5 ms
Bandwidth	260 Hz/Px

**Sequence - Part 2**

Define	Turbo factor
Echo trains per slice	33
Phase correction	Automatic
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Hyperecho	Off
WARP	Off
Red. EC sensitivity	Off
Turbo factor	3

**Sequence - Assistant**

Mode	Off
Allowed delay	60 s