

For *syngo* iFlow no separate DSA runs are necessary

Courtesy of

PD Dr. Jan Hinrichs, MD Institute for Diagnostic and Interventional Radiology, Medical School Hannover, Germany

Supported by syngo iFlow

System & Software Artis pheno VE 10

Case Description

Patient history

74-year-old female patient with history of recurrent pulmonary embolism and chronic embolic pulmonary hypertension (CTEPH).

Diagnosis

CTEPH with perfusion defects.

Treatment

In the third balloon angioplasty procedure, we focused on the right middle and right upper lobe.

General comments

With syngo iFlow we were able to visualize the improvement of peripheral filling of the treated vessels.

ROIs create contrast concentration versus time curves.

Tips and tricks

For the use of *syngo* iFlow, 4 F/s DSA run is sufficient. No separate DSA runs are necessary.

Coordinate timing of injection for direct pre- and posttreatment comparison.



Arthrography of lunatummalacia right wrist using syngo DynaCT Micro

Acquisition protocol	DSA 4 F/s ~27 frames
Injection protocol	
Catheter position	Upper right lung lobe artery
Contrast medium (CM)	300 mg iodine/mL
Dilution (CM/Saline):	60%
Injection volume	8 mL
Injection rate	Hand injection
Duration of injection	10 s
X-ray delay	Injection starts right after mask image
Power injector used	No

Clinical Images

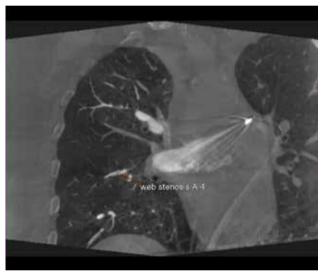




Figure 1: Web stenoses visible on *syngo* DynaCT images

Clinical Images

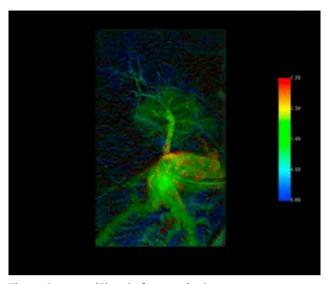
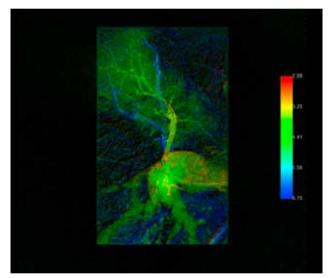


Figure 2: syngo iFlow before angioplasty



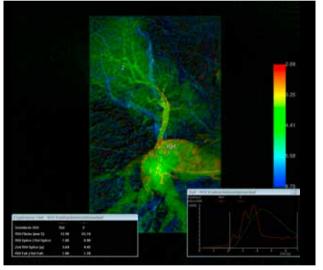


Figure 3: syngo iFlow after angioplasty visualizes the improvement of peripheral blood flow

Siemens Healthineers Headquarters

Siemens Healthcare GmbH

Henkestr. 127

91052 Erlangen, Germany Phone: +49 9131 84-0 siemens-healthineers.com The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.

All rights reserved.