



CleanGuide **Comprehensive cleaning concept**

ARTIS pheno® – Because infection control matters

siemens.com/artis-pheno

SIEMENS
Healthineers

CleanGuide

Improve infection control

Purpose of this document

This CleanGuide is a non-binding recommendation developed jointly by Siemens Healthineers and ECOLAB on how to clean and disinfect ARTIS pheno®, and is intended both for intermediate and terminal cleaning. Adaptations due to application- or region-specific requirements need to be considered, as well. This document contains ECOLAB and Siemens Healthineers specific information. Please also observe the individual product information in the operator manual.

Indications and caution

Antibiotic-resistant pathogens are not characterized by a higher resistance to disinfectants. A patient's infection status (e.g. "septic" patients with chronic viral disease like HIV and hepatitis, or patients with multi-resistant pathogens like MRSA and MRGN) does not necessarily indicate:

- changes to the intermediate disinfection of the treatment rooms and ORs or
- an extension of the contact time: extra wetting of the surfaces is not necessary.

All surfaces in the examination rooms must be completely dry before the rooms can be entered or used! In the case of acute infectious diseases with non-enveloped viruses or spores, the standard disinfection process must be adjusted (see recommended materials).

Caution:

- Do not let cleaning liquids seep into the openings of the system, e.g. air openings, gaps between covers.
- Use only substances for cleaning and disinfection, which are recommended.

Recommended materials

In this paragraph, we distinguish between materials used to disinfect ARTIS pheno, and those used to disinfect the floor directly surrounding ARTIS pheno:



ARTIS pheno disinfection
Standard

ECOLAB® Incidin® OxyWipes or
ECOLAB® Incidin® S OxyWipes



ARTIS pheno disinfection
Non-enveloped viruses or spores (e.g.
hepatitis A, clostridia, norovirus)

ECOLAB® Incidin® S OxyWipes



Floor disinfection & ARTIS pheno
disinfection using the ECOLAB Incidin®
Dry Wipe system

ECOLAB® Incidin® Pro 0.5
impregnated mops & dry wipes

General cleaning and disinfection instructions

- While cleaning and disinfecting, wear gloves with chemical resistance.
- Always work from top to bottom and from clean to dirty areas.
- After removing blood, discard the wet wipe or mop. Use a new one to continue cleaning.
- Use mops and wipes only as long as they leave a closed liquid film on the surface.
- Dispose of used mops and wipes according the instructions of your facility.

1. Preparation

- Disinfection of the floor directly surrounding ARTIS pheno system: An adequate number of mops must be prepared in the morning (according to the dilution instructions of Incidin® Pro, and the instructions for impregnation of mops at your facility).
- Surface disinfection of ARTIS pheno system:
 - The buckets of Incidin® Dry Wipes for disinfection of surfaces must be filled with Incidin® Pro 0.5 %.
 - Incidin® OxyWipes or Incidin® OxyWipes S must be on hand.
- The start of any intermediate disinfection is determined by the nurse. If necessary, the staff can give instructions to remove drapes from areas further away from the patient (e.g. C-arm) and disinfect those while the patient is present in the examination room.
- Drive ARTIS pheno into “cleaning position” (RAO 90°) and activate block radiation and block unit movements
- If used, remove the drapes from C-arm, patient table, control modules, and footswitch.



2. Intermediate disinfection

- A. In case there is visible soiling of the floor: disinfect the floor below and directly around the patient table.
B. Use wet or impregnated dry wipes to disinfect the following surfaces, in the following order:



C-arm: wipe from detector to tube



Patient mattress: wipe from head-end to foot-end



Positioning aids and any other accessories used



Handles of the monitors
• Wipe from the top down in each case



If necessary:
• Base of the patient table
• Trolley
Wipe from the top down in each case



Control modules:

- Joysticks, buttons, surfaces
- Give attention to the gaps between

Upper- and lower-body radiation protection:
wipe from the top down in each case

Operating light, in this order:

- Illuminated area
- Outer surface up to the first joint
- Handles

Work from clean to dirty areas.



Footswitch

C. Use the impregnated mops to disinfect the floor. Work from clean to dirty areas.

D. The surfaces in the OR must be completely dry before entering.

E. Dispose used wipes, mops and, gloves after you have finished cleaning.

3. Terminal disinfection of ARTIS pheno®

The procedure of the terminal disinfection is the same as the procedure of the intermediate disinfection. Additionally, after step i), the followings steps of disinfection need to be carried out (before disinfecting the floor):



- j Mattress: all sides
- k Monitor: all sides, remove any residues
- l Patient table surface: wipe from the top down
- m all surfaces of ARTIS pheno beyond the C-arm: wipe toward the base

4. Cleaning

To ensure good hygiene practice, preserve material integrity long-term and avoid any buildup of residues, a regular cleaning step is required. This cleaning step is important, no matter whether cleaner-disinfectants or just disinfectants are used for disinfection.

A cleaner-disinfectant enables a one-step process in most cases, pre-empting a cleaning step preceding each disinfection step. However, use of a cleaner-disinfectant does not replace a regular cleaning process. If ARTIS pheno is used heavily, a weekly cleaning step is strongly recommended.

Periodic cleaning of surfaces close to the patient

For cleaning the areas close to the patient, please refer to the process for disinfecting these areas of ARTIS pheno, i.e. steps 2 to 4, above.

Cleaning of surfaces not close to the patient

Surfaces which are not close to the patient do not need to be disinfected after each patient, or even every day. The following surfaces have to be cleaned regularly and only disinfected if there is visible contamination:

- Elements beyond the joint of the C-arm:
 - C-arm: cable routing
 - Cover of the robot incl. sealing rubbers and rotating elements
- Surfaces below the patient table
 - Electrical connections of the table
 - Accessory rail at the table
 - Cover of the height-adjustable table
- Display and display mounts

Intermediate disinfection

Preparation steps

Drive ARTIS pheno® into
"cleaning position"

Remove drapes from C-arm, patient
table, and control modules

Start of disinfection

- a C-arm: wipe from detector to tube
- b Patient mattress: wipe from head-end to foot-end
- c Control modules:
 - Joysticks, buttons, surfaces
 - Give attention to the gaps between
- d Upper- and lower-body radiation protection
- e Positioning aids and any other accessories used
- f Handles of the monitors
- g Operating light: handles, illuminated area and outer surface up to the first angle
- h Footswitch
- i If necessary: bases of the patient table and trolley

General Instructions

- While cleaning and disinfecting, wear gloves with chemical resistance.
- Always work from top to bottom and from clean to dirty areas.
- After removing blood, discard the wet wipe or mop. Use a new one to continue cleaning.
- Use mops and wipes only as long as they leave a closed liquid film on the surface.
- Dispose of used mops and wipes according to the instructions of your facility.

Terminal disinfection

Continue disinfection

- j Mattress: all sides
- k Monitor: all sides, remove any residues
- l Patient table surface
- m All surfaces of the ARTIS pheno beyond the C-arm: wipe toward the base

Use pre-soaked mops to
disinfect the floor

End of disinfection

General Instructions

The surfaces in the examination room must be completely dry before entering.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and are subject to change without prior notice.

Some/ALL of the features and products described herein may not be available in the United States or other countries.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features that do not always have to be present in individual cases.

Siemens reserves the right to modify the design, packaging, specifications and options described herein without prior notice. Please contact your local Siemens sales representative for the most current information.

In the interest of complying with legal requirements concerning the environmental compatibility of our products (protection of natural resources and waste conservation), we recycle certain components. Using the same extensive quality assurance measures as for factory-new components, we guarantee the quality of these recycled components.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

Caution: Federal law restricts this device to sale by or on the order of a physician.

ECOLAB products are not sold by Siemens. They have to be ordered, installed, and serviced by way of separate contracts with the vendor in question.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone +49 9131 84-0
[siemens.com/healthineers](https://www.siemens.com/healthineers)