
Mechanical thrombectomy of left MCA occlusion

Interventional Neuroradiology

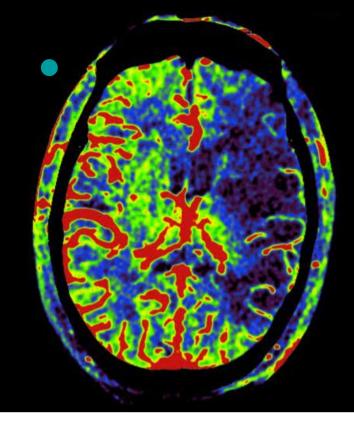

The individual X-ray delay for syngo Neuro PBV IR can be analyzed by syngo iFlow.

Courtesy of

Prof. Wan-You Guo, Department of Radiology, Taipei Veterans General Hospital, Taiwan

Supported by

syngo DynaCT syngo Neuro PBV IR syngo iFlow

System & Software

Artis zee biplane VC14 syngo X Workplace VB15

Case Description

Patient history

A 67-year-old man presented with symptoms of acute right hemiplegia of less than 6 hours duration since onset. The patient was taken to CT immediately to exclude hemorrhage with native CT.

Diagnosis

Left middle cerebral artery (MCA) occlusion. Noncontrast CT demonstrated a hyperdense MCA sign and excluded intracranial hemorrhage. Flat-detector CT (FDCT) angiography demonstrated the total occlusion of left MCA. A parenchymal cerebral blood volume (syngo Neuro PBV IR) map depicted a large area of hypoperfusion in the corresponding left MCA territory, which was similar to the results of multidetector CT perfusion imaging (not shown). After intra-arterial mechanical thrombectomy, recanalization of the left MCA was demonstrated by FDCT

angiography. A *syngo* Neuro PBV IR map depicted the recovery of CBV values in part of the hypoperfused parenchyma after revascularization.

Treatment

Revascularization with intra-arterial approach. MCA occlusion could be successfully reopened.

General comments

By combining C-arm rotational acquisition with intraarterial contrast medium injection from the aortic arch, we can obtain

- (1) non-contrast syngo DynaCT,
- (2) a three-dimensional volume of intracranial vasculature (syngo DynaCT reconstruction of fill run), and
- (3) a syngo Neuro PBV IR map.

Related publication

BioMed Research International Volume 2013, Article ID 873614, 7 pages http://dx.doi. org/10.1155/2013/873614



Mechanical thrombectomy of left MCA occlusion

Acquisition protocol	8s Neuro PBV IR
Injection protocol	
Catheter position	Aortic arch 4 F pigtail catheter
Contrast medium (CM)	340 mg iodine/mL
Dilution	50 %
Injection volume	85 mL
Injection rate	5 mL/s
Duration of injection	17 s
X-ray delay	Individually analyzed with <i>syngo</i> iFlow Tmax = 9 seconds for this patient in the superior sagittal sinus
Power injector used	Yes

Reconstructions	Primary	Secondary
Name	Neuro PBV HU Smooth	DynaCT Head Nat Fill HU ¹
VOI size	Full	
Slice matrix	512×512	
Kernel type	HU	HU
Image characteristics	Smooth	Normal
Reconstruction mode	Neuro PBV	Nat fill
Viewing preset	Neuro PBV	DynaCT Head

Clinical Images

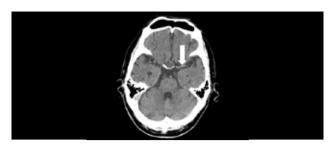


Figure 1: Noncontrast CT demonstrated a hyperdense MCA sign (arrow) and excluded intracranial hemorrhage

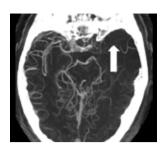


Figure 2: Thick MIP FDCT angiography demonstrated the total occlusion of left MCA (arrow)

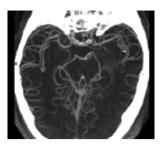


Figure 3: Thick MIP
After intra-arterial
mechanical
thrombectomy,
recanalization of the left
MCA was demonstrated
by FDCT

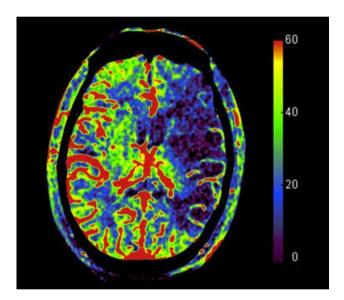


Figure 4: A parenchymal cerebral blood volume (*syngo* Neuro PBV IR) map depicted a large area of hypoperfusion in the corresponding left MCA territory

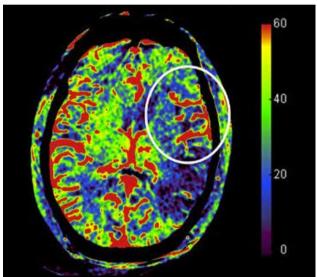


Figure 5: A *syngo* Neuro PBV IR map depicted the recovery of CBV values (circle) in part of the hypoperfused parenchyma after revascularization

Siemens Healthineers Headquarters

Siemens Healthcare GmbH Henkestr. 127

91052 Erlangen, Germany Phone: +49 9131 84-0 siemens-healthineers.com The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.

All rights reserved.