



Have you wondered why breasts have to be compressed during a mammogram?

Compression is key to getting a good mammogram. It prevents movement that can lead to blurred images. Compression spreads out the normal dense tissue of your breast, making it easier for radiologists to see through the tissue and detect abnormalities.

Compression is important, but it doesn't have to be painful anymore.

www.facilitywebsite.com

CONTACT US

FACILITY LOGO



+123-456-7890



hello@facilitywebsite.com



www.facilitywebsite.com



123 Anywhere St., Any City



NOT ALL MAMMOGRAMS ARE ALIKE

Introducing the
mammogram that's
easier on you

Comfort is just the beginning

Every woman is different so we wanted a mammography system as unique as our patients.

We chose the Siemens Healthineers MAMMOMAT Revelation mammography system because it makes comfort personal. The MAMMOMAT Revelation automatically compresses only as long as your breast is soft and pliable. This means exactly the right amount of compression is used for your breast.

The system's soft-edges and shape fit your breast's curves so comfort is there right from the start.

While you remain comfortable, your doctor is getting high-quality 3D images thanks to the system's unique wide-angle technology. This makes it easier to find small areas of concern with greater accuracy.

Frequently Asked Questions

Will the exam with the MAMMOMAT Revelation be different?

It should feel more comfortable! But other than that, it is the same. Your technologist will help position you at the system and explain what you need to do.

I have dense breasts. Is MAMMOMAT Revelation right for me?

Yes! MAMMOMAT Revelation is right for every type of breast tissue. It is also the first system to offer automated breast density measurement during the mammogram to maximize your personalized care.

Is MAMMOMAT Revelation a 3D mammogram?

MAMMOMAT Revelation is actually the next generation of 3D mammography because it features a wide-angle, high-definition technology that captures more detail. This helps uncover abnormalities that might otherwise be hidden by breast tissue.

How do I prepare for a MAMMOMAT Revelation exam?

You don't have to do anything differently. On the day of your mammogram you should not wear any deodorant on your underarms or lotion on your torso because these can interfere with the quality of your images. Remember to schedule your exam when your breasts are the least likely to be tender.

