


## Study Protocol

# Transarterial chemo-embolization of HCC using *syngo* DynaPBV Body

Interventional Oncology

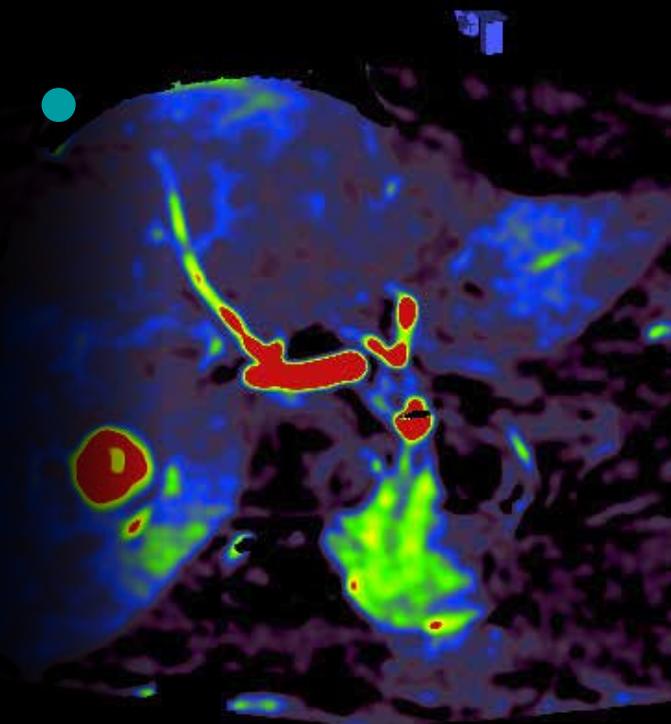

The post-embolization *syngo* DynaPBV acquisition confirmed complete treatment by showing no contrast opacification within the tumor. The one-month follow-up four-phase CT liver imaging confirmed results indicated by *syngo* DynaPBV Body.

---

### Courtesy of

Jeff McCann, MD,  
Ronan Ryan, MD,  
Department of Interventional  
Radiology,  
St. Vincent University Hospital,  
Dublin, Ireland

### Supported by

*syngo* DynaPBV Body

### System & Software

Artis Q ceiling VD10  
*syngo* X Workplace VC10

---

## Case Description

### Patient history

51-year-old male with Hep C. Hepatocellular BCLC stage A carcinoma (HCC), which is not amenable to radiofrequency (RF) ablation due to its proximity to the gall bladder. Patient is awaiting liver transplant and recommended for TACE treatment.

### Diagnosis

Pre-procedural four-phase CT of the liver measured a maximum of 3.5 cm in length of the segment V HCC lesion. The lesion demonstrated arterial hyperenhancement with portal venous and delayed phase washout centrally consistent with a HCC.

### Treatment

Selective chemoembolization of the 3.5 cm segment V HCC was performed on the patient. Chemoembolization was performed using 150 mg Doxorubicin adsorbed upon two vials of 100-300  $\mu$ m DC Beads,

followed by bland embolization using approximately 30% of one vial of 500-700  $\mu$ m Embosphere microspheres. The patient tolerated the procedure well and there were no immediate complications.

### General comments

The pre-embolization *syngo* DynaPBV Body showed that the hypervascular tumor in the inferior right lobe derived supply from the segment V artery. Chemoembolization followed by bland embolization of the arterial supply was satisfactory with excellent angiographic response. The post-embolization PBV run confirmed complete treatment by showing no contrast opacification within the tumor.

### Tips and tricks

Don't oversedate the patient as patient cooperation with breathing is very important. Arms should be put above the head during *syngo* DynaPBV Body acquisition.

# Transarterial chemoembolization of HCC using *syngo* DynaPBV Body

Acquisition protocol	5s DynaPBV Body (automatic)
----------------------	-----------------------------

## Injection protocol

Catheter position	Proper hepatic
Contrast medium (CM)	340 mg iodine/mL
Dilution (CM/Saline):	33 %
Injection volume	36 mL
Injection rate	3 mL/s
Duration of injection	12 s
X-ray delay	Manual CM injection started when C-arm finished mask run 7 s acquisition delay as C-arm returns for fill run
Power injector used	Yes

Reconstructions	Primary	Secondary
Name	DynaPBV Body Dual PBV	Reconstruct the Nat Fill run
VOI size	Large	Large
Slice matrix	512 × 512	512 × 512
Kernel type	HU	HU
Image characteristics	Smooth	Smooth
Reconstruction mode	Dual (Sub and Mask)	NatFill
Viewing preset	PBV Body	DynaCT Body

# Clinical Images



Figure 1: Secondary reconstruction of the fill run of the pre-procedural syngo DynaPBV Body run gives good visualization of vessel tree (esp. showing the looped tumor-feeding vessel)

## Clinical Images

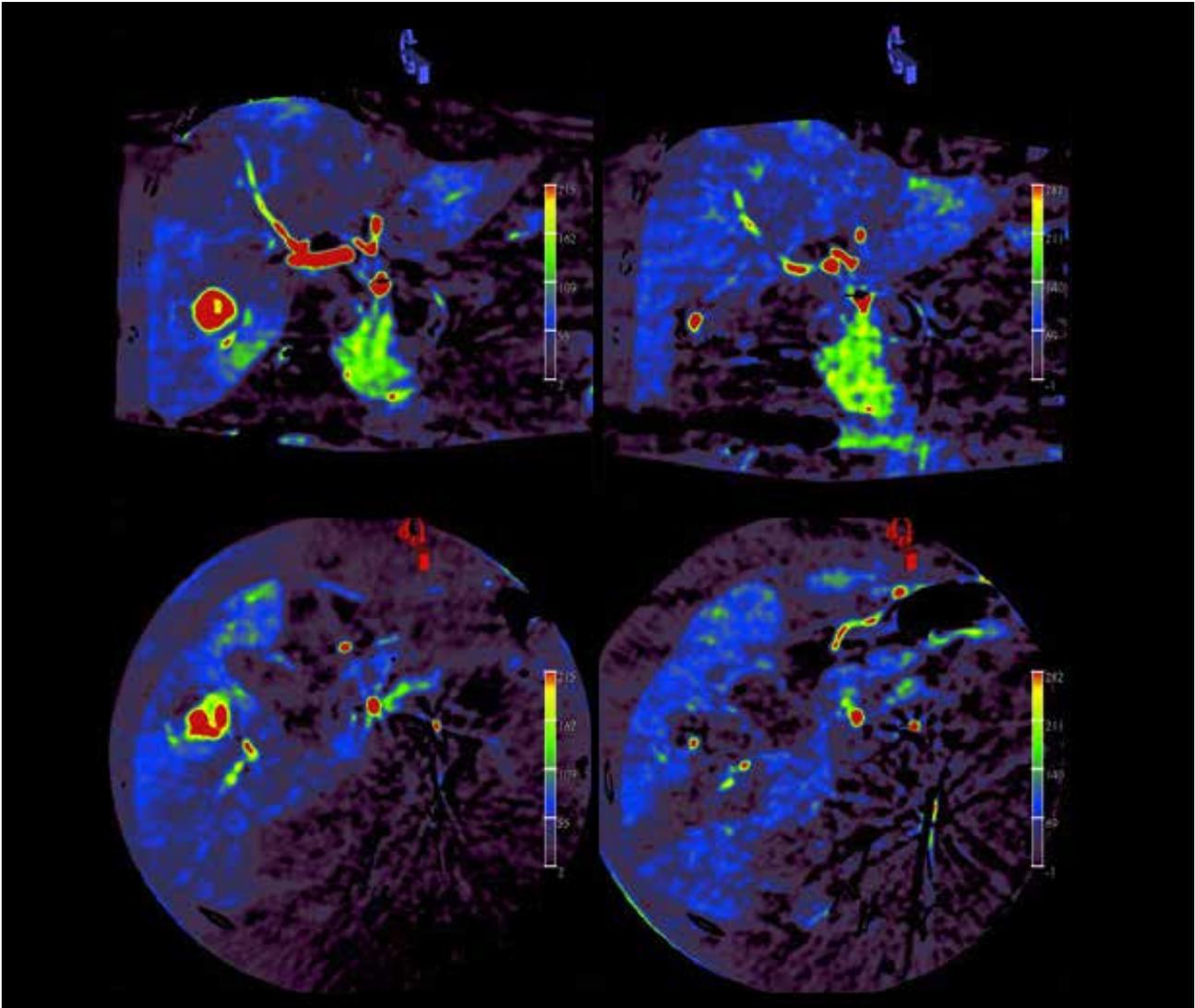


Figure 2: Pre- and post-embolization syngo DynaPBV imaging to confirm treatment success

---

### Siemens Healthineers Headquarters

Siemens Healthcare GmbH  
Henkestr. 127  
91052 Erlangen, Germany  
Phone: +49 9131 84-0  
siemens-healthineers.com

*The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.*

*On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.*

*All rights reserved.*