



## Study Protocol

# Follow-up of aneurysm clipping using *syngo* DynaCT with IV injection

Neuro Interventions

The *syngo* Dyna4D clearly showed how slow the inflow of blood into the aneurysmal structure had become after flow diverter placement, confirming the expected effect.

---

### Courtesy of

Prof. Peter Mitchell, MD,  
Department of Radiology,  
The Royal Melbourne Hospital,  
Melbourne, Australia

### Supported by

*syngo* DynaCT DSA  
*syngo* DualVolume

### System & Software

Artis Q biplane VD11,  
*syngo* X Workplace with  
*syngo* Application Software  
VD11

---

## Case Description

### Patient history

Past history of anterior communicating artery clipping (13 years ago), with subsequent elective middle cerebral artery embolization. Small bleb-like aneurysm adjacent to tip of aneurysm clips on DSA.

### Diagnosis

Intracranial aneurysms at both the anterior communicating and middle cerebral arteries. Small bleb-like aneurysm for assessment.

### Treatment

Clip occlusion > 10 years ago, routine follow-up showed small new bleblike aneurysm on DSA. Difficult to identify on MRA. *syngo* DynaCT with intravenous contrast injection in DualVolume mode offered less invasive imaging, able to exclude growth into a true saccular aneurysm, and allowed ongoing surveillance without repeat DSA.

### Tips & Tricks

- Once the injector is armed, set the 3D protocol to manual mode (injection starts automatically after returning from mask spin)
- Start acquiring the fill run when the carotid artery is displayed on the bolus watch (approx. 8–15 sec)
- Head positioning: symmetrical patient head positioning with strap fixing to reduce patient movement during both spins

### General comments

When alternative non-invasive arterial imaging is limited, due to dense coil and stent packing, or certain types and numbers of clips, *syngo* DynaCT with intravenous contrast injection in DualVolume mode offers a technique that is less impacted by these artifacts, particularly when the clinical area of suspicion is at the base of the previously treated aneurysm.

# Follow-up of aneurysm clipping using *syngo* DynaCT with IV injection

Acquisition protocol	10s DSA DCT Head	
Injection protocol		
Catheter position	IV injection antecubital vein 18 g cannula	
Contrast medium (CM)	350 mg iodine/mL	
Test bolus	No	
Dilution	No	
Injection volume	96 mL	
Injection rate	8 mL/s	
Duration of injection	12 s	
X-ray delay	DSA bolus tracking	
Power injector used	Yes	
Reconstructions	Primary	Secondary
Name	DynaCT DSA Dual Head (reconstructs nat fill, nat mask + sub volume)	DynaCT DSA Dual Head
VOI size	Full (nat fill volume)	Medium (nat mask + sub volume)
Slice matrix	512 × 512	512 × 512
Kernel type	HU (nat fill + nat mask volume)	EE (sub volume)
Image characteristics	Auto	Auto
Reconstruction mode	Vasc head	Vasc head
Viewing preset	Dual Volume 2	Dual Volume 2

## Clinical Images

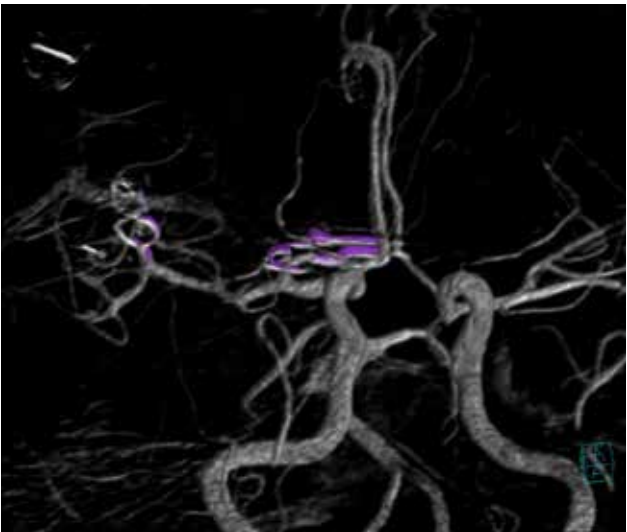


Figure 1: *syngo* DualVolume visualization of vessels and clips

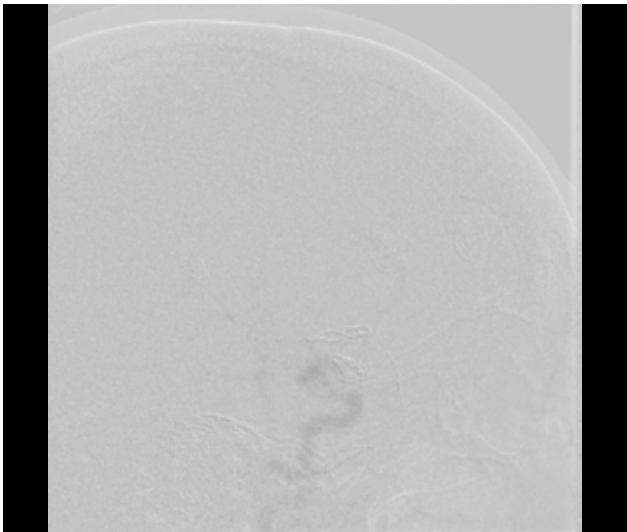


Figure 2: Bolus watching phase for second spin

## Clinical Images

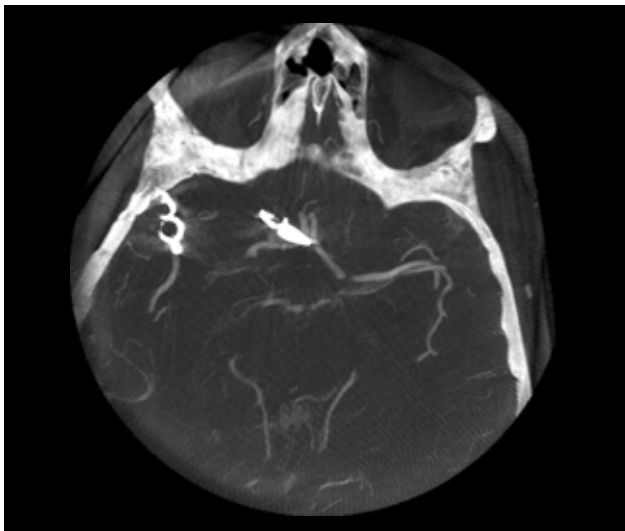


Figure 3: Transversal MIP 5 mm

---

### Siemens Healthineers Headquarters

Siemens Healthcare GmbH  
Henkestr. 127  
91052 Erlangen, Germany  
Phone: +49 9131 84-0  
[siemens-healthineers.com](http://siemens-healthineers.com)

*The statements by Siemens' customers presented here are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results.*

*On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this case are available throughout the Siemens sales organization worldwide.*

*All rights reserved.*