


**N Latex FLC kappa and lambda assays**

# Improving diagnostic accuracy in patients with monoclonal gammopathies

One reference range for simplified result interpretation and confident patient management

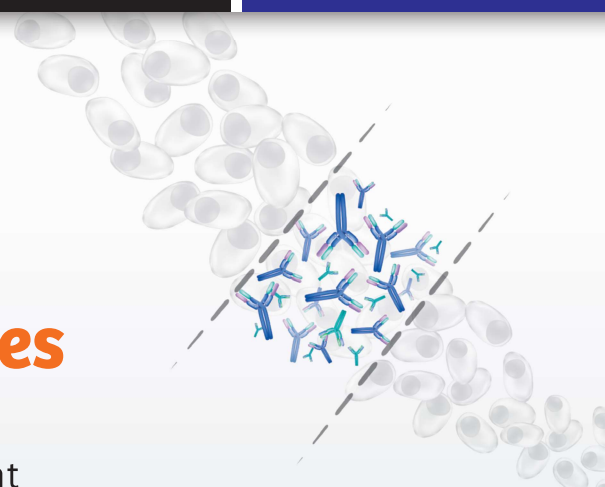

**Unlocking insight into monoclonal diseases**

The International Myeloma Working Group (IMWG) and other international societies recommend serum free light chains (sFLC) testing on nephelometry as an **indicator and differentiator** for the different stages of monoclonal gammopathies.<sup>1-5</sup>

N Latex FLC kappa and lambda assays enable **more reliable management** of patients with paraproteinemias, including evaluation of monoclonal gammopathies of undetermined significance (MGUS).

**Clinical utility**

Free light chain assay measurements may be used as an aid in the diagnosis, assessment, and monitoring of monoclonal diseases, including:

- Multiple myeloma
- Waldenström’s macroglobulinemia
- AL amyloidosis
- Light chain deposition disease
- Lymphocytic neoplasm

**Healthcare impact**

**Achieve consistent results**

Monoclonal antibodies support **consistency of results** across reagent lots for unbiased monitoring of disease activity.

**Simplify interpretation**

One **easy-to-interpret reference range** for all patients—including those with impaired renal function—reduces the risk of patient misclassification.

**Confidently identify patients**


Nephelometry enables precise measurements at both very high and very low concentrations for **accurate FLC ratio determinations**.

**Broad portfolio of nephelometric assays for consistent results and confident management of monoclonal diseases**

**Nephelometric assay portfolio for gammopathies**

β2-Microglobulin	IgA	▶ <b>FLC kappa</b>
α1-microglobulin	IgG	▶ <b>FLC lambda</b>
α2-microglobulin	IgM	Ig/Light Chain, type kappa
IgG subclasses 1–4		Ig/Light Chain, type lambda

**One reference range across all patient populations<sup>6</sup>**

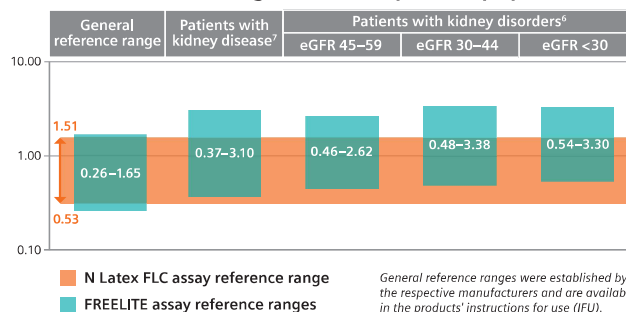

## Sensitivity compared to immunofixation†

Assay	IFE κ Positive (60)	IFE λ Positive (59)
N Latex FLC kappa	60 (100%)	–
FREELITE kappa	59 (98.3%)	–
N Latex FLC lambda	–	58 (98.3%)
FREELITE lambda	–	56 (94.9%)

## Assay highlights

	N Latex FLC kappa	N Latex FLC lambda
Assay principle	Nephelometric quantification on BN II and BN ProSpec Systems	
Sample types	Serum, heparin/EDTA plasma	
Sample volume	50 µL	50 µL
Initial dilution	1:100	1:20
Analytical measuring range	3.4–110 mg/L	1.9–60 mg/L
Time to first result	12 min.	13 min.

## Ordering information

Catalog No.	Product	No. of Tests
OPJA	N Latex FLC kappa	3 x 37 tests
OPJB	N Latex FLC lambda	3 x 37 tests
OPJC	N FLC Supplement Reagent	3 x 0.5 mL Supp A 3 x 2 mL Supp B
OPJD	N FLC Standard SL	3 x 1 mL
OPJE	N FLC Control SL1	3 x 1 mL
OPJF	N FLC Control SL2	3 x 1 mL

### References:

- Rajkumar SV, et al. International Myeloma Working Group updated criteria for the diagnosis of multiple myeloma. *Lancet Oncol.* 2014;15(12):e538-48.
- Moreau P, et al. Multiple myeloma: ESMO Clinical Practice Guidelines for diagnosis, treatment and follow-up. *Ann Oncol.* 2017;28(suppl 4):iv52-iv61.
- Mateos MV, et al. International Myeloma Working Group risk stratification model for smoldering multiple myeloma (SMM). *Blood Cancer J.* 2020 Oct 16;10(10):102.
- Kumar SK, et al. Multiple myeloma, version 3.2017, NCCN Clinical Practice Guidelines in Oncology. *J Natl Compr Canc Netw.* 2017;15(2):230-69.
- Bird JM, et al. Haemato-oncology Task Force of British Committee for Standards in Haematology (BCSH) and UK Myeloma Forum. Guidelines for the diagnosis and management of multiple myeloma 2011. *Br J Haematol.* 2011;154(1):32-75.
- Long TE, et al. Defining new reference intervals for serum free light chains in individuals with chronic kidney disease: results of the iStopMM study. *Blood Cancer J.* 2022 Sep 14;12(9):133.
- Hutchinson CA, et al. Quantitative Assessment of serum and urinary polyclonal free light chains in patients with chronic kidney disease. *Clin J Am Soc Nephrol.* 2008 Nov;3(6): 1684-90.

All trademarks are the property of their respective owners. Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.


### Siemens Healthineers Headquarters

Siemens Healthineers AG  
Siemensstr. 3  
91301 Forchheim, Germany  
Phone: +49 9191 18-0  
siemens-healthineers.com

### Published by

Siemens Healthcare Diagnostics  
Products GmbH  
Specialty Lab Solutions  
Emil-von-Behring-Strasse 76  
35041 Marburg, Germany

## Free light chain measurements (low-end concentration)

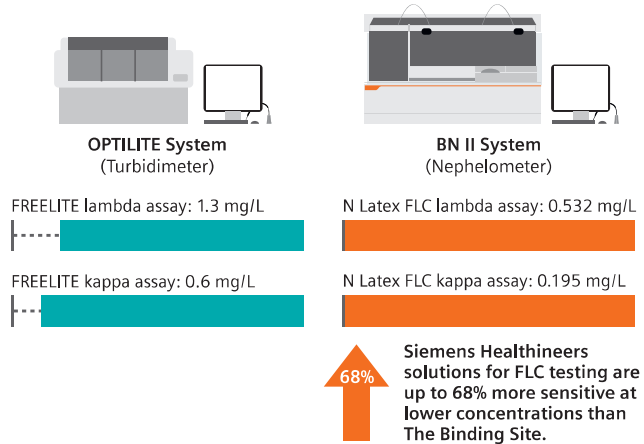

Limit of quantitation using the FREELITE assays on a turbidimeter (OPTILITE system from The Binding Site) and N Latex FLC assays on a nephelometer (BN II System from Siemens Healthineers).

Source: *Manufacturers' IFU.*

Visit [siemens-healthineers.com/flc](https://www.siemens-healthineers.com/flc) to learn more about N Latex FLC assays—the first assays to be FDA cleared for evaluation of MGUS.

†Data obtained during validation at Jeroen Bosch Hospital, Den Bosch, NL