Simple-to-follow procedure for trained professionals

CLINITEST® Rapid COVID-19 Antigen Test

siemens-healthineers.com



Do not use test kit after the expiration date.

Used materials should be discarded as biohazardous waste in accordance with local regulations.

Bring specimen and test components to room temperature if needed (15–30°C), and use the appropriate personal protective equipment.

Specimen Collection

Nasopharyngeal OR Nasal Swab



Nasopharyngeal Swab

Carefully insert the sterile swab into patient's nostril, gently rotate the swab several times over the surface of the posterior nasopharynx, and gently remove.




Nasal Swab

Carefully insert the sterile swab into patient's nostril (2–4 cm), roll swab 5 times along the mucosa to collect mucous and cells, and gently remove.

Repeat for other nostril.

Sample Preparation

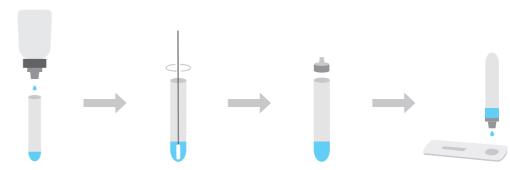

Add 10 drops (~300 µL) of sample extraction buffer into extraction tube, keeping tube upright using stand provided. Roll swab 6+ times inside tube, squeeze against bottom and sides of tube, let stand for 1 minute, and squeeze several more times. Safely remove and discard swab, and insert dropper tip into tube with specimen. Lay cassette flat, label, and add four drops (~100 μL) of test sample into sample well. Results will be ready in 15 minutes. Do not interpret the result after 20 minutes.

Results Interpretation





At Siemens Healthineers, our purpose is to enable healthcare providers to increase value by empowering them on their journey toward expanding precision medicine, transforming care delivery, and improving patient experience, all made possible by digitalizing healthcare.

An estimated 5 million patients globally benefit every day from our innovative technologies and services in the areas of diagnostic and therapeutic imaging, laboratory diagnostics, and molecular medicine, as well as digital health and enterprise services.

We are a leading medical technology company with over 120 years of experience and 18,000 patents globally. Through the dedication of more than 50,000 colleagues in 75 countries, we will continue to innovate and shape the future of healthcare.

CLINITEST is a trademark of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product is under FDA review for Emergency Use Authorization (EUA) and not available for sale in the U.S. Product availability may vary by country. Distributed by Siemens Healthineers.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH Henkestr. 127 91052 Erlangen, Germany Phone: +49 913184-0 siemens-healthineers.com Published by

Siemens Healthcare Diagnostics Inc. Point of Care Diagnostics 2 Edgewater Drive Norwood, MA 02062-4637 USA

Phone: +1 781-269-3000