

**DICOM Conformance Statement**

# **FLUOROSPOT®**

## **Compact VI11**

Multi-platform imaging system used by:

Luminos dRF Max VF11  
Luminos Agile Max VF11  
LUMINOS Lotus Max VF11  
Uroskop Omnia Max VF11  
Multitom Rax VF11  
LUMINOS Impulse VF11

[siemens-healthineers.com](http://siemens-healthineers.com)



<b>Network Conformance Statement .....</b>	<b>6</b>
<b>1      Introduction .....</b>	<b>6</b>
1.1      Overview.....	6
1.2      Scope and Field .....	6
1.3      Audience .....	6
1.4      Remarks.....	6
1.5      Definitions, Terms and Abbreviations.....	8
1.6      References .....	8
1.7      Structure.....	8
<b>2      Implementation Model Verification .....</b>	<b>9</b>
2.1      Application Data Flow Diagram.....	9
2.2      Functional Definitions of Applications .....	9
2.3      Sequencing of Real-World Activities .....	9
<b>3      Application Entity Specification Verification.....</b>	<b>10</b>
3.1      Verification AE Specification .....	10
3.1.1      Association Establishment Policies.....	10
3.1.1.1      General.....	10
3.1.1.2      Number of Associations.....	10
3.1.1.3      Asynchronous Nature .....	10
3.1.1.4      Implementation Identifying Information.....	10
3.1.2      Association Initiation Policy.....	10
3.1.2.1      Associated Real-World Activity.....	10
3.1.3      Association Acceptance Policy.....	11
<b>4      Implementation Model Storage .....</b>	<b>12</b>
4.1      Application Data Flow Diagram.....	12
4.2      Functional Definitions of Application Entities.....	12
4.3      Sequencing of Real-World Activities .....	13
<b>5      Application Entity Specification Storage.....</b>	<b>14</b>
5.1      Storage AES Specification .....	14
5.1.1      Association Establishment Policies.....	15
5.1.1.1      General.....	15
5.1.1.2      Number of Associations.....	15
5.1.1.3      Asynchronous Nature .....	15
5.1.1.4      Implementation Identifying Information.....	15
5.1.2      Association Initiation Policy.....	15
5.1.2.1      Associated Real-World Activity – Storage SCU .....	15
5.1.3      Association Acceptance Policy.....	18
5.1.3.1      Associated Real-World Activity – Storage SCP.....	18
<b>6      Implementation Model Storage Commitment .....</b>	<b>22</b>
6.1      Application Data Flow Diagram.....	22
6.2      Functional Definitions of Application Entities.....	22
6.3      Sequencing of Real-World Activities .....	22
<b>7      AE Specification Storage Commitment .....</b>	<b>23</b>
7.1      Storage Commitment AE Specification .....	23
7.1.1      Association Establishment Policies.....	23
7.1.1.1      General.....	23
7.1.1.2      Number of Associations.....	23
7.1.1.3      Asynchronous Nature .....	23
7.1.1.4      Implementation Identifying Information.....	23
7.1.2      Association Initiation Policy.....	23
7.1.2.1      Real World Activity – Storage Commitment SCU .....	24
7.1.3      Association Acceptance Policy.....	24
7.1.3.1      Real World Activity – Storage Commitment SCU .....	25

<b>8</b>	<b>Implementation Model Query / Retrieve .....</b>	26
8.1	Application Data Flow Diagram.....	26
8.2	Functional Definitions of Application Entities .....	26
8.3	Sequencing of Real-World Activities.....	27
<b>9</b>	<b>AE Specification Query/Retrieve .....</b>	28
9.1	Query/Retrieve Service Aes Specification.....	28
9.1.1	Association Establishment Policies.....	28
9.1.1.1	General .....	28
9.1.1.2	Number of Associations.....	28
9.1.1.3	Asynchronous Nature .....	28
9.1.1.4	Implementation Identifying Information.....	28
9.1.2	Association Initiation Policy.....	28
9.1.2.1	Real World Activity – Find SCU .....	29
9.1.2.2	Real-World Activity – Move SCU.....	31
9.1.3	Association Acceptance Policy.....	33
<b>10</b>	<b>Implementation Model Modality Worklist .....</b>	34
10.1	Application Data Flow Diagram.....	34
10.2	Functional Definitions of Application Entities .....	34
10.3	Sequencing of Real-World Activities.....	34
<b>11</b>	<b>AE Specification Modality Worklist.....</b>	35
11.1	Modality Worklist Service Aes Specification.....	35
11.1.1	Association Establishment Policies.....	35
11.1.1.1	General .....	35
11.1.1.2	Number of Associations.....	35
11.1.1.3	Asynchronous Nature .....	35
11.1.1.4	Implementation Identifying Information.....	35
11.1.2	Association Initiation Policy.....	35
11.1.2.1	Real-World Activity.....	36
11.1.3	Association Acceptance Policy.....	37
<b>12</b>	<b>Implementation Model Modality Performed Procedure Step .....</b>	38
12.1	Application Data Flow Diagram.....	38
12.2	Functional Definitions of Application Entities .....	38
12.3	Sequencing of Real-World Activities.....	38
<b>13</b>	<b>AE Specification Modality Performed Procedure Step .....</b>	39
13.1	Modality Performed Procedure Step Service Aes Specification.....	39
13.1.1	Association Establishment Policies.....	39
13.1.1.1	General .....	39
13.1.1.2	Number of Associations.....	39
13.1.1.3	Asynchronous Nature .....	39
13.1.1.4	Implementation Identifying Information.....	39
13.1.2	Association Initiation Policy.....	40
13.1.2.1	Real-World Activity.....	40
13.1.3	Association Acceptance Policy .....	41
<b>14</b>	<b>Implementation Model Print.....</b>	42
14.1	Application Data Flow Diagram.....	42
14.2	Functional Definition of Application Entities .....	42
14.3	Sequencing of Real-World Activities.....	42
<b>15</b>	<b>Application Entity Specification Print .....</b>	43
15.1	Print Management AE Specification.....	43
15.1.1	Association Establishment Policies.....	43
15.1.1.1	General .....	43
15.1.1.2	Number of Associations.....	43
15.1.1.3	Asynchronous Nature .....	43
15.1.1.4	Implementation Identifying Information.....	43
15.1.2	Association Initiation Policy.....	43

15.1.2.1	Associated Real-World Activity.....	44
15.1.3	Association Acceptance Policy.....	48
<b>16</b>	<b>Communication Profiles.....</b>	<b>49</b>
16.1	Supported Communication Stacks.....	49
16.1.1	TCP/IP Stack.....	49
16.1.1.1	API.....	49
16.1.1.2	Physical Media Support.....	49
<b>17</b>	<b>Extensions / Specializations / Privatizations.....</b>	<b>50</b>
17.1.1	Standard Extended / Specialized / Private SOPs.....	50
17.1.2	Private Transfer Syntaxes.....	50
<b>18</b>	<b>Configuration.....</b>	<b>51</b>
18.1	AE Title/Presentation Address Mapping.....	51
18.1.1	DICOM Verification.....	51
18.1.2	DICOM Storage SCP AE Title.....	51
18.1.3	DICOM Storage SCU AE Title.....	51
18.1.4	DICOM Query/Retrieve AE Title.....	51
18.1.5	DICOM Storage Commitment AE Title.....	51
18.1.6	DICOM Worklist AE Title.....	52
18.1.7	DICOM MPPS AE Title.....	52
18.1.8	DICOM Print AE Title.....	52
18.2	Configurable Parameters.....	52
18.2.1	Storage and Query/Retrieve.....	52
18.2.2	Storage Commitment.....	53
18.2.3	Modality Worklist.....	53
18.2.4	Modality Performed Procedure Step.....	53
18.2.5	Print.....	53
18.3	Default Parameters.....	53
<b>19</b>	<b>Support of Extended Character Sets.....</b>	<b>55</b>
<b>20</b>	<b>Security.....</b>	<b>56</b>
20.1	Security Profiles.....	56
20.1.1	Secure Transport Connection Profiles.....	56
<b>1</b>	<b>Application Profile Conformance Statement .....</b>	<b>57</b>
<b>1.1</b>	<b>Introduction .....</b>	<b>58</b>
1.1.1	Purpose .....	58
1.1.2	Scope.....	58
1.1.3	Definitions, Terms and Abbreviations.....	58
1.1.4	References .....	58
1.1.5	Remarks.....	58
<b>2</b>	<b>Implementation Model .....</b>	<b>60</b>
2.1	Application Data Flow Diagram.....	60
2.2	Functional Definitions of Aes .....	60
2.3	Sequencing of Real-World Activities .....	60
2.4	File Meta Information Options .....	61
<b>3</b>	<b>AE Specifications .....</b>	<b>62</b>
3.1	DICOM Archive Specification.....	62
3.1.1	File Meta Information for the Application Entity .....	62
3.1.2	Real-World Activities for this Application Entity .....	62
3.1.2.1	Real-World Activity: Browse Directory Information.....	62
3.1.2.2	Real-World Activity: Import into local Storage .....	62
3.1.2.3	Real-World Activity: Export to local Archive Media .....	64
<b>4</b>	<b>Augmented and Private Profiles.....</b>	<b>65</b>
4.1	Augmented Application Profiles.....	65
4.2	Private Application Profiles .....	65
<b>5</b>	<b>Extensions, Specializations and Privatizations of SOP Classes and Transfer Syntaxes.....</b>	<b>66</b>

5.1	SOP Specific Conformance Statement for Basic Directory.....	67
5.1.1	Extension, Specialization for "Packed CD" .....	67
5.1.2	DICOMDIR Attributes.....	67
<b>6</b>	<b>Configuration.....</b>	<b>70</b>
6.1	AE Title Mapping.....	70
6.1.1	DICOM Media Storage AE Title.....	70
<b>7</b>	<b>Support of Extended Character Sets.....</b>	<b>71</b>
<b>A</b>	<b>APPENDIX .....</b>	<b>72</b>
A.1	Common attributes in image and structured report IODs.....	72
A.2	Multiframe True Color SC Storage Standard Extended SOP Class.....	75
A.3	XRF Standard Extended SOP Class.....	76
A.3.1	configuration "XRF – Full" .....	76
A.3.2	configuration "XRF – Ready Processed".....	79
A.4	CR,DX Standard Extended SOP Class.....	82
A.5	X-Ray Radiation Dose SR SOP Class.....	85
A.5.1	Projection X-Ray Radiation Dose (TID 10001) .....	86
A.5.2	Accumulated X-Ray Dose (TID 10002).....	87
A.5.3	Irradiation Event X-Ray Data (TID 10003).....	88
A.6	EXAM Protocol as Secondary Capture Image.....	90
A.7	XA Standard Extended SOP Class.....	91
A.8	SOP Common Module – Image Type Extensions.....	92
A.9	Private Data .....	92
A.9.1	General Private Data.....	92
A.10	FLUOROSPOT Compact Worklist Identifier.....	97
A.11	Modality Performed Procedure Step Identifiers.....	100
A.12	Grayscale Image Consistency .....	103

# Network Conformance Statement

## 1 Introduction

### 1.1 Overview

This Conformance Statement describes the DICOM network interface for Siemens Healthineers X-Ray acquisition modalities based on the FLUOROSPOT Compact VI11 imaging system according to part PS 3.2 of [DICOM].

This introduction describes the application's implemented DICOM functionality in general terms.

### 1.2 Scope and Field

The Siemens Healthineers product FLUOROSPOT Compact is a multifunctional imaging system for fluoroscopic, urologic and radiographic procedures. This DICOM Conformance Statement is valid for the following products listed on the front page.

The supported DICOM Services allow a smooth integration into the clinical network. Depending on system type images can be stored utilizing the DICOM XRF, XA, CR or DX IOD. The DICOM service 'Storage Commitment (Push Model) Service Class' indicates established image storage in the DICOM image archive. Images, acquired at the FLUOROSPOT Compact, can be retrieved from another network node by using the DICOM service 'DICOM Query/Retrieve'. With 'DICOM Basic Worklist Management Service Class' the worklist containing patient data can be retrieved from the information system and afterwards examination data can be send back to the information by using 'Modality Performed Procedure Step Service Class'. FLUOROSPOT Compact supports the "Print Management Service Class" and the 'Media Storage Service Class' (General Purpose CDR Application Profile) and is able to encrypt the DICOM communication according to the 'Basic TLS Secure Transport Connection Profile'.

This DICOM Conformance Statement refers to FLUOROSPOT Compact based products only.

### 1.3 Audience

This document is intended for hospital staff, health system integrators, software designers or implementers. It is assumed that the reader has a working understanding of DICOM.

### 1.4 Remarks

DICOM, by itself, does not guarantee interoperability. However, the Conformance Statement facilitates a first-level validation for interoperability between different applications supporting the same DICOM functionality as SCU and SCP, respectively.

This Conformance Statement is not intended to replace validation with other DICOM equipment to ensure proper exchange of information intended.

The scope of this Conformance Statement is to facilitate communication with Siemens Healthineers and other vendors' Medical equipment. The Conformance Statement should be read and understood in conjunction with the DICOM 3.0 Standard [DICOM]. It is, however, by itself not guaranteed to ensure the desired interoperability and a successful interconnectivity.

The user should be aware of the following important issues:

- The comparison of different conformance statements is the first step towards assessing interconnectivity between Siemens Healthineers and non-Siemens Healthineers equipment.
- Test procedures should be defined and tests should be performed by the user to validate the connectivity desired. DICOM itself and the conformance parts do not specify this.
- The standard will evolve to meet the user's future requirements. Siemens Healthineers is actively involved in developing the standard further and therefore reserves the right to make changes to its products or to discontinue its delivery.
- Siemens Healthineers reserves the right to modify the design and specifications contained herein without prior notice. Please contact your local Siemens Healthineers representative for the most recent product information.

### 1.5 Definitions, Terms and Abbreviations

Definitions, terms and abbreviations used in this document are defined within the different parts of the DICOM standard.

Abbreviations and terms are as follows:

AE	DICOM Application Entity
AET	DICOM Application Entity Title
ASCII	American Standard Code for Information Interchange
CR	Computed Radiography
CSE	Customer Service Engineer
DCS	DICOM Conformance Statement
EXI	Exposure Index
FD	Flat panel detector
IOD	DICOM Information Object Definition
ISO	International Standard Organization
MPPS	Modality Performed Procedure Step
NEMA	National Electrical Manufacturers Association
NL	Nesting Level
R	Required Key Attribute
O	Optional Key Attribute
OPG	Organ Program
RIS	Radiology Information System
PDU	DICOM Protocol Data Unit
SC	Secondary Capture
SCU	DICOM Service Class User (DICOM client)
SCP	DICOM Service Class Provider (DICOM server)
SID	Source Image Receptor Distance
SOD	Source Object Distance SOP
	DICOM Service-Object Pair
U	Unique Key Attribute
UI	User Interface
UID	Unique Identifier
US	Ultrasound
XA	X-Ray Angiography
XRF	X-Ray Radiofluoroscopy

### 1.6 References

[DICOM] Digital Imaging and Communications in Medicine (DICOM), NEMA PS 3.1-3.20.

The DICOM Standard is under continuous maintenance, the current official version is available at <http://dicom.nema.org>.

### 1.7 Structure

The FLUOROSPOT Compact Conformance Statement is subdivided into three Parts which relate to individual documents needed to declare Conformance according to the requirements of "Part 2 - Conformance" of the DICOM Standard.

Those parts are:

- "DICOM Conformance Statement" for Network related Services
- "Offline Media Conformance Statement" to support local archive media.
- A general Appendix.

## 2 Implementation Model Verification

The FLUOROSPOT Compact DICOM application requests Verification to verify the ability of another DICOM application on a remote node to respond to DICOM messages.

Responding to Verification requests from remote nodes is handled by the Storage SCP application.

### 2.1 Application Data Flow Diagram

The FLUOROSPOT Compact DICOM application acts as SCU for the C-ECHO DICOM network service.

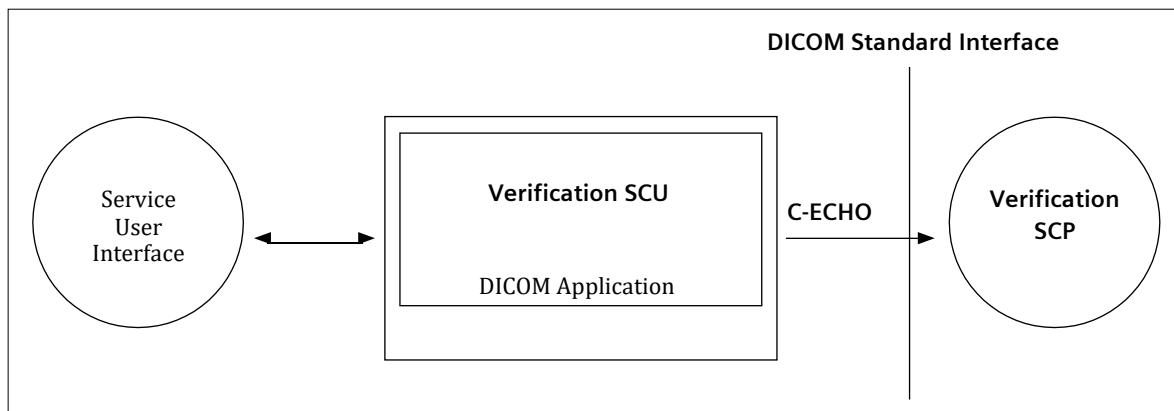


Figure 1: FLUOROSPOT Compact Application Flow Diagram – Verification SCU

### 2.2 Functional Definitions of Applications

The Siemens Healthineers FLUOROSPOT Compact DICOM application opens an association when a “C-ECHO” of a remote application is requested from the Service User Interface. This can be done to verify the correct setup of a remote destinations configuration data.

### 2.3 Sequencing of Real-World Activities

Not applicable

# 3 Application Entity Specification Verification

## 3.1 Verification AE Specification

- Association Establishment Policies

### 3.1..1 General

The Siemens Healthineers FLUOROSPOT Compact DICOM application attempts to open an association for verification request whenever the "C-Echo" function is activated.

### 3.1..2 Number of Associations

The Siemens Healthineers FLUOROSPOT Compact DICOM application initiates one association at a time to request verification.

### 3.1..3 Asynchronous Nature

The Siemens Healthineers FLUOROSPOT Compact DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 3.1..4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

- Association Initiation Policy

The Siemens Healthineers FLUOROSPOT Compact DICOM application attempts to initiate a new association for

- DIMSE C-ECHO service operations.

### 3.1..1 Associated Real-World Activity

#### 3.1..1.1 Associated Real-World Activity – C-Echo

The associated Real-World activity is a C-ECHO request initiated by Service User Interface environment whenever a verification of a destination configuration is requested. If an association to a remote Application Entity is successfully established, verification with the configured AET is requested via the open association. If the C-ECHO Response from the remote Application is received, a message will be displayed indicating that the C-Echo was successful, and the association is closed.

### 3.1..1.2 Proposed Presentation Contexts

The Siemens Healthineers FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table – Verification SCU**

Abstract Syntax		Transfer Syntax		Role	Ext.
Name	UID	Name List	UID List	Neg.	
Verification	1.2.840.10008.1.1	Implicit VR Little Endian Explicit VR Big Endian Explicit VR Little Endian	1.2.840.10008.1.2 1.2.840.10008.1.2.2 1.2.840.10008.1.2.1	SCU	None

### 3.1..1.3 SOP Specific Conformance – Verification SCU

None.

- **Association Acceptance Policy**

The Verification SCP is part of the Storage SCP – see section 5.1.3.

## 4 Implementation Model Storage

The Siemens Healthineers FLUOROSPOT Compact DICOM Application Entity originates associations for Storage of DICOM Composite Information Objects in Remote Application Entities and receives association requests for Storage from Remote Application Entities.

“Send to...” is performed on the user’s request for each study completed or for specific images selected.

“Send to...” can also be performed for images, which are deleted (copy to Recycle Bin) but have not been archived yet.

### 4.1 Application Data Flow Diagram

The FLUOROSPOT Compact DICOM application acts as SCU and SCP for the C-STORE DICOM network service and as SCP for the C-ECHO DICOM network service.

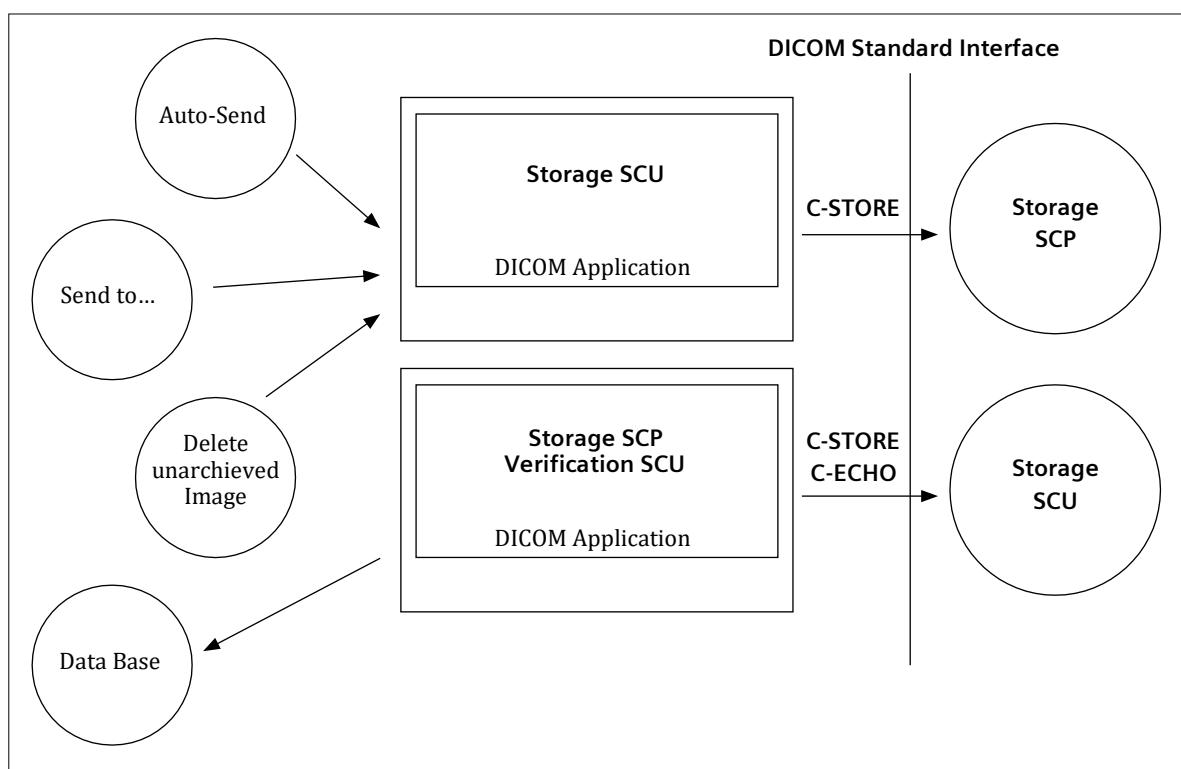


Figure 2: FLUOROSPOT Compact Application Data Flow Diagram – Storage SCU/SCP, Verification SCP

### 4.2 Functional Definitions of Application Entities

With “Send to...” the marked images can be sent automatically to multiple destinations (up to 3). Each request is forwarded to the job queue and processed individually. For each destination it can be configured during Service session, which IOD type (selection depending on system type) has to be used when sending images. For multiple destinations it is required, that all of the desired destination will support the same IOD type. If this is not the case, then only one target at a time can be addressed.

With “Send to..” initiated on user’s request in patient mode selected studies can be exported to a DICOM node chosen from a list of configured destinations.

With "Send to..." also the "Exam Protocol as Secondary Captured Image" and the DICOM Dose Structured Report can be send to defined network destinations.

With "Auto Send" the "Send To..." mechanism is invoked automatically for a set of predefined or marked images. The rules and destinations for "Auto Send" can be configured in the OGP editor and can be specified for each OGP.

With "Delete unarchived image" an image will be copied into the local Recycle Bin. Additionally for this image a "Send To..." could be invoked to a selected DICOM node.

The Storage SCU is invoked by the job control interface that is responsible for processing network archival tasks. The job consists of data describing the composite image objects marked for storage and the destination. An association is negotiated with the destination application entity and the image data is transferred using the C-STORE DIMSE-Service. Status of the transfer is reported through the job control interface. Only one job will be active at a time.

The Storage SCP component of the Siemens Healthineers FLUOROSPOT Compact DICOM application is operating as background server process. It is running when the system is triggered to import images and waits then for Storage association requests. Upon accepting an association with a negotiated Presentation Context it starts to receive the Composite Image Objects and imports them to local database.

Verification requests will be processed and responded by the Storage SCP.

#### 4.3 Sequencing of Real-World Activities

not applicable

# 5 Application Entity Specification Storage

## 5.1 Storage AES Specification

The FLUOROSPOT Compact Storage SCU/SCP applications use two different AETs when initiating/receiving associations to/from remote DICOM nodes.

The FLUOROSPOT Compact imaging system supports several Siemens Healthineers X-Ray acquisition modalities.

SOP Classes used for storage depend on the acquisition system. FLUOROSPOT Compact provides Standard Conformance to the following DICOM SOP Classes as SCU:

SOP Class Name	SOP Class UID
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7
X-Ray Radiation Dose SR	1.2.840.10008.5.1.4.1.1.88.67
X-Ray Angiographic Image Storage <sup>1</sup>	1.2.840.10008.5.1.4.1.1.12.1
X-Ray Radiofluoroscopic Image Storage <sup>2</sup>	1.2.840.10008.5.1.4.1.1.12.2
Computed Radiography Image Storage	1.2.840.10008.5.1.4.1.1.1
Digital X-Ray Image Storage – For Presentation <sup>3</sup>	1.2.840.10008.5.1.4.1.1.1
Multi-frame True Color Secondary Capture Image Storage <sup>4</sup>	1.2.840.10008.5.1.4.1.1.7.4

Siemens Healthineers FLUOROSPOT Compact imaging system provides Standard Conformance to the following DICOM SOP Classes as SCP:

SOP Class Name	SOP Class UID
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1
X-Ray Radiofluoroscopic Image Storage	1.2.840.10008.5.1.4.1.1.12.2
Computed Radiography Image Storage	1.2.840.10008.5.1.4.1.1.1
Digital X-Ray Image Storage – For Presentation	1.2.840.10008.5.1.4.1.1.1
Multi-frame True Color Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7.4
CT Image Storage	1.2.840.10008.5.1.4.1.1.2
MR Image Storage	1.2.840.10008.5.1.4.1.1.4
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1
Ultrasound Multiframe Image Storage	1.2.840.10008.5.1.4.1.1.3.1
Verification	1.2.840.10008.1.1

<sup>1</sup> Only for Multitom Rax

<sup>2</sup> Only for Fluorosystems

<sup>3</sup> Only for Radsystems

<sup>4</sup> Only for UROSKOP Omnia

## 5.1.1 Association Establishment Policies

### 5.1.1.1 General

The existence of a job queue entry with network destination will activate the DICOM Storage Application. An association request is sent to the destination AE and upon successful negotiation of a Presentation Context the transfer is started. If the association cannot be opened, the related job is set to an error state and can be restarted by user job control interface. It is configurable by the CSE if a retry for failed jobs is initiated automatically.

The default PDU size used will be 64 KB.

### 5.1.1.2 Number of Associations

The FLUOROSPOT Compact DICOM application initiates one association at a time, subsequently for each destination to which a transfer request is being processed in the active job queue list. Only one job will be active at a time, the others remain pending until the active job is completed or failed.

### 5.1.1.3 Asynchronous Nature

The FLUOROSPOT Compact software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 5.1.1.4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

## 5.1.2 Association Initiation Policy

If a job with network destination gets active in the job list the FLUOROSPOT Compact DICOM application attempts to initiate a new association for

- DIMSE C-STORE service operations.

### 5.1.2.1 Associated Real-World Activity – Storage SCU

#### 5.1.2.1.1 Associated Real-World Activity –Send to ...

The associated Real-World activity is a C-STORE request initiated by an internal daemon process triggered by a job with network destination. If the process successfully establishes an association to a remote Application Entity, it will transfer each marked image one after another via the open association. The used storage IOD is depending on the system type and on the destination's configuration. If the C-STORE Response from the remote Application contains a status other than Success or Warning, the association is aborted. The related job is switched to a failed state. It can be restarted any time by user interaction. An automatic retry can be configured by the CSE.

Processing of images of a send job and sending of processed images is done in parallel. That means, as soon as the first image of a job is processed an association is initiated and the sending of the processed image is started.

An established association is closed as soon as no processed image is available to be sent.

Due to different speeds of sending and processing the FLUOROSPOT Compact might initiate more than one association for each send job!

### 5.1.2.1.2 Proposed Presentation Context –Send to...

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax		Transfer Syntax	Role	Ext.
Name	UID	Name List	UID List	Neg.
Computed	1.2.840.10008.5.1.4.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	
Radiography		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Image Storage		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
Service Class <sup>1</sup>				
Digital X-Ray	1.2.840.10008.5.1.4.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	
Image Storage –		Explicit VR Big Endian	1.2.840.10008.1.2.2	
For Presentation <sup>5</sup>		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
Secondary Cap-	1.2.840.10008.5.1.4.1.1.7	Implicit VR Little Endian	1.2.840.10008.1.2	
ture Image		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Storage Service		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
Class				
X-Ray Radia-	1.2.840.10008.5.1.4.1.1.88.67	Implicit VR Little Endian	1.2.840.10008.1.2	
tionDose SR		Explicit VR Big Endian	1.2.840.10008.1.2.2	
		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
X-Ray Angio-	1.2.840.10008.5.1.4.1.1.12.1	Implicit VR Little Endian	1.2.840.10008.1.2	
graphic Image		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Storage Service		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
Class <sup>2</sup>				
X-Ray Radio-	1.2.840.10008.5.1.4.1.1.12.2	Implicit VR Little Endian	1.2.840.10008.1.2	
fluoroscopic		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Image Storage		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
Service Class <sup>3</sup>				
Multi-frame	1.2.840.10008.5.1.4.1.1.7.4	Implicit VR Little Endian	1.2.840.10008.1.2	
True Color		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Secondary		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None
Capture Image				
Storage <sup>4</sup>				

If multiple destinations are addressed within one network job, then all of those are required to support the same IOD.

<sup>1</sup>Only for radsystems

<sup>2</sup>Only for Multitom Rax

<sup>3</sup>Only for Fluorosystems

<sup>4</sup>Only for Uroskop Omnia

#### 5.1.2.1.3 SOP specific Conformance Statement –Send to...

The DICOM images created by the DICOM interface of the FLUOROSpot Compact conform to the DICOM CR, XRF, XA, DX or Multi-frame True Color Secondary Capture IOD definitions. Private elements are contained in the objects. The IODs will be a Standard Extended Image Storage SOP Classes.

The DICOM Dose Structured Report created by the DICOM interface of the FLUOROSpot Compact conforms to the X-Ray Radiation Dose SR IOD definitions. Private elements are contained in the objects.

Image Laterality (0020,0062) and Patient Orientation (0020,0020) might not be included in Images sent in DX IOD.

For the projection images that are used for 3D reconstruction Multitom Rax uses the X-Ray Angiographic Image Storage Service Class. The reconstruction is carried out on a different workstation. The resulting 3D images will be encoded as CT Image Storage SOP Class. Please refer to A.7 for the definition of the attributes that are specific for XA objects used for the projection images.

Imported DICOM images cannot be exported again.

#### 5.1.2.1.3.1 Optional Attributes

A list of all DICOM IOD attributes of type 2 and 3 which are encoded by the FLUOROSpot Compact applications can be found in the APPENDIX (page 75 ff).

#### 5.1.2.1.3.2 Image Pixel Attribute Description for Grayscale Images

The Siemens Healthineers FLUOROSpot Compact DICOM application supports the "MONOCHROME 2" Photometric Interpretation with the unsigned integer 16-bit (8 bit for downsized images) gray scale pixel format:

### 5.1.3 Association Acceptance Policy

The FLUOROSPOT Compact DICOM application attempts to accept a new association for

- DIMSE C-ECHO
- DIMSE C-STORE

service operations. Any Information Objects transmitted on that association will be checked on conformance and stored in database if check was successful.

#### 5.1.3.1 Associated Real-World Activity – Storage SCP

##### 5.1.3.1.1 Associated Real-World Activity – Receiving Images from a Remote Node (with Retrieve)

Receiving is possible whenever an import request (C-MOVE) is sent to a remote destination. As long as it is active, the receiver process will accept an association and will receive images transmitted on that association and will store the images on disk in the own data base if the conformance check is performed successfully.

The FLUOROSPOT Compact DICOM application is designed for retrieving images that were acquired by the system itself.

The FLUOROSPOT Compact DICOM application will accept Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax		Transfer Syntax	Role	Ext.
Name	UID	Name List	UID List	Neg.
Computed	1.2.840.10008.5.1.4.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	
Radiography		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Image Storage		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Service Class				
Digital X-Ray	1.2.840.10008.5.1.4.1.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	
Image Storage -		Explicit VR Big Endian	1.2.840.10008.1.2.2	
For Presentation		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Secondary Cap-	1.2.840.10008.5.1.4.1.1.7	Implicit VR Little Endian	1.2.840.10008.1.2	
ture Image Stor-		Explicit VR Big Endian	1.2.840.10008.1.2.2	
age Service Class		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
X-Ray Radio-	1.2.840.10008.5.1.4.1.1.12.2	Implicit VR Little Endian	1.2.840.10008.1.2	
fluoroscopic		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Image Storage		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Service Class				
X-Ray Angio-	1.2.840.10008.5.1.4.1.1.12.1	Implicit VR Little Endian	1.2.840.10008.1.2	
graphic Image		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Storage Service		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Class				
Multi-frame	1.2.840.10008.5.1.4.1.1.7.4	Implicit VR Little Endian	1.2.840.10008.1.2	
True Color		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Secondary		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Capture Image				
Storage Class				
CT Image	1.2.840.10008.5.1.4.1.1.2	Implicit VR Little Endian	1.2.840.10008.1.2	
Storage		Explicit VR Big Endian	1.2.840.10008.1.2.2	
		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
MR Image	1.2.840.10008.5.1.4.1.1.4	Implicit VR Little Endian	1.2.840.10008.1.2	
Storage		Explicit VR Big Endian	1.2.840.10008.1.2.2	
		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Ultrasound	1.2.840.10008.5.1.4.1.1.6.1	Implicit VR Little Endian	1.2.840.10008.1.2	
Image Storage		Explicit VR Big Endian	1.2.840.10008.1.2.2	
		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Ultrasound	1.2.840.10008.5.1.4.1.1.3.1	Implicit VR Little Endian	1.2.840.10008.1.2	
Multi-frame		Explicit VR Big Endian	1.2.840.10008.1.2.2	
Image Storage		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None
Verification	1.2.840.10008.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	
		Explicit VR Big Endian	1.2.840.10008.1.2.2	
		Explicit VR Little Endian	1.2.840.10008.1.2.1	SCP None

### 5.1.3.1.2 SOP-specific Conformance Statement – Receiving Images from a Remote Node (with Retrieve)

The FLUOROSPOT Compact DICOM application conforms to the Full Storage Class at Level 1. In the event of a successful C-STORE operation, the image has successfully been written on disk.

The FLUOROSPOT Compact DICOM receiver returns the status Success upon successful operation otherwise one of the following status codes is returned and the association is aborted:

- Refused (A700):

This error status indicates a lack of Resources (e.g. not enough disk space) on the FLUOROSPOT Compact modality.

- Error (A900 or C000):

An error occurred while processing the image that makes it impossible to proceed. The image will not be stored and the association aborted.

If an image instance is received that is identified by a SOP Instance UID that is already used by an Instance stored in database then the actual received image will be refused. The existing Instance is not superseded.

### 5.1.3.1.2.1 Image Pixel Attribute Description for Receiving of Images

The FLUOROSPOT Compact DICOM receiver accepts pixel data with unsigned integer and 8 or 16 bits allocated.

Accepted values:

Pixel plane

- Samples per Pixel (0028,0002) = 1
- Photometric Interpretation (0028,0004) = "MONOCHROME2" + "RGB"
- Only Pixel Aspect Ratio (0028,0034) 1:1 is supported
- Pixel Representation (attribute 0028,0103) = 0
- Bits Allocated (0028,0100) = 8, 16
- Bits Stored (0028,0101) = 8, 10, 12, 16
- High Bit (0028,0102) = 7, 9, 11, 15

Furthermore following restrictions are valid:

- For VOI LUT, only the linear LUT (Window Center/Width) and not the VOI LUT Sequence is supported.
- Display of overlay planes is not supported.
- No manipulations except windowing are allowed on imported images not generated by a FLUOROSPOT Compact imaging system.
- Multiframe objects with identical image type are expected to be separated in series level.

### 5.1.3.1.3 Presentation Context Acceptance Criterion

The FLUOROSPOT Compact DICOM application will accept any number of verification or storage SOP classes that are listed above. The number of presentation contexts accepted is limited to the maximum of 127 (DICOM limit). In the event that the FLUOROSPOT Compact DICOM application runs out of resources, it will reject the association request.

#### 5.1.3.1.4 Transfer Syntax Selection Policies

The FLUOROSpot Compact DICOM application currently supports the Implicit VR Little Endian, the Explicit VR Little Endian and Explicit VR Big Endian Transfer Syntaxes

Any proposed presentation context including one of these transfer syntaxes will be accepted. Any proposed presentation context that does not include one of these transfer syntaxes will be rejected.

Explicit Little Endian Transfer Syntax will be preferred over the implicit Default Transfer Syntax.

The FLUOROSpot Compact DICOM application will remove any Private Attributes not known to the application.

# 6 Implementation Model Storage Commitment

The Storage Commitment Service Class defines an application-level class-of-service that facilitates commitment of storage. The Storage Commitment Service Class enables an Application Entity (AE) acting as SCU to request another Application Entity (AE) acting as an SCP to make the commitment for the safekeeping of the SOP Instances (i.e. that the SOP Instances will be kept for an implementation specific period of time and can be retrieved).

The FLUOROSPOT Compact DICOM application will support the Storage Commitment Push Model SOP Class as SCU.

## 6.1 Application Data Flow Diagram

“Send Image complete” is performed as a result of a completed Storage request. Successful completion of a Send to... request will result in automated initiation of a Storage Commitment Service. Each request results in an initiation for an association. The association will be closed after a maximum of 10 seconds (configurable in Service) of waiting for the response. After that receiving the Storage Commitment status is expected on a different association.

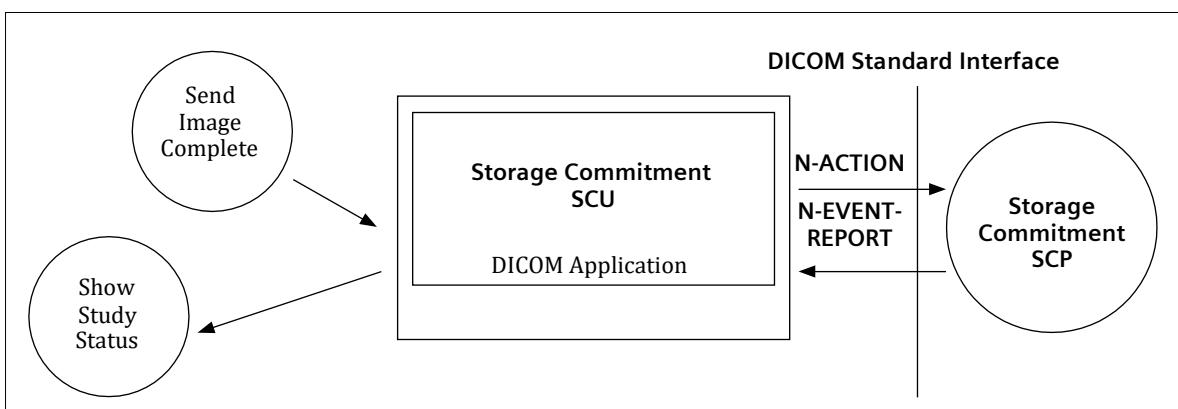


Figure 3: FLUOROSPOT Compact Application Data Flow Diagram – Storage Commitment

## 6.2 Functional Definitions of Application Entities

With each successfully completed send operation the FLUOROSPOT Compact DICOM Application will create a Storage Commitment Push Model Identifier from the SOP Instances sent. Then a Storage Commit Request is triggered. The FLUOROSPOT Compact will wait up to 10 seconds for responses, after that it expects the related response-trigger on a different association.

The commitment status derived from the related trigger response will be indicated in the Study list. Additionally the status will be evaluated previously to delete operations to inform the user about possible loss of information due to incomplete send operations and/or commit failures.

In any case, commitment will only be requested for previously and successfully sent images.

Storage Commitment is possible to different configured network nodes. Each Storage SCP can have its own Storage Commitment Server associated.

## 6.3 Sequencing of Real-World Activities

The Storage Commitment trigger is automatically derived from the successful completion of a Send operation.

# 7 AE Specification Storage Commitment

## 7.1 Storage Commitment AE Specification

The FLUOROSPOT Compact DICOM application will use one AE when performing Storage Commitment request and when awaiting notification from the Storage Commitment Provider.

Siemens Healthineers FLUOROSPOT Compact DICOM products provide Standard Conformance to the following DICOM SOP Classes as SCU:

SOP Class Name	SOP Class UID
Storage Commitment Push Model	1.2.840.10008.1.20.1

- **Association Establishment Policies**

### 7.1..1 General

The DICOM application will generate a Storage Commitment Identifier which references to all Instances of a selected study in the scope of the FLUOROSPOT Compact application. The request is then sent over a single opened association. The FLUOROSPOT Compact will wait for status responses to the Storage Commitment Request. If the Provider accepts the Storage Commitment with Success status, the generated Transaction UID, together with study identification data and a time-stamp, is kept in a Pending Request Queue. The association is then closed. Multiple Storage Commitment Requests can be held in the Pending Request Queue.

The default PDU size used will be 64 KB.

### 7.1..2 Number of Associations

The FLUOROSPOT Compact will initiate one association to send Storage Commitment requests to a configured Archive destination.

### 7.1..3 Asynchronous Nature

The FLUOROSPOT Compact DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 7.1..4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

- **Association Initiation Policy**

If a Storage Commitment Request needs to be sent to an Archive destination the FLUOROSPOT Compact DICOM application attempts to initiate a new association for

- DIMSE N-ACTION service operations.

### 7.1..1 Real World Activity – Storage Commitment SCU

#### 7.1..1.1 Associated Real-World Activity – Send Image Complete

Whenever a Store Request to a Storage SCP with associated Storage Commitment SCP is completed in the job queue, a Storage Commitment Identifier is generated for the same scope of Instances as transferred through the successful Storage Request. The identifier is then sent with the N-ACTION service to the Storage Commitment SCP. The association will wait up to 10 seconds for responses after the Storage Commitment Provider accepted the request.

Storage Commitment SCP nodes can be configured by the CSE.

A number of retries for failed commit requests and a Storage Commitment timeout can be configured in User Settings.

#### 7.1..1.2 Proposed Presentation Contexts – Send Image Complete

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax Name	UID	Transfer Syntax Name List	UID List	Role	Ext. Neg.
Storage	1.2.840.10008.1.20.1	Implicit VR Little Endian	1.2.840.10008.1.2		
Commitment		Explicit VR Big Endian	1.2.840.10008.1.2.2		
Push Model		Explicit VR Little Endian	1.2.840.10008.1.2.1 SCU		None

#### 7.1..1.3 SOP-specific Conformance Statement – Send Image Complete

The FLUOROSPOT Compact DICOM application will NOT support the Storage Media File Set ID attributes.

Siemens Healthineers FLUOROSPOT Compact DICOM application will request Storage commitment for the following SOP Classes as SCU:

SOP Class Name	SOP Class UID
Computed Radiography Image Storage <sup>1</sup>	1.2.840.10008.5.1.4.1.1.1
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7
X-Ray Radiofluoroscopic Image Storage <sup>2</sup>	1.2.840.10008.5.1.4.1.1.12.2
Digital X-Ray Image Storage – For Presentation <sup>9</sup>	1.2.840.10008.5.1.4.1.1.1

- Association Acceptance Policy**

The FLUOROSPOT Compact Storage Commitment DICOM application attempts to accept a new association for

- DIMSE N-EVENT-  
REPORT service  
operations.

<sup>1</sup> Only for Radsystems

<sup>2</sup> Only for Fluorosystems

### 7.1.3.1 Real World Activity – Storage Commitment SCU

#### 7.1.3.1.1 Associated Real-World Activity – Show Study Status

Consecutive to start-up, the FLUOROSpot Compact DICOM application will await Storage commitment Notification triggers. Any incoming Notification will be checked for validity, that is, if the related Transaction UID is still part of the Pending Request Queue. If not, the related Notification trigger is rejected with an “Unable to process” error. The related study is already set to commit failed due to time-out on notification.

If the Notification is valid, the Notification Identifier is evaluated and the related Instances are marked with the related status. The over-all Commit Status of the study is derived from propagation of the States of all instances included by a study.

The Study Status indicated in the Study list can be one of

- <no Status> (-) [not sent or send failed]
- Sent (S) [all images successfully sent to DICOM Send node, commitment successful for the complete study]
- Partially Sent (s) [some or all instances of the Study successfully sent to DICOM Send node, commitment outstanding for at least 1 image of the study]
- Archived (A) [all images successfully sent to Archive node, commitment successful for the complete study]
- Partially Archived (a) [some or all instances of the Study successfully sent to Archive node, commitment outstanding for at least 1 image of the study]

The FLUOROSpot Compact delete function will be sensitive on the Study Status and will protect Studies. The user can override the protection by confirming the respective delete dialogues.

In case of failure the user has to repeat the transfer of images to the Archive destination. Another Storage Commitment will be performed after sending is completed successfully.

#### 7.1.3.1.2 Accepted Presentation Contexts – Show Study Status

The FLUOROSpot Compact DICOM application will accept Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax		Transfer Syntax		Role	Ext.
Name	UID	Name List	UID List	Neg.	
Storage Commitment	1.2.840.10008.1.20.1	Implicit VR LittleEndian	1.2.840.10008.1.2		
		Explicit VR BigEndian	1.2.840.10008.1.2.2		
Push Model		Explicit VR LittleEndian	1.2.840.10008.1.2.1	SCU*1	None

\*1 according to the rules defined for Storage Commitment N-EVENT-REPORT, the FLUOROSpot Compact itself assumes the SCU role and therefore accepts the SCP role negotiation of the Storage Commitment Provider, even if initiator of the association negotiation.

#### 7.1.3.1.3 SOP-specific Conformance Statement – Show Study Status


The FLUOROSpot Compact DICOM application will NOT support the Storage Media File Set ID attributes.

# 8 Implementation Model Query / Retrieve

The query/retrieve service class defines an application-level class of services that facilitates the management of images and patient data against the well-defined information model of DICOM and allows a DICOM AE to retrieve images from a remote DICOM node. The FLUOROSPOT Compact DICOM query/retrieve application supports the query/retrieve services to act as SCU.

## 8.1 Application Data Flow Diagram

The FLUOROSPOT Compact DICOM application acts as SCU for the query/retrieve network service.

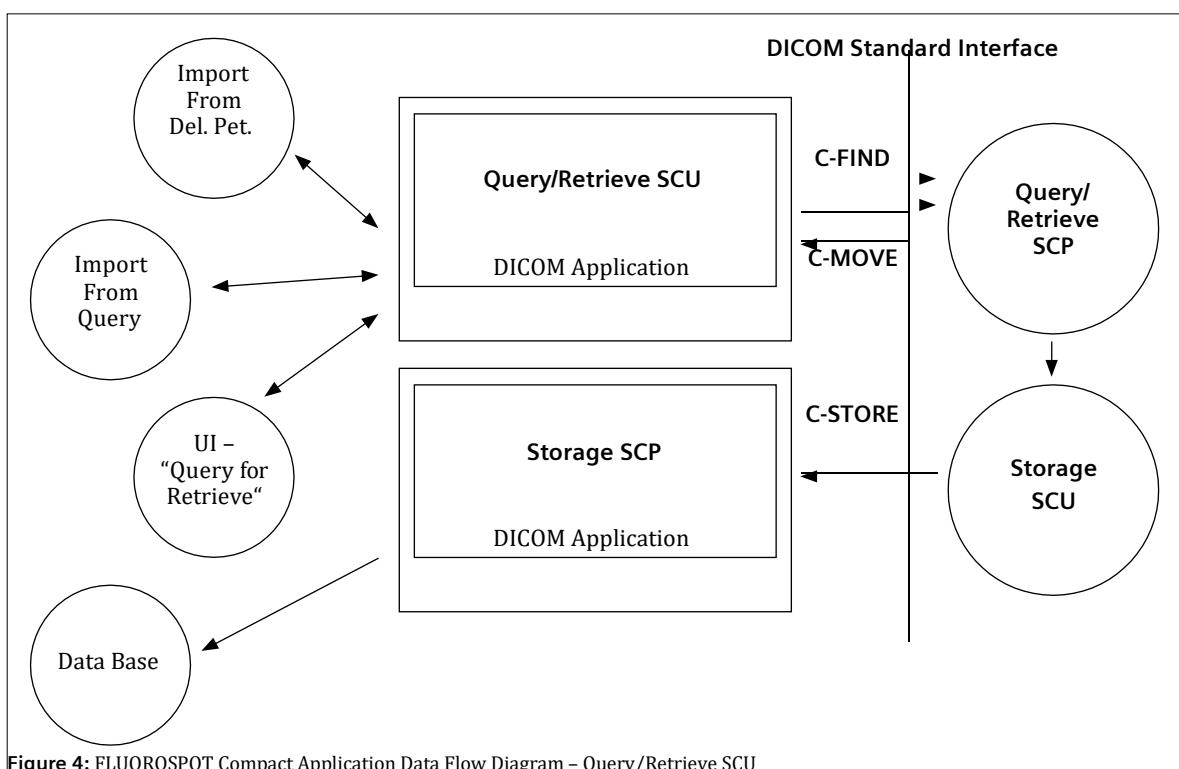

## 8.2 Functional Definitions of Application Entities

The FLUOROSPOT Compact DICOM query/retrieve SCU requests the remote query/retrieve SCP to perform a search (C-FIND) and match to the keys specified in the request in order to display the results in the FLUOROSPOT Compact user interface. If the defined limit of Query results is reached, the FLUOROSPOT Compact will cancel the running query and discard any further incoming query result responses. Queries can be operated in parallel to one running import (C-MOVE) activity.

Depending on user action (Import) the FLUOROSPOT Compact DICOM query/retrieve SCU sends a C-MOVE DIMSE service to initiate a C-STORE sub-operation on the SCP to start an image transfer from remote Storage SCU (running on Query/Retrieve SCP) to the FLUOROSPOT Compact Storage SCP. The user has the possibility to cancel a running import by an "Import Cancel" function. The FLUOROSPOT Compact DICOM application will perform only one running import operation at a time.

### 8.3 Sequencing of Real-World Activities

Import operation has to be completed or successfully canceled before a new import can be initiated at the user interface.

# 9 AE Specification Query/Retrieve

## 9.1 Query/Retrieve Service Aes Specification

The Query/Retrieve SCU requests that the remote SCP performs a match of all keys specified in the request, against the information in its database.

Siemens Healthineers FLUOROSPOT Compact DICOM products provide Standard Conformance to the following DICOM SOP Classes as SCU:

SOP Class Name	SOP Class UID
Study Root Query/Retrieve Information Model – FIND	1.2.840.10008.5.1.4.1.2.2.1
Study Root Query/Retrieve Information Model – MOVE	1.2.840.10008.5.1.4.1.2.2.2

- **Association Establishment Policies**

### 9.1..1 General

With the “Query for Retrieve” function the query data are input and the DICOM query/retrieve application is started. A query request will be sent out to the selected remote node and the response data will be displayed for the user. Upon request (Import), the retrieval of selected items is initiated.

Running Query requests will be automatically canceled whenever the defined limit has been reached. A running import can be canceled upon user's request.

The default PDU size used will be 64 KB.

### 9.1..2 Number of Associations

The FLUOROSPOT Compact DICOM application initiates one association for each query request being processed to a remote node. Only one Import (C-MOVE) request can be active at a time.

### 9.1..3 Asynchronous Nature

The FLUOROSPOT Compact DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 9.1..4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

- **Association Initiation Policy**

The query user interface will request the query-data (combined input of patient- and study-related data) from user and triggers one or more C-FIND request to the selected remote node. The response data will be displayed in the query UI as a combined Patient/Study list. Navigation (further queries) to Series-Level is supported by the FLUOROSPOT Compact, navigation to Image-Level is not supported. A running Query will be canceled (C-CANCEL) by the DICOM Application if the response limit has been reached.

When requesting Import of a related item, the browser requests the DICOM retrieve application to send a C-MOVE request to the related remote node. SOP Instances will then be received by the Storage SCP as described in the related section. An import can be terminated by user request (C-CANCEL).

#### 9.1.2.1 Real World Activity – Find SCU

##### 9.1.2.1.1 Associated Real-World Activity – Query for Retrieve

The associated Real-World activity is to fill out a query form with search data and pass it as a query request to the network application that issues one or more C-FIND requests over a previously built association. The remote SCP will respond with related data-entries that will be passed to a browser application. When data transfer is finished, the association is closed.

In case of error, the list is built up to the point where the error occurred. The user is informed about incomplete processing of the query results.

##### 9.1.2.1.2 Proposed Presentation Contexts – Query for Retrieve

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax Name	UID	Transfer Syntax Name List	UID List	Role	Ext. Neg.
Study Root	1.2.840.10008.5.1.4.1.2.2.1	Implicit VR LittleEndian	1.2.840.10008.1.2		
Query/Retrieve		Explicit VR BigEndian	1.2.840.10008.1.2.2		
Model – FIND		Explicit VR LittleEndian	1.2.840.10008.1.2.1 SCU		None

Extended Negotiation to request “relational query” is not supported by the FLUOROSPOT Compact DICOM application.

### 9.1.2.1.3 SOP Specific Conformance Statement – Query for Retrieve

The FLUOROSPOT Compact DICOM Query/Retrieve SCU supports hierarchical queries with the search keys listed below. The Query SCU does not support the interactive querying of attributes on IMAGE level. The following table describes the matching keys / return keys that the SCU supports:

Attribute Name	Tag	Type	Matching	User Input	Return Value Display
<b>Patient Level</b>					
Patient's Name <sup>1</sup>	(0010,0010)	R	Wildcard	Enter value	Yes
Patient ID	(0010,0020)	U / R	Wildcard	Enter value	Yes
Patient's Birth Date	(0010,0030)	O	Universal (Null)	Enter value	Yes
Patient's Sex	(0010,0040)	O	Universal (Null)	Select value	Yes
<b>Study Level</b>					
Study Instance UID	(0020,000D)	U	Universal (Null) / Single Value	-- / Select from List	-- / Yes
Study ID	(0020,0010)	R	Universal (Null)	Enter value	No
Study Date	(0008,0020)	R	Universal (Null)	--	Yes
Study Time	(0008,0030)	R	Universal (Null)	--	Yes
Accession Number	(0008,0050)	R	Universal (Null)	Enter value	Yes
Study Description	(0008,1030)	O	Universal (Null)	--	Yes
Number of Study Related Series	(0020,1206)	O	Universal (Null)	--	Yes
Number of Study Related Instances	(0020,1208)	O	Universal (Null)	--	Yes
<b>Series Level</b>					
Series Instance UID	(0020,000e)	U	Single Value	Select from List	Yes
Series Number	(0020,0011)	R	Universal (Null)	--	Yes
Modality	(0008,0060)	R	Universal (Null)	--	Yes
Number of Series Related Instances	(0020,1209)	O	Universal (Null)	--	Yes
Series Description	(0008,103e)	O	Universal (Null)	--	Yes

<sup>1</sup> In the Patient Name always a “\*” is added

The Find SCU interprets following status codes:

Service Status	Meaning	Error Codes	Related Fields
Refused	Out of Resources	A700	(0000,0902)
Failed	Identifier does not match SOP Class	A900	(0000,0901) (0000,0902)
	Unable to process	Cxxx	(0000,0901) (0000,0902)
Cancel	Matching terminated due to Cancel request	FE00	None
Success	Matching is complete – No final Identifier is supplied	0000	None
Pending	Matches are continuing – Current Match is supplied and any Optional Keys were supported in the same manner as Required Keys	FF00	Identifier
	Matches are continuing – Warning that one or more Optional Keys were not supported for existence and/or matching for this identifier	FF01	Identifier

### 9.1.2.2 Real-World Activity – Move SCU

#### 9.1.2.2.1 Associated Real-World Activity – Import from Query

When selecting a data entry in the Query UI and activating the “Import” function, a retrieval request is passed to the archival application that issues a C-MOVE service according to the Study Root query model. (The Storage Service Class Conformance Statement SCP section describes the response behavior to the C-STORE request sub-operation that is generated by the SCP when processing the C-MOVE request.)

The transferred image data are processed as described in the Storage Class SCP descriptions.

The possibility to request the remote C-MOVE SCP to move data to an application entity other than the FLUOROSPOT Compact DICOM application is NOT USED.

#### 9.1.2.2.2 Proposed Presentation Contexts – Import from Query

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax Name	UID	Transfer Syntax Name List	UID List	Role	Ext. Neg.
Study Root	1.2.840.10008.5.1.4.1.2.2.2	Implicit VR LittleEndian Explicit VR BigEndian	1.2.840.10008.1.2		
Query/Retrieve Model – MOVE		Explicit VR LittleEndian	1.2.840.10008.1.2.1 SCU		None

### 9.1.2.2.3 SOP Specific Conformance Statement – Import from Query

At association establishment time the C-MOVE presentation context shall be negotiated. The C-STORE sub-operations must be done on a different association to transfer images to the own Storage Service Class SCP.

The FLUOROSPOT Compact will make use of the “list of UID” mechanism when specifying the Instances to be retrieved.

The Move SCU interprets following status codes:

Service Status	Meaning	Error Codes	Related Fields
Refused	Out of Resources – Unable to calculate number of matches	A701	(0000,0902)
	Out of Resources – Unable to perform sub operations		(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Failed	Identifier does not match SOP Class	A900	(0000,0901) (0000,0902)
	Unable to process (0000,0902)		CXXX (0000,0901)
Cancel	Sub-operations terminated due to Cancel Indication	FE00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
	Sub-operations Complete – One or more Failures of Warnings		
Warning	Sub-operations Complete – No Failures or Warning	B000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
	Sub-operations are continuing		
Success	Sub-operations Complete – No Failures or Warning	0000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
	Sub-operations are continuing		
Pending	Sub-operations are continuing	FF00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
	Sub-operations are continuing		

### 9.1.2.2.4 Associated Real-World Activity – Import from “Deleted Patients”

When selecting a data entry in the “Deleted Patients” UI and activate the “Import” function, a retrieval request is passed to the archival application which issues a C-MOVE service according to the Study Root query model. (The Storage Service Class Conformance Statement SCP section describes the response behavior to the C-STORE request sub-operation that is generated by the SCP when processing the C-MOVE request.). Instances can only be transferred to the system that originated the Import request.

The transferred image data are processed as described in the Storage Class SCP descriptions.

#### 9.1.2.2.5 Proposed Presentation Contexts – Import from “Deleted Patients”

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax Name	UID	Transfer Syntax Name List	UID List	Role	Ext. Neg.
Study Root	1.2.840.10008.5.1.4.1.2.2.2	Implicit VR LittleEndian	1.2.840.10008.1.2		
Query/Retrieve Model – MOVE		Explicit VR BigEndian	1.2.840.10008.1.2.2		
		Explicit VR LittleEndian	1.2.840.10008.1.2.1 SCU		None

#### 9.1.2.2.6 SOP Specific Conformance Statement – Import from “Deleted Patients”

At association establishment time the C-MOVE presentation context shall be negotiated. The C-STORE sub-operations must be done on a different association to transfer images to the own Storage Service Class SCP.

The Move SCU interprets following status codes:

Service Status	Meaning	Error Codes	Related Fields
Refused	Out of Resources – Unable to calculate number of matches	A701	(0000,0902)
	Out of Resources – Unable to perform sub operations	A702	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Failed	Identifier does not match SOP Class	A900	(0000,0901) (0000,0902)
	Unable to process	CXXX	(0000,0901) (0000,0902)
Cancel	Sub-operations terminated due to Cancel Indication	FE00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Warning	Sub-operations Complete – One or more Failures of Warnings	B000	(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Success	Sub-operations Complete – No Failures or Warning 0000		(0000,1020) (0000,1021) (0000,1022) (0000,1023)
Pending	Sub-operations are continuing	FF00	(0000,1020) (0000,1021) (0000,1022) (0000,1023)

- Association Acceptance Policy**

Not applicable.

# 10 Implementation Model Modality Worklist

The Basic Worklist Management Service Class defines an application-level class-of-service that facilitates access to worklists. The worklist is queried by the AE (modality) and supplies the SCU with the tasks scheduled and which have to be performed on the modality. The FLUOROSPOT Compact DICOM application will support the Modality Worklist SOP class as SCU.

## 10.1 Application Data Flow Diagram

Worklist Update is performed as a result of an operator request. Each request results in an initiation for an association. Under normal condition the association will be closed after receiving a "Success" response from the Information System.

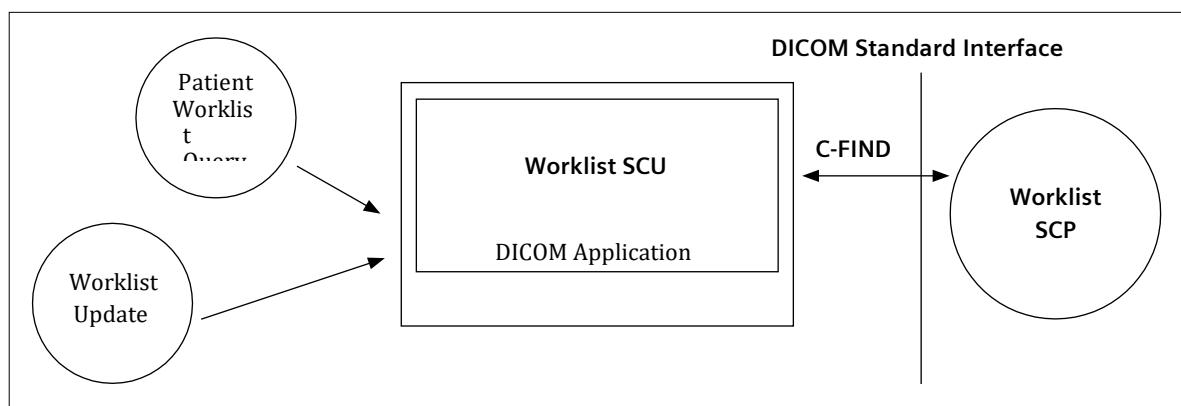


Figure 5: FLUOROSPOT Compact Application Data Flow Diagram – Modality Worklist

## 10.2 Functional Definitions of Application Entities

The request for a Worklist Update is initiated either by user interaction, i.e. pressing the button "Worklist Update" or automatically at configurable time intervals and at the switch from Patient Mode to Examination Mode. With "Worklist Update" the automated Query mechanism is performed immediately on request, while with "Patient Worklist Query" a dialog to enter search criteria is opened and an interactive query can be performed.

Upon initiation of the request, the FLUOROSPOT Compact will build an Identifier for the C-FIND request, will initiate an association to send the request and will wait for Worklist responses. After retrieval of all responses, FLUOROSPOT Compact will access the local database to add or update patient demographic data. To protect the system from overflow, the FLUOROSPOT Compact will limit the number of processed Worklist responses to a configurable maximum.

With automated worklist queries (including "Worklist Update") the FLUOROSPOT Compact always requests all items for a configured Scheduled Procedure Step Start Date, Modality Type and the Scheduled Station AE Title.

If any other SCP response status than "Success" or "Pending" is received by FLUOROSPOT Compact, a message "update failed" will appear on the user interface.

The FLUOROSPOT Compact Worklist Request Identifier is described in Annex A.10.

## 10.3 Sequencing of Real-World Activities

Not applicable.

# 11 AE Specification Modality Worklist

## 11.1 Modality Worklist Service Aes Specification

Siemens Healthineers FLUOROSPOT Compact DICOM products provide Standard Conformance to the following DICOM SOP Classes as SCU:

SOP Class Name	SOP Class UID
Modality Worklist Information Model – FIND	1.2.840.10008.5.1.4.31

- **Association Establishment Policies**

### 11.1..1 General

Worklist can be queried in automated mode – either cyclic or with “Update” – or with an interactive query with search criteria entered by application.

The default PDU size used will be 64 KB.

### 11.1..2 Number of Associations

The FLUOROSPOT Compact DICOM application initiates one association at a time for a Worklist request.

### 11.1..3 Asynchronous Nature

The FLUOROSPOT Compact DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 11.1..4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

- **Association Initiation Policy**

The FLUOROSPOT Compact DICOM Application Entity acts as a Service Class User (SCU) for the Modality Worklist Service Class (to request a Worklist from a RIS).

To do so, the FLUOROSPOT Compact will issue a

- C-FIND DIMSE according to the Modality Worklist Information Model or a
- C-FIND-CANCEL DIMSE to stop processing of running worklist query.

### 11.1..1 Real-World Activity

#### 11.1..1.1 Associated Real-World Activity – Patient Worklist Query

Patient Worklist Query will display a dialog for entering data as search criteria. When the Query is started on user request, only the data from the dialog will be inserted as matching keys into the query. An association is established and the query is sent. During receiving the worklist response items are counted and the query processing is canceled if the configurable limit of items (maximum 999) is reached. The results will be displayed in a separate list. All worklist data from previous queries will be deleted when new data is received.

#### 11.1..1.2 Proposed Presentation Contexts – Patient Worklist Query

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax	Transfer Syntax	Role	Ext.	
Name	UID	Name List	UID List	Neg.
Modality Work- Information	1.2.840.10008.5.1.4.31	Implicit VR LittleEndian 1.2.840.10008.1.2 list		
Model – FIND		Explicit VR BigEndian 1.2.840.10008.1.2.2		
		Explicit VR LittleEndian 1.2.840.10008.1.2.1	SCU	None

#### 11.1..1.3 SOP Specific Conformance Statement – Patient Worklist Query

Please refer to Annex A.10 for the description of the FLUOROSPOT Compact Worklist Request Identifier.

If any other SCP response status than “Success” or “Pending” is received by FLUOROSPOT Compact, a message “query failed” will appear on the user interface.

#### 11.1..1.4 Associated Real-World Activity –Worklist Update

Worklist Update attempts to download a Worklist from a remote node. If the FLUOROSPOT Compact AE establishes an association to a remote AE, it will transfer all worklist items via the open association. Also here the incoming worklist response items are counted and the query processing is canceled if the configurable limit of items (maximum 999) is reached. The results will be displayed in a separate list. All worklist data from previous queries will be deleted when new data is received.

#### 11.1..1.5 Proposed Presentation Contexts –Worklist Update

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax	Transfer Syntax	Role	Ext.	
Name	UID	Name List	UID List	Neg.
Modality Work- Information	1.2.840.10008.5.1.4.31	Implicit VR LittleEndian 1.2.840.10008.1.2 list		
Model – FIND		Explicit VR BigEndian 1.2.840.10008.1.2.2		
		Explicit VR LittleEndian 1.2.840.10008.1.2.1	SCU	None

#### 11.1..1.6 SOP Specific Conformance Statement –Worklist Update

Please refer to Annex A.10 for the description of the FLUOROSPOT Compact Worklist Request Identifier.

If any other SCP response status than “Success” or “Pending” is received by FLUOROSPOT Compact, a message “update failed” will appear on the user interface.

- **Association Acceptance Policy**

Not applicable.

# 12 Implementation Model

## Modality Performed Procedure Step

The FLUOROSPOT Compact DICOM application will support the Modality Performed Procedure Step (MPPS) SOP class as SCU.

### 12.1 Application Data Flow Diagram

MPPS Update is performed as a result of an operator request. Applying dose through acquisition of images or fluoroscopy mode will result in automated creation on a MPPS Instance. Each request results in an initiation for an association. The association will be closed after receiving a "Success" response from the Information System.

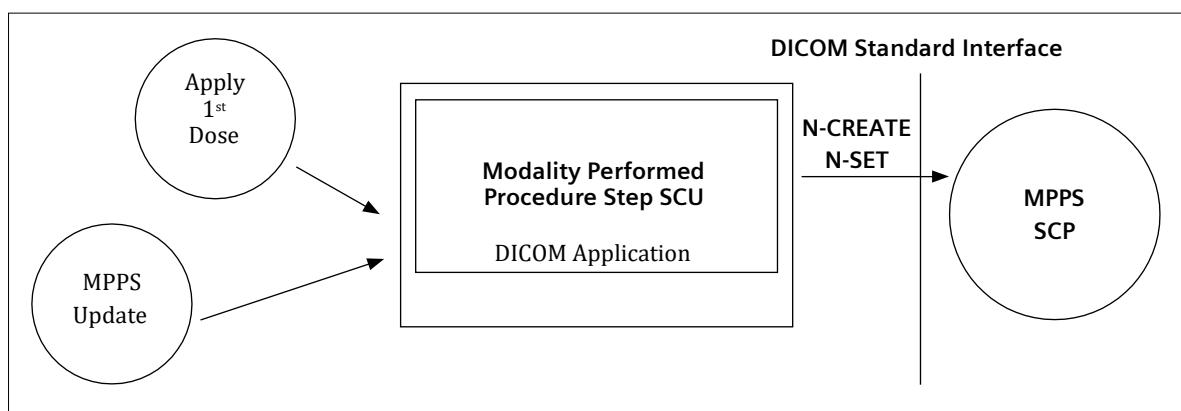


Figure 6: FLUOROSPOT Compact Application Data Flow Diagram – MPPS

### 12.2 Functional Definitions of Application Entities

With the 1st Dose applied consecutive to registering a Patient (Scheduled Procedure Step from Worklist), the FLUOROSPOT Compact DICOM application will create a MPPS Instance and communicate it to the MPPS SCP.

To inform the MPPS SCP about the progress or the completion of an MPPS instance, an update can be performed. It is configurable for the user how the update is performed:

- Automatic (status "COMPLETED" is sent after a patient gets inactive)
- Automatic with MPPS Dialog ("COMPLETED", "IN PROGRESS" or "DISCONTINUED" can be selected. Dialog is displayed after a patient gets inactive)
- Manual with MPPS Dialog ("COMPLETED", "IN PROGRESS" or "DISCONTINUED" can be selected.)

After a state of "COMPLETED" or "DISCONTINUED", the FLUOROSPOT Compact DICOM application will no longer allow updates on the related MPPS Instance.

The FLUOROSPOT Compact will support creation of "unscheduled cases" by allowing MPPS Instances to be communicated for locally registered Patients.

### 12.3 Sequencing of Real-World Activities

Not applicable.

# 13 AE Specification Modality Performed Procedure Step

## 13.1 Modality Performed Procedure Step Service Aes Specification

Siemens Healthineers FLUOROSPOT Compact DICOM products provide Standard Conformance to the following DICOM SOP Classes as SCU:

SOP Class Name	SOP Class UID
Modality Performed Procedure Step	1.2.840.10008.3.1.2.3.3

- **Association Establishment Policies**

### 13.1..1 General

The creation of a MPPS Instance is done automatically by FLUOROSPOT Compact after the first application of radiation ("1st dose applied"). For emergency patients (patient data incomplete) the creation will be delayed until completion of the procedure step.

If the 1st dose is applied, while a worklist update is in progress, the MPPS N-Create is delayed until the next update of the Procedure Step.

Further updates on the MPPS data can either be done interactively from the related MPPS user interface or will automatically be performed if configured by the user.

The default PDU size used will be 64 KB.

### 13.1..2 Number of Associations

The FLUOROSPOT Compact DICOM application initiates one association at a time for a communication of MPPS information.

### 13.1..3 Asynchronous Nature

The FLUOROSPOT Compact DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 13.1..4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

- **Association Initiation Policy**

The FLUOROSPOT Compact DICOM Application Entity acts as a Service Class User (SCU) for the Modality Performed Procedure Step Service Class (to notify a RIS about status of a procedure while it is performed).

To do so, the FLUOROSPOT Compact will issue a

- N-CREATE DIMSE according to the CREATE Modality Performed Procedure Step SOP Instance operation or a
- N-SET DIMSE to update the contents and state of the MPPS according to the SET Modality Performed Procedure Step Information operation.

### 13.1..1 Real-World Activity

#### 13.1..1.1 Associated Real-World Activity – Apply 1st Dose

Consecutive to registration of Patient, the FLUOROSPOT Compact is awaiting the 1st application of radiation to the patient ("1st dose applied"). For emergency patients (patient data incomplete) the creation will be delayed until completion of the procedure step. From this event the trigger to create a MPPS Instance is derived. The related Instance is then immediately communicated to the configured RIS system. An association is established and the MPPS Instance is sent.

#### 13.1..1.2 Proposed Presentation Contexts – Apply 1st Dose

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax		Transfer Syntax	Role	Ext.
Name	UID	Name List	UID List	Neg.
Modality	1.2.840.10008.3.1.2.3.3	Implicit VR LittleEndian	1.2.840.10008.1.2	
Performed		Explicit VR BigEndian	1.2.840.10008.1.2.2	
Procedure Step		Explicit VR LittleEndian	1.2.840.10008.1.2.1	SCU None

#### 13.1..1.3 SOP Specific Conformance Statement – Apply 1st Dose

Please refer to Annex A.11 for the description of the FLUOROSPOT Compact Modality Performed Procedure Step CREATE Request Identifier.

#### 13.1..1.4 Associated Real-World Activity – MPPS Update

With a dedicated MPPS UI it is possible to set additional information and set the state to finalize the MPPS. Sending updates of the MPPS Instance can be interactively triggered. A MPPS Instance that has been sent with a state of "COMPLETED" or "DISCONTINUED" can no longer be updated.

A MPPS state of "COMPLETED" can also be sent automatically after a patient gets inactive. This automatic update without any user input is configurable for the user.

### 13.1..1.5 Proposed Presentation Contexts – MPPS Update

The FLUOROSPOT Compact DICOM application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax	Transfer Syntax	Role	Ext.
Name	Name List	UID List	Neg.
Modality Performed	Implicit VR Little Endian Explicit VR Big Endian	1.2.840.10008.1.2 1.2.840.10008.1.2.2	
Procedure Step	Explicit VR Little Endian	1.2.840.10008.1.2.1	SCU None

### 13.1..1.6 SOP Specific Conformance Statement – MPPS Update

Please refer to Annex A.11 for the description of the FLUOROSPOT Compact Modality Performed Procedure Step SET Information Request Identifier.

The MPPS Set SCU interprets the following additional status code:

Service Status	Meaning	Error Codes
Processing Failure updated	Performed Procedure Step Object may no longer be A710	

If any other SCP response status than “Success” or “Pending” is received by FLUOROSPOT Compact, a message “MPPS update failed” will appear on the user interface.

- **Association Acceptance Policy**

Not applicable.

# 14 Implementation Model Print

The Print Management Service Classes define an application-level class of services that facilitate the printing of images on a hardcopy medium. The print management SCU and the print management SCP are peer DICOM print management application entities. The FLUOROSPOT Compact DICOM print application supports the print management DIMSE services to act as SCU.

## 14.1 Application Data Flow Diagram

The FLUOROSPOT Compact DICOM application acts as SCU for the print management network service.

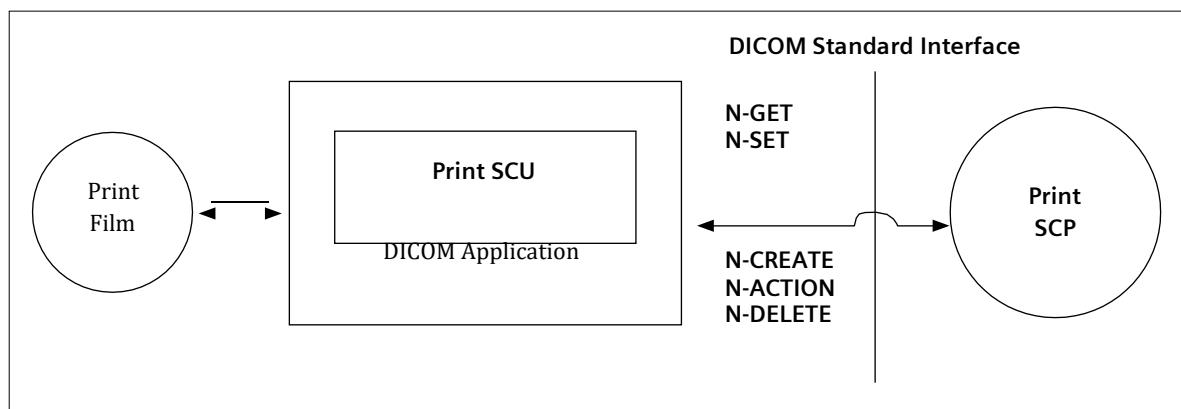


Figure 7: FLUOROSPOT Compact DICOM Application Data Flow Diagram – Print SCU

## 14.2 Functional Definition of Application Entities

The “Print Film” could either be invoked by the user interface whenever an image is ready to be printed on film, or automatically for a predefined set of images when study is completed.

The Print SCU will hold and maintain all data needed to compile a complete film-sheet from the data (images, layout, configuration) received. Whenever a film-sheet is ready to print, the related data is used to supply the information to the SOP Classes of the Basic Grayscale Print Management Meta SOP Class. A queue is maintained, in order to intermediately store up to 10 film-sheets in case of resource problems on printer. The SCU will only supply and require the mandatory SOP Classes of the Basic Grayscale Print Management Meta SOP Class.

It is configurable by the CSE to automatically initiate a retry for failed jobs.

## 14.3 Sequencing of Real-World Activities

Not applicable

# 15 Application Entity Specification Print

## 15.1 Print Management AE Specification

The FLUOROSPOT Compact print management SCU invokes print management DIMSE services to transfer images from the local AE to the remote SCP AE to print images with defined layout on a selected network-based DICOM hardcopy printer.

Siemens Healthineers FLUOROSPOT Compact DICOM products provide Standard Conformance to the following DICOM Print Management Meta SOP Classes as SCU:

SOP Class Name	SOP Class UID
Basic Grayscale Print Management Meta SOP Class	1.2.840.10008.5.1.1.9

- **Association Establishment Policies**

### 15.1..1 General

Whenever a film is completely set up, either automatically or by command, the job is prepared for processing. As soon as the queue is ready for processing the job, it is activated and encoded according the processing data. The related Print application will initiate an association to the print destination and process the printing of the related information.

The default PDU size used will be 64 KB.

### 15.1..2 Number of Associations

The FLUOROSPOT Compact DICOM application initiates one association at a time for each different print device configured.

### 15.1..3 Asynchronous Nature

The FLUOROSPOT Compact DICOM software does not support asynchronous communication (multiple outstanding transactions over a single association).

### 15.1..4 Implementation Identifying Information

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

- **Association Initiation Policy**

Triggered by the Print job queue the Print Management SCU establishes an association by using the DICOM association services. With the help of the N-GET request for the Printer SOP Class the Status is determined before printing.

### 15.1..1 Associated Real-World Activity

#### 15.1..1.1 Associated Real-World Activity – Print Film

Whenever a film-sheet is prepared by the user, it is forwarded to the Printer Job queue. As soon as the associated Printer device is available the job is activated and association is set up.

The film sheet is set up with the related Film Session and Film Box services. The images are internally processed and then sent as image boxes as needed. Status is controlled by awaiting any N-EVENT message all through the transfer until the last image or film-sheet is sent.

If the response from the remote application contains a status other than "Success" or "Warning" the association is aborted. It can be restarted any time by user interaction. An automatic retry can be configured by a CSE. The FLUOROSPOT Compact displays all warning and failure messages in the DICOM Film job list and writes them also into a log file.

#### 15.1..1.2 Proposed Presentation Context – Print Film

The FLUOROSPOT Compact DICOM print application will propose Presentation Contexts as shown in the following table:

**Presentation Context Table**

Abstract Syntax Name	UID	Transfer Syntax Name List	UID List	Role	Ext. Neg.
Basic Grayscale	1.2.840.10008.5.1.1.9	Implicit VR LittleEndian	1.2.840.10008.1.2		
Print Management Meta SOP class		Explicit VR LittleEndian	1.2.840.10008.1.2.1		
		Explicit VR BigEndian	1.2.840.10008.1.2.2	SCU	None
Presentation LUT SOP class	1.2.840.10008.5.1.1.23	Implicit VR LittleEndian	1.2.840.10008.1.2		
		Explicit VR LittleEndian	1.2.840.10008.1.2.1		
		Explicit VR BigEndian	1.2.840.10008.1.2.2	SCU	None

#### 15.1..1.3 SOP specific Conformance Statement – Meta SOP Classes

The FLUOROSPOT Compact DICOM print management SCU conforms to the DICOM Basic Grayscale Print Management Meta SOP Class.

The application uses a setting platform to define the properties of the connected DICOM Print SCP, e.g.:

- maximum number of print copies: 9
- supported film sizes of the connected DICOM Print SCP
- supported film formats of the DICOM Print SCP

The printing is only suspended in the case of a failure return status of the Print SCP.

### 15.1.2.1.3.1 Basic Film Session SOP class

The Basic Film Session information object definition describes all the user-defined parameters that are common for all the films of a film session. The Basic Film Session refers to one or more Basic Film Boxes that are printed on one hardcopy printer.

The FLUOROSPOT Compact DICOM print management SCU supports the following DIMSE Service elements for the Basic Film Session SOP Class as SCU:

- N-CREATE
- N-DELETE

The Basic Film Session SOP Class N-CREATE-RQ (SCU) uses the following attributes:

Attribute Name	Tag	Usage SCU	Supported Values
Number of Copies	(2000,0010)	U	Set by user
Print Priority	(2000,0020)	U	MED

be kept internally and used  
below:

The affected SOP Instance UID received with N-CREATE-RSP message will  
for later requests (e.g. N-DELETE-RQ) on the Basic Film Session – see

Attribute Name	Tag	Source of Information
Requested SOP Instance UID	(0000,1000) -> (0000,1001)	Affected SOP Instance UID of N-CREATE-RSP on Basic Film Session

The N-DELETE-RQ on the Basic Film Session SOP Class is used to remove the complete Basic Film Session SOP Instance hierarchy.

The Basic Film Session SOP class interprets the following status codes (from N-CREATE-RSP, N-DELETE-RSP messages):

Service Status	Meaning	Error Codes
Failed	Film session SOP instances hierarchy does not contain film box SOP instances Unable to create print job, print queue is full Image size is larger than images box size	C600 C601 C603
Warning	Memory allocation not supported Film session printing is not supported Film box does not contain image box (empty page)	B600 B601 B602
Success	Film box belonging to the film session is accepted for printing	0000

### 15.1.2.1.3.2 Basic Film Box SOP class

The Basic Film Box information object definition describes all the user-defined parameter of one film of the film session. The Basic Film Box information description defines the presentation parameters that are common for all images on a given sheet of film.

The Basic Film Box refers to one or more Image Boxes. Supported Service Elements as SCU are:

- N-CREATE
- N-ACTION

The Basic Film Box SOP class N-CREATE-RQ message uses the following attributes (the actual values for each attribute depend on DICOM printer configuration within the FLUOROSPORT Compact DICOM print management SCU):

Attribute Name	Tag	Usage SCU	Supported Values
Image Display Format	(2010,0010)	M	1*1
Referenced Film Session Sequence	(2010,0500)	M	
> Referenced SOP Class UID	(0008,1150)	M	1.2.840.10008.5.1.1.1
> Referenced SOP Instance UID	(0008,1155)	M	
Film Orientation	(2010,0040)	U	(from Camera Configuration) PORTRAIT or LANDSCAPE
Film Size ID	(2010,0050)	U	(from Camera Configuration) 8INX10IN, 10INX12IN, 11INX14IN,, 14INX14IN, 14INX17IN
Referenced Presentation LUT Sequence	(2050,0500)	MC	Only if Print Presentation LUT is configured for the DICOM Print SCP
> Referenced SOP Class UID	(0008,1150)	M	1.2.840.10008.5.1.1.23
> Referenced SOP Instance UID	(0008,1155)	M	Instance UID of presentation LUT object

The N-CREATE-RSP message from the Print SCP includes the Referenced Image Box Sequence with SOP Class/Instance UID pairs that will be kept internally to be further used for the subsequent Basic Image Box SOP Class N-SET-RQ messages.

When all Image Boxes (including parameters) for the film-sheet have been set, the FLUOROSPORT Compact DICOM print manager will issue an N-ACTION-RQ message with the SOP Instance UID of the Basic Film Box and the Action Type ID of 1.

The affected SOP Instance UID received with N-CREATE-RSP message will be kept internally and used for later requests (e.g. N-ACTION-RQ) on the Basic Film Box – see below:

Attribute Name	Tag	Source of Information
Requested SOP Instance UID	(0000,1000) -> (0000,1001)	Affected SOP Instance UID of N-CREATE-RSP on Basic Film Box

The Basic Film Box SOP class interprets the following status codes:

Service Status	Meaning	Error Codes
Failure	Unable to create print job, print queue is full Image size is larger than images box size	C601 C603
Warning	Film box does not contain image box (empty page) Requested MinDensity or MaxDensity outside of Printer's operating range	B603 B605
Success	Film accepted for printing	0000

#### 15.1.2.1.3.3 Basic Grayscale Image Box SOP Class

The Basic Grayscale Image Box information object definition is the presentation of an image and image related data in the image area of a film. The Basic Image Box information describes the presentation parameters and image pixel data that apply to a single image of a sheet of film.

The Grayscale Image Box SOP Class uses only the N-SET-RQ with the following attributes:

Attribute Name	Tag	Usage SCU	Supported Values
Image Position	(2020,0010)	M	Depending on display format
Basic Grayscale Image Sequence		(2020,0110)	M
> Samples per Pixel	(0028,0002)	M	1
> Photometric Interpretation	(0028,0004)	M	MONOCHROME2
> Rows	(0028,0010)	M	
> Columns	(0028,0011)	M	
> Pixel Aspect Ratio	(0028,0034)	M	1\1
> Bits Allocated	(0028,0100)	M	8
> Bits Stored	(0028,0101)	M	8
> High Bit	(0028,0102)	M	7
> Pixel Representation	(0028,0103)	M	0
> Pixel Data	(7FE0,0010)	M	

The Grayscale Image Box SOP class interprets the following status codes:

Service Status	Meaning	Error Codes
Failure	Image contains more pixel than printer can print in Image Box Insufficient memory in printer to store the image	C603 C605
Warning	Requested Min Density or Max Density outside of Printer's operating range	B605
Success		0000

#### 15.1.2.1.3.4 Printer SOP Class

The Printer SOP Class is the possibility to monitor the status of the hardcopy printer in a synchronous and an asynchronous way.

The FLUOROSPOT Compact DICOM print manager can directly ask the Printer (SCP) for its status or receive events from the Printer asynchronously:

- N-GET as SCU
- N-EVENT-REPORT as SCU

In both cases the following information is supported:

Used Printer N-EVENT Report attributes

Event-type Name	Event	Attributes	Tag	Usage SCU
Normal	1			
Warning	2	Printer Status Info	(2110,0020)	U
Failure	3	Printer Status Info	(2110,0020)	U

Mandatory Printer N-GET-RSP, N-EVENT-REPORT-RQ attributes

Attribute Name	Tag	Usage SCP	Supported Values
Printer Status	(2110,0010)	M	NORMAL, FAILURE, WARNING
Printer Status Info	(2110,0020)	M	

#### 15.1..1.4 SOP specific Conformance Statement – Print Presentation LUT SOP class

The objective of the Presentation LUT is to realize image hardcopy printing tailored for specific modalities, applications and user preferences.

The output of the Presentation LUT is Presentation Values (P-Values). P-Values are approximately related to human perceptual response. They are intended to facilitate common input for hardcopy. P-Values are intended to be independent of the specific class or characteristics of the hardcopy device.

The FLUOROSPOT Compact DICOM print management SCU supports the following DIMSE Service elements for the Print Presentation LUT SOP Class as SCU:

- N-CREATE
- N-DELETE

The Print Presentation LUT SOP Class N-CREATE-RQ (SCU) uses the following attributes:

Attribute Name	Tag	Usage SCU	Supported Values
Presentation LUT Shape	(2050,0020)	U	IDENTITY

#### 15.1.3 Association Acceptance Policy

Not applicable

# 16 Communication Profiles

## 16.1 Supported Communication Stacks

The FLUOROSPOT Compact DICOM application provides DICOM TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

- **TCP/IP Stack**

The FLUOROSPOT Compact DICOM application uses the TCP/IP stack from the underlying Operating System upon which it executes. It uses the MergeCOM-3 subroutine library from Merge Technologies Inc. that is based on a Berkeley socket interface.

### 16.1..1 API

The FLUOROSPOT Compact DICOM application uses the MergeCOM library that is based on a TCP/IP socket interface.

### 16.1..2 Physical Media Support

The FLUOROSPOT Compact DICOM application is indifferent to the physical medium over which TCP/ IP executes; it inherits this from the underlying operating system upon which it executes.

# 17 Extensions / Specializations / Privatizations

## 17.1.1 Standard Extended / Specialized / Private SOPs

Please refer to Appendix A.1 and A.3 for all information on these topics. A detailed overview is given there.

## 17.1.2 Private Transfer Syntaxes

Not applicable

# 18 Configuration

## 18.1 AE Title/Presentation Address Mapping

To ensure unique identification within the network the hostname could be used as part of the AE Titles (see examples below, hostname = FLC1). The string can be up to 16 characters long and must not contain any extended characters, only 7-bit ASCII characters (excluding Control Characters) are allowed according to DICOM Standard.

- **DICOM Verification**

The Verification Service uses the AE configuration of the DICOM Service that is checked with the C-ECHO message. E.g. Verification will use the Storage SCU AE, if initiated to check the configuration of a remote DICOM storage provider.

- **DICOM Storage SCP AE Title**

The DICOM Storage Provider application provides an application entity title that can be configured via Service UI; this AE Title must be different from the Storage SCU AE Title:  
e.g. FLC\_STORE SCP

There are 2 fixed listening port numbers used:

- 104 for unencrypted communication
- 2762 for TLS-based encrypted communication.

- **DICOM Storage SCU AE Title**

The DICOM Storage User application provides an application entity title that can be configured via Service UI:

e.g. FLC\_STORE\_SCU

For each Storage destination the AET, Port and IP-Address can be configured.

- **DICOM Query/Retrieve AE Title**

The DICOM Query/Retrieve application provides an own application entity title that can be configured via Service UI:

e.g. FLC\_QR\_SCU

For each Query/Retrieve provider the AET, Port and IP-Address can be configured.

- **DICOM Storage Commitment AE Title**

The DICOM Storage Commitment application provides the application entity title that can be configured via Service UI:

e.g. FLC\_STC\_SCU

There are 2 fixed listening port numbers used:

- 104 for unencrypted communication
- 2762 for TLS-based encrypted communication.

For each Storage Commitment provider the AET, Port and IP-Address can be configured.

- **DICOM Worklist AE Title**

The DICOM Worklist application provides an application entity title that can be configured via Service UI:  
e.g. FLC\_WK\_SCU

For one Worklist provider the AET, Port and IP-Address can be configured.

- **DICOM MPPS AE Title**

The DICOM Modality Performed Procedure Step application uses the same application entity title as configured for the DICOM Worklist application 18.1.6.

For one MPPS provider the AET, Port and IP-Address can be configured.

- **DICOM Print AE Title**

The DICOM Print application provides the application entity title:  
e.g. FLC\_PRINT\_SCU

For each Print provider the AET, Port and IP-Address can be configured

## 18.2 Configurable Parameters

The Application Entity Titles, host names and port numbers are configured using the Siemens Healthineers FLUOROSPOT Compact Service/Installation Tool. For each AET the list of services supported can be configured.

- **Storage and Query/Retrieve**

The Siemens Healthineers FLUOROSPOT Compact Service/Installation Tool can be used to set the AETs, port-numbers, host-names, IP-addresses and capabilities for the remote nodes (SCPs).

Storage:

(General Settings)

- Retry delay for failed jobs (range: 5 to 180 minutes, 0 means no retry). Maximum 5 retries will occur (per AE target).
- Usage of Pixel Spacing in addition to Imager Pixel Spacing
- 1 image per series (default data model of FLUOROSPOT Compact) or n images per series / Singleframe per series (all single frame images, belonging to the same procedure step, are in the same series)
- Downscaling of Fluoroloops

(Node specific settings)

- Usage of Ready Window. Ready Window means, that windowing is calculated into the pixel data. Bit depth is not changed.
- Ready Processing (burn in of shutter, overlay, zoom and pan) for XRF IODs
- Archive flag
- Associated Storage Commitment Server

- **Storage Commitment**  
One destination (AET, Port and IP) can be configured for Storage Commitment requests.
  - A timeout (1-10s) for a Storage Commitment result within the same association can be defined.
  - A Storage Commitment timeout and the number of retries could be configured in User Settings.
- **Modality Worklist**  
One destination (AET, Port and IP) can be configured for Worklist requests.
  - Time-out waiting for a response from the worklist provider (Query waiting time: maximum 999 s).
  - Maximum of worklist entries received (Max. Query Match: maximum 999).
  - Time between automated worklist queries (0 .. 1440 min.; 0 min means no automatic query) and used modality (0008,0060) in worklist query.
  - Usage of own AET in worklist query (Query by AET)
  - Time range used in worklist query
- **Modality Performed Procedure Step**  
One destination (AET, Port and IP) can be configured for MPPS data. The "MPPS Complete" method can be selected in the user configuration.
- **Print**  
The Siemens Healthineers FLUOROSPOT Compact Service/Installation Tool can be used to configure the SCPs (DICOM-Printers). The AET, host-name, IP-address and port-number can be set.  
Print: Retry delay for failed jobs (range: 5 to 180 minutes, 0 means no retry). Maximum 5 retries will occur.
  - Usage of Print Presentation LUT

### 18.3 Default Parameters

This installation tool also uses some default parameters:

- max. PDU size set to 65536 Bytes (64 KB)
- time-out for accepting/rejecting an association request: 60 s
- time-out for responding to an association open/close request: 60 s
- time-out for accepting a message over network: 60 s
- time-out for waiting for data between TCP/IP-packets: 60 s
- timeout for a network connect: 15s

The time-outs for waiting for a Service Request/Response message from the remote node are as follows:

- for Storage SCU: 30 s for single frame images, 600 s for multiframe
- for Storage SCP: 600 s
- for Query/Retrieve SCP/SCU: 600 s
- for Print Management SCU:
- time-out for Response to N-SET-RQ: 25 s
- time-out for Response to other Requests: 25 s

# 19 Support of Extended Character Sets

The Siemens Healthineers FLUOROSPOT Compact DICOM application supports the

- ISO\_IR 100 (ISO 8859-1:1987 Latin Alphabet No. 1 supplementary set)
- GB18030 (used when Chinese patient data input is configured)
- ISO 2022 IR 13, ISO 2022 IR 87 and ISO 2022 IR 159 (used when Japanese patient data input is configured)
- ISO\_IR 144 (used when Russian patient data input is configured)

# 20 Security

## 20.1 Security Profiles

- **Secure Transport Connection Profiles**

FLC supports the “Basic TLS Secure Transport Connection Profile” as specified in [DICOM].

The following TLS versions are supported: 1.2.

The cypher suites that are provided by the underlying Operating System are used.

The private key and certificate used by an FLC based system and Certificate Authorities (CA) to validate certificates received from remote nodes are stored in the local Certificate Store of the underlying Operating System.

The listening port used for the DICOM services is 2762.

# Application Profile Conformance Statement

This chapter contains the Conformance Statement to all “Offline Media Application Profiles (incl. private extensions)” supported by the FLUOROSPOT Compact archive options.

Those application profiles supported shall be:

- General Purpose CDR
- General Purpose DVD
- General Purpose USB

# 1 Introduction

## 1.1 Purpose

This DICOM Conformance Statement describes the DICOM offline interface for Siemens Healthineers X-ray acquisition modalities based on the FLUOROSPOT Compact imaging system according to part PS 3.2 of [DICOM].

## 1.2 Scope

The FLUOROSPOT Compact DICOM offline media storage service implementation acts as FSC, FSU and/or FSR for the specified application profiles and the related SOP Class instances.

## 1.3 Definitions, Terms and Abbreviations

ACR	American College of Radiology
AE	DICOM Application Entity
ASCII	American Standard Code for Information Interchange <sup>1</sup>
CR	Computed Radiography
DCS	DICOM Conformance Statement
DICOM	Digital Imaging and Communications in Medicine
DIMSE	DICOM Message Service Element
DIMSE-C	DICOM Message Service Element with Composite information objects
	FSC
	File Set Creator
FSR	File Set Reader
FSU	File Set Updater
GSDF	Grayscale Standard Display Function
IOD	DICOM Information Object Definition
FD	Flat panel detector
ISO	International Standard Organization
MOD	Magneto-optical Disk
NEMA	National Electrical Manufacturers Association
	O
	Optional Key Attribute
PDU	DICOM Protocol Data Unit
R	Required Key Attribute
RWA	Real-World Activity
U	Unique Key Attribute
XA	X-Ray Angiography
XRF	X-Ray Radiofluoroscopy

## 1.4 References

[DICOM] Digital Imaging and Communications in Medicine (DICOM), NEMA PS 3.1-20, 2014.

## 1.5 Remarks

DICOM, by itself, does not guarantee interoperability. However, the Conformance Statement facilitates a first-level validation for interoperability between different applications supporting the same DICOM functionality as SCU and SCP, respectively.

This Conformance Statement is not intended to replace validation with other DICOM equipment to ensure proper exchange of information intended.

The scope of this Conformance Statement is to facilitate communication with Siemens Healthineers and other vendors' Medical equipment. The Conformance Statement should be read and understood in conjunction with the DICOM Standard [DICOM]. It is, however, by itself not guaranteed to ensure the desired interoperability and a successful interconnectivity.

The user should be aware of the following important issues:

- The comparison of different conformance statements is the first step towards assessing interconnectivity between Siemens Healthineers and non-Siemens Healthineers equipment.
- Test procedures should be defined and tests should be performed by the user to validate the connectivity desired. DICOM itself and the conformance parts do not specify this.
- The standard will evolve to meet the users' future requirements. Siemens Healthineers is actively involved in developing the standard further and therefore reserves the right to make changes to its products or to discontinue its delivery.
- Siemens Healthineers reserves the right to modify the design and specifications contained herein without prior notice. Please contact your local Siemens Healthineers representative for the most recent product information.

## 2 Implementation Model

### 2.1 Application Data Flow Diagram

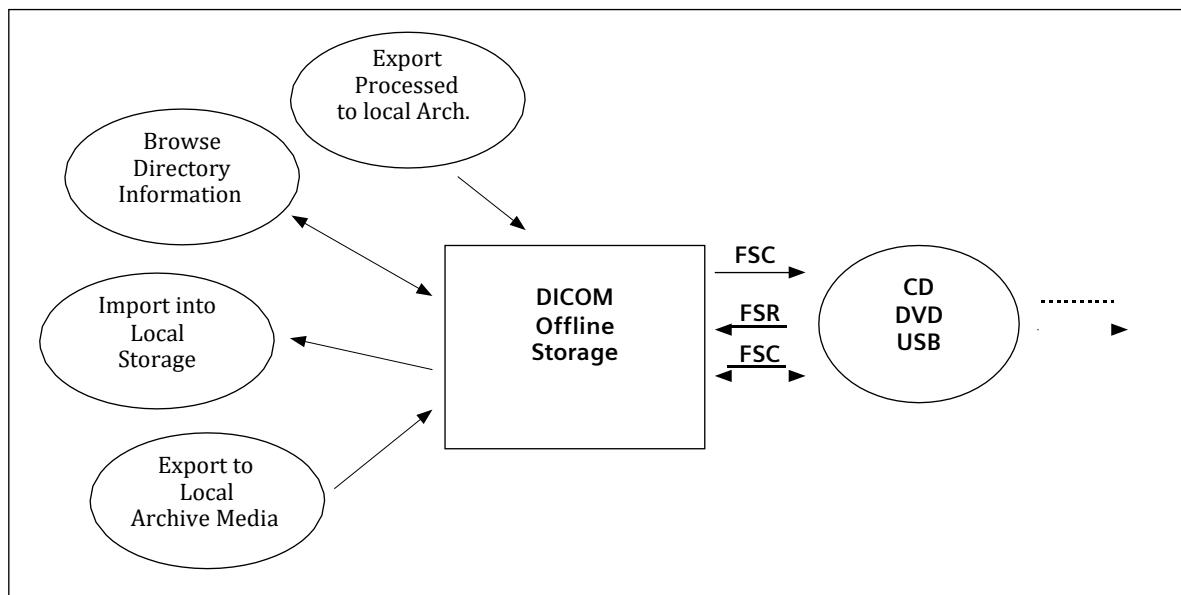


Figure 8: FLUOROSPOT Compact DICOM Media Storage Implementation Model

The DICOM Offline Storage application will serve as an interface to the CD, DVD and USB offline medium. It serves interfaces to include the offline media directory into the browser and to copy SOP instances to a medium or retrieve SOP Instances from medium into local storage.

The DICOM Offline Storage application will support the 120 mm CD-R/DVD-R/DVD+R medium and USB connected removable storage devices.

The FSU role will update new SOP Instances only to media with pre-existing File-sets conforming to the Application Profiles supported.

The contents of the DICOMDIR will be temporarily stored in a local Archive-Database.

### 2.2 Functional Definitions of Aes

The FLUOROSPOT Compact DICOM offline media storage application consists of the DICOM application entity serving all interfaces to access offline media. The DICOM Offline Storage application is capable of

1. creating a new File-set onto an unwritten medium (Export to...),
2. updating an existing File-set by writing new SOP Instances onto the medium (Export to...),
3. importing SOP Instances from the medium onto local storage,
4. reading the File-sets DICOMDIR information into temporary database and pass it to display applications,
5. writing/updating File-Sets with processed image export.

### 2.3 Sequencing of Real-World Activities

Not applicable.

## 2.4 File Meta Information Options

Implementation Class UID	1.3.12.2.1107.5.3.4
Implementation Version Name	Siemens_FLC_80

# 3 AE Specifications

## 3.1 DICOM Archive Specification

The DICOM Offline Storage provides Standard conformance to Media Storage Service Class (Interchange Option). Details are listed in following Table:

Application Profiles Supported	Real-World Activity	Role	SC Option
STD-GEN-CD	Browse Directory Information	FSR	Interchange
	Import into local Storage	FSR	Interchange
	Export to local Archive Media	FSC, FSU	Interchange
STD-GEN-DVD	Browse Directory Information	FSR	Interchange
	Import into local Storage	FSR	Interchange
	Export to local Archive Media	FSC, FSU	Interchange
STD-GEN-USB12	Browse Directory Information	FSR	Interchange
	Import into local Storage	FSR	Interchange
	Export to local Archive Media	FSC, FSU	Interchange

### 3.1.1 File Meta Information for the Application Entity

The Source Application Entity Title is set by configuration. See Chapter "Configuration" for details.

### 3.1.2 Real-World Activities for this Application Entity

#### 3.1.2.1 Real-World Activity: Browse Directory Information

The DICOM Offline Storage application acts as FSR using the interchange option when requested to read the media directory.

The DICOM Offline Storage application will read the DICOMDIR and insert those directory entries valid for the application profiles supported into a local database. The database can then be used for browsing media contents.

During operation no "Attribute Value Precedence" is applied to the SOP Instances. Detached Patient Management is not supported.

##### 3.1.2.1.1 Application Profiles for the RWA: Browse Directory Information

See Table in section 3.1 for the Application Profiles listed that invoke this Application Entity for the Browse Directory Information RWA.

#### 3.1.2.2 Real-World Activity: Import into local Storage

The DICOM Offline Storage application acts as FSR using the interchange option when requested to read SOP Instances from the medium into the local storage.

The SOP Instance selected from the media directory will be copied into the local storage. Only SOP Instances, that are valid for the application profile supported, can be retrieved from media storage. This is due to the fact that the Browse Directory Information will filter all SOP Instances not matching the Application profiles supported.

During operation no "Attribute Value Precedence" is applied to the SOP Instances. Detached Patient Management is not supported.

Compressed Transfer Syntaxes are not supported by the DICOM Offline Storage application neither as FSR/FSC or FSU.

For media conforming to the STD-GEN-CD, STD-GEN-DVD and STD-GEN-USB profiles the following SOP classes will be supported as an FSR:

Information Object Definition	SOP Class UID	Transfer Syntax UID
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian 1.2.840.10008.1.2.1
X-Ray Radiation Dose SR	1.2.840.10008.5.1.4.1.1.88.67	Explicit VR Little Endian 1.2.840.10008.1.2.1
X-Ray Angiographic Image Storage	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR Little Endian 1.2.840.10008.1.2.1
X-Ray Radiofluoroscopic Image Storage (only for Fluorosystems)	1.2.840.10008.5.1.4.1.1.12.2	Explicit VR Little Endian 1.2.840.10008.1.2.1
Computed Radiography Image Storage (only for Radsystems)	1.2.840.10008.5.1.4.1.1.1	Explicit VR Little Endian 1.2.840.10008.1.2.1
Digital X-Ray Image Storage – For Presentation	1.2.840.10008.5.1.4.1.1.1.1	Explicit VR Little Endian 1.2.840.10008.1.2.1
Multi-frame True Color Secondary Capture Image Storage (only for Uroskop Omnia)	1.2.840.10008.5.1.4.1.1.7.4	Explicit VR Little Endian 1.2.840.10008.1.2.1
CT Image Storage	1.2.840.10008.5.1.4.1.1.2	Explicit VR Little Endian 1.2.840.10008.1.2.1
MR Image Storage	1.2.840.10008.5.1.4.1.1.4	Explicit VR Little Endian 1.2.840.10008.1.2.1
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Little Endian 1.2.840.10008.1.2.1
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Little Endian 1.2.840.10008.1.2.1

### 3.1.2.2.1 Application Profiles for the RWA: Import into local Storage

See Table in section 3.1 for the Application Profiles listed that invoke this Application Entity for the Import into Local Storage RWA.

### 3.1.2.3 Real-World Activity: Export to local Archive Media

The DICOM Offline Storage application acts as FSU (for media with existing DICOM file-set) or FSC (media not initialized) using the interchange option when requested to copy SOP Instances from the local storage to local Archive Medium.

The DICOM Archive application will receive a list of marked SOP Instances to be copied to the local archive medium. According to the state of the medium inserted (new medium, Medium with DICOM file-set) the validity of the SOP Instances according to the applicable profile is checked. Only valid SOP Instances are accepted. The user can select if "all" images are marked or the marks from image acquisition will be retained.

The user can choose the IOD format of the instances (CR or DX for RAD images). For XRF it can be configured to store images "ready processed".

The DICOM archive application will not close the medium in case of CD-R/DVD-R or DVD+R.

Conforming to the STD-GEN-CD, STD-GEN-DVD and STD-GEN-USB profiles the following SOP classes will be supported as an FSC and FSU:

Information Object Definition	SOP Class UID	Transfer Syntax UID
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Explicit VR LittleEndian 1.2.840.10008.1.2.1
X-Ray Radiation Dose SR	1.2.840.10008.5.1.4.1.1.88.67	Explicit VR LittleEndian 1.2.840.10008.1.2.1
X-Ray Radiofluoroscopic Image Storage (only for Fluorosystems)	1.2.840.10008.5.1.4.1.1.12.2	Explicit VR LittleEndian 1.2.840.10008.1.2.1
X-Ray Angiographic Image Storage (only for Multitom Rax)	1.2.840.10008.5.1.4.1.1.12.1	Explicit VR LittleEndian 1.2.840.10008.1.2.1
Computed Radiography Image Storage (only for Radsystems)	1.2.840.10008.5.1.4.1.1.1	Explicit VR LittleEndian 1.2.840.10008.1.2.1
Digital X-Ray Image Storage For Presentation (only for Radsystems)	1.2.840.10008.5.1.4.1.1.1.1	Explicit VR LittleEndian 1.2.840.10008.1.2.1
Multi-frame True Color Secondary Capture Image Storage (only for Uroskop Omnia)	1.2.840.10008.5.1.4.1.1.7.4	Explicit VR LittleEndian 1.2.840.10008.1.2.1

#### 3.1.2.3.1 Application Profiles for the RWA: Export to local Archive Media

See Table in section 3.1 for the Application Profiles listed that invoke this Application Entity for the Export to local Archive Media RWA.

## 4 Augmented and Private Profiles

### 4.1 Augmented Application Profiles

Not applicable.

### 4.2 Private Application Profiles

Not applicable.

## 5 Extensions, Specializations and Privatizations of SOP Classes and Transfer Syntaxes

The SOP Classes listed refer in majority to those created by the equipment to which this Conformance Statement is related to. For SOP classes not listed in this section, please refer to the Storage section of the DICOM Conformance Statement of the product. This will include all SOP Instances that can be received and displayed and therefore will be included into offline media storage even though these SOP Instances are not created by the equipment serving the Media Storage Service.

Restrictions and Extensions <sup>1</sup>:

1. It is possible to store images on offline media in a “packed” manner. This will result in a DICOMDIR containing “PRIVATE” records instead of “IMAGE” records. This functionality is not intended as exchange media with other systems.
2. The user can choose the IOD format of the instances. RAD images can be stored in CR or DX IOD. For XRF it can be configured to store images “ready processed”.
3. It is possible to update an offline media containing unpacked images with packed images and vice versa.

<sup>1</sup> Not valid for Mobilett Mira as Mira does not store images in a “packed” manner.

## 5.1 SOP Specific Conformance Statement for Basic Directory

### 5.1.1 Extension, Specialization for "Packed CD"

The PRIVATE directory records will be identified according to the information in the following table<sup>1</sup>:

Attribute	Tag	Value used
Private Record UID	(0004,1432)	1.3.12.2.1107.5.3.4.1

### 5.1.2 DICOMDIR Attributes

The DICOMDIR file will contain the following attributes for the levels Patient – Study – Series – Image – Private

Attribute Name	Tag	Type	Notes
<b>File Set Identification</b>			
File-set ID	(0004,1130)	2	Volume label of media
<b>Directory Information</b>			
Offset of the First Directory Record of the Root Directory Entry	(0004,1200)	1	
Offset of the Last Directory Record of the Root Directory Entry	(0004,1202)	1	
File-set Consistency Flag	(0004,1212)	1	0000h
Directory Record Sequence	(0004,1220)	2	
> Offset of Next Directory Record	(0004,1400)	1C	
> Record In-use Flag	(0004,1410)	1C	FFFFh
> Offset of Referenced Lower-Level Directory Entity	(0004,1420)	1C	
> Directory Record Type	(0004,1430)	1C	PATIENT, STUDY, SERIES, IMAGE, PRIVATE
> Private Record UID	(0004,1432)	1C	For PRIVATE Directory Record
> Referenced File ID	(0004,1500)	1C	Filename and path, only for IMAGE Directory Records
> Referenced SOP Class UID in File	(0004,1510)	1C	For IMAGE Directory Records
> Referenced SOP Instance UID in File	(0004,1511)	1C	For IMAGE Directory Records
> Referenced Transfer Syntax UID in File	(0004,1512)	1C	For IMAGE Directory Records
> Record Selection Keys	See below	n. a.	
<b>Patient Keys</b>			
Specific Character Set	(0008,0005)	1C	
Patient's Name	(0010,0010)	2	
Patient ID	(0010,0020)	1	
Date of Birth	(0010,0030)	3	
Patient's Sex	(0010,0040)	3	
<b>Study Keys</b>			
Specific Character Set	(0008,0005)	1C	
Study Date	(0008,0020)	1	

<sup>1</sup> Not valid for Mobilett Mira.

Attribute Name	Tag	Type	Notes
Study Time	(0008,0030)	1	
Accession Number	(0008,0050)	2	
Study Description	(0008,1030)	2	
Study Instance UID	(0020,000D)	1C	
Study ID	(0020,0010)	1	
<b>Series Keys</b>			<b>Directory Record Type SERIES</b>
Specific Character Set	(0008,0005)	1C	
Series Date	(0008,0021)	3	
Series Time	(0008,0031)	3	
Modality	(0008,0060)	1	
Institution Name	(0008,0080)	3	
Institution Address	(0008,0081)	3	Zero length
Series Description	(0008,103E)	3	
Performing Physician	(0008,1050)	3	
Series Instance UID	(0020,000E)	1	
Series Number	(0020,0011)	1	
<b>Image Keys</b>			<b>Directory Record Type IMAGE</b>
Image Type	(0008,0008)		
Instance Number	(0020,0013)	1	
Number of Frames	(0028,0008)	3	
Icon Image Sequence	(0088,0200)	3	
>Samples per Pixel	(0028,0002)	1C	1
>Photometric Interpretation	(0028,0004)	1C	MONOCHROME2
>Rows	(0028,0010)	1C	128
>Columns	(0028,0011)	1C	128
>Bits Allocated	(0028,0100)	1C	8
>Bits Stored	(0028,0101)	1C	8
>High Bit	(0028,0102)	1C	7
>Pixel Representation	(0028,0103)	1C	0000H
>Pixel Data	(7FE0,0010)	1C	Icon Image
<b>SR Document Keys</b>			<b>Directory Record Type "SR DOCUMENT"</b>
Specific Character Set	(0008,0005)	1C	
Instance Number	(0020,0013)	1	
Completion Flag	0040,A491	1	"COMPLETE"
Verification Flag	0040,A493	1	"UNVERIFIED"
Content Date	0008,0023	1	
Content Time	0008,0033	1	
Concept Name Code Sequence	0040,A043	1	
Content Sequence	0040,A730	1C	
>Relationship Type	0040,A010	1	HAS CONCEPT MOD
>Value Type	0040,A040	1	
>Concept Name Code Sequence	0040,A043	1C	

Attribute Name	Tag	Type	Notes
>> Code Value	0008,0010	1	
>> Coding Scheme Designator	0008,0102	1	
>> Code Meaning	0008,0104	1	
>Concept Code Sequence	0040,A168	1C	
>> Code Value	0008,0010	1	113704
>> Coding Scheme Designator	0008,0012	1	DCM
>> Code Meaning	0008,0014	1	Projection X-Ray
>Content Sequence	0040,A730	1C	
>>Relationship Type	0040,A010	1	HAS CONCEPT MOD
>>Value Type	0040,A040	1	CODE
>> Concept Name Code Sequence	0040,A043	1C	
>>> Code Value	0008,0010	1	G-C0E8
>>> Coding Scheme Designator	0008,0012	1	SRT
>>> Code Meaning	0008,0014	1	Has Intent
>>Concept Code Sequence	0040,A168	1C	
>>> Code Value	0008,0010	1	R-408C3
>>> Coding Scheme Designator	0008,0012	1	SRT
>>> Code Meaning	0008,0014	1	Diagnostic Intent
<b>Private Keys</b>			<b>Directory Record Type PRIVATE</b>
Private Record UID	(0004,1432)	1	
Instance Number	(0020,0013)	1	

# 6 Configuration

## 6.1 AE Title Mapping

### 6.1.1 DICOM Media Storage AE Title

The DICOM Storage application provides the application entity title:

Not applicable.

## 7 Support of Extended Character Sets

The Siemens Healthineers FLUOROSPOT Compact DICOM application supports the

- ISO\_IR 100 (ISO 8859-1:1987 Latin Alphabet No. 1 supplementary set)
- ISO\_IR 144 (Cyrillic, supplementary set of ISO 8859, used when Russian patient data input is configured)
- GB18030 (used when Chinese patient data input is configured)
- ISO 2022 IR 13, ISO 2022 IR 87 and ISO 2022 IR 159 (used when Japanese patient data input is configured)

# A APPENDIX

## A.1 Common attributes in image and structured report IODs

Overview of supplied attributes – all type of images and structured report

Attribute Name	Tag	Value
Specific Character Set	(0008,0005)	dependent on system configuration
Image Type	(0008,0008)	see A.7
SOP Class UID	(0008,0016)	dependent on the SOP Class (see chapter 5.1)
SOP Instance UID	(0008,0018)	System generated
Study Date	(0008,0020)	<yyymmdd>
Series Date	(0008,0021)	<yyymmdd>
Acquisition Date	(0008,0022)	<yyymmdd>
Content Date	(0008,0023)	<yyymmdd>
Study Time	(0008,0030)	<hhmmss>
Series Time	(0008,0031)	<hhmmss>
Acquisition Time	(0008,0032)	<hhmmss>
Image Time	(0008,0033)	<hhmmss>
Accession Number	(0008,0050)	From Modality Worklist or user input
Manufacturer	(0008,0070)	SIEMENS
Institution Name	(0008,0080)	From Configuration or value from Worklist
Referring Physician's Name	(0008,0090)	From Modality Worklist or user input
Station Name	(0008,1010)	From Configuration (hostname)
Study Description	(0008,1030)	Requested Procedure Description (0032,1060) from Modality Worklist or first Exam Name of the study
Procedure Code Sequence	(0008,1032)	(0032,1064) from Modality Worklist
> Code Value	(0008,0100)	(0032,1064) from Modality Worklist
> Coding Scheme Designator	(0008,0102)	(0032,1064) from Modality Worklist
> Coding Scheme Version	(0008,0103)	(0032,1064) from Modality Worklist
> Code Meaning	(0008,0104)	(0032,1064) from Modality Worklist
Series Description	(0008,103E)	Organ program name
Institutional Department Name	(0008,1040)	
Performing Physician's Name	(0008,1050)	Physician field in Study list, default (0040,0006) from Modality Worklist or user input
Operator's Name	(0008,1070)	User input
Admitting Diagnoses Description	(0008,1080)	From Modality Worklist or user input
Manufacturer's Model Name	(0008,1090)	For Dose Reports: the name of the system (e.g. "Multitom_Rax"). Otherwise: "Fluorospot Compact FD"
Referenced Study Sequence	(0008,1110)	From Modality Worklist
> Referenced SOP Class UID	(0008,1150)	From Modality Worklist
> Referenced SOP Instance UID	(0008,1155)	From Modality Worklist

Attribute Name	Tag	Value
Referenced Patient Sequence	(0008,1120)	From Modality Worklist
> Referenced SOP Class UID	(0008,1150)	From Modality Worklist
> Referenced SOP Instance UID	(0008,1155)	From Modality Worklist
Referenced Performed Procedure Step Sequence		(0008,1111) Only with MPPS
> Referenced SOP Class UID	(0008,1150)	1.2.840.10008.3.1.2.3.3 (MPPS)
> Referenced SOP Instance UID	(0008,1155)	Reference to MPPS
Derivation Description 1)	(0008,2111)	
Irradiation Event UID 1) 2)	(0008,3010)	
Patient's Name	(0010,0010)	From Modality Worklist or user input (all 5 components)
Patient ID	(0010,0020)	From Modality Worklist or user input
Patient's Birth Date	(0010,0030)	From Modality Worklist or user input
Patient's Sex	(0010,0040)	From Modality Worklist or user input
Other Patient IDs	(0010,1000)	From Modality Worklist
Other Patient Names	(0010,1001)	From Modality Worklist
Patient's Age	(0010,1010)	
Patient's Size	(0010,1020)	From Modality Worklist or user input
Patient's Weight	(0010,1030)	From Modality Worklist or user input
Ethnic Group	(0010,2160)	From Modality Worklist
Additional Patient History	(0010,21B0)	From Modality Worklist
Patient Comments	(0010,4000)	From Modality Worklist
Device Serial Number	(0018,1000)	From Configuration
Software Version	(0018,1020)	From Configuration
Protocol Name <sup>1</sup>	(0018,1030)	Organ program name
Date of Last Calibration <sup>1,2</sup>	(0018,1200)	
Study Instance UID	(0020,000D)	From Modality Worklist or generated by FLC
Series Instance UID	(0020,000E)	System generated
Study ID	(0020,0010)	User Input, default (0040,1001) from Modality Worklist
Series Number	(0020,0011)	System generated
Acquisition Number <sup>1,2</sup>	(0020,0012)	System generated
Instance Number	(0020,0013)	System generated
Patient Orientation <sup>1,2</sup>	(0020,0020)	not available
Laterality <sup>1,2</sup>	(0020,0060)	Zero length if not available, not sent when <ul style="list-style-type: none"> <li>• Laterality is "both" or "unpaired"</li> <li>• "Series Model" configuration is used</li> <li>• In case of DX image</li> </ul>
Image Comments <sup>1,2</sup>	(0020,4000)	From user input
Burned in Annotations 1)	(0028,0301)	"NO"

<sup>1</sup>Not used for X-Ray Radiation Dose SR SOP Class<sup>2</sup>Not used for EXAM Protocol as Secondary Capture Image

Attribute Name	Tag	Value
Window Center <sup>1</sup>	(0028,1050)	Dependent on user input and bit depth
Window Width <sup>1</sup>	(0028,1051)	Dependent on user input and bit depth
Lossy Image Compression <sup>1</sup>	(0028,2110)	"00"
Study Comments	(0032,4000)	User Input
Current Patient Location	(0038,0300)	From Modality Worklist
Performed Procedure Step Start Date	(0040,0244)	= (0008,0020)
Performed Procedure Step Start Time	(0040,0245)	= (0008,0030)
Performed Procedure Step ID	(0040,0253)	Internally created
Performed Procedure Step Description	(0040,0254)	= (0008,1030)
Performed Protocol Code Sequence <sup>1,2</sup>	(0040,0260)	Only present, if usage of protocol codes is configured in user settings
> Code Value	(0008,0100)	
> Coding Scheme Designator	(0008,0102)	
> Coding Scheme Version	(0008,0103)	
> Code Meaning	(0008,0104)	
Request Attributes Sequence	(0040,0275)	
>Scheduled Procedure Step Description	(0040,0007)	From Modality Worklist
> Scheduled Protocol Code Sequence	(0040,0008)	From Modality Worklist
>> Code Value	(0008,0100)	From Modality Worklist
>> Coding Scheme Designator	(0008,0102)	From Modality Worklist
>> Coding Scheme Version	(0008,0103)	From Modality Worklist
>> Code Meaning	(0008,0104)	From Modality Worklist
>Scheduled Procedure Step ID	(0040,0009)	From Modality Worklist
>Requested Procedure ID	(0040,1001)	From Modality Worklist or user input
>Reason for the Requested Procedure <sup>3</sup>	(0040,1002)	From Modality Worklist
Presentation LUT Shape <sup>1</sup>	(2050,0020)	"IDENTITY"
Pixel Data <sup>1</sup>	(7FE0,0010)	System generated

<sup>1</sup> Not used for X-Ray Radiation Dose SR SOP Class

<sup>2</sup> Not used for EXAM Protocol as Secondary Capture Image

<sup>3</sup> Only stored for X-Ray Radiation Dose SR SOP Class

## A.2 Multiframe True Color SC Storage Standard Extended SOP Class

Overview of specific attributes – Multiframe True Color SC image

Attribute Name	Tag	Value
Modality	(0008,0060)	"OT"
Conversion Type	(0008,0064)	"WSD"
Series Description	(0008,103E)	not present
Derivation Description	(0008,2111)	not present
Body Part Examined	(0018,0015)	
Protocol Name	(0018,1030)	not present
Samples per Pixel	(0028,0002)	3
Photometric Interpretation	(0028,0004)	RGB
Planar Configuration	(0028,0006)	0
Number of Frames	(0028,0008)	1 for CAP images ≥1 for Medical Report
Frame Increment Pointer	(0028,0009)	(0018,1063)
Rows	(0028,0010)	
Columns	(0028,0011)	
Bits Allocated	(0028,0100)	8
Bits Stored	(0028,0101)	8
High Bit	(0028,0102)	7
Pixel Representation	(0028,0103)	0000H
Pixel Intensity Relationship	(0028,1040)	"LIN"
Window Center	(0028,1050)	not present
Window Width	(0028,1051)	not present

**A.3 XRF Standard Extended SOP Class****A.3.1 configuration "XRF – Full"**

Overview of specific attributes – XRF image (Full)

Attribute Name	Tag	Value
Modality	(0008,0060)	"RF"
Recommended Display Frame Rate	(0008,2144)	Only for multi-frame
Contrast/Bolus Agent	(0018,0010)	Contrast or Bolus Agent as predefined in the OGP or entered by the user
Body Part Examined	(0018,0015)	- value from OGP - Zero length if not available
KVP	(0018,0060)	From Acquisition
Device Serial Number	(0018,1000)	From Configuration
Frame Time	(0018,1063)	Only if multi-frame
Distance Source to Detector	(0018,1110)	SID from Acquisition (not available for all devices)
Distance Source to Patient	(0018,1111)	SOD (only if table object distance, image receptor table distance and SID are available)
Estimated Radiographic Magnification Factor	(0018,1114)	only if SID and SOD are available (=SID/SOD)
Table Motion	(0018,1134)	"STATIC" (not available for all devices)
Table Angle	(0018,1138)	From Acquisition (not available for all devices)
Table Type	(0018,113A)	"TILTING" (not available for all devices)
Field of View Shape	(0018,1147)	"RECTANGLE"
Field of View Dimension(s)	(0018,1149)	rows\columns in mm
Exposure Time	(0018,1150)	From Acquisition, not available for all devices (e.g. Mobilett Mira)
Exposure	(0018,1152)	From Acquisition
Exposure in $\mu$ As	(0018,1153)	From Acquisition
X-Ray Tube Current	(0018,1151)	From Acquisition, not available for all devices
Filter Type	(0018,1160)	Defined Terms: "NONE", "CU_0.1_MM", "CU_0.2_MM", "CU_0.3_MM"
Average Pulse Width	(0018,1154)	From Acquisition
Radiation Setting	(0018,1155)	GR or SC
Radiation Mode	(0018,115A)	PULSED
Image Area Dose Product	(0018,115E)	If dose measurement is available, in $\text{dGy cm}^2$ . Only present for acquisitions, not for fluoro.
Imager Pixel Spacing	(0018,1164)	For detector images: "0.148\0.148" or "0.296\0.296" or "0.444\0.444" For II: images: zoomed image size/1024
Grid	(0018,1166)	"NONE", "FOCUSED", "UNIVERSAL"

Attribute Name	Tag	Value
Date of Last Calibration	(0018,1200)	
Acquisition Device Processing Description	(0018,1400)	Description of processing parameter set
Acquisition Device Processing Code	(0018,1401)	Code to identify processing parameter set
Relative X-Ray Exposure	(0018,1405)	Exposure Index (EXI)
Column Angulation	(0018,1450)	From Acquisition
Tomo Layer Height	(0018,1460)	only for tomo acquisitions
Tomo Angle	(0018,1470)	only for tomo acquisitions
Tomo Time	(0018,1480)	only for tomo acquisitions
Positioner Type	(0018,1508)	device dependent
Shutter Shape	(0018,1600)	RECTANGULAR for detector images CIRCULAR for I.I images
Shutter Left Vertical Edge	(0018,1602)	From Acquisition / Post-processing
Shutter Right Vertical Edge	(0018,1604)	From Acquisition / Post-processing
Shutter Upper Horizontal Edge	(0018,1606)	From Acquisition / Post-processing
Shutter Lower Horizontal Edge	(0018,1608)	From Acquisition / Post-processing
Center of Circular Shutter	(0018,1610)	for I.I images
Radius of Circular Shutter	(0018,1612)	for I.I images
Patient Position	(0018,5100)	As predefined in OGP or entered by the user, otherwise empty.
View Position	(0018,5101)	Only available if entered by the user, otherwise empty.
Sensitivity	(0018,6000)	
Detector Conditions Nominal Flag	(0018,7000)	Set to "NO" if user was notified about possible image quality compromise, e.g. because detector temperature is out of range
Detector Temperature	(0018,7001)	the last received detector temperature
Detector Type	(0018,7004)	"SCINTILLATOR"
Detector Description	(0018,7006)	only for detector images Detector data: - Factory Serial Number - Factory Date Code - Configuration Version - Factory Part Number
Detector ID	(0018,700A)	Factory Serial Number
Date of Last Detector Calibration	(0018,700C)	<yyyymmdd>
Time of Last Detector Calibration	(0018,700E)	<hhmmss>
Field of View Origin	(0018,7030)	
Field of View Rotation	(0018,7032)	Enumerated Values: "0", "90", "180", "270"
Field of View Horizontal Flip	(0018,7034)	Enumerated Values: "NO", "YES"
Grid Focal Distance	(0018,704C)	"1150", "1250", "1500", "1800", "3000" only if Grid (0018,1166) = FOCUSED"
X-Ray Tube Current in $\mu$ A	(0018,8151)	From Acquisition, not available for all devices
Samples per Pixel	(0028,0002)	1

Attribute Name	Tag	Value
Photometric Interpretation	(0028,0004)	MONOCHROME2
Number of Frames	(0028,0008)	
Frame Increment Pointer	(0028,0009)	<0018,1063> only if multi-frame
Rows	(0028,0010)	Depending on detector and acquisition mode
Columns	(0028,0011)	Depending on detector and acquisition mode
Pixel Spacing	(0028,0030)	only if configured in service value = 0018,1164 if no SOD is present otherwise: value = (0018,1164) * SOD / SID
Bits Allocated	(0028,0100)	8, 16 depending on user selection
Bits Stored	(0028,0101)	8, 16 depending on user selection
High Bit	(0028,0102)	7, 15 depending on user selection
Pixel Representation	(0028,0103)	0
Pixel Intensity Relationship	(0028,1040)	LIN or LOG
Recommended Viewing Mode	(0028,1090)	NAT, SUB, only if multi-frame and (0028,1040) = LOG
Window Center & Width Explanation	(0028,1055)	only if (0028,1040) = LOG
Modality LUT Sequence	(0028,3000)	No Modality LUT is written even in case the Pixel Intensity Relationship is set to "LOG"
> LUT Descriptor	(0028,3002)	
> Modality LUT Type	(0028,3004)	
> LUT Data	(0028,3006)	
Representative Frame Number	(0028,6010)	Only if multi-frame
Mask Subtraction Sequence	(0028,6100)	Only if multi-frame and (0028,1040) = LOG
> Mask Operation	(0028,6101)	"AVG_SUB"
> Mask Frame Numbers	(0028,6110)	Mask Frame Number (always only one mask)
> Mask Sub-pixel Shift	(0028,6114)	From Post-processing

### A.3.2 configuration "XRF – Ready Processed"

Overview of specific attributes – XRF image (Ready Processed)

Attribute Name	Tag	Value
Modality	(0008,0060)	"RF"
Recommended Display Frame Rate	(0008,2144)	Only for multi-frame
Contrast/Bolus Agent	(0018,0010)	Contrast or Bolus Agent as predefined in the OGP or entered by the user
Body Part Examined	(0018,0015)	- value from OGP - Zero length if not available
KVP	(0018,0060)	From Acquisition
Device Serial Number	(0018,1000)	From Configuration
Frame Time	(0018,1063)	Only if multi-frame
Distance Source to Detector	(0018,1110)	SID from Acquisition (not available for all devices)
Distance Source to Patient	(0018,1111)	SOD (only if table object distance, image receptor table distance and SID are available)
Estimated Radiographic Magnification Factor	(0018,1114)	only if SID and SOD are available (=SID/SOD)
Table Motion	(0018,1134)	"STATIC"
Table Angle	(0018,1138)	From Acquisition
Table Type	(0018,113A)	"TILTING"
Field of View Shape	(0018,1147)	"RECTANGLE"
Field of View Dimension(s)	(0018,1149)	rows\columns in mm
Exposure Time	(0018,1150)	From Acquisition, not available for all devices (e.g. Mobilett Mira)
Exposure	(0018,1152)	From Acquisition
Exposure in $\mu$ As	(0018,1153)	From Acquisition
X-Ray Tube Current	(0018,1151)	From Acquisition, not available for all devices
Filter Type	(0018,1160)	Defined Terms: "NONE","CU_0.1_MM","CU_0.2_MM", "CU_0.3_MM"
Average Pulse Width	(0018,1154)	From Acquisition
Radiation Setting	(0018,1155)	GR or SC
Radiation Mode	(0018,115A)	PULSED
Image Area Dose Product	(0018,115E)	If dose measurement is available, in $\text{dGy cm}^2$ . Only present for acquisitions, not for fluoro.
Imager Pixel Spacing	(0018,1164)	For detector images: "0.148\0.148" or "0.296\0.296" or "0.444\0.444" For I.I: images: zoomed image size/1024
Grid	(0018,1166)	"NONE","FOCUSED","UNIVERSAL"
Date of Last Calibration	(0018,1200)	
Acquisition Device Processing Description	(0018,1400)	Description of processing parameter set

Attribute Name	Tag	Value
Acquisition Device Processing Code	(0018,1401)	Code to identify processing parameter set
Relative X-Ray Exposure	(0018,1405)	Exposure Index (EXI)
Column Angulation	(0018,1450)	From Acquisition
Tomo Layer Height	(0018,1460)	only for tomo acquisitions
Tomo Angle	(0018,1470)	only for tomo acquisitions
Tomo Time	(0018,1480)	only for tomo acquisitions
Positioner Type	(0018,1508)	device dependent
Shutter Shape	(0018,1600)	RECTANGULAR for detector images CIRCULAR for I.I images
Shutter Left Vertical Edge	(0018,1602)	From Acquisition / Post-processing
Shutter Right Vertical Edge	(0018,1604)	From Acquisition / Post-processing
Shutter Upper Horizontal Edge	(0018,1606)	From Acquisition / Post-processing
Shutter Lower Horizontal Edge	(0018,1608)	From Acquisition / Post-processing
Center of Circular Shutter	(0018,1610)	for I.I images
Radius of Circular Shutter	(0018,1612)	for I.I images
Patient Position	(0018,5100)	As predefined in OGP or entered by the user, otherwise empty.
View Position	(0018,5101)	Only available if entered by the user, otherwise empty.
Sensitivity	(0018,6000)	
Detector Conditions Nominal Flag	(0018,7000)	Set to "NO" if user was notified about possible image quality compromise, e.g. because detector temperature is out of range
Detector Temperature	(0018,7001)	the last received detector temperature
Detector Type	(0018,7004)	"SCINTILLATOR"
Detector Description	(0018,7006)	only for detector images Detector data: - Factory Serial Number - Factory Date Code - Configuration Version - Factory Part Number
Detector ID	(0018,700A)	Factory Serial Number
Date of Last Detector Calibration	(0018,700C)	<yyyymmdd>
Time of Last Detector Calibration	(0018,700E)	<hhmmss>
Field of View Origin	(0018,7030)	
Field of View Rotation	(0018,7032)	Enumerated Values: "0", "90", "180", "270"
Field of View Horizontal Flip	(0018,7034)	Enumerated Values: "NO", "YES"
Grid Focal Distance	(0018,704C)	"1150", "1250", "1500", "1800", "3000" only if Grid (0018,1166) = FOCUSED"
X-Ray Tube Current in $\mu$ A	(0018,8151)	From Acquisition, not available for all devices
Samples per Pixel	(0028,0002)	1
Photometric Interpretation	(0028,0004)	
Number of Frames	(0028,0008)	

Attribute Name	Tag	Value
Frame Increment Pointer	(0028,0009)	<0018,1063> only if multi-frame
Rows	(0028,0010)	Depending on detector and acquisition mode
Columns	(0028,0011)	Depending on detector and acquisition mode
Pixel Spacing	(0028,0030)	only if configured in service value = 0018,1164 if no SOD is present otherwise: value = (0018,1164) * SOD / SID
Bits Allocated	(0028,0100)	8, 16 depending on user selection
Bits Stored	(0028,0101)	8, 16 depending on user selection
High Bit	(0028,0102)	7, 15 depending on user selection
Pixel Representation	(0028,0103)	0
Pixel Intensity Relationship	(0028,1040)	LIN
Representative Frame Number	(0028,6010)	Only if multi-frame

**A.4 CR,DX Standard Extended SOP Class**

Overview of supplied attributes –CR/DX image

Attribute Name	Tag	Value
Modality	(0008,0060)	“CR” or “DX”
Presentation Intent Type	(0008,0068)	“FOR PRESENTATION” (DX only)
Anatomic Region Sequence	(0008,2218)	Empty (DX only)
Contrast/Bolus Agent	(0018,0010)	Contrast or Bolus Agent as predefined in the OGP or entered by the user
Body Part Examined	(0018,0015)	- value from OGP - Zero length if not available
KVP	(0018,0060)	From Acquisition
Device Serial Number	(0018,1000)	From Configuration
Software Version	(0018,1020)	From Configuration
Protocol Name	(0018,1030)	Organ Program Name
Distance Source to Detector	(0018,1110)	SID from Acquisition (not available for all devices)
Distance Source to Patient	(0018,1111)	- SOD (only if table object distance, image receptor table distance and SID are available) - not available for free exposures
Table Angle	(0018,1138)	From Acquisition
Table Type	(0018,113A)	“TILTING”
Field of View Shape	(0018,1147)	“RECTANGLE”
Field of View Dimension(s)	(0018,1149)	rows\columns in mm
Exposure Time	(0018,1150)	From Acquisition, not available for all devices (e.g. Mobilett Mira)
Exposure	(0018,1152)	From Acquisition
Exposure in $\mu$ As	(0018,1153)	From Acquisition
X-Ray Tube Current	(0018,1151)	From Acquisition, not available for all devices
Filter Type	(0018,1160)	Defined Terms: “NONE”, “CU_0.1_MM”, “CU_0.2_MM”, “CU_0.3_MM”
Radiation Setting	(0018,1155)	GR
Radiation Mode	(0018,115A)	PULSED
Image Area Dose Product	(0018,115E)	Only if dose measurement is available, in $\text{dGy cm}^2$ .
Imager Pixel Spacing	(0018,1164)	detector dependent
Grid	(0018,1166)	“NONE”, “FOCUSED”, “UNIVERSAL” Always “NONE” for Multix Fusion Max
Acquisition Device Processing Description	(0018,1400)	Description of processing parameter set
Acquisition Device Processing Code	(0018,1401)	Code to identify processing parameter set
Relative X-Ray Exposure	(0018,1405)	Exposure Index (EXI)
Exposure Index	(0018,1411)	Clinical EXI value, (in units of 100.uGy)
Target Exposure Index	(0018,1412)	From OGP, (in units of 100.uGy)
Deviation Index	(0018,1413)	

Attribute Name	Tag	Value
Column Angulation	(0018,1450)	From Acquisition
Tomo Layer Height	(0018,1460)	only for tomo acquisitions
Tomo Angle	(0018,1470)	only for tomo acquisitions
Tomo Time	(0018,1480)	only for tomo acquisitions
Positioner Type	(0018,1508)	device dependent
Patient Position	(0018,5100)	As predefined in OGP or entered by the user, otherwise empty.
View Position	(0018,5101)	Only available if entered by the user
Sensitivity	(0018,6000)	
Detector Conditions Nominal Flag	(0018,7000)	Set to "NO" if user was notified about possible image quality compromise, e.g. because detector temperature is out of range
Detector Temperature	(0018,7001)	the last received detector temperature
Detector Type	(0018,7004)	"SCINTILLATOR"
Detector Description	(0018,7006)	only for detector images Detector data: - Factory Serial Number - Factory Date Code - Configuration Version - Factory Part Number
Detector ID	(0018,700A)	Factory Serial Number
Date of Last Detector Calibration	(0018,700C)	<yyyymmdd>
Time of Last Detector Calibration	(0018,700E)	<hhmmss>
Field of View Origin	(0018,7030)	
Field of View Rotation	(0018,7032)	Enumerated Values: "0", "90", "180", "270"
Field of View Horizontal Flip	(0018,7034)	Enumerated Values: "NO", "YES"
Grid Focal Distance	(0018,704C)	"1150", "1250", "1500", "1800", "3000" only if Grid (0018,1166) = FOCUSED not available for Multix Fusion Max
X-Ray Tube Current in $\mu$ A	(0018,8151)	From Acquisition, not available for all devices
Image Laterality	(0020,0062)	User Input / Organ Program /empty (DX only)
Samples per Pixel	(0028,0002)	1
Photometric Interpretation	(0028,0004)	MONOCHROME2
Rows	(0028,0010)	Depending on detector and acquisition mode
Columns	(0028,0011)	Depending on detector and acquisition mode
Pixel Spacing	(0028,0030)	only if configured in service value = 0018,1164 if no SOD is present otherwise: value = (0018,1164) * SOD / SID
Bits Allocated	(0028,0100)	16
Bits Stored	(0028,0101)	16
High Bit	(0028,0102)	15

Attribute Name	Tag	Value
Pixel Representation	(0028,0103)	0
Pixel Intensity Relationship	(0028,1040)	DISP or LIN for DX
Rescale Intercept	(0028,1052)	0 (DXonly)
Rescale Slope	(0028,1053)	1 (DXonly)
Rescale Type	(0028,1054)	US (DXonly)
Pixel Intensity Relationship Sign	(0028,1041)	“-1” for “Bones White” “1” for “Bones Black” Remark: DX only, uncertainty of LUT handling

### A.5 X-Ray Radiation Dose SR SOP Class

The FLUOROSPOT Compact will create X-Ray Radiation Dose SRs implementing TID 10001 Projection X-Ray Radiation Dose as indicated in Figure 9.

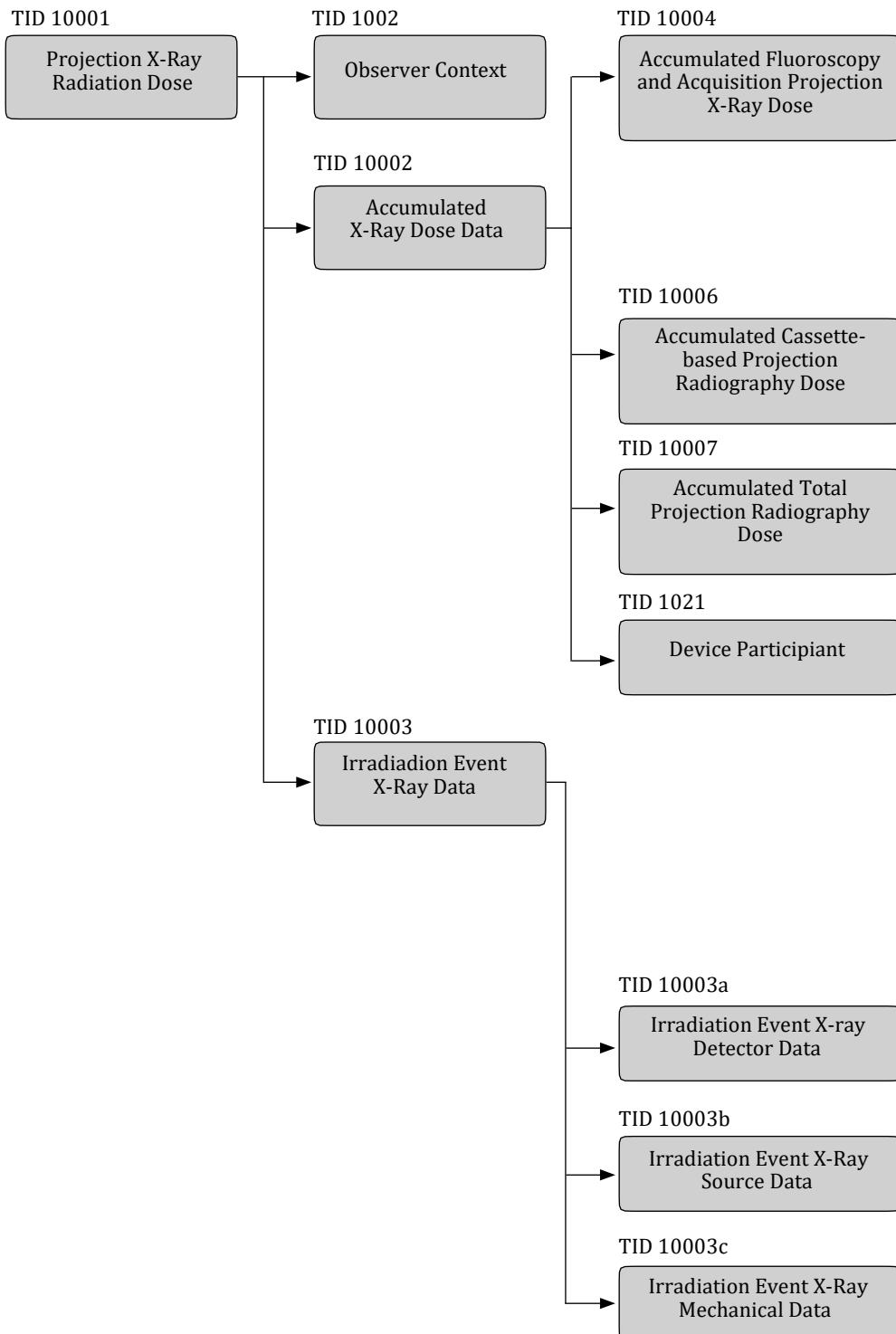


Figure 9: X-Ray Radiation Dose SR IOD Template Structure

The scope of accumulation is "Study" or "Performed Procedure Step" if MPPS is configured.

The report is kept open (Completion Flag (0040,A491) = "PARTIAL") to support continuation of the study. Closing a study will set the Completion Flag (0040,A491) to "COMPLETED". Updating of a SR will not change the SOP Instance UID.

All patient level, study level and equipment information is taken from the acquired images. The related attribute mapping from Modality Worklist to X-Ray Radiation Dose SR is equal to the mapping into acquired images. Please refer to A.10 FLUOROSPOT Compact Worklist Identifier for mapping of worklist information.

#### A.5.1 Projection X-Ray Radiation Dose (TID 10001)

NL	Relation with Parent	VT	Concept Name	Value
		CONTAINER	EV (113701, DCM, "X-Ray Radiation Dose Report")	Root node.
>	HAS CONCEPT MOD	CODE	EV (121058, DCM, "Procedure reported")	DT (113704, DCM, "Projection X-Ray")
>>	HAS CONCEPT MOD	CODE	EV (G-C0E8, SRT, "Has Intent")	EV (R-408C3, SRT, "Diagnostic Intent")
<b>DTID 1002 "Observer Context"</b>				
>	HAS OBS CONTEXT	CODE	EV (121005, DCM, "Observer Type")	EV (121007, DCM, "Device")
>	HAS OBS CONTEXT	UIDREF	EV (121012, DCM, "Device Observer UID")	Root UID including serial number
>	HAS OBS CONTEXT	TEXT	EV (121013, DCM, "Device Observer Name")	Value from Station Name (0008,1010)
>	HAS OBS CONTEXT	TEXT	EV (121014, DCM, "Device Observer Manufacturer")	Value from Manufacturer (0008,0070)
>	HAS OBS CONTEXT	TEXT	EV (121015, DCM, "Device Observer Model Name")	Value from Manufacturer's Model Name (0008,1090)
>	HAS OBS CONTEXT	TEXT	EV (121016, DCM, "Device Observer Serial Number")	Value from Device Serial Number (0018,1000)
>>	HAS PROPERTIES	UIDREF	EV (113705, DCM, "Scope of Accumulation") or EV (110180, DCM, "Study Instance UID")	EV (113016, DCM, "Performed Procedure Step") or EV (113014, DCM, "Study") Study Instance UID or MPPS UID
>	CONTAINS	INCLUDE	DTID 10002 "Accumulated X-Ray Dose" See A.5.2	
>	CONTAINS	INCLUDE	DTID 10003 "Irradiation Event X-Ray For each Irradiation Data" See A.5.3	
>	CONTAINS	CODE	EV (113854, DCM, "Source of Dose Information")	EV("A-2C090", SRT, "Dosimeter")
>	CONTAINS	TEXT	EV (121106, DCM, "Comment")	Formatted report.

## A.5.2 Accumulated X-Ray Dose (TID 10002)

NL	Relation with Parent VT	Concept Name	Value
	CONTAINER	EV (113702, DCM, „Accumulated X-Ray Dose Data“)	
>	HAS CONCEPT MOD	CODE	EV (113764, DCM, „Acquisition Plane“)
>	CONTAINS	CONTAINER	EV (122505, DCM, “Calibration”)
>>	HAS CONCEPT MOD	CODE	EV (113794, DCM, “Dose Measurement Device”)
>>	CONTAINS	DATETIME	EV (113723, DCM, “Calibration Date”)
>>	CONTAINS	NUM	EV (122322, DCM, “Calibration Factor”)
>>	CONTAINS	NUM	EV (113763, DCM, “Calibration Uncertainty”)
>>	CONTAINS	TEXT	EV (113724, DCM, “Calibration Responsible Party”)
>>	CONTAINS	TEXT	EV (113720, DCM, “Calibration Protocol”)
>	CONTAINS	NUM	EV (113722, DCM, “Dose Area Product Total”)
>	CONTAINS	NUM	EV (113725, DCM, “Dose (RP) Total”)
>	CONTAINS	NUM	EV (113726, DCM, “Fluoro Dose Area Product Total”)
>	CONTAINS	NUM	EV (113728, DCM, “Fluoro Dose (RP) Total”) <sup>1</sup>
>	CONTAINS	NUM	EV (113730, DCM, “Total Fluoro Time”) <sup>1</sup>
>	CONTAINS	NUM	EV (113727, DCM, “Acquisition Dose Area Product Total”)
>	CONTAINS	NUM	EV (113729, DCM, “Acquisition Dose (RP) Total”)
>	CONTAINS	CODE	EV (113780, DCM, “Reference Point Definition”)
>	CONTAINS	NUM	EV (113863, DCM, “30cm above table”)
			Luminos Agile:
			EV (113862, DCM, “1cm above table”)
			Ysio:
			EV (1000, SMS_XP, “Skin Focus Distance Configuration”)
>	CONTAINS	NUM	EV (113855, DCM, “Total Acquisition Time”)
			Accumulated acquisition time.

<sup>1</sup> Only present in case of at least one FLUORO event.

## A.5.3 Irradiation Event X-Ray Data (TID 10003)

NL	Relation with Parent VT	Concept Name	Value
>	CONTAINS	CONTAINER	EV (113706, DCM, "Irradiation Event X-Ray Data")
>>	HAS CONCEPT MOD	CODE	EV (113764, DCM, "Acquisition Plane")
>>	CONTAINS	DATETIME	DT (111526,DC,"DateTime Started")
>>	CONTAINS	CODE	EV (113721, DCM, "Irradiation Event Type")
>>	CONTAINS	TEXT	EV (125203, DCM, "Acquisition Protocol")
>>	CONTAINS	CODE	EV (113745, DCM, "PatientTable Relationship")
>>	CONTAINS	CODE	EV (113743, DCM, "Patient Orientation")
>>>	HAS CONCEPT MOD	CODE	EV (113744, DCM, "Patient Orientation Modifier")
>>	CONTAINS	CODE	EV (113780, DCM, " Reference Point Definition")
>>	CONTAINS	UIDREF	EV (113769, DCM, "Irradiation Event UID")
>>	CONTAINS	NUM	EV (122130, DCM, "Dose Area Product")
>>	CONTAINS	NUM	EV (113845, DCM, "Exposure Index")
>>	CONTAINS	NUM	EV (113846, DCM, "Target Exposure Index")
>>	CONTAINS	NUM	EV (113847, DCM, "Deviation Index")
>>	CONTAINS	NUM	EV (113738, DCM, "Dose (RP)")
>>	CONTAINS	NUM	Entrance Dose for this irradiation event.
>>	CONTAINS	NUM	Value from Column Angulation (0018,1450)
>>	CONTAINS	CONTAINER	EV (113771, DCM, "X-Ray Filters")
>>>	CONTAINS	CODE	EV (113772, DCM, "X-Ray Filter Type")
>>>	CONTAINS	CODE	EV (111609, DCM, "No Filter") or EV (113650, DCM, "Strip filter")
>>>	CONTAINS	CODE	EV (C-127F9,SRT,"Copper or Copper compound")
>>>	CONTAINS	NUM	EV (113758, DCM, "X-Ray Filter Thickness Minimum") <sup>1</sup>
>>>	CONTAINS	NUM	EV (113773, DCM, "X-Ray Filter Thickness Maximum") <sup>1</sup>
>>	CONTAINS	CODE	EV (113732, DCM, "Fluoro Mode") <sup>2</sup>
>>	CONTAINS	NUM	EV (113631, DCM, "Pulsed")
>>	CONTAINS	NUM	EV (113791, DCM, "Pulse Rate") <sup>2</sup>
			FLUORO frame rate.

>>	CONTAINS	NUM	EV (113768, DCM, "Number of Pulses")
>>	CONTAINS	NUM	EV (113733, DCM, "KVP")

<sup>1</sup>Only if filter was used.

<sup>2</sup>Only for FLUORO irradiation events

NL	Relation with Parent	VT	Concept Name	Value
>>	CONTAINS	NUM	EV (113734, DCM, "X-Ray Tube Current")	
>>	CONTAINS	NUM	EV (113824, DCM, "Exposure Time")	
>>	CONTAINS	NUM	EV (113793, DCM, "Pulse Width")	
>>	CONTAINS	NUM	EV (113736, DCM, "Exposure")	
>>	CONTAINS	NUM	EV (113766, DCM, "Focal Spot Size")	
>>	CONTAINS	NUM	EV (113790, DCM, "Collimated Field Area")	
>>	CONTAINS	NUM	EV (113788, DCM, "Collimated Field Height")	
>>	CONTAINS	NUM	EV (113789, DCM, "Collimated Field Width")	
>>	CONTAINS	NUM	EV (113742, DCM, „Irradiation Duration“)	
>>	CONTAINS	NUM	EV (113750, DCM, „Distance Source to Detector“)	
>>	CONTAINS	CODE	EV (123014, DCM, „Target Region“)	Filled with entry from CID 4031 "Common Anatomic Regions" - part 16 of [DICOM]
>>	CONTAINS	CODE	EV (113876, DCM, „Device Role in Procedure“)	EV (113859, DCM, „Irradiating Device“)
>>>	HAS PROPERTIES	TEXT	EV (113878, DCM, „Device Manufacturer“)	SIEMENS
>>>	HAS PROPERTIES	TEXT	EV (113879, DCM, „Device Model Name“)	Fluorospot Compact FD
>>>	HAS PROPERTIES	TEXT	EV (113880, DCM, „Device Serial Number“)	Value from Device Serial Number (0018,1000).
>>>	HAS PROPERTIES	UIDREF	EV (121012, DCM, „Device Observer UID“)	Root UID including serial number
>>	CONTAINS	IMAGE	EV (113795, DCM, "Acquired Image")	Reference to the image (but only in case an image was created for this event). Not present for FLUOROS.
>>	CONTAINS	NUM	EV(113737, DCM, "Distance Source to Reference Point")	According to the definition of the Reference Point.
>>	CONTAINS	NUM	EV(113751, DCM, "Table Longitudinal Position")	Only for table acquisitions. <sup>1</sup>
>>	CONTAINS	NUM	EV(113752, DCM, "Table Lateral Position")	Only for table acquisitions. <sup>1</sup>
>>	CONTAINS	NUM	EV(113753, DCM, "Table Height Position")	Only for table acquisitions. <sup>1</sup>
>>	CONTAINS	NUM	EV(113759, DCM, "Table Longitudinal End Position")	Only for table acquisitions. <sup>1</sup>
>>	CONTAINS	NUM	EV(113760, DCM, "Table Lateral End Position")	Only for table acquisitions. <sup>1</sup>
>>	CONTAINS	NUM	EV(113761, DCM, "Table Height End Position")	Only for table acquisitions. <sup>1</sup>

<sup>1</sup> Only for LUMINOS Impulse

#### A.6 EXAM Protocol as Secondary Capture Image

The FLUOROSPOT Compact will create a secondary capture image containing the "Exam Protocol" as image. The Secondary Capture Image Storage Service Class (1.2.840.10008.5.1.4.1.1.7) is used for this.

The value for Modality (0008,0060) depends on the images, acquired for this procedure step. If only RAD images are present, "CR" is used. If DFR images are present "RF" is used.

Overview of supplied attributes –SC image

Attribute Name	Tag	Value
Modality	(0008,0060)	"CR" or "RF"
Conversion Type	(0008,0064)	"DI"
Rows	(0028,0010)	2100
Columns	(0028,0011)	2970
Bits Allocated	(0028,0100)	16
Bits Stored	(0028,0101)	12
High Bit	(0028,0102)	11

Additionally the following private attributes are used for documentation of accumulated dose values:

Tag	Private Owner Code	Name	VR	VM
(0017,xxD0)	SIEMENS_FLCOMPACT_VA01A_PROC	Accumulated fluorotime in sec	US	1
(0017,xxD1)	SIEMENS_FLCOMPACT_VA01A_PROC	Accumulated DAP in dGy*cm*cm	LO	1
(0017,xxD2)	SIEMENS_FLCOMPACT_VA01A_PROC	Accumulated Skin Dose in mGy	LO	1

The scope of accumulation is the Procedure Step.

#### A.7 XA Standard Extended SOP Class

Overview of specific attributes – XA image

Attribute Name	Tag	Value
Image Type	(0008,0008)	ORIGINAL\PRIMARY\ SINGLE PLANE\3DSCAN\3DRAX
Distance Source to Detector	(0018,1110)	Contains the SID.
Distance Source to Patient	(0018,1111)	Contains the SOD.
Estimated Radiographic Magnification Factor	(0018,1114)	Ratio of SID over SOD.
Intensifier Size	(0018,1162)	480
Positioner Motion	(0018,1500)	STATIC if the imaging table moves during a multi-frame acquisition, but the X-Ray positioner does not move. Otherwise DYNAMIC.
Positioner Primary Angle	(0018,1510)	0
Positioner Secondary Angle	(0018,1511)	0
Positioner Primary Angle Increment	(0018,1520)	(0,...,0) for each frame
Positioner Secondary Angle Increment	(0018,1521)	(0,...,0) for each frame
Detector Primary Angle	(0018,1530)	Not contained
Detector Secondary Angle	(0018,1531)	Not contained
Collimator Left Vertical Edge	(0018,1702)	According to initial collimator settings.
Collimator Right Vertical Edge	(0018,1704)	According to initial collimator settings.
Collimator Upper Horizontal Edge	(0018,1706)	According to initial collimator settings.
Collimator Lower Horizontal Edge	(0018,1708)	According to initial collimator settings.
Patient Position	(0018,5100)	Always set as the Patient Position is needed for reconstruction to set the Patient Orientation.

#### Private Creator "SIEMENS SMS-AX ACQ 1.0"

Please refer to A.9.1

### A.8 SOP Common Module – Image Type Extensions

Overview of settings in tag Image Type (0008,0008)

Type of Scene / Image	Image Type
Standard DFR Acquisition Image (Single and Scene)	ORIGINAL\PRIMARY\SINGLE PLANE
Fluoro Single Image	ORIGINAL\PRIMARY\SINGLE PLANE\FLUORO\SINGLE
Fluoro Scene	ORIGINAL\PRIMARY\SINGLE PLANE\FLUORO\LOOP
DSA Single Image	ORIGINAL\PRIMARY\SING LE PLANE\STORE MONITOR\DSA
DSA Peak Opacification	ORIGINAL\PRIMARY\SINGLE PLANE\STORE MONITOR\PEAKOP
DSA Series	ORIGINAL\PRIMARY\SINGLE PLANE\DSA
Store Monitor Average	DERIVED\SECONDARY\SINGLE PLANE\SUM
Standard Roadmap Series	ORIGINAL\PRIMARY\SIN GLE PLANE\ROADMAP\LOOP
Roadmap Single Image	ORIGINAL\PRIMARY\SIN
Fluoro Image Last Image Hold	GLE PLANE\ROADMAP\SINGLE
Twinview/Quadview Image	ORIGINAL\PRIMARY\SIN GLE PLANE\FLUORO\LIH
Tomography Image	DERIVED\SECONDARY\SINGLE PLANE\COMBI
Tomography Image (Image Intensifier)	ORIGINAL\PRIMARY\TOMO
PLANE\TOMO Scan Mode Image	ORIGINAL\PRIMARY\SINGLE
PLANE\SCAN	ORIGINAL\PRIMARY\SINGLE
Radiography Image	ORIGINAL\PRIMARY\RAD
Tilting Ortho Mode Image	ORIGINAL\PRIMARY\ORTHO\TILTING
Exam Protocol as Secondary Capture	DERIVED\SECONDARY\SINGLE PLANE\EXAM
PROTOCOL Composed Image	DERIVED\SECONDARY\algorithm\"version"
Videomanager Image	ORIGINAL\PRIMARY\SINGLE PLANE\CAP
Store Monitor of Standard Single Image	DERIVED\SECONDARY\SINGLE PLANE\STORE
MONITOR Multitom Rax Projection Images	ORIGINAL\PRIMARY\SINGLE PLANE\3DSCAN\3DRAX

### A.9 Private Data

#### A.9.1 General Private Data

The following tables describe private data that may be used for the FLC generated IODs.

Private Creator "SIEMENS\_FLCOMPACT\_VA01A\_PROC"

Tag	Name	VR	VM
(0017,xx0A)	Blackening Correction	SS	1
(0017,xx0B)	Dose Level	SS	1
(0017,xx0C)	SDM	SS	1
(0017,xx0D)	Frame Rate	FL	1
(0017,xx0E)	Characteristic Curve	LO	1
(0017,xx0F)	Acquisition Filter	LO	1
(0017,xx10)	Skin Dose	SS	1
(0017,xx11)	Focus	SS	1

Tag	Name	VR	VM
(0017,xx14)	Native Edge Enhancement Percent Gain	US	1
(0017,xx16)	Bone Level Black White	US	1
(0017,xx17)	Contrast Native	US	1
(0017,xx18)	Brightness Native	US	1
(0017,xx19)	ShutterX	US	1
(0017,xx1A)	ShutterXY	US	1
(0017,xx1B)	FlipH	US	1
(0017,xx1C)	FlipV	US	1
(0017,xx1E)	Zoom	US	1
(0017,xx1F)	PanX	SS	1
(0017,xx20)	PanY	SS	1
(0017,xx21)	Harmonization	US	1
(0017,xx22)	Mask Frame Number	US	1
(0017,xx23)	DSA Viewing Mode	US	1
(0017,xx24)	DSA Contrast Subtracted	US	1
(0017,xx25)	DSA Brightness Subtracted	US	1
(0017,xx26)	Subtracted Edge Enhancement Percent Gain	US	1
(0017,xx27)	Anatomical Background Percent	US	1
(0017,xx28)	Pixel Shift X	FL	1
(0017,xx29)	Pixel Shift Y	FL	1
(0017,xx48)	Series Number	SS	1
(0017,xx49)	Label	SS	1
(0017,xx4A)	Label Position	SS	1
(0017,xx4B)	Acquisition Number String	SS	1
(0017,xx4C)	Modification Number String	LO	1
(0017,xx4D)	Single Series Flag	SS	1
(0017,xx4E)	FlcSeriesDate	LO	1
(0017,xx4F)	FlcSeriesTime	LO	1
(0017,xx50)	Image Type	SS	1
(0017,xx51)	Calibration Factor	FL	1
(0017,xx52)	Edge Enhancement Kernel Native	SS	1
(0017,xx53)	Edge Enhancement Kernel Subtracted	SS	1
(0017,xx54)	Harmonization Kernel	SS	1
(0017,xx55)	DSA Flag	SS	1
(0017,xx5A)	Overlay List V1	OW	1
(0017,xx5B)	Overlay List V2	OW	1
(0017,xx5C)	Overlay List V3	OW	1
(0017,xx64)	Native Edge Enhancement Percent Gain Acq.	FL	1
(0017,xx66)	Bone Level Black White Acq.	US	1
(0017,xx67)	Contrast Native Acq.	US	1
(0017,xx68)	Brightness Native Acq.	US	1
(0017,xx85)	Harmonization Acq.	US	1
(0017,xx86)	Mask Frame Number Acq.	US	1

Tag	Name	VR	VM
(0017,xx87)	DSA Viewing Mode Acq.	US	1
(0017,xx88)	DSA Contrast Subtracted Acq.	US	1
(0017,xx89)	DSA Brightness Subtracted Acq.	US	1
(0017,xx8A)	Subtracted Edge Enhancement Percent Gain Acq.	US	1
(0017,xx8B)	Anatomical Background Percent Acq.	US	1
(0017,xx8C)	Pixel Shift X Acq.	FL	1
(0017,xx8D)	Pixel Shift Y Acq.	FL	1
(0017,xx8E)	Shutter X Acq.	US	1
(0017,xx8F)	Shutter Y Acq.	US	1
(0017,xx90)	Internal Data	US	1
(0017,xx91)	Internal Data	US	1
(0017,xx92)	Internal Data	US	1
(0017,xx93)	Internal Data	US	1
(0017,xxA0)	Laterality	US	1
(0017,xxA1)	Label1 Flag	US	1
(0017,xxA2)	Position Label 1	LO	1
(0017,xxAA)	System Type	US	1
(0017,xxAB)	Internal Data	US	1
(0017,xxB0)	Detector Calibration Temperature	DS	1
(0017,xxBF)	name of the performed OGP	LO	1
(0017,xxC0)	name of the performed Exam	LO	1
(0017,xxC1)	Internal Data	US	1
(0017,xxC2)	Internal Data	US	1
(0017,xxC3)	Stored marking number	US	1
(0017,xxC4)	"Name for RIS" of the performed exam	LO	1

## Private Creator "Thorax/Multix FD Lab Settings"

Tag	Name	VR	VM
(0019,xx06)	Table Object Distance	FD	1
(0019,xx07)	Table Detector Distance	FD	1
(0019,xx08)	Ortho Step Distance	US	1-n
(0019,xx09)	Asymmetric Collimation	LO	1

## Private Creator "Thorax/Multix FD Post Processing"

Tag	Name	VR	VM
(0021,xx08)	Auto Window Flag	US	1
(0021,xx09)	Auto Window Center	SL	1
(0021,xx0A)	Auto Window Width	SL	1
(0021,xx0B)	Filter ID	SS	1
(0021,xx13)	Internal Value	US	1
(0021,xx14)	Anatomic Correct View	US	1
(0021,xx15)	Auto Window Shift	SS	1
(0021,xx16)	Auto Window Expansion	DS	1
(0021,xx17)	System Type	LO	1
(0021,xx18)	Detector Type	LO	1
(0021,xx30)	Anatomic Sort Number	SH	1
(0021,xx31)	Acquisition Sort Number	SH	1

## Private Creator "Siemens: Thorax/Multix FD Raw Image Settings"

Tag	Name	VR	VM
(0025,xx00)	Raw Image Amplification	SS	1
(0025,xx0C)	Harmonization Kernel	SS	1
(0025,xx0D)	Harmonization Gain	FL	1
(0025,xx0E)	Edge Enhancement Kernel	SS	1
(0025,xx0F)	Edge Enhancement Gain	FL	1
(0025,xx10)	Internal Value	LT	1
(0025,xx18)	Auto Gain	US	1
(0025,xx19)	Ortho Subsampling	US	1
(0025,xx1A)	Image Crop Upper Left	US	2
(0025,xx1B)	Image Crop Upper Right	US	2
(0025,xx1C)	Image Crop Lower Left	US	2
(0025,xx1D)	Image Crop Lower Right	US	2
(0025,xx30)	Manual Cropping	US	1
(0025,xx31)	Gamma LUT Parameter 1	SS	1
(0025,xx32)	Gamma LUT Parameter 2	DS	1
(0025,xx33)	Gamma LUT Parameter 3	SS	1
(0025,xx34)	Gamma LUT Parameter 4	SS	1
(0025,xx35)	Gamma LUT Name	LO	1
(0025,xx36)	Physical EXI	DS	1
(0025,xx37)	Clinical EXI	DS	1

## Private Creator "SIEMENS SMS-AX ACQ 1.0"

Tag	Name	VR	VM	Comment
(0021,xx0F)	Xray Quadruples	SL	1-n	(voltage, current in $\mu$ A, pulse time in $\mu$ s, $\mu$ As) per projection
(0021,xx62)	Reconstruction Preset	US	1	1973
(0021,xxF0)	FlcXmlParams3D	UT	1	Contains XML structure with reconstruction parameters and parameters for Geometry and I0-Calibration.
(0021,xxF1)	Automatic Gain Correction (AGC)	FL	1-n	Contains the AGC value for each frame.

### A.10 FLUOROSpot Compact Worklist Identifier

Module Name	Attribute Name	Tag	M	R	Q	UI	IOD
<b>SOP Common</b>							
	Specific Character Set	(0008,0005)		x			x
<b>Scheduled Procedure Step</b>							
	Scheduled Procedure Step Sequence	(0040,0100)		x			
	> Scheduled Station AET	(0040,0001)	S				
	> Scheduled Procedure Step Start Date	(0040,0002)	S				
	> Scheduled Procedure Step Start Time	(0040,0003)	S				
	> Modality	(0008,0060)	S				
	> Scheduled Performing Physician's Name	(0040,0006)	S	x	x	x	x <sup>1</sup>
	> Scheduled Procedure Step Description	(0040,0007)		x			x
	> Scheduled Protocol Code Sequence	(0040,0008)		x			x
	>> Code Value	(0008,0100)		x			x
	>> Coding Scheme Designator	(0008,0102)		x			x
	>> Coding Scheme Version	(0008,0103)		x			x
	>> Code Meaning	(0008,0104)		x			x
	> Scheduled Procedure Step Location	(0040,0011)		x			
	> Pre-Medication	(0040,0012)		x			
	> Scheduled Procedure Step ID	(0040,0009)		x			x
	> Scheduled Procedure Step Status	(0040,0020)		x			
	> Requested Contrast Agent	(0032,1070)		x			
<b>Requested Procedure</b>							
	Requested Procedure ID	(0040,1001)	P	x	x	x	x <sup>2</sup>
	Reason for the Requested Procedure	(0040,1002)		x			x
	Requested Procedure Description	(0032,1060)		x			x <sup>3</sup>
	Requested Procedure Code Sequence	(0032,1064)		x			x <sup>4</sup>
	> Code Value	(0008,0100)		x			x <sup>4</sup>
	> Coding Scheme Designator	(0008,0102)		x			x <sup>4</sup>
	> Coding Scheme Version	(0008,0103)		x			x <sup>4</sup>
	> Code Meaning	(0008,0104)		x			x <sup>4</sup>
	Study Instance UID	(0020,0000D)		x			x
	Referenced Study Sequence	(0008,1110)		x			x
	> Referenced SOP Class UID	(0008,1150)		x			x
	> Referenced SOP Instance UID	(0008,1155)		x			x
	Requested Procedure Priority	(0040,1003)		x			
	Names of Intended Recipients of Results	(0040,1010)		x			
	Requested Procedure Comments	(0040,1400)		x			
<b>Imaging Service Request</b>							
	Accession Number	(0008,0050)	P	x	x	x	x
	Requesting Physician	(0032,1032)		x		x	
	Referring Physician's Name	(0008,0090)	P	x	x	x	x
	Requesting Service	(0032,1033)		x			
	Imaging Service Request Comments	(0040,2400)		x			
<b>Visit Identification</b>							
	Institution Name	(0008,0080)		x		x	x
	Admission ID	(0038,0010)		x			
<b>Visit Status</b>							
	Current Patient Location	(0038,0300)		x	x		x
<b>Visit Admission</b>							
	Admitting Diagnoses Description	(0008,1080)		x			x

Module Name	Attribute Name	Tag	M	R	Q	UI	IOD
<b>Visit Relationship</b>							
	Referenced Patient Sequence	(0008,1120)		x		x	x
	> Referenced SOP Class UID	(0008,1150)		x		x	x
	> Referenced SOP Instance UID	(0008,1155)			x		xx
<b>Patient Identification</b>							
	Patient Name	(0010,0010)	P	x	x	x	x
	Patient ID	(0010,0020)	P	x	x	x	x
	Other Patient Ids	(0010,1000)		x			x
	Other Patient Names	(0010,1001)		x			x
<b>Patient Demographic</b>							
	Patient's Birth Date	(0010,0030)		x		x	x <sup>5</sup>
	Patient's Sex	(0010,0040)		x		x	x
	Patient's Weight	(0010,1030)		x		x	x
	Patient's Size	(0010,1020)		x		x	x
	Patient's Address	(0010,1040)		x			
	Military Rank	(0010,1080)		x		x	x
	Ethnic Group	(0010,2160)		x			x
	Patient Comments	(0010,4000)		x			x
	Confidentiality Constraints on Patient Data	(0040,3001)		x			x
<b>Patient Medical</b>							
	Patient State	(0038,0500)		x			
	Patient Weight	(0010,1030)		x			x
	Pregnancy Status	(0010,21C0)		x			
	Medical Alerts	(0010,2000)		x <sup>6</sup>			
	Contrast Allergies	(0010,2110)		x			
	Special Needs	(0038,0050)		x			
	Smoking Status Additional	(0010,21A0)		x			
	Patient History Last	(0010,21B0)		x			x
	Menstrual Date	(0010,21D0)		x			

**Table 1:** Basic Worklist C-FIND-RSP Return Key Attributes

The table should be read as follows:

Attribute Name: Attributes supported to build a FLUOROSPOT Compact Worklist Request Identifier.

Tag: Appropriate DICOM tag for this attribute.

M: Matching keys for Worklist Update. A "S" will indicate that FLUOROSPOT Compact will supply an attribute value for Single Value Matching.  
It can be configured that the attribute is used as Return Key only  
A "P" indicated, that the user can enter a single value for a Patient based worklist query.

R: Return keys. A "x" will indicate that FLUOROSPOT Compact will supply this attribute as Return Key with zero length for Universal Matching.

Q: Interactive Query Key. A "x" will indicate that FLUOROSPOT Compact will supply this attribute as matching key, if entered in the Query Patient Worklist dialog. The other keys will then be Return Keys only.

IOD: An "x" indicates that this Worklist attribute is included into all Object Instances created during performance of the related Procedure Step.

The default Query Configuration is set to "Modality" and "Date". Optionally, matching for the own "AE Title" and "Date" is configurable. For "Date" one of the following settings could be configured: "Today", "Yesterday - Today", "Today +/- 12 hours", "Today +/- 24 hours", and "Use no Date".

In Patient based worklist update the usage of date and time can be deactivated! The Scheduled AE Title is used as Return Key only.

x1 : "Scheduled Performing Physician's Name (0040,0006)" is not directly included in the header. However its value is stored in the header as "Performing Physician's Name (0008,1050)". It can be modified by user during Patient Registration.

x2 : Requested Procedure ID (0040,1001) is directly included in the header. Additionally its value is stored in the header as Study ID (0020,0010). Study ID can be modified by the user during Patient Registration.

x3: Requested Procedure Description (0032,1060) is not directly included in the header. However its value is stored in the header as "Study Description (0008,1030)". It can be modified by user during Patient Registration.

x4: Requested Procedure Code Sequence (0032,1064) is not directly included in the header. However its value is stored in the header as "Procedure Code Sequence (0008,1032)". It is not sent, when the scheduled protocol codes differ from the performed protocol codes.

x5: If a date of birth (0010,0030) with zero\_length value is received via worklist, the date of birth will internally set to 1850/01/01.

The value 1850/01/01 for the date of birth is not used in a DICOM header (DICOM Send, MPPS, CD/DVD Export), but again replaced by zero\_length.

x6: Medical Alerts (0010,2000) has a VM of 1-n. Only 20 entries are accepted.

**A.11 Modality Performed Procedure Step Identifiers**

Attribute Name	Tag	N-CREATE no value	N-CREATE with value	N-SET Final
Specific Character Set	(0008,0005)		value depends on regional settings	./.
Modality	(0008,0060)		"CR", "RF"	./.
Procedure Code Sequence	(0008,1032)	- If not received via worklist - for unscheduled case	If received via worklist	./.
> Code Value	(0008,0100)		From Modality Worklist (0032,1064)	
> Coding Scheme Designator	(0008,0102)		From Modality Worklist (0032,1064)	
> Coding Scheme Version	(0008,0103)		From Modality Worklist (0032,1064)	
> Code Meaning	(0008,0104)		From Modality Worklist (0032,1064)	
Referenced Patient Sequence	(0008,1120)	Y if not received via worklist	From Modality Worklist (0008,1120)	./.
Patient's Name	(0010,0010)		From Modality Worklist or user input (all 5 components)	./.
Patient ID	(0010,0020)		From Modality Worklist or user input	./.
Patient's Birth Date	(0010,0030)		From Modality Worklist or user input	./.
Patient's Sex	(0010,0040)		From Modality Worklist or user input	./.
Distance Source to Detector (SID)	(0018,1110)	Y		minimum value
Image Area Dose Product	(0018,115E)	Y		includes fluoro and cassette images
Study ID	(0020,0010)		From Modality Worklist (0040,1001) or user input	./.
Performed Station AE Title	(0040,0241)		AE Title of MPPS SCU	./.
Performed Station Name	(0040,0242)		Hostname	./.
Performed Location	(0040,0243)	Y		./.
Performed Procedure Step Start Date	(0040,0244)		Y	./.
Performed Procedure Step Start Time	(0040,0245)		Y	./.
Performed Procedure Step End Date	(0040,0250)	Y		./. (if IN PROGRESS) Actual end date (ELSE)

Attribute Name	Tag	N-CREATE no value	N-CREATE with value	N-SET Final
Performed Procedure Step End Time	(0040,0251)	Y		./. (if IN PROGRESS) Actual end time (ELSE)
Performed Procedure Step Status	(0040,0252)		IN PROGRESS	IN PROGRESS DISCONTINUED COMPLETED
Performed Procedure Step ID	(0040,0253)		Internally created	./.
Performed Procedure Step Description	(0040,0254)		From Modality Work- list (0040,0007) or user input	(0040,0007) or user input
Performed Procedure Type Description	(0040,0255)	Y		zero Length
Performed Protocol Code Sequence <sup>1</sup>	(0040,0260)			./.
> Code Value	(0008,0100)		From performed Exam	From performed Exam
> Coding Scheme Designator	(0008,0102)		From performed Exam	From performed Exam
> Coding Scheme Version	(0008,0103)		From performed Exam	From performed Exam
> Code Meaning	(0008,0104)		From performed Exam	From performed Exam
Scheduled Step Attributes Sequence	(0040,0270)		If 1st dose applied results in an Instance	./.
> Accession Number	(0008,0050)		From Modality Worklist	./.
> Referenced Study Sequence	(0008,1110)			./.
>> Referenced SOP Class UID	(0008,1150)		From Modality Worklist	./.
>> Referenced SOP Instance UID	(0008,1155)		From Modality Worklist	./.
> Study Instance UID	(0020,000D)		From Modality Worklist	./.
> Requested Procedure Description	(0032,1060)		From Modality Worklist	./.
> Scheduled Procedure Step Description	(0040,0007)		From Modality Worklist	./.
> Scheduled Protocol Code Sequence	(0040,0008)			./.
>> Code Value	(0008,0100)		From Modality Work- list (0040,0008)	./.
>> Coding Scheme Designator	(0008,0102)		From Modality Work- list (0040,0008)	./.

<sup>1</sup> items only present if system is configured to use protocol codes, otherwise zero length

Attribute Name	Tag	N-CREATE no value	N-CREATE with value	N-SET Final
>> Coding Scheme Version	(0008,0103)		From Modality Worklist (0040,0008)	./.
>> Code Meaning	(0008,0104)		From Modality Worklist (0040,0008)	./.
> Scheduled Procedure Step ID	(0040,0009)		From Modality Worklist	./.
> Requested Procedure ID	(0040,1001)		From Modality Worklist	./.
Performed Procedure Step Discontinuation Reason Code Sequence	(0040,0281)	Y		only if (0040,0252) = "DISCONTINUED"
> Code Value				See Context ID 9300 DICOM 3.16
> Coding Scheme Designator	(0008,0102)			DCM
> Code Meaning	(0008,0104)			See Context ID 9300
Total Time of Fluoroscopy	(0040,0300)	Y		Total time
Total Number of Exposures	(0040,0301)	Y		Number of exposures
Distance Source to Entrance	(0040,0306)	Y		Minimum value
Exposure Dose Sequence	(0040,030E)	Y		Zero or more items
> KVP	(0018,0060)			Current value
> Protocol Name	(0018,1030)			Current value, if extended MPPS is configured
> X-Ray Tube Current in $\mu$ A	(0018,8151)			Current value
> Exposure Time	(0018,1150)			Current value
> Filter Type	(0018,1160)			Current value
> Image Area Dose Product Extended MPPS is configured	(0018,115E)			Current value, if
> Entrance Dose in mGy	(0040,8302)			Current value, if Extended MPPS is configured
Comments on Radiation Dose	(0018,0310)	Y		Formatted table of dose information for all acquisition events format: <OGP  dGy*cm <sup>2</sup>   kV  mAs  Filter  Physical EXI>\r\n
Billing Procedure Sequence	(0040,0320)	Y		Zero or more items

Attribute Name	Tag	N-CREATE no value	N-CREATE with value	N-SET Final
Film Consumption Sequence	(0040,0321)	Y		Zero or more items
> Film Size ID	(2010,0050)			Y if Seq. present
> Number of Films	(2100,0170)			Y if Seq. present
Performed Series Sequence	(0040,0340)	Y		One or more items
> Retrieve AE Title	(0008,0054)			AE Title of archive
> Series Description	(0008,103E)	--		Organ Program Name
> Performing Physician's Name	(0008,1050)	--		from Modality Worklist (0040,0006) or user input
> Operator's Name	(0008,1070)	--		user input
> Referenced Image Sequence	(0008,1140)	--		One or more items
>> Referenced SOP Class UID	(0008,1150)	--		System dependent
>> Referenced SOP Instance UID	(0008,1155)	--		Y
> Protocol Name	(0018,1030)	--		Organ Program
> Series Instance UID	(0020,000E)	--		Y
> Referenced Non-Image Composite SOP Instance Sequence	(0040,0220)			Zero length
Entrance Dose in mGy	(0040,8302)	Y		Accumulated value, includes fluoro and cassette images

#### A.12 Grayscale Image Consistency

The used monitors come with a DICOM Grayscale Standard Display Function (GSDF) compliant factory pre-setting. A typical working environment setup is assumed for ambient light.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens Healthineers sales organization worldwide. Availability and packaging may vary by country and are subject to change without prior notice.

Some/All of the features and products described herein may not be available in the United States or other countries.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features that do not always have to be present in individual cases.

Siemens Healthineers reserves the right to modify the design, packaging, specifications and options described herein without prior notice. Please contact your local Siemens Healthineers sales representative for the most current information.

In the interest of complying with legal requirements concerning the environmental compatibility of our products (protection of natural resources and waste conservation), we recycle certain components. Using the same extensive quality assurance measures as for factory-new components, we guarantee the quality of these recycled components.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

Caution: Federal law restricts this device to sale by or on the order of a physician.

---

**Siemens Healthineers Headquarters**  
Siemens Healthcare GmbH Henkestr.  
127  
91052 Erlangen, Germany  
Phone +49 9131 84-0  
[siemens-healthineers.com](http://siemens-healthineers.com)

**Legal Manufacturer**  
Siemens Healthcare GmbH  
Henkestr. 127  
91052 Erlangen, Germany