

DICOM Conformance Statement

AI-Rad Companion Chest X-ray *VC10A*



August 2025

DICOM Conformance Statement

AI-Rad Companion Chest X-ray is a cloud-based/on-premises(edge) application, which communicates indirectly with other DICOM nodes since it makes use of the network services provided by teamplay DICOM Hub and teamplay Receiver.

AI-Rad Companion Chest X-ray:

- Receives the input DICOM data from teamplay DICOM Hub after a storage request to teamplay Receiver using the configurable AET (E.g., "AIRC").
- Displays images to a user (browser-based viewer application).
- Generates result objects in DICOM Secondary Capture format, DICOM Structured Report (TID 1500), DICOM Grayscale Softcopy Presentation States (GSPS).
- Stores result DICOM data via teamplay DICOM Hub and teamplay Receiver to one or several target DICOM nodes configured in teamplay DICOM Hub.

AI-Rad Companion Chest X-ray conforms to the DICOM Standard [\[2\]](#) and supports the network services through teamplay DICOM Hub as described in Table 1 - Network Services.

Please refer to the DICOM Conformance Statements of teamplay DICOM Hub and teamplay Receiver [\[1\]](#) for further information on the provided network services.

NOTE: - This DICOM Conformance Statement is applicable for **AI-Rad Companion Chest X-ray** of version VC10A and later until superseded by a more recent document applicable to a more recent version.

Table 1: Storage SOP Classes

SOP Classes	SOP Class UID	User of Service (SCU)		Provider of Service (SCP)	
SOP Classes created by AI-Rad Companion Chest X-ray					
		Create	Send	Store	Display
Comprehensive SR Storage	1.2.840.10008.5.1.4.1.1.88.33	Yes	No ¹	No ¹	No
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Yes	No ¹	No ¹	No
Grayscale Softcopy Presentation State Storage	1.2.840.10008.5.1.4.1.1.11.1	Yes	No ¹	No ¹	No




¹ Network communication is performed by teamplay DICOM Hub and teamplay Receiver. Please refer to the DICOM Conformance Statements of teamplay DICOM Hub and teamplay Receiver [\[1\]](#) for further information.

Table 2: Media Services

Media Storage Application Profile	Write Files (FSC or FSU)	Read Files (FSR)
NOT APPLICABLE		

Table 3: Implementation Identifying Information

Name	Value
Implementation Class UID	1.3.12.2.1107.5.8.21
Implementation Version Name	AI-Rad Companion

	1 INTRODUCTION	6
	1.1 Revision History	6
	1.2 Audience	6
	1.3 Remarks	6
	1.4 Definitions, Terms and Abbreviations	6
	1.5 References	7
	2 NETWORKING	8
	2.1 Implementation Model	8
	2.1.1 Application Data Flow	9
	2.1.2 Functional Definitions of AEs	9
	2.1.3 Sequencing of Activities	9
	2.2 AE Specifications	9
	2.3 Network Interfaces	9
	2.3.1 Physical Network Interface	9
	2.3.2 Additional Protocols	9
	2.3.3 IPv4 and IPv6 Support	9
	2.4 Configuration	9
	3 MEDIA INTERCHANGE	10
	4 TRANSFORMATIONS OF DICOM TO CDA	10
	5 SUPPORT OF EXTENDED CHARACTER SETS	11
	6 ATTRIBUTE CONFIDENTIALITY PROFILES	14
	6.1 De-identification	14
	7 SECURITY	15
	7.1 Security Profiles	15
	7.2 Association Level Security	15

7.3	Application Level Security	15
8	ANNEXES	16
8.1	IOD Contents	16
8.1.1	Created SOP Instances	16
8.1.2	Usage of Attributes from Received IODs	25
8.1.3	Attribute Mapping	25
8.1.4	Coerced/Modified Fields	25
8.2	Data Dictionary of Private Attributes	25
8.3	Coded Terminology and Templates	25
8.3.1	Context Groups	25
8.3.2	Template Specifications	26
8.3.3	Private Code definitions	29
8.4	Grayscale Image Consistency	29
8.5	Standard Extended / Specialized / Private SOP Classes	29
8.6	Private Transfer Syntaxes	29

1 Introduction

1.1 Revision History

Version	Date	Change
R1.0	06/08/2025	<p>Final release for VC10A.</p> <ul style="list-style-type: none">Modified table "Imaging Measurement Report for DICOM SR" to include details for Non Actionable Finding.Modified table "Private Code definitions" to add Non Actionable Disease private tag.

1.2 Audience

This document is intended for the audience listed below. It is assumed that the reader has a working knowledge of the DICOM Standard. The document structure was designed for easier access to relevant information for different user groups:

List of audience shall be hospital staff, health system integrators, software designers or implementers.

1.3 Remarks

The scope of this DICOM Conformance Statement is to facilitate integration between **AI-Rad Companion Chest X-ray** and other DICOM products. The Conformance Statement should be read and understood in conjunction with the DICOM Standard [\[2\]](#). DICOM by itself does not guarantee interoperability.

The Conformance Statement does, however, facilitate a first-level comparison for interoperability between different applications supporting compatible DICOM functionality.

This Conformance Statement is not supposed to replace validation with other DICOM equipment to ensure proper exchange of intended information. In fact, the user should be aware of the following important issues:

- The comparison of conformance statements is the first step towards assessing interconnectivity and interoperability between **AI-Rad Companion Chest X-ray** and other DICOM conformant equipment.
- Test procedures should be defined and executed to validate the required level of interoperability with specific compatible DICOM equipment, as established by the healthcare facility.
- Siemens Healthineers reserves the right to modify the design and specifications contained herein without prior notice. Please contact your local Siemens Healthineers representative for the most recent product information.

1.4 Definitions, Terms and Abbreviations

Definitions, terms, and abbreviations used in this document are defined within the different parts of the DICOM standard.

Additional Abbreviations and terms are as follows:

AE	DICOM Application Entity
AET	Application Entity Title
ASCII	American Standard Code for Information Interchange
DCS	DICOM Conformance Statement
DICOM	Digital Imaging and Communications in Medicine
FSC	File Set Creator

FSR	File Set Reader
FSU	File Set Updater
GSDF	Grayscale Standard Display Function
GSPS	Grayscale Softcopy Presentation State
IOD	DICOM Information Object Definition
ISO	International Standard Organization
N/A	Not Applicable
NEMA	National Electrical Manufacturers Association
O	Optional Key Attribute
PDU	DICOM Protocol Data Unit
R	Required Key Attribute
ROI	Region Of Interest
SC	Secondary Capture
SCU	DICOM Service Class User (DICOM client)
SCP	DICOM Service Class Provider (DICOM Server)
SOP	DICOM Service-Object Pair
SR	Structured Report
TID	Template ID
U	Unique Key Attribute
UID	Unique Identifier
UTF-8	Unicode Transformation Format-8
VR	Value Representation

1.5 References

- [1] DICOM Conformance Statements of teamplay – <https://www.siemens-healthineers.com/en-in/services/it-standards/dicom-conformance-statements-digital-and-automation/teamplay>
- [2] NEMA PS3 / ISO 12052, Digital Imaging and Communications in Medicine (DICOM) Standard, National Electrical Manufacturers Association, Rosslyn, VA, USA (available free at <https://www.dicomstandard.org/>)
- [3] AI-Rad Companion Data Privacy and Security White Paper – Based on on-demand request from the end users.

2 Networking

Please refer to the latest version of DICOM Conformance Statement of teamplay DICOM Hub [\[1\]](#) for further information on the provided networking capabilities for AI-Rad Companion Chest X-ray.

Table 4: Supported Image Encoding

Abstract Syntax		Transfer Syntax	
SOP Classes	SOP Class UID	Name List	UID List
Digital X-Ray Image Storage – For Presentation	1.2.840.10008.5.1.4.1.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2
		Explicit VR Little Endian	1.2.840.10008.1.2.1
		Explicit VR Big Endian (Retired)	1.2.840.10008.1.2.2
		JPEG Lossless, Non-Hierarchical, First-Order Prediction (Process 14 [Selection Value 1])	1.2.840.10008.1.2.4.70
		JPEG 2000 Image Compression (Lossless Only)	1.2.840.10008.1.2.4.90
Comprehensive SR Storage	1.2.840.10008.5.1.4.1.1.88.33	Explicit VR Little Endian	1.2.840.10008.1.2.1
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Explicit VR Little Endian	1.2.840.10008.1.2.1
Computed Radiography Image Storage	1.2.840.10008.5.1.4.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2
		JPEG Lossless, Non-Hierarchical, First-Order Prediction (Process 14 [Selection Value 1])	1.2.840.10008.1.2.4.70
		JPEG 2000 Image Compression (Lossless Only)	1.2.840.10008.1.2.4.90
		Explicit VR Big Endian (Retired)	1.2.840.10008.1.2.2
		Explicit VR Little Endian	1.2.840.10008.1.2.1
Grayscale Softcopy Presentation State Storage	1.2.840.10008.5.1.4.1.1.11.1	Explicit VR Little Endian	1.2.840.10008.1.2.1

2.1 Implementation Model

NOT APPLICABLE

2.1.1 Application Data Flow

NOT APPLICABLE

2.1.2 Functional Definitions of AEs

NOT APPLICABLE

2.1.3 Sequencing of Activities

NOT APPLICABLE

2.2 AE Specifications

NOT APPLICABLE

2.3 Network Interfaces

2.3.1 Physical Network Interface

NOT APPLICABLE

2.3.2 Additional Protocols

NOT APPLICABLE

2.3.3 IPv4 and IPv6 Support

NOT APPLICABLE

2.4 Configuration

NOT APPLICABLE

3 Media Interchange

Please refer to the latest version of "DICOM Conformance Statements of teamplay DICOM Hub [1] for further information on the provided Media Interchange for AI-Rad Companion Chest X-ray".

4 Transformations of DICOM to CDA

NOT APPLICABLE

5 Support of Extended Character Sets

AI-Rad Companion Chest X-ray supports the following character sets as defined in the tables in this section.

Table 5: Single-Byte Character Sets without Code Extension

Character Set Description	Defined Term	ISO Registration Number	Character Set
Default repertoire	None	ISO_IR 6	ISO 646
Latin alphabet No. 1	ISO_IR 100	ISO_IR 100	Supplementary set
		ISO_IR 6	ISO 646
Latin alphabet No. 2	ISO_IR 101	ISO_IR 101	Supplementary set
		ISO_IR 6	ISO 646
Latin alphabet No. 3	ISO_IR 109	ISO_IR 109	Supplementary set
		ISO_IR 6	ISO 646
Latin alphabet No. 4	ISO_IR 110	ISO_IR 110	Supplementary set
		ISO_IR 6	ISO 646
Latin alphabet No. 5	ISO_IR 148	ISO_IR 148	Supplementary set
		ISO_IR 6	ISO 646
Cyrillic	ISO_IR 144	ISO_IR 6	Supplementary set
		ISO_IR 6	ISO 646
Arabic	ISO_IR 127	ISO_IR 127	Supplementary set
		ISO_IR 6	ISO 646
Greek	ISO_IR 126	ISO_IR 126	Supplementary set
		ISO_IR 6	ISO 646
Hebrew	ISO_IR 138	ISO_IR 138	Supplementary set
		ISO_IR 6	ISO 646
Japanese	ISO_IR 13	ISO_IR 13	JIS X 0201: Katakana
		ISO_IR 14	JIS X 0201: Romaji
Thai	ISO_IR 166	ISO_IR 166	TIS 620-253 (1990)
		ISO_IR 6	ISO 646

Table 6: Single-Byte Characters Sets with Code Extension

Character Set Description	Defined Term	Standard for Code Extension	ESC sequence	ISO Registration Number	Character Set
Default repertoire	ISO 2022 IR 6	ISO 2022	ESC 02/08 04/02	ISO-IR 6	ISO 646
Latin alphabet No.1	ISO 2022 IR 100	ISO 2022	ESC 02/13 04/01	ISO-IR 100	Supplementary set
		ISO 2022	ESC 02/08 04/02	ISO-IR 6	ISO 646
Latin alphabet No.2	ISO 2022 IR 101	ISO 2022	ESC 02/13 04/02	ISO-IR 101	Supplementary set
		ISO 2022	ESC 02/08 04/02	ISO-IR 6	ISO 646
Latin alphabet No.3	ISO 2022 IR 109	ISO 2022	ESC 02/13 04/03	ISO-IR 109	Supplementary set
		ISO 2022	ESC 02/08 04/02	ISO-IR 6	ISO 646
Latin alphabet No. 4	ISO 2022 IR 110	ISO 2022	ESC 02/13 04/04	ISO-IR 110	Supplementary set
		ISO 2022	ESC 02/08 04/02	ISO-IR 6	ISO 646
Latin alphabet No. 5	ISO 2022 IR 148	ISO 2022	ESC 02/13 04/13	ISO-IR 148	Supplementary set
		ISO 2022	ESC 02/08 04/02	ISO-IR 6	ISO 646

Table 7: Multi-Byte Character Sets without Code Extension

Character Set Description	Defined Term	ISO Registration Number	Character Set
Unicode	ISO_IR 192	ISO 10646	Unicode in UTF-8
Chinese	GB18030	GB18030	GB 18030-2000 (China Association for Standardization)

Table 8: Multi-Byte Character Sets with Code Extension

Character Set Description	Defined Term	Standard for Code Extension	ESC sequence	ISO Registration Number	Character Set
Japanese	ISO 2022 IR 159	ISO 2022	ESC 02/04 02/08 04/04	ISO-IR 159	JIS X 0212: Supplementary Kanji set
Korean	ISO 2022 IR 149	ISO 2022	ESC 02/04 02/09 04/03	ISO-IR 149	KS X 1001: Hangul and Hanja

All Special Character Sets(SCS) listed above are supported for incoming Data.

Three categories of character sets have to be differentiated because of their different encoding formats:

- Conventional ISO character sets: ISO_IR 6, ISO 2022 IR 6, ISO_IR 100, etc. → Encoded in ISO 2022
- ISO_IR 192 → Encoded in UTF-8
- GB18030 → Encoded in GB18030

It is not possible to recognize the following mismatches automatically on receiving or importing:

- An attribute value is encoded in ISO_IR 192 \leftrightarrow (0008,0005) contains a conventional ISO character set as primary character set
- An attribute value is encoded in GB18030 \leftrightarrow (0008,0005) contains a conventional ISO character set as primary character set
- An attribute value is encoded in ISO 2022 \leftrightarrow (0008,0005) contains ISO_IR 192
- An attribute value is encoded in ISO 2022 \leftrightarrow (0008,0005) contains GB18030

An IOD that contains one of the above mentioned inconsistencies does not conform to DICOM. As these kinds of inconsistencies cannot be recognized by the system, the IOD will not be rejected but the character data might be corrupted.

AI-Rad Companion Chest X-ray supports Kanji characters in the byte zone after 74 (79, 7A, 7B and 7C).

6 Attribute confidentiality profiles

6.1 De-identification

Please refer to the AI-Rad Companion Data Privacy and Security White Paper [\[3\]](#) for further information on the support of de-identification of attributes natively for AI-Rad Companion Cloud deployment. In case of edge (on-premises) deployment, no de-identification happens from teamplay.

AI-Rad Companion Chest X-ray does not support High privacy and Restrictive privacy levels.

7 Security

7.1 Security Profiles

AI-Rad Companion Chest X-ray does not support any specific security measures.

7.2 Association Level Security

It is possible to configure whether the SCP will only answer to known AETs or to any AET.

7.3 Application Level Security

NOT APPLICABLE

8 Annexes

The tables in these section uses a number of abbreviations.

The abbreviations used in the "Presence" column are:

VNAP	Value is Not Always Present. Attribute is sent zero length if no value is present.
ANAP	Attribute Not Always Present.
ALWAYS	Attribute and Value are always present.
EMPTY	Attribute is sent zero length.

The abbreviations used in the "Source" Column are:

MWL	The attribute value is copied from Modality Worklist.
USER	The attribute value is entered by the user.
AUTO	The attribute value is generated by the system.
CONFIG	The attribute value is obtained by configuration
COPY	The attribute value is copied from the source image

8.1 IOD Contents

8.1.1 Created SOP Instances

8.1.1.1 General Modules

Table 9 lists all Attributes that are supported in the Patient Module.

Table 9: Patient Module

Attribute	Tag	Source	Value	Presence	Comments
Patient`s Name	(0010,0010)	COPY	Copied from source image	VNAP	
Patient ID	(0010,0020)	COPY	Copied from source image	VNAP	
Patient`s Birth Date	(0010,0030)	COPY	Copied from source image	VNAP	
Patient`s Sex	(0010,0040)	COPY	Copied from source image	VNAP	

Table 10 lists all Attributes that are supported in the General Study Module

Table 10: General Study Module

Attribute	Tag	Source	Value	Presence	Comments
Study Date	(0008,0020)	COPY	Copied from source image	VNAP	

Attribute	Tag	Source	Value	Presence	Comments
Study Time	(0008,0030)	COPY	Copied from source image	VNAP	
Accession Number	(0008,0050)	COPY	Copied from source image	VNAP	
Referring Physician's Name	(0008,0090)	COPY	Copied from source image	VNAP	
Study Description	(0008,1030)	COPY	Copied from source image	ANAP	
Study Instance UID	(0020,000D)	COPY	Copied from source image	ALWAYS	
Study ID	(0020,0010)	COPY	Copied from source image	VNAP	

Table 11 lists all Attributes that are supported in the Patient Study Module

Table 11: Patient Study Module

Attribute	Tag	Source	Value	Presence	Comments
Patient's Age	(0010,1010)	COPY	Copied from source image	ANAP	

Table 12 lists all Attributes that are supported in the General Equipment Module

Table 12: General Equipment Module

Attribute	Tag	Source	Value	Presence	Comments
Manufacturer	(0008,0070)	AUTO	Siemens Healthineers	ALWAYS	
Manufacturer's Model Name	(0008,1090)	AUTO	AI-Rad Companion Chest X-ray	ALWAYS	
Device Serial Number	(0018,1000)	AUTO	AI-Rad Companion system IVK number + teamplay serial number"	ALWAYS	
Secondary Capture Device Software Version	(0018,0019)	AUTO	VXXXX where XXXX indicates the used version of Chest X-ray. For Example:- VC10A.	ALWAYS	

Table 13 lists all Attributes that are supported in the SOP Common Module

Table 13: SOP Common Module

Attribute	Tag	Source	Value	Presence	Comments
Specific Character Set	(0008,0005)	COPY	Copied from input image	ALWAYS	
Instance Creation Date	(0008,0012)	AUTO	Date when series is created	ALWAYS	
Instance Creation Time	(0008,0013)	AUTO	Time when series is created	ALWAYS	
SOP Class UID	(0008,0016)	AUTO	Secondary Capture: 1.2.840.10008.5.1.4.1.1.7 Comprehensive SR :	ALWAYS	

Attribute	Tag	Source	Value	Presence	Comments
			1.2.840.10008.5.1.4.1.1.88.33 GSPS: 1.2.840.10008.5.1.4.1.1.11.1		
SOP Instance UID	(0008,0018)	AUTO	Unique Identifier of the instance	ALWAYS	
Timezone Offset from UTC	(0008,0201)	COPY	Copied from the input image. If not present then it is +0000	ALWAYS	

8.1.1.2 Secondary Capture Image Storage IOD

Table 14: Secondary Capture Image IOD Modules

Information Entity	Module	Reference	Presence of Module
Patient	Patient Module	Table 9	ALWAYS
Study	General Study Module	Table 10	ALWAYS
	Patient Study Module	Table 11	ALWAYS
Series	General Series Module	Table 15	ALWAYS
Equipment	General Equipment Module	Table 12	ALWAYS
	SC Equipment Module	Table 16	ALWAYS
Image	General Image Module	Table 17	ALWAYS
	Image Pixel Module	Table 18	ALWAYS
	SC Image Module	Table 19	ALWAYS
	SOP Common	Table 13	ALWAYS

Table 15 lists all Attributes that are supported in the General Series IOD Module

Table 15: General Series IOD Modules

Attribute	Tag	Source	Value	Presence	Comments
Series Date	(0008,0021)	AUTO	Date when series is created	ALWAYS	
Series Time	(0008,0031)	AUTO	Time when series is created	ALWAYS	
Modality	(0008,0060)	AUTO	OT	ALWAYS	
Series Description	(0008,103E)	AUTO	AI-Rad Companion-Results	ALWAYS	
Series Instance UID	(0020,000E)	AUTO	Unique identifier of the series.	ALWAYS	
Series Number	(0020,0011)	AUTO	Set to high value	ALWAYS	

Attribute	Tag	Source	Value	Presence	Comments
Patient Position	(0018,5100)	COPY	Copied from input image	VNAP	
Laterality	(0020,0060)	COPY	Copied from input image	VNAP	

Table 16 lists all Attributes that are supported in the SC Equipment IOD Module

Table 16: SC Equipment IOD Modules

Attribute	Tag	Source	Value	Presence	Comments
Conversion Type	(0008,0064)	AUTO	WSD	ALWAYS	

Table 17 lists all Attributes that are supported in the General Image IOD Module

Table 17: General Image IOD Modules

Attribute	Tag	Source	Value	Presence	Comments
Image Type	(0008,0008)	AUTO	DERIVED\SECONDARY	ALWAYS	
Content Date	(0008,0023)	AUTO	Date when series is created	ALWAYS	
Content Time	(0008,0033)	AUTO	Time when series is created	ALWAYS	
Instance Number	(0020,0013)	AUTO	0	ALWAYS	
Patient Orientation	(0020,0020)	AUTO	Copied from input image	ANAP	

Table 18 lists all Attributes that are supported in the Image Pixel IOD Module

Table 18: Image pixel IOD Modules

Attribute	Tag	Source	Value	Presence	Comments
Samples per Pixel	(0028,0002)	AUTO	3	ALWAYS	
Photometric Interpretation	(0028,0004)	AUTO	RGB	ALWAYS	
Planar Configuration	(0028,0006)	AUTO	0	ALWAYS	
Rows	(0028,0010)	AUTO	Variable	ALWAYS	
Columns	(0028,0011)	AUTO	Variable	ALWAYS	
Bits Allocated	(0028,0100)	AUTO	8	ALWAYS	
Bits Stored	(0028,0101)	AUTO	8	ALWAYS	
High Bit	(0028,0102)	AUTO	7	ALWAYS	
Pixel Representation	(0028,0103)	AUTO	0	ALWAYS	
Pixel Data	(7FE0,0010)	AUTO	A data stream of the pixel samples that comprise the Image	ALWAYS	

Attribute	Tag	Source	Value	Presence	Comments
Pixel Aspect Ratio	(0028,0034)	AUTO	Integer ratio. Vertical Size\Horizontal Size Example: 6\5	ALWAYS	

Table 19 lists all Attributes that are supported in the SC Image IOD Module

Table 19: SC Image IOD Modules

Attribute	Tag	Source	Value	Presence	Comments
Pixel Spacing	(0028,0030)	AUTO	Copied from input image	ALWAYS	

8.1.1.3 Structured Report Document IOD

Table 20: Structured Report Document IOD Modules

Information Entity	Module	Reference	Presence of Module
Patient	Patient	Table 9	ALWAYS
Study	General Study	Table 10	ALWAYS
	Patient Study	Table 11	ALWAYS
Series	SR Document Series	Table 21	ALWAYS
Equipment	General Equipment	Table 12	ALWAYS
Document	SR Document General	Table 22	ALWAYS
	SR Document Content	Table 23	ALWAYS
	SOP Common	Table 13	ALWAYS

Table 21 lists all Attributes that are supported in the SR Document Series Module

Table 21: SR Document Series Module

Attribute	Tag	Source	Value	Presence	Comments
Modality	(0008, 0060)	AUTO	SR	ALWAYS	
Series Instance UID	(0020, 000E)	AUTO	Unique identifier of the Series	ALWAYS	
Series Number	(0020, 0011)	AUTO	A number that identifies the Series.	ALWAYS	
Series Date	(0008,0021)	AUTO	Date the Series started.	ALWAYS	
Series Time	(0008,0031)	AUTO	Time the Series started.	ALWAYS	

Attribute	Tag	Source	Value	Presence	Comments
Protocol Name	(0018, 1030)	AUTO	Description of the conditions under which the Series was performed.	ALWAYS	
Series Description	(0018, 103E)	AUTO	Description of the series (AI-Rad Companion-SR)	ALWAYS	
Referenced Performed Procedure Step Sequence	(0008,1111)	AUTO	Uniquely identifies the Performed Procedure Step SOP Instance for which the Series is created.	ALWAYS	

Table 22 lists all Attributes that are supported in the SR Documents General Module

Table 22: SR Document General Module

Attribute	Tag	Source	Value	Presence	Comments
Verification Flag	(0040, A493)	AUTO	Unverified	ALWAYS	
Completion Flag	(0040, A491)	AUTO	COMPLETE	ALWAYS	
Content Date	(0008,0023)	AUTO	The date the document content creation started.	ALWAYS	
Content Time	(0008,0033)	AUTO	The time the document content creation started.	ALWAYS	
Study Instance UID	(0020,000D)	AUTO	Unique identifier of the study	ALWAYS	Copied from input image
Accession Number	(0008,0050)	AUTO	Copied from input image	VNAP	
Performed Procedure Code Sequence	(0040, A372)	AUTO	(36554-4, LOINC, XR Chest Single view)	ALWAYS	

Table 23 lists all Attributes that are supported in the SR Document Content Module

Table 23: SR Document Content Module

Please refer to [Table 32](#): Imaging Measurement Report for DICOM SR for details regarding SR Document Content Module.

8.1.1.4 Grayscale Softcopy Presentation State Storage

Table 24: Grayscale Softcopy Presentation State IOD Modules

Information Entity	Module	Reference	Presence of Module
Patient	Patient	Table 9	ALWAYS
Study	General Study	Table 10	ALWAYS
	Patient Study	Table 11	ALWAYS
Series	General Series	Table 15	ALWAYS

	Presentation Series	Table 25	ALWAYS
Equipment	General Equipment	Table 12	ALWAYS
Presentation State	Presentation State Identification	Table 26	ALWAYS
	Presentation State Relationship	Table 27	ALWAYS
	Displayed Area	Table 28	ALWAYS
	Graphic Annotation	Table 29	ALWAYS
	SOP Common	Table 13	ALWAYS

Table 25: Presentation Series Module

Attribute	Tag	Source	Value	Presence	Comments
Modality	(0008, 0060)	AUTO	PR	ALWAYS	

Table 26: Presentation State Identification Module

Attribute	Tag	Source	Value	Presence	Comments
Presentation Creation Date	(0070,0082)	AUTO	Date on which this presentation was created.	ALWAYS	
Presentation Creation Time	(0070,0083)	AUTO	Time at which this presentation was created.	ALWAYS	
Instance Number	(0020,0013)	AUTO	1	ALWAYS	
Content Label	(0070,0080)	AUTO	AIRC AI RESULTS	ALWAYS	
Content Description	(0070,0081)	AUTO	AIRC AI Results	ALWAYS	
Content Creator's Name	(0070,0084)	AUTO		EMPTY	

Table 27: Presentation State Relationship Module

Attribute	Tag	Source	Value	Presence	Comments
Referenced Series Sequence	(0008,1115)	AUTO		ALWAYS	
> Series Instance UID	(0020,000E)	AUTO	Unique identifier of a Series that is part of the Study defined by the Study Instance UID (0020,000D) in the enclosing data set.	ALWAYS	
> Referenced Image Sequence	(0008,1140)			ALWAYS	
>> Referenced SOP Class UID	(0008,1150)	AUTO	Uniquely identifies the referenced SOP Class.	ALWAYS	
>> Referenced SOP Instance UID	(0008,1155)	AUTO	Uniquely identifies the referenced SOP Instance.	ALWAYS	

Table 28: Presentation State Shutter Module

Attribute	Tag	Source	Value	Presence	Comments
Shutter Presentation Value	(0018,1622)	COPY	A single grayscale unsigned value used to replace those parts of the image occluded by the shutter, when rendered on a monochrome display	VNAP	
Shutter Presentation- Color CIE Lab Value	(0018,1624)	COPY	A color triplet value used to replace those parts of the image occluded by the shutter, when rendered on a color display.	VNAP	

Table 29: Displayed Area

Attribute	Tag	Source	Value	Presence	Comments
Displayed Area Selection Sequence	(0070,005A)	AUTO	sequence of Items each of which describes the displayed area selection for a group of images or frames.	ALWAYS	
>Displayed Area Top Left Hand Corner	(0070,0052)	AUTO	The top left (after spatial transformation) pixel in the referenced image to be displayed,	ALWAYS	
>Displayed Area Bottom Right Hand Corner	(0070,0053)	AUTO	The bottom right (after spatial transformation) pixel in the referenced image to be displayed,	ALWAYS	
>Presentation Size Mode	(0070,0100)	AUTO	SCALE TO FIT	ALWAYS	
>Presentation Pixel Spacing	(0070,0101)	AUTO	Physical distance between the center of each pixel in the referenced image.	ALWAYS	
>Presentation Pixel Aspect Ratio	(0070,0102)	AUTO	Ratio of the vertical size and the horizontal size of the pixels in the referenced image. Example: 1:1	ALWAYS	

Table 30: Graphic Annotation Module

Attribute	Tag	Source	Value	Presence	Comments
Graphic Annotation Sequence	(0070,0001)	AUTO		ALWAYS	

Attribute	Tag	Source	Value	Presence	Comments
>Graphic Layer	(0070,0002)	AUTO	FINDINGS	ALWAYS	
>Text Object Sequence	(0070,0008)	AUTO		ALWAYS	
>>Bounding Box Annotation Units	(0070,0003)	AUTO		ALWAYS	
>>Anchor Point Annotation Units	(0070,0004)	AUTO		ALWAYS	
>>Unformatted Text Value	(0070,0006)	AUTO		ALWAYS	
>>Bounding Box Top Left Hand Corner	(0070,0010)	AUTO		ALWAYS	
>>Bounding Box Bottom Right Hand Corner	(0070,0011)	AUTO		ALWAYS	
>>Bounding Box Text Horizontal Justification	(0070,0012)	AUTO	CENTER	ALWAYS	
>>Anchor Point	(0070,0014)	AUTO		ALWAYS	
>>Anchor Point Visibility	(0070,0015)	AUTO	Y	ALWAYS	
>Graphic Object Sequence	(0070,0009)	AUTO		ALWAYS	
>>Graphic Annotation Units	(0070,0005)	AUTO	PIXEL	ALWAYS	
>>Graphic Dimensions	(0070,0020)	AUTO	2	ALWAYS	
>>Number of Graphic Points	(0070,0021)	AUTO	Number of data points in this graphic.	ALWAYS	
>>Graphic Data	(0070,0022)	AUTO	Coordinates that specify this graphic annotation.	ALWAYS	
>>Graphic Type	(0070,0023)	AUTO	POLYLINE	ALWAYS	
>>Graphic Filled	(0070,0024)	AUTO	N	ALWAYS	
>Line Style Sequence	(0070,0232)	AUTO		ALWAYS	
>>Shadow Style	(0070,0244)	AUTO	NORMAL	ALWAYS	
>>Shadow Offset X	(0070,0245)	AUTO		ALWAYS	
>>Shadow Offset Y	(0070,0246)	AUTO		ALWAYS	
>>Shadow Color CIELab Value	(0070,0247)	AUTO		ALWAYS	
>>Pattern On Color CIELab Value	(0070,0251)	AUTO		ALWAYS	
>>Line Thickness	(0070,0253)	AUTO		ALWAYS	

Attribute	Tag	Source	Value	Presence	Comments
>>Line Dashing Style	(0070,0254)	AUTO		ALWAYS	
>>Shadow Opacity	(0070,0258)	AUTO		ALWAYS	
>>Pattern On Opacity	(0070,0284)	AUTO		ALWAYS	

8.1.2 Usage of Attributes from Received IODs

NOT APPLICABLE

8.1.3 Attribute Mapping

NOT APPLICABLE

8.1.4 Coerced/Modified Fields

NOT APPLICABLE

8.2 Data Dictionary of Private Attributes

Table 31 lists all private attributes created by AI-Rad Companion Chest X-ray which may be included in the generated instances

Table 31: Private Data Element Dictionary

Tag	Private Owner Code	Name	VR	VM	Description
(0015, xx10)	AI-Rad Companion Chest X-ray	Pneumothorax AI score	IS	1	Integer string between 1 to 10
(0015, xx11)	AI-Rad Companion Chest X-ray	Pleural Effusion AI score	IS	1	Integer string between 1 to 10
(0015, xx12)	AI-Rad Companion Chest X-ray	Lesions AI score	IS	1	Integer string between 1 to 10
(0015, xx13)	AI-Rad Companion Chest X-ray	Consolidation AI score	IS	1	Integer string between 1 to 10
(0015, xx14)	AI-Rad Companion Chest X-ray	Atelectasis AI score	IS	1	Integer string between 1 to 10
(0015, xx15)	AI-Rad Companion Chest X-ray	Abnormality indicator	IS	1	Integer string – 0 or 1: '1' if there is at least one finding abnormal, '0' otherwise.

8.3 Coded Terminology and Templates

8.3.1 Context Groups

Not Applicable.

8.3.2 Template Specifications

8.3.2.1 TID 1500 Imaging Measurement Report for DICOM SR

Table 32: Imaging Measurement Report for DICOM SR

Nesting Level	Relationship Type	Value Type	Concept Name	Values	TID
		CONTAINER	EV (126000, DCM, "Imaging Measurement Report")		1500
>	CONTAINS	Code	EV(111017,DCM,CAD Processing and Findings Summary)	(111241, DCM, All algorithms succeeded; without findings) OR (111242, DCM, All algorithms succeeded; with findings) OR (111245, DCM, No algorithms succeeded; without findings)	4001
>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>	CONTAINS	Code	EV (111064, DCM, "Summary of Detections")	(111222, DCM, Succeeded) (111225, DCM, "Not Attempted")	4000
>	INFERRED FROM	CONTAINER	EV (111063, DCM, "Successful Detections")		4015
>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>	CONTAINS	Code	EV (111022, DCM, "Detection Performed")	(CHESTXRAY001, 99SHSAIRC, AI-Rad Companion Chest X-ray) AND/OR (CHESTXRAY002, 99SHSAIRC, AI-Rad Companion Chest X-ray Endotracheal Tube) AND/OR (CHESTXRAY004, 99SHSAIRC, AI-Rad Companion Non Actionable Finding)	4017
>>>	HAS PROPERTIES	TEXT	EV (111001, DCM, "Algorithm Name")	Assessment of AP view for 2 findings (Pneumothorax, Consolidation) OR Assessment of Endotracheal Tube OR Assessment of PA view for 5 findings (Pneumothorax, Pleural Effusion, Pulmonary Lesions, Consolidation, Atelectasis) OR Assessment of PA view for 1 finding(Pulmonary Nodule) OR Assessment of Non Actionable Finding	4019
>>>	HAS PROPERTIES	TEXT	EV (111003, DCM, "Algorithm Version")	Version of the algorithm where the findings were found For example: 10.0 or VA12A	4019
>>>	CONTAINS	IMAGE		Contains the reference to the original input image	4017
		CONTAINER	EV (126000, DCM, "Imaging Measurement Report")		1500
>	CONTAINS	CONTAINER	EV (126010, DCM, "Imaging Measurements")		1500
>>	CONTAINS	CONTAINER	EV (125007, DCM, "Measurement Group")		1410

Nesting Level	Relationship Type	Value Type	Concept Name	Values	TID
>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>	HAS OBS CONTEXT	TEXT	DT (112039, DCM, "Tracking Identifier")	LES / PEF / PTX / CO /AT/No finding/ ET/ Dist / CAR/NonActionableFinding/PotentialActionableFinding The findings of same category are enumerated (Eg: LES1, LES2 LES n.)	1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>	HAS OBS CONTEXT	UIDREF	EV (112040, DCM, "Tracking Unique Identifier")	Uniquely generated	1410
>>>	CONTAINS	CODE	EV (121071, DCM, "Finding")	A coded representation of the finding, using Radlex codes, that can take one of the following values: <ul style="list-style-type: none"> • (RID28493, RADLEX, Atelectasis) • (RID34539 ,RADLEX, Pleural effusion) • (RID43255, RADLEX, Consolidation) • (RID38780, RADLEX, Lesion) • (RID5352, RADLEX, Pneumothorax) • (111241, DCM, All algorithms succeeded;without findings) • (RID5557, RADLEX, EndotrachealTube) • (RID1248, RADLEX, Carina) • (DCM, 121206, Distance) • (111245, DCM, No algorithms succeeded; without findings) • (RID49484, RADLEX, F0:indeterminate) • (RID49486, RADLEX, F2:abnormal-insignificant) 	1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>	CONTAINS	TEXT	EV (121106, DCM, "Comment")	Detected View Position: AP/PA. AI results are auto generated and can be incomplete or incorrect. Assess original images for any decision. AI Confidence score should be always interpreted as the non-diagnostic likelihood of the findings OR "Input Data Error: Error description"	1410
>>>	CONTAINS	NUM	(DCM, 121206, Distance)	UNITS = EV(mm,UCUM,milimeter)	1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>	HAS CONCEPT MOD	CODE	(SCT, 106233006, Topographical modifier)	(RID5557, RADLEX, Endotracheal Tube) AND/OR (RID1248, RADLEX, Carina)	1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD

Nesting Level	Relationship Type	Value Type	Concept Name	Values	TID
>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>	HAS CONCEPT MOD	CODE	EV ("363698007", "SCT", "Finding site")	("RID1301", RADLEX, "Lung")	1419
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>	CONTAINS	SCOORD	EV (111030, DCM, "Image Region")		1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>>	SELECTED FROM	IMAGE		Original image	1410
>>>	CONTAINS	NUM	(RID29, RADLEX, "Confidence")	UNITS = EV (1, UCUM, "no units")	1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>>>>	HAS CONCEPT MOD	TEXT	EV (111012, DCM, "Certainty of Finding")	VALUE = Confidence range (1 - low to 10 - high)	1410
>>>		UIDREF	Observation UID ³	Uniquely generated	SR IOD
>	HAS CONCEPT MOD	CODE	EV (121049, DCM, "Language of Content Item and Descendants")	(eng, "RFC5646", "English")	1204
>>	HAS CONCEPT MOD	CODE	EV (121046, DCM, "Country of Language")	(US, ISO3166_1, "United States")	1204
>	HAS OBS CONTEXT	CODE	EV (121005, DCM, "Observer Type")	(121007, DCM, "Device")	1002
>	HAS OBS CONTEXT	UIDREF	EV (121012, DCM, "Device Observer UID")	Same as Device UID (0018,1002)	1004
>	HAS OBS CONTEXT	TEXT	EV (121014, DCM, "Device Observer Manufacturer")	Same as Manufacturer (0008,0070)	1004
>	HAS OBS CONTEXT	TEXT	EV (121015, DCM, "Device Observer Model Name")	Same as Manufacturer's Model Name (0008,1090)	1004
>	HAS CONCEPT MOD	CODE	EV (121058, DCM, "Procedure reported")	(36554-4, LOINC, XR Chest Single view)	1500
>	CONTAINS	CONTAINER	EV (111028, DCM, "Image Library")		1600
>>	CONTAINS	CONTAINER	EV (126200, DCM, "Image Library Group")		1600
>>>	CONTAINS	IMAGE		Contains 2 images - One for Original image/Secondary Capture image	1601
>>>>	HAS ACQ CONTEXT	CODE	EV (111031, DCM, "Image View")	(110871, DCM, Posterior to Anterior) OR (110870, DCM, Anterior To Posterior) OR (114010, DCM, Measurement not available: Value unknown)	1602
>>>>	HAS ACQ CONTEXT	CODE	EV (121139, DCM, "Modality")	Digital Radiography (DX, DCM, Digital Radiography) / Computed Radiography (CR, DCM, Computed Radiography) / other modality (OT) (RID49585, RADLEX, other modality (DICOM))	1602

Nesting Level	Relationship Type	Value Type	Concept Name	Values	TID
>>>>	HAS ACQ CONTEXT	DATE	EV (111060, DCM, "Study Date")	Copied from Original image	1602
>>>>	HAS ACQ CONTEXT	TIME	EV (111061, DCM, "Study Time")	Copied from Original image	1602

Note:

- 1) TID 1410 is extended to include field EV (121106, DCM, "Comment")
- 2) TID 1410 is extended to include field EV (111012, DCM, "Certainty of Finding")
- 3) Observation UID (0040,A171) is attribute in the representation of that Content Item. Hence relationship type is empty.
- 4) Error description will depend on the type of error.

8.3.3 Private Code definitions

The following tables list all private codes created by AI-Rad Companion Chest X-ray which may be included in the generated instances.

Table 33: Private Code definitions

Code Value	Code Meaning	Coding Scheme Designator
CHESTXRAY001	AI-Rad Companion Chest X-ray	99SHSAIRC
CHESTXRAY002	AI-Rad Companion Chest Lines & Tubes	99SHSAIRC
CHESTXRAY003	Processing Algorithm	99SHSAIRC
CHESTXRAY004	AI-Rad Companion Non Actionable Finding	99SHSAIRC

8.4 Grayscale Image Consistency

NOT APPLICABLE

8.5 Standard Extended / Specialized / Private SOP Classes

NOT APPLICABLE

8.6 Private Transfer Syntaxes

NOT APPLICABLE

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and are subject to change without prior notice.

Some / All the features and products described herein may not be available in the United States or other countries.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features that do not always have to be present in individual cases.

Siemens reserves the right to modify the design, packaging, specifications, and options described herein without prior notice. Please contact your local Siemens sales representative for the most current information.

In the interest of complying with legal requirements concerning the environmental compatibility of our products (protection of natural resources and waste conservation), we recycle certain components. Using the same extensive quality assurance measures as for factory- new components, we guarantee the quality of these recycled components.

Siemens Healthineers Headquarters

Siemens Healthineers AG

Siemensstr. 3

91301 Forchheim

Germany

Phone: +49 9191 18-0

siemens-healthineers.com



Siemens Healthcare GmbH

Henkestr. 127

91052 Erlangen

Germany

Published by Siemens Healthineers AG · ©Siemens Healthcare GmbH, 2025